Reduced Gap Deluxe Full-Length Bed Rail

Assembly, Installation and Operating Instructions

Model Nos. 6628 and 6628-TSS (extra-wide)

Yes, you can.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the dealer/carrier for further instructions.

To ensure the safe use of the reduced gap deluxe full-length bed rail, these instructions **MUST** be followed:

⚠ WARNINGS

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

These bed rails are intended to prevent an individual from inadvertently rolling out of bed. **DO NOT** use for restraint purposes.

ENTRAPMENT MAY OCCUR. Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, and mattress thickness, size or density could increase the risk of entrapment. Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Review **A Guide to Bed Safety**, published by the Hospital Bed Safety Workgroup, located at www.invacare.com. Use the link located under each bed rail product entry to access this bed safety guide.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Although bed rails are not rated to any specific weight limitation, the bed rails may become deformed or broken if excessive side pressure is exerted on the bed rails. This bed rail is not an assist rail for getting into or out of bed. **DO NOT** use the bed rails as push handles when moving the bed.

After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely. Bed rails with dimensions different from the original equipment supplied or specified by the bed manufacturer may not be interchangeable and may result in entrapment or other injury.

For more information regarding Invacare products, parts, and services,
please visit www.invacare.com

⚠ WARNING

On 6628 rail only, one crossbrace MUST be positioned between the sixth and seventh springs at the head end of the bed. The remaining crossbrace MUST be positioned between the second and third springs at the foot end of the bed. On 6628-TSS rail only, one crossbrace MUST be positioned between the fifth and sixth springs at the head end of the bed. The remaining crossbrace MUST be positioned between the second and third springs at the foot end of the bed. Otherwise, either the bed rail or the bed may be damaged when the head end and/or foot end are raised.

To ensure proper bed rail installation, arrows on crossbrace decal MUST point to center of bed.

The outer telescoping tube of the bed rail MUST be positioned at the head end of the bed to prevent a pinch point when raising/lowering bed.

To reduce the risk of entrapment, make certain that the crossbraces DO NOT exceed the width of the mattress. Mattress MUST fit bed frame and bed rails snugly to reduce the risk of entrapment.

Model 6628-TSS is designed and intended for use with powered air mattresses only. To reduce the risk of entrapment, model 6628-TSS must only be used with powered air mattresses.

Unless the bed rail is in the lowest position, ensure that the plunger pin fully protrudes through the same respective adjustment hole of each bed rail tube. This ensures that the bed rail is securely locked in position and an even height adjustment is achieved. (The lowest position for the bed rail occurs when the bed rail rests against the bed rail stop. Spring loaded pins DO NOT engage into an adjustment hole when the bed rail in the lowest position.)

Install bed rails fully extended on bed. DO NOT use full length bed rails as half rails.

Installing the Crossbraces

NOTE: READ AND UNDERSTAND WARNINGS before proceeding with procedure.

NOTE: For this procedure, refer to FIGURE 1 on page 4.

1. Remove the mattress from the bed.
2. Perform one of the following:
 - On 6628 Rails - Place the bed in the flat position and position one crossbrace between the sixth and seventh springs at the head end of the bed frame (Detail "A").

NOTE: If the bed frame contains BLACK springs at the sixth and seventh spring locations at the head end of the bed, position the crossbrace between these springs.

- On 6628-TSS Rails - Place the bed in the flat position and position one crossbrace between the fifth and sixth springs at the head end of the bed frame (Detail "A").

-
3. When holding either end of the crossbrace assembly, turn the barrels so plunger knobs are on top (Detail "A").

NOTE: Ensure the long end of the barrel faces the center of the bed with the decal to the center of bed as shown in FIGURE 1 on page 4. If long end faces the opposite way, rotate crossbrace 180°.

4. Position the bed hooks so they are on the bottom side of the crossbrace (Detail "B").
5. Perform one of the following:
 - Standard Hardware Installation - Perform the following steps:
 - i. Attach one bed hook onto the bed spring angle iron and push in on the spring loaded bed rail until you are able to attach the opposite end to the bed spring angle iron (Detail "B").
 - ii. Rotate the barrel so that the long ends face down (Detail "B").

NOTE: Ensure plunger pins are facing the center of the bed.

- Optional Hardware Installation - Perform the following steps:
 - i. Attach wingnuts to bed hooks.
 - ii. Tighten until the wingnut reaches the bed spring angle iron.
 - iii. Rotate the barrel so that the long ends face down.
6. Visually inspect crossbrace to confirm proper orientation.
 7. Position the remaining crossbrace between the second and third springs at the foot end of the bed.

NOTE: If the bed frame contains BLACK springs at the second and third spring locations at the foot end of the bed, position the crossbrace between these springs.

8. Repeat STEPS 3-7 for the foot end crossbrace.
9. Place mattress on bed frame.
10. Install the bed rails. Refer to [Installing the Bed Rails](#) on page 4.

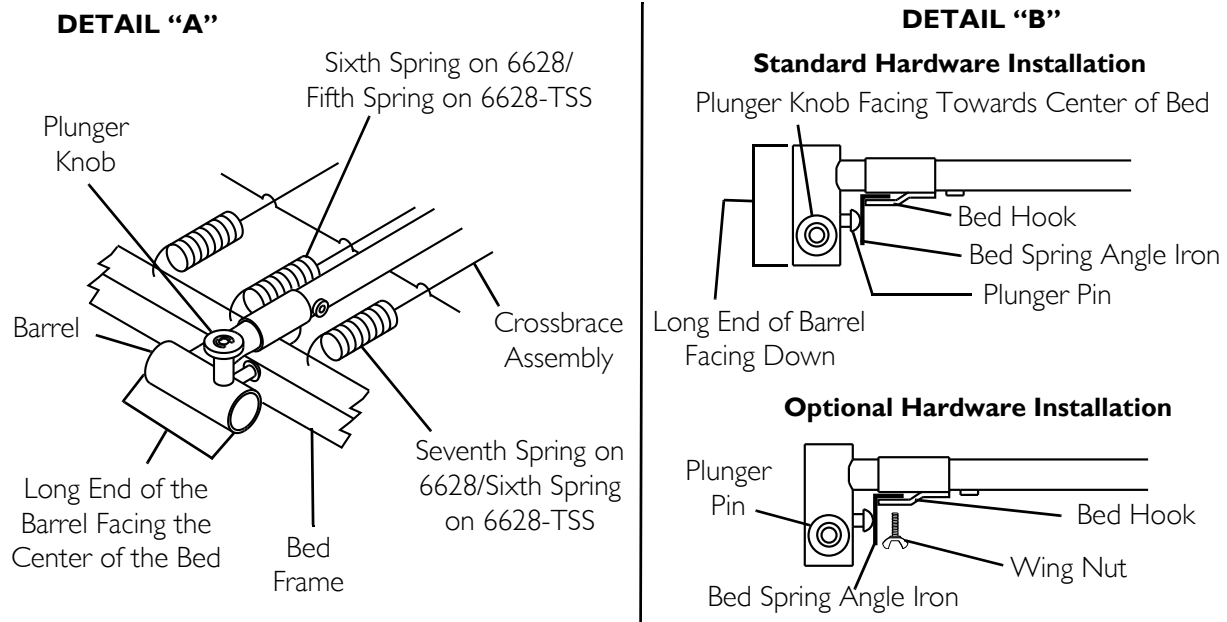
FIGURE 1 Installing the Crossbraces

Installing the Bed Rails

⚠ WARNING


The outer telescoping tube of the bed rail **MUST be positioned at the head end of the bed to prevent a pinch point when raising/lowering bed.**

NOTE: READ AND UNDERSTAND WARNINGS before proceeding with procedure.

NOTE: For this procedure, refer to FIGURE 2.

1. Extend one bed rail to reach the head end and foot end crossbraces.
2. Insert the outer telescoping tube of the bed rail into the barrel of the head end crossbrace.
3. Pull and hold the plunger knob on the barrel while lowering the bed rail.
4. Release the plunger knob on the barrel.
5. Raise or lower the bed rail until the plunger pin clicks securely into the adjustment hole or until the rail stop securely rests against the barrel.
6. Insert the inner telescoping tube of the bed rail into the barrel of the foot end crossbrace.
7. Repeat STEPS 3-5 for the inner telescoping tube of the bed rail, adjusting the bed rail to the position reached in STEP 5.
8. Repeat STEPS 1-7 to install the remaining bed rail.

WARNING
Patient entrapment with bed side rails may cause injury or death. Mattress must fit bed frame and side rails snugly to prevent patient entrapment. Follow the manufacturer's instructions. Monitor patient frequently. Read and understand the Owner's/Operator's Manual prior to using this equipment. Manual is available at www.invacare.com or your dealer. PIN 1100183 Rev A - 1104

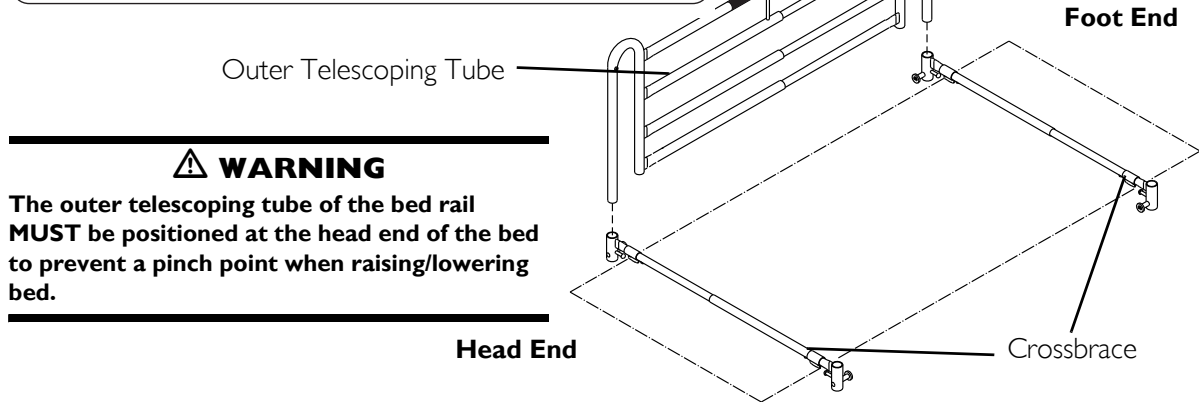WARNING
The outer telescoping tube of the bed rail **MUST** be positioned at the head end of the bed to prevent a pinch point when raising/lowering bed.

FIGURE 2 Installing the Bed Rails

Operating Instructions

NOTE: READ AND UNDERSTAND WARNINGS before proceeding with procedure.

Lowering/Raising the Bed Rail

1. Hold bed rail while pulling the plunger pin.
2. Lower/raise bed rail.
3. Release plunger pin and lower/raise the bed rail until plunger pin clicks into position unless bed rail is at lowest position.

NOTE: The lowest position for the bed rail is when the bed rail rests against the bed rail stop. Spring loaded pins DO NOT have an adjustment hole to engage into when the bed rail in the lowest position.

4. Repeat STEPS 1-3 for opposite end of bed rail.

NOTE: Ensure head end and foot end of bed rails are adjusted to the same height.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for one year for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO NORMAL WEAR AND TEAR OR FAILURE TO ADHERE TO THE PRODUCT INSTRUCTIONS.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2010 Invacare Corporation. All rights reserved.
Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Corporation www.invacare.com
© 2010 Invacare Corporation