Patient Slings

DEALER: This manual must be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

For more information regarding
Invacare products, parts, and services,
please visit www.invacare.com

Yes, you can.

WARNING

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

REGISTER YOUR PRODUCT

The benefits of registering include:

1. Safeguarding your investment.
2. Ensuring long-term maintenance and servicing of your product.
3. Receiving updates with product information, maintenance tips and industry news.

Register ONLINE at warranty.invacare.com

Please have your model number and purchase date available to complete your registration.

Any registration information you submit will only be used by Invacare Corporation and protected as required by applicable laws and regulations.

NOTE: Updated versions of this manual are available on www.invacare.com.

TABLE OF CONTENTS

REGISTER YOUR PRODUCT	2
SPECIAL NOTES	4
SECTION 1—GENERAL GUIDELINES	5
Using the Sling.....	5
SECTION 2—MODEL NUMBERS	7
SECTION 3—STANDARD SLINGS	8
Fabric.....	8
Body.....	9
Positioning the Sling.....	9
Positioning the Patient on the Sling.....	10
Attaching the Sling Hardware.....	11
Replacing the Four Metal Support Bars in the Sling.....	12
Attaching the Chains or Straps to the Slings.....	13
SECTION 4—FULL BODY, DIVIDED LEG AND TOILETING SLINGS	14
Model No. R140 - R141 Heavy Duty Slings.....	14
Model Nos. R110-R117 Full Body Slings.....	14
Fabric	14
Description.....	15
Positioning the Full Body Slings.....	15
Positioning the Patient on the Sling.....	16
SECTION 5—DIVIDED LEG/TOILETING SLING	18
Model Nos. R100-R102 Divided Leg and Model No. R121 Toileting Sling.....	18
Positioning the Patient in the Divided Leg Sling.....	18
SECTION 6—ATTACHING SLINGS TO LIFTS	21
Slings Using Chains or Straps.....	21
Slings Using Color Coded Straps.....	22
SECTION 7—STAND UP AND TRANSPORT SLINGS	23
Patient Slings.....	23
Standing Sling - Models R130 and R131	24
Transport Sling - Models R134 and R135	24
Lifting Preparation.....	24
Positioning the Slings Around the Patient.....	24
Attaching the Slings to the Stand Up Lift.....	27
LIMITED WARRANTY	30

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

MAINTENANCE

Maintenance MUST be performed ONLY by qualified personnel.

SECTION I—GENERAL GUIDELINES

⚠ WARNING

SECTION I - GENERAL GUIDELINES contains important information for the safe operation and use of this product.

DO NOT move a person suspended in a sling any distance. The Invacare patient lift or the sling are **NOT** transport devices. They are intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair). Otherwise, injury or damage may occur.

Check all parts for shipping damage before using. In case of damage, **DO NOT** use the equipment. Contact the dealer for further instructions.

DO NOT attempt any transfer without approval of the patient's physician, nurse or medical assistant. Thoroughly read the instructions in this owner's manual, observe a trained team of experts perform the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient.

Use common sense in all lift procedures. Special care must be taken with people with disabilities who cannot cooperate while being lifted.

ALWAYS keep hands and fingers clear of moving parts to avoid injury.

DO NOT exceed maximum weight limitation of the patient lift. These weights will vary from 350 to 600 lbs. depending on the type of patient lift purchased. For maximum weight limitation, refer to the Typical Product Parameters section in the patient lift manual or contact customer service.

Invacare slings are made specifically for use with Invacare lifts. For the safety of the patient, **DO NOT** intermix slings and lifts of different manufacturers.

Using the Sling

Standing Slings - Individuals that use the standing sling **MUST** be able to support the majority of their own weight, otherwise injury can occur. Before lifting the patient, make sure the bottom edge of the standing sling is positioned on the lower back of the patient and the patient's arms are outside the standing sling. The belt **MUST** be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer, possibly causing injury.

Transport Slings - Before lifting the patient, make sure the bottom edge of the transport sling is at the base of the spine and the patient's arms are outside the transport sling. **DO NOT** raise the patient to a full standing position while using the transport sling, otherwise injury may occur.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and sling material that may cause the patient to slide out of the sling during transferring.

ALWAYS use the color coded strap on the standing sling closest to the patient while still maintaining patient stability and comfort.

Use an Invacare approved sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual being lifted.

Laundering should always be done with dark colors. The sling should be washed regularly in a water temperature not to exceed 180°F (82°C). DO NOT bleach. Air dry or dry at low temperature. Refer to tagged washing instructions on the sling.

After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching.

Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.

DO NOT alter slings.

DO NOT move the patient if the sling is not properly connected to the attachment points of the lift. Check that the sling is properly connected to the attachment points prior to lifting a patient. Attachment points include swivel bar, straps, S-hooks and D or O-rings. If any attachments are not properly in place, correct the problem. When the sling is elevated a few inches off the stationary surface and before moving the patient, check again to make sure that all sling attachments are secure. If any attachments are not properly in place, lower the patient back onto the stationary surface and correct this problem - otherwise, injury or damage may occur.

During transfer, with patient suspended in a sling attached to the lift, DO NOT roll caster base over objects such as carpet, raised carpet bindings, door frames, or any uneven surfaces or obstacles that would create an imbalance of the lift and could cause the lift to tip over. Use the steering handles on the mast at all times to push or pull the patient lift.

SECTION 2—MODEL NUMBERS

MODEL NOS. STD.	DESCRIPTIONS
9042	Polyester Sling
9043	Polyester Sling With Commode Opening
9046	Polyester Mesh Sling
9047	Polyester Mesh Sling With Commode Opening
9070	Strap Kit
9071	Chain Kit

Note: Slings listed above require a strap or chain kit for hook-up.

CHAIN-FREE ALL FABRIC SLINGS	
*R110 - R117	Full Body Slings With or Without Commode Openings
*R101 - R102	Divided Leg Slings
*R121	Toileting Sling
**R140 AND R141	Heavy Duty With or Without Commode Openings
***R130, R131, R134, R136	Standing and Transport Slings

*Note: Can ONLY be used with Invacare Models 9700, 9701, 9805, 9806, 9807 and Reliant Lifts on the large hooks of the swivel bar or on converted model nos. 9801, 9802 and 9803 (9800A Boom Retrofit Kit). The small hooks are to be used for chain/strap kits only.

**Note: Can ONLY be used with Invacare Model Reliant 450 and 600 Lifts.

***Note: Can ONLY be used with Invacare Model Reliant 350 and 440 Lifts.

⚠ WARNING

DO NOT exceed maximum weight limitation of the patient lift. The weight capacity is limited to the lowest rated capacity of any one of the components in use (e.g. patient lift, sling, or scale). The patient's weight MUST not exceed the lowest rated capacity of any component.

SECTION 3—STANDARD SLINGS

MODEL NUMBERS STD.	DESCRIPTIONS
9042	Polyester
9043	Polyester w/ Commode Opening
9046	Polyester Mesh
9047	Polyester Mesh w/ Commode Opening

Invacare slings are designed specifically for use with Invacare Patient Lifts and made to support the patient during lift and transfer procedures. Slings attach to the lift with either adjustable straps or chains (interchangeable). The slings are reinforced at all points of attachment for safety. The fabrics used in these slings are of durable materials that resist deterioration from exposure to moisture and laundering. This design will accommodate patients of heavy or light weight with support from the shoulders to the knees. The contour shape provides comfort during the lift by avoiding the bulk of the excess material between the back and the seat sections.

Fabric

The inner side of the fabric is smooth while the pocket on the outer side has binding around it.

NOTE: Always place the sling under the patient with the smooth side next to the body.

NOTE: Laundering should always be done with dark colors. The sling should be washed regularly in a water temperature not to exceed 180°F (82°C). DO NOT bleach. Air dry or dry at low temperature. Inspect with each use. Refer to tagged washing instructions on the sling.

⚠ WARNING

After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching. Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.

Body

Slings may be ordered at the time of purchase with a commode opening in the seat section for use over a toilet or a commode chair.

Each sling includes four metal support bars that have a hole at the center to attach the “S” hook.

When shipped from the factory, these reinforcing metal support bars are enclosed in the sling pocket with the hole exposed.

Positioning the Sling

NOTE: For this procedure, refer to FIGURE 3.1.

1. Position the patient in the center of the bed and laying flat on his/her back. Refer to Detail “A”.
2. Fold the sling in half (length-wise) and place the fabric sections of the sling beside the patient.

NOTE: The sling’s french seams (uneven surface) will be against the bed surface and facing the attendant when folded. The opposite side of the fabric is smooth. After the sling is positioned for lifting purposes, the smooth side will be next to the patient.

3. The top edge of the sling fabric (nearest to the patient’s head) should be slightly above the armpit level of the patient. Refer to Detail “B”.
4. The bottom edge of the sling fabric should then be a few inches above the back of the patient’s knee.
5. With both attendants holding the patient, push the folded sling under the patient without rolling him/her over.

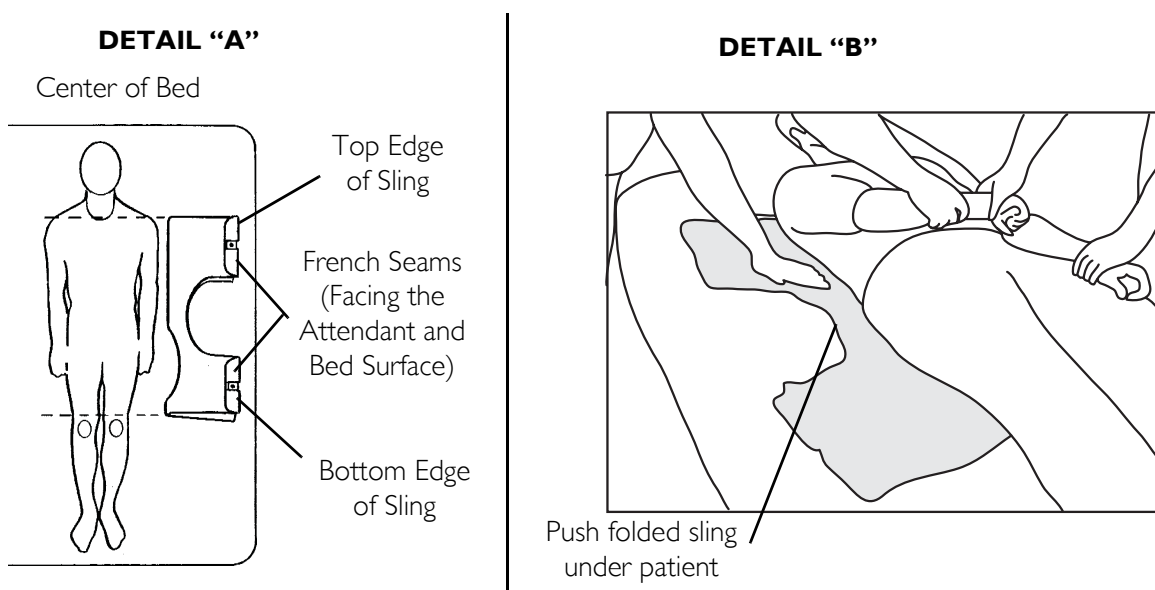
FIGURE 3.1 Positioning of the Standard Sling

Positioning the Patient on the Sling

NOTE: For this procedure, refer to FIGURE 3.2 and FIGURE 3.3 on page 11.

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

NOTE: Although Invacare recommends that two assistants be used for all lifting preparation, transferring from and transferring to procedures, our equipment will permit proper operation by one assistant. The use of one assistant is based on the evaluation of the health care professional for each individual case. Bed rails may be used to minimize patient movements during sling positioning.

1. If the patient is to be rolled to their left-side, elevate the patient's right knee until the right foot is flat on the bed. Refer to Detail "A".
2. With an attendant on each side of the bed and up against the mattress, the attendant on the left-hand side of the bed will position his/her right-hand on the elevated knee and his/her left-hand under the patient's right shoulder, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side.
3. With the patient on their side, push the fabrics of the seat and back gently under them. Refer to Detail "B".

NOTE: The upper edge of the back section should be positioned slightly above the armpit level and the lower edge of the seat section positioned a few inches above the back of the patient's knees.

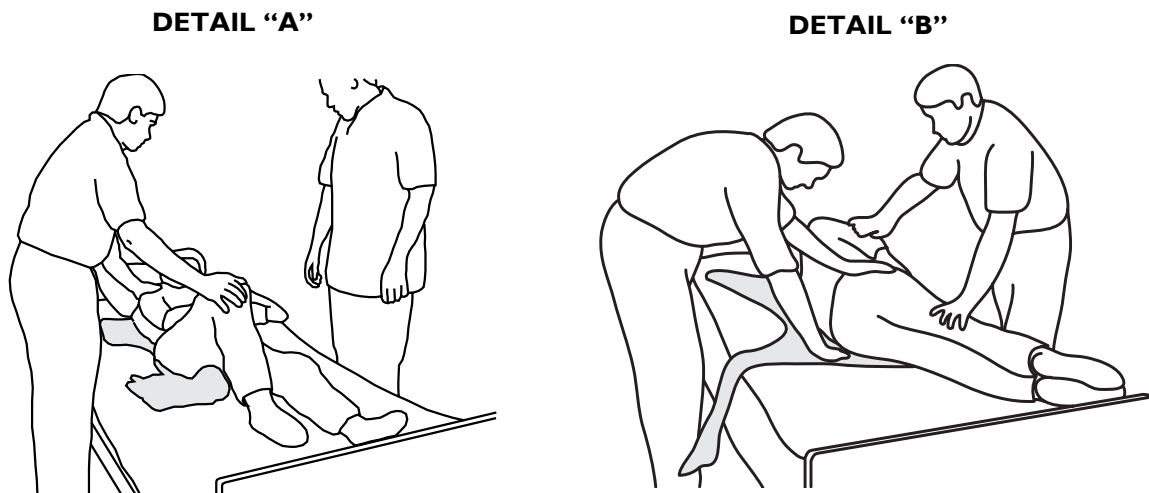
FIGURE 3.2 Rolling the Patient onto His/Her Side

4. Roll the patient on to his/her back.
Refer to Detail "A" of FIGURE 3.3.
5. After the patient has been positioned once again on his/her back, roll the patient to their right-side (facing the attendant on the left).
6. With an attendant on each side of the bed and up against the mattress, the attendant on the right-hand side of the bed will elevate the left knee and position his/her left-hand on the elevated knee and his/her right-hand under the patient's left shoulder, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side.
Refer to Detail "B" of FIGURE 3.3.
7. Pull the fabrics of the seat and back across the mattress until smooth.
8. Roll the patient onto his back and he should be approximately centered on the sling section. Refer to Detail "C" of FIGURE 3.3

NOTE: The sling is now ready to be attached to the chain or strap. Refer to Attaching the Sling Hardware.

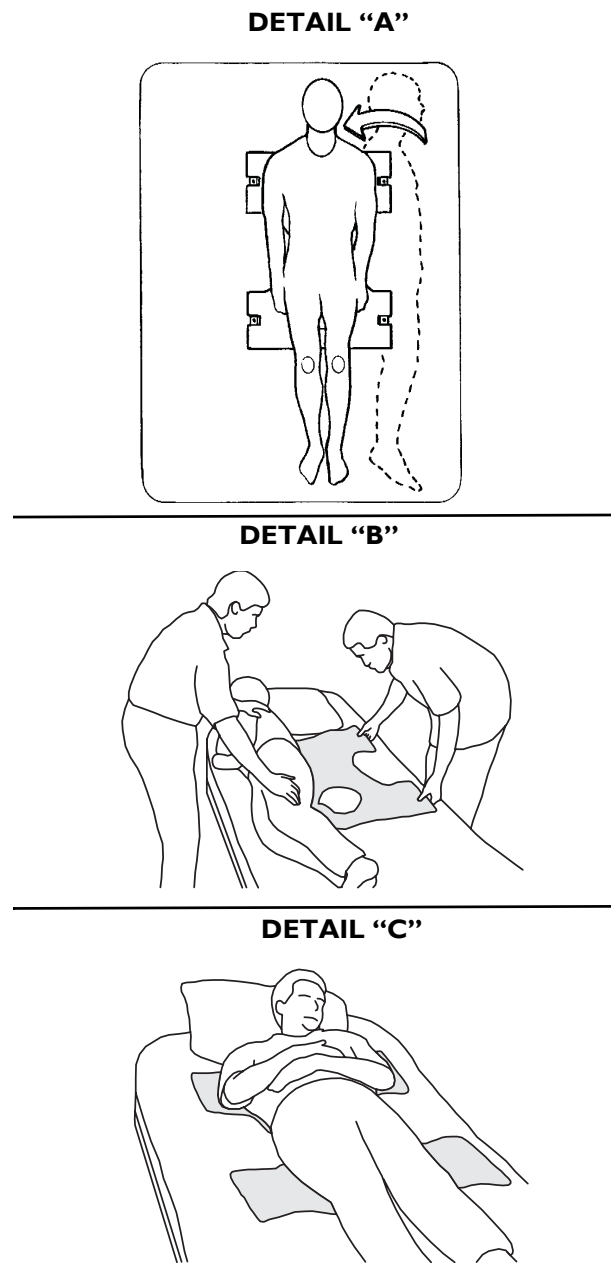
FIGURE 3.3 Rolling the Patient onto His/ Her Opposite Side and Back

Attaching the Sling Hardware

MODEL NUMBERS	DESCRIPTIONS
9070	Strap Kit (Can be used with Models 9042, 9043, 9046, 9047)
9071	Chain Kit (Can be used with Models 9042, 9043, 9046, 9047)

The hardware used with the standard sling includes four metal support bars. The support bars insert into the pockets at the edge of the fabric. The pocket that will hold the metal bar has an opening in the center. The pocket is sewn closed on one end and has an open end with a strap on the other.

NOTE: When the slings are laundered, remove the four metal support bars. They MUST be replaced or the sling can not be used for lifting purposes.


Replacing the Four Metal Support Bars in the Sling

NOTE: For this procedure, refer to FIGURE 3.4.

NOTE: When inserting the support bar into the sling pocket, align the metal support bar in the center of the strap (after it has been pulled down) and then push the metal support bar into the sling pocket. The sling is sewn on both sides so the center location is critical.

1. To insert the metal support bar, pull the strap down allowing the bar to pass through the opening in the sling pocket. Refer to Detail "A".
2. Push the metal support bar through until the mounting hole clears the center opening of the sling pocket. Refer to Details "C", "D", and "E".
3. Once the opposite end of the bar is inside the sewn pocket, push the strap over the metal support bar to prevent it from coming out. Refer to Detail "B".

NOTE: The hole in the metal bar should be in the center of the pocket opening. This is where the "S" hook of the chain or strap will be attached.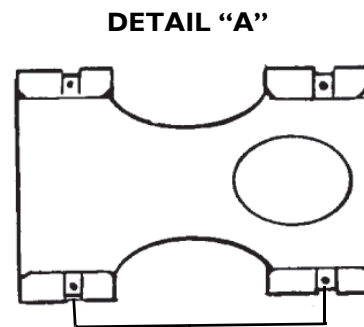

Metal Support Bars with Openings for Chains or Straps (2 of 4)

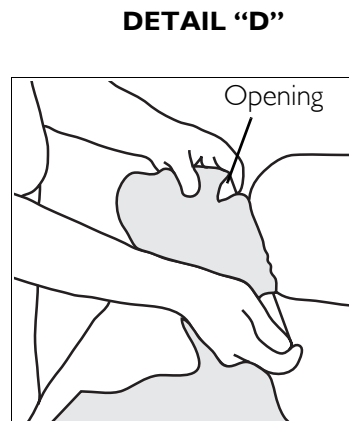
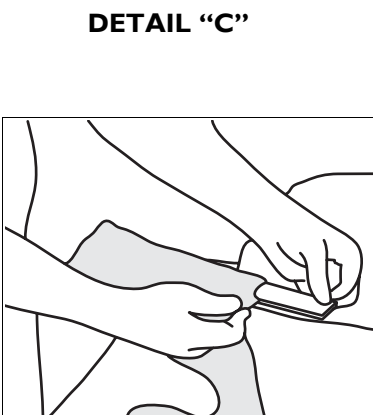
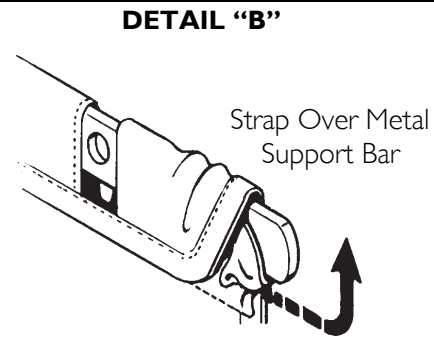
FIGURE 3.4 Replacing the Four Metal Support Bars in the Sling

Attaching the Chains or Straps to the Slings

NOTE: For this procedure, refer to FIGURE 3.5 on page 13.


NOTE: Each sling will work with either a pair of chain assemblies or a pair of strap assemblies.

CAUTION


When using either the chains or straps to connect the sling to the patient lift, the shortest of the two sections of the chains or straps **MUST** be attached to the back section of the sling. Using the long section will leave little or no support for the patient's back.

1. With the patient laying on his/her back and positioned properly in the sling, lay the chains or straps on the bed next to the sling. Refer to Detail "A".
2. The short section of chain or strap will be placed parallel to the patient's back. This will position the longer section of chain or strap from the waist to the mid section of the thigh.
3. The "S" hooks on the ends of the chains or straps are inserted into the metal support bar of the sling.
4. The "S" hook **MUST** be inserted through the hole in the support bar with the open end facing away from the patient. Refer to Detail "C".
5. Position the lift for use. Refer to Attaching Slings to Lifts on page 21.

NOTE: The lift must be positioned for use before the slings can be attached.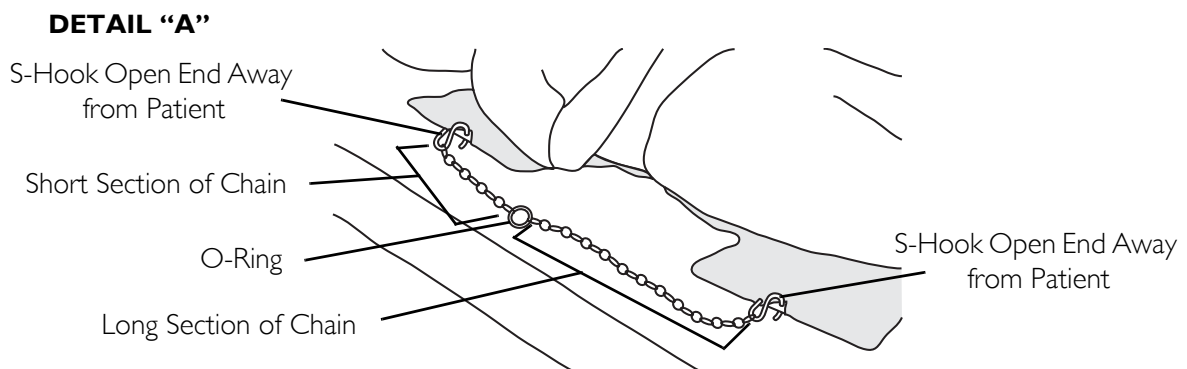

DETAIL "B"

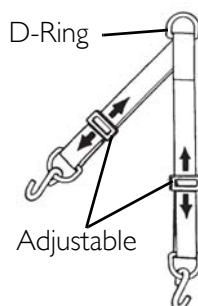DETAIL "C"

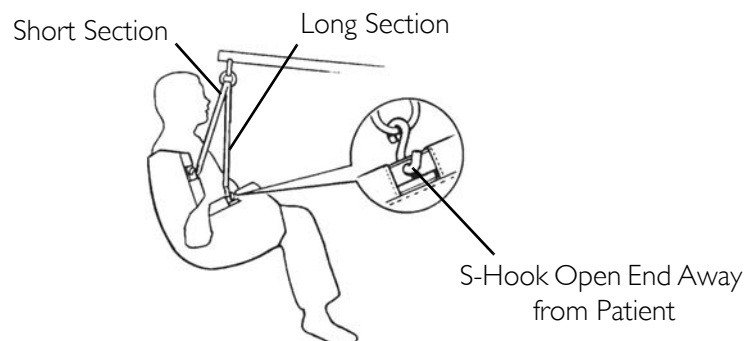
FIGURE 3.5 Attaching the Chains or Straps to the Slings

SECTION 4—FULL BODY, DIVIDED LEG AND TOILETING SLINGS

Model No. R140 - R141 Heavy Duty Slings

NOTE: For this section, refer to FIGURE 4.1.

Model nos. R140 - R141 heavy duty slings have a full body mesh sling with optional commode opening, two straps per side, and supports up to 600 lbs.

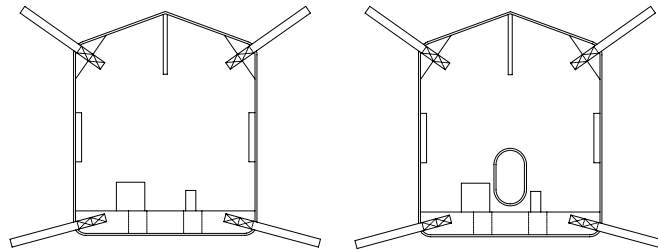
FIGURE 4.1 Model No. R140 - R141 Heavy Duty Slings

Model Nos. R110-R117 Full Body Slings

NOTE: For this section, refer to FIGURE 4.2 on page 15.

NOTE: Model Nos. R110-R117 are full body slings with or without a commode opening.

Invacare slings are designed specifically for use with Invacare Patient Lifts and made to support the patient during lift and transfer procedures. Slings attach to the lift with color coded straps for easy positioning. The slings are reinforced at all points of attachment for safety. The fabrics used in these slings are of durable materials that resist deterioration from exposure to moisture and laundering. Designs will accommodate patients of heavy or light weight with full support from the head to the knees.

Fabric

The outer side has a reinforced "grab handle" for assistance in positioning the patient in a wheelchair, commode, etc. The edging is soft yet durable with extra reinforced stitching for safety.

NOTE: Always place the sling under the patient with the handle away from the patient.

NOTE: Laundering should always be done with dark colors. The sling should be washed regularly in a water temperature not to exceed 180°F (82°C). DO NOT bleach. Air dry or dry at low temperature. Inspect with each use. Refer to tagged washing instructions on the sling.

⚠ WARNING

After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching. Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.

Description

Slings may be ordered at the time of purchase with a commode opening in the seat section for use over a toilet or a commode chair. Slings are constructed with a high back section that provides additional support for the head and neck and a padded under thigh section for comfort.

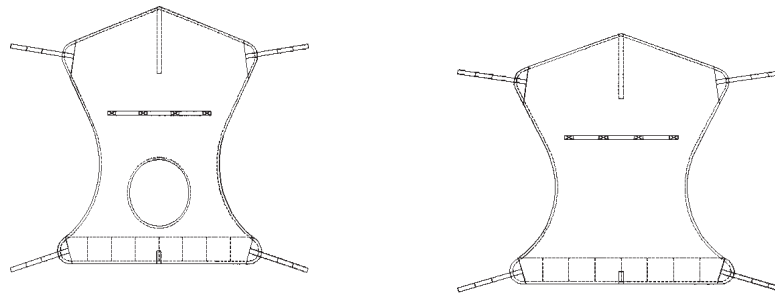
FIGURE 4.2 Model Nos. R110-R117 Full Body Slings

Positioning the Full Body Slings

NOTE: For this procedure, refer to FIGURE 4.3 on page 16.

NOTE: Model Nos. R110 - R117 are full body slings with or without a commode opening.

NOTE: Although Invacare recommends that two assistants be used for all lifting preparation, transferring from and transferring to procedures, our equipment will permit proper operation by one assistant. The use of one assistant is based on the evaluation of the health care professional for each individual case. Bed rails may be used to minimize patient movements during sling positioning.

1. Position the patient in the center of the bed and laying flat on his/her back. Refer to Detail "A".
 2. Fold the sling in half (length-wise) and place the sling beside the patient.
- NOTE: The closed-end or commode opening of the sling with positioning handle should be facing the patient when folded.*
3. The top edge of the sling fabric should be slightly above the patient's head.
 4. The bottom edge of the sling fabric should then be a few inches above the back of the patient's knees.
 5. With one assistant holding the patient, the second assistant pushes the folded sling under the patient without rolling him/her over. Refer to Detail "B".

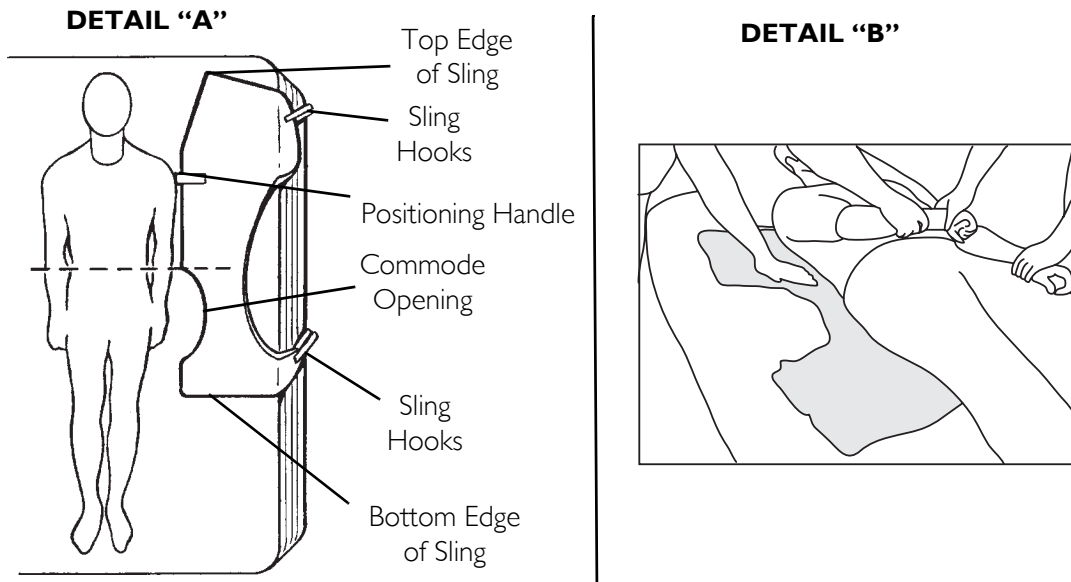
FIGURE 4.3 Positioning the Sling

Positioning the Patient on the Sling

NOTE: For this procedure, refer to FIGURE 4.4 and FIGURE 4.5 on page 17.

NOTE: Although Invacare recommends that two assistants be used for all lifting preparation, transferring from and transferring to procedures, our equipment will permit proper operation by one assistant. The use of one assistant is based on the evaluation of the health care professional for each individual case. Bed rails may be used to minimize patient movements during sling positioning.

1. If the patient is to roll to their left-side, then elevate the patient's right knee until the right foot is flat on the bed. Refer to Detail "A".
2. With an assistant on each side of the bed and up against the mattress, the assistant on the left-hand side of the bed will position his/her right-hand on the elevated knee and his/her left-hand under the patient's right shoulder, slowly push on the knee and assist with a slight lift of the shoulder. The patient will easily roll onto their side.
3. With the patient on their side, push the fabrics of the seat and back gently under them. Refer to Detail "B".

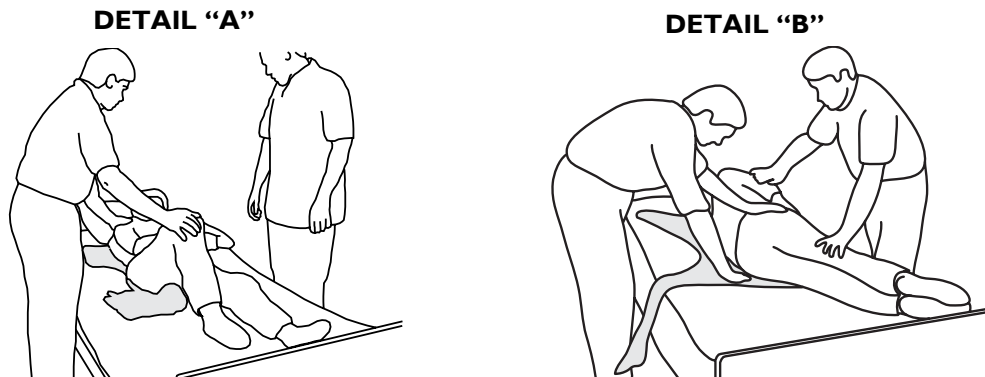
FIGURE 4.4 Rolling Patient Onto His/Her Side

NOTE: The patient's head should be positioned in the headrest just below the top edge for maximum comfort and the lower edge of the seat section positioned a few inches above the back of the patient's knees. Refer to Detail "A" of FIGURE 4.3 on page 16.

4. Roll the patient on to his/her back. Refer to Detail "A" of FIGURE 4.5.

NOTE: Assistants will reverse roles.

5. After the patient has been positioned once again on his/her back, you now need to roll the patient to their right-side (facing the assistant on the left).

6. With an assistant on each side of the bed and up against the mattress, the assistant on the right-hand side of the bed will elevate the left knee and position his/her left-hand on the elevated knee and his/her right-hand under the patient's left shoulder, slowly push on the knee and assist with a slight lift of the shoulder and the patient will easily roll onto their side. Refer to Detail "B" of FIGURE 4.5.

7. Pull the fabrics of the seat and back across the mattress until they are smooth.

8. Roll the patient onto their back and they should be approximately centered on the sling section. Refer to Detail "C" of FIGURE 4.5.

9. Position the lift for use. Refer to Attaching Slings to Lifts on page 21.

NOTE: The lift must be positioned for use before the slings can be attached.

FIGURE 4.5 Rolling Patient Onto His/Her Opposite Side and Back

SECTION 5—DIVIDED LEG/ TOILETING SLING

NOTE: Laundering should always be done with dark colors. The sling should be washed regularly in a water temperature not to exceed 180°F (82°C). DO NOT bleach. Air dry or dry at low temperature. Inspect with each use. Refer to tagged washing instructions on the sling.

⚠ WARNING

After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching. Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.

Model Nos. R100-R102 Divided Leg and Model No. R121 Toileting Sling

NOTE: For this procedure, refer to FIGURE 5.1 to FIGURE 5.4 on page 20.

Divided leg design is best suited for high level spinal cord injuries or patients on which sling removal is more difficult. Support ranges from the upper back to beneath the thighs. Can be positioned with patient seated. Greatly simplifies the use of restroom facilities.

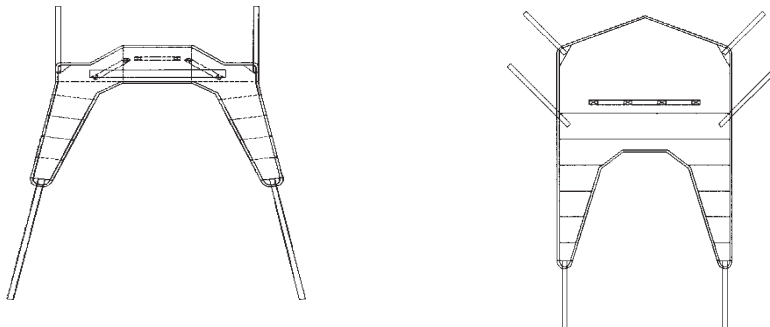
FIGURE 5.1 Model Nos. R100-R102 Divided Leg and Model No. R121 Toileting Sling

Positioning the Patient in the Divided Leg Sling

NOTE: Although Invacare recommends that two assistants be used for all lifting preparation, transferring from and transferring to procedures, our equipment will permit proper operation by one assistant. The use of one assistant is based on the evaluation of the health care professional for each individual case.

NOTE: Use the following method to easily move the patient and avoid strain to yourself.

⚠ WARNING

If the patient is in a wheelchair, secure the wheel locks in place to prevent the wheelchair from moving forward or backwards.

1. The rear wheel locks of the wheelchair are locked to prevent movement of the wheelchair.
2. With the patient sitting in a wheelchair one assistant in the front and the other assistant in the back, lean the patient forward. Refer to Detail "A".

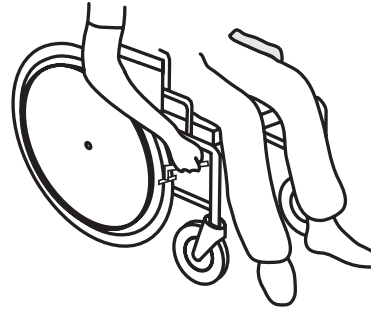
FIGURE 5.2 Rear Wheel Locks

NOTE: The front assistant will be supporting the weight of the patient.

3. Place the divided leg sling behind the patient (with "grab handle" on the outside) and bring the flaps out alongside the patient's legs. Refer to Detail "B".

NOTE: Toileting slings have a waist buckle which helps hold the sling in place during positioning.

NOTE: For the Divided Leg Sling: The back of the sling should be parallel to the patient's upper arms and be positioned between the top of the patient's head and the base of the spine.

NOTE: For the Toileting Sling: The back of the sling should be parallel to the patient's upper arms and be positioned between the top of the patient's midback and the base of the spine.

4. With the back of the sling positioned properly, push the bottom of the back section to the seat.
5. Lean the patient back into the wheelchair with the assistant in the rear supporting his/her weight.
6. Lift the patient's legs (one at a time) and reach under the patient's leg to pull the front of the sling until it is behind the patient's knees (about 3 inches). The back of the sling should remain in position. Refer to Details "C" to "F".


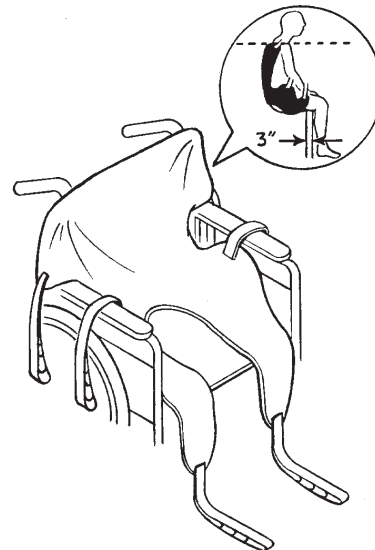
FIGURE 5.3 Positioning the Patient in the Divided Leg/Toilet Slings

NOTE: The straps may be around, crossed between or underneath the patient's legs. Refer to Detail "G".

7. Position the lift for use. Refer to [Attaching Slings to Lifts](#) on page 21.

NOTE: The lift must be positioned for use before the slings can be attached.

DETAIL "A"

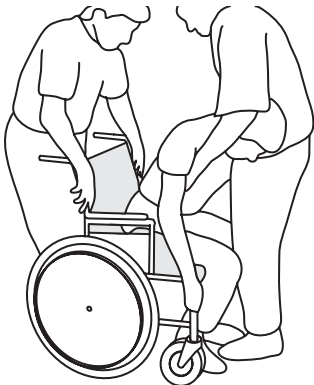
DETAIL "B"


DETAIL "C"


DETAIL "D"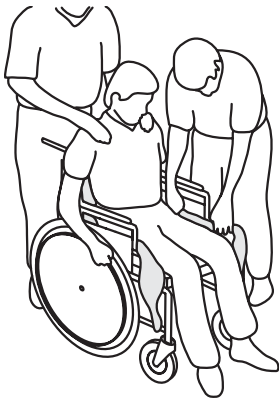


DETAIL "E"

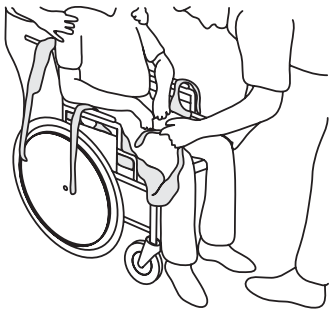DETAIL "F"

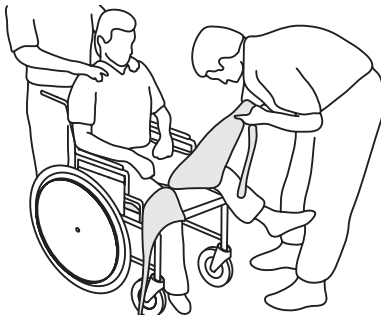DETAIL "G"

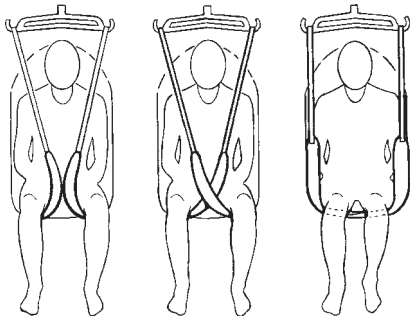Around Between Underneath

FIGURE 5.4 Positioning the Patient in the Divided Leg/Toilette Slings

SECTION 6—ATTACHING SLINGS TO LIFTS

Slings Using Chains or Straps

NOTE: For this procedure, refer to FIGURE 6.1.

1. The D-ring (strap) or O-ring (chain) is the suggested location to attach the swivel bar, however, by changing to different links or adjusting the strap, you can reposition the patient into a reclining or more upright position.

⚠ WARNING

When changing to different lengths, DO NOT disconnect the O-rings of the chains from the swivel bar. Position BOTH the O-rings and additional chain link onto the hooks of the swivel bar.

2. The straps can be simply adjusted to achieve the same effect.
3. With the sling positioned under the patient, and the support bars in place, you are now ready to move the lift into position to attach the sling hardware to the swivel bar.
4. Refer to Lifting the Patient in the Portable Patient Lift and Sling Owner's Manual.

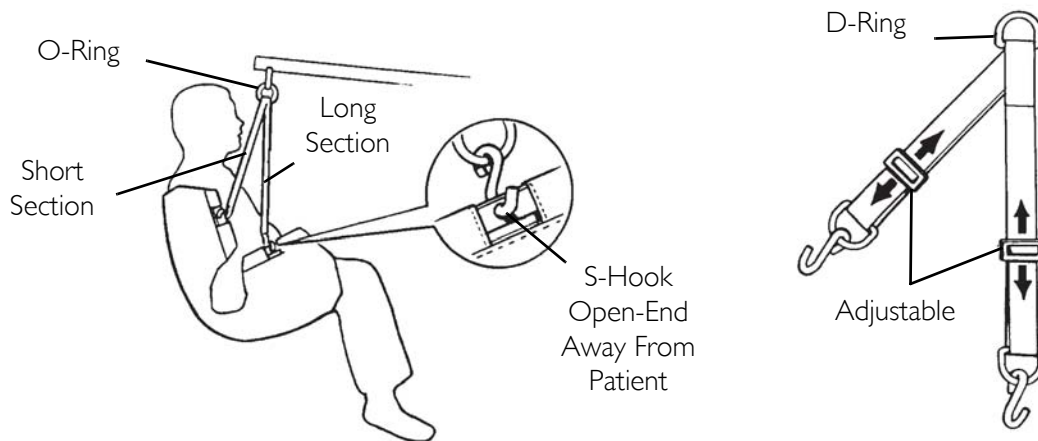
FIGURE 6.1 Attaching Slings to Lift - Slings Using Chains or Straps

Slings Using Color Coded Straps

NOTE: For this procedure, refer to FIGURE 6.2.

1. Place the straps of the sling over hooks of the swivel bar. Match the corresponding colors on each side of the sling for an even lift of the patient.

CAUTION

When connecting sling to the patient lift, the shortest of the straps **MUST be at the back of patient for support. Using long section will leave little or no support for patient's back. The loops of the sling are color coded and can be used to place patient in various positions. The colors make it easy to connect both sides of the sling equally. Make sure that there is sufficient head support when lifting a patient.**

NOTE: Model Nos. R110 - R117 Full Body Slings and Model No. 121 Toileting Sling have four sling straps. Model Nos. 100 - 102 Divided Leg Slings have six sling straps.

2. Refer to the Lifting the Patient in the Portable Patient Lift and Sling Owner's Manual.

NOTE: Invacare Lift Swivel Bars have three hookup points per side. The middle hookup is **ONLY** used for slings that have three sets of straps per side or slings that use chains or adjustable straps.

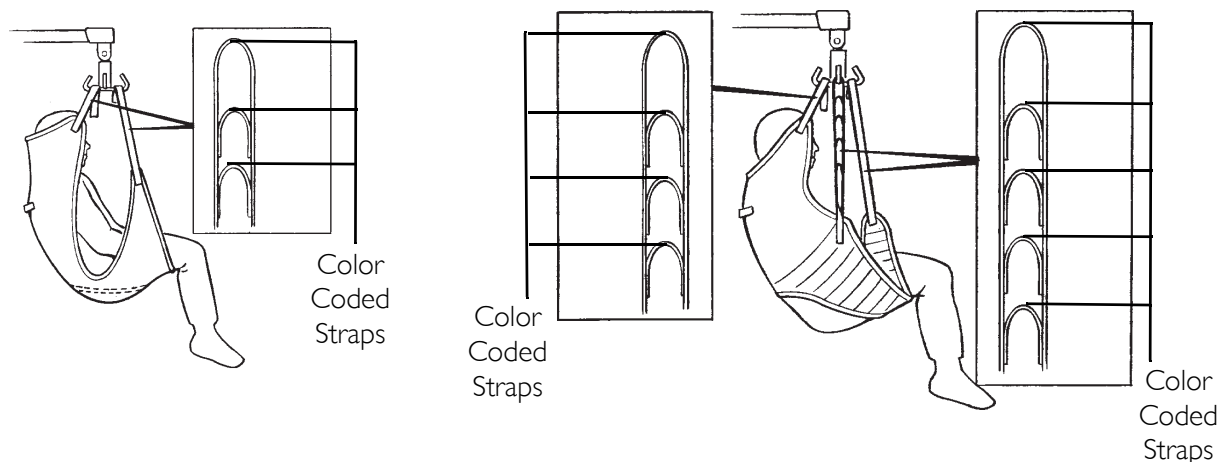
FIGURE 6.2 Attaching Slings to Lift - Slings Using Color Coded Straps

SECTION 7—STAND UP AND TRANSPORT SLINGS

Patient Slings

⚠ WARNING

Invacare standing and transport slings are specifically designed to be used in conjunction with Invacare Stand Up Lifts. Slings and accessories designed by other manufacturers or other Invacare slings are not to be utilized as a component of Invacare's stand up lift system. Use of these products is prohibited and will void Invacare's stand up lift warranty. Use only Invacare standing and transport slings and lift accessories to maintain patient safety and product utility. Use the sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual being lifted.

Read sling label for compatible lift usage before using any sling/stand up lift combination.

Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard Immediately.

DO NOT alter slings.

Refer to the stand-up lift owner's manual for additional information and guidelines.

Standing (Model R130 and R131) and Transport (Model R134 and R136) slings attach to the lift with color coded straps. The color coded straps allow for adjustment.

Standing (Model R130 and R131) and Transport (Model R134 and R136) slings are constructed of durable materials which resist deterioration from exposure to moisture or laundering.

NOTE: Laundering should always be done with dark colors. The sling should be washed regularly in a water temperature not to exceed 180°F (82°C). DO NOT bleach. Air dry or dry at low temperature. Refer to tagged washing instructions on the sling.

⚠ WARNING

After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching. Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.

Standing Sling - Models R130 and R131

⚠ WARNING

Individuals that use the Standing Sling (Model R130 and R131) **MUST** be able to support the majority of their own weight, otherwise injury can occur.

NOTE: For this section, refer to FIGURE 7.1.

Standing slings assist individuals that can support the majority of their own weight in a standing position. Multiple hook up points adjust to body sizes.

Models R130 and R131 (Back View of Sling)

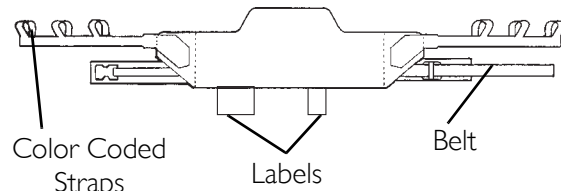
FIGURE 7.1 Standing Sling

Transport Sling - Models R134 and R135

⚠ WARNING

DO NOT raise the patient to a full standing position while using the transport sling (Models R134 and R136), otherwise injury can occur.

NOTE: For this section, refer to FIGURE 7.2.

Transport slings transfer fully dependent individuals in a seated position. Transport slings hook under the patient's legs and support their torso. Multiple hook up points adjust to body sizes.

Models R134 and R136 (Back View of Sling)

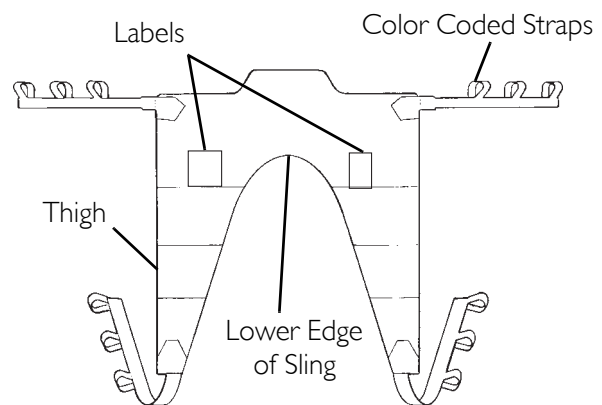
FIGURE 7.2 Transport Slings

Lifting Preparation

Positioning the Slings Around the Patient

Standing Sling (Models R130 and R131)

⚠ WARNING

Individuals that use the Standing Sling (Models R130 and R131) **MUST** be able to support the majority of their own weight, otherwise injury can occur.

DO NOT use the standing sling in combination with the stand up lift as a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair). Moving a person using the standing sling in combination with the stand up lift over any distance is not recommended.

⚠ WARNING

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and sling material that may cause the patient to slide out of the sling during transferring.

Before lifting the patient, make sure the bottom edge of the standing sling is positioned on the lower back of the patient and the patient's arms are outside the standing sling.

NOTE: For this procedure, refer to FIGURE 7.3.

1. Position the standing sling around the patient's torso just below the shoulder blades and under the arms. Refer to Detail "A".

NOTE: Make sure the bottom edge of the sling is on the lower back and the patient's arms are outside the standing sling.

NOTE: The belt should be towards the patient with the labels at the bottom of the standing sling.

2. Secure the standing sling to the patient by buckling the belt. Refer to Detail "B".

⚠ WARNING

The belt **MUST** be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer, possibly causing injury.

3. Adjust the belt for a snug, but comfortable fit. Refer to Detail "C".
4. Attach the standing sling to the stand up lift. Refer to [Attaching the Slings to the Stand Up Lift](#) on page 27.

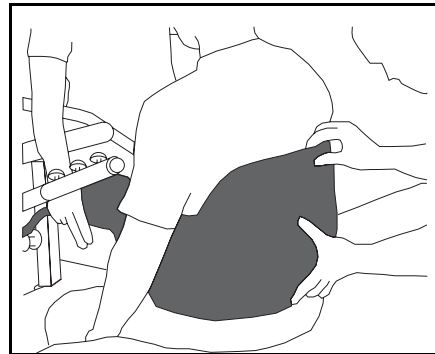
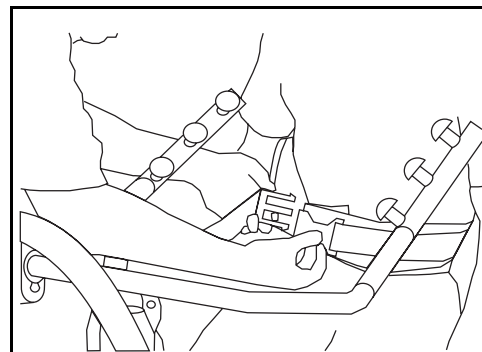
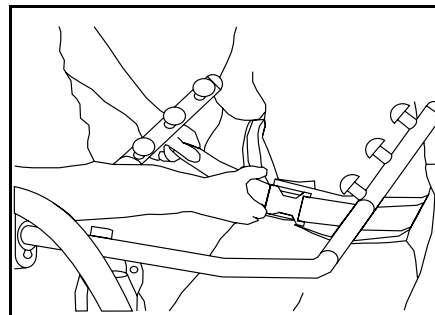
DETAIL "A"**DETAIL "B"****DETAIL "C"**

FIGURE 7.3 Positioning the Slings Around the Patient - Standing Slings.

Transport Sling (Model R134 and R136)

⚠ WARNING

DO NOT raise the patient to a full standing position while using the transport sling (model R134 and R136), otherwise injury can occur.

DO NOT use any kind of material (such as a plastic back incontinence pad or seating cushion) between the patient and sling material that may cause the patient to slide out of the sling during transferring.

Before lifting the patient, make sure the bottom edge of the transport sling is at the base of the spine and the patient's arms are outside the transport sling.

NOTE: For this procedure, refer to FIGURE 7.4.

1. Position the transport sling behind the patient. Make sure the bottom edge of the sling is at the base of the spine. Refer to Detail "A".


NOTE: The labels on the transport sling should be up and facing away from the patient.

2. Lift up one of the patient's legs and position the thigh support under the leg. Refer to Detail "B".
3. Repeat STEP 2 for the opposite thigh support.
4. Pull the thigh support of the transport sling under each thigh of the patient until the thigh supports reach completely around the thigh's of the patient.

NOTE: Make sure the transport sling fabric is smooth underneath the thighs of the patient.

5. Attach the transport sling to the stand up lift. Refer to [Attaching the Slings to the Stand Up Lift](#) on page 27.

DETAIL "A"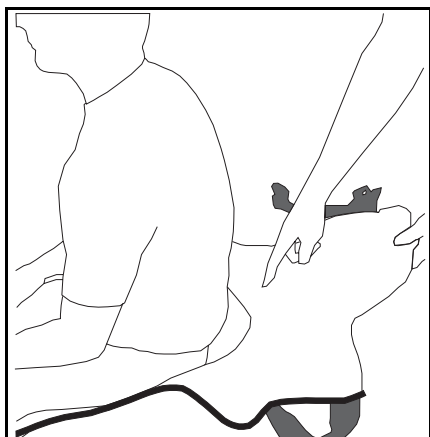

DETAIL "B"

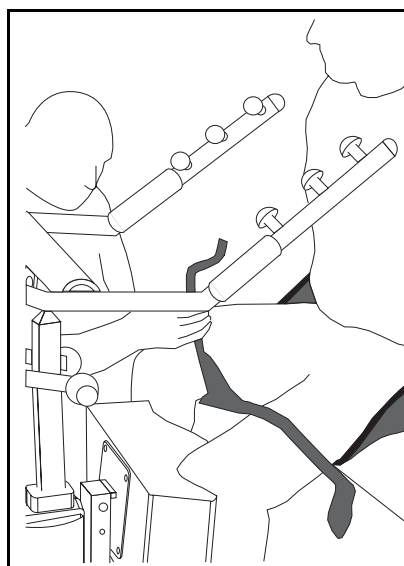
FIGURE 7.4 Positioning the Slings Around the Patient - Transport Slings

Attaching the Slings to the Stand Up Lift

Standing Sling (Model R130 and R131)

NOTE: For this procedure, refer to FIGURE 7.5.

1. Determine the best attachment point for the standing sling on each lift arm. Use the following chart as a guideline to determine attachment point. Refer to Detail "A".

NOTE: The following chart is for reference ONLY.

ATTACHMENT POINT	PATIENT HEIGHT RANGE
Short	under 5 ft 6 in.
Medium	5 ft 6 in. to 5 ft 10 in.
Tall	over 5 ft 10 in.


⚠ WARNING

ALWAYS use the color coded strap on the standing sling closest to the patient while still maintaining patient stability and comfort.


Make sure the color coded strap sits flush against the attachment knob in Detail "B", otherwise injury or damage can result.

NOTE: The color coded straps can be used to place the patient in various positions. The colors make it easy to connect both sides of the sling equally.

2. From the outside of the stand up lift, attach the standing sling to the stand up lift at the attachment point determined in STEP 1. Refer to Detail "A".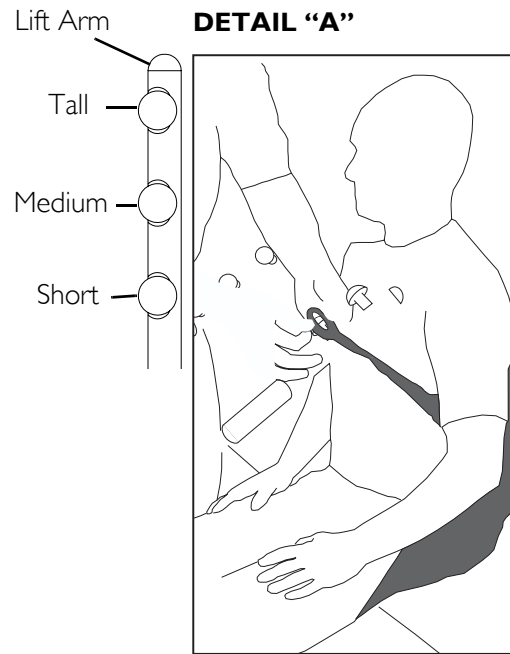

DETAIL "B"

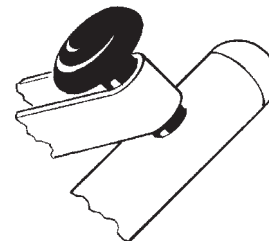DETAIL "C"

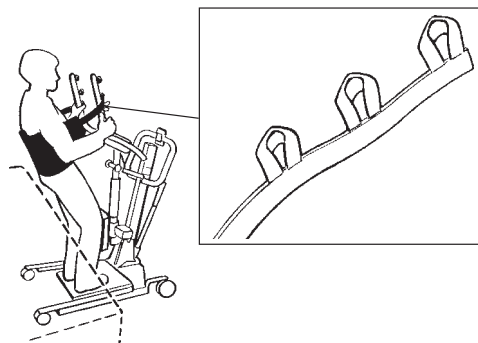
FIGURE 7.5 Attaching the Slings to the Stand Up Lift - Standing Slings

3. Make sure of the following:

- The standing sling is wrapped around the outside of each lift arm.
- The closest color coded strap to the patient is used to maintain stability and comfort.
- The color coded strap sits flush against the attachment knob. Refer to Detail “B”.
- The same color coded strap is used on each side of the standing sling. Refer to Detail “C”.
- The same attachment point is used on each lift arm.

NOTE: If necessary, adjust the attachment point of the transport sling to each lift arm.

4. Once a stable and comfortable position is found for the patient, mark the attachment points and color coded strap used in the patient file for future use.
5. Transfer the patient. Refer to Transferring the Patient in the Stand Up Lift Owner's Manual.

Transport Sling (Model RI34 and RI36)

NOTE: For this procedure, refer to FIGURE 7.6 and FIGURE 7.7 on page 29.

1. Determine the best attachment point for the transport sling on each lift arm. Use the following chart as a guideline to determine attachment point. Refer to Detail “A”.

NOTE: The following chart is for reference ONLY.

⚠ WARNING

ALWAYS use the color coded strap on the transport sling closest to the patient while still maintaining patient stability and comfort.

Make sure the color coded strap sits flush against the attachment knob in Detail “A” of FIGURE 7.7 on page 29, otherwise injury or damage can result.

NOTE: The color coded straps can be used to place the patient in various positions. The colors make it easy to connect both sides of the sling equally.

2. Pull the thigh supports forward and attach to the lower attachment points just above the pivoting knee pad.
3. Attach the top portion of the transport sling to each lift arm at the attachment point determined in STEP 1.

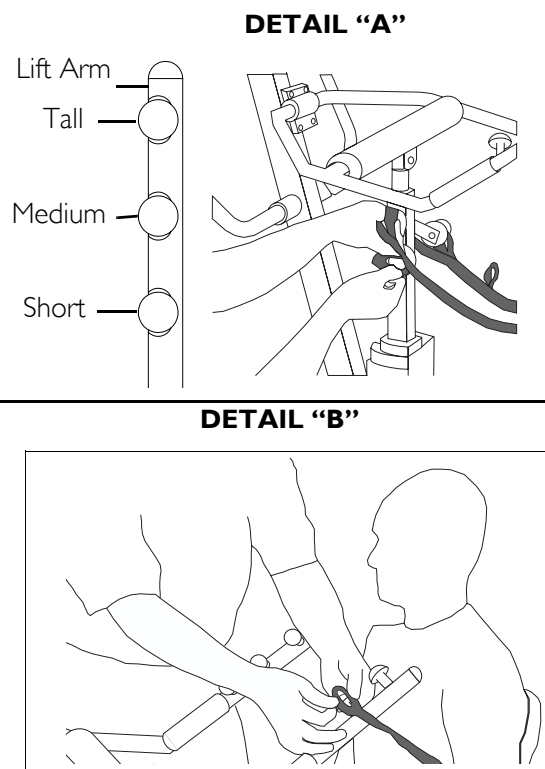
FIGURE 7.6 Attaching Slings to the Stand Up Lift - Transport Slings

4. Make sure of the following:

- The transport sling is wrapped around the outside of each lift arm. Refer to Detail “B” of FIGURE 7.6 on page 28.
- The closest color coded strap to the patient is used to maintain stability and comfort. Refer to Detail “B” of FIGURE 7.6 on page 28.
- The colored coded strap sits flush against the attachment knob. Refer to Detail “B” of FIGURE 7.7.
- The same top color coded strap is used on each side of the transport sling. Refer to Detail “A” of FIGURE 7.7
- The same attachment point is used on each lift arm. Refer to Detail “B” of FIGURE 7.6 on page 28. Refer to Detail “B” of FIGURE 7.7
- The same bottom color coded strap is used on each side of the transport sling.

NOTE: The top and bottom colored straps do not have to be the same.

NOTE: If necessary, adjust the attachment point of the transport sling to each lift arm.

5. Once a stable and comfortable position is found for the patient, mark the attachment points and color coded strap used in the patient file for future use.

6. Transfer patient. Refer to Transferring a Patient in the Stand Up Lift Owner's Manual.

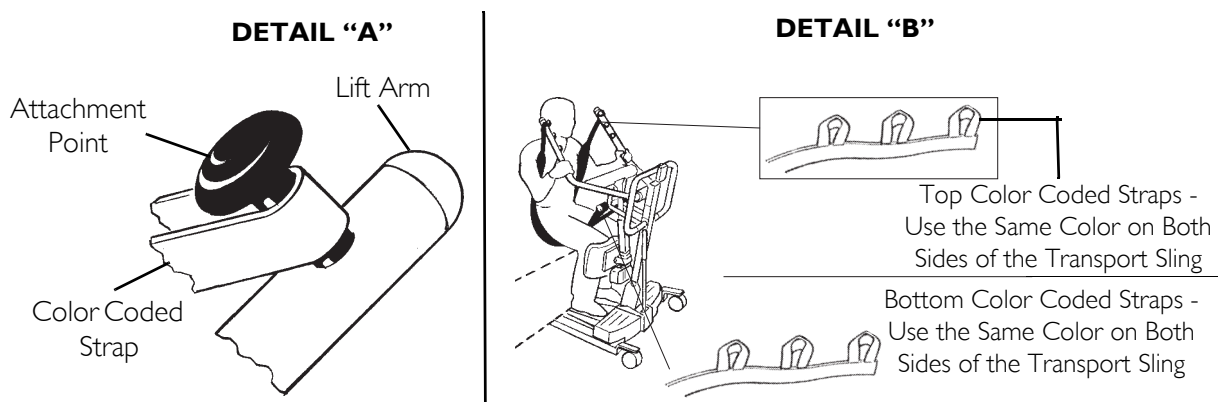
FIGURE 7.7 Attaching Slings to the Stand Up Lift - Transport Slings

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the products manufactured to be free from defects in materials and workmanship for a period of one year from the date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, date of purchase, indicate nature of the defect.

Invacare Corporation will issue a serialized return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable as identification within 30 days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTIONS IN THIS MANUAL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE OR PROVINCIAL LAWS AND REQUIREMENTS.

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All rights reserved. Trademarks are identified by the symbols [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
Invacare Corporation www.invacare.com
© 2007 Invacare Corporation