Leg Extensions and Wheeled Walker Accessories

Assembly, Installation and Operating Instructions

Model Nos. 6263, 6264, 6265, 6266 and 6274

Yes, you can.™

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

SAFETY SUMMARY

To ensure the safe installation of the Leg Extensions and Wheeled Walker Accessories, these instructions **MUST** be followed:

NOTE: Invacare recommends the leg accessory combinations indicated on the chart by an "X".

FRONT WHEELS MODEL NOS.	REAR LEG EXTENSION OPTIONS MODEL NOS.					*KIT MODEL NOS.
	6263 13-inch Sure Glide	6264 10-inch Brakes with Wheels	6265 13-inch Brakes with Wheels	6266 13-inch Leg Extension	6274 10-inch Sure Glide	Kit = Front Wheel Model + Rear Leg Ext. Model
6273 3-inch Swivel	X*		X	X*		*6272 = 6273 + 6263 *6269 = 6273 + 6266
6267 5-inch Swivel	X*		X	X		*6268 = 6267 + 6263
6270 3-inch Fixed		X			X	NA
6271 5-inch Fixed	X		X	X		NA
6277 Tall 3-inch Fixed	X		X	X*		*6278 = 6277 + 6266
Tall Leg Extensions (Kit Only)				X*		*6275 = 6266 + 6266

NOTE: When adding wheeled accessories to the walker, you **MUST** have front wheels installed **BEFORE** adding rear leg extension options.

NOTE: Front wheels can be used in conjunction with the rear glide tips included or by adding the allowable rear wheel option as shown in the chart above.

NOTE: When using 5-inch swivel accessories on the front of the walker, 13-inch rear leg accessories **MUST** be used. Otherwise, height adjustment will be limited.

*NOTE: There are **ONLY** five (5) kits available which allow you to order one (1) number to get both **FRONT** wheels and **REAR** leg extensions. All other available combinations **MUST** be ordered **INDIVIDUALLY**.

GENERAL WARNING

DO NOT install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

STABILITY WARNINGS

Check to make sure that rubber tips and/or plastic glide tips are not ripped, worn or missing. Replace them **BEFORE** using the walker.

DO NOT use rubber tips on front leg extensions when using rear wheel/brake options.

DO NOT use glide tips and rubber tips at the same time.

STABILITY WARNING

Glide tips are not recommended for use on rough surfaces including surfaces that may be experienced outdoors.

Glide tips are used on rear leg extensions **ONLY** to allow the walker to roll easily over indoor carpeted areas without the need to lift the walker off the ground.

Glide tips on rear leg extensions should only be used with front wheel attachments.

Glide tips are **NOT** recommended for use on all four (4) leg extensions at once.

When using rear leg extension with rubber tips or glide tips, all four legs **MUST** be placed on the floor for stability/support. Do not use only the two (2) rear legs for stability/support.

Glide tips do not grip the floor the way rubber tips grip flooring, therefore, extra caution should be observed when in use.

When using glide tips on a walker, short steps should be taken to assure that weight is distributed evenly and directly over walker legs. These precautions will prevent uncontrollable movement of the walker.

INSTALLATION WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that leg extensions are securely locked in position.

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch lower than the front.

Swivel casters should be periodically checked for tightness and/or damage. If loose, tighten securely. If damaged, replace immediately.

After installation, ensure that the walker and accessories are properly and securely locked in place and hardware is tight **BEFORE** using.

Wheel Accessories are **ONLY** to be used on front walker legs with exception of Model Nos. 6264 - 10-inch Rear Wheel Brakes and 6265 - 13-inch Rear Wheel Brakes.

INSTALLING WHEEL ATTACHMENTS/LEG EXTENSIONS (FIGURE 1)

NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Depress the snap buttons on the walker frame to remove the two (2) EXISTING FRONT leg extensions from the walker.

NOTE: The front leg extensions will not be reassembled to the walker when using the wheel attachments/leg extensions.


NOTE: For 3 and 5-inch fixed wheels, position the wheels on the outside of walker.

2. Depress the snap button on the walker frame and install the NEW wheel attachment/leg extension onto FRONT leg of the walker.
3. Repeat STEP 2 for remaining FRONT leg.
4. Adjust the wheel attachment/leg extension to one (1) of the adjustment holes.
5. Perform one (1) of the following:
 - A. Install the REAR leg extension options, if necessary. Refer to INSTALLING REAR LEG EXTENSION OPTIONS in this instruction sheet.
 - B. Remove rear rubber tips and replace with glide tips included. Refer to INSTALLING THE GLIDE TIP in this instruction sheet.

NOTE: When using 10-inch REAR leg extensions, the adjustment ranges are reduced on Model Nos. 6273 and 6267.

6. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker is level.


MODEL NOS. 6270 AND 6271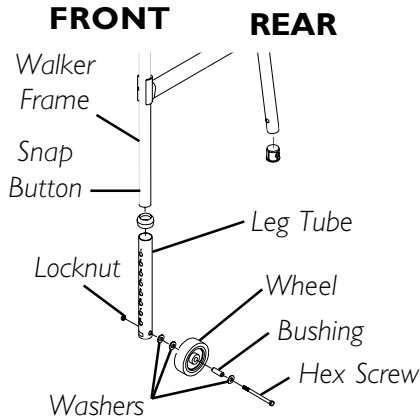


FIXED WHEEL ATTACHMENT

MODEL NOS. 6273 AND 6267

SWIVEL WHEEL ATTACHMENT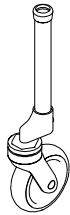

MODEL NO. 6275

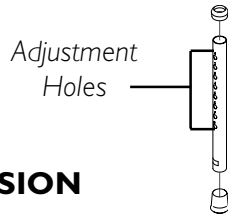LEG EXTENSION

FIGURE 1 - INSTALLING WHEEL ATTACHMENTS/LEG EXTENSIONS

INSTALLING REAR LEG EXTENSION OPTIONS (FIGURE 2)


NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Depress the snap buttons to remove the two (2) EXISTING REAR leg extensions from the walker.


NOTE: The REAR leg extensions will not be reassembled to the walker when using the REAR wheel attachments.

2. Depress the snap button on the walker frame and install the NEW wheel attachment/leg extension onto REAR leg of the walker.
3. Repeat STEP 2 for remaining REAR leg.
4. Adjust the wheel attachment/leg extension to one (1) of the adjustment holes.
5. Repeat STEP 4 for remaining wheel attachment/leg extension.
6. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker sits level.

MODEL NOS. 6264 AND 6265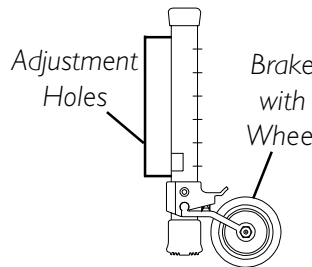

MODEL NO. 6266

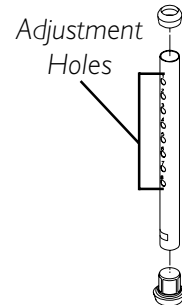MODEL NOS. 6263 AND 6274

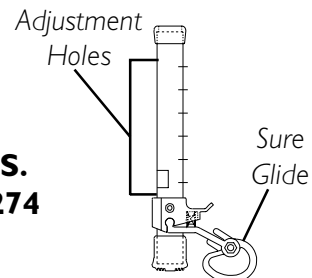
FIGURE 2 - INSTALLING REAR LEG EXTENSION OPTIONS

INSTALLING THE GLIDE TIP (FIGURE 3)

NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Depress the snap buttons on the walker frame to remove the rear leg extension from the walker.
2. Firmly grasp the leg extension and perform one of the following:
 - A. Remove the EXISTING glide tip from the leg extension.
 - B. Remove the rubber tip from the leg extension.

NOTE: When properly installed, the tube edge will rest inside the top edge of the new glide tip. If the tube edge is visible, the tip is not properly installed.

3. Insert the NEW glide tip firmly into the leg extension.
4. Ensure the NEW glide tip is properly installed as shown in DETAIL "A" of FIGURE 3. If necessary, tap new glide tip into position.
5. Depress the snap button and install the leg extension with NEW glide tip onto the walker frame leg. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker sits level.
6. Repeat STEPS 1-5 on remaining rear leg extension.

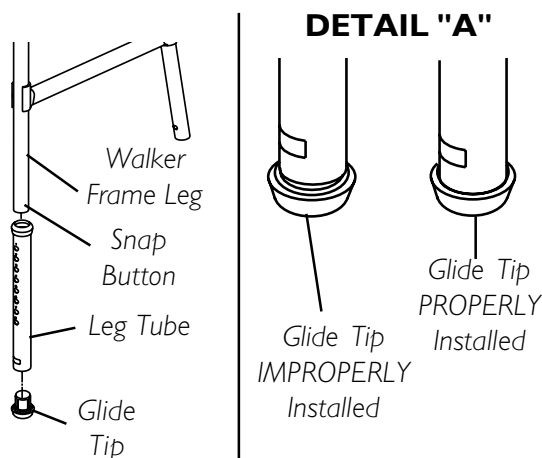
FIGURE 3 - INSTALLING THE GLIDE TIP

REPLACING FIXED WHEELS/SWIVEL CASTERS

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

FIXED WHEELS - MODEL NOS. 6270 AND 6271 (FIGURE 1)

Tools Required:

7/16-inch Boxed End Wrench (QTY. 2)

1. Position one (1) wrench on the hex screw and remove the locknut by using the other wrench and turning COUNTERCLOCKWISE.
2. Remove the washers, wheel, bushing and hex screw that secure the wheel to the leg extension.
3. Inspect washers, bushing and hex screw for wear and/or damage and replace if necessary.
4. To install new wheel, reverse STEPS 1-2.

SWIVEL CASTERS - MODEL NOS. 6273 AND 6267 (FIGURE 4)

Tools Required:

10-inch crescent wrench

1. Grip leg tube and using the crescent wrench, apply to hex nut.
2. Loosen swivel caster and unscrew from swivel caster housing.
3. Inspect swivel caster housing and leg tube for any damage. If damaged, replace immediately.
4. Insert new swivel caster by screwing it into swivel caster housing and tightening with the crescent wrench.

TIGHTENING SWIVEL CASTERS (FIGURE 4)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove swivel wheel attachments from front legs by depressing snap button.
2. Hold one (1) swivel wheel attachment with the wheel up as shown in FIGURE 4.

3. Turn hex nut **CLOCKWISE** with a wrench to tighten swivel caster.
4. Repeat **STEPS 2-3** for remaining swivel wheel attachment.
5. Depress the snap buttons and slide the swivel wheel attachments onto the front legs of the walker. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker sits level.

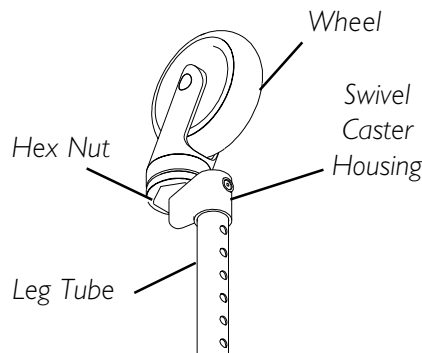
FIGURE 4 - TIGHTENING SWIVEL CASTERS

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of the product for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges

incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare
Invacare Corporation www.invacare.com