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
Turn-Q Plus

Automatic Turning Mattress
Low Air Loss Therapy System

Automatisk Vendemadrass
Low Air Loss Terapi System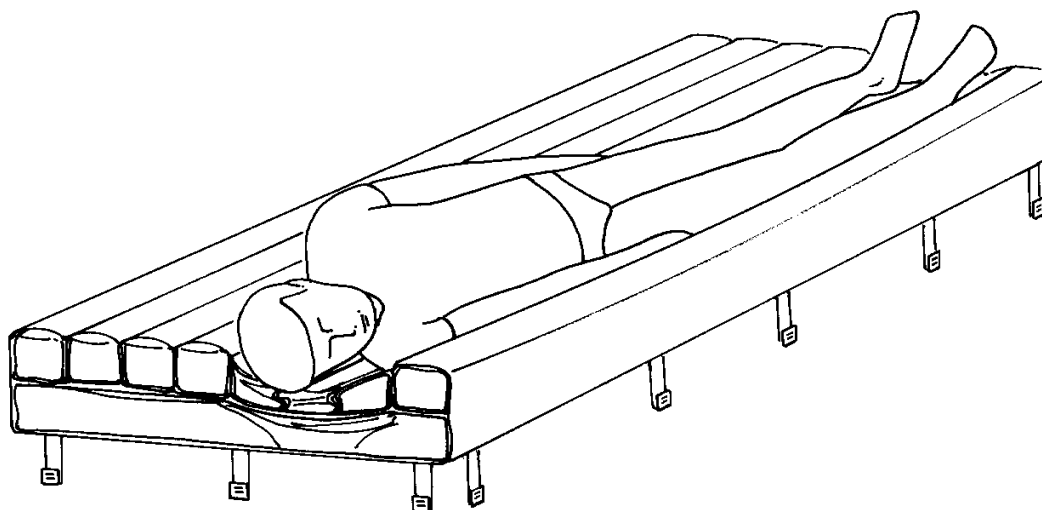

DEALER: THIS MANUAL MUST BE GIVEN TO THE USER OF THE TURN-Q PLUS MATTRESS SYSTEM.

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USER: BEFORE USING THIS TURN-Q PLUS MATTRESS SYSTEM, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE.

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S
P
E
C
I
A
L

N
O
T
E
S

WARNING/CAUTION notices used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

WARNING

W
A
R
N
I
N
G

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL IF APPLICABLE BEFORE ATTEMPTING TO USE THIS EQUIPMENT. OTHERWISE INJURY OR DAMAGE MAY RESULT.

SAVE THESE INSTRUCTIONS

TABLE OF CONTENTS

SPECIAL NOTES	2	CLEANING AND DISINFECTING	15
SAFETY SUMMARY	4	RECOMMENDED CLEANING MATERIALS ...	15
SPECIFICATIONS	5	CLEANING AT THE PATIENT SITE	16
FEATURES		CLEANING AND DECONTAMINATING	
QUICK REFERENCE -		SPILLS OF BODY FLUIDS	16
TURN-Q PLUS	7	FIELD CLEANING OF POWER UNITS	16
UNPACKING	11	LAUNDRY PROCEDURE	16
UNPACKING AND INSPECTION	11	MATTRESS DISASSEMBLY	16
COMPONENTS SUPPLIED IN THE POWER		PRESOAK	16
UNIT PACKAGE	11	WASHING CYCLE	16
COMPONENTS SUPPLIED IN MATTRESS		MACHINE DRYING METHOD	16
ASSEMBLY PACKAGE	11	MATTRESS COMPONENTS	16
PREPARING FOR OPERATION	11	TRANSPORT MAT	17
BED PLACEMENT AND PREPARATION	11	PACKAGING OF LAUNDERED SYSTEM	
POWER UNIT INSTALLATION	11	COMPONENTS	17
PRE-ASSEMBLED MATTRESS		MATTRESS ASSEMBLY	17
INSTALLATION	11	NON-STANDARD OPERATION	18
AIR HOSE CONNECTION	12	POWER UNIT RELOCATION	18
THERAPY PAD INSTALLATION	12	NARROW WIDTH BEDS	18
POWER CORD	12	MAINTENANCE	18
POWER TO THE SYSTEM	12	AIR FILTER REPLACEMENT	18
PATIENT SETUP	13	TROUBLESHOOTING	19
PATIENT PLACEMENT AND		SPARE PARTS AND OPTIONAL	
MANAGEMENT	13	EQUIPMENT	19
HEAD RAISED POSITION	13	APPENDIX	20
PATIENT TRANSFER FROM/TO A		SYSTEM COMPONENTS CLEANING	
GURNEY	14	CHART	20
PATIENT TRANSFER FROM/TO A		INSPECTION AND TEST REPORT	21
WHEELCHAIR	14	SOURCES FOR GERMICIDAL	
ADJUSTABLE WIDTH SIDERAILS	14	CHEMICALS	23
SYSTEM SERVICING/CLEANING AND		INSPECTION, MAINTENANCE AND TEST	
DISINFECTING/ LAUNDRY	14	INTERVALS	23
DRESS STANDARDS	14	LIMITED WARRANTY	24
UNIT SETUP	14		
ROUTINE MAINTENANCE	14		
COLLECTING	15		

SAFETY SUMMARY

**S
A
F
E
T
Y

S
U
M
M
A
R
Y**

WARNING

CONTRAINDICATIONS

Always consult the patient's physician before using the Turn-Q Plus system. Certain medical conditions preclude the use of an automatic turning bed e.g., unstable spinal cord, etc.

PHYSICIAN ORDERS

Always receive orders for patient treatment and turning times from the patient's physician before placing them on the Turn-Q Plus system.

INSTALLATION

The Turn-Q Plus **MUST** be installed on medical beds frames or medical-grade homecare bed with full length side rails. After placing the patient on the mattress, verify the side rails provide adequate protection for the patient. The side rails **MUST** be in the raised position, whenever a patient is on the bed.

A few bed frames have controls on the footboard that may be obstructed by the Power Unit. It will be necessary to relocate the Power Unit. See **NON-STANDARD SECTION** for alternate Power Unit locations.

Before placing a patient on the bed, be sure Air Hoses and Power Cord are clear of moving bed components. Operate all motorized functions through their full range of motion to be certain there is no pulling, interference, or pinching.

The Therapy Pad (supplied with each mattress) is essential for skin moisture control. **DO NOT** place a patient directly on the Mattress without a Therapy Pad. The Therapy Pad protects the Mattress and components from contamination.

SMOKING IN THE PATIENT ENVIRONMENT

Turn-Q Plus systems use room air for circulation through the mattress. Smoking by the patient is **NOT** permitted. A cigarette can burn a hole in the bed surface and cause deflation of the bed. Also, patient clothing, bed sheets, etc. may be combustible and cause a fire. Smoking by visitors in the room will contaminate the system. Therefore, visitor smoking is **NOT** permitted.

OXYGEN AND ANESTHESIA EQUIPMENT

Consult expert before use. Explosion risk if used with flammable anesthetics. There is a possible fire hazard when used with oxygen administering equipment other than nasal mask or 1/2 bed tent type. The oxygen tent should **NOT** extend below mattress support level.

ELECTRICAL

Before performing any maintenance to the Power Unit, disconnect the Power Cord from the wall outlet. Refer servicing to qualified personnel only. Grounding reliability depends upon a properly grounded wall outlet. **DO NOT** use extension cords. **ALWAYS** use the 18 ft. (5.5 meter) hospital-grade Power Cord provided.

This unit is equipped with a battery back-up to retain program settings. In the event of a power outage, check the unit to confirm that the settings have been retained. If the unit has not returned to its' default settings, contact your dealer for service.

SERVICING

Other than the Air Filter element, the Turn-Q Plus contains no user-serviceable parts. **DO NOT** attempt to open the Power Unit - refer all repair questions to Customer Service. The Turn-Q Plus **MUST** be maintained, inspected and tested according to the schedule in **TROUBLESHOOTING SECTION** to deliver consistent performance.

If your system requires factory service, contact Customer Service the address on the back of this manual for a return shipment authorization number. Package systems in the original shipping box. Clean all system components before shipment - unsanitary systems will be returned without servicing.

SPECIFICATIONS (USA ONLY)

TurnQPLUS	
AIR TUBES	
Washable:	Yes
Waterproof:	Yes
Number:	Eight (8) cylindrical tubes, One (1) lower mattress
Pressure Sections:	Two (2) side, Three (3) left, Three (3) right, One (1) lower
Tube Seam Bonding:	RF Welded
PATIENT/MATTRESS INTERFACE	
Vapor Permeable:	Yes
FILTERS	
Air Filter:	Yes
WEIGHTS/CAPACITY	
Patient Weight:	500 lbs. (227) kg Maximum
System Weight:	Power Unit: 32 lbs. 14.5 kg Mattress: 28 lbs. 13.0 kg Total: 60 lbs. 27.5 kg
AIR BLOWER	
Service Life (est.):	24,000 hrs.
DIMENSIONS	
Mattress:	82 x 37 in. 208 x 94 cm Standard Width 82 x 32 in. 208 x 82 cm Narrow Width
MATTRESS SANITIZATION	
Fluid Entrapment:	No (Seamless)
Detergent. Resist:	Yes
PATENTS	
U.S. and Foreign Patent:	U.S. 5,375,273, U.S. 5,745,942 EPO 0673217
ELECTRICAL	
Power:	115 VAC 50/60 Hz, 2A (Operating)
SAFETY	
Listing Agency:	Underwriters Laboratories
Listing Specification:	UL 544
Fire:	Cal 106

SPECIFICATIONS (UK ONLY)

**S
P
E
C
I
F
I
C
A
T
I
O
N
S**

TurnQPLUS	
AIR TUBES	
Washable:	Yes
Waterproof:	Yes
Number:	Eight (8) cylindrical tubes, One (1) lower mattress
Pressure Sections:	Two (2) side, Three (3) left, Three (3) right, One (1) lower
Tube Seam Bonding:	RF Welded
PATIENT/MATTRESS INTERFACE	
Vapor Permeable:	Yes
FILTERS	
Air Filter:	Yes
WEIGHTS/CAPACITY	
Patient Weight:	227 kg Maximum
System Weight:	Power Unit: 14.5 kg Mattress: 13.0 kg
AIR BLOWER	
Service Life (est.):	24,000 hrs.
DIMENSIONS	
Mattress:	208 x 94 cm Standard Width 208 x 82 cm Narrow Width
MATTRESS SANITIZATION	
Fluid Entrapment:	No (Seamless)
Detergent. Resist:	Yes
PATENTS	
U.S. and Foreign Patent:	U.S. 5,375,273, U.S. 5,745,942 EPO 0673217
ELECTRICAL	
Power:	207 - 253 V, 50 Hz, 3A Normal, 5A Max. (Operating)
Fuses:	IEC 127, 250 VAC Bus Fuses, 5A - 5mm x 20mm, Time Lag
Classification:	Class I Type B
Ingress Protection:	IPXO
SAFETY	
Listing Agency:	CE Underwriters Laboratories

FEATURES

QUICK REFERENCE - TURN-Q PLUS

Low Air Loss Therapy

Since its introduction in the early 1970's, Low Air Loss Therapy has become an increasingly important and effective treatment modality for pressure ulcers. Low air loss therapy has been shown to reduce the incidence of skin lesions induced by maceration, shear and the loss of capillary blood circulation. It also minimizes the formation of bacterial growth sites in sedentary or bed-bound patients.

Low air loss therapy maintains peripheral circulation by distributing the patient's weight over several low pressure mattress sections. The even distribution of pressure on the skin prevents capillary closure (capillary occlusion typically occurs at skin interface pressures greater than 32 mm Hg), tissue viability is maintained around bony prominences, such as the trochanter and coccyx. In addition, controlled air loss from the mattress surface removes perspiration and other bacteria supporting fluids. A polyester filled, low-friction therapy pad is located beneath the patient. The top surface of the pad, has a high vapor permeability that regulates moisture removal without the drying effect of a direct air stream.

Lateral Turning Therapy

Turning immobile patients laterally at least every two (2) hours (Q2H) is a fundamental tenet of basic patient care. Patient turning is particularly critical in the prevention and treatment of decubitus ulcers. In addition, frequent lateral turning is often ordered to promote pulmonary toilet. Turning helps reduce complications such as pneumonia in patients with head trauma and spinal injuries. Lateral patient turning may increase circulation, urinary output, peristalsis, venous flow in the legs, and remove lung fluid density. Improvements in patient skin tone and general well being may be observed in some cases. Whether in the home, long term care or hospital, automatic patient turning with the Turn-Q Plus Automatic Turning Mattress ensures that your patients are turned according to doctor's orders.

Low Air Loss enhances the therapeutic effectiveness of the Turn-Q Plus mattress by managing the micro-climate at the patient's skin. The air flow keeps the patient's skin naturally dry, reducing skin friction and shear. Excess moisture evaporates without risk of dehydrating the skin or moist wound dressings.

About the Turn-Q Plus

The Turn-Q Plus is a deep flotation, true low air loss therapy system that can directly replace specialty hospital beds. It is portable, and readily attached to most hospital and nursing home beds. The Turn-Q Plus replaces the mattress on standard medical beds. It provides continuous flotation in all turning modes assuring that patients do not "bottom out" on a hard bed surface. The mattress will maintain the patient on an air flotation surface during a power failure. When power is restored, the system will resume operation without intervention by the care giver.

The Turn-Q Plus gently turns patients between 30 and 40 degrees of lateral rotation to the left or right of center (supine). Operation is fully automatic and the turn interval can be set between four (4) minutes and two (2) hours. There are seven (7) selectable turning modes, three (3) fixed position and four (4) automatic turning. The patient is not forced to lie on a trochanter pressure ulcer because turning positions can be set to exclude the right or left turns.

Products Covered by this Manual

This manual details the features, and operating instructions for the Turn-Q Plus Portable Low Air Loss Therapy System. It is important that the end-user thoroughly read and understand the material contained in this document before attempting to operate the equipment.

QUICK REFERENCE - TURN-Q PLUS (Cont.)

Front Panel Features

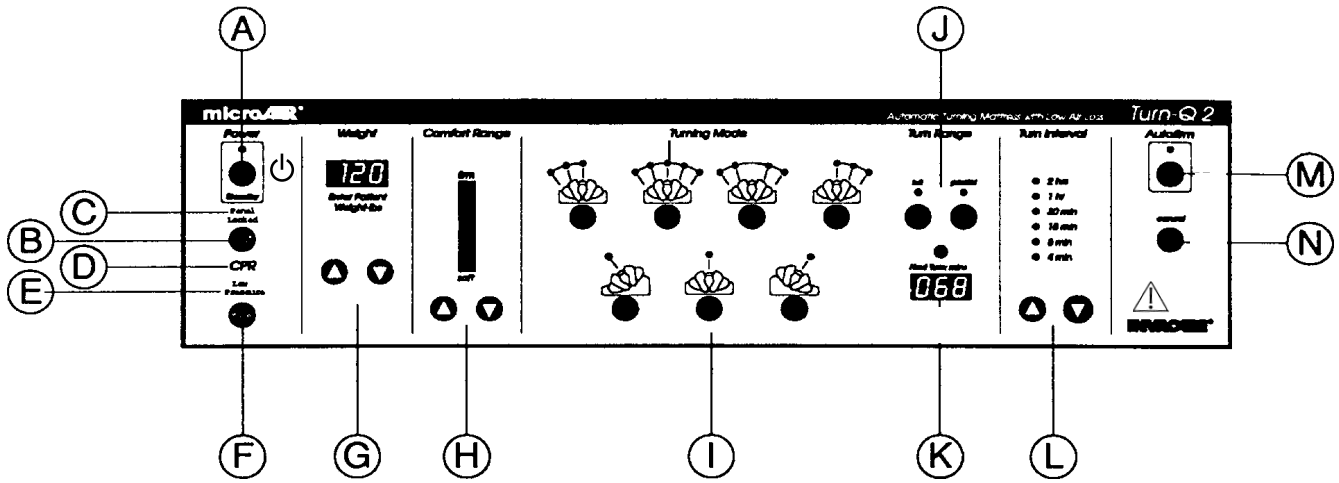
FIGURE 1 - TURN-Q PLUS CONTROL PANEL

- (A) **Power Button** Press the button to turn “ON” the Power Unit. Press the button again to turn “OFF” the Power Unit. The control panel power is always “ON”, as indicated by the STANDBY light. The Power Button turns “ON” system operation.
- (B) **Lock Out Button** Press the Lockout Button to cancel all other control panel functions. Press the button again to restore control panel operation.
- (C) **Panel Locked** Indicator lights to notify the Panel Locked mode is engaged.
- (D) **CPR** Indicator lights to notify that CPR mode is engaged. Pull the handle on the left side to engage CPR mode.
- (E) **Low Pressure** Indicator lights to notify that mattress pressure is too low for patient support. Check the system for leaks or damage. Check the Air Filter.
- (F) **Mute Button** When the Low Pressure indicator turns “ON”, a sonic alarm draws attention to the condition. Press the Mute Button to turn “OFF” the alarm.
- (G) **Weight** Press the **UP** or **DOWN** button to set the patients weight during patient setup. This power setting provides a mattress pressure suitable to turn the patient 30-40 degrees from supine depending on total weight.
- (H) **Comfort Range** Press the **UP** or **DOWN** buttons to alter the relative firmness of the mattress, if the patient requires a change. The normal setting is the center of the display.
- (I) **Turning Mode** Press the button that corresponds to the desired Turning mode. The indicator arcs above the button display either full or partial turn angles.
- (J) **Turn Range** Press the button that activates either Full or Partial Turning mode. Full turn is normal. Partial turn is used to acclimate new patients to the motion.
- (K) **Next Turn Display** The Next Turn display indicates the number of minutes until the next turn starts.
- (L) **Turn Interval** Press the **UP** or **DOWN** button to set the Turning Interval time. The turning motion starts after each time interval. Example: In Left/Right mode set to 30 minutes Turn interval, the patient will turn Right 30 minutes then turn Left 30 minutes continuously.

QUICK REFERENCE - TURN-Q PLUS (Cont.)

(M) Auto-Firm Button Press the Auto-Firm Button to engage auto firm. The Mattress will become firm for nursing procedures. An indicator lights to notify the bed is in Auto-Firm mode. A timer will cancel auto firm in 30 Minutes. Should the nurse be called away, an unattended patient will not be left on a firm surface.

WARNING

DO NOT leave an unattended patient on auto-firm to prevent excessive pressure. Auto firm also increases the risk of the patient falling out of bed.

(N) Cancel Button Press the Cancel Button to cancel Auto-Firm mode.

 **- ATTENTION: Refer to Operation Manual**

Side Panel Features

SIDE PANEL FEATURES - LEFT.

- (O) CPR Handle** Pull CPR Handle to deflate the Mattress rapidly for CPR mode. Push the handle closed to turn Power Unit back “ON” and return system to operation.
- (P) Air Hose Couplers** Five (5) color coded quick connect fittings that connect to the hoses.
- (Q) Cord Connector** Push in the cord plug to install the Power Cord. Use only the 18 ft. (5.5 meter) hospital grade cord to ensure safety ground. The cord connector has built-in fuses. For fuse specification, refer to the SPECIFICATION Section of this manual.

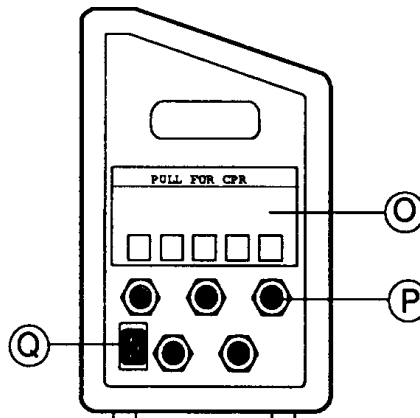
FIGURE 2 - TURN-Q PLUS LEFT SIDE PANEL

SIDE PANEL FEATURES - RIGHT.

- (R) Hour Meter** Inside the handle opening is an elapsed hour meter that records the total time the Power Unit has been operating in hours. Use the timer to change the air filter every 1,500 hours.
- (S) Filter Door** Open the Filter Door to change the Air Filter. Rotate counterclockwise to open.
- (T) Low Pressure Alarm** Alarm to notify that mattress pressure is too low for patient support. Check the system for leaks or damage. Check the Air Filter.

QUICK REFERENCE - TURN-Q PLUS (Cont.)

SIDE PANEL FEATURES - RIGHT (Cont.).

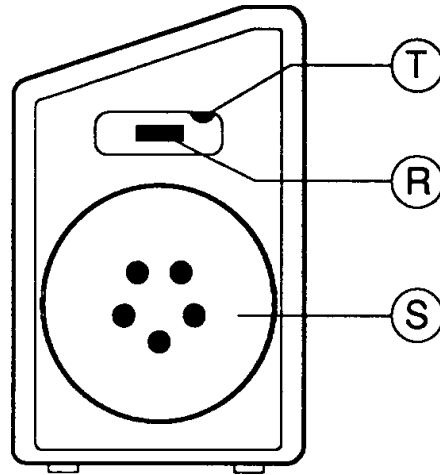
FIGURE 3 - TURN-Q PLUS RIGHT SIDE PANEL

REAR PANEL FEATURES.

- (U) Mounting Brackets Narrow and wide swing-out Brackets fit most bed footboards.
- (V) Chassis Ground Chassis grounding point to equalize potential.

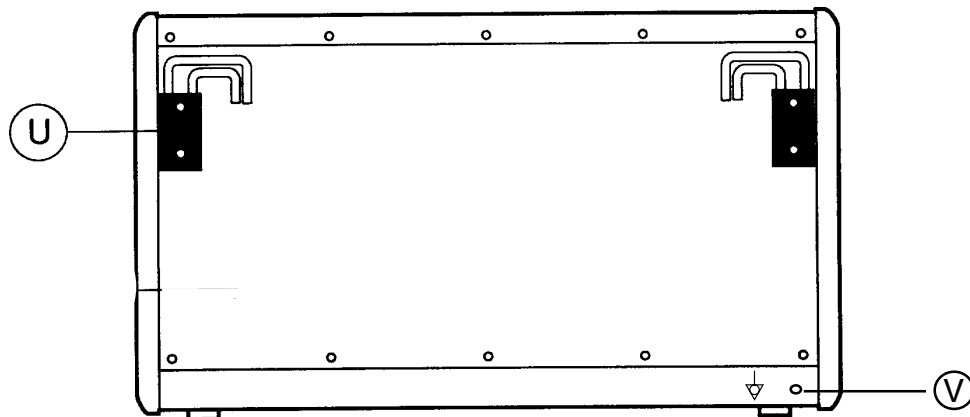
FIGURE 4 - TURN-Q PLUS REAR PANEL

UNPACKING

Unpacking and Inspection

CAUTION

The mattress assembly **MUST** be at room temperature **BEFORE** it is unrolled. Attempting to unroll the mattress assembly when it is cold may cause "splits" or "cracks" to develop in it. These "splits" or "cracks" will **NOT** allow the mattress assembly to inflate/function properly. Therefore, when the mattress assembly is transported in the winter, it is important to wait approximately 24 hours **AFTER** transport **BEFORE** unrolling it. **ANY** attempt to unroll the mattress assembly when it is cold will void the warranty.

Each Turn-Q Plus is shipped in two (2) boxes - one (1) contains the Power Unit and the other contains the Mattress Assembly. Verify that no components are damaged or missing. Report any damage to the transport carrier immediately.

Components Supplied in the Power Unit Package

- (A) - One (1) Power Unit
- (B) - One (1) Operation and Maintenance Manual
- (C) - One (1) Power Cord

Components Supplied in the Mattress Assembly Package

- (A) - One (1) Mattress Assembly with Air Hoses Yellow, Red, Blue, and Green
- (B) - One (1) Nylon Duffel Bag

PREPARING FOR OPERATION

Turn-Q Plus systems are designed for simple, reliable operation. Once the initial adjustments are made, little attention is required other than routine service.

Bed Placement and Preparation

The bed **MUST** be a standard hospital bed or medical-grade home care bed.

WARNING

For patient safety, the bed **MUST** have side rails and the rails **MUST** be in the up position when the patient is lying on the bed.

Remove the original, bed mattress and place it in storage.

Power Unit Installation

The standard power unit installation is on the upper edge of the footboard. Select the appropriate width swing-out mounting bracket (two (2) sizes are available on the rear panel) and place the power unit on the footboard. Run the power cord under the center of the bed.

Pre-Assembled Mattress Installation

The mattress and therapy pad are pre-assembled when shipped from the factory. In actual service, the pre-assembly point should be at the cleaning facility or dealer location.

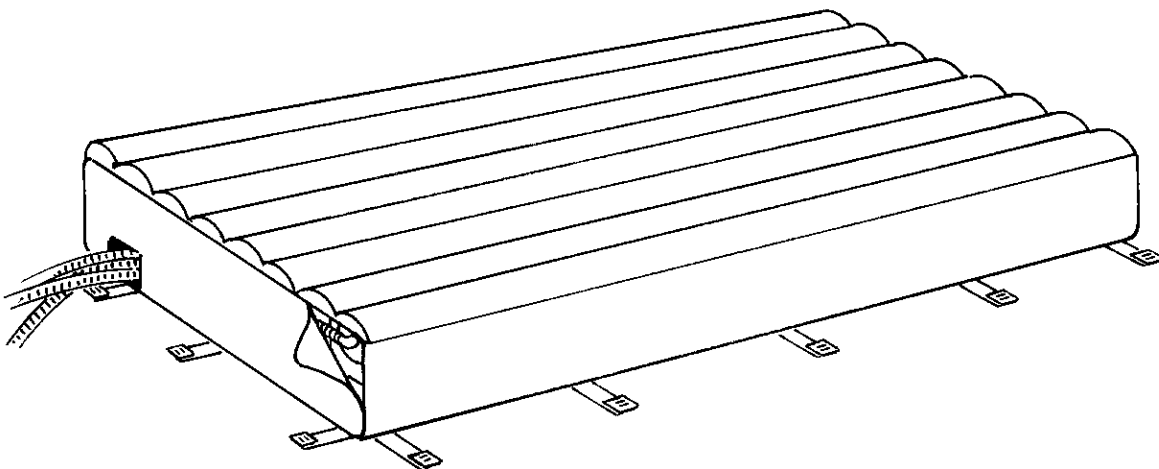
FIGURE 1 - MATTRESS COMPONENTS WITHIN THE MATTRESS HOUSING

1. To install the Mattress, lower the side rails to facilitate installation.
2. Roll out the Mattress onto the bed frame.
3. Secure the tie-down straps to the ends and sides of the bed frame at convenient locations.

NOTE: Make certain that the side rails can move freely.

4. Thread the strap ends through the buckles provided (FIGURE 2) and take up the excess slack.

NOTE: A minimum of two (2) straps are required in the center and two (2) each at the head and foot.

5. To release strap tension, lift the tab on the buckle.

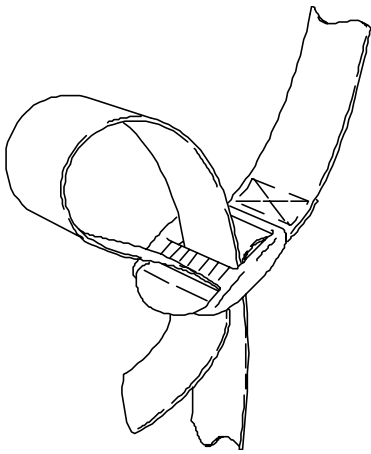
FIGURE 2 - PROPER METHOD OF THREADING FRICTION BUCKLES.

Air Hose Connection

Connect the color-coded hoses to the Power Unit. Push in the snap ring at the Power Unit. Insert the hose fitting. Release the snap ring to lock the fitting if the ring does not snap back, push the hose fitting in until it does.

Therapy Pad Installation

Starting at the foot end of the bed, lay the Therapy Pad (quilted side down) on top of the Mattress. Connect the Therapy Pad snaps to the corresponding snaps on the straps located on the Mattress (near the bed frame). A patient should never be placed directly on the Mattress without a Therapy Pad.

A loose (non-fitted) cotton sheet between the patient and Therapy Pad is optional. It may increase skin friction and shear if the sheet is taut.

Power Cord

1. Plug the Power Cord into a wall outlet.
2. Verify that Air Hoses and Power Cord are clear of moving bed components.
3. Operate all of the bed's motorized functions through their full range.
4. Check for pulling, interference or pinching, and correct any problems.

Power to the System

1. Press the Power Button to start the Power Unit.
2. The Mattress will start to fill slowly.
3. Press the Auto-Firm Button to inflate the mattress quickly. Wait about three (3) minutes and press the Cancel Button. If the Mattress does NOT inflate, see TROUBLESHOOTING SECTION for suggestions.

PATIENT SETUP

1. Set the patient weight with the Weight selection buttons (G).
2. Press the "Center" static Turning Mode button (I).
3. Ask the nursing staff to place the patient on the bed.
4. Center the patient on the bed from side-to-side and head-to-foot.
5. Use a regular pillow under the head for comfort and to prevent the head from rolling excessively.
6. Raise side rails.
7. Press the desired Turning Mode button (O).
8. Press the Full Turn Range button (J).
9. Set the Turn Interval time with the up/down buttons (L).
10. If the patient feels the bed is too soft/hard press the up/down Comfort Range buttons accordingly (H).

NOTE: Special positioning may be required with contracted patients to provide comfortable positions.

5. Verify the patient's limbs assume natural positions and are not pinched.
6. Use pillows, foam or sheepskin appliances as required to cushion potential body-to-body contact areas.
7. Position the patient's arms to protect the elbows from abrasion or excess pressure. It may be necessary to use elbow appliances or pillows.
8. If the patient's knees or ankles contact or rub during turning, it may be necessary to use a pillow to separate these points.
9. Set time as per instructions from patient's physician.

Patient Placement and Management

1. After placing the patient, make certain no objects will fall under the patient when turning, such as feeding tubes, IV's etc.
2. Use a regular pillow to help support and stabilize the patient's head. It is particularly important in ventilator patients to keep their head from moving excessively during turning.
3. Before leaving the patient in any of the automatic turning positions, observe the patient through at least two (2) full turning cycles.
4. Set time to eight (8) minutes so that the patient will be fully turned during each cycle.

Head Raised Position

When elevating the head of the bed, elevate the knee section first. The Therapy Pad is slippery to minimize skin friction. Head elevation without knee elevation may cause the patient to slide toward the foot of the bed.

The head of the bed should be maintained at the lowest degree of elevation consistent with medical conditions (AHCPR Pub. 92-0047, P.24). The lowest skin pressure is maintained with 0° to 20° head raise angle. Angles above 30° place more body weight over the seat area, reducing pressure relief. Instruct the care-giver to avoid head angles over 30° for extended periods, unless medically necessary (U.S. Department of Health and Human Services publication No. 92-0048).

If the head of the bed is raised more than 30 degrees, DO NOT leave the patient in automatic rotation.

Press the center Static Button (I). The Static Positions are for nursing procedures or when manual control is needed. A patient left in Static Position should be turned manually at least every two (2) hours.

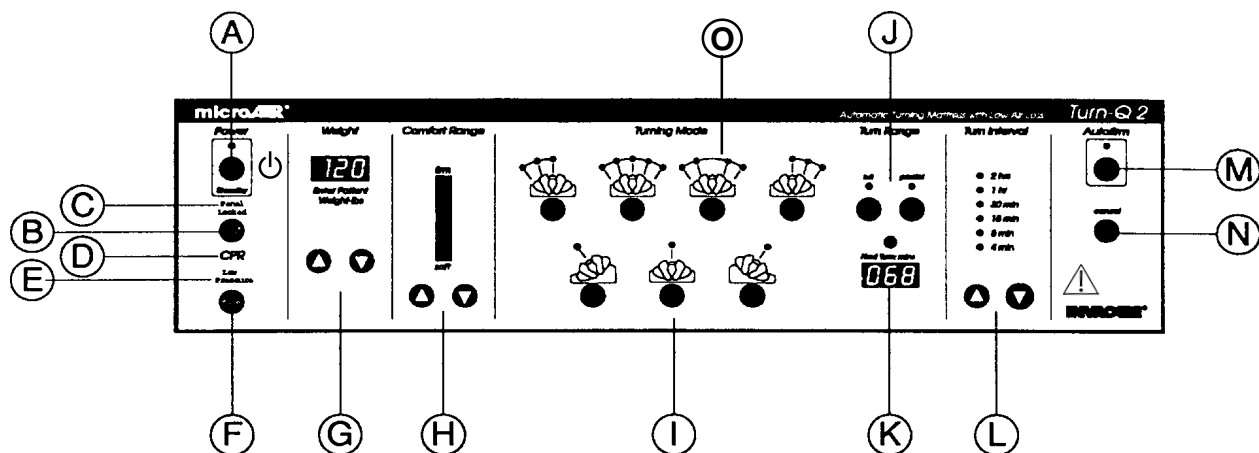
FIGURE 1 - ADJUSTING THE CONTROL PANEL

Patient Transfer from/to a Gurney

1. Press Auto-Firm to achieve maximum mattress pressure.
2. Raise or lower the bed to match the gurney height.
3. In two (2) to three (3) minutes the mattress will become very firm so the patient can be slid onto the gurney.

WARNING

Always engage bed and gurney wheel locks and secure moving parts of bed and gurney before patient transfers.

4. Press cancel to return mattress pressure to the original setting.

Patient Transfer from/to a Wheelchair

1. Press the POWER Button to turn “OFF” the unit to deflate the mattress so the patient is on the same level as the wheelchair.
2. After transferring to a Turn-Q Plus, have the patient lie on their back and raise the bed side rails to the safety position.
3. Press the POWER Button to turn “ON” the power.
4. Press the Auto-Firm Button and wait until the mattress is inflated.
5. Press the Cancel Button.

Adjustable-Width Side Rails

Some home-care beds have adjustable-width side rails.

WARNING

Set the side rail width so the rails touch the mattress. This is necessary to prevent the patient slipping into the gap between the mattress and a side rail.

**SYSTEM SERVICING /
CLEANING AND
DISINFECTING / LAUNDRY**

SYSTEM SERVICING PROCEDURES

The following describes general procedures concerning system setup, maintenance and collection. It is intended as a guideline.

Dress Standards

Standard clothing consists of street clothes, a lab coat, and two (2) pairs of gloves (one (1) pair is a spare). Protective clothing may include a mask, waterproof gown, face shield or other equipment. Gloves should be worn when touching the patient, dressings or body fluids.

Proper procedures for donning and removal of protective clothing should be observed.

Unit Setup

The Turn-Q Plus system is made patient-ready at the dealership before delivery to the installation site. The mattress is pre-assembled and the power unit has been tested and serviced according to the **MAINTENANCE Section** in this manual.

Ask the caregiver or nursing staff what isolation precautions are required when performing adjustments with the patient on the bed. Typically, if protective clothing is required, the institution will provide it to all health care workers.

Routine Maintenance

Disposable gloves are worn on all maintenance visits. They are disposed of after visiting each patient. Hand washing is required before and after gloves are worn.

For lighter soil, the therapy pad can be cleaned by topical wipe down with a PH balanced soap and water solution normally used to wash the patient. Rinse with water. If grossly soiled, the therapy pad should be replaced, cleaned, and decontaminated.

Filter maintenance should be scheduled. Consult the **MAINTENANCE Section** for details. Typically, this includes changing the air filter element according to the prescribed schedule.

Collecting

1. When collecting the Turn-Q Plus system, put on gloves and other protective equipment if required.
2. Press the POWER Button to turn “ON” the power unit.
3. Check the underside of the therapy pad and mattress folds for sharps (e.g. scissors, needles, etc.). These should be removed and discarded before proceeding with any further cleaning or disassembly.
4. Examine the surface of the power unit, therapy pad and mattress assembly components for visible blood or body fluids. If blood is present, decontaminate. Ensure that bedding is bagged and labeled before removal from the patient site.
5. If blood is not present, remove any gross soil from the Therapy Pad with paper towels.
6. Dispose of towels according to institutional policy.
7. Wipe the top (patient) surface down with a dilute detergent solution of quaternary cleaner disinfectant or other germicidal detergent solution, using a clean sponge or paper towel.
8. Press the POWER Button to turn “OFF” the power unit.
9. Clean any soiled areas of the power unit and hose fittings with detergent solution.
10. Disconnect hoses from the power unit and allow air to vent from the mattress.
11. Gently roll up the mattress with minimal handling and agitation.
12. Verify the therapy pad surface is inside the roll.
13. Place the used bedding in a laundry transport bag, appropriately labeled. Seal the bag before removing it from the room.

WARNING

NEVER place used bedding or soiled components in the duffel bag because it will become contaminated.

14. Used gloves should be disposed of properly according to institutional policy.
15. Move the entire system from the patient’s room to the transport vehicle.

CLEANING AND DISINFECTING

All equipment should be inspected. Any item that is visibly soiled with the patient’s blood or other body fluids should be properly cleaned or removed.

Recommended Cleaning Materials

Disinfectant chemicals should be registered, hospital-grade, and have effectiveness claims for destroying or inactivating harmful microorganisms. Use only in the manufacturer’s recommended dilution. Never mix different chemicals together in the same container.

Phenolic disinfectants are supplied in highly concentrated form. They are tuberculocidal when used at the recommended dilution. They provide a high level of disinfection. It is also effective in decontaminating blood spills.

Quaternary cleaner disinfectants are supplied in highly concentrated form. They are usually capable of cleaning, deodorizing and disinfecting in a single operation.

They are especially suitable for cleaning equipment at the patient site where low-level disinfecting or odor control is indicated.

Laundry disinfectant presoaks can be either phenol or quaternary chemicals. These products are suitable for adding to water as a presoak.

Cleaning At the Patient Site

Whenever the therapy pad is visibly soiled, cleaning is done with the patient off the bed. Use a PH balanced soap and water solution normally used to wash the patient. Rinse with water. Because of potential risk of infectious exposure, cleaning with the patient on the bed is not recommended.

Cleaning and Decontaminating Spills of Body Fluids

Because of the risk of contracting HIV and hepatitis from contact with blood spills, cleaning MUST be done with protective clothing. Gloves MUST be worn during cleanup and hands should be washed thoroughly after completion. First, all visible soil should be removed with disposable paper towels. Next, the area should be scrubbed with freshly prepared effective phenolic detergent disinfectant solution. In general, chemical germicides that are registered for use as hospital disinfectants can be used to decontaminate blood spills.

Field Cleaning of Power Units

1. Turn off the power unit and disconnect the power cord.
2. Wipe all controls and chassis, including the hose fittings, with a quaternary disinfectant solution.
3. Where needed, gentle agitation with a nylon brush may be useful since crevices can harbor microorganisms.
4. All treated surfaces should be allowed to air dry before connecting the power cord and restarting the system.

LAUNDRY PROCEDURE

It is important that laundry employees treat all soiled bedding as if it were contaminated with pathogenic microorganisms. Laundry workers should wear gloves, gowns and additional protective equipment if needed. Soiled bedding should be transported in covered carts and be handled as little as possible.

Separate soiled and clean bedding areas should be designated so that intermixing cannot occur. Soiled bedding should NOT be placed on the floor. Space should be provided for sorting, folding and bagging of clean bedding and to reassemble the mattress.

Mattress Disassembly

1. To launder the Turn-Q Plus Mattress, place the Mattress Assembly on a suitable work table.
2. Unsnap the Therapy Pad.
3. Unzip the hose end. Slide out the Air Tube/Hose assemblies.
4. Slide out the Lower Mattress Assembly.
5. Follow the laundry procedure for the Mattress Cover and Therapy Pad.
6. Inflate the Vinyl Air Tubes and Lower Mattress sections.
7. Clean and disinfect with a hospital grade Quaternary or Phenolic cleaner disinfectant.

Presoak

1. Remove the items to be laundered from the bag with as little handling and agitation as possible.
2. Place the items in the washing machine and fill the tub with warm water (below 60°C [140°F]).
3. Add laundry disinfectant presoak.
4. Follow the manufacturer's dilution directions. Typical directions require agitation of the laundry for three (3) minutes to ensure saturation, followed by a 10 minute soak.

Washing Cycle

1. Place laundry into the washing machine. DO NOT overload the machine.
2. Wash therapy pad separate from mattress cover.
3. Wash using the delicate load setting with warm water (60°C [140°F] max.) using detergent.
4. Follow the manufacturer's instructions. DO NOT use concentrated bleach since it may damage the fabric coatings.

Machine Drying Method

Laundry workers should always wash their hands before working with clean bedding. Dry therapy pads below 50°C (120°F) in an industrial clothes dryer with accurate controls. (High air temperatures will damage the fabrics and will void the warranty.)

Mattress Components

1. After disassembling the mattress, inflate the air tubes, and lower mattress.
2. Wipe down with a properly diluted solution of quaternary cleaner disinfectant and allow to air dry.

Transport Mat

Wipe down with a properly diluted solution of quaternary cleaner disinfectant and allow to air dry.

Packaging of Laundered System Components

1. Dry bedding should be placed in covered carts and transported to the packaging/reassembling area.
2. Each clean air tube subassembly should be reassembled into the mattress cover.
3. Completed mattresses should be packaged in labeled plastic bags then transferred to duffel bags.

Mattress Assembly

1. Place the Mattress Cover on a clean work table.
2. Unzip both ends.
3. Slide the Lower Mattress assembly into the lower pocket of the Mattress Cover. The hose should extend out the foot end.

4. Insert the two (2) side barrier Air Tubes as an assembly into the far left and right pockets. Align BLACK line on TOP for vent holes on top.
5. Insert the three (3) Left turning Air Tubes into the second, third and fourth pockets. Align BLACK line on TOP for vent holes on top.
6. Insert the three (3) Right turning Air Tubes into the fifth, sixth, and seventh pockets. Align BLACK line on TOP for vent holes on top.
7. Route the hoses through the opening on the Mattress Cover end flap.
8. If the components are hard to insert, reach in from the head end and pull the Tubes even with the top of the Mattress Cover edge.
9. Connect the hoses to the Power Unit and turn on power.
10. If the top netting should have any wrinkles, turn off power and have two workers give a brief pull from each end of the Mattress Cover to stretch the netting.
11. Cover the Mattress with Therapy Pad and snap in place.

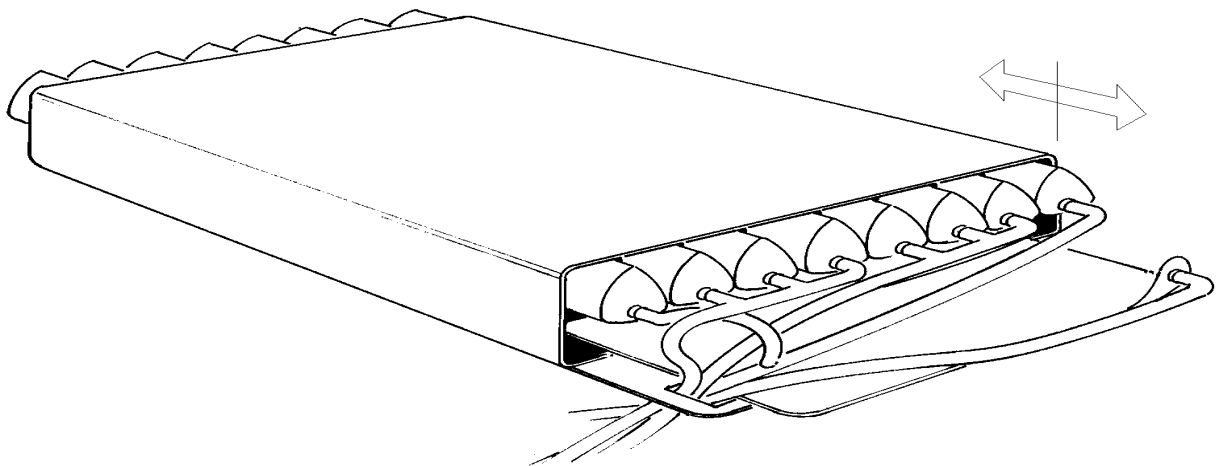
FIGURE 1 - MATTRESS DISASSEMBLY/ASSEMBLY

NON-STANDARD OPERATION

The majority of installations are successfully handled using the procedures detailed in this manual. When a difficult situation arises, call your customer service representative for assistance.

POWER UNIT RELOCATION

Some beds have controls that are obstructed by the standard power unit installation. It will be necessary to relocate the power unit if essential control functions are not duplicated elsewhere on the bed.

The power unit can be installed on the headboard, or hung under the frame at the foot-end, depending on the make and model of the bed.

NARROW WIDTH BEDS

WARNING

DO NOT use narrow width mattress assembly on standard width medical/home-care bed.

Invacare offers an optional narrow width mattress assembly for use with 82 cm wide ICU-type beds. Whatever the application, the side rails should touch the Mattress.

MAINTENANCE

AIR FILTER REPLACEMENT

The Turn-Q Plus system is equipped with an air filter. The filter can be changed without moving the bed. Regular servicing will insure the blower operates at maximum power.

The air filter should be changed every 1,500 hours (9 weeks) of continuous operation, or before the next patient is placed on the system. If the filter becomes excessively dirty, the system pressure will decrease.

The air filter is located behind the filter door (inside the power unit). The element is replaced as follows:

1. Open filter door by counterclockwise rotation.
2. Remove the used filter element and discard.
3. Install a new air filter element.
4. Close the filter door by clockwise rotation.

TROUBLESHOOTING

The Turn-Q Plus is constructed of industrial-grade components and built to high quality standards. While failures are unlikely, the following check points may be helpful:

PROBLEM	WHAT TO CHECK FOR
No tubes inflate; blower cannot be heard.	Make sure the power switch is "ON" and the power cord is plugged into a grounded receptacle. If the power switch is not lit, check the electrical outlet for line voltage.
One (1) Tube doesn't fully inflate or is totally deflated.	Check the tube for cuts, tears or material separation. Check each hose coupling is fully seated in the power unit. Check hoses for leaks.
All Air Tubes are too soft.	Increase the weight setting. Listen and feel for air leaks in the system. Follow the setup instructions in the PATIENT SETUP Section. The air filter may require replacement. Refer to the MAINTENANCE Section.
All Tubes are uninflated; the blower is running.	Check for air flow from the power unit. Check hose connections.
Patient DOESN'T turn.	Verify Turning Mode Selection and Turning Time. Check that ALL tubes have BLACK line aligned on TOP so air vents through netting.

SPARE PARTS AND OPTIONAL EQUIPMENT

SPARE PARTS

A list of available parts is shown below. Contact your customer service representative for supplies and service.

Air Filter

Therapy Pad - Standard or narrow width

Lower Air Mattress - Standard or narrow width

Air Tubes - standard or narrow width

Mattress Cover - Standard or narrow width

Air Supply Hoses - standard lengths, color coded:
Yellow, Red, Blue, and Green

Nylon Duffel Bag (for Mattress Assembly)

Operation and Maintenance Manual

Power Cord

Bumper Set

CPR Flag/Plug assembly

Power Unit

OPTIONAL EQUIPMENT

A list of optional equipment is shown below. Contact your customer service representative for applications assistance.

Air Supply Hoses

Hoses can be supplied for any length.

Narrow Mattress Assembly

This Mattress is fully compatible with the Power Unit and is specifically designed for use with narrow, intensive care beds.

APPENDIX

SYSTEM COMPONENTS CLEANING CHART - TURN-Q PLUS

COMPONENT	WIPE DOWN	PRESOAK	LAUNDER
Fabric Mattress Cover		●	●
Upper Air Tubes & Hoses	●	○	
Lower Mattress & Hoses	●		
Therapy Pad		●	●
Power Unit	●		

Legend

- As needed
- Required

INSPECTION AND TEST REPORT FOR TURN-Q PLUS SERIES LOW AIR LOSS THERAPY SYSTEMS FOR HOSPITAL INSTALLATIONS

SHEET 2 OF 2

ELECTRICAL INSPECTION CHECKLIST

ITEM	PARAMETER	REQUIREMENTS
Electrical leakage: Normal Reverse Ground connected	_____ $\mu\alpha$ _____ $\mu\alpha$ _____ $\mu\alpha$	No reading shall exceed 100 microamps
Ground resistance from plug to Power Unit chassis:	_____ milliohms	Resistance shall not exceed 100 milliohms

This unit has: ____ passed ____ failed the above inspections. If failed, the following repairs and corrections are required _____

I certify that this unit conforms to all of the above test and inspection criteria and that scheduled maintenance has been performed as indicated.

I certify that this unit is suitable for service.

Signature of Service Person

Date

SOURCES FOR GERMICIDAL CHEMICALS*

The following companies offer germicidal chemicals through distributors:

National Laboratories (Lysol Brand Disinfectants)

Calgon Vestal (LpHse, TBQ)

Convatel - Division Bristol Myers - Squibb

* Lysol, Convatel, LpHse, and TBQ are registered trademarks of their respective manufacturers.

INSPECTION, MAINTENANCE AND TEST INTERVALS FOR TURN-Q PLUS SERIES LOW AIR LOSS THERAPY SYSTEMS

The following procedures are to be performed by the designated service organization, dealer or institutional Biomed department:

TIME INTERVAL	DESCRIPTION OF REQUIRED ACTIVITY
1,500 hours (9 weeks of continuous operation)	<ul style="list-style-type: none"> - Remove and replace Air Filter cartridge - Inspect Air Hose connections & Mattress Assembly function - Inspect Power Unit and Power Cord
After decontamination of system	<ul style="list-style-type: none"> - Perform general physical inspection of Mattress Assembly, Hoses, Power Unit and Power Cord. - Perform electrical line leakage test - Perform electrical ground integrity test
Annually, or after 8,000 hours of operation	<ul style="list-style-type: none"> - Perform general physical inspection of Mattress Assembly, Hoses, Power Unit and Power Cord. - Perform electrical line leakage test - Perform electrical ground integrity test - Remove and replace Air Filter cartridge - Check for loose hardware or physical damage which might affect operation or safety

LIMITED WARRANTY (USA ONLY)

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants Turn-Q Plus Power Units to be free from defects in materials and workmanship for a period of two (2) years from date of purchase. Invacare warrants Soft Goods for the Turn-Q Plus system to be free of defects in material and workmanship for one (1) half year (180 days) from the date of purchase. Soft Goods means all system components excluding the Power Unit. If within such warranty period the product shall be proven to Invacare's reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare's option ; Invacare's sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare at the address on the bottom of the back cover during normal working hours..Upon receiving notice of an alleged defect in a product, Invacare will issue a serialised return authorisation. The product shall then be returned to either Invacare's plant or service center as specified by Invacare. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorisation date. DO NOT return products to our factory without prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THESE INSTRUCTIONS. A CHANGE IN OPERATING NOISE, PARTICULARLY RELATIVE TO MOTORS AND GEARBOXES DOES NOT CONSTITUTE A FAILURE. ALL SUCH DEVICES WILL EXHIBIT A CHANGE IN OPERATING NOISE DUE TO AGING.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

LIMITED WARRANTY (UK ONLY)

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare warrants Turn-Q Plus Power Units to be free from defects in materials and workmanship for a period of two (2) years from date of purchase. Invacare warrants Soft Goods for the Turn-Q Plus system to be free of defects in material and workmanship for one (1) half year (180 days) from the date of purchase. Soft Goods means all system components excluding the Power Unit. If within such warranty period the product shall be proven to Invacare's reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare's option ; Invacare's sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare at the address on the bottom of the back cover during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare will issue a serialised return authorisation. The product shall then be returned to either Invacare's plant or service center as specified by Invacare. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorisation date. DO NOT return products to our factory without prior consent.

LIMITATIONS AND EXCLUSIONS : THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORISED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE.

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© 2002 Invacare Corporation
Invacare is a registered trademark of Invacare Corporation.
Yes, you can. is a trademark of Invacare Corporation.
Invacare Corporation