


# **Nutron<sup>®</sup> Series**

## **R5I<sup>™</sup> and R5I<sub>LXP</sub>**

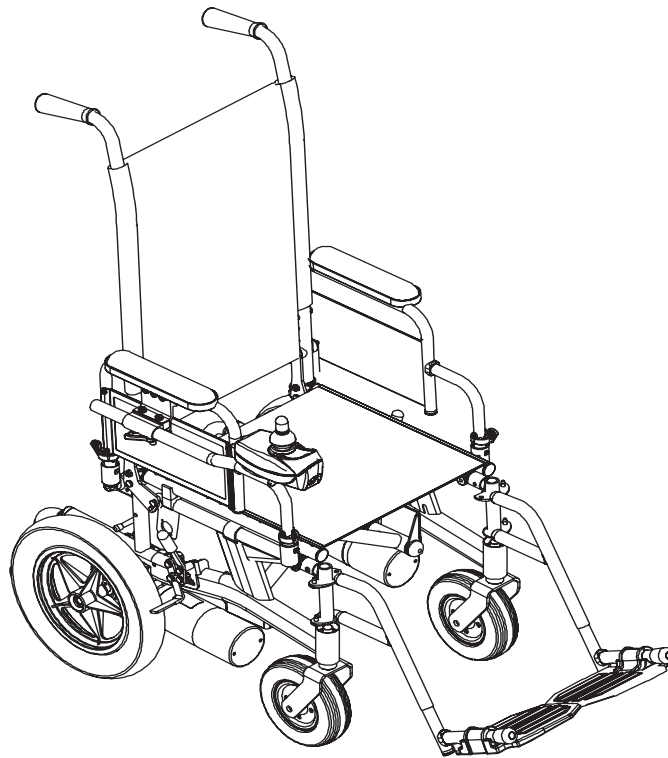

**DEALER:** This manual **MUST** be given to the user of the wheelchair.

**USER:** **BEFORE** using this wheelchair, read this manual and save for future reference.

For more information regarding  
Invacare products, parts, and services,  
please visit [www.invacare.com](http://www.invacare.com)



**Yes, you can.<sup>®</sup>**

## **WARNING**

A qualified technician **MUST** perform the initial set up of this wheelchair. Also, a qualified technician **MUST** perform all procedures in the service manual.

---

## **DANGER**

**Risk of Death, Serious Injury or Damage**

**Improper use of this product may cause injury or damage**

**If you are unable to understand the warnings, cautions or instructions, contact a health care professional or dealer before attempting to use this equipment.**

**DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment.**

---

# REFERENCE DOCUMENTS

DOCUMENT	PART NUMBER
Quad Link and Height Adjustable Joystick Mount Instruction Sheet	1134844
MK <sub>5</sub> <sup>™</sup> NX <sup>™</sup> Electronics Manual	1110532

*NOTE: Updated versions of this manual are available on [www.invacare.com](http://www.invacare.com).*

# TABLE OF CONTENTS

<b>REFERENCE DOCUMENTS .....</b>	<b>2</b>
<b>SPECIAL NOTES .....</b>	<b>7</b>
<b>SPECIFICATIONS .....</b>	<b>8</b>
<b>SECTION 6—GENERAL GUIDELINES .....</b>	<b>9</b>
Accessories Information .....	9
Batteries.....	9
Charging Batteries .....	10
Grounding Instructions .....	10
Operating Information.....	11
Repair or Service Information .....	13
Rain Test.....	14
Tire Pressure .....	15
Weight Training .....	15
Weight Limitation.....	15
<b>SECTION 7—SAFETY GUIDELINES .....</b>	<b>16</b>
<b>SECTION 8—EMI INFORMATION .....</b>	<b>19</b>
<b>SECTION 9—SAFETY/HANDLING OF WHEELCHAIRS .....</b>	<b>21</b>
Stability and Balance.....	21
Coping With Everyday Obstacles.....	21
A Note to Wheelchair Assistants.....	22
Tilting.....	22
Tilting - Curbs.....	22
Stairways.....	23
Escalators .....	24
Transferring To and From Other Seats .....	24
Reaching, Leaning and Bending - Forward .....	25
Reaching and Bending - Backward .....	25
<b>SECTION 10—SAFETY INSPECTION/TROUBLESHOOTING .....</b>	<b>27</b>
Safety Inspection Checklists.....	27
Inspect/Adjust Initially .....	27
Inspect/Adjust Weekly.....	28
Inspect/Adjust Monthly.....	28
Inspect/Adjust Periodically.....	28
Troubleshooting Guide.....	29

# TABLE OF CONTENTS

Troubleshooting - Electrical.....	29
SPJ™ + Joysticks.....	29
Information Gauge Display Diagnostics .....	30
Service Indicator Light Diagnostics .....	30
Checking Battery Charge Level.....	31
<b>SECTION 11—WHEELCHAIR OPERATION .....</b>	<b>32</b>
Operating the Wheelchair.....	32
Turning the Power On/Off.....	32
Using the Joystick to Drive the Wheelchair.....	32
SPJ™ + Joystick Switches and Indicators .....	33
On/Off Button .....	34
Speedometer.....	34
Speed Control Buttons.....	34
Joystick.....	34
Charger/Programming Input.....	35
Service Indicator.....	35
Information Gauge Display .....	35
<b>SECTION 12—FRONT RIGGINGS .....</b>	<b>36</b>
Installing/Removing the Footrests.....	36
Installing.....	36
Removing .....	36
Adjusting the Footrest Height.....	36
Model PW93, PW93E, and PW93ST Footrests .....	36
Model 93M, 904A, PAL4A, and PAS4A Footrests .....	37
Replacing Heel Loops .....	37
Installing/Removing Elevating Legrests.....	38
Installing.....	38
Removing .....	38
Raising/Lowering Elevating Legrests and/or Adjusting Calfpads .....	39
Raising/Lowering Elevating Legrests .....	39
Adjusting Calfpads.....	39
Adjusting/Replacing Telescoping Front Rigging Support .....	39
<b>SECTION 13—ARMS .....</b>	<b>41</b>
Adjusting Armrest Height.....	41
Using Swing-Back Arms.....	42
Replacing Desk/Full Length Armrest Pad and/or Clothing Guards .....	42
Armrest Pad .....	42
Clothing Guard.....	42

# TABLE OF CONTENTS

Removing/Installing Armrest.....	43
Removing .....	43
Installing.....	43
<b>SECTION 14—BACK .....</b>	<b>44</b>
Adjusting the Back Height .....	44
Using the Fold Down Back Canes .....	45
Changing the Back Angle .....	45
<b>SECTION 15—UPHOLSTERY/SEAT POSITIONING STRAP .....</b>	<b>47</b>
Replacing Seat Upholstery .....	47
Replacing Back Upholstery .....	47
Replacing Seat Positioning Strap .....	48
<b>SECTION 16—JOYSTICK .....</b>	<b>50</b>
Preparing the Joystick For Use.....	50
Repositioning the Joystick.....	51
<b>SECTION 17—CASTERS .....</b>	<b>52</b>
Adjusting Forks .....	52
<b>SECTION 18—BATTERIES .....</b>	<b>53</b>
Warnings for Handling and Replacing Batteries .....	53
Using the Proper Batteries.....	54
Recommended Battery Types .....	54
Installing/Removing the Battery Boxes .....	55
Installing/Removing the Battery Tray.....	56
Installing/Removing the Batteries Into/From the Battery Boxes .....	57
Connecting Battery Cables .....	59
Direct Mount Method.....	59
Battery Clamp Method .....	62
Replacing the Battery Box Retaining Strap.....	65
Cleaning Battery Terminals .....	66
When to Charge Batteries.....	67
SPJ™ + Joysticks.....	67
Description and Use of Battery Chargers.....	68
Connecting the Battery Charger .....	68
<b>SECTION 19—CLUTCH/MOTOR LOCK .....</b>	<b>69</b>
Engaging/Disengaging the Clutches.....	69

# TABLE OF CONTENTS

Disengaging/Engaging the Motor Lock Levers - R51LXP Only .....	70
Four Pole Motors.....	70
<b>SECTION 20—WHEEL LOCKS .....</b>	<b>71</b>
Installing/Adjusting/Using the Wheel Locks .....	71
Installing/Adjusting .....	71
Using .....	72
<b>SECTION 21—ANTI-TIPPERS .....</b>	<b>73</b>
Installing/Removing the Anti-Tippers.....	73
Installing.....	73
Removing .....	73
<b>SECTION 22—TRANSPORTING .....</b>	<b>74</b>
Transporting the Nutron.....	74
Unfolding/Folding the Wheelchair .....	74
Unfolding .....	74
Folding .....	74
<b>SECTION 23—WEAR AND TEAR INFORMATION .....</b>	<b>76</b>
General Information .....	76
<b>GLOBAL LIMITED WARRANTY (EXCLUDING CANADA) .....</b>	<b>79</b>
<b>CANADA LIMITED WARRANTY .....</b>	<b>80</b>

# SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

## NOTICE

**THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.**

### WHEELCHAIR USER

As a manufacturer of wheelchairs, Invacare endeavors to supply a wide variety of wheelchairs to meet many needs of the end user. However, final selection of the type of wheelchair to be used by an individual rests solely with the user and his/her healthcare professional capable of making such a selection.

### WHEELCHAIR TIE-DOWN RESTRAINTS AND SEAT RESTRAINTS

Wheelchair users should **NOT** be transported in vehicles of any kind while in wheelchairs. As of this date, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type.

It is Invacare's position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the auto industry. Invacare cannot and does not recommend any wheelchair transportation systems.

## DANGER

### Risk of Death or Serious Injury

Not wearing your seat positioning strap could result in death or serious injury.

**ALWAYS** wear your seat positioning strap. Your seat positioning strap helps reduce the possibility of a fall from the wheelchair. The seat positioning strap is a positioning belt only. It is not designed for use as a safety device withstanding high stress loads such as auto or aircraft safety belts. If signs of wear appear, seat positioning strap **MUST** be replaced **IMMEDIATELY**.

## SPECIFICATIONS

<b>SEAT WIDTH RANGE</b>	14, 16, 18 or 20 inches
<b>SEAT DEPTH RANGE</b>	16, 17 or 18 inches (17 and 18 in. by extension)
<b>BACK HEIGHT RANGE STANDARD</b>	16, 17 or 18 inches
<b>SEAT-TO-FLOOR</b>	17¾ or 19¾ inches
<b>OVERALL WIDTH</b>	21¾ inches to 27¾ inches (With joystick)
<b>OVERALL HEIGHT</b> LOW SEAT ADULT	33¾ inches to 37¾ inches 35¾ inches to 39¾ inches
<b>OVERALL LENGTH</b>	36½ inches to 47¼ inches
<b>WEIGHT</b> W/O BATTERIES AND FRONT RIGGINGS W/ BATTERIES AND PW93 FRONT RIGGINGS UI GEL CELL (14-INCH CHAIRS) 22NF GEL CELL (16, 18 OR 20 INCH CHAIRS) SHIPPING	82 pounds 145 pounds 110 pounds 157 pounds 128 pounds
<b>DRIVE WHEELS/TIRES</b> (FLAT FREE OR PNEUMATIC)	12½ inches x 2¼ inches
<b>CASTERS W/PRECISION SEALED BEARINGS</b>	8 x 1¾ inches semi pneumatic (standard) 8 x 2 inches pneumatic (optional) 6 x 2 inches semi pneumatic (optional)
<b>ANTI-TIPPERS</b>	Rotating, removable (standard)
<b>FOOTREST/LEGREST</b>	Swingaway, removable
<b>ARMRESTS</b> FIXED HEIGHT ADJUSTABLE HEIGHT	Desk and full length (standard) Desk and full length (optional)
<b>UPHOLSTERY</b>	Black nylon
<b>BATTERIES</b> BATTERY/SIZE (NOT SUPPLIED) TWO REQUIRED	UI gel cell 22NF gel cell (not available on 14-inch wide wheel-chairs)
<b>SPEED (M.P.H.)</b>	0 to 4 (R51) or 0 to 6 (R51LXP)
<b>TURNING RADIUS</b>	35 inches
<b>*RANGE (VARIABLE)</b>	12 - 14 miles with UI batteries 18 - 22 miles with 22NF batteries
<b>**WEIGHT LIMITATION</b> R51 R51LXP	250 lbs 300 lbs
<b>OPERATING TEMPERATURE</b> <b>STORAGE TEMPERATURE</b>	122 F (50 C) Maximum to -13 F (-25 C) Minimum 149 F (65 C) Maximum to -58 F (-40 C) Minimum

\*NOTE: Values for range are calculated for maximum chair weight rating using largest batteries applicable (22NF), per test procedures described in ANSI/RESNA WC/VOL2-1998 Section 4 and meet federal reimbursement requirements for this product. While considered typical, they are derived based on certain ideal conditions. Variances in battery condition, user weight, usage pattern or overall terrain conditions will result in actual values for range that differ from these stated values. Users should become accustomed to how their unique conditions impact their individual results. Users should become familiar with the battery discharge indicator on the joystick to determine the range of their wheelchair. Refer to When to Charge Batteries on page 67 for more information about the battery discharge indicator.

\*\*NOTE: Includes seating system and accessories.



---

# SECTION 6—GENERAL GUIDELINES

---

## **⚠ WARNING**

**SECTION 1 - GENERAL GUIDELINES** contains important information for the safe operation and use of this product.

---

## Accessories Information

---

### **⚠ WARNING**

#### **Risk of Serious Injury or Damage**

Use of non-Invacare accessories may result in serious injury or damage.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

**DO NOT** use non-Invacare accessories.

To obtain Invacare accessories, contact Invacare by phone or at [www.invacare.com](http://www.invacare.com).

---

### **⚠ WARNING**

Extreme care should be exercised when using oxygen in close proximity to electric circuits and other combustible materials. Contact your oxygen supplier for instruction in the use of oxygen.

---

## Batteries

---

### **⚠ DANGER**

When using an extension cord, use only a three wire extension cord having at least 16 AWG (American Wire Gauge) wire and the same or higher electrical rating as the device being connected. Use of improper extension cord could result in a risk of fire and electric shock. Three prong to two prong adapters should not be used. Use of three prong adapters can result in improper grounding and present a shock hazard to the user.

The warranty and performance specifications contained in this manual are based on the deep cycle gel cell batteries. Invacare strongly recommends their use as the power source for this unit.

Carefully read battery/battery charger information prior to installing, servicing or operating your wheelchair.

---

## Charging Batteries

---

### **⚠ WARNING**

**NEVER** attempt to recharge the batteries by attaching cables directly to the battery terminals.

**DO NOT** attempt to recharge the batteries and operate the wheelchair at the same time.

**DO NOT** operate wheelchair with extension cord attached to the AC cable.

**DO NOT** attempt to recharge the batteries when the wheelchair has been exposed to any type of moisture.

**DO NOT** attempt to recharge the batteries when the wheelchair is outside.

**DO NOT** sit in the wheelchair while charging the batteries.

**READ** and **CAREFULLY** follow the manufacturer's instructions for each charger (supplied or purchased). If charging instructions are not supplied, consult a qualified technician for proper procedures.

After charging batteries, **ALWAYS** make sure that the battery charger cord is unplugged. Failure to do this could result in damage to the cord or personal injury to the user or bystanders.

Ensure the pins of the extension cord plug are the same number, size, and shape as those on the charger.

**DO NOT** under any circumstances cut or remove the round grounding plug from the charger AC cable plug or the extension cord plug.

---

## Grounding Instructions

---

### **⚠ WARNING**

**DO NOT**, under any circumstances, cut or remove the round grounding prong from any plug used with or for Invacare products. Some devices are equipped with three-prong (grounding) plugs for protection against possible shock hazards. Where a two-prong wall receptacle is encountered, it is the personal responsibility and obligation of the customer to contact a qualified electrician and have the two-prong receptacle replaced with a properly grounded three-prong wall receptacle in accordance with the National Electrical Code. If you must use an extension cord, use only a three-wire extension cord having the same or higher electrical rating as the device being connected. In addition, Invacare has placed **RED/ORANGE WARNING TAGS** on some equipment. **DO NOT** remove these tags.

---

---

## Operating Information

---

### **⚠ DANGER**

#### **Risk of Death, Serious Injury or Damage**

Operating the wheelchairs outdoors or in areas of poor lighting may result in death, serious injury, or damage.

Operating the wheelchair near motor vehicles may result in death, serious injury or damage.

**DO NOT** operate on roads, streets or highways.

Use caution when operating the wheelchair outdoors at night or in areas with poor lighting.

**ALWAYS** be aware of motor vehicles when using the wheelchair.

#### **Risk of Death, Serious Injury or Damage**

Continued use of the wheelchair that is not set to the correct specifications may cause erratic behavior of the wheelchair resulting in death, serious injury, or damage.

Performance adjustments should only be made by professionals of the healthcare field or persons fully conversant with this process and the driver's capabilities.

After the wheelchair has been set up/adjusted, check to make sure that the wheelchair performs to the specifications entered during the set up procedure. If the wheelchair does not perform to specifications, turn the wheelchair **Off** immediately and reenter set up specifications. Contact Invacare, if wheelchair still does not perform to correct specifications.

#### **Risk of Death, Serious Injury or Damage**

Misuse of the wheelchair may cause component failure and/or the wheelchair to start smoking, sparking, or burning. Death, serious injury, or damage may occur due to fire.

**DO NOT** use the wheelchair other than its intended purpose. If the wheelchair starts smoking, sparking, or burning, discontinue using the wheelchair and seek service **IMMEDIATELY**.

#### **Risk of Death, Serious Injury, or Damage**

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury, or damage.

Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

**DO NOT** smoke while using this wheelchair.

---

**⚠ WARNING**

**Risk of Serious Injury or Damage**

Operating the wheelchair with a ground clearance of less than 3 inches between the footplates and the ground/floor may cause serious injury or property damage.

**ALWAYS** maintain a minimum of 3 inches between the bottom of the footplates and ground/floor to ensure proper ground clearance while the wheelchair is in motion. If necessary, adjust the footplates height to achieve proper ground clearance. After footplates height adjustment, if the wheelchair dips forward and the footplates touch the ground while in motion, please contact your dealer for an inspection and avoid use of the wheelchair if possible. This warning applies to the use of the footboard as well.

**DO NOT** shift your weight or sitting position toward the direction you are reaching as the wheelchair may tip over.

**DO** determine and establish your particular safety limits by practicing bending, reaching and transferring activities in the presence of a qualified healthcare professional **BEFORE** attempting active use of the wheelchair.

**DO NOT** lean over the top of the back upholstery to reach objects from behind as this may cause the wheelchair to tip over.

**DO NOT** use an escalator to move a wheelchair between floors. Serious bodily injury may occur.

**DO NOT** engage or disengage the motor release levers until the power is in the off position.

Invacare strongly recommends proceeding down ramps or slopes at half speed or slower and to avoid hard braking or sudden stops.

**DO NOT** attempt to lift the wheelchair by lifting on any removable (detachable) parts. Lifting by means of any removable (detachable) parts of a wheelchair may result in injury to the user or damage to the wheelchair.

---

**⚠ WARNING**

**Risk of Serious Injury**

Hot surfaces can cause severe burns

Be mindful of potential hot surfaces and avoid touching.

---

**⚠ WARNING**

Before performing any maintenance, adjustment or service verify that the ON/OFF switch on the joystick is in the off position.

Avoid storing or using the wheelchair near open flame or combustible products. Serious injury or damage to property may result.

**DO NOT** engage or disengage the motor locks/clutches until the power is in the OFF position.

**DO NOT** climb, go up or down ramps or traverse slopes greater than 9°.

**DO NOT** attempt to drive over curbs or obstacles. Doing so may cause your wheelchair to turn over and cause bodily harm or damage to the chair.

**DO NOT** leave the power button in the on position when entering or exiting your wheelchair.

**DO NOT** stand on the frame of the wheelchair.

**DO NOT** use the footplates as a platform. When getting in or out of the wheelchair, make sure that the footplates are in the upward position or swing footrests towards the outside of the chair.

Anti-tippers **MUST** be used at all times. When outdoors on wet, soft ground or gravel surfaces, anti-tippers may not provide the same level of protection against tipover. Extra caution must be observed when traversing such surfaces.

---

**Repair or Service Information**

---

**⚠ DANGER**

**Risk of Death, Serious Injury or Damage**

Use of incorrect or improper replacement (service) parts may cause death, serious injury, or damage.

Replacement parts **MUST** match original Invacare parts.

**ALWAYS** provide the wheelchair serial number to assist in ordering the correct replacement parts.

**Risk of Death, Serious Injury, or Damage**

Incorrect repair and/or servicing of this wheelchair performed by users/caregivers or unqualified technicians can result in death, serious injury, or damage.

**Users/Caregivers — DO NOT** attempt to repair and/or service this wheelchair.

Repair and/or service of this wheelchair **MUST** be performed by a qualified technician. Contact a dealer or Invacare technician.

---

**⚠ WARNING**

Set-up of the Electronic Control Unit is to be performed **ONLY** by a qualified technician. The final adjustments of the controller may affect other activities of the wheelchair. Damage to the equipment could occur if improperly set-up or adjusted.

---

**⚠ DANGER**

**Risk of Death, Serious Injury, or Damage**

Corroded electrical components due to water, liquid exposure, or incontinent users can result in death, serious injury, or damage.

Minimize exposure of electrical components to water and/or liquids. Electrical components damaged by corrosion **MUST** be replaced immediately.

Wheelchairs that are used by incontinent users and/or are frequently exposed to water/liquids may require replacement of electrical components more frequently.

---

## Rain Test

---

**⚠ CAUTION**

**Risk of Damage**

Operating the wheelchair in rain or dampness may cause the wheelchair to malfunction electrically and mechanically; may cause the wheelchair to prematurely rust or may damage the upholstery.

**DO NOT** leave wheelchair in a rain storm of any kind.

**DO NOT** use wheelchair in a shower.

**DO NOT** leave wheelchair in a damp area for any length of time.

Check to ensure that the battery covers are secured in place, joystick boot is **NOT** torn or cracked where water can enter and that all electrical connections are secure at all times. **DO NOT** use if the joystick boot is torn or cracked. If the joystick boot becomes torn or cracked, replace **IMMEDIATELY**.

---

*NOTE: Invacare has tested its power wheelchairs in accordance with ISO 7176 "Rain Test". This provides the end user or his/her attendant sufficient time to remove his/her power wheelchair from a rain storm and retain wheelchair operation.*

## Tire Pressure

---

### **⚠ WARNING**

**DO NOT** use your wheelchair unless it has the proper tire pressure (p.s.i.). **DO NOT** overinflate the tires. Failure to follow these suggestions may cause the tire to explode and cause bodily harm. The recommended tire pressure is listed on the side wall of the tire.

---

## Weight Training

---

### **⚠ WARNING**

Invacare **DOES NOT** recommend the use of its wheelchairs as a weight training apparatus. Invacare wheelchairs have not been designed or tested as a seat for any kind of weight training. If occupant uses said wheelchair as a weight training apparatus, Invacare shall not be liable for bodily injury, and the warranty is void.

---

## Weight Limitation

---

### **⚠ WARNING**

The Nutron R5I has a weight limitation, including the seating system, of 250 lbs. The Nutron R5ILXP has a weight limitation of 300 lbs. Refer to Coping With Everyday Obstacles on page 21.

Invacare recommends that only heavy-duty wheelchairs be used for individuals weighing more than 250 lbs. Further, the activity level of the individual wheelchair user is important. For instance, a 170 lbs active wheelchair user could subject the wheelchair to more stress than a 250 lbs user. Invacare recommends that very active users consider using heavy-duty wheelchairs.

---

## SECTION 7—SAFETY GUIDELINES

---

### **⚠ WARNING**

#### **Risk of Serious Injury or Damage**

Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage

After **ANY** adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

---

### **⚠ DANGER**

#### **Risk of Death, Serious Injury, or Damage**

Missing attaching hardware could cause instability resulting in death, serious injury or damage.

Ensure all attaching hardware is present and tightened securely.

---

### **⚠ WARNING**

#### **Risk of Serious Injury or Damage**

Loss of power due to loose electrical connections could cause the wheelchair to suddenly stop resulting in serious injury or damage.

**ALWAYS** ensure that all electrical connections are tightly connected so they don't vibrate loose.

---

### **⚠ WARNING**

#### **Risk of Injury or Damage**

Incorrect set up of this wheelchair performed by users/caregivers or unqualified technicians can result in injury or damage.

User/Caregivers- **DO NOT** attempt to set up this wheelchair.

Initial set up of this wheelchair **MUST** be performed by a qualified technician.

---

### **⚠ WARNING**

#### **Risk of Minor to Serious Injury**

Pinch points can cause minor to serious injury.

Be mindful of potential pinch points and use caution when using this product.

---

### **⚠ WARNING**

#### **Risk of Serious Injury**

Impacting objects in the surrounding environment can cause serious injury.

When maneuvering the wheelchair around, **ALWAYS** have assured cleared distance with all objects in environment.

---



---

**⚠ WARNING****Risk of Serious Injury**

Sharp edges can cause serious injury.

Be mindful that some parts may have sharp edges. Use caution when encountering these sharp edges.

---

**⚠ WARNING****Risk of Death, Serious Injury, or Damage**

Operating the wheelchair without anti-tippers could result in death, serious injury or damage.

**DO NOT** operate the wheelchair without antitippers installed.

---

**⚠ WARNING****Risk of Serious Injury or Damage**

Accidental activation of wheelchair caused by pets, children, etc. can result in serious injury or damage.

**ALWAYS** turn power off when around pets and/or children to prevent unintended movement.

---

**⚠ DANGER****Risk of Death or Serious Injury**

Traveling on inclines with wet, slippery, icy or oily surfaces could cause loss of traction resulting in death or serious injury.

**DO NOT** use on inclines with wet, slippery, icy or oily surfaces. This may include certain painted or otherwise treated wood surfaces.

---

**⚠ WARNING****Risk of Death or Serious Injury**

Braking hard and/or sudden stops while on inclines could cause loss of stability resulting in death or serious injury.

While on inclines, **ALWAYS** travel at a reduced, constant speed to maintain stability. Traveling down ramps at high speeds will reduce traction and increase stopping distance.

**DO NOT** brake hard and avoid sudden stops while traveling on an incline.

If stopping becomes necessary while on an incline, release the joystick and allow the wheelchair to come to a full stop. Then proceed at a slower speed.

---

---

**⚠ DANGER**

**Risk of Death or Serious Injury**

Electric shock can cause death or serious injury

To avoid electric shock, inspect plug and cord for cuts and/or frayed wires. Replace cut cords or frayed wires immediately.

---

**⚠ WARNING**

**Risk of Serious Injury or Damage**

Dropping the battery can result in serious injury or property damage.

Batteries can weigh up to 52 lbs (23.6 kg). **ALWAYS** use a battery lifting strap when lifting the battery. It is the most reliable method of carrying a battery and preventing serious injury.

---

**⚠ WARNING**

**Risk of Death, Serious Injury or Damage**

Exceeding the weight capacity of the wheelchair/seating system could cause instability resulting in death or serious injury.

**DO NOT** exceed the weight capacity.

---

**⚠ WARNING**

**Risk of Serious Injury or Death**

Failure to observe this warning can result in serious injury or death.

Loss of traction on ramps and inclines can occur for a variety of reasons including; water, ramp material, surface conditions, steepness or grade etc. Lighter weight users may be at an increased risk for loss of traction. As such, when using on ramps or inclines always reduce speed and proceed with caution.

---

**⚠ WARNING**

**Risk of Death, Serious Injury, or Damage**

Improperly connected joystick could cause loss of power resulting in death, serious injury, or damage.

Ensure the joystick is securely connected to controller.

---

**⚠ DANGER**

**Risk of Death, Serious Injury or Damage**

Malfunctioning joystick could cause unintended/erratic movement resulting in death, serious injury, or damage.

If unintended/erratic movement occurs, stop using the wheelchair immediately and contact a qualified technician.

---

---

# SECTION 8—EMI INFORMATION

---

## **⚠ WARNING**

**CAUTION: IT IS VERY IMPORTANT THAT YOU READ THIS INFORMATION REGARDING THE POSSIBLE EFFECTS OF ELECTROMAGNETIC INTERFERENCE ON YOUR POWERED WHEELCHAIR.**

### **Electromagnetic Interference (EMI) From Radio Wave Sources**

Powered wheelchairs and motorized scooters (in this text, both will be referred to as powered wheelchairs) may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two way radios, and cellular phones. The interference (from radio wave sources) can cause the powered wheelchair to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the powered wheelchair's control system. The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each powered wheelchair can resist EMI up to a certain intensity. This is called its "immunity level." The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20 V/m immunity level, which would provide useful protection from the more common sources of radiated EMI.

There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warnings listed below, your risk to EMI will be minimized.

The sources of radiated EMI can be broadly classified into three types:

- 1) Hand-held Portable transceivers (transmitters-receivers with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, "walkie talkie", security, fire and police transceivers, cellular telephones, and other personal communication devices).

*NOTE: Some cellular telephones and similar devices transmit signals while they are ON, even when not being used.*

- 2) Medium-range mobile transceivers, such as those used in police cars, fire trucks, ambulances and taxis. These usually have the antenna mounted on the outside of the vehicle; and
- 3) Long-range transmitters and transceivers, such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.

*NOTE: Other types of hand-held devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, cassette players, and small appliances, such as electric shavers and hair dryers, so far as we know, are not likely to cause EMI problems to your powered wheelchair.*

---

**⚠ WARNING****Powered Wheelchair Electromagnetic Interference (EMI)**

Because EM energy rapidly becomes more intense as one moves closer to the transmitting antenna (source), the EM fields from hand-held radio wave sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the powered wheelchair's control system while using these devices. This can affect powered wheelchair movement and braking. Therefore, the warnings listed below are recommended to prevent possible interference with the control system of the powered wheelchair.

Electromagnetic interference (EMI) from sources such as radio and TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones can affect powered wheelchairs and motorized scooters.

**FOLLOWING THE WARNINGS LISTED BELOW SHOULD REDUCE THE CHANCE OF UNINTENDED BRAKE RELEASE OR POWERED WHEELCHAIR MOVEMENT WHICH COULD RESULT IN SERIOUS INJURY.**

- 1) Do not operate hand-held transceivers (transmitters receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the powered wheelchair is turned ON;
- 2) Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them;
- 3) If unintended movement or brake release occurs, turn the powered wheelchair OFF as soon as it is safe;
- 4) Be aware that adding accessories or components, or modifying the powered wheelchair, may make it more susceptible to EMI (NOTE: There is no easy way to evaluate their effect on the overall immunity of the powered wheelchair); and
- 5) Report all incidents of unintended movement or brake release to the powered wheelchair manufacturer, and note whether there is a source of EMI nearby.

**Important Information**

- 1) 20 volts per meter (V/m) is a generally achievable and useful immunity level against EMI (as of May 1994) (the higher the level, the greater the protection);
- 2) This device has been tested to a radiated immunity level of 20 volts per meter.
- 3) The immunity level of the product is unknown.

Modification of any kind to the electronics of this wheelchair as manufactured by Invacare may adversely affect the EMI immunity levels.

---

# SECTION 9—SAFETY/HANDLING OF WHEELCHAIRS

“Safety and Handling” of the wheelchair requires the close attention of the wheelchair user as well as the assistant. This manual points out the most common procedures and techniques involved in the safe operation and maintenance of the wheelchair. It is important to practice and master these safe techniques until you are comfortable in maneuvering around the frequently encountered architectural barriers.

Use this information only as a “basic” guide. The techniques that are discussed on the following pages have been used successfully by many.

Individual wheelchair users often develop skills to deal with daily living activities that may differ from those described in this manual. Invacare recognizes and encourages each individual to try what works best for him/her in overcoming architectural obstacles that they may encounter, however, ALL WARNINGS and CAUTIONS given in this manual MUST be followed. Techniques in this manual are a starting point for the new wheelchair user and assistant with “safety” as the most important consideration for all.

## Stability and Balance

---

### **WARNING**

**Be aware that carrying heavy objects on your lap while occupying the wheelchair may adversely affect the stability of the wheelchair, resulting in serious bodily injury to the user, damage to the wheelchair and surrounding property.**

**This wheelchair has been designed to accommodate one individual. If more than one individual occupies the wheelchair this may adversely affect the stability of the wheelchair, resulting in serious bodily injury to the user and passenger and damage to the wheelchair and surrounding property.**

---

To assure stability and proper operation of your wheelchair, you must at all times maintain proper balance. Your wheelchair has been designed to remain upright and stable during normal daily activities as long as you do not move beyond the center of gravity. DO NOT lean forward out of the wheelchair any further than the length of the armrests.

## Coping With Everyday Obstacles

Coping with the irritation of everyday obstacles can be alleviated somewhat by learning how to manage your wheelchair. Keep in mind your center of gravity to maintain stability and balance.

## **A Note to Wheelchair Assistants**

When assistance to the wheelchair user is required, remember to use good body mechanics. Keep your back straight and bend your knees whenever tilting wheelchair or traversing curbs, or other impediments.

---

### **⚠ WARNING**

**DO NOT attempt to lift the wheelchair by any removable (detachable) parts. Lifting by means of any removable (detachable) parts of a wheelchair may result in injury to the user or damage to the wheelchair.**

---

Also, be aware of detachable parts such as arms or leg-rests. These must NEVER be used for hand-hold or lifting supports, as they may be inadvertently released, resulting in possible injury to the user and/or assistant(s).

When learning a new assistance technique, have an experienced assistant help you before attempting it alone.

## **Tilting**

---

### **⚠ WARNING**

**DO NOT tilt the wheelchair without assistance.**

---

When tilting the wheelchair, an assistant should grasp the back of the wheelchair on a non-removable (non-detachable) part. Inform the wheelchair occupant before tilting the wheelchair and remind him/her to lean back. Be sure the occupant's feet and hands are clear of all wheels.

### **Tilting - Curbs**

After mastering the techniques of tilting the wheelchair, use this procedure to tackle curbs, short stairs, etc.

This procedure requires two assistants. The second assistant should be positioned at the front of the wheelchair lifting upward on a non-removable (non-detachable) part of the wheelchair frame when lifting the wheelchair and stabilizing the wheelchair when the wheelchair is being lowered to the ground.

The first assistant should turn the anti-tippers so the wheels are pointing up, apply a continuous downward motion until the balance point is achieved and the front casters clear the curb. At this point, the assistants will feel a difference in the weight distribution.

**⚠ WARNING**

**DO NOT** let the wheelchair drop the last few inches to the ground. This could result in injury to the occupant.

**Anti-tippers MUST BE** used at all times. When outdoors on wet, soft ground or gravel surfaces, anti-tippers may not provide the same level of protection against tipover. Extra caution must be observed when traversing such surfaces.

---

Roll the wheelchair forward and SLOWLY lower the wheelchair in one continuous movement. Push the wheelchair forward until the rear wheels roll up and over the curb.

---

**⚠ WARNING**

**Make sure anti-tipper wheels are pointing down towards the ground/floor before using the wheelchair.**

---

Turn the anti-tipper wheels down toward the ground/floor.

## Stairways

---

**⚠ WARNING**

**DO NOT** attempt to move an occupied power wheelchair between floors using a stairway. Use an elevator to move an occupied power wheelchair between floors. If moving a power wheelchair between floors by means of a stairway, the occupant **MUST** be removed and transported independently of the power wheelchair.

Extreme caution is advised when it is necessary to move an unoccupied power wheelchair up or down the stairs. Invacare recommends using two assistants and making thorough preparations. Make sure to use only secure, non-detachable parts for hand-hold supports.

**DO NOT** attempt to lift the wheelchair by any removable (detachable) parts. Lifting by means of any removable (detachable) parts of a wheelchair may result in injury to the user or damage to the wheelchair.

The weight of the wheelchair without the user and batteries is between 80 and 127 lbs. Use proper lifting techniques (lift with your legs) to avoid injury.

---

1. Remove the occupant from the wheelchair.
2. Remove the battery boxes from the wheelchair. Refer to [Installing/Removing the Battery Boxes](#) on page 55.
3. If necessary, fold the wheelchair. Refer to [Transporting the Nutron](#) on page 74.
4. Bend your knees and keep your back straight.
5. Using non-removable (non-detachable) parts of the wheelchair, lift the wheelchair off of the ground and transfer the wheelchair up or down the stairs.

- The wheelchair should not be lowered until the last stair has been negotiated and the wheelchair has been carried away from the stairway.

## Escalators

---

### **⚠ WARNING**

**DO NOT** use an escalator to move a wheelchair between floors. Serious bodily injury may occur.

---

## Transferring To and From Other Seats

---

### **⚠ WARNING**

#### **Risk of Serious Injury or Damage**

Improper transfer techniques may cause serious injury or damage.

**Before attempting transfers, consult a health care professional to determine proper transfer techniques for the user and type of wheelchair.**

**Reduce gap between transfer surface and wheelchair seat to the minimum distance necessary to perform transfer.**

**Align casters parallel to the drive wheels to improve stability during transfer.**

**ALWAYS** turn the wheelchair power off.

**ALWAYS** engage both motor locks/clutches and free wheel hubs (if equipped) to prevent the wheels from moving before transferring into or from the wheelchair.

---

### **CAUTION**

**When transferring, position yourself as far back as possible in the seat. This will prevent broken screws, damaged upholstery and the possibility of the wheelchair tipping forward.**


---

*NOTE: For this procedure, refer to FIGURE 9.1.*

*NOTE: This activity may be performed independently provided you have adequate mobility and upper body strength.*

During independent transfer, little or no seat platform will be beneath you. Use a transfer board if at all possible.

- Position the wheelchair as close as possible along side the seat to which you are transferring, with the casters pointing parallel to it.
- Engage clutches. Refer to [Engaging/Disengaging the Clutches](#) on page 69.
- Shift body weight into seat with transfer.

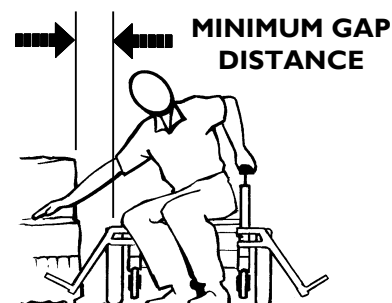

**FIGURE 9.1** Transferring To and From Other Seats



## Reaching, Leaning and Bending - Forward

*NOTE: For this procedure, refer to FIGURE 9.2 on page 25.*

Many activities require the wheelchair user to reach, bend and transfer in and out of the wheelchair. These movements will cause a change to the normal balance, center of gravity, and weight distribution of the wheelchair. To determine and establish your particular safety limits, practice bending, reaching and transferring activities in several combinations in the presence of a qualified healthcare professional before attempting active use of the wheelchair.

1. Position the front casters as far forward as possible.
2. Engage clutches and, if applicable, wheel locks\*.

---

### **⚠ WARNING**


#### **Risk of Serious Injury or Damage**

**Improper positioning while leaning or bending could cause the wheelchair to tip forward resulting in serious injury or damage.**

**To assure stability and proper operation of your wheelchair, you must at all times maintain proper balance. Your wheelchair has been designed to remain upright and stable during normal daily activities as long as you DO NOT move beyond the center of gravity. DO NOT lean forward out of the wheelchair any further than the length of the armrests.**

**DO NOT attempt to reach objects if you have to move forward in the seat or pick them up from the floor by reaching down between your knees.**

---

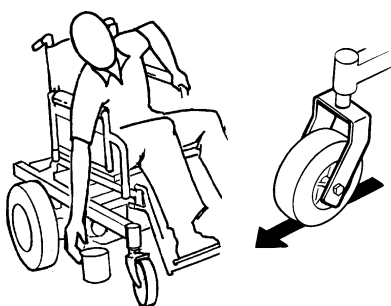

**FIGURE 9.2** Reaching, Leaning and Bending - Forward

## Reaching and Bending - Backward

---

### **⚠ WARNING**

**DO NOT lean over the top of the back upholstery. This will change your center of gravity and may cause you to tip over.**

---

*NOTE: For this procedure, refer to FIGURE 9.3.*

1. Position wheelchair as close as possible to the desired object.
2. Point the front casters as far forward as possible to create the longest possible wheelbase.
3. Reach back only as far as your arm will extend without changing your sitting position.



**FIGURE 9.3** Reaching and Bending - Backward

# SECTION 10—SAFETY INSPECTION/TROUBLESHOOTING

*NOTE: Every six months or as necessary take your wheelchair to a qualified technician for a thorough inspection and servicing. Regular cleaning will reveal loose or worn parts and enhance the smooth operation of your wheelchair. To operate properly and safely, your wheelchair must be cared for just like any other vehicle. Routine maintenance will extend the life and efficiency of your wheelchair.*

## Safety Inspection Checklists

Initial adjustments should be made to suit your personal body structure needs and preference. Thereafter follow these maintenance procedures:

### Inspect/Adjust Initially

- Ensure wheelchair rolls straight (no excessive drag or pull to one side).
- Ensure arms are secure but easy to release and adjustment levers engage properly.
- Ensure adjustable height arms operate and lock securely.
- Ensure arm pivot points are free of wear and looseness.
- Inspect seat and back upholstery for rips or sagging.
- Ensure armrest pads sit flush against arm tubes.
- Ensure seat release latch is functional. Replace if necessary.
- Ensure wheel mounting bolts are secure on drive wheels.
- Ensure no excessive side movement or binding when drive wheels are lifted and spun when disengaged (freewheeling).
- Ensure wheel/fork assembly has proper tension when caster is spun. Caster should come to a gradual stop.

---

### CAUTION

**As with any vehicle, wheels and tires should be checked periodically for cracks and wear and should be replaced as necessary.**

- 
- Loosen/tighten caster locknut if wheel wobbles noticeably or binds to a stop.
  - Ensure all caster/wheel/fork/headtube fasteners are secure.
  - Inspect tires for flat spots and wear.
  - Clean upholstery and armrests.
  - Ensure that casters are free of debris.

### **Inspect/Adjust Weekly**

- Inspect tires for flat spots and wear.
- Ensure that casters are free of debris.

### **Inspect/Adjust Monthly**

- Ensure arm pivot points are free of wear and looseness.
- Ensure wheel mounting bolts are secure on drive wheels.
- Ensure no excessive side movement or binding when drive wheels are lifted and spun when disengaged (free-wheeling).
- Ensure wheel/fork assembly has proper tension when caster is spun. Caster should come to a gradual stop.

---

### **CAUTION**

**As with any vehicle, wheels and tires should be checked periodically for cracks and wear and should be replaced as necessary.**

---

- Loosen/tighten caster locknut if wheel wobbles noticeably or binds to a stop.
- Ensure all caster/wheel/fork/headtube fasteners are secure.
- Inspect for any loose hardware on the wheelchair.
- Inspect seat positioning strap for any signs of wear. Ensure buckle latches. Verify hardware that attaches strap to frame is secure and undamaged. Replace if necessary.
- Ensure that casters are free of debris.

### **Inspect/Adjust Periodically**

- Ensure wheelchair rolls straight (no excessive drag or pull to one side).
- Ensure arms are secure but easy to release and adjustment levers engage properly.
- Ensure adjustable height arms operate and lock securely.
- Ensure armrest pads sit flush against arm tubes.
- Inspect seat and back upholstery for rips or sagging.
- Ensure seat release latch is not worn. Replace if necessary.
- Clean upholstery and armrests.
- Inspect charger AC power cord for damage. Replace if necessary.
- Check that all labels are present and legible. Replace if necessary.
- Ensure that casters are free of debris.
- Inspect electrical components for signs of corrosion. Replace if corroded or damaged.

## Troubleshooting Guide

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
Batteries draw excessive current when charging.	Battery failure.  Electrical malfunction.	Have batteries checked for shorted cell. Replace if necessary.  Contact dealer/Invacare for service.
Battery indicator flashes the charge level is low - immediately after recharge.	Battery failure.  Malfunctioning battery charger.  Electrical malfunction	Check batteries for shorted cell. Replace if necessary.  Contact dealer/Invacare for service.  Contact dealer/Invacare for service.
Battery indicator flashes the charge level is low - too soon after being recharged.	Batteries not charged.  Weak batteries.	Have charger checked.  Replace batteries if necessary. Contact dealer/Invacare for service.
Motor “chatters” or runs irregular.	Electrical malfunction.	Contact dealer/Invacare for service.
Joystick erratic or does not respond as desired.	Damaged motor coupling.  Electrical malfunction.  Controller programmed improperly.	Contact dealer/Invacare for service.  Contact dealer/Invacare for service.  Reprogram controller (refer to electronics owner’s manual supplied with wheelchair).
Wheelchair does not respond to commands.	Poor battery terminal connection.	Have terminals cleaned.
Power indicator off - even after recharging.	Electrical malfunction.	Contact dealer/Invacare for service.


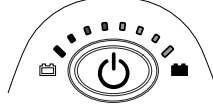
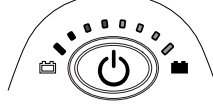
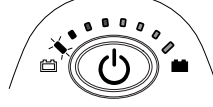
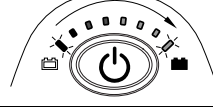

## Troubleshooting - Electrical

*NOTE: For additional troubleshooting information and explanation of error codes, refer to the individual Electronics Manual supplied with each wheelchair*

### SPJ™ + Joysticks

The joystick information gauge and the service indicator give indications of the type of fault or error detected by the control module. When a fault is detected, the wheelchair may stop and not drive. The LEDs on the information gauge may flash in a particular pattern or the service indicator light will flash. The number or type of flashes indicates the nature of the error. If multiple errors are found, only the first error encountered by the control module will be displayed.

## Information Gauge Display Diagnostics

DISPLAY	DESCRIPTION	DEFINITION	COMMENTS
 <p>Information Gauge Display</p>			
	All LEDs are off.	Power is off.	
	All LEDs are on.	Power is on.	Fewer than three LEDs on implies reduced battery charge.
	Left RED LED is flashing.	Battery charge is low.	The batteries should be charged as soon as possible.
	Left to Right “chase” alternating with steady display.	Joystick is in programming, inhibit and/or charging mode.	The steady LEDs indicate the current state of the battery charge.
	All LEDs are flashing slowly.	Joystick has detected Out-of-Neutral-at-Power-Up mode.	Release the joystick back to Neutral.

## Service Indicator Light Diagnostics

NUMBER OF FLASHES	ERROR CODE DESCRIPTION	POSSIBLE SOLUTION
1	User Fault	Release joystick to neutral and try again.
2	Battery Fault	Charge the batteries. Refer to <a href="#">Charging Batteries</a> on page 10. Check that battery cables are connected properly. Refer to <a href="#">Connecting Battery Cables</a> on page 59. If necessary, replace batteries. Refer to <a href="#">Batteries</a> on page 53.
3	Left Motor Fault	Contact Invacare/Dealer for service.
4	Right Motor Fault	Contact Invacare/Dealer for service.
5	Left Park Brake Fault	Contact Invacare/Dealer for service.
6	Right Park Brake Fault	Contact Invacare/Dealer for service.
7	Remote Fault	Check to make sure joystick is connected properly. Contact Invacare/Dealer for service.
8	Controller Fault	Contact Invacare/Dealer for service.
9	Communications Fault	Contact Invacare/Dealer for service.
10	General Fault	Contact Invacare/Dealer for service.
11	Incompatible or incorrect Remote	Wrong type of remote connected. Contact Invacare/Dealer for service.

## Checking Battery Charge Level

The following “Do’s” and “Don’ts” are provided for your convenience and safety.

DON'T	DO
Don't perform any installation or maintenance without first reading this manual.	Read and understand this manual and any service information that accompanies a battery and charger before operating the wheelchair.
Don't perform installation or maintenance of batteries in an area that could be damaged by battery spills.	Move the wheelchair to a work area before cleaning terminals, or opening battery box.
Don't make it a habit to discharge batteries to the lowest level.	Recharge as frequently as possible to maintain a high charge level and extend battery life.
Don't use randomly chosen batteries or chargers.	Follow recommendations in this manual when selecting a battery or charger.
Don't put new batteries into service before charging.	Fully charge a new battery before using.
Don't tip or tilt batteries.	Use a carrying strap to remove, move or install a battery.
Don't tap on clamps and terminals with tools.	Push battery clamps on the terminals. Spread clamps wider if necessary.
Don't mismatch your battery and chargers.	Use <b>ONLY</b> a GEL charger for a GEL battery.

# SECTION 11—WHEELCHAIR OPERATION

## ⚠ WARNING

After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware is tightened securely - otherwise injury or damage may result.


Set-up of the Electronic Control Unit is to be performed **ONLY** by a qualified technician. The final adjustments of the controller may affect other activities of the wheelchair. Damage to the equipment could occur if improperly set-up or adjusted.

## Operating the Wheelchair

### Turning the Power On/Off

*NOTE: FIGURE 11.1.*

1. To turn the power On, press the On/Off button.
2. Turning the power Off can be achieved by pressing the On/Off button.

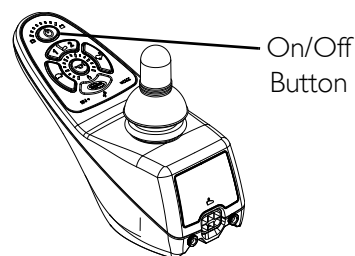

**FIGURE 11.1** Turning the Power On/Off

### Using the Joystick to Drive the Wheelchair

## ⚠ WARNING

**DO NOT** operate wheelchair on an incline while in an elevated position. Otherwise, the wheelchair may tip over and injury or damage may occur.

*NOTE: For this procedure, refer to FIGURE 11.2.*

The joystick is located on the joystick housing and provides smooth control of speed and direction. It is equipped with 360 degrees of mobility for ease of operation. The joystick is spring-loaded, and automatically returns to the upright (neutral) position when released. Pushing the joystick in a given direction causes the wheelchair to move in that direction.

The joystick has proportional drive control, meaning that the further it is pushed from the upright (neutral) position, the faster the wheelchair moves. The maximum speed, however, is limited by the speed control setting.

To slow the wheelchair to a stop, simply release the joystick. The wheelchair has automatic speed and direction compensation to minimize corrections.


When first learning to drive, select a slow speed and try to drive the wheelchair as slowly as possible by pushing the joystick slightly forward. This exercise will help you learn to utilize the full potential of the proportional control and allow you to start and stop smoothly.



To drive the wheelchair, perform the following:

1. Adjust speed. Refer to Speed Control Buttons on page 34.
2. Turn the power On. Refer to Turning the Power On/Off on page 32.
3. Maneuver the joystick in the following manner:


MOVEMENT	ACTION
FORWARD	Push joystick forward, towards the front of the wheelchair.
REVERSE	Pull joystick back, towards the rear of the wheelchair.
Turn RIGHT	Move joystick toward the right side of the wheelchair.
Turn LEFT	Move joystick toward the left side of the wheelchair.
STOP	Release the joystick and the wheelchair will slow to a stop.

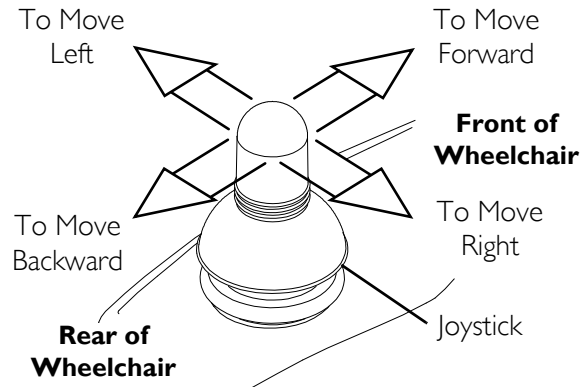

**FIGURE 11.2** Using the Joystick to Drive the Wheelchair

## SPJ™ + Joystick Switches and Indicators

*NOTE: For the following information, refer to FIGURE 11.3.*

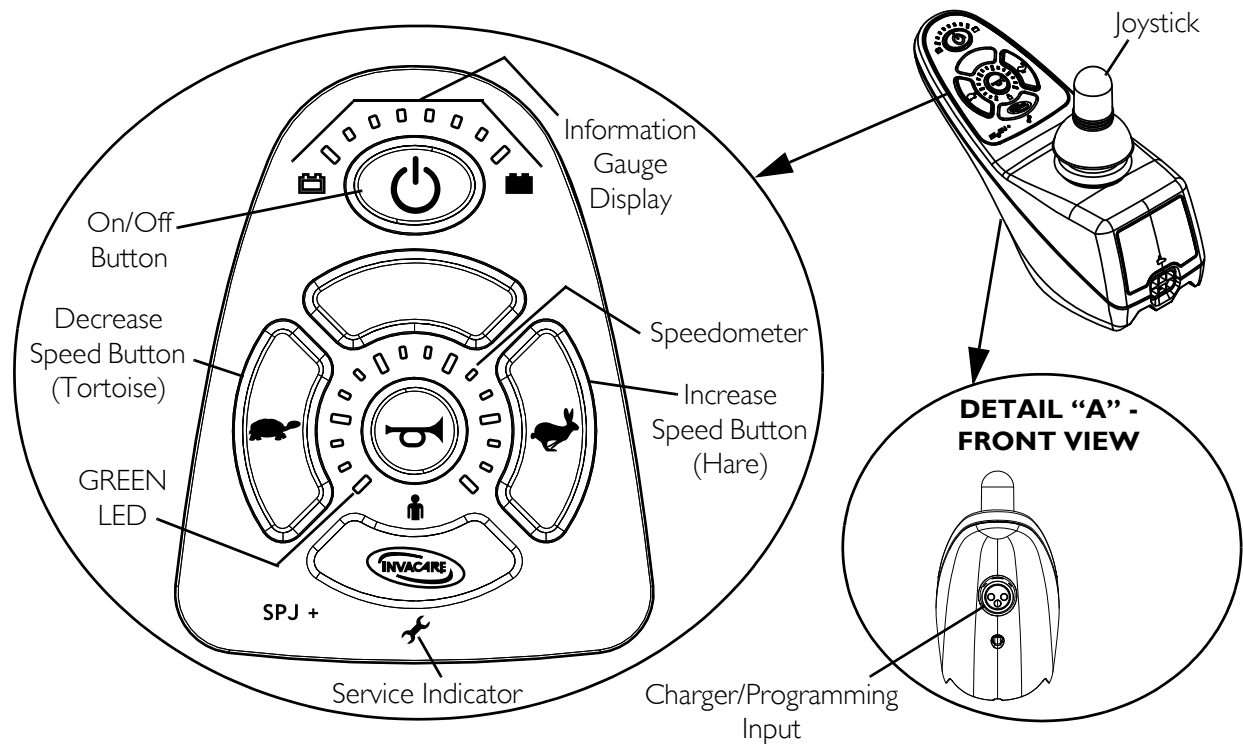

**FIGURE 11.3** SPJ™ + Joystick Switches and Indicators



## On/Off Button









This button is located at the front of the joystick housing. It is used to turn the wheelchair on and off, to remove the joystick from sleep mode (if programmed) and to lock or unlock the joystick (if programmed).

## Speedometer

The speedometer is used to show the maximum speed. The right-most LED indicates current maximum speed setting. The bottom left GREEN LED flashes to indicate that the joystick is in speed limit mode. Speed limit mode limits the drive speed to a pre-programmed value, typically when the seat has been elevated and the wheelchair is required to drive at 20% speed.

## Speed Control Buttons

The speed control buttons (tortoise button () and hare button () are used to set and adjust the maximum speed.

1. To adjust the speed, perform one of the following:
  - Adjust Speed in 20% Increments (5 Speed Mode) - Press the tortoise button () or hare button () to decrease/increase the speed in 20% increments. The larger bars in the speedometer will light.
  - Adjust Speed in Smaller Increments (VSP Mode) - Perform the following steps:
    - i. Press and hold both the tortoise button () and hare button () until the joystick beeps.
    - ii. Perform one of the following:
      - Press the tortoise button () or hare button () to decrease/increase the speed in 20% increments. The larger bars in the speedometer will light.
      - Press and hold the tortoise button () or hare button () to decrease/increase the speed in smaller increments. The smaller bars in the speedometer will light.

## Joystick

The joystick has proportional drive control, meaning that further the joystick is pushed from the upright (neutral) position, the faster the wheelchair or seat moves. Your top speed, however, is limited by the programmed settings.

To slow the wheelchair to a stop, simply release the joystick. The wheelchair has automatic speed and direction compensation to minimize corrections.

## Charger/Programming Input

The charger/programming input is located at the front of the joystick housing. This provides easy access for charging the wheelchair batteries. This port also serves as the Remote Programmer Communication connection. Driving is prevented while the system is charging.

## Service Indicator

The AMBER service indicator will light when an error or fault occurs. Refer to electronics manual supplied with wheelchair for a listing of the flash codes and what they indicate.

## Information Gauge Display

Located on the front of the joystick housing, it provides the following information to the user on the status of the wheelchair -

1. Power is on.
2. True state-of-battery-charge, including notification of when the battery requires charging:
  - A. GREEN LEDs are lit, indicating well charged batteries.
  - B. AMBER LEDs are lit, indicating batteries are moderately charged. Recharge batteries before taking a long trip.
  - C. RED LEDs are lit, indicating batteries are running out of charge. Recharge batteries as soon as possible.

The Information Gauge display also serves as a system diagnostic device when a fault is detected by the control module. A specific number of flashes of the LEDs indicate the type of fault detected. Refer to Information Gauge Display Diagnostics on page 30 for the diagnostic indications of the wheelchair status.

# SECTION 12—FRONT RIGGINGS

## ⚠ WARNING

After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware is tightened securely - otherwise injury or damage may result.

While the wheelchair is moving, minimum ground clearance for the front rigging is three inches. If the wheelchair is not moving, the front rigging **MUST** maintain a minimum of one inch ground clearance - otherwise personal injury and damage may result.

## Installing/Removing the Footrests

*NOTE: For this procedure, refer to FIGURE 12.1.*

### Installing

1. Turn footrest to the side (open footplate is perpendicular to wheelchair) and position mounting holes in the footrest hinge plates with hinge pins on the wheelchair frame.


2. Install the footrest hinge plates onto the hinge pins on the wheelchair frame.

3. Push the footrest towards the inside of the wheelchair until it locks into place.

*NOTE: The footplate will be on the inside of the wheelchair when locked in place.*

4. Repeat STEPS 1-3 for the opposite side of the wheelchair.

3. Repeat STEPS 1-2 for the opposite side of the wheelchair

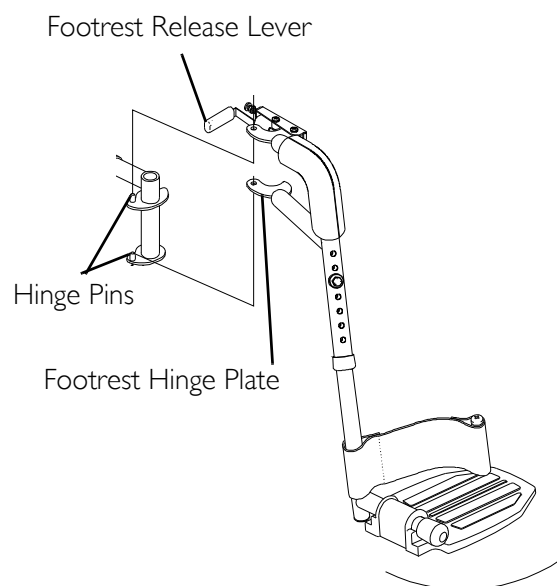

**FIGURE 12.1** Installing/Removing the Footrests

### Removing

1. Push the footrest release lever inward and rotate the footrest outward.
2. Lift up on footrest and remove from the wheelchair.

## Adjusting the Footrest Height


### Model PW93, PW93E, and PW93ST Footrests

*NOTE: For this procedure, refer to FIGURE 12.2.*

1. Remove any accessory from the footrest(s).
2. Remove the footrest from the wheelchair. Refer to Installing/Removing the Footrests on page 36.

*NOTE: Lay footrest on a flat surface to simplify procedure.*

3. Remove the hex bolt, coved washers and locknut that secure the lower footrest to the footrest support.
4. Reposition the lower footrest to the desired height.
5. Reinstall hex bolt, coved washers and locknut that secure lower footrest to footrest support. Tighten securely.
6. Repeat STEPS 1-5 for the other footrest, if necessary.
7. Reinstall the footrest(s) onto the wheelchair. Refer to Installing/Removing the Footrests on page 36.
8. Reinstall any accessory onto the footrest(s).

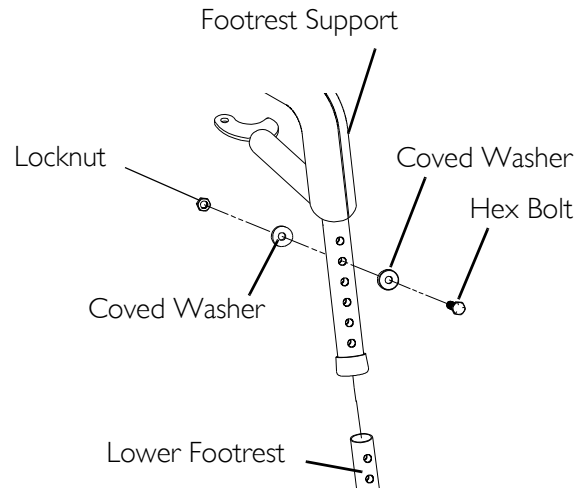


**FIGURE 12.2** Adjusting the Footrest Height - Model PW93, PW93E, and PW93ST Footrests

### Model 93M, 904A, PAL4A, and PAS4A Footrests

*NOTE: For this procedure, refer to FIGURE 12.3.*

1. Loosen, but do not remove the lug bolt and locknut that secure the lower footrest to the footrest support.
2. Reposition the lower footrest to the desired height.
3. Securely tighten the lug bolt and locknut that secure the lower footrest to the footrest support.
4. Repeat STEPS 1-3 for the other footrest, if necessary.

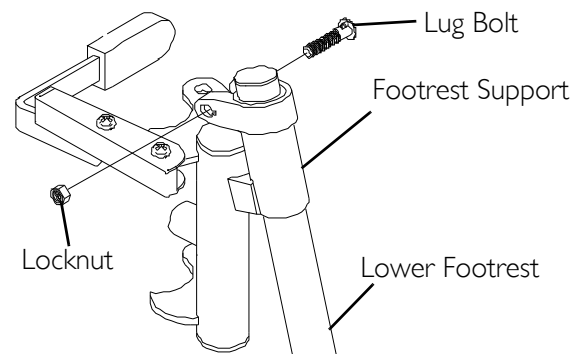

*how. 904A, PAL4A and PAS4A footrests adju*


**FIGURE 12.3** Adjusting the Footrest Height - Model 93M, 904A, pal4a, and PAS4A Footrests

### Replacing Heel Loops

*NOTE: For this procedure, refer to FIGURE 12.4.*

1. Note the position of hex bolt, coved washers and locknut for reinstallation.

2. Remove the hex bolt, coved washers and locknut that secure the lower footrest to the footrest support.
3. Remove the lower footrest.
4. Remove the phillips bolt, spacer and locknut that secure the existing heel loop to the lower footrest.
5. Slide the existing heel loop off the lower footrest.
6. Replace heel loop.
7. Reverse STEPS 1-6 to reassemble.

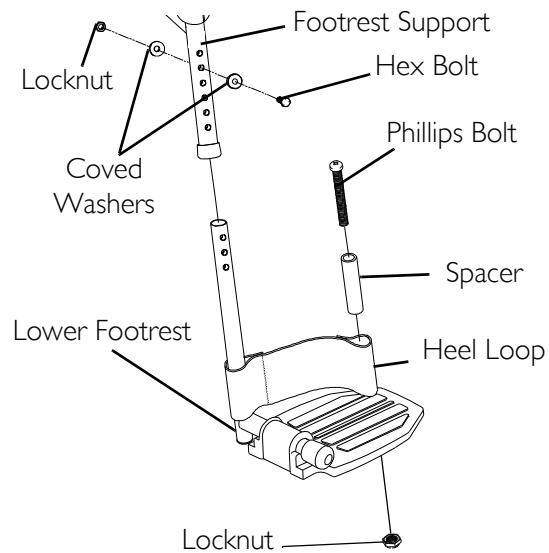

**FIGURE 12.4** Replacing Heel Loops

*NOTE: When securing heel loop to lower footrest, tighten the phillips screw and locknut until the spacer is secure.*


## Installing/Removing Elevating Legrests

*NOTE: For this procedure, refer to FIGURE 12.5.*

### Installing

1. Turn legrest to side (open footplate is perpendicular to wheelchair) and position mounting holes in the legrest hinge plates with hinge pins on the wheelchair frame.
2. Install the legrest hinge plates onto the hinge pins on the wheelchair frame.
3. Rotate legrest toward the inside of the wheelchair until it locks in place.

*NOTE: The footplate will be on the inside of the wheelchair when locked in place.*

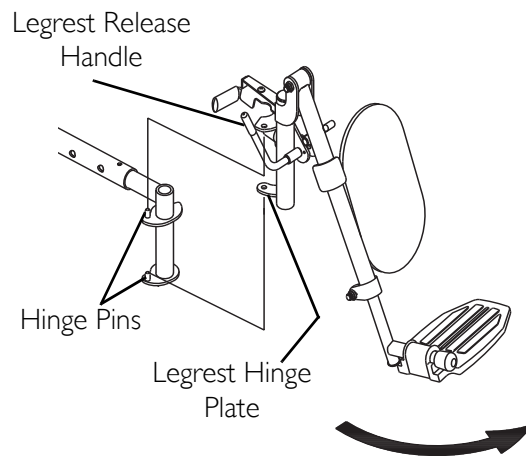

**FIGURE 12.5** Installing/Removing Elevating Legrests

4. Repeat STEPS 1-3 for the other legrest.
5. After seated in wheelchair, adjust footplate to correct height by loosening nut and sliding the lower footrest assembly up or down until desired height is achieved.

### Removing

1. Push the legrest release handle toward the inside of the wheelchair (facing the front of the wheelchair) and swing the legrest to the outside of the wheelchair.

2. Lift up on the legrest and remove from the wheelchair.
3. Repeat STEPS 1-2 for the other legrest.

## Raising/Lowering Elevating Legrests and/or Adjusting Calfpads

*NOTE: For this procedure, refer to FIGURE 12.6 on page 39.*

### Raising/Lowering Elevating Legrests


1. Perform one of the following:
  - Raising - Pull back on the release lever until the leg is at the desired height.
  - Lowering - Support leg with one hand. Push release lever downward with other hand.

### Adjusting Calfpads

1. Turn calfpad towards the outside of the wheelchair.
2. Slide calfpad up or down until desired position is obtained.

*NOTE: If one of the top two calfpad adjustment positions is being used, the legrest will need to be raised to avoid interference with the front stabilizers while going over obstacles or going up and down ramps. Refer to Raising/Lowering Elevating Legrests on page 39.*

3. Turn the calfpad toward the inside of the wheelchair.

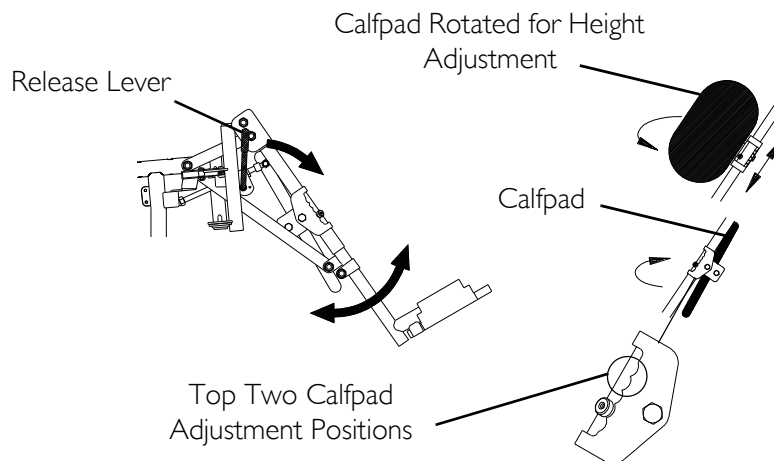

**FIGURE 12.6** Raising/Lowering Elevating Legrests and/or Adjusting Calfpads

## Adjusting/Replacing Telescoping Front Rigging Support

*NOTE: For this procedure, refer to FIGURE 12.7 on page 40.*

*NOTE: When adjusting the telescoping front rigging support depth, ensure the footplate does not interfere with the caster wheel rotation.*


1. Lift up on the crossbraces until they release from the H-blocks on the wheelchair frame.
2. Remove the long socket bolt and locknut that secure the front H-block and spacer to the wheelchair frame and telescoping front rigging support.
3. Remove the front, and rear short socket bolts and the two locknuts that secure telescoping front rigging support to wheelchair.
4. Perform one of the following:
  - Adjusting:

*NOTE: Telescoping front rigging supports may be extended up to 2-inches from the wheelchair frame in 1-inch increments. This adjustment does not effect seat depth.*

- i. Position the telescoping front rigging support to the desired depth.
- Replacing:
    - i. Remove the existing telescoping front rigging support from the wheelchair frame.
    - ii. Insert the new telescoping front rigging support into the wheelchair frame.
    - iii. Position the telescoping front rigging support to the desired depth.

*NOTE: The footplate will be on the inside of the wheelchair when locked in place.*

5. Using the front and rear socket bolts and two locknuts, secure the telescoping front rigging support to the wheelchair frame as shown in FIGURE 12.7.
6. Using the long socket bolt and locknut, secure the H-block and spacer to the wheelchair frame and the telescoping front rigging support as shown in FIGURE 12.7.
7. Snap crossbraces into H-blocks. Wiggle crossbraces to make sure they are secured in the H-blocks.
8. If necessary, repeat STEPS 1-8 on remaining telescoping front rigging support.

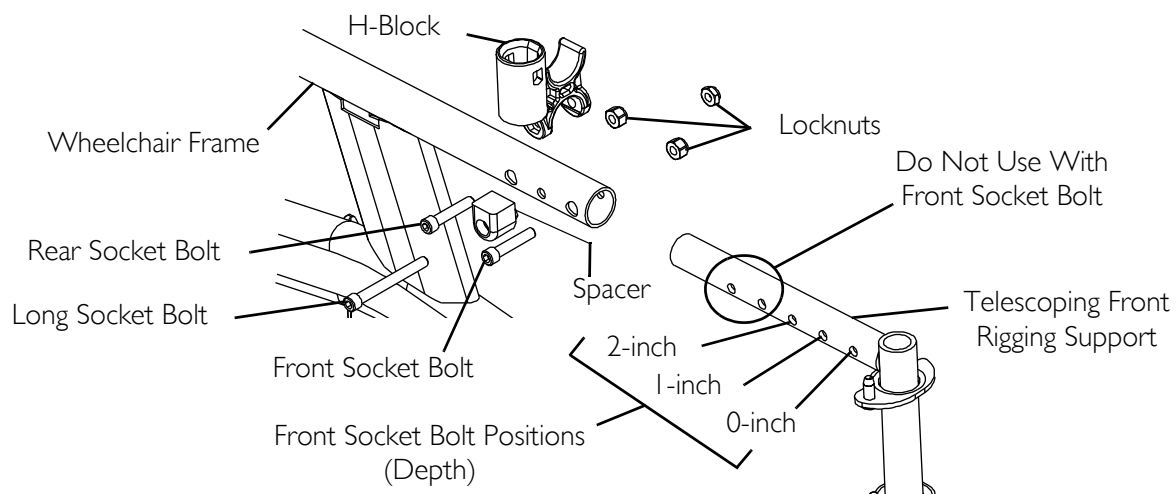

**FIGURE 12.7** Adjusting/Replacing Telescoping Front Rigging Support



# SECTION 13—ARMS

## ⚠ WARNING

After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

## Adjusting Armrest Height

NOTE: For this procedure, refer to FIGURE 13.1.


## ⚠ WARNING

Make sure the height adjustment lever is in the locked position before using the wheelchair.

1. Unlock the armrest by flipping the height adjustment lever on the top front of the armrest to the up (horizontal) position.
2. Adjust armrest to one of five positions.

NOTE: Height adjustment lever MUST be in the unlocked position when placing armrest into the arm assembly.

3. Lock the armrest by pressing the height adjustment lever into the down (vertical) position when the desired armrest height is achieved.
4. Repeat STEPS 1-3 for other armrest.

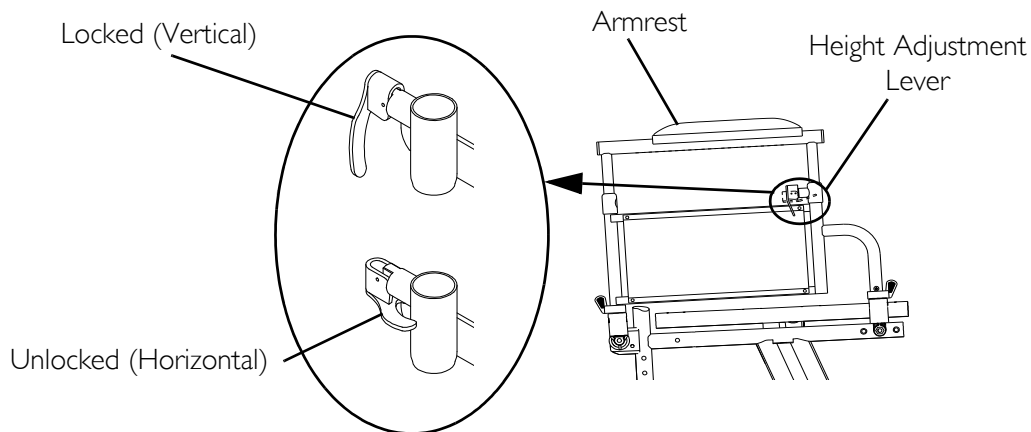

**FIGURE 13.1** Adjusting Armrest Height

## Using Swing-Back Arms

### **⚠ WARNING**


**Make sure the armrest release lever is in the locked position before using the wheelchair.**

*NOTE: For this procedure, refer to FIGURE 13.2.*

1. Unlock the swing-back arms by rotating the armrest release lever toward the outside of the wheelchair.
2. Pull the front of the swing-back arm straight up/out of the arm socket and toward the rear of the wheelchair.
3. To use the swing-back arm, push the swing-back arm toward the front of the wheelchair and then downward into the arm socket.

*NOTE: Armrest release lever MUST be in the unlocked position when placing armrest into the arm sockets.*

4. Lock the swing-back arms by rotating the armrest release lever toward the inside of the wheelchair.

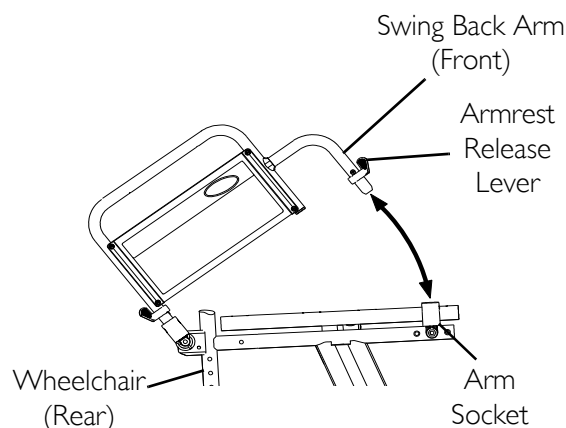

**FIGURE 13.2** Using Swing-Back Arms

## Replacing Desk/Full Length Armrest Pad and/or Clothing Guards

*NOTE: For this procedure, refer to FIGURE 13.3 on page 43.*


*NOTE: This procedure is for fixed height arms only.*

### **Armrest Pad**

1. Remove the two mounting screws that secure the armrest pad to the armrest assembly.
2. Replace armrest pad and securely tighten with the existing mounting screws.
3. Repeat STEPS 1 and 2 for the other armrest.

### **Clothing Guard**

1. Remove the four mounting screws that secure the existing clothing guard to the armrest assembly.
2. Install new clothing guard and securely tighten with the existing mounting screws.
3. Repeat STEPS 1 and 2 for the other clothing guard.

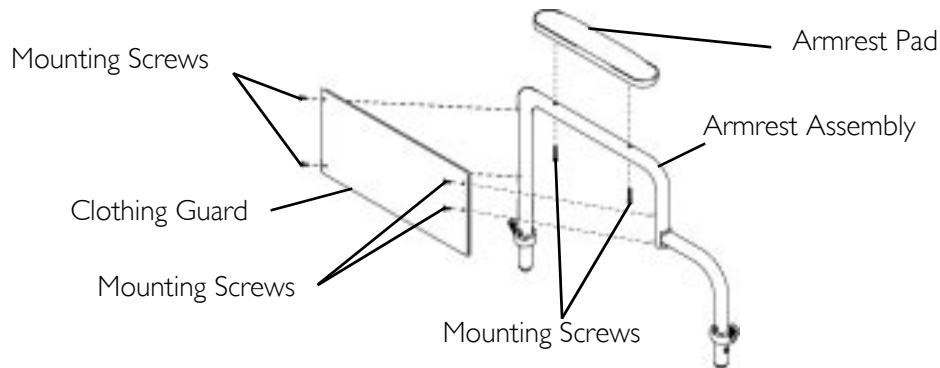

**FIGURE 13.3** Replacing Desk/Full Length Armrest Pad and/or Clothing Guards

## Removing/Installing Armrest

### **⚠ WARNING**

**Make sure the armrest release lever is in the locked position before using the wheelchair.**

*NOTE: For this procedure, refer to FIGURE 13.4.*


### Removing

1. Unlock the armrest by turning the armrest release levers towards the outside of the wheelchair.
2. Lift armrest completely out of arm sockets from the wheelchair.

### Installing

*NOTE: Armrest release levers MUST be in the unlocked position when placing armrests into the arm sockets.*

1. Place armrest into arm sockets located on the side of the wheelchair.
2. Lock the armrest by turning the armrest release levers towards the inside of the wheelchair.

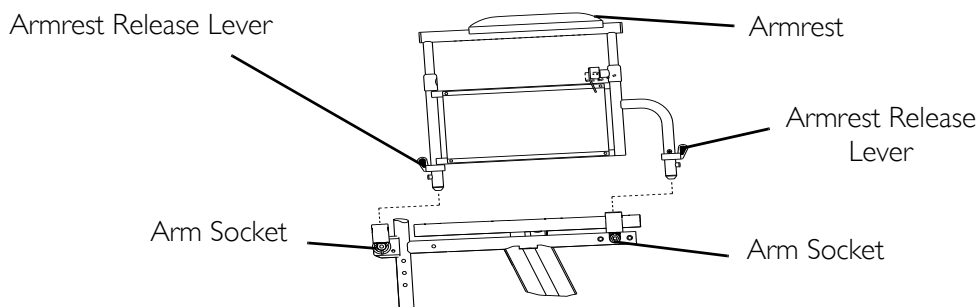

**FIGURE 13.4** Removing/Installing Armrest

# SECTION 14—BACK

## ⚠ WARNING

After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.


## Adjusting the Back Height

NOTE: For this procedure, refer to FIGURE 14.1.

NOTE: Back canes must be adjusted to the same height.

NOTE: Take note of the position and orientation of the back cane mounting hardware before removing. The fold down back canes use two mounting screws to secure the two back canes to the wheelchair frame, the adjustable angle back canes use four mounting screws.

1. Remove the mounting screws and locknuts that secure the two back canes to the back cane mounting tube.
2. Reposition the back canes to the desired height adjustment position as shown in FIGURE 14.1.
3. Reinstall the mounting screws and locknuts that secure the back canes to the back cane mounting tube and securely tighten.

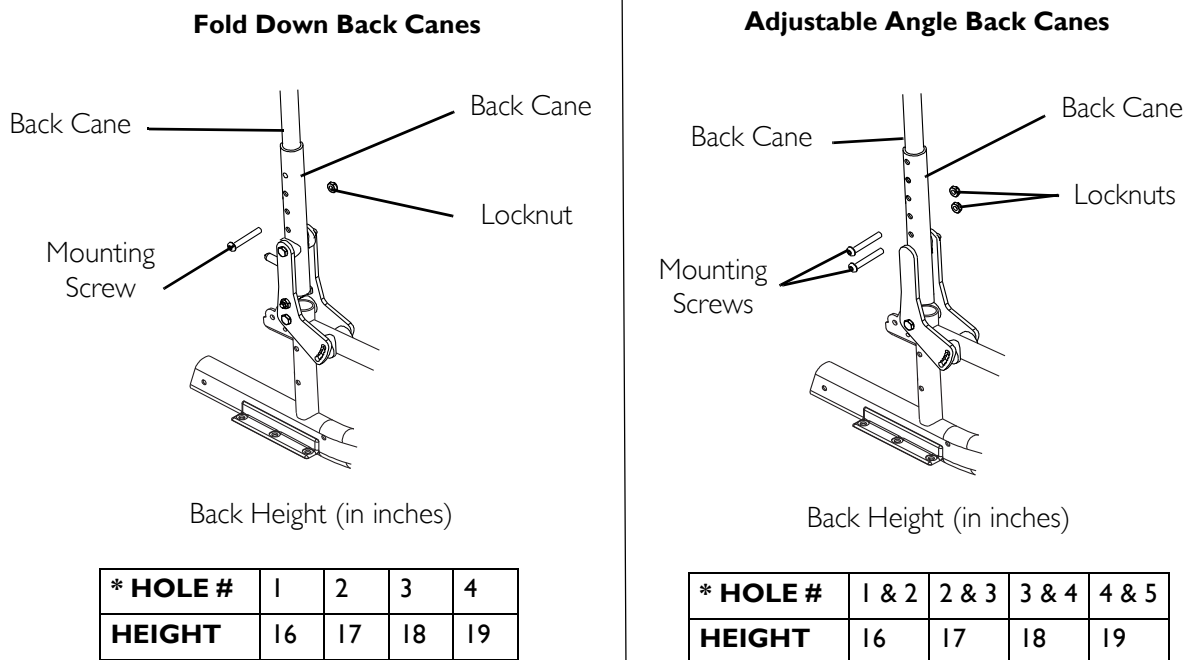

\*NOTE: Holes numbered from bottom to top for reference only. There are no numbers on the back canes or wheelchair frame.

**FIGURE 14.1** Adjusting the Back Height


## Using the Fold Down Back Canes

### **⚠ WARNING**

The back **MUST** be locked securely in place before using the wheelchair.

*NOTE: For this procedure, refer to FIGURE 14.2.*

1. Perform one of the following:
  - Unfold - Push/pull the back toward rear of wheelchair until it locks in place.
  - Fold - Pull up on the back release latches and push the back toward the front of the wheelchair.

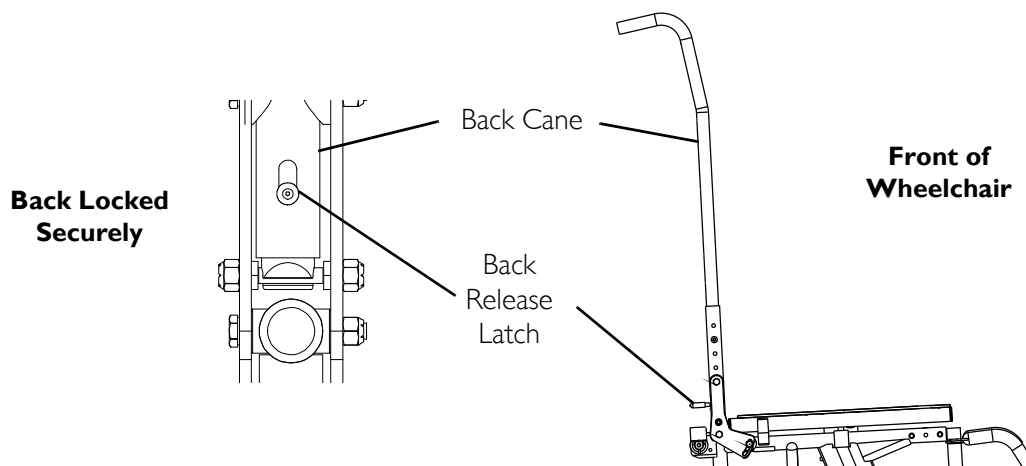

**FIGURE 14.2** Using the Fold Down Back Canes

## Changing the Back Angle

*NOTE: For this procedure, refer to FIGURE 14.3 on page 46.*


### **⚠ WARNING**

The following procedure must be performed by a qualified technician.

The seat depth and back angle of the wheelchair, as well as the large wheel position directly relate to the stability of the wheelchair. Any change to one or any combination of the three may cause the wheelchair to decrease in stability. Use **EXTREME** caution when using a new seating position for the first time.

1. Loosen, but do not remove the rear hex screw that secures the back angle plate to the wheelchair frame.
2. Remove the front hex screw, washer, and locknut that secure the back angle plate to the wheelchair frame.

3. Position the back angle plate to one of six back angles. Refer to Detail "A" of FIGURE 14.3.
4. Reinstall the front hex screw and washer, securely tighten with locknut. Refer to FIGURE 14.3 for correct hardware orientation.
5. Securely tighten the rear hex screw that secures the back adjustment plate to the wheelchair frame.

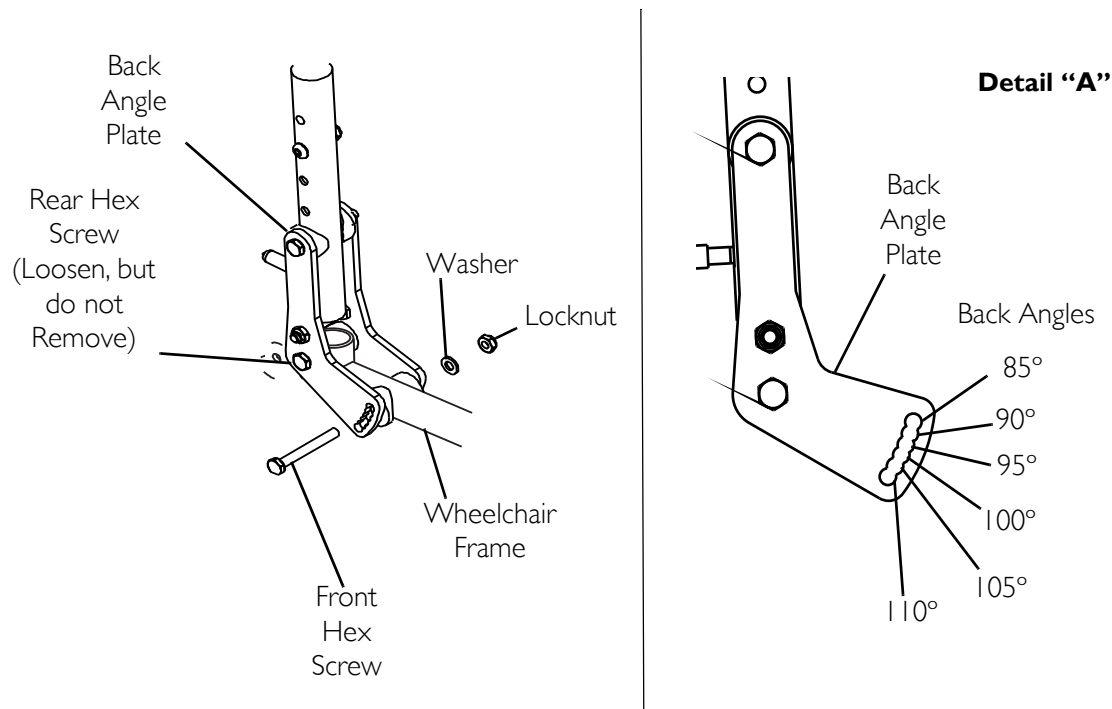

**FIGURE 14.3** Changing the Back Angle

# SECTION 15—UPHOLSTERY/SEAT POSITIONING STRAP


## ⚠ WARNING

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

## Replacing Seat Upholstery

*NOTE: For this procedure, refer to FIGURE 15.1.*

1. Remove the phillips screws and washers that secure the existing seat upholstery to the crossbraces.
2. Remove existing seat upholstery from the crossbraces.
3. Line up mounting holes in new seat upholstery with mounting holes in the crossbraces.
4. Securely tighten with existing phillips screws and washers.

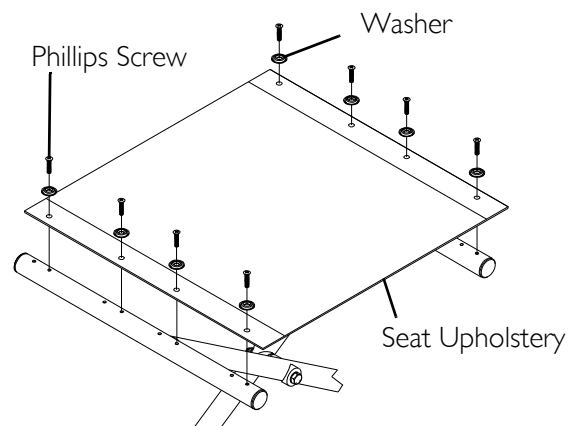

**FIGURE 15.1** Replacing Seat Upholstery

*NOTE: Clean upholstery with warm DAMP cloth and mild detergent to remove superficial soil.*

## ⚠ WARNING

**Laundering or moisture will reduce flame retardation of the upholstery.**


## Replacing Back Upholstery

*NOTE: For this procedure, refer to FIGURE 15.2 on page 48.*

*NOTE: Observe the position and orientation of the back cane mounting hardware before removing. The fold down back canes use two mounting screws to secure the two back canes to the wheelchair frame, all other back canes use four mounting screws.*

1. Remove the two upholstery mounting screws and washers that secure the existing back upholstery to the back canes.
2. Remove the screws and locknuts that secure the two back canes to the wheelchair frame.
3. Remove the two back canes and the back upholstery from the wheelchair frame.

4. Pull the loose back canes out of the existing back upholstery.
5. Slide the loose back canes into the new back upholstery.
6. Secure the back canes to the wheelchair frame with the mounting screws and locknuts removed in STEP 1.
7. Secure the new back upholstery to the back canes with the two existing upholstery mounting screws and washers.

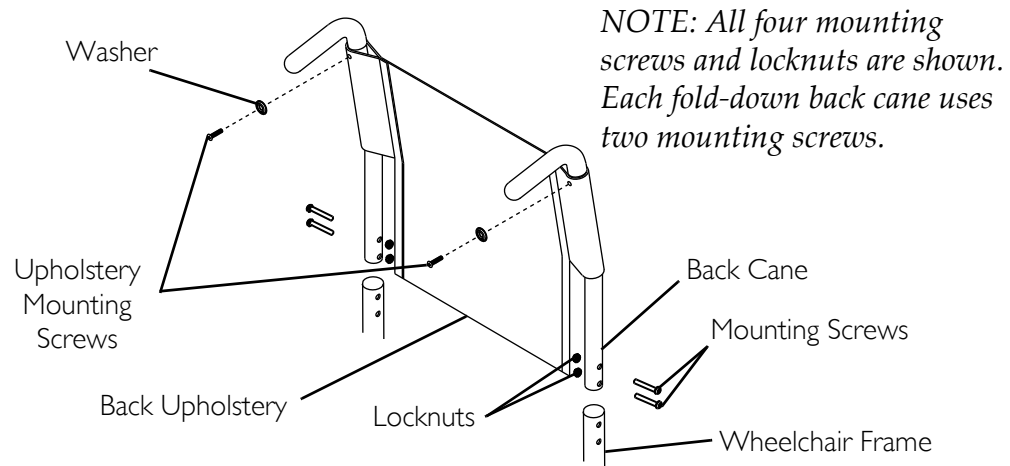

**FIGURE 15.2** Replacing Back Upholstery

## Replacing Seat Positioning Strap


*NOTE: For this procedure, refer to FIGURE 15.3 on page 49.*

1. If necessary, remove the seat cushion from the seat upholstery.
2. Remove the two rear phillips screws and washers that secure the seat upholstery and seat positioning strap to the crossbraces.
3. Remove the two halves of the seat positioning strap from the crossbraces.
4. Position the two new seat positioning strap halves between the seat upholstery and the crossbraces.
5. Reinstall the rear phillips screws and washers that secure the seat upholstery and seat positioning strap to the crossbraces.

*NOTE: Washer is positioned on top of seat upholstery.*

6. If necessary, reinstall the seat cushion onto the seat upholstery.



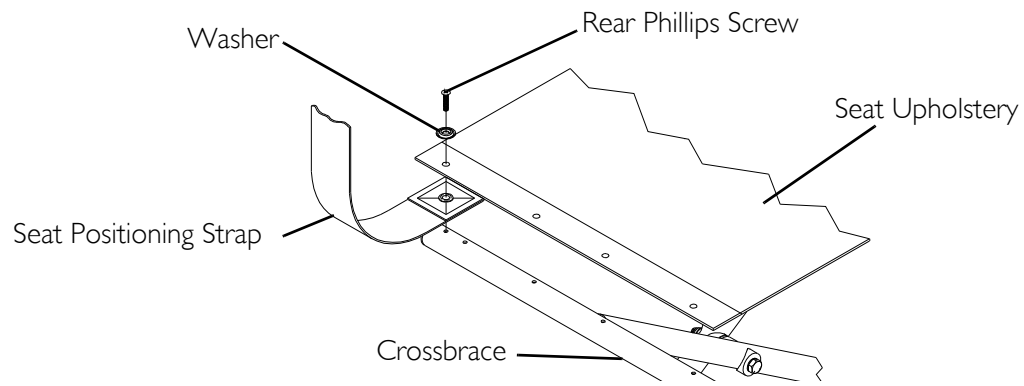

**FIGURE 15.3** Replacing Seat Positioning Strap

# SECTION 16—JOYSTICK

## ⚠ WARNING


After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

## Preparing the Joystick For Use

NOTE: For this procedure, refer to FIGURE 16.1.

NOTE: The joystick is factory installed on the right side of the wheelchair. To reposition the joystick onto the left side of the wheelchair refer to Repositioning the Joystick on page 51. The joystick should be repositioned by a qualified technician.

1. Turn the adjustment lock lever to release the joystick mounting tube from the mounting bracket.
2. Slide joystick mounting tube to the desired position.
3. Turn the adjustment lock lever to secure the joystick mounting tube in the mounting bracket.

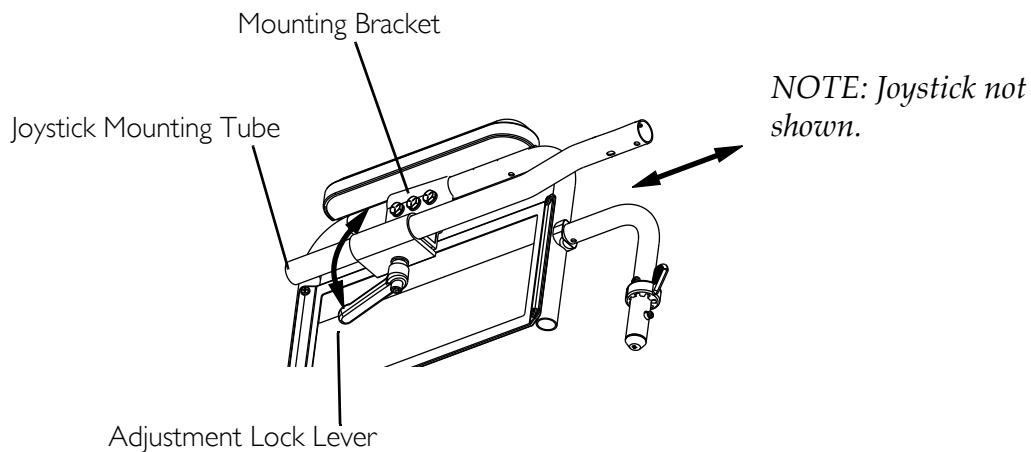


**FIGURE 16.1** Preparing the Joystick For Use

## Repositioning the Joystick

*NOTE: For this procedure, refer to FIGURE 16.2.*

1. Turn the adjustment lock lever to release the joystick mounting tube from the mounting bracket.
2. Remove the joystick from the wheelchair.
3. Remove the three hex screws that secure the halves of the mounting bracket to the arm tube.
4. Reposition mounting bracket on opposite arm tube ensuring the threaded plate of the mounting bracket is on the inside of the arm tube as shown in FIGURE 16.2.
5. Using the three hex mounting screws and washers, secure the halves of the mounting bracket to the arm tube.
6. Slide the joystick mounting tube through the mounting bracket to the desired position.
7. Turn the adjustment lock lever to secure the joystick mounting tube into the mounting bracket.

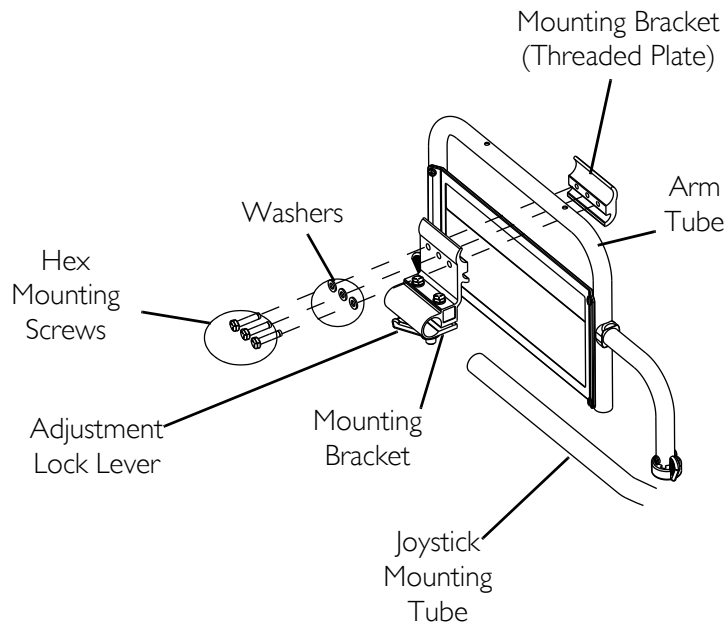

*NOTE: Joystick not shown.*

**FIGURE 16.2** Repositioning the Joystick

# SECTION 17—CASTERS

## ⚠ WARNING

After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware is tightened securely - otherwise injury or damage may result.

Before performing any maintenance, adjustment or service verify that **ON/OFF** switch on the joystick is in the **OFF** position.

## CAUTION

As with any vehicle, the wheels, casters and tires should be checked periodically for cracks and wear and should be replaced.

## Adjusting Forks

NOTE: For this procedure, refer to FIGURE 17.1.

NOTE: It may be necessary to extend the telescoping front rigging support to gain access to the caster headtube hardware. Refer to Adjusting/Replacing Telescoping Front Rigging Support on page 39.

1. Remove the dust cover from the caster headtube.
2. To properly tighten caster journal system and guard against flutter, perform the following check:
  - A. Remove battery boxes from wheelchair. Refer to Installing/Removing the Battery Boxes on page 55.
  - B. Tip back the wheelchair to floor.
  - C. Pivot both forks and casters to top of their arc simultaneously.
  - D. Let casters drop to bottom of arc (wheels should swing once to one-side, then immediately rest in a straight downward position).
  - E. Adjust locknuts according to freedom of caster swing.
3. Test wheelchair for maneuverability.
4. Readjust locknuts, if necessary, and repeat STEPS 1-3 until correct.
5. Snap dust cover into the caster headtube.

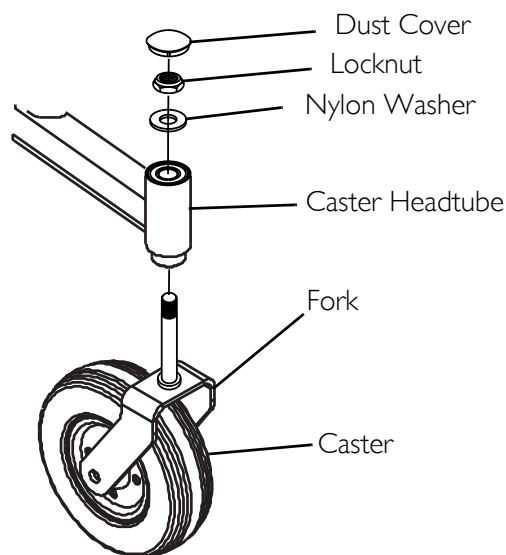


FIGURE 17.1 Adjusting Forks

# SECTION 18—BATTERIES

## Warnings for Handling and Replacing Batteries

### **⚠ DANGER**

#### **Risk of Death or Serious Injury**

Failure to observe these warnings can cause an electrical short resulting in death, serious injury, or damage to the electrical system.

The **POSITIVE (+) RED** battery cable **MUST** connect to the **POSITIVE (+)** battery terminal(s)/post(s).

The **NEGATIVE (-) BLACK** battery cable **MUST** connect to the **NEGATIVE (-)** battery terminal(s)/post(s).

**NEVER** allow any of your tools and/or battery cable(s) to contact **BOTH** battery post(s) at the same time. An electrical short may occur and serious injury or damage may occur.

Install protective caps on positive and negative battery terminals.

Replace cable(s) immediately if cable(s) insulation becomes damaged.

**DO NOT** remove fuse or mounting hardware from **POSITIVE (+)** red battery cable mounting screw.

### **⚠ WARNINGS**

After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware is tightened securely - otherwise injury or damage may result.

Make sure power to the wheelchair is off before performing this section.

The use of rubber gloves and chemical goggles or face shields is recommended when working with batteries.

Invacare strongly recommends that battery installation and battery replacement **ALWAYS** be done by a qualified technician.

**ALWAYS** use a battery lifting strap when lifting a battery. It is the most convenient method and assures that the battery acid will not spill. It also helps to prolong the life of the battery.

**DO NOT** tip the batteries. Keep the batteries in an upright position.

*NOTE: If there is battery acid in the bottom of the battery tray or on the sides of the battery(ies), apply baking soda to these areas to neutralize the battery acid. Before reinstalling the existing or new battery(ies), clean the baking soda from the battery tray or battery(ies) being sure to avoid contact with skin and eyes. Determine source of contamination. Never install/reinstall a battery with a cracked or otherwise damaged case.*

## Using the Proper Batteries


NOTE: For this procedure, refer to FIGURE 18.1 on page 54.

1. Place battery on ground/flat surface.
2. Visually draw a horizontal and vertical centerline through the middle of battery (FIGURE 18.1).
3. Position the battery so that the terminals are above the horizontal centerline.
4. Visually inspect the battery to ensure the correct position of the POSITIVE and NEGATIVE terminals (FIGURE 18.1).

### **⚠ WARNING**

**Batteries with terminal configuration as shown in FIGURE 18.1 must be used. Batteries that have the reverse terminal configuration MUST NOT be used - otherwise injury and damage may occur.**

**Terminals MUST have a cross hole in them as shown below.**

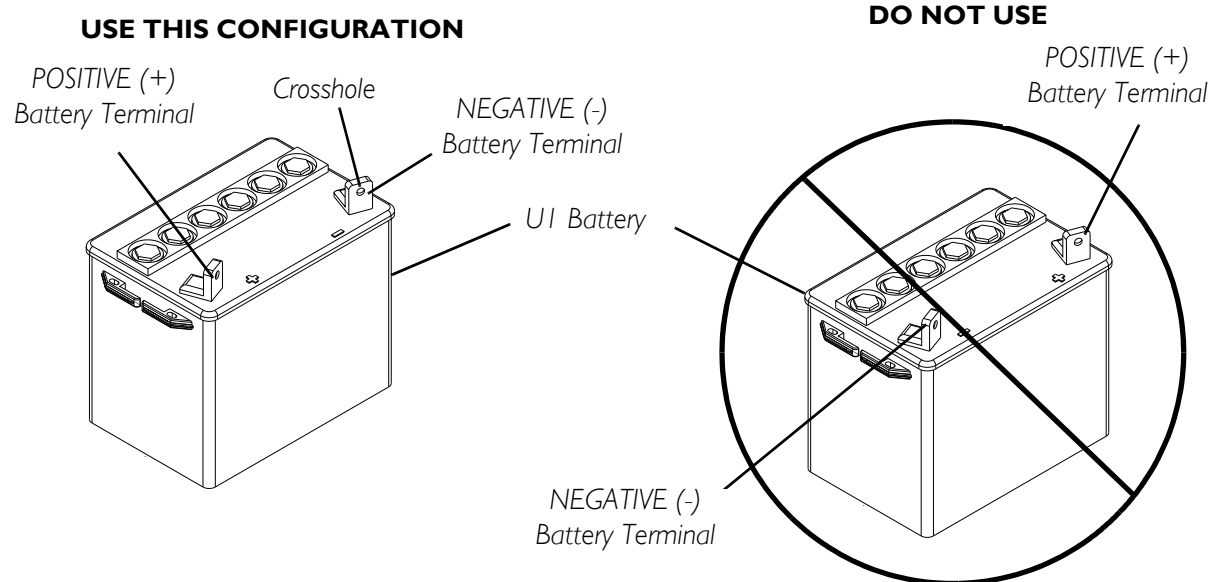

**FIGURE 18.1** Using the Proper Batteries

## Recommended Battery Types

### **⚠ WARNING**

The warranty and performance specifications contained in this manual are based on the use of deep cycle gel cell batteries. Invacare strongly recommends their use as the power source for this unit.

**CAUTION**

**Failure to use the correct battery size and/or voltage may cause damage to your wheelchair and give you unsatisfactory performance.**

CHAIRSEATWIDTH	QTY	VOLTS	BATTERY SIZE	REMARKS
14 inches	2	12	UI	Deep Cycle
16, 18, 20 inches	2	12	22NF or UI	Deep Cycle

1. Remove the battery boxes and tray from the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55 of the manual.
2. Remove the existing batteries from the battery boxes. Refer to Installing/Removing the Batteries Into/From the Battery Boxes on page 57.
3. Clean the new battery terminals. Refer to Cleaning Battery Terminals on page 66.
4. Install the new batteries into the battery boxes. Refer to Installing/Removing the Batteries Into/From the Battery Boxes on page 57 of this manual.
5. Install the battery boxes onto the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55 of the manual.

**Installing/Removing the Battery Boxes****⚠ WARNING**

**NEVER unplug the battery cables by pulling on the wires. Unplug the battery cables by pulling on the connector ONLY. Otherwise injury or damage may result.**

*NOTE: For this procedure, refer to FIGURE 18.2 on page 56.*



*NOTE: To remove the battery boxes from the wheelchair, reverse the following procedure.*

1. Verify that the on/off switch on the joystick is in the off position.
2. Place the front battery box onto the battery tray and connect controller connector to the front battery box.
3. Slide front battery box toward the front of the wheelchair.
4. Place rear battery box onto the battery tray and connect the rear battery box to the front battery box.
5. Connect the front battery box to the controller.

**⚠ WARNING**

**The battery box retaining strap MUST be fastened securely and adjusted properly to hold battery boxes in place before using the wheelchair.**

6. Connect the battery box retaining strap clip together and adjust if necessary.

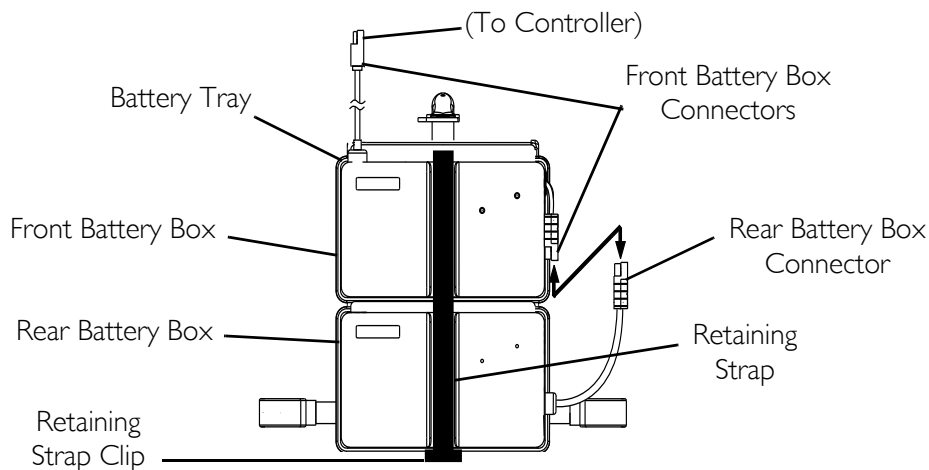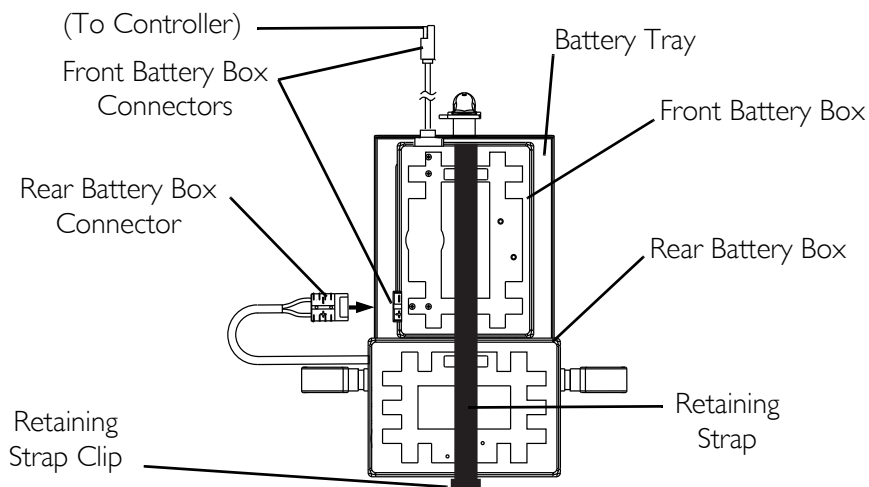**Detail “A” 16, 18, and 20-Inch Wide Wheelchair****Detail “B” 14-Inch Wide Wheelchair****FIGURE 18.2** Installing/Removing the Battery Boxes

## Installing/Removing the Battery Tray

### **⚠ WARNING**

#### **Risk of Serious Injury**


**Improperly installed battery tray can cause instability resulting in serious injury. Ensure batteries and battery tray are installed properly to maintain stability.**

*NOTE: For these procedures, refer to FIGURE 18.3 on page 57.*

*NOTE: To remove the battery tray from the wheelchair, reverse the following procedure.*



1. Attach the key slot bracket located at the front of the battery tray over the crossbrace bolt bushing under the seat upholstery of the wheelchair.
2. Attach the hanger brackets to the wheelchair frame.

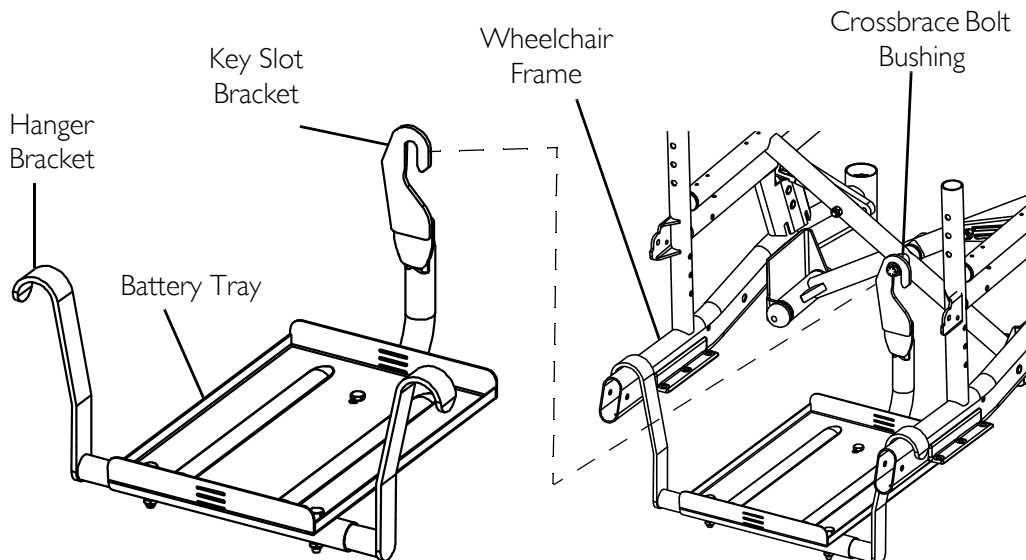

**FIGURE 18.3** Installing/Removing the Battery Tray

## Installing/Removing the Batteries Into/From the Battery Boxes

### **⚠ WARNING**

**Make sure power to the wheelchair is OFF before performing this procedure.**

**The use of rubber gloves and chemical goggles or face shields is recommended when working with batteries.**

**Invacare strongly recommends that battery installation and battery replacement ALWAYS be done by a qualified technician.**

**After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.**

**ALWAYS use a battery lifting strap when lifting a battery. It is the most convenient method and assures that the battery acid will not spill. It also helps to prolong the life of the battery.**

**DO NOT tip the batteries. Keep the batteries in an upright position.**

*NOTE: For this procedure, refer to FIGURE 18.4 on page 58.*

*NOTE: For changing or removing/installing the battery boxes, refer to Installing/Removing the Battery Boxes on page 55.*

*NOTE: To remove a battery from the battery box, reverse the following procedure.*

NOTE: Have the following tools available:

TOOL	QTY	COMMENTS
Battery lifting strap	1	Refer to Note
1/2-inch (6 pt) box wrench	1	Not supplied
7/16-inch (6pt) Box Wrench	1	Not supplied
3/8-inch (6pt) Box Wrench	1	Not supplied
Diagonal cutters	1	Not supplied

NOTE: The battery lifting strap supplied is for Group 22 batteries ONLY. Refer to the battery manufacturer for the proper lifting strap and/or battery tools for U1 battery removal/installation.

NOTE: If there is battery acid in the bottom or on the sides of the battery box(es) or battery(ies), apply baking soda to these areas to neutralize the battery acid. Before reinstalling the new or existing battery(ies), clean the baking soda from the battery box(es) or battery(ies).


NOTE: When securing battery lifting strap to battery, observe polarity markings located on the ends of the battery lifting strap, (+) side to POSITIVE (+) battery terminal/post and (-) side to NEGATIVE (-) battery terminal/post.

1. If necessary, remove the battery boxes from the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55.
2. Secure the battery lifting strap to battery terminal(s)/post(s).

**CAUTION**

Some battery manufacturers mold a carrying strap and/or hold down flanges directly into the battery case. Batteries which interfere with the battery box cannot be used for these applications. Attempting to “wedge” a battery into a battery box may damage the box and/or the battery.

3. Place batteries into battery box bottom.

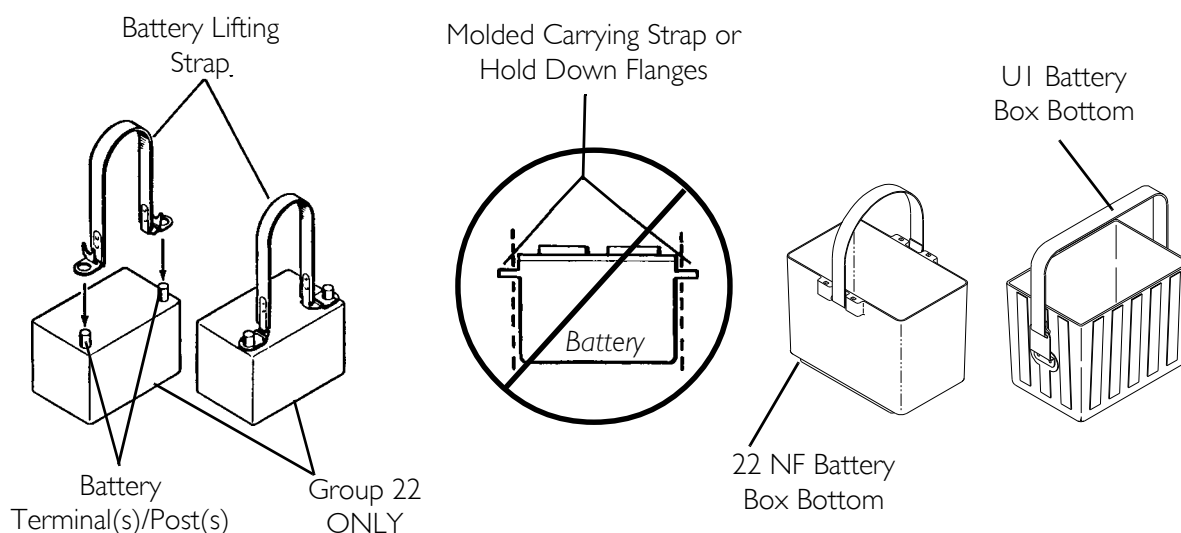

**FIGURE 18.4** Installing/Removing the Batteries Into/From the Battery Boxes

## Connecting Battery Cables

### **⚠ WARNING**

The use of rubber gloves and chemical goggles or face shields is recommended when working with batteries.

**NEVER** allow any of your tools and/or battery cable(s) to contact **BOTH** battery terminal(s)/post(s) at the same time. An electrical short may occur and serious personal injury or damage may occur.


Perform one of the following methods for connecting the battery cable(s):

- Dual U1 Batteries - Use direct mount method. Refer to FIGURE 18.5 and FIGURE 18.6.
- Dual Group 22NF Batteries with Mounting Holes in the Terminal(s)/Post(s) - Use direct mount method. Refer to FIGURE 18.5 and FIGURE 18.6.
- Dual Group 22NF Batteries without Mounting Holes in the Terminal(s)/Post(s) - Use battery clamp method. Refer to FIGURE 18.7, FIGURE 18.8 and FIGURE 18.9.

### **Direct Mount Method**

*NOTE: For this procedure, refer to FIGURE 18.5 and FIGURE 18.6 on page 61.*

1. Install battery terminal cap(s) onto battery cable(s):
  - A. Dual U1 Batteries -
    - i. ORANGE battery terminal cap onto RED battery cable.
    - ii. GREY battery terminal cap onto BLACK battery cable.
  - B. Dual Group 22NF Batteries -
    - i. RED battery terminal cap onto RED battery cable.
    - ii. BLACK battery terminal cap onto BLACK battery cable.

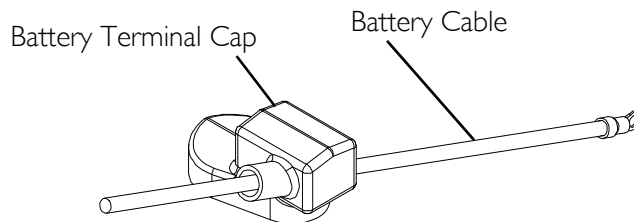

*NOTE: Only one battery cable and terminal cap shown for clarity. Both caps install in the same manner.*

**FIGURE 18.5** Installing Battery Terminal Cap

### CAUTION

**When connecting the battery cables to the battery(ies), the battery cable(s) MUST be connected to the battery terminal(s)/post(s) as shown in Detail “A” or Detail “B” of FIGURE 18.6 (depending on battery type), otherwise damage to the battery cable may result when installing battery terminal caps.**

---

2. Connect battery cable(s) to battery(ies) terminal(s)/post(s) as shown in Detail “A” or Detail “B” of FIGURE 18.6, depending on battery type:
  - A. NEGATIVE (-) BLACK battery cable to NEGATIVE (-) battery terminal/post.
  - B. POSITIVE (+) RED battery cable to POSITIVE (+) battery terminal/post.
3. Secure the battery cable(s)/ring terminal(s) to the battery terminal(s)/post(s), BLACK to NEGATIVE (-) and RED to POSITIVE (+), with the provided 1/4-20 x 7/8-inch hex flange screw and hex flange locknut as shown in Detail “A” or Detail “B” of FIGURE 18.6, depending on battery type. Securely tighten.
4. Verify all battery cable(s)/ring terminal(s) are correctly installed and securely tightened.
5. Slide terminal cap(s) down battery cable(s) and onto battery clamps (FIGURE 18.6).
6. Using the 11½-inch long tie-wrap, secure each terminal cap in place (Detail “C” of FIGURE 18.6).


*NOTE: It will be necessary to trim excess tie-wrap in order to install the battery box top(s).*

7. Install the battery box top(s).
8. Install the battery box(es) into the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55.


*NOTE: New battery(ies) MUST be fully charged BEFORE using, otherwise the life of the battery(ies) will be reduced.*

9. If necessary, charge the battery(ies). Refer to When to Charge Batteries on page 67.

**Detail “A” - Dual UI Batteries**

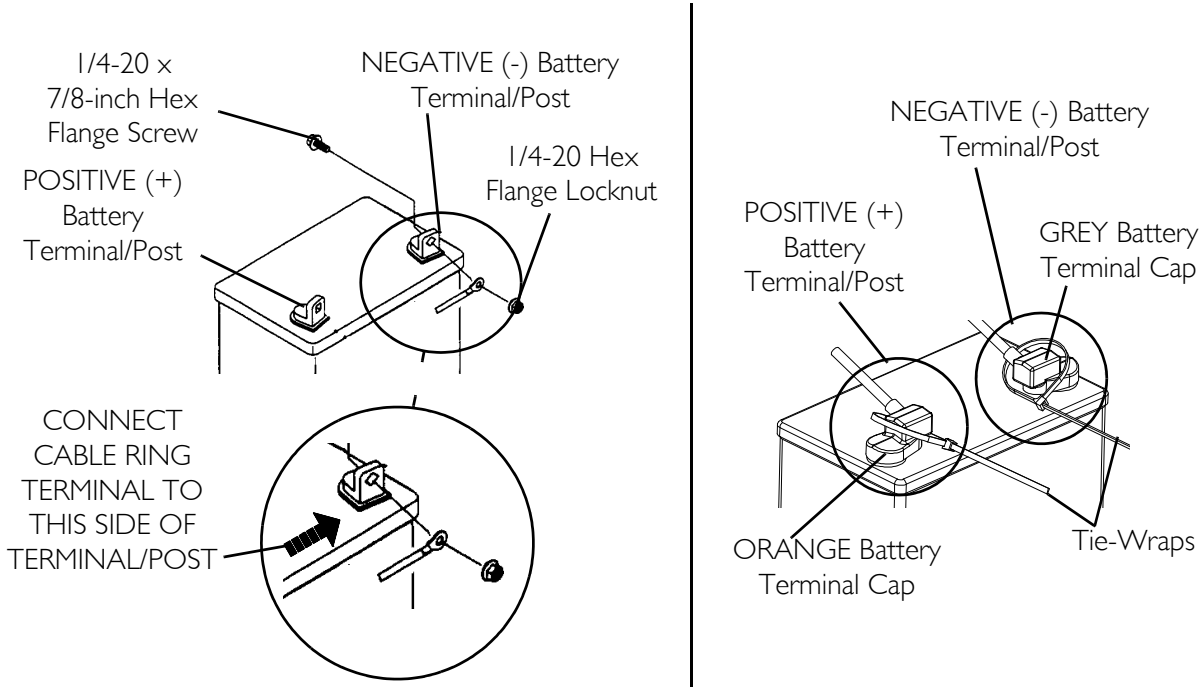

**Detail “B” - Dual Group 22 NF Batteries**

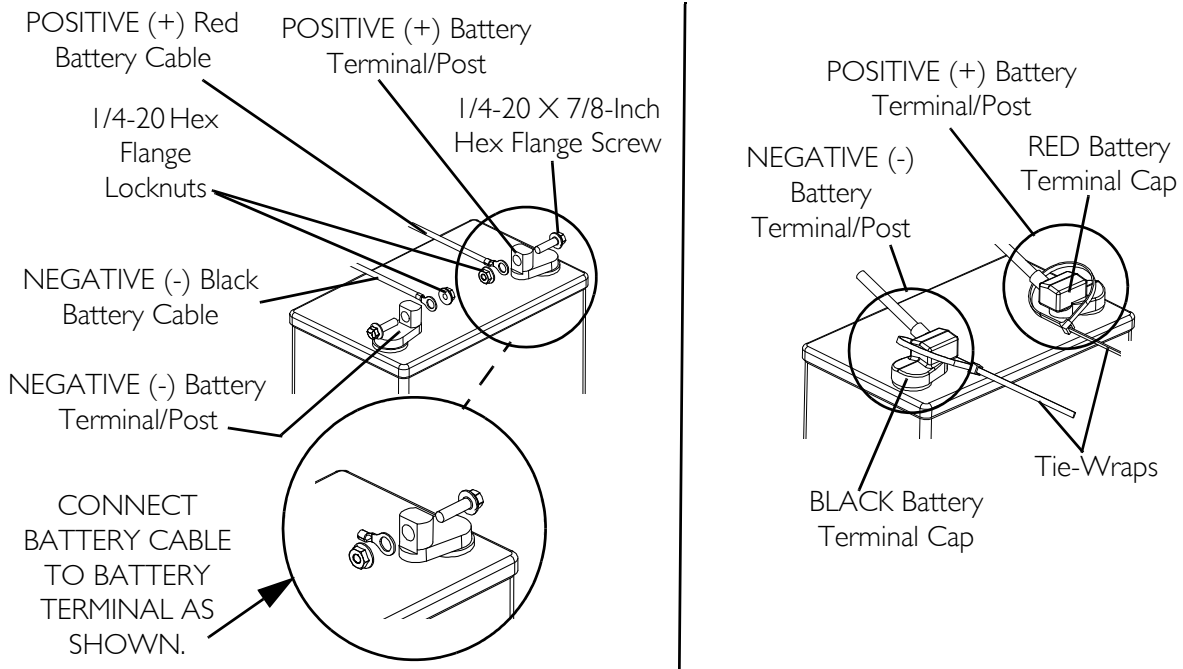


**FIGURE 18.6** Connecting Battery Cables - Direct Mount Method

## Battery Clamp Method

### ⚠ WARNING

The battery clamp of the **POSITIVE (+)** battery terminal/post **MUST** be mounted in the position shown in **FIGURE 18.7**, otherwise the battery box top cannot be installed properly.

*NOTE: For this procedure, refer to FIGURE 18.7, FIGURE 18.8 on page 63 and FIGURE 18.9 on page 64.*

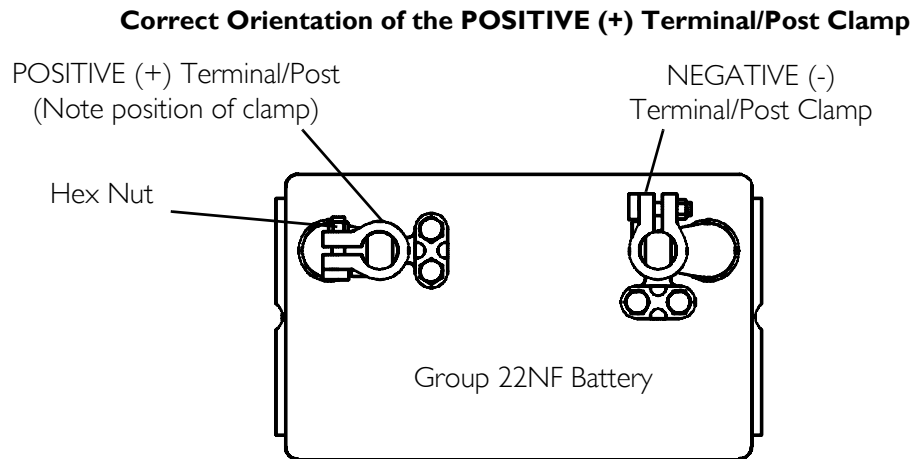

**FIGURE 18.7** Connecting Battery Cables - Battery Clamp Method

1. Perform one of the following:
  - If the battery clamp of the POSITIVE (+) battery terminal/post is NOT mounted in the orientation shown in FIGURE 18.7, perform the following:
    - i. Loosen the hex nut that secures the battery clamp to the POSITIVE (+) battery terminal/post.


### CAUTION

**When tightening the clamps, always use a box wrench. Pliers will “round off” the nuts. NEVER wiggle the battery terminal(s)/post(s) when tightening. The battery may become damaged.**

- ii. Remove the battery clamp from the POSITIVE (+) battery terminal/post.
  - iii. Reposition the battery clamp on the POSITIVE (+) battery terminal/post as shown in FIGURE 18.7.
  - iv. Securely tighten the hex nut that secures the battery clamp to the positive (+) battery terminal/post.
- If the battery clamp on the POSITIVE (+) battery terminal/post is positioned as shown in FIGURE 18.7, proceed to STEP 2.

2. Install battery clamp covers onto battery cables (FIGURE 18.8) -
  - A. RED battery clamp cover onto RED battery cable.
  - B. BLACK battery clamp cover onto BLACK battery cable.

*NOTE: Only one battery cable and battery clamp cover are shown for clarity. Both battery clamp covers install in the same manner.*

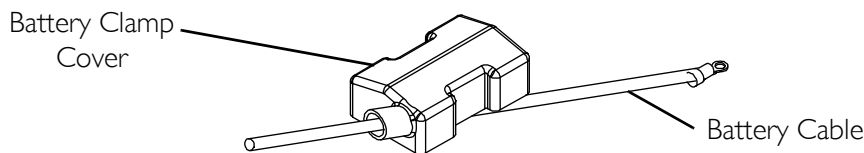

**FIGURE 18.8** Installing Battery Clamp Cover

### CAUTION

**When connecting the battery cable/ring terminal(s) to the battery(ies) clamp(s), the battery cable(s) MUST be connected in the position shown in Detail “A” of FIGURE 18.9. Otherwise, damage can occur to the battery cable and/or battery clamp covers.**

3. Connect battery cable(s) to battery(ies) terminal(s)/post(s) as shown in Detail “A” of FIGURE 18.9:
  - A. NEGATIVE (-) BLACK battery cable/ring terminal between the mounting plate and battery clamp of NEGATIVE (-) battery terminal/post.
  - B. POSITIVE (+) RED battery cable/ring terminal between the mounting plate and battery clamp of POSITIVE (+) battery terminal/post.
4. Secure the battery cable(s)/ring terminal(s) to the battery clamp(s), BLACK to NEGATIVE (-) and RED to POSITIVE (+), with existing hex screws as shown in Detail “A” of FIGURE 18.9. Securely tighten.
5. Verify battery cable ring terminal(s) are correctly installed and securely tightened.
6. Slide battery clamp covers down battery cables and onto battery terminals as shown in Detail “B” of FIGURE 18.9.
7. Using 11½-inch long tie-wraps, secure each terminal cap in place as shown in Detail “B” of FIGURE 18.9.


*NOTE: It will be necessary to trim excess tie-wrap in order to install the battery box top(s).*

8. Install the battery box top(s).
9. Install the battery box(es) into the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55.


*NOTE: New battery(ies) MUST be fully charged BEFORE using, otherwise the life of the battery(ies) will be reduced.*

10. If necessary, charge the battery(ies). Refer to When to Charge Batteries on page 67.

**Exploded View**

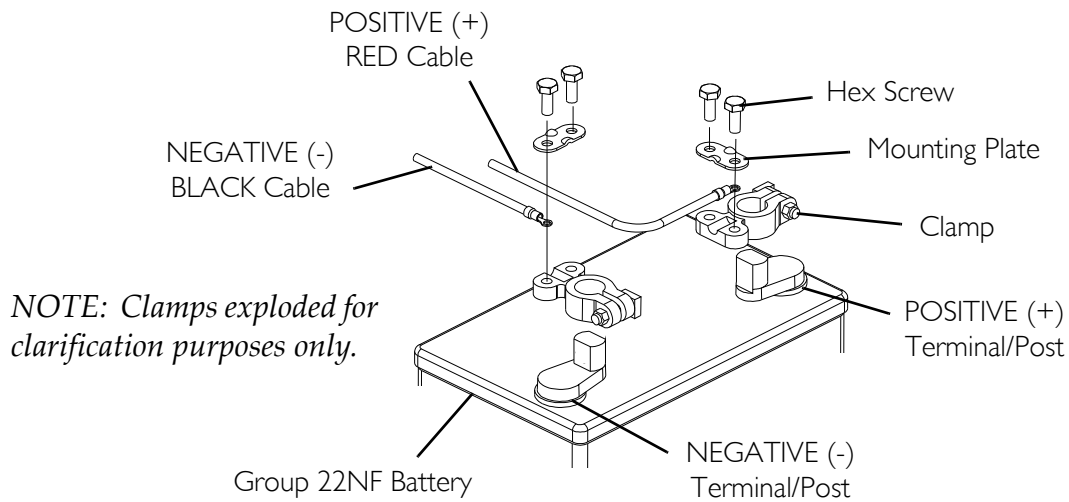

**Top View**

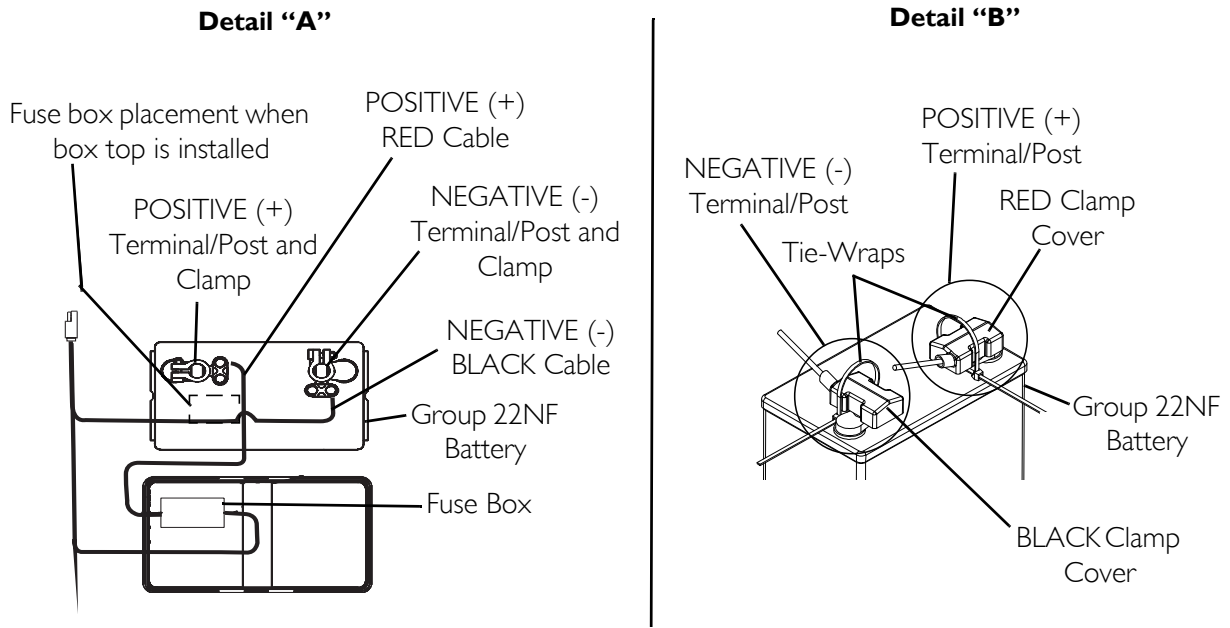

**FIGURE 18.9** Connecting Battery Cables -Battery Clamp Method



---

## Replacing the Battery Box Retaining Strap

*NOTE: For this procedure, refer to FIGURE 18.10 on page 66.*

---

### **⚠ WARNING**

**The battery box retaining strap MUST be fastened securely and adjusted properly to hold battery boxes in place before using the wheelchair.**

- 
1. Remove the battery boxes from the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55.
  2. Remove the battery tray from the wheelchair. Refer to Installing/Removing the Battery Tray on page 56.

---


### **⚠ WARNING**

**The retaining strap MUST be fed through the adjustable end of the battery box retaining strap buckle as shown in FIGURE 18.10.**

---

*NOTE: Observe the position and orientation of the battery box retaining strap as it passes through the slots of the battery tray and the adjustable end of the battery box retaining strap buckle.*

3. Feed the existing battery box retaining strap through the adjustable end of the battery box retaining strap buckle.
4. Feed the existing battery box retaining strap through the slots in the battery box tray.
5. Install the NEW battery box retaining strap by reversing STEPS 3 and 4 as shown in FIGURE 18.10.
6. Reinstall the battery tray from the wheelchair. Refer to Installing/Removing the Battery Tray on page 56.
7. Reinstall the battery boxes from the wheelchair. Refer to Installing/Removing the Battery Boxes on page 55.

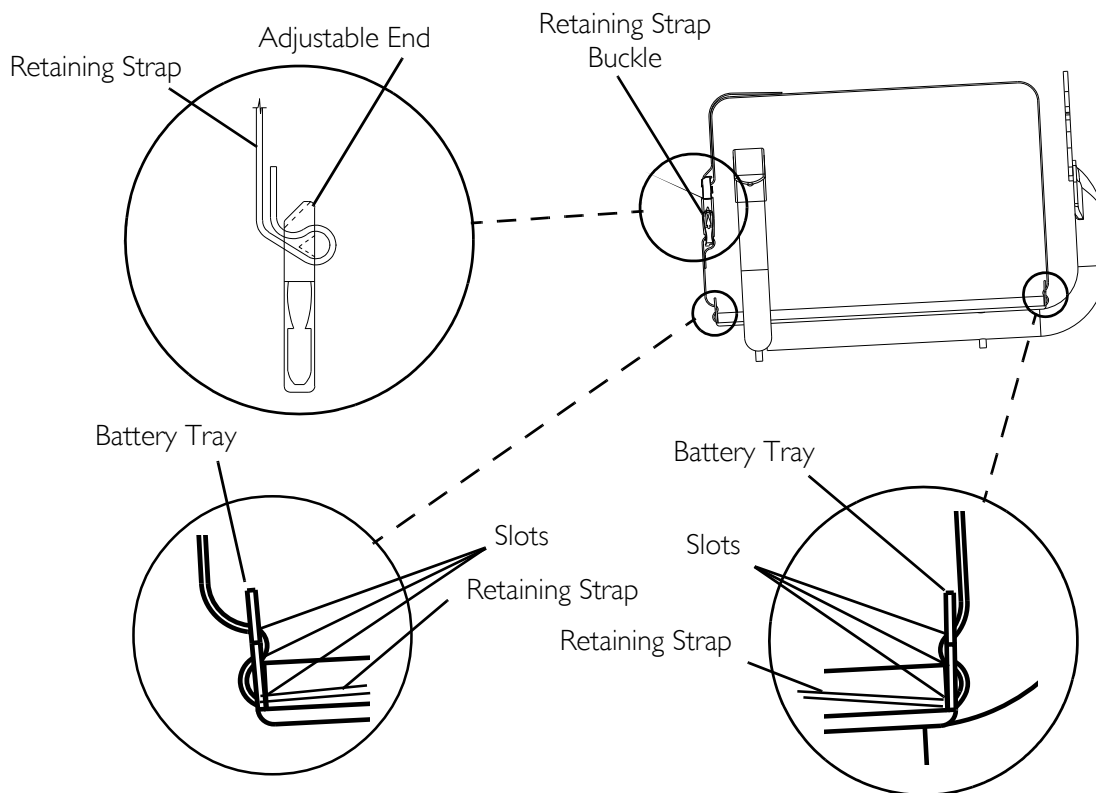

**FIGURE 18.10** Replacing the Battery Box Retaining Strap

## Cleaning Battery Terminals

### ⚠ WARNING

Most batteries are not sold with instructions. However, warnings are frequently displayed on the cell caps. Read them carefully.

**DO NOT** allow the liquid in the battery to come in contact with skin, clothes or other possessions. It is a form of acid and harmful or damaging burns may result. Should the liquid touch your skin, wash the area immediately and thoroughly with cool water. In serious cases or if eye contact is made, seek medical attention immediately.

**NEVER** smoke or strike a match near batteries. If the caps of the battery cells are removed, **NEVER** look directly into them while charging batteries.

1. Examine battery clamps and terminals for corrosion.
2. Verify that the plastic caps are in place over the battery cell holes.
3. Clean the terminals and inside the battery clamps by using a battery cleaning tool, wire brush, or medium grade sand paper.

*NOTE: When done, these areas should be shiny, not dull.*

4. Carefully dust off all metal particles.


## When to Charge Batteries

### SPJ™ + Joysticks

*NOTE: For this procedure, refer to FIGURE 18.11.*

The Information Gauge Display located on the front of the joystick housing, it provides the state-of-battery charge, including notification of when the battery requires charging. It also provides the following information to the user on the status of the wheelchair:

- A. GREEN LEDs are lit, indicating well charged batteries.
- B. AMBER LEDs are lit, indicating batteries are moderately charged. Recharge batteries before taking a long trip.
- C. RED LEDs are lit, indicating batteries are running out of charge. Recharge batteries as soon as possible.

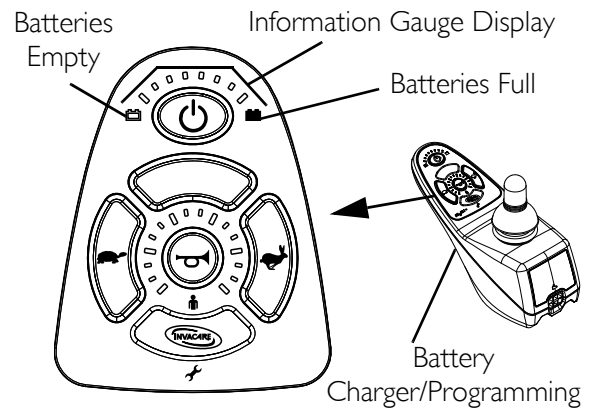

**FIGURE 18.11** SPJ™ + Joysticks

### ⚠ WARNINGS

**Never attempt to recharge the batteries by attaching cables directly to the battery terminals or clamps. Use the recharging plug located on either the rear of the wheelchair frame or on the front of the joystick.**

**DO NOT attempt to recharge the batteries and operate the power wheelchair at the same time.**

**DO NOT attempt to recharge the batteries when the wheelchair has been exposed to ANY type of moisture.**

**DO NOT attempt to recharge the batteries when the wheelchair is outside.**

**DO NOT sit in the wheelchair while recharging the batteries.**

**During use and charging, unsealed batteries will vent hydrogen gas which is explosive in the right concentration with air.**

### CAUTION

**New batteries MUST be fully charged prior to initial use of the wheelchair.**

**Always charge new batteries before initial use or battery life will be reduced.**

*NOTE: As a general rule, batteries should be recharged daily to assure the longest possible life and minimize the required charging time. Plan to recharge the batteries when it is anticipated the wheelchair will not be used for a long period of time.*

The range per battery charge using recommended batteries should be approximately 5 to 9 hours of typical operation. Extensive use on inclines may substantially reduce per charge mileage.

## Description and Use of Battery Chargers

The charger automatically reduces the charge from an initially high rate to a zero reading at a fully charged condition. If left unattended, the charger should automatically shut-off when full charge is obtained.

Some basic concepts which will help you understand this automatic process are:

The amount of electrical current drawn within a given time to charge a battery is called “charge rate”. If, due to usage, the charge stored in the battery is low, the charge rate is high, as indicated by the green light on the charger. Initially, the green light will stay illuminated for a short period of time followed by a longer period of off time. As a charge builds up, the charge rate is reduced, and the green light will stay illuminated for a longer period of time followed by a shorter off time.

*NOTE: If performing the charging procedures independently, READ and CAREFULLY follow the individual instructions for each charger (supplied or purchased).*

*NOTE: If charging instructions are not supplied, consult a qualified service technician for proper procedures. Required items:*

TOOL	QUANTITY	COMMENTS
Battery Charger	1	Supplied
Extension Cord*	1	Not Supplied


*NOTE: 3-prong plug, 15 ampere current rating; industrial type*

## Connecting the Battery Charger

*NOTE: For this procedure, refer to FIGURE 18.12.*

1. Attach the battery charger connector to the charger port on the front of the joystick.
2. Plug the charger’s AC power cord, or extension, into the grounded 120 VAC wall outlet.
3. Wait until charging is complete.

*NOTE: If the batteries need to be charged more often or take longer to charge than normal, they may need to be replaced. Contact an Invacare dealer for service.*

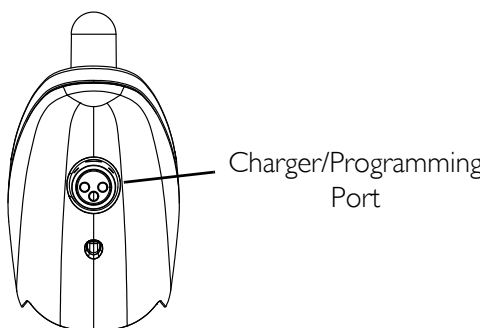

**FIGURE 18.12** When to Charge Batteries

# SECTION 19—CLUTCH/MOTOR LOCK

## ⚠ WARNING

After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

## Engaging/Disengaging the Clutches

## ⚠ WARNING

**DO NOT** engage or disengage the clutches until the power is off.


*NOTE: For this procedure, refer to FIGURE 19.1.*

The clutch engagement/disengagement allows free-wheeling or joystick controlled operation. Free-wheeling allows an attendant to maneuver the wheelchair without power. To engage/disengage the clutches:

1. Locate the clutch handles on motors as shown in FIGURE 19.1.

## CAUTION

If clutch handles are forced to engage in the wrong direction as shown in FIGURE 19.1, the motors will be damaged and will need to be replaced.

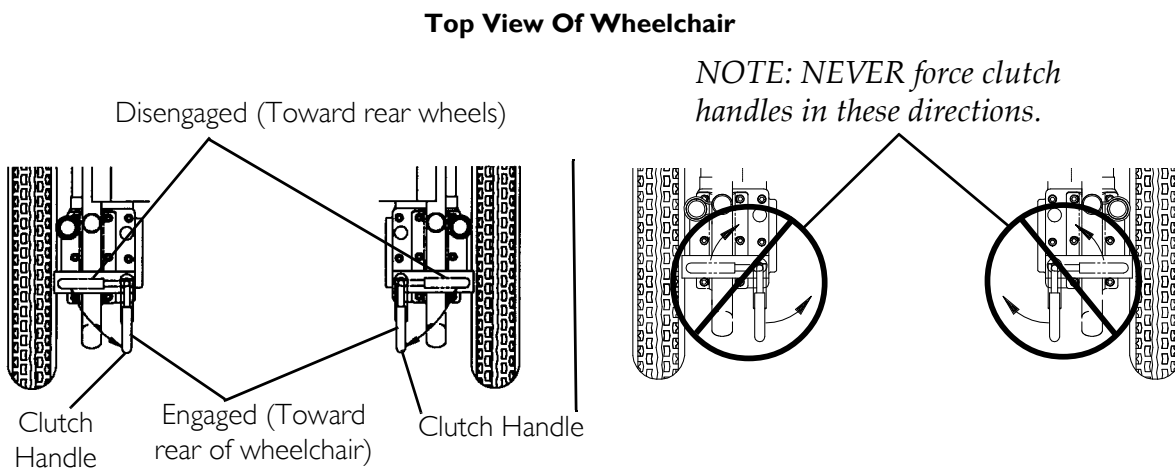

**FIGURE 19.1** Engaging/Disengaging the Clutches

2. Perform one of the following:
  - To Engage: turn the clutch handles until they are pointing toward the rear of the wheelchair. NEVER try to turn the clutch handles toward the front of the wheelchair.

- To Disengage: turn the clutch handles until they are pointing toward the rear wheels. NEVER try to turn the clutch handles toward the inside of the wheelchair.

## Disengaging/Engaging the Motor Lock Levers - R5 ILXP Only

---

### **⚠ WARNING**

**DO NOT** engage or disengage motor locks until the power is in the off position.


---

*NOTE: For this procedure, refer to FIGURE 19.2.*

*NOTE: Motor lock disengagement/engagement allows free-wheeling or joystick controlled operation. Free-wheeling allows an assistant to maneuver the wheelchair without power.*

### Four Pole Motors

1. Perform one of the following:
  - Disengage (push) - Push motor lock levers downward.
  - Engage (drive) - Pull motor lock levers upward.

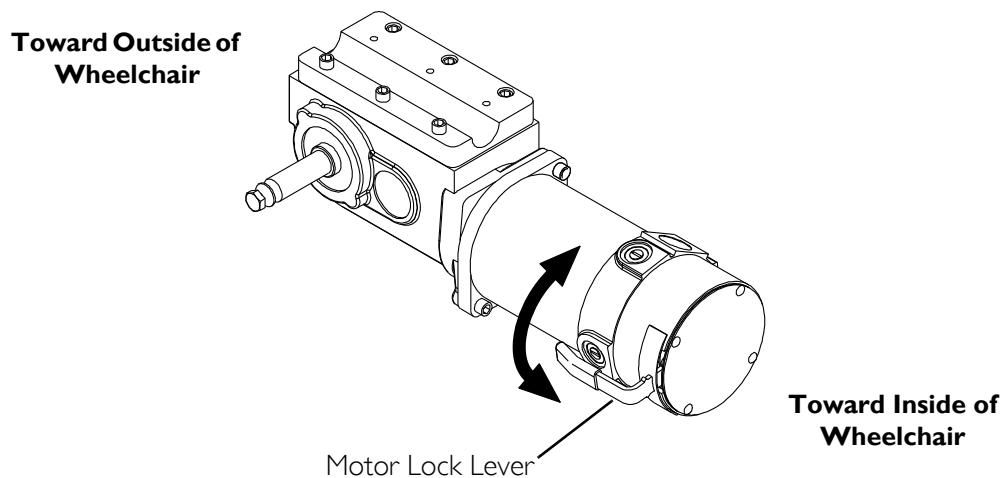

**FIGURE 19.2** Disengaging/Engaging the Motor Lock Levers - R5 ILXP Only

---

# SECTION 20—WHEEL LOCKS

---

## **⚠ WARNING**

**After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely. Otherwise, injury or damage may result.**

---

*NOTE: For this procedure, refer to FIGURE 20.1 on page 72.*

## **Installing/Adjusting/Using the Wheel Locks**

---

## **⚠ WARNING**

**Wheel locks are an option on RWD (you can order with or without wheel locks). Transfer to and from the wheelchair in the presence of a qualified health care professional to determine individual safety limits. Invacare strongly recommends ordering the wheel locks as an additional safeguards for the wheelchair user.**


---

### **Installing/Adjusting**

*NOTE: Before adjusting or replacing the wheel lock assemblies, ensure that the tires are inflated to the recommended psi on the side wall of tire.*

*NOTE: If necessary, remove threaded plastic insert from the wheel lock mounting hole in the wheelchair frame.*

1. Position the wheel lock on the wheelchair frame.
2. Loosely install the hex screw and locknut that secures the wheel lock to the wheelchair frame.
3. Make sure wheel lock is disengaged from rear wheel.
4. Measure the distance between the wheel lock shoe and the rear wheel.
5. Slide the wheel lock along the wheelchair until the measurement is between 5/32 and 5/16-inches.
6. Tighten the wheel lock to the wheelchair frame.
7. Repeat STEPS 2-6 for the opposite wheel lock.
8. Disengage the clutches. Refer to Engaging/Disengaging the Clutches on page 69.
9. Engage the wheel locks and push against the wheelchair to determine if the wheel locks engage the rear wheels enough to hold the wheelchair.
10. Repeat STEPS 3-9 until the wheel locks engage the rear wheels enough to hold the wheelchair.
11. Engage clutches. Refer to Engaging/Disengaging the Clutches on page 69.

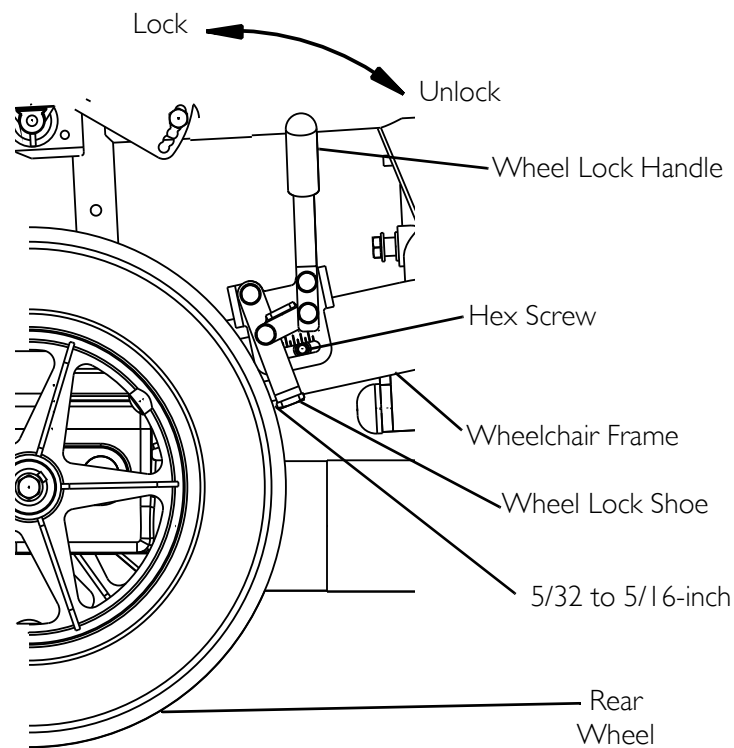

**FIGURE 20.1** Installing/Adjusting/Using the Wheel Locks

## Using

The wheelchair is equipped with a pair of independently operated wheel locks located just in front of the rear wheels.

1. To engage the wheel locks, grip the handle and push forward to the lock position.
2. To disengage, reverse STEP 1.

---

## CAUTION

**DO NOT use the wheel locks when the wheelchair power is on and the clutches are engaged - otherwise damage to the wheelchair may result.**

---

*NOTE: Use the wheel locks whenever the clutches are disengaged and the wheelchair is being pushed.*



# SECTION 21—ANTI-TIPPERS

## ⚠ WARNING

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

## Installing/Removing the Anti-Tippers

## ⚠ WARNING

Anti-tippers **MUST** be fully engaged and spring buttons fully protruding out of adjustment holes **BEFORE** using the wheelchair.

Ensure both anti-tippers have the same ground clearance.

A 1½ to 2-inch clearance between the bottom of the anti-tipper wheels and the ground/floor **MUST** be maintained at all times.

Anti-tippers **MUST** be used at all times. When outdoors on wet, soft ground or gravel surfaces, anti tippers may not provide the same level of protection against tipover. Extra caution must be observed when traversing such surfaces.


*NOTE: For this procedure, refer to FIGURE 21.1.*

### Installing

1. Press the two release buttons in and insert the anti-tippers with anti-tipper wheels pointing toward the ground/floor into the support tubes until the release buttons lock in place.
2. Measure the distance between the bottom of the anti-tipper wheels and the ground/floor.
3. If the distance between the bottom of the anti-tipper wheels and the ground/floor is not 1½-2-inches, do not use the anti-tippers. Replace anti-tippers and repeat STEPS 1 and 2.

### Removing

1. Press release buttons in and remove the anti-tippers from the support tubes.

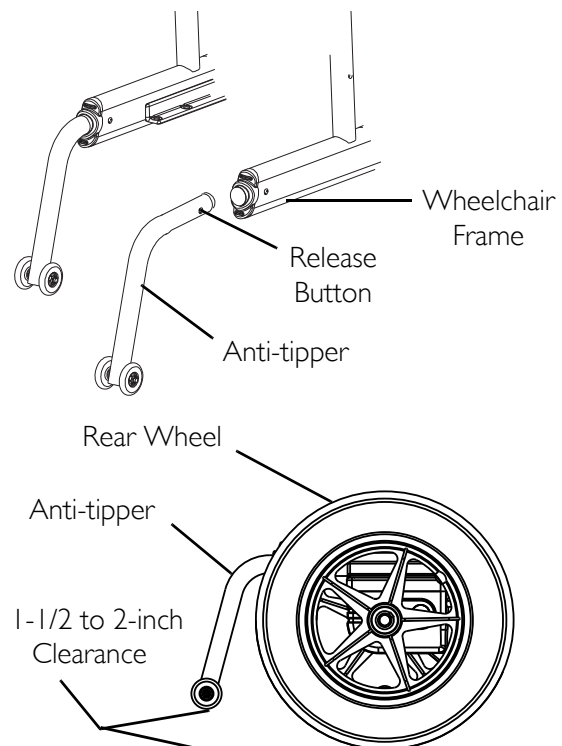

**FIGURE 21.1** Installing/Removing the Anti-Tippers

## SECTION 22—TRANSPORTING

---

### **⚠ WARNING**

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

---

## Transporting the Neutron

---

### **⚠ WARNING**

The weight of the wheelchair without the user and batteries is approximately 79 lbs. Use proper lifting techniques (lift with your legs) to avoid injury. Extreme caution is advised when it is necessary to move an unoccupied power wheelchair. Invacare recommends using two assistants and making thorough preparations. Make sure to use **ONLY** secure, non-detachable parts for hand-hold supports.

---

*NOTE: For this procedure, refer to FIGURE 22.1 on page 75.*

## Unfolding/Folding the Wheelchair

---

### **⚠ WARNING**

Keep hands and fingers clear of moving parts to avoid injury.

**DO NOT** place hand or fingers on the underside of the seat rail when opening or closing the wheelchair.

**DO NOT** sit or transfer into the wheelchair unless it is fully open and the seat rails are fully seated into the side frame H-blocks.

---

### **Unfolding**


1. Push down on the seat rails until the seat rails are positioned in the seat guides.
2. Assemble the wheelchair by following the instructions in this manual.

### **Folding**

1. Remove the anti-tippers. Refer to Installing/Removing the Anti-Tippers on page 73.
  2. Remove the batteries. Refer to Installing/Removing the Battery Boxes on page 55.
  3. Remove the footrests/legrests from the wheelchair. Refer to Installing/Removing the Footrests on page 36 or Installing/Removing Elevating Legrests on page 38.
  4. Remove the battery tray from the wheelchair. Refer to Installing/Removing the Battery Tray on page 56.
  5. Lift up on the seat rails of the wheelchair.
-

Refer to Installing/Removing the Battery Boxes on page 55  
and Installing/Removing the Battery Tray on page 56.

Refer to Installing/Removing the Anti-Tippers  
on page 73.

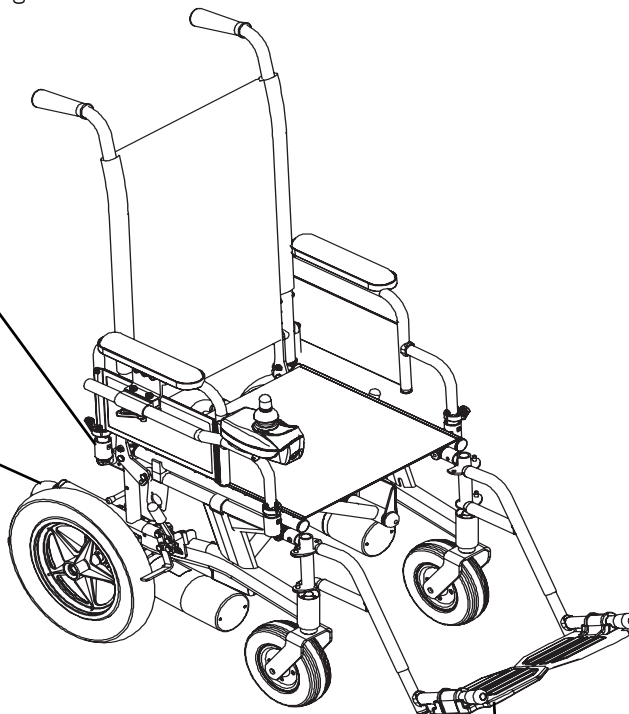

Refer to Installing/Removing the Footrests on page 36  
or Installing/Removing Elevating Legrests on page 38.

**FIGURE 22.1** Transporting the Nutron

# SECTION 23—WEAR AND TEAR INFORMATION

## General Information

Normal wear and tear items and components include but are not limited to: all upholstery items including seat and back upholstery, arm and calf pads, cushions, wheels, tires and casters, all types of batteries, joystick overlays and inductive rubberized protective boots.

Invacare reserves the right to ask for any item back that has an alleged defect in workmanship. Refer to the Warranty section in this manual for specific warranty information.

Refer to the Inspection Checklists in this manual for proper preventative maintenance schedule.

This is just a general guideline and does not include items damaged due to abuse and misuse.

<b>PRODUCT TYPE</b>	<b>PRODUCT WEAR AND TEAR</b>
<b>Wheelchairs</b>	Wheels, Brake Assembly, Hand Grips
<b>Scooters</b>	Wheels, Braking System, Armrest, Seat
<b>Mobility Hardware and Electronics</b>	Rubber Urethane Tires and Casters, Handgrips, Joystick Inductive Tops, Joystick Overlays, Motors and Gearboxes (if exposed to prolonged moisture, urine, etc.), Stability Lock
<b>Upholstery and Seating</b>	Arm pads, Seat Cushion Foam, Seat Cushion Covers, Back Cushion Foam, Back Cushion Covers, Headrest Foam, Headrest Covers, Footplate Covers, Calf Pad (if applicable)Foam and Cover
<b>Batteries</b>	Lead acid/Lithium, Coin cell (watch type), Gel (6 months)

# NOTES

# NOTES

# **GLOBAL LIMITED WARRANTY (EXCLUDING CANADA)**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser who purchases this product within any country excluding CANADA when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner.

Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. For product purchased in Canada, please refer to the Canada Limited Warranty.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the base frame to be free from defects in materials and workmanship for a period of three (3) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants the seat frame to be free from defects in materials and workmanship for a period of three (3) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all electronics and electrical components (excluding batteries), motors and gearboxes to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all batteries to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all remaining components (excluding all upholstered materials, padded materials, tires and wheels) to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty periods any such product component shall be proven to be defective, the product component shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the bottom of the back cover. Provide dealer's name address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS); PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR AND TEAR OR FAILURE TO ADHERE TO THE PRODUCT INSTRUCTIONS. A CHANGE IN OPERATING NOISE, PARTICULARLY RELATIVE TO MOTORS AND GEARBOXES DOES NOT CONSTITUTE A FAILURE OR DEFECT AND WILL NOT BE REPAIRED; ALL DEVICES WILL EXHIBIT CHANGES IN OPERATING NOISE DUE TO AGING.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN AND INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER; SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

# CANADA LIMITED WARRANTY

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser who purchases this product within Canada when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the base frame to be free from defects in materials and workmanship for a period of three (3) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants the seat frame to be free from defects in materials and workmanship for a period of three (3) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all electronics and electrical components (excluding batteries), motors and gearboxes to be free from defects in materials and workmanship for a period of two (2) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all batteries to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all remaining components (excluding all upholstered materials, padded materials, tires and wheels) to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty periods any such product component shall be proven to be defective, the product component shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the bottom of the back cover. Provide dealer's name address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS); PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR AND TEAR OR FAILURE TO ADHERE TO THE PRODUCT INSTRUCTIONS. A CHANGE IN OPERATING NOISE, PARTICULARLY RELATIVE TO MOTORS AND GEARBOXES DOES NOT CONSTITUTE A FAILURE OR DEFECT AND WILL NOT BE REPAIRED; ALL DEVICES WILL EXHIBIT CHANGES IN OPERATING NOISE DUE TO AGING.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN AND INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER; SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

Invacare Corporation [www.invacare.com](http://www.invacare.com)



*Yes, you can.®*

**USA**  
One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900

**Canada**  
570 Matheson Blvd E Unit 8  
Mississauga Ontario  
L4Z 4G4 Canada  
800-668-5324

All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.  
© 2016 Invacare Corporation

Part No. 1106645

Rev I-01 - 4/16

Making Life's Experiences Possible™

