

SAVE THESE INSTRUCTIONS

Yes, you can.

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

⚠ WARNING

- **DO NOT** install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.
- Each individual should **ALWAYS** consult with their physician or therapist to determine proper adjustment and usage.
- **DO NOT** sit on the knee walker. **DO NOT** use the knee walker as a wheelchair or transport device. The knee walker is not intended to be propelled while seated.
- The brakes **MUST** be in the locked position before using the knee pad.
- A physical/occupational therapist should assist in the height adjustment of the walker for maximum support and correct brake activation.
- Care should be taken to ensure that all hand and height adjustments are secure, and that casters and moving objects are in good working order before using this or any mobility aid.
- All wheels **MUST** be in contact with the floor at all times during use. This will ensure the walker is properly balanced.
- When using the walker in a stationary position, the hand brakes **MUST** be locked.
- **ALWAYS** observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.
- If push handle is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips **DO NOT** twist on push handle - otherwise damage or injury may occur.
- **DO NOT** attempt to reach objects if you have to move forward on the knee pad. Reaching for these objects will cause a change to the weight distribution of the walker and may cause the walker to tip, resulting in injury or damage.
- **DO NOT** hang anything from the frame of the walker. Items should be placed in the basket. The basket can hold up to 10 lbs.
- Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use **EXTREME** caution when reaching for any object.

Patient Height Specifications

NOTE: For models not listed, please check labeling on product.

MODEL	PATIENT HEIGHT MIN. / MAX
65950	5' - 6'3"

Using the Knee Walker

Folding/Opening the Knee Walker

NOTE: To open the knee walker, reverse this procedure.

1. If necessary, remove basket. Refer to [Installing/Removing the Basket](#).
2. Lift up on the free end of the knee pad to fold the knee walker.


Using the Brake

1. Pull the brake handle in towards the push handle.
2. Perform one of the following:
 - To remain stationary - continue to apply pressure to the brake handle.
 - To continue mobility - release the brake handle.
3. To lock the brake, push down on the brake handle.
4. To unlock the brake, pull up on the brake handle.

Adjusting Push Handle Height

1. Press the snap buttons on the frame tubes.
2. Adjust the push handle to the desired height and slide the snap buttons to one of seven height adjustment holes on either side of the frame tube.

NOTE: There will be an audible click.

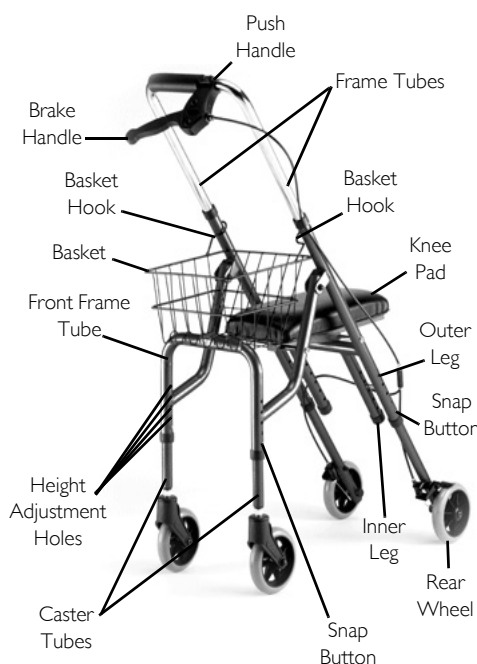

Installing/Adjusting Caster Tubes

1. Press the snap button on the caster tube.
2. Insert the caster tube into the leg and slide the snap button to one of five height adjustment holes.

NOTE: There will be an audible click.

3. Repeat steps 1-2 for the other caster tubes.

Changing Rear Wheel Position

NOTE: To move the rear wheel out of the way of the propelling leg, attach it to the inner leg on the same side as the propelling leg. (Example: When using the left leg to propel the knee walker, attach the rear wheel to the inner leg on the left side. Attach the right rear wheel to the outer leg on the right side to ensure stability.)

1. Press the snap button on the caster tube.
2. Slide the caster tube out of the leg.
3. Insert the caster tube into either the outer or inner leg.

Installing/Removing the Basket

Perform one of the following:

- To install the basket - Put the basket hooks onto the frame tube as shown. Rotate the basket into position to rest on the front frame tube.
- To remove the basket, rotate it up and release the basket hooks from the frame tube.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the frame and brakes to be free from defects in materials and workmanship for one year for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

Invacare Corporation

www.invacare.com



Yes, you can.

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

Canada

570 Matheson Blvd E Unit 8
Mississauga Ontario
L4Z 4G4 Canada
800-668-5324

All rights reserved. Trademarks are identified by the symbols [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.

© 2008 Invacare Corporation

Part No 1145772

Rev C - 06/08