Jasmine™ Patient Lift Boom Hardware Replacement Kit

Assembly, Installation and Operating Instructions

Yes, you can.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact carrier/dealer for further instruction.
To ensure the safe installation, operation and use of the Jasmine Patient Lift, these instructions MUST be followed:

⚠ WARNING

DO NOT install this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions, and Instructions**, contact Invacare Technical Support before attempting to install this kit - otherwise, injury or damage may result.
After ANY adjustments, repair or service and before use, make sure that all attaching component parts are secure.

Kit Description

This hardware kit is used when replacing the boom, (Invacare P/N 1146891), on the Jasmine Patient Lift.

DESCRIPTION	PART NUMBER	QTY	USE
Covers	1152349	2	Attaches Boom to Mast Assembly
Connector	1152344	1	
Screws, Hex Head	1152343	2	
Screw, Socket Head	1152353	1	
Washer, Plain	1152341	1	
Washer, Plain	1154622	1	
Nut, Hex Lock	1148547	1	
Nuts, Lock	1045608	2	
Washers, Nylon	1152351	2	
Bushing	1152352	1	
Bearing, Teflon	1155390	2	
Cover, Boom	1152348	1	

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

Overview

The Boom replacement kit contains components should it be necessary to replace the hardware that attaches the boom to the mast assembly. The following checklist provides an overview of the procedures necessary to replace the hardware contained in this kit.

- Remove the swivel bar.
- Detach the electric actuator from the boom actuator mounting bracket.
- Remove the existing boom.
- Install the replacement boom.
- Attach the electric actuator to the boom actuator mounting bracket.
- Install the swivel bar.

Replacing the Swivel Bar or Optional Cradle Attachment

⚠ WARNING

After the first year of use, the hooks of the swivel bar and mounting brackets of the boom should be inspected every six months to determine the extent of wear. If these parts become worn, replacement **MUST be made.**

NOTE: For this procedure, refer to FIGURE 1.

1. Remove the side covers.

NOTE: Support the swivel bar or cradle attachment from underneath while removing the attaching hardware.

2. Remove the socket head screw, washer, bushings, nylon spacers, washer and nut that attach the swivel bar or cradle attachment to the boom.

NOTE: When the attaching hardware is removed, the pin, washer and o-ring will become loose.

3. To install the swivel bar, reverse STEPS 1-2.
-

CAUTION

DO NOT overtighten the nut and socket head screw. This damages the mounting bracket.

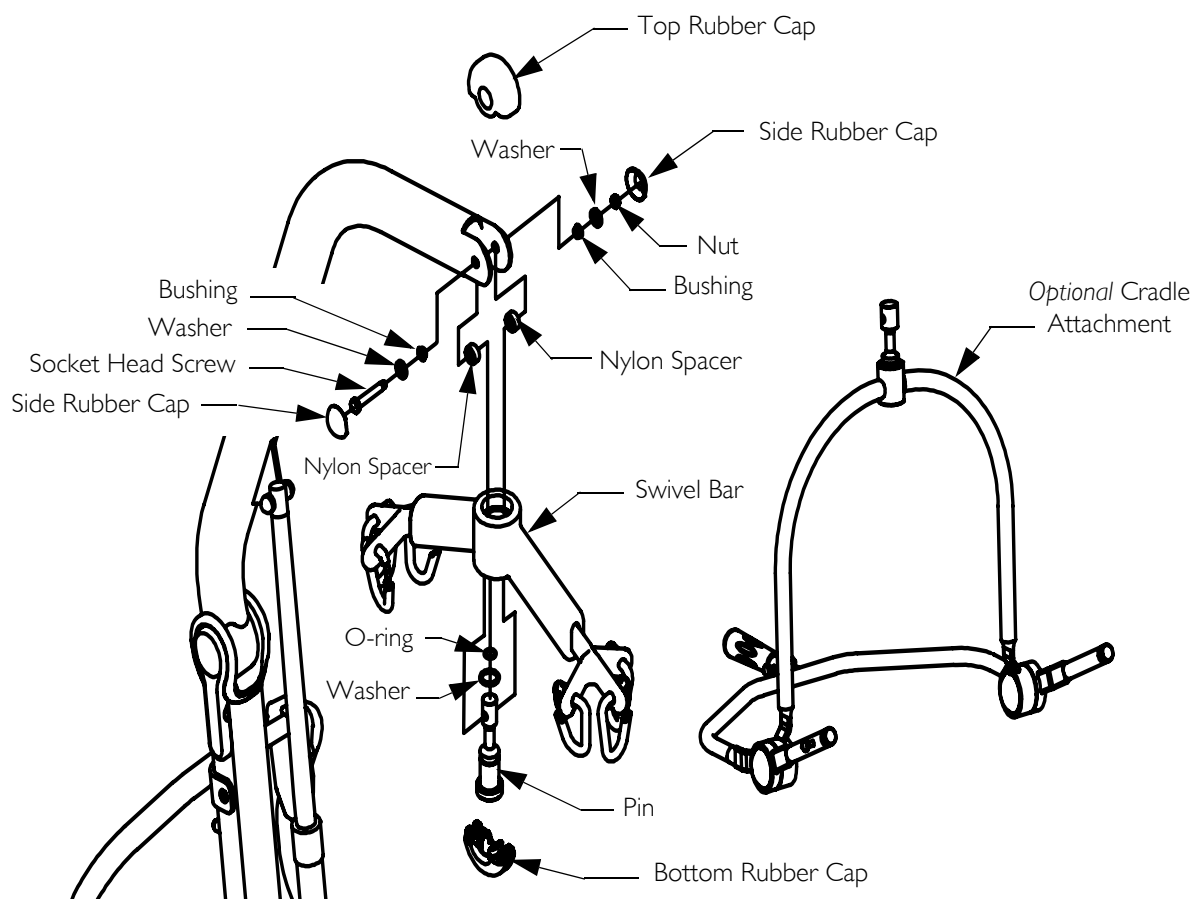
FIGURE 1 Replacing the Swivel Bar or Optional Cradle Attachment

Replacing the Boom Actuator

NOTE: For this procedure, refer to FIGURE 2.

NOTE: If possible, use an assistant while removing and replacing the boom actuator.

1. Unplug the boom actuator from the control box on the mast assembly. Thread the actuator lead through the actuator lead routing hole in the mast assembly.
2. Remove the hardware that secures the top of the boom actuator to the boom actuator mounting bracket.
 - A. Remove the rubber caps from the locknut and socket head screw.
 - B. Remove the socket head screw, washer, bearing, nylon washers, bushing, bearing, washer, flat washer and locknut.

NOTE: Lower the boom carefully to a resting position after the top of the boom actuator is removed from the boom actuator mounting bracket.

3. Remove the hardware that secures the bottom of the boom actuator to the boom actuator mast mounting bracket.
 - A. Remove the rubber caps from the locknut and socket head screw.
 - B. Remove the socket head screw, washer, bearing, nylon washers, bushing, bearing, washer, flat washer and locknut.
4. Reverse STEPS 1-3 to replace the boom actuator.

CAUTION

DO NOT overtighten the locknuts and socket head screws. This damages the mounting bracket.

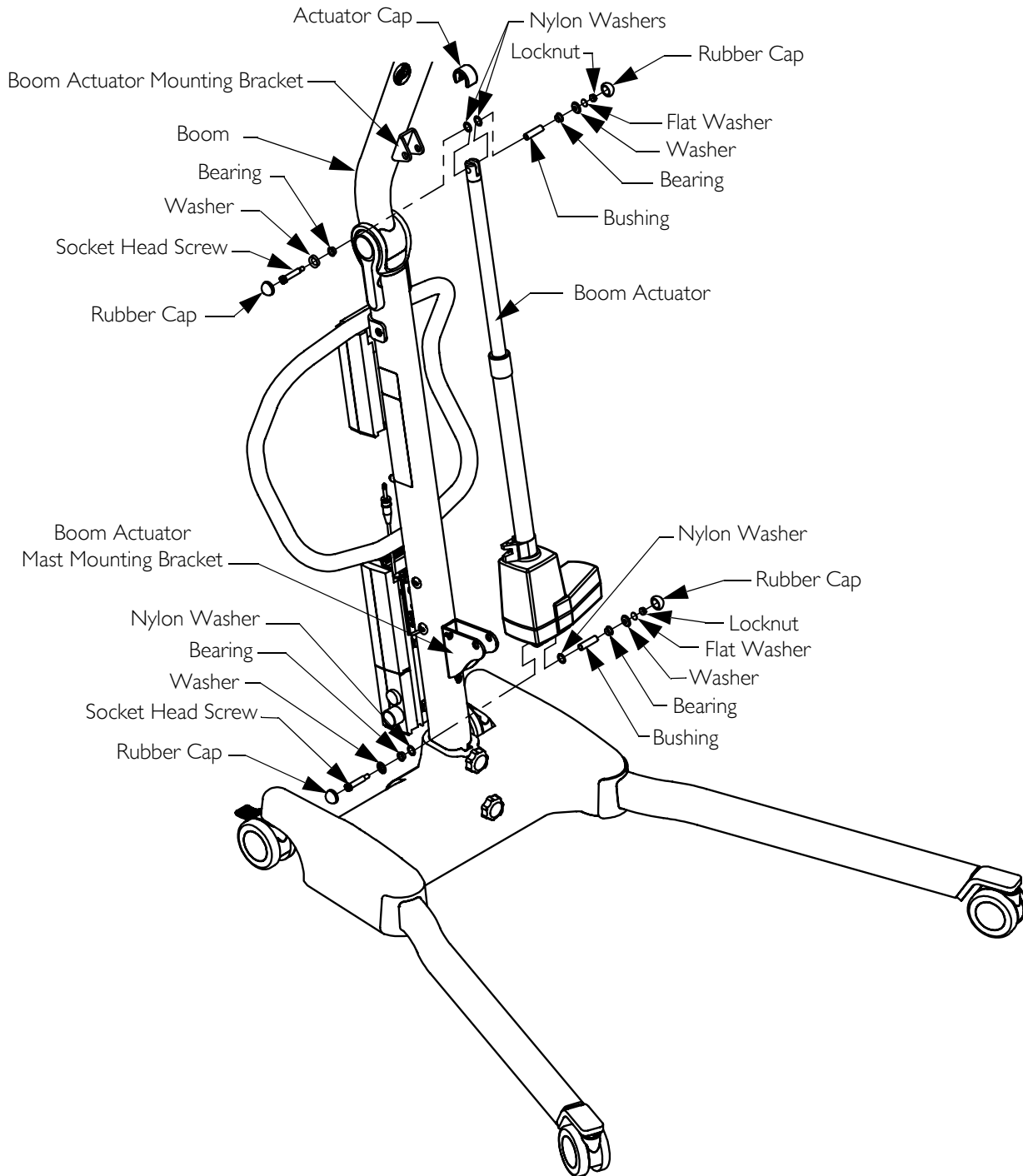
FIGURE 2 Replacing the Boom Actuator

Removing/Installing the Boom

NOTE: For this procedure, refer to FIGURE 3.

1. To gain access to the attaching hardware, remove the covers that are on either side of the boom.
2. Remove the screw, washers, nylon washers, teflon bearings, bushing and locknut that attach the boom to the mast connector. Pull the boom away from the mast connector.
3. Remove the screws and lock nuts that attach the mast connector to the mast.
4. To install the mast connector and boom, reverse STEPS 1-3.

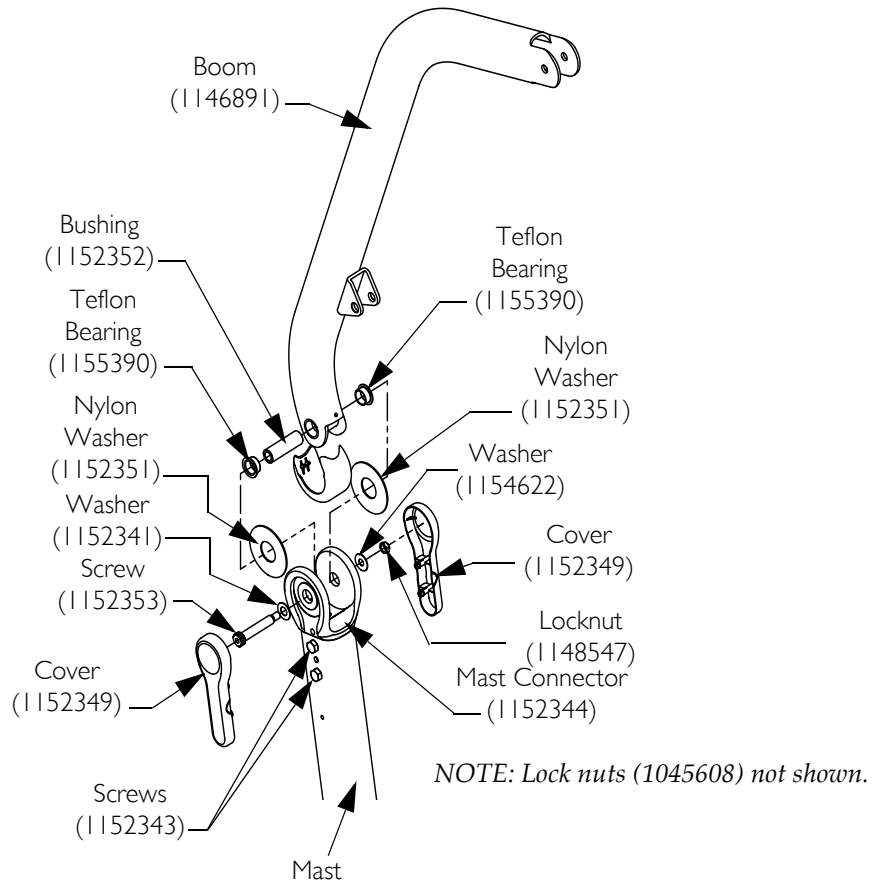
FIGURE 3 Removing/Installing the Boom

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the products manufactured to be free from defects in materials and workmanship for a period of six months from the date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product, product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, date of purchase, indicate nature of the defect.

Invacare Corporation will issue a serialized return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable as identification within 30 days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTIONS IN THIS MANUAL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE OR PROVINCIAL LAWS AND REQUIREMENTS.

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© 2009 Invacare Corporation. All rights reserved.
Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.