# Roze™ Patient Lift Liftarm Assembly Hardware Replacement Kit

Assembly, Installation and Operating Instructions

Yes, you can.

## SAVE THESE INSTRUCTIONS

*NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact carrier/dealer for further instruction.*

To ensure the safe installation, operation and use of the Roze Patient Lift, these instructions MUST be followed:

### **⚠ WARNING**

**DO NOT** install this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions, and Instructions**, contact Invacare Technical Support before attempting to install this kit - otherwise, injury or damage may result.

**After ANY adjustments, repair or service and before use**, make sure that all attaching component parts are secure.

## Kit Description

This hardware kit is used when replacing the liftarm assembly (Invacare P/N 1148554), on the Roze Patient Lift.

DESCRIPTION	PART NUMBER	QTY	USE
Screws, Socket Head, 1/2 x 2.00	1148530	2	Attaches Liftarm Assembly to Mast Assembly
Washers, Nylon	1148531	2	
Plugs, Tube	1148532	2	
Bushings, Mast Bronze	1154556	2	
Bushings	1148527	2	
Washers, Plain	1146913	2	
Knobs, Sling	1070177	2	Replacement for Existing Sling Knobs
Screws, Socket Head, 3/8 x 2.25	1146879	2	Attaches Sling Knob to Liftarm Assembly
Caps, Knob	1070902	2	
Washers, Step	1154620	2	
Nuts	1148559	2	
Caps, Nut	1154619	2	
Ring, Hook	1155290	2	New Hook Rings

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

---

## Overview

The following checklist provides an overview of the procedures necessary to replace the hardware contained in this kit.

- Remove the electric actuator.
- Remove the liftarm assembly.
- Install liftarm assembly.
- Install the electric actuator.


## Removing/Installing the Electric Actuator

*NOTE: For this procedure, refer to FIGURE 1 on page 3.*

1. Unplug the electric actuator from the control box on the mast assembly. Thread the actuator lead through the actuator lead routing hole in the mast assembly.
2. Remove the hardware that secures the top of the electric actuator to the lift arm actuator mounting bracket.
  - A. Remove the rubber caps from the nut and screw that attach the electric actuator to the lift arm actuator mounting bracket.
  - B. Remove the screw, washer, bearing, nylon washer, actuator cap, bushing, nylon washer, bearing, washer, flat washer and nut.

*NOTE: The actuator cap will become loose when the attaching hardware is removed. Inspect the actuator cap for damage. Replace the actuator cap if damage is found.*

3. Remove the hardware that secures the bottom of the electric actuator to the mast assembly actuator mounting bracket.
  - A. Remove the rubber caps from the nut and screw that attach the electric actuator to the mast assembly actuator mounting bracket.
  - B. Remove the screw, washer, bearing, nylon washer, bushing, nylon washer, bearing, washer, flat washer, and nut.
4. To install the electric actuator, reverse STEPS 1-3.

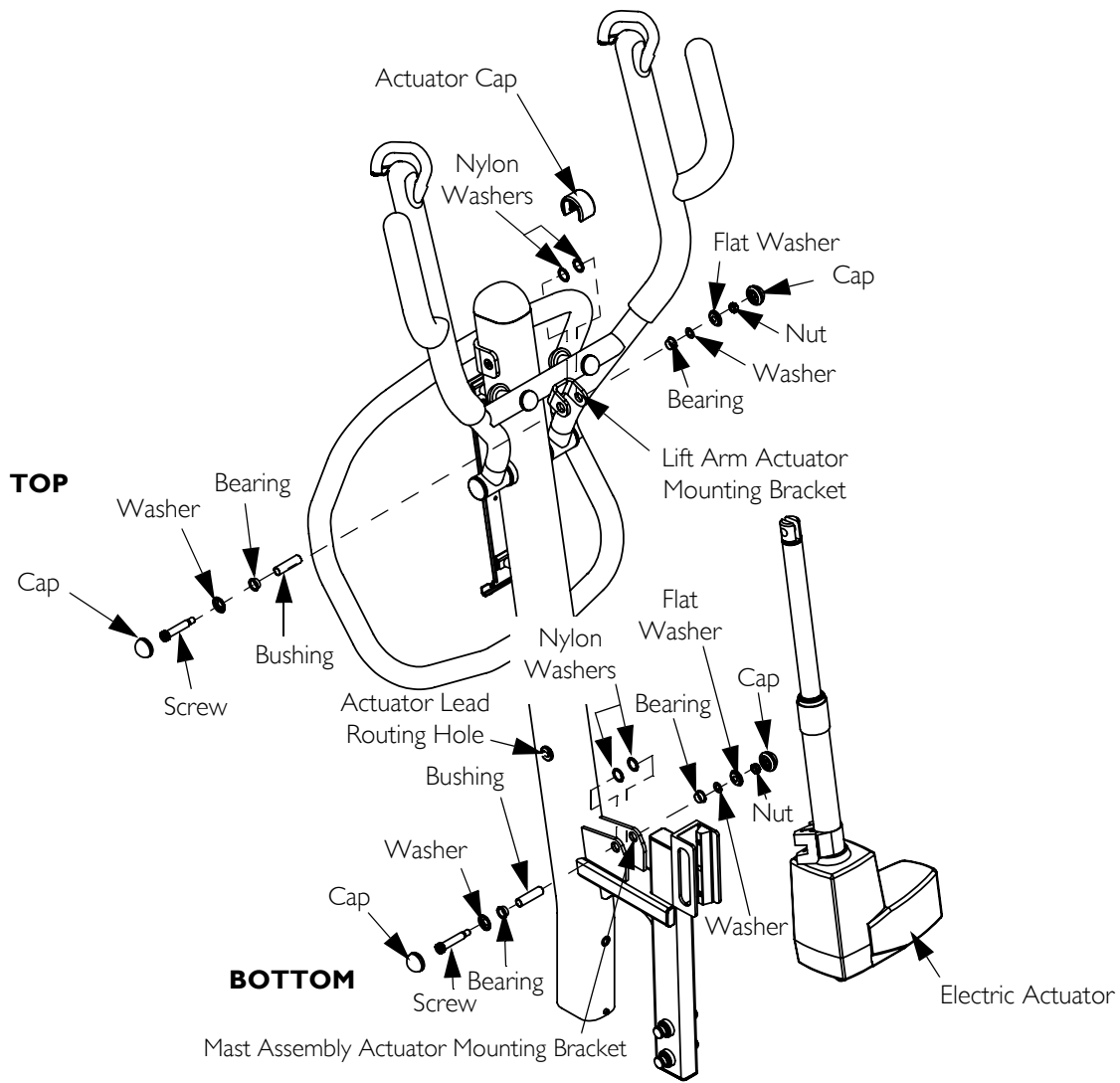**FIGURE I** Removing/Installing the Electric Actuator

---

## Removing/Installing the Liftarm Assembly

*NOTE: For this procedure, refer to FIGURE 2 on page 4.*


1. Remove the hardware that secures the liftarm assembly to the mast assembly on both the right and left hand side of the patient lift.
  - A. Remove the tube plugs from both sides of the liftarm assembly with the tip of a screwdriver.
  - B. Remove the socket head screws, plain washers and bushings that secure the liftarm assembly to the mast assembly.
  - C. Lift the liftarm assembly away from the mast assembly. Discard the existing hardware and liftarm assembly.
2. Install the replacement liftarm assembly.
  - A. Put a new nylon washer on the inside of the liftarm assembly mounting tube. See FIGURE 2 on page 4 for part orientation.
  - B. Put the liftarm assembly into place on the mast assembly.
  - C. On each side of the patient lift, attach the liftarm assembly to the mast assembly with a socket head screw, plain washer and bushings. See FIGURE 2 on page 4 for part orientation.

## Installing the Sling Knobs and Hook Caps

*NOTE: For this procedure, refer to FIGURE 2 on page 4.*

1. Attach the sling knob to the liftarm assembly with a socket head screw, step washer and nut. Push the knob cap into the opening at the top of the sling knob.
2. Repeat STEP 1 to install the other sling knob.
3. Snap the hook caps over the hooks on the end of the lift arm assembly.

**Roze™ Patient Lift**

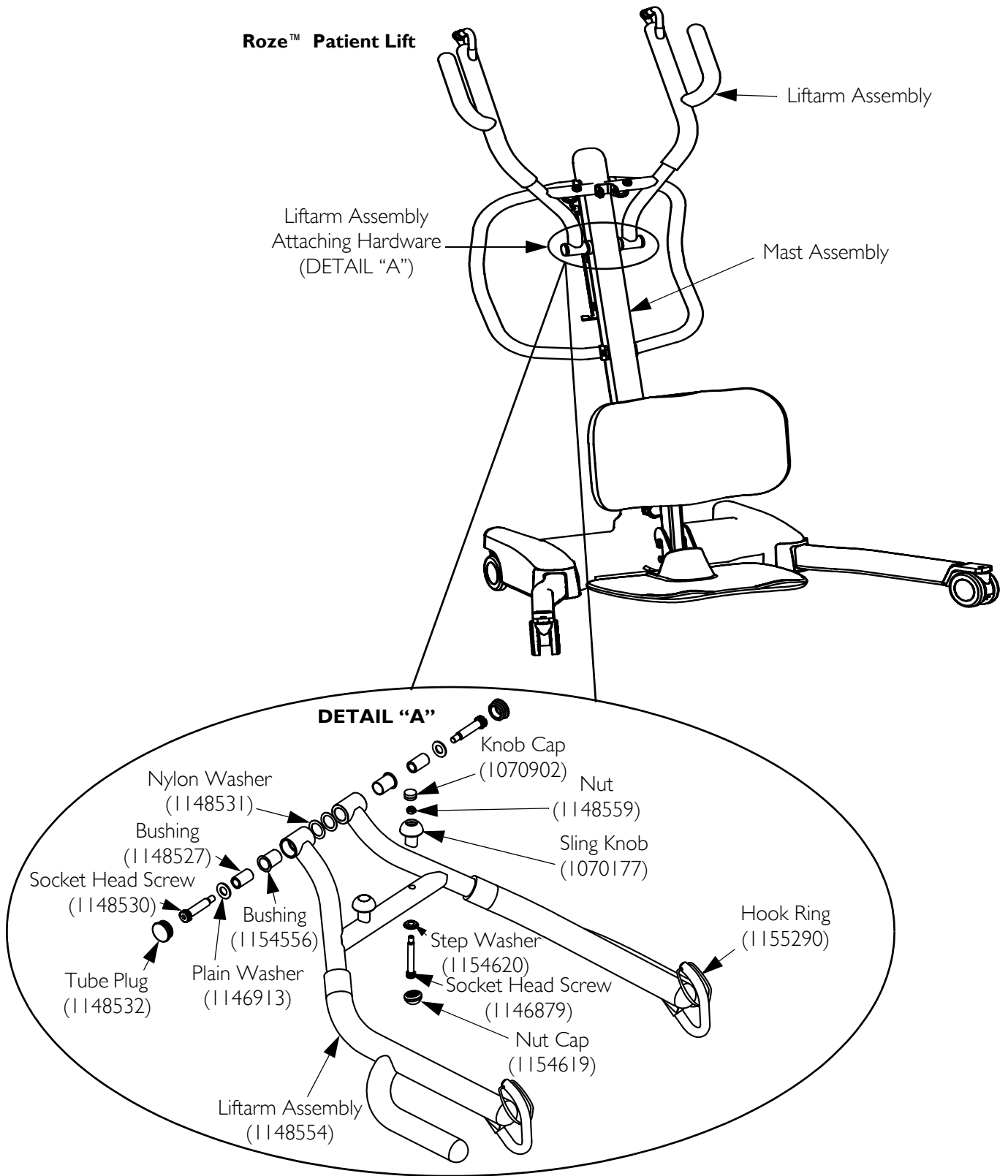**FIGURE 2** Removing/Installing the Liftarm Assembly

---

# LIMITED WARRANTY

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the products manufactured to be free from defects in materials and workmanship for a period of six months from the date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, date of purchase, indicate nature of the defect.

Invacare Corporation will issue a serialized return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable as identification within 30 days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTIONS IN THIS MANUAL.**

**THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**SOME STATES DO NOT ALLOW EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE OR PROVINCIAL LAWS AND REQUIREMENTS.**

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