

Installing Mattress Keepers



Assembly, Installation and Operating Instructions

Yes, you can!

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer/Carrier for further instructions.

Safety Summary

⚠ WARNING

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

⚠ ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

1. Remove mattress from bed.
2. Align mattress keeper with the corner of the mattress deck.
3. Secure the mattress keeper to the mattress deck using two mounting screws and locknuts. Tighten securely.
4. Repeat STEPS 2-3 to install the three remaining mattress keepers.
5. Install mattress on to bed.

Correct Mounting Position For Mattress Keeper

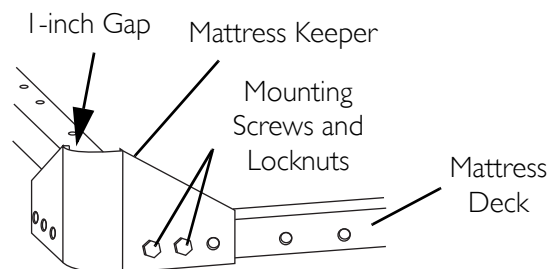


FIGURE I Installing Mattress Keepers

For more information regarding Invacare products, parts, and services, please visit www.invacare.com



Yes, you can.®

Invacare Corporation

www.invacare.com

USA
One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

Canada
570 Matheson Blvd E
Unit 8
Mississauga Ontario
L4Z 4G4 Canada
800-668-5324

All rights reserved. Trademarks are identified by the symbols [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

© 2009 Invacare Corporation

Part No 1163188

Rev A - 8/09

