Invacare® Canes

Walking Aids

User Manual

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Manuale d'uso

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Bruksanvisning

Brukerveiledning

Käyttöopas

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EN

FR

IT

DE

ES

PT

NL

SV

NO

FI

RO

LT

This manual MUST be given to the user of the product.

BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

This product complies with Directive 93/42/EEC concerning medical devices.

| | | | | | |
|----|----------------------------|-----------|----|-------------------------------|-------------|
| FR | Manuel d'utilisation | page 11 | SV | Bruksanvisning..... | sidan 51 |
| IT | Manuale d'uso | pagina 19 | NO | Brukerveiledning..... | side 57 |
| DE | Gebrauchsanweisung..... | Seite 25 | FI | Käyttöopas..... | sivu 63 |
| ES | Manual del usuario | página 31 | RO | Manualul utilizatorului | pagina 69 |
| PT | Manual de Utilização..... | página 39 | LT | Naudotojo vadovas..... | puslapis 75 |
| NL | Gebruiksaanwijzing | pagina 45 | | | |

© 2010 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

User Manual

Invacare[®] Canes

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

| | | |
|----------|---|----------|
| 1 | GENERAL | 4 |
| | Symbols | 4 |
| | Service Life..... | 4 |
| | Product Labeling..... | 4 |
| | Specifications and Weight Limitation | 4 |
| 2 | SAFETY | 5 |
| | Intended Use..... | 5 |
| | General Warnings..... | 5 |
| 3 | USAGE | 6 |
| 4 | CARE AND MAINTENANCE | 7 |
| 5 | AFTER USE | 7 |
| | Disposal..... | 7 |
| | Reuse | 7 |
| 6 | WARRANTY | 8 |
| | Limited Warranty - North America..... | 8 |
| | Customer Service and Warranty Information | 9 |

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.


Gives useful tips, recommendations and information for efficient, trouble-free use.

I.2 Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Maintenance should be done at least every six months.

I.3 Product Labeling

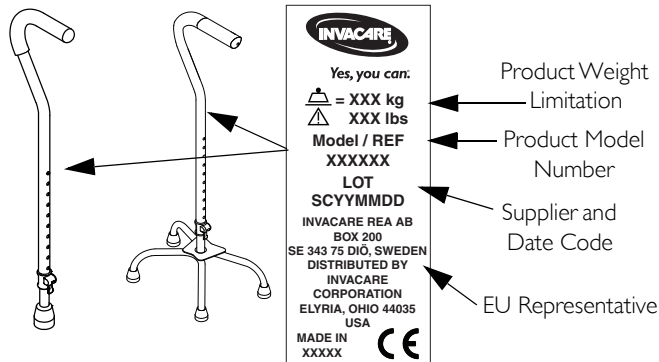I.4 Specifications and Weight Limitation

WARNING

ALWAYS observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.

| ITEM | ADJUSTMENT | USER HEIGHT MIN. / MAX | WEIGHT LIMITATION |
|--------------------------------|------------------------|-------------------------|-------------------|
| 8940-6, 8940-1 | 29-38" (74-97 cm) | 4'10"-6'4" (147-193 cm) | 700 lb (318 kg) |
| 7835-1, 7835-2 | 29.5-38.5" (75-98 cm) | 4'11"-6'5" (150-196 cm) | 700 lb (318 kg) |
| 7831-1, 7831-2 (Not CE Marked) | 29.5-38.5" (75-98 cm) | 4'11"-6'5" (150-196 cm) | 700 lb (318 kg) |
| 3920-1, 3920-2 | 29.25-38.25" (74-97) | n/a | 250 lb (113 kg) |
| 7827 | 29.25-38.5" (74-98 cm) | n/a | 250 lb (113 kg) |
| 8917 | 29-38" (74-97 cm) | 4'4"-6'4" (132-193 cm) | 250 lb (113 kg) |
| 40918-1, 40918-6 | 29-38" (74-97 cm) | 4'4"-6'4" (132-193 cm) | 250 lb (113 kg) |
| 8916 | 30-39" (76-99 cm) | 4'6"-6'6" (137-198 cm) | 250 lb (113 kg) |

2 Safety

-  Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

2.1 Intended Use

The cane is intended to be used solely as a walking aid and all other use can cause risk of injury. The cane can be used inside and outside the house.

2.2 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

You should ALWAYS consult with your physician or therapist to determine proper adjustment and usage. A physical/occupational therapist should assist in the height adjustment for maximum support.

Check rubber tips for rips, tears, cracks or wear. If any of these conditions exist, replace the rubber tips IMMEDIATELY.

The cane is not designed to support the total weight of an individual.

When using a quad cane, all four legs of the cane base must be in contact with and perpendicular to the walking surface. The base of the cane may be larger than the surface on stairways. Make sure that the longest or flattest portion of the base is closest to the user's foot - otherwise, injury or damage may occur.

Exercise extra care when walking on a slippery or wet surface.

After adjustments and before use, make sure that the cam lever is locked in place.

Care should be taken to ensure that all hand and height adjustments are secure before using this or any mobility aid.

If hand grips are exposed to extreme temperature [above 100°F (37°C) or below 32°F (0°C)], high humidity and/or becomes wet, prior to use, ensure hand grips do not twist on cane handle - otherwise damage or injury may occur.

If exposed to extreme temperature [above 100°F (37°C)], the cane will be hot and first degree burns may occur.

3 Usage

3.1 Adjusting the Cane Height

1. Lift up on the cam lever.
2. Press the double snap button.
3. Slide the extension up or down until the desired height adjustment is achieved. Allow the double snap button to click into the height adjustment holes in the cane.

i There will be an audible “click”. Make sure the snap button is fully engaged in the height adjustment holes. See Detail “A”.


4. Push down on the cam lever to lock it in place.

3.2 Adjusting the Quad Cane for Left or Right Hand Use

1. Lift up on the cam lever.
2. Press the double snap button.
3. Turn the cane handle 180°. Allow the double snap button to click into the holes in the cane.


i There will be an audible “click”. Make sure the snap button is fully engaged in the height adjustment holes. See Detail “A”.

4. Press down on the cam lever to lock.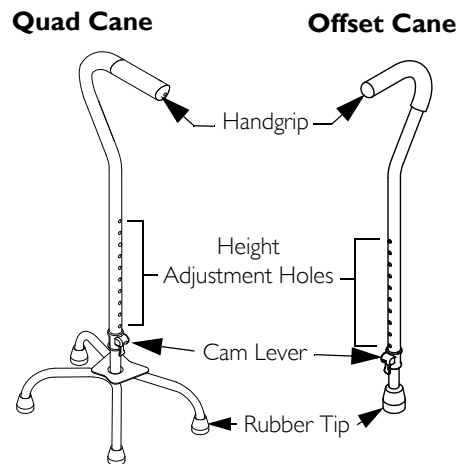

DETAIL “A” - SNAP BUTTON

Not engaged Fully Engaged

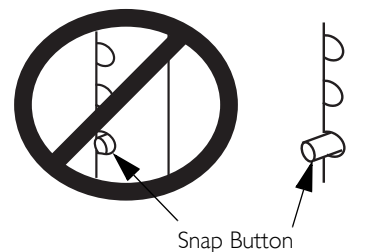4 Care and Maintenance

Maintenance should be done at least every six months. DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the walker.

1. Regular cleaning of the cane is recommended. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
2. Regular inspection of parts including hardware, brackets, and plastics, for deformation, corrosion, breakage wear or compression is recommended. Replace the cane if any of these conditions exist.
3. If the hand grip is loose, DO NOT use the cane. Contact your Invacare dealer.
4. If rubber tips are worn or cracked, replace the rubber tips IMMEDIATELY.

5 After Use

5.1 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly. Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Reuse

WARNING: Risk of Injury

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, DO NOT use the product. Otherwise injury may occur.

6 Warranty

6.1 Limited Warranty - North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for three years for the original purchaser (excluding tires and wheels). If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

6.2 Customer Service and Warranty Information

| | |
|---|---|
| United Kingdom | Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ Tel: (44) (0) 1656 776 222, Fax: (44) (0) 1656 776 220, www.invacare.co.uk, UK@invacare.com |
| Ireland | Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland Tel: (353) 1 810 7084, Fax: (353) 1 810 7085, www.invacare.co.uk, ireland@invacare.com |
| France | Invacare France Opération, Route de Saint Roch, 37230 Fondettes Tel: (33) (0) 2 47 62 64 66, Fax: (33) (0) 2 47 42 12 24, www.invacare.fr, contactfr@invacare.com |
| Belgium & Luxemburg | Invacare nv, Autobaan 22, B-8210 Loppem, Brügge Tel: (32) (0) 50 83 10 10, Fax: (32) (0) 50 83 10 11, www.invacare.be, belgium@invacare.com |
| Switzerland | Invacare AG, Benkenstrasse 260, CH-4108 Witterswil Tel: (41) (0) 61 487 70 80, Fax: (41) (0) 61 487 70 81, switzerland@invacare.com |
| Deutschland | Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny Tel: (49) (0) 75 62 7 00 0, Fax: (49) (0) 75 62 7 00 66, www.invacare-aquatec.de, info@invacare-aquatec.com |
| Eastern Europe, Middle East & Africa | Invacare European Distributor Organisation, Kleisestraße 49, D-32457 Porta Westfalica Tel: (49) (0) 57 31 754 540, Fax: (49) (0) 57 31 754 541, www.invacare.eu.com, edo@invacare.com |
| Portugal | Invacare Lda, Rua Estrada Velha, 949, P-4465-784, Leça do Balio Tel.: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39, portugal@invacare.com |
| Norge | Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo Tel: (47) (0) 22 57 95 00, Fax: (47) (0) 22 57 95 01, norway@invacare.com, island@invacare.com |
| Italia | Invacare Mecc San s.r.l., Via dei Pini 62, I-3016 Thiene (VI) Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34, www.invacare.it, italia@invacare.com |
| España | Invacare SA, c/Areny s/n, Poligon Industrial de Celrà, E-17460 Celrà (Girona) Tel: (34) (0) 972 49 32 00, Fax: (34) (0) 972 49 32 20, www.invacare.es, contactsp@invacare.com |
| Canada | Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 Tel: (905) 890-8300, Fax: (905) 890-5244, www.invacare.ca |
| Australia | Invacare Australia Pty Ltd., 1 Lenton Place, North Rocks, NSW 2151 Australia Tel: (61) 2 8839 5333, Fax: (61) 2 8839 5353, www.invacare.com.au, sales@invacare.com.au |
| New Zealand | Invacare New Zealand, 4 Westfield Place, Mt. Wellington, Auckland New Zealand Tel: (64) 9 917 3939, Fax: (64) 9 917 3957, www.invacare.co.nz, sales@invacare.co.nz |
| Denmark | Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby Tel: (45) (0) 36 90 00 00, Fax: (45) (0) 36 90 00 01, denmark@invacare.com |

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Invacare Rea AB