NOTE: Check all parts for shipping damage. If damage is found, DO NOT use. Contact Carrier/Invacare for further instructions.

IMPORTANT SAFEGUARDS

⚠ WARNING

IMPORTANT SAFEGUARDS contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a health care professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

CAUTION

“Caution: Federal law restricts this device to sale by or on order of a physician, or any other practitioner licensed by the law of the State in which he practices to use or order the use of this device.”

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

⚠ DANGER

To Reduce The Risk Of Electrocution:

ALWAYS unplug this product **IMMEDIATELY** after using and store in a dry place.

DO NOT use while bathing

DO NOT place or store this product where it can fall or be pulled into water or other liquid.

DO NOT place in or drop into water or other liquid.

DO NOT reach for product if it has fallen into liquid. **Unplug IMMEDIATELY.**

⚠ WARNING

To Reduce The Risk Of Burns, Electrocution, Explosion, Fire, Electrical Shock, Or Injury To Persons:

DO NOT disassemble. Disassembly or attempted repairs if accomplished incorrectly can create electrical shock hazard and/or damage to the product. Only qualified technicians should perform servicing of this compressor unit.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

⚠ WARNING

Keep all matches, lighted cigarettes or other sources of ignition out of the room in which this product is located.

This product should **NEVER** be left unattended when plugged in.

Close supervision is necessary when this product is used near children. **NEVER** allow children to operate the unit.

Use this product only for its intended use as described in this instructions sheet.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

NEVER operate this product if it has a damaged cord or plug. If it is not working properly, if it has been dropped or damaged, or if it has fallen into water, return the product to a service center for examination and repair.

Keep the cord away from **HEATED** surfaces.

NEVER block any air openings (inlet) of the product or place it on a soft surface where the opening may be blocked. Keep all air openings free of lint, dirt and other foreign objects.

NEVER use while sleepy or drowsy.

NEVER drop or insert fingers or any other object into any openings.

NEVER operate this product outdoors in the rain or in a wet area.

DO NOT use this product in or near explosive atmospheres or where aerosol (spray) products are being used.

DO NOT operate this product in an oxygen enriched environment, i.e. oxygen tent or oxygen hood.

If this unit is supplied with a three-pronged plug, connect unit to a properly grounded outlet only. Refer to Grounding Instructions on page 3.

Improper use of the grounding plug can result in a risk of electric shock.

DO NOT pump anything other than atmospheric air.

DO NOT use this product near flames.

DO NOT use the compressor for continuous operation more than 10 hours, this can result in overheating and damage to the compressor.

DO NOT pump combustible liquids or vapors with this product or use in or near an area where flammable or explosive liquids or vapors may exist.

This unit may be thermally protected and can automatically restart when the protector resets. Always disconnect power source before servicing.

Use only in well ventilated areas.

All electrical products generate heat. To avoid serious burns **NEVER** touch unit during or immediately after operation.

⚠ WARNING

After any adjustments, repair or service and before use, make sure that all attaching component parts are secure.

SAVE THESE INSTRUCTIONS

Regulatory Listing In Compliance With:

UL 1431- Personal Hygiene and Health Care Appliances

CSA C22.2 No. 68 - Motor Operated Appliances

Grounding Instructions

⚠ WARNING

Improper use of the grounding plug can result in a risk of electric shock.

NOTE: For this procedure, refer to FIGURE 1.

NOTE: This product should be grounded. In the event of an electrical short circuit, grounding reduces the risk of electrical shock by providing an escape wire for the electrical current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

NOTE: If repair or replacement of the cord or plug is necessary, do not connect the grounding wire to either flat blade terminal. The wire with insulation having an outer surface that is green with or without yellow strips is the grounding wire.

NOTE: This product is for use on a nominal 120 V circuit, and has a grounding plug that looks like the plug illustration in SKETCH A of FIGURE 1. A temporary adapter, which looks like the adapter illustrated in SKETCHES B and C, may be used to connect this plug to 2-pole receptacle as shown in SKETCH B if a properly grounded outlet is not available. The temporary adapter should be used only until a properly grounded outlet (SKETCH A) can be installed by a qualified electrician. The green colored rigid ear, lug and the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by the screw.

NOTE: Check with a qualified electrician or serviceman if the grounding instructions are not completely understood, or if in doubt as to whether the product is properly grounded.

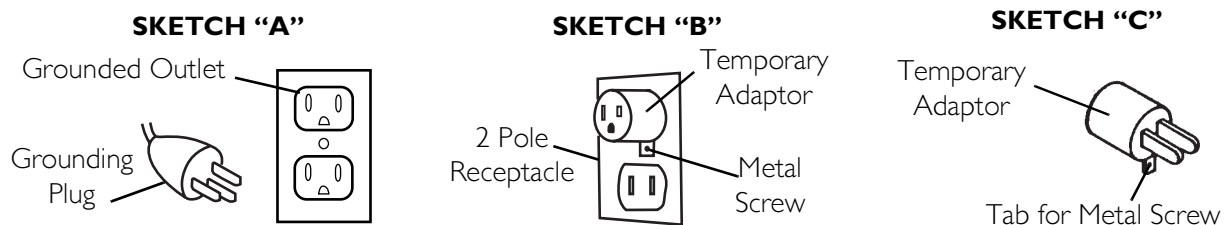
FIGURE 1 Grounding Instructions

Electrical - Compressor

If the overload protector shuts off the motor frequently, you may have a low voltage situation. Low voltage can also be suspected when:

1. The motor does not get up to full power or speed.
2. Fuses or circuit breakers activate when starting compressor.
3. Lights dim or remain dim when compressor is started.
4. Other motor operated appliances fail to operate properly. Too many motor operated appliances on same circuit.

NOTE: The motor of this compressor has a thermal overload protector. If the motor should overheat, the overload protector will shut off the motor. If this should occur, turn the ON/OFF switch to the OFF position and allow the motor to cool (approximately 5 minutes). Restart compressor. If the compressor fails to start, wait an additional 5 minutes before attempting to restart. If this fails, check for a blown fuse on the household electric circuits. If the unit still does not restart, the motor must be replaced. Call your dealer immediately.

Electrical - Extension Cords

NOTE: Make certain extension cord is in good condition and is NOT LESS than 18 gauge for 25 feet, 16 gauge for 50 feet and 14 gauge for 100 feet.

Operation

WARNING

Proper operation of this product is necessary to avoid physical harm. Improper use can cause personal injury and/or tissue damage. This product should not be used unless the operator has been instructed by a physician or other qualified health care professional.

DO NOT use the compressor for continuous operation more than 10 hours, this can result in overheating and damage to the compressor.

NOTE: For this procedure, refer to FIGURE 2.

1. Familiarize yourself with this unit.
2. Place unit on table and push down hard so that suction feet hold unit firmly in place.
3. Check to see that the ON/OFF switch is in the OFF position. Plug power cord into electrical outlet.

NOTE: Before attempting to start a cold unit, let the unit reach room temperature in a heated area, then press the ON/OFF switch to start the unit.

4. Turn the pressure control knob fully counterclockwise.

NOTE: As the knob is turned counterclockwise the pressure will decrease, as the knob is turned clockwise the pressure will increase.

5. Attach the nebulizer tubing to the outlet port.
6. Turn the ON/OFF switch to the ON position.

⚠ WARNING

Use only the pressure setting directed by your physician or qualified healthcare professional.

1. Turn the pressure control knob clockwise to adjust the pressure to the setting specified by your physician's instructions.

NOTE: The pressure gauge needle will vibrate. Adjust the pressure gauge setting as closely as possible to the specified pressure.

2. Proceed with therapy per physician's instructions.

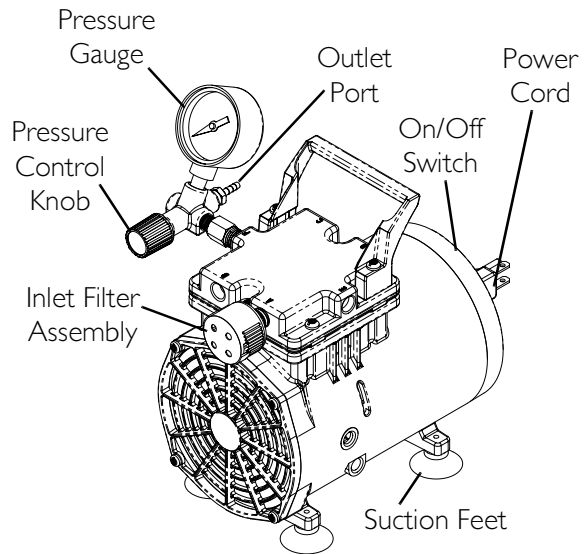
FIGURE 2 Operation

Cleaning/Replacing The Air Filters

NOTE: For this procedure, refer to FIGURE 2.

NOTE: Check the filters at least once each month. If the compressor begins to lose pressure in normal use, check the air filters.

1. Remove the filter cap from the Inlet Filter Assembly.
2. If the foam filter appears dirty, wash with mild detergent and water. Rinse thoroughly and let air dry.

CAUTION

DO NOT wash the felt filter. Water damages the felt filter.

3. Replace the felt filter if it appears dark in color, becomes clogged, or is damaged.

⚠ WARNING

DO NOT operate this compressor without filters installed.

Accessories

⚠ WARNING

DO NOT use the compressor for continuous operation more than 10 hours, this can result in overheating and damage to the compressor.

J-Tube and DISS Connector for large volume nebulizer.

Replacement Parts

Felt filter, Foam Filter, Filter Cap

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the 50 PSI Compressor to be free from defects in materials and workmanship for a period of one (1) year from date of purchase. If within such warranty period any such product shall be proven to Invacare's satisfaction to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include normal wear and tear or shipping charges incurred in replacement part installation or repair of such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement.

For warranty service, please contact Invacare's service department at the toll free number on the back page during normal working hours.

Upon receiving notice of an alleged defect in a product, Invacare will issue a serialized return authorization. It shall be the responsibility of the purchaser, at the purchaser's cost, to return the unit, pack the unit in a manner to avoid shipping damage and to ship the unit to either Invacare's plant or service center as specified by Invacare in advance. Defective units must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. Do not return products to our factory without prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, SOOT OR SMOKE-FILLED ENVIRONMENTS, COMMERCIAL USE, OR OTHER THAN NORMAL APPLICATION, USE OR SERVICE, OR TO PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE OR TO PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT APPLY TO YOU.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2006 Invacare Corporation
Invacare, Yes, you can, and the Medallion Design are registered trademarks of Invacare Corporation. Platinum is a trademark of Invacare Corporation.
Invacare Corporation