

9000 Jymni™ Wheelchair



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 00-104. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2022 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are trademarks; Jymni is a trademark of Invacare Corporation.

TABLE of CONTENTS

Description	Page Number
Accessories - Forward Rear Wheel Positioner (1")	77
Accessories - I.V./ Oxygen Holder	79
Accessories - Miscellaneous	87
Accessories - Oxygen Holder	80
Accessories - Recliner Headrest & Push Handles - Chrome Finish	83
Accessories - Recliner Headrest & Push Handles - Color Finish	85
Accessories - Solid Back & Solid Seat Inserts	78
Accessories - Stroller Handle	82
Arms - Conventional Adjustable Height	19
Arms - Conventional Fixed Height	27
Arms - Space Saver Adjustable Height	23
Arms - Space Saver Fixed Height	30
Back - Recliner Assembly	12
Canes - Back	15
Casters - Front, Forks and Mounting Hardware	37
Crossbraces	17
Frame Finishes, Packaging, Literature and Upholstery Colors	5
Frames - Hardware	10
Frames - Side	7
Front Riggings - Calf Pads	75
Front Riggings - Footplate with Hardware Assembly - J932A/J933A	54

TABLE of CONTENTS

Description	Page Number
Front Riggings - Footrest & Legrest Accessories	76
Front Riggings - Footrest Support Assembly J932A	56
Front Riggings - Footrest Support Assembly J932A -14	58
Front Riggings - Footrest Support Assembly J933A	60
Front Riggings - Legrest Support Assembly J934A - 14 - Chrome	66
Front Riggings - Legrest Support Assembly J934A - Black	64
Front Riggings - Legrest Support Assembly J934A - Chrome	62
Front Riggings - Legrest Support Assembly J934A -14 - Black	68
Front Riggings - Pivot Slide Tube with Footplate Assembly J932A -14	70
Front Riggings - Pivot Slide Tube with Footplate Assembly J934A	72
Front Riggings - Pivot Slide Tube with Footplate Assembly J934A -14	74
Seat Extension Kit	36
Upholstery - Back	33
Upholstery - Seat	34
Wheel Locks - Disk Style, Conventional & Space Saver Arm (After 8/14/03)	48
Wheel Locks - Link-Pivot Style - Conventional Arm, Builder Chairs (After 4/15/07)	50
Wheel Locks - Link-Pivot Style - Space Saver Arm, Builder Chairs (After 4/15/07)	52
Wheel Locks - Link-Pivot Style, Conventional Arm (Before 8/14/03)	44
Wheel Locks - Link-Pivot Style, Space Saver Arm (Before 8/14/03)	46
Wheels - Handrims and Attaching Hardware	43
Wheels - Rear	39

TABLE of CONTENTS

Description	Page Number
Wheels - Rear - Hardware	41

Frame Finishes, Packaging, Literature and Upholstery Colors

NOTE: When adjusting or replacing parts, please refer to your Owner's Manual for assistance.

Frame Finishes

Color Number	Part Number	Color Description
08P	1031456-NLA	Deep Purple
24P	1031450-NLA	Wet Black
30P	1031448-NLA	Sunny Yellow
33P	1031455-NLA	Red
34P	N/A	Flat Black
61P	1046321-NLA	Electric Red
62P	1046320-NLA	Electric Blue
66P	1046322-NLA	Electric Purple
71P	N/A	Silver Vein
83P	1073895-NLA	Royal Blue with Grape
84P	1073896-NLA	Lavender
104P	1101316-NLA	Electric Teal
105P	1101326-NLA	Midnight Blue

Frame Finishes, Packaging, Literature and Upholstery Colors

NOTE: When adjusting or replacing parts, please refer to your Owner's Manual for assistance.



Packaging and Literature

Part Number	Description
1101927	Complete Packaging
1100868	Owner's Manual

Upholstery Colors

Color Number	Description
U240	Black Nylon
U67	Black Vinyl
U550	Midnight Blue Vinyl

Frames without Hardware

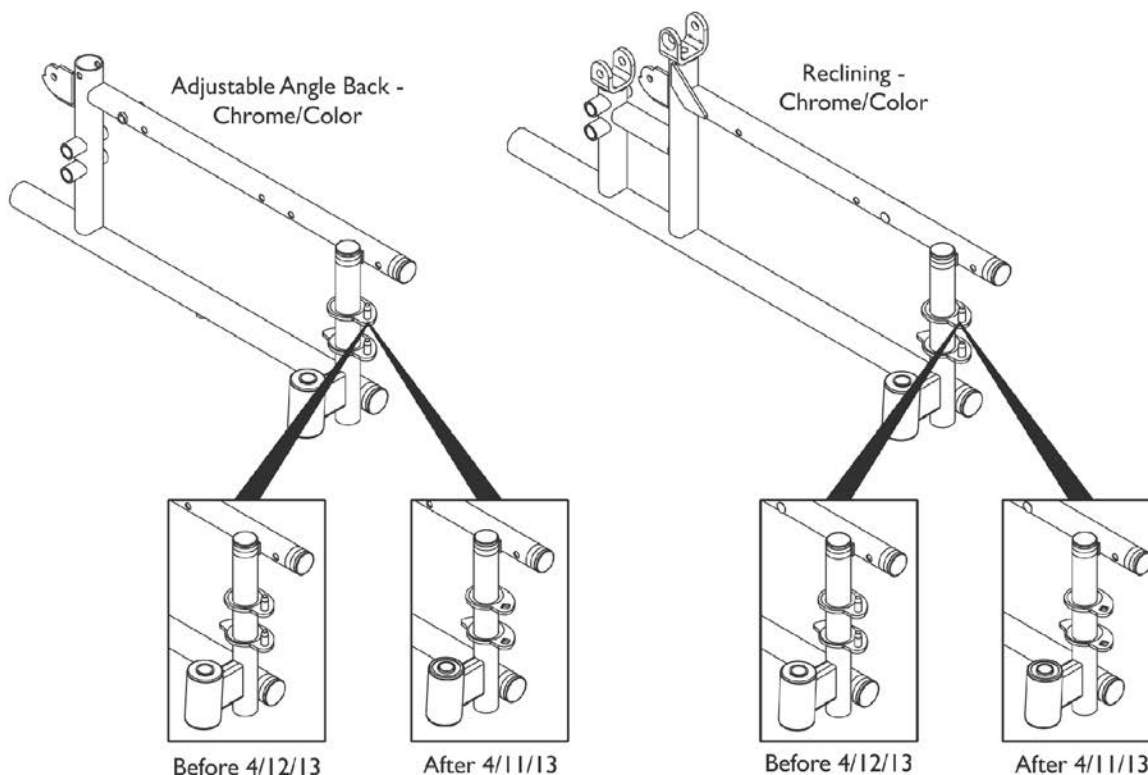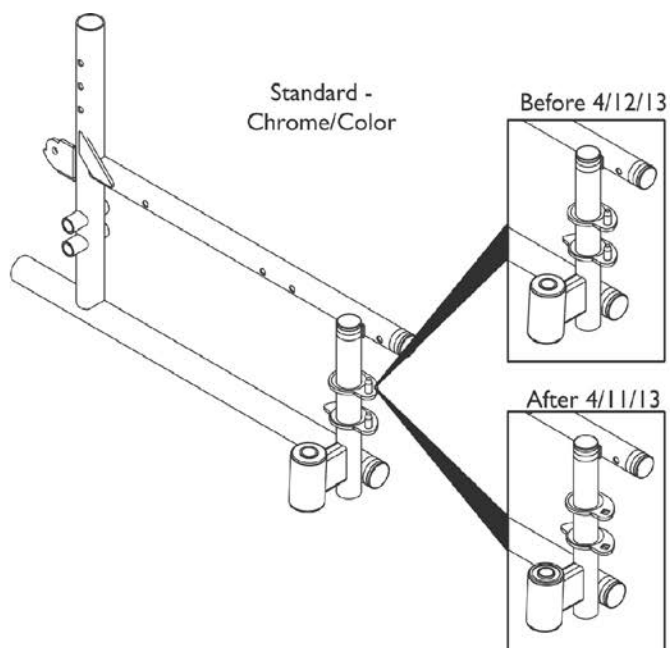

Frames without Hardware

Standard

Side	Chrome	Color
10" Deep - Right	1100935-NLA See Note(s) B1, B	8881100933 See Note(s) B, B1, A
10" Deep - Left	1100936-NLA See Note(s) B1, B	8881100934 See Note(s) B1, B, A
12" Deep - Right	1100939-NLA See Note(s) B1, B	8881100937 See Note(s) B1, B, A
12" Deep - Left	1100940-NLA See Note(s) B1, B	8881100938 See Note(s) B, A, B1
14" Deep - Right	1100943-NLA See Note(s) B1, B	8881100941 See Note(s) B, B1, A
14" Deep - Left	1100944-NLA See Note(s) B, B1	8881100942 See Note(s) B, B1, A
NOTE: A - Specify frame finish, see category titled "Frame Finishes, Packaging, Literature and Upholstery Colors". B - Hinge Pin Replacement Hardware Kit located on the Frame Hardware page is used on Frames w/ welded-on Hinge Pins manufactured before 4/12/13. Side Frames after 4/11/13 have Hinge Pin Saddle Plates w/ square holes to attach REMOVABLE Non-Welded Hinge Pins. B1 - All Side Frames ordered after 4/11/13 require new Side Frame Hardware that will include the removable Hinge Pins also. Removable Hinge Pins are not sold separately.		

Reclining


Side	Chrome	Color
10" Deep - Right	1100947-NLA See Note(s) B, B1	8881100945 See Note(s) A, B, B1
10" Deep - Left	1100948-NLA See Note(s) B1, B	8881100946 See Note(s) A, B, B1
12" Deep - Right	1100951-NLA See Note(s) B1, B	8881100949 See Note(s) B1, B, A
12" Deep - Left	1100952-NLA See Note(s) B1, B	8881100950 See Note(s) B, A, B1
14" Deep - Right	1100955-NLA See Note(s) B1, B	8881100953 See Note(s) B1, B, A
14" Deep - Left	1100956-NLA See Note(s) B1, B	8881100954 See Note(s) B1, B, A
NOTE: A - Specify frame finish, see category titled "Frame Finishes, Packaging, Literature and Upholstery Colors". B - Hinge Pin Replacement Hardware Kit located on the Frame Hardware page is used on Frames w/ welded-on Hinge Pins manufactured before 4/12/13. Side Frames after 4/11/13 have Hinge Pin Saddle Plates w/ square holes to attach REMOVABLE Non-Welded Hinge Pins. B1 - All Side Frames ordered after 4/11/13 require new Side Frame Hardware that will include the removable Hinge Pins also. Removable Hinge Pins are not sold separately.		

Frames without Hardware

Adjustable Angle Back

Side	Chrome	Color
10" Deep - Right	1100959-NLA See Note(s) B1, B	8881100957 See Note(s) B1, B, A
10" Deep - Left	1100960-NLA See Note(s) B1, B	8881100958 See Note(s) B1, B, A
12" Deep - Right	1100963-NLA See Note(s) B1, B	8881100961 See Note(s) B1, B, A
12" Deep - Left	1100964-NLA See Note(s) B1, B	8881100962 See Note(s) B1, B, A
14" Deep - Right	1100967-NLA See Note(s) B1, B	8881100965 See Note(s) B1, B, A
14" Deep - Left	1100968-NLA See Note(s) B1, B	8881100966 See Note(s) B1, A, B
NOTE: A - Specify frame finish, see category titled "Frame Finishes, Packaging, Literature and Upholstery Colors". B - Hinge Pin Replacement Hardware Kit located on the Frame Hardware page is used on Frames w/ welded-on Hinge Pins manufactured before 4/12/13. Side Frames after 4/11/13 have Hinge Pin Saddle Plates w/ square holes to attach REMOVABLE Non-Welded Hinge Pins. B1 - All Side Frames ordered after 4/11/13 require new Side Frame Hardware that will include the removable Hinge Pins also. Removable Hinge Pins are not sold separately.		


Frame Hardware

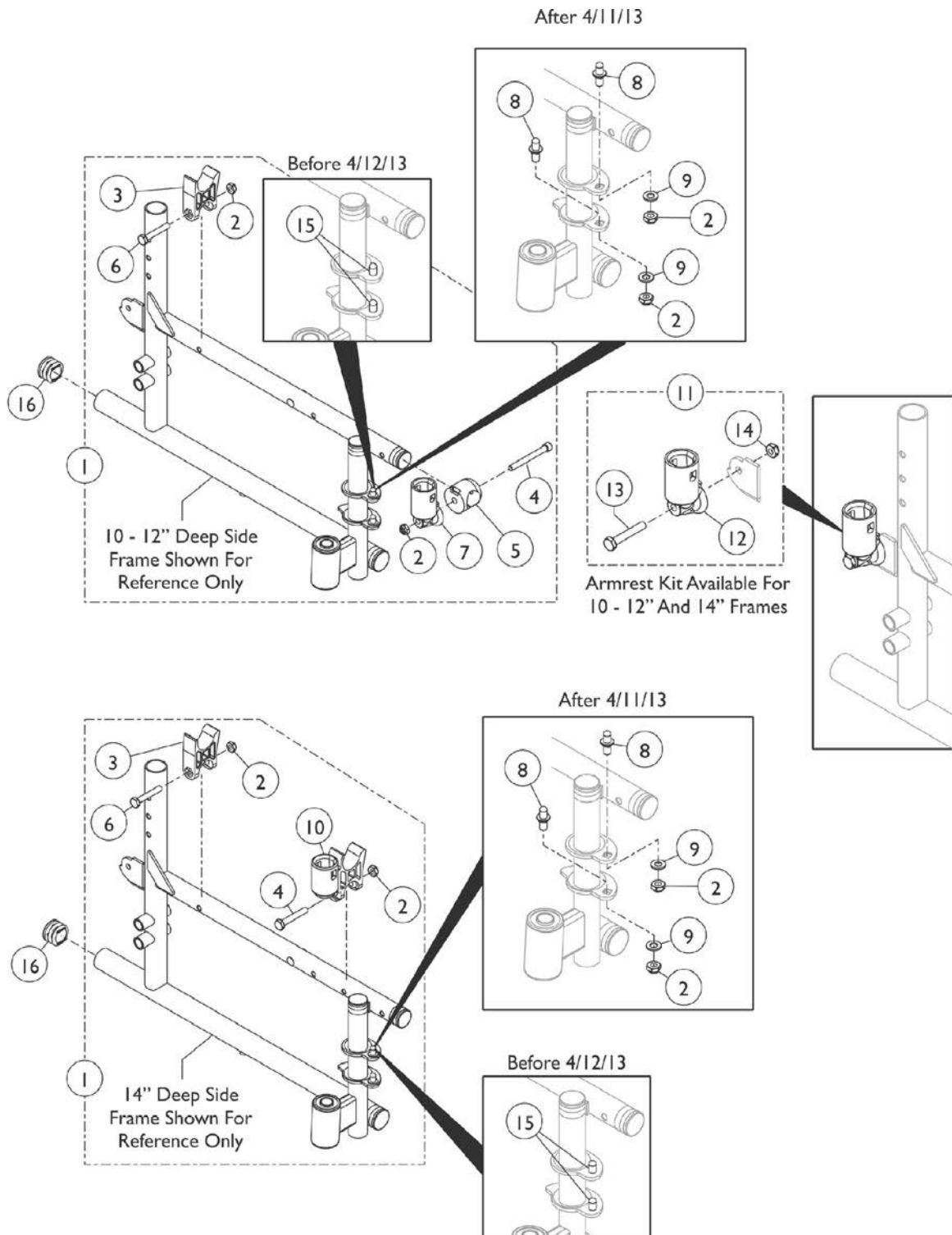

Frame Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1100969	Kit, Side Frame Hardware (10 - 12" D), (Pair)	1	1
2			Locknut (1/4-20)	1	8
3			H-Block	1	2
4			Screw, Socket Head (1/4-20 x 2-1/4")	1	2
5			Spacer, Arm Socket	1	1
6			Screw, Hex Head (1/4-20 x 1-1/2")	1	2
7			Cup, Socket Pivot	1	2
8			Pin, Hinge	1	4
9			Washer, Plain (1/4 x 1/2 x 9/16")	1	2
1	B	1056983-NLA	NLA - Kit, Side Frame Hardware (14" D), (Pair)	1	1
2			Locknut (1/4-20)	1	8
3			H-Block	1	2
6			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
8			Pin, Hinge	1	4
9			Washer, Plain (1/4 x 1/2 x 9/16")	1	4
10			NLA - H-Block with Socket Cup	1	2
2		60275X000	Locknut (1/4-20)	1	2
3		1107805	H-Block	1	1
4		60285X006	Screw, Socket Head (1/4-20 x 2-1/4")	1	2
5	C	1098397	Spacer, Arm Socket	1	1
6		16720X004	Screw, Hex Head (1/4-20 x 1-1/2")	1	2
7		1056406	Cup, Socket Pivot	1	2
10		1056405-NLA	NLA - H-Block with Socket Cup	1	1
11		1058139	Kit, Armrest Hardware	1	1
12			Cup, Socket Pivot	1	1
13			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
14			Locknut (1/4-20)	1	1
15	E,E1	1096781	Kit, Hinge Pin Replacement Hardware (Pair)	1	1
15			NLA - Wrench, Allen (3/32")	1	1
15			Cap, Threaded	1	4
15			Washer, Locking Split (#10)	1	4
15			Screw, Socket Button Head Cap, Regulator (8-32 x 3/8")	1	4
16	D	1029501	Plug Button, Black (1")	1	2
NOTE: A - Includes items 2-9. B - Includes items 2, 3, 6, 8-10. C - Includes items 12-14 D - Chairs without anti-tippers require plug buttons. E - Hinge Pin Replacement Hardware Kit is only used on Side Frames w/ welded-on Hinge Pins manufactured before 4/12/13. Side Frames after 4/11/13 have Hinge Pin Saddle Plates w/ square holes to attach REMOVABLE Non-Welded Hinge Pins. E1 - All Side Frames ordered after 4/11/13 require new Side Frame Hardware that will include the removable Hinge Pins also. Removable Hinge Pins are not sold separately.					

Recliner Back

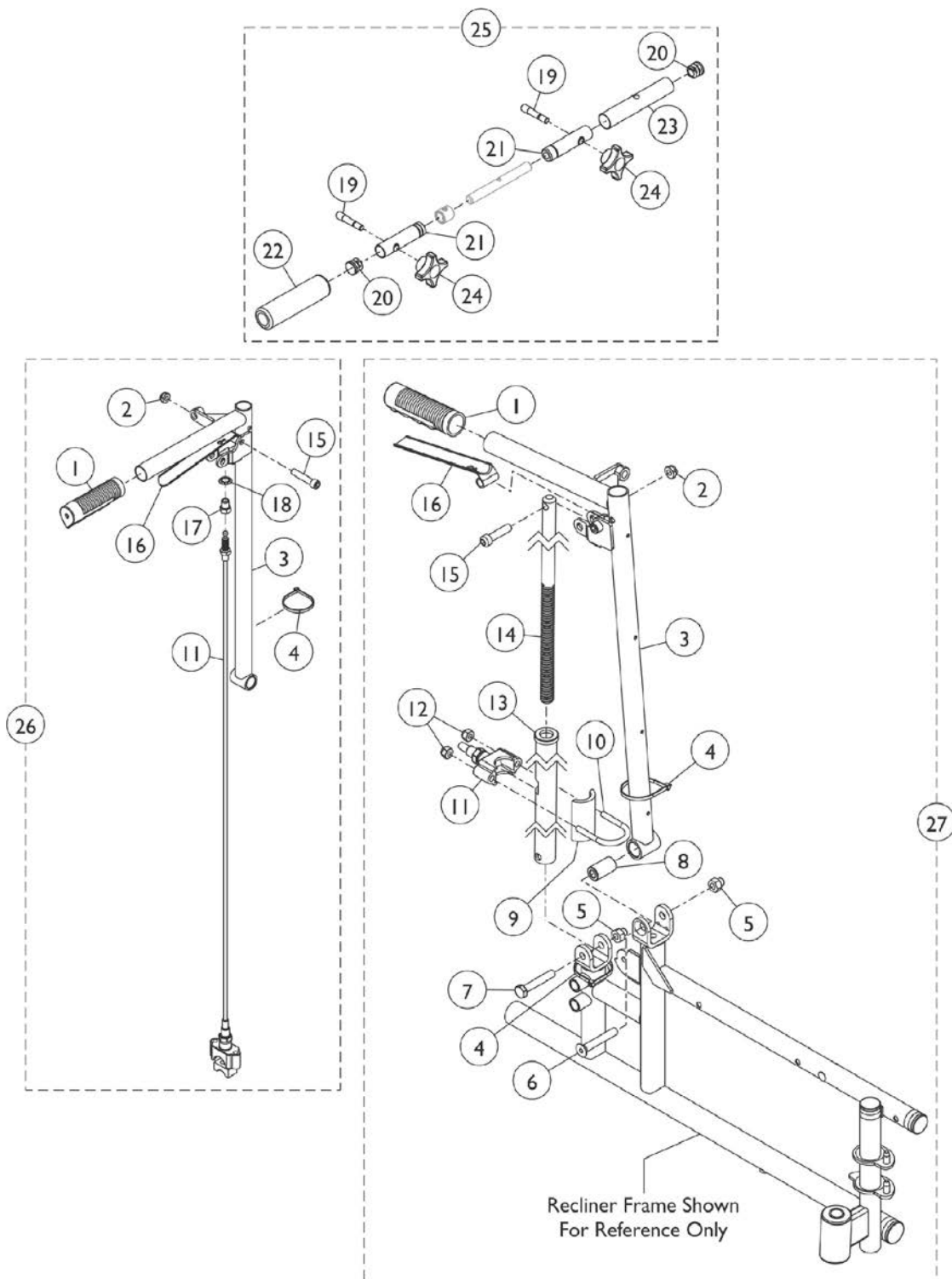


Recliner Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1021261	Grip, Hand (6")	1	1
2		60290X002	Locknut (5/16-18)	1	1
3		1101925	Tube, Back Reclining, Black - Right	1	1
3		1101926	Tube, Back Reclining, Black - Left	1	1
4		1021260	Tie Wrap (5-5/8" L)	1	2
5		1060057	Nut, Cap (5/16-24)	1	2
6		1058920	Screw, Socket Flat Head (5/16-24 x 2")	1	1
7		11239X000-NLA	NLA - Screw, Hex Head (5/16-24 x 1-3/4")	1	1
8		1058938	Bushing, Pivot (5/16 x 5/8 x 1-1/8")	1	1
9		1023329-NLA	NLA - Plate Support	1	1
10		1023328	U-Bolt (1/4-20 x 1-13/16")	1	1
11		1074471	Cable and Housing Assembly	1	1
11			Cable Assembly	1	1
11			Nut, Hex (5/16"-18)	1	2
11			Housing, Support	1	1
11			NLA - Spring, Compression (27/64 x 1 x 3/64")	1	1
11			NLA - Sector, Threaded	1	1
11			Tubing, Heatshrink Adhesive (1-1/2")	1	1
11			NLA - Nut, Swivel (5/16-18 x 21/32")	1	1
12		7930307-NLA	NLA - Locknut (1/4-20")	25	2
13		1098445	Tube and Cap Assembly, Black	1	1
14		1100700	Rod, Locking	1	1
15		1021344	Screw, Socket Head (5/16-18 x 1-1/2")	1	1
16		1021085	Lever, Brake Release, Black	1	1
17		1070840-NLA	NLA - Nut, Swivel (5/16-18 x 21/32")	1	1
18		1071676	Fastener, Spring Grip	1	1
19		1021091	Stud (5/16-18 x 1-3/4")	1	2
20		1027097	Plug Button, Black (3/4")	1	2
21		1101934-NLA	NLA - Tube, Right	1	1
21		1101935-NLA	NLA - Tube, Left	1	1
22		1101930	Grip, Handle, Black	1	1
23	G	1101932-NLA	NLA - Cover, Tube, Chrome (10 - 12" W)	1	1
23	G	1021353-NLA	NLA - Cover, Tube, Chrome (14 - 15" W)	1	1
23	G	1021088-NLA	NLA - Cover, Tube, Chrome (16 - 19" W)	1	1
24		1021092	Knob, Black	1	2
25	A	1101936-NLA	NLA - Bar, Adjustable, Spreader, Chrome (10 - 12" W)	1	1
25	A	1052597-NLA	NLA - Bar, Adjustable, Spreader, Chrome (14 - 15" W)	1	1
25	A	1021276-NLA	NLA - Bar, Adjustable, Spreader, Chrome (16 - 19" W)	1	1
25	A,F	8881181933	Bar, Adjustable, Spreader, Color (10 - 12" W)	1	1
25	A,F	8881181932	Bar, Adjustable, Spreader, Color (14 - 15" W)	1	1
25	A,F	8881181931	Bar, Adjustable, Spreader, Color (16 - 19" W)	1	1
26	B	1102013	Recliner Assembly - Right	1	1


Recliner Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
26	B	1102014	Recliner Assembly - Left	1	1
27	E	1102015	Recliner Back Assembly - Right	1	1
27	E	1102016	Recliner Back Assembly - Left	1	1
	C,D	1098443	Spacer, Tube (9/16 x 5/8 x 1/2")	1	1
	D	1054595-NLA	NLA - Headrest Extension Tube Assembly, Black	1	2
	D,H	8881181852	Headrest Extension Tube Assembly, Color	1	2
NOTE: A - Includes items 19-24. B - Includes items 1-4, 11, 15 - 18. C - Tube spacer is placed over non-threaded end of Locking Rod (item #14) before inserting into Tube & Cap Weldment (item #13). D - Not Shown. E - Includes items 1-16. F - Specify 71P Flat Black when ordering. G - The Color Tube Cover is not available for color frame models. Must order item #25 as a replacement. H - Specify 71P Silver Vein when ordering.					


Back Canes and Hardware

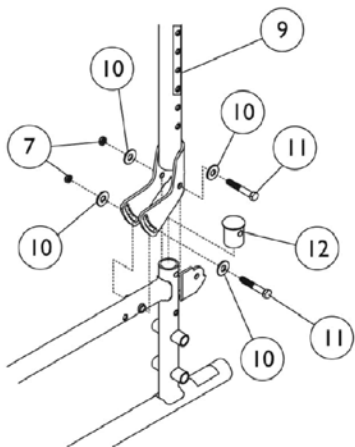


Adjustable Angle Back Hardware

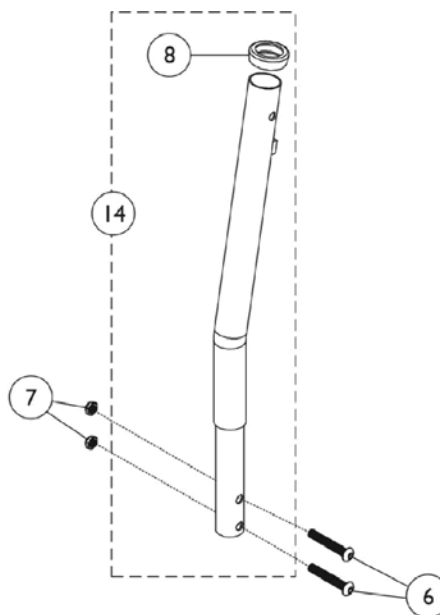

Adjustable Height Back with
10 Degree Bend for Stroller Handles

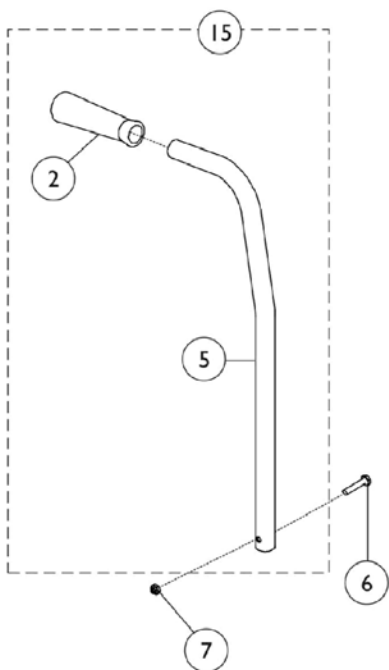

Upright Extended Assembly
with 10 Degree Bend

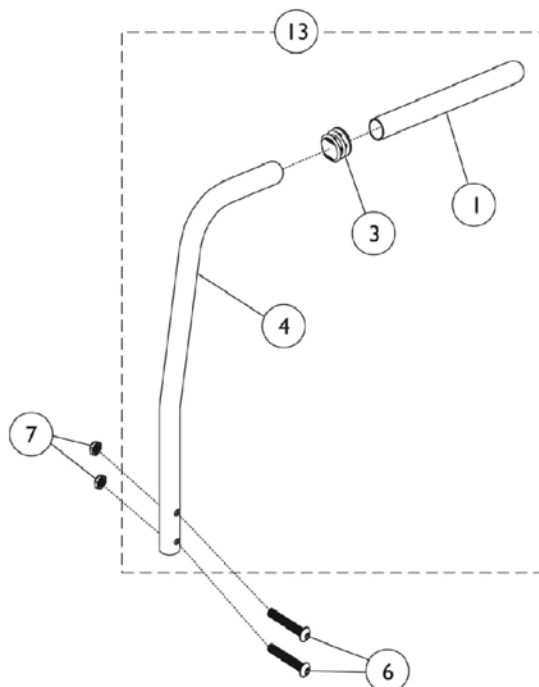


Adjustable Height Back
with 10 Degree Bend

Back Canes and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1006229	Grip, Foam (8")	1	1
2		1057248	Grip, Hand, Foam (4-3/4")	1	1
3		1027097	Plug Button, Black (3/4")	1	1
4		1020545-NLA	NLA - Cane, Back with Bend	1	1
5		1064524	Cane, Back, 10 Degree Bend	1	1
6		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
7		60275X000	Locknut (1/4-20)	1	2
8		15680X013	Anti-Rattle (1")	1	1
9		1100615-NLA	NLA - Bracket, Adjustable Angle, Chrome	1	1
9		8881100613	Bracket, Adjustable Angle, Color	1	1
10		16348X009-NLA	NLA - Washer, Steel (3/16 x 9/16 x 1/16")	1	4
11		1013046-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
12		1063137	Plug, End	1	1
13	A	1028372	Back Cane Assembly, 10 Degree Bend	1	1
14	B	1101949	Adjustable Back Cane Assembly	1	1
15	C	1064525-NLA	NLA - Cane Assembly, Extended with 10 Degree Bend	1	1
			Grip, Hand, Foam (4-3/4")	1	1
			Cane, Back, 10 Degree Bend	1	1
	D,E	1042623	Hardware, Mounting	1	2
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	4
			Locknut (1/4-20)	1	4
NOTE: A - Includes items 1, 3 and 4. B - Includes Cane, Bushing and Anti-Rattle. C - Includes items 2 and 5. D - Includes items 6-7. E - Not Shown.					


Crossbraces and Hardware

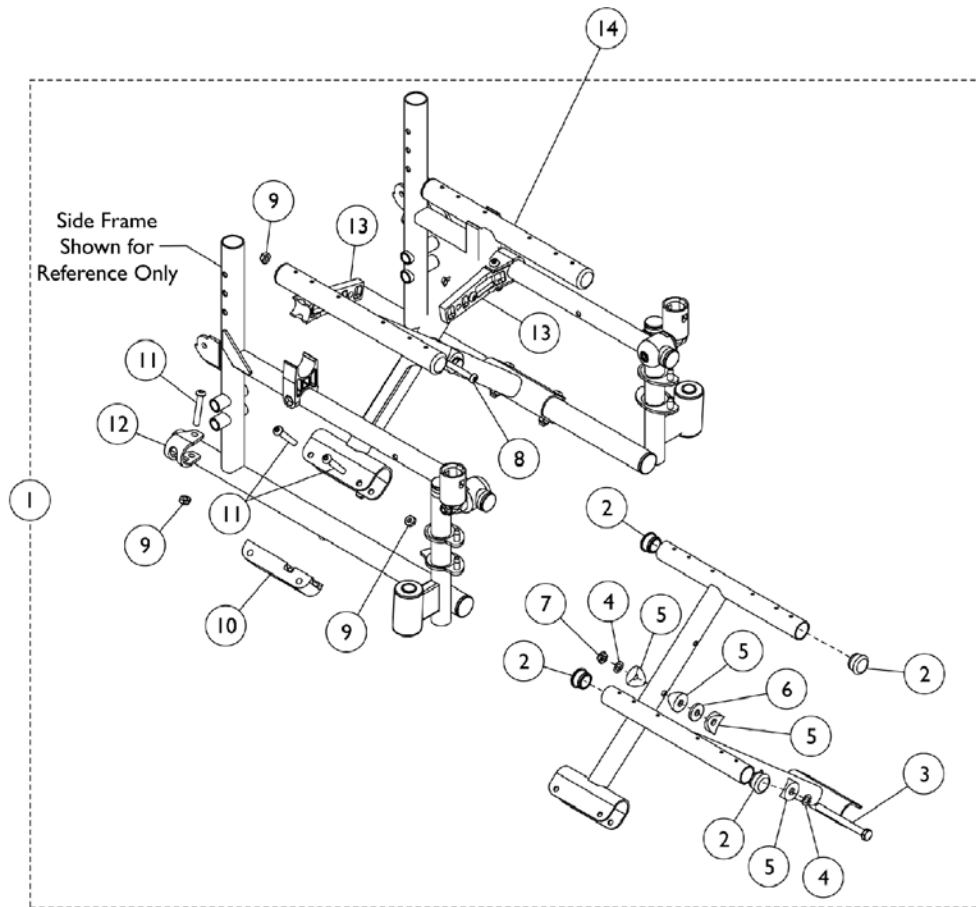

Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1100971	Hardware, Crossbrace (10 - 12" W)	1	1
1	B,C	1046925	Hardware, Crossbrace (14 - 15" W)	1	1
1	B,D	1034077	Hardware, Crossbrace (16 - 18" W)	1	1
2	F	1056328	Plug Button, Black (1")	1	4
2	C,D	1029501	Plug Button, Black (1")	1	4
3		1016261	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
4		1016652	Washer (5/16 x 9/16 x 1/32")	1	2
5		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
6		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
7		60264X001	Locknut (5/16-24)	1	1
8		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	2
9		60275X000	Locknut (1/4-20)	1	8
10		1012172-NLA	NLA - Clamp, Saddle	1	2
11		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	6
12		1012180	Bracket, Pivot, Black	1	2
13	A	1100610	Link, Pivot, Black (10 - 12" W)	1	2
13	C	1046501	Link, Pivot, Black (14 - 15" W)	1	2
13	D	1012173	Link, Pivot, Black (16 - 18" W)	1	2
14		1098453	Crossbrace w/o Hardware, Black (10" W x 10" D)	1	2
14		1098455	Crossbrace w/o Hardware, Black (12" W x 10" D)	1	2
14	E	1098454	Crossbrace w/o Hardware, Black (12" W x 12" D)	1	2
14	E	1098456	Crossbrace w/o Hardware, Black (14" W x 12" D)	1	2
14	E	1119543	Crossbrace w/o Hardware, Black (16" W x 12" D)	1	2
NOTE: A - For 10-12" Wide Chairs. B - Includes items 2-13. C - For 14-15" Wide Chairs. D - For 16-18" Wide chairs. E - A Seat Extension Kit is needed for chairs over 12" deep. See category titled, "Seat Extension Kits." F - Use 1029501 as replacement part.					

Conventional Adjustable Height Arms

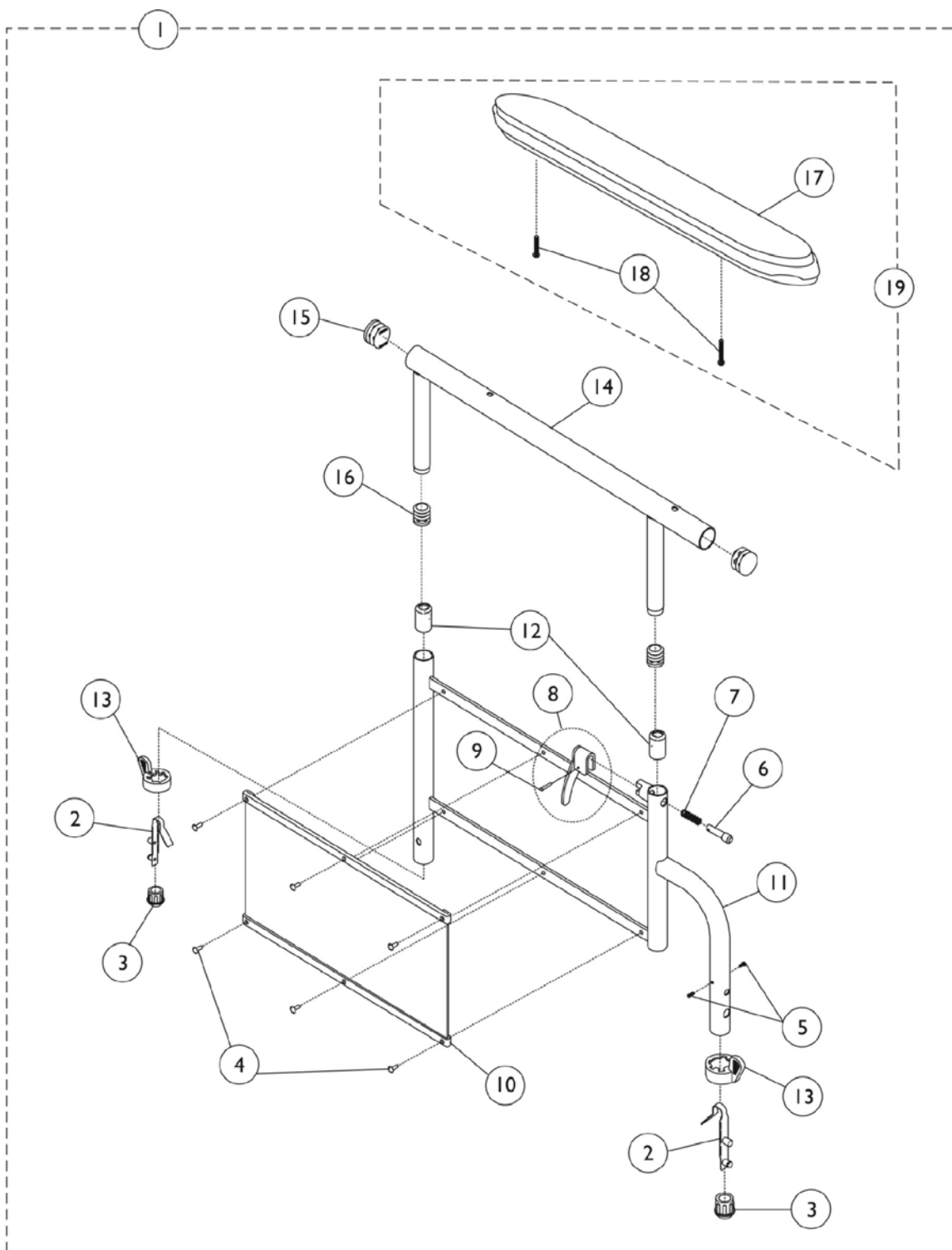

Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,E	8881130809	Kit, Armrest, Full Length, Black - Right (Style 19JY)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Right	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Arm Assembly, Adjustable Height (2" Lower)	1	1
12			Anti-Rattle	1	2
13			Handle, Quad Release	1	2
14			Tube, Armrest, Full Length, Black (2" Lower)	1	1
15			Plug Button, Black (7/8")	1	2
16			Plug Button, Black (5/8")	1	2
17			NLA - Pad, Arm, Full Length w/ Law Label	1	1
18			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
1	A,E	8881130810	Kit, Armrest, Full Length, Black - Left (Style 19JY)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Left	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Arm Assembly, Adjustable Height (2" Lower)	1	1
12			Anti-Rattle	1	2
13			Handle, Quad Release	1	2
14			Tube, Armrest, Full Length, Black (2" Lower)	1	1
15			Plug Button, Black (7/8")	1	2
16			Plug Button, Black (5/8")	1	2
17			NLA - Pad, Arm, Full Length w/ Law Label	1	1
18			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
1		8881130807	Kit, Armrest, Desk Length, Black - Right (Style 29JY)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Right	1	1

Conventional Adjustable Height Arms


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Arm Assemblby, Adjustable Height (2" Lower)	1	1
12			Anti-Rattle	1	1
13			Handle, Quad Release	1	1
14			Tube, Armrest, Desk Length, Black (2" Lower)	1	1
15			Plug Button, Black (7/8")	1	2
16			Plug Button, Black (5/8")	1	2
17			NLA - Arm Pad, Desk Length w/ Law Label	1	1
18			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
1	A,E	8881130808	Kit, Armrest, Desk Length, Black - Left (Style 29JY)	1	1
2			Spring, Torsion with Button Lock	1	2
3			Plug, Armrest, Black (3/4") (18GA)	1	2
4			Rivet	1	6
5			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
6			Pin, Locking	1	1
7			Spring, Arm Lock	1	1
8			Handle, Lock, Black - Left	1	1
9			Rivet, Tubular	1	1
10			Guard, Clothing	1	1
11			Arm Assemblby, Adjustable Height (2" Lower)	1	1
12			Anti-Rattle	1	1
13			Handle, Quad Release	1	1
14			Tube, Armrest, Desk Length, Black (2" Lower)	1	1
15			Plug Button, Black (7/8")	1	2
16			Plug Button, Black (5/8")	1	2
18			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17			NLA - Arm Pad, Desk Length w/ Law Label	1	1
19	B,E	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
17			NLA - Pad, Arm, Full Length w/ Law Label	1	1
18			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
19	B,E	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
17			NLA - Arm Pad, Desk Length w/ Law Label	1	1
18			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
	C	1092472-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Right/Left	1	1
	C	1092469-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Right/Left	1	1
	D	1092459-NLA	NLA - Arm Base Assembly w/o Arm Pad, Full/ Desk Length, Chrome - Right	1	1
	D	1092460-NLA	NLA - Arm Base Assembly w/o Arm Pad, Full/ Desk Length, Chrome - Left	1	1
	F	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	1

Conventional Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	F	7100054	Pin, Locking	1	1
			Spring, Arm Lock	1	1
			Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	1
			Pin, Locking	1	1
			Spring, Arm Lock	1	1
NOTE: A - Includes items 2-16. B - Includes items 17 and 18. C - Not Shown. Includes items 14-16. D - Not Shown. Includes items 2-13. E - Specify color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging and Literature." F - Not shown. Includes items 6-9.					

Space Saver Adjustable Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

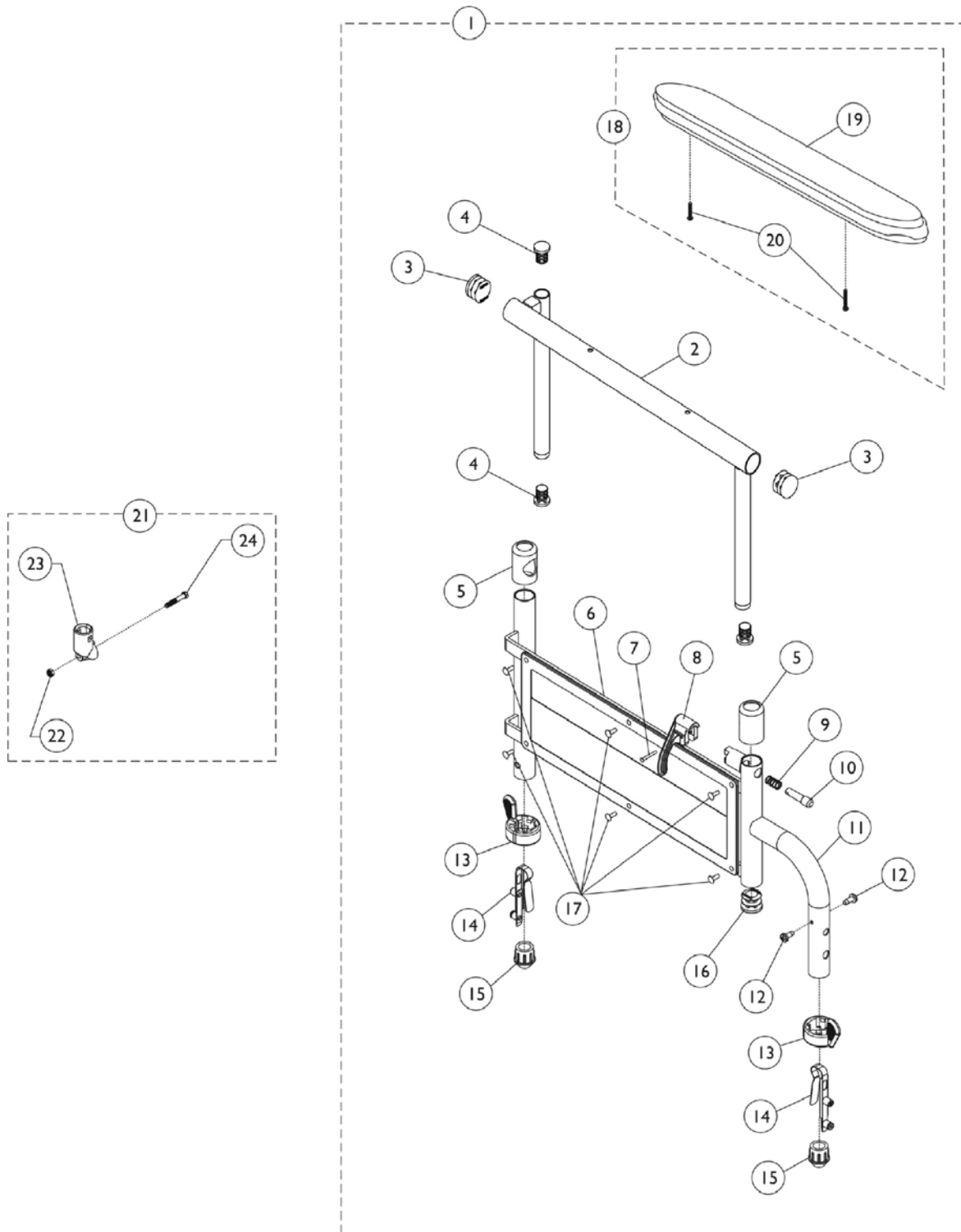

Space Saver Adjustable Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,G	8881130874	Kit, Armrest, Full Length, Black - Right (Style 79JY)	1	1
2			Tube, Armrest, Full Length, Black - Right	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Right	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black -Right	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
1	A,G	8881130875	Kit, Armrest, Full Length, Black - Left (Style 79JY)	1	1
2			Tube, Armrest, Full Length, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black - Left	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
1	A,G	8881130862	Kit, Armrest, Desk Length, Black - Right (Style 89JY)	1	1
2			Tube, Armrest, Desk, Black - Right	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2

Space Saver Adjustable Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Right	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black -Right	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			NLA - Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
1	A,G	8881130863	Kit, Armrest, Desk Length, Black - Left (Style 89JY)	1	1
2			Tube, Armrest, Desk, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Tube, Armrest Base, Adjustable Height, Black - Left	1	1
12			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	2
16			Plug Button, Black (3/4")	1	1
17			Rivet	1	6
19			NLA - Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
18	B,G	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
18	B,G	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
19			NLA - Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
22	C	1058139	Kit, Armrest Hardware	1	1
23			Cup, Socket Pivot	1	1
24			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1





Space Saver Adjustable Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
25			Locknut (1/4-20)	1	1
	D,E	1102510-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Right	1	1
	D,E	1102511-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Left	1	1
	D,E	1102512-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Right	1	1
	D,E	1102513-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Left	1	1
	E,F	1102528-NLA	NLA - Arm Base Assembly w/o Arm Pad, Full/ Desk Length, Chrome - Right	1	1
	E,F	1102529-NLA	NLA - Arm Base Assembly w/o Arm Pad, Full/ Desk Length, Chrome - Left	1	1
	H	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	1
			Pin, Locking	1	1
			Spring, Arm Lock	1	1
	H	7100054	Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	1
			Pin, Locking	1	1
			Spring, Arm Lock	1	1
NOTE: A - Includes items 2-20. B - Includes items 19 and 20. C - Includes items 23-25. D - Includes 2-4. E - Not Shown. F - Includes 5-17. G - Specify color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging and Literature." H - Not shown. Includes items 7-10.					

Conventional Fixed Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

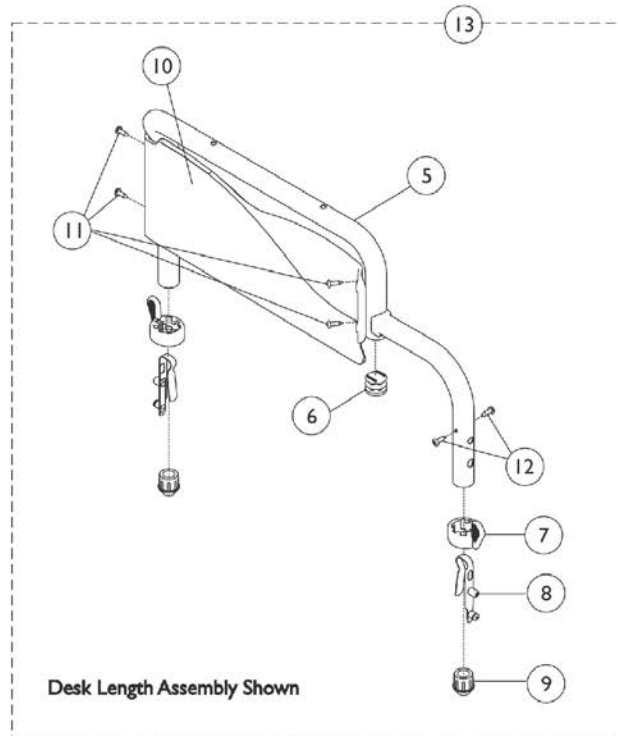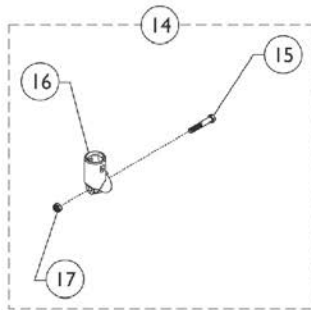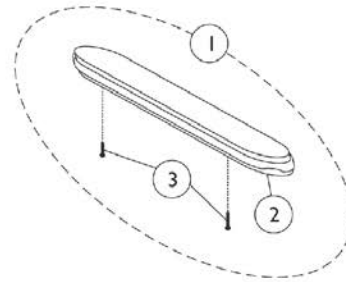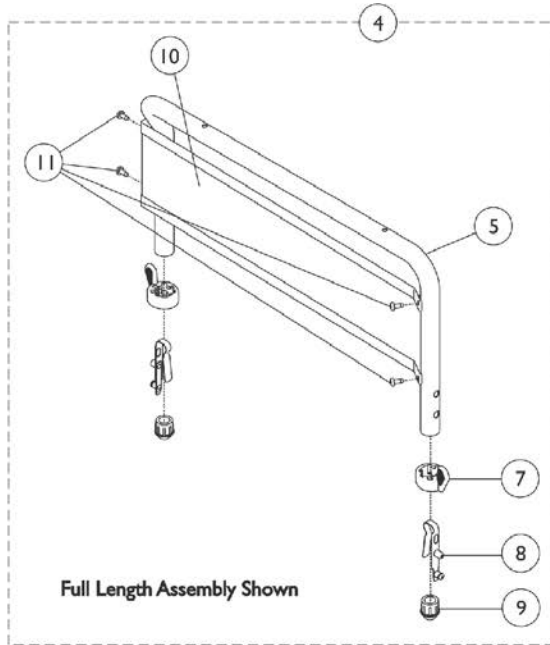

Conventional Fixed Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
2			NLA - Pad, Arm, Full Length w/ Law Label	1	1
3			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
1	A,B	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
2			NLA - Arm Pad, Desk Length w/ Law Label	1	1
3			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4	C,D	1100986	Assembly, Armrest, Full Length, Black - Right/Left	1	1
5			Tube, Armrest, Full Length, Black - Right/Left	1	1
7			Handle, Quad Release	1	2
8			Spring, Torsion with Button Lock	1	2
9			Plug, Armrest, Black (3/4") (20GA)	1	2
10			Guard, Clothing, Black Steel, Full Length - Right/Left	1	1
11			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
13		1100979	Assembly, Conventional Armrest, Desk Length, Black - Right	1	1
5			Tube, Armrest, Desk Length, Black - Right	1	1
6			Plug Button, Black (3/4")	1	1
7			Handle, Quad Release	1	2
8			Spring, Torsion with Button Lock	1	2
9			Plug, Armrest, Black (3/4") (20GA)	1	2
10			Guard, Clothing, Desk Length - Right	1	1
11			Screw, Self Tapping (#6-20 x 1/2")	1	4
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13	D,E	1100980	Assembly, Conventional Armrest, Desk Length, Black - Left	1	1
5			Tube, Armrest, Desk Length, Black - Left	1	1
6			Plug, Armrest, Black (3/4") (20GA)	1	2
7			Handle, Quad Release	1	2
9			Plug Button, Black (3/4")	1	1
8			Spring, Torsion with Button Lock	1	2
10			Guard, Clothing, Desk Length - Left	1	1
11			Screw, Self Tapping (#6-20 x 1/2")	1	4
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
14	F	1058139	Kit, Armrest Hardware	1	1
15			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
16			Cup, Socket Pivot	1	1
17			Locknut (1/4-20)	1	1
	G	1117367-NLA	NLA - Kit, Clothing Guard, Desk Length	1	1
			Screw, Self Tapping (#6-20 x 1/2")	1	8
			Guard, Clothing, Desk Length - Right	1	1
			Guard, Clothing, Desk Length - Left	1	1
	G	1117368	NLA - Kit, Clothing Guard, Full Length	1	1
			Screw, Self Tapping (#6-20 x 1/2")	1	8





Conventional Fixed Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
			Guard, Clothing, Full Length - Right	1	1
			Guard, Clothing, Full Length - Left	1	1
NOTE: A - Specify color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging and Literature." B - Includes items 2 and 3. C - Includes items 5, 7-11. D - Due to style change, it is recommended when ordering an arm a clothing guard kit also be ordered for chairs built before 11/25/03. E - Includes items 5-12. F - Includes items 15-17. G - Not shown.					

Space Saver Fixed Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

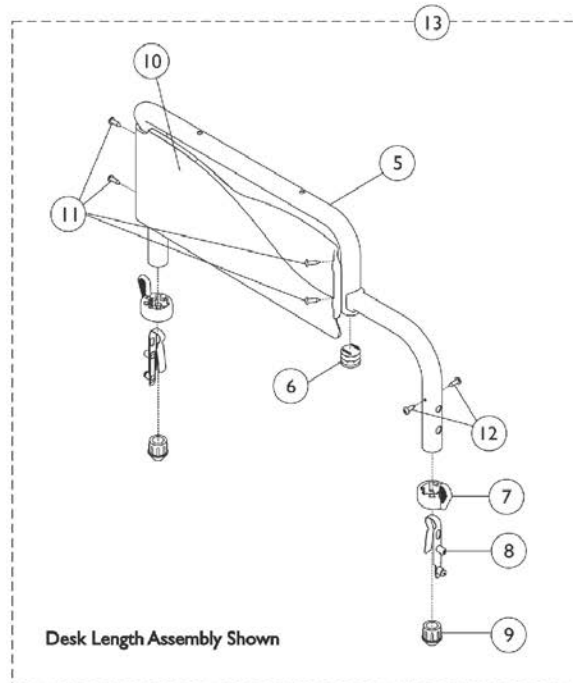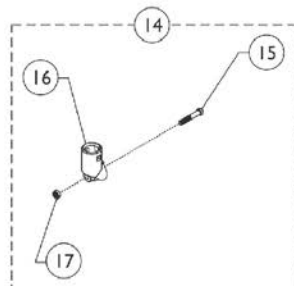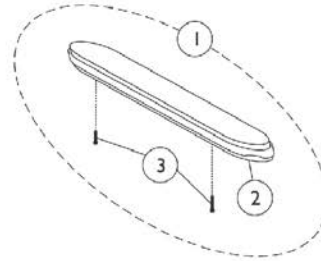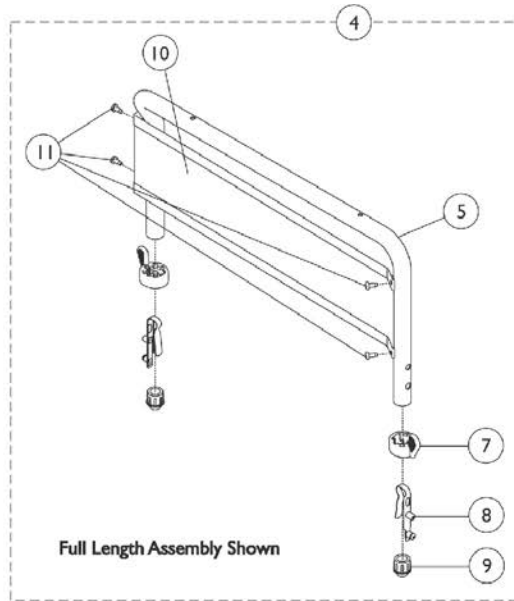

Space Saver Fixed Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
2			NLA - Pad, Arm, Full Length w/ Law Label	1	1
3			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
1	A,B	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
2			NLA - Arm Pad, Desk Length w/ Law Label	1	1
3			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
4	C,D	1100983	Assembly, Space Saver Fixed, Armrest, Full Length, Black - Right	1	1
5			Tube, Armrest, Full Length, Black - Right	1	1
7			Handle, Quad Release	1	2
8			Spring, Torsion with Button Lock	1	2
9			Plug, Armrest, Black (3/4") (20GA)	1	2
10			Guard, Clothing, Black Steel, Full Length - Right	1	1
11			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
4		1100984	Assembly, Space Saver Fixed, Armrest, Full Length, Black - Left	1	1
5			Tube, Armrest, Full Length, Black - Left	1	1
7			Handle, Quad Release	1	2
8			Spring, Torsion with Button Lock	1	2
9			Plug, Armrest, Black (3/4") (20GA)	1	2
10			Guard, Clothing, Black Steel, Full Length - Left	1	1
11			Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
12	C,E	1100975	Assembly, Space Saver Fixed, Armrest, Desk Length, Black - Right	1	1
5			Tube, Armrest, Desk Length, Black - Right	1	1
6			Plug Button, Black (3/4")	1	1
7			Spring, Torsion with Button Lock	1	2
8			Handle, Quad Release	1	2
9			Plug, Armrest, Black (3/4") (20GA)	1	2
10			Guard, Clothing, Desk Length - Right	1	1
12			Screw, Self Tapping (#6-20 x 1/2")	1	4
13			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
12		1100976	Assembly, Space Saver Fixed, Armrest, Desk Length, Black - Left	1	1
5			Tube, Arm, Desk Length, Black - Left	1	1
6			Plug Button, Black (3/4")	1	1
7			Spring, Torsion with Button Lock	1	2
8			Handle, Quad Release	1	2
9			Plug, Armrest, Black (3/4") (20GA)	1	2
10			Guard, Clothing, Desk Length - Left	1	1
12			Screw, Self Tapping (#6-20 x 1/2")	1	4
13			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
14	F,G	1058139	Kit, Armrest Hardware	1	1

Space Saver Fixed Height Arms

NOTE: Color arms are no longer available (NLA) and have been replaced with black arms.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
15			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
16			Cup, Socket Pivot	1	1
17			Locknut (1/4-20)	1	1
	G,H	1123305-NLA	NLA - Kit, Clothing Guard, Desk Length w/ Screws (Pair)	1	1
	G,H	1123306-NLA	NLA - Kit, Clothing Guard, Full Length w/ Screws (Pair)	1	1
NOTE: A - Specify color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging and Literature." B - Includes items 2 and 3. C - Due to style change, it is recommended when ordering an arm a clothing guard kit also be ordered for chairs built before 11/25/03, . D - Includes items 5, 7-11. Does not include item 6. E - Includes items 5-10, 12 and 13. F - Includes items 15-17. G - Not shown. H - Kit includes clothing guards, screw and decals. Clothing Guards are not sold individually.					

Back Upholstery without Hardware

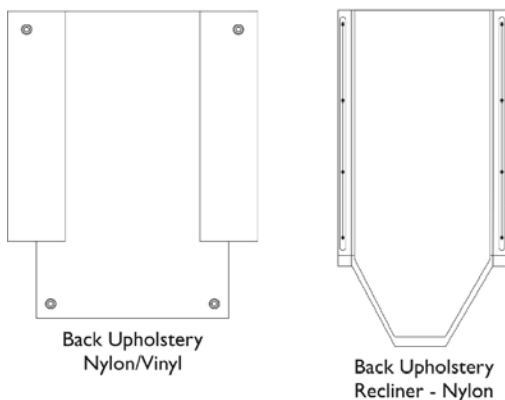


Back Upholstery

Width	Nylon	Vinyl	Recliner
10" Wide	1100670	8881101971 See Note(s) A	1102532 See Note(s) C, B
12" Wide	1100671	8881101972 See Note(s) A	1102533 See Note(s) C, B
14" Wide	1100672	8881101973 See Note(s) A	1102534 See Note(s) C, B
16" Wide	1119545	8881119546 See Note(s) A	1119549 See Note(s) C, B
NOTE: A - Specify upholstery color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging." B - Nylon upholstery is used on all chairs with reclining backs. C - At this time, this part number is only available in U240 Nylon and only comes in Black.			

Back Upholstery Hardware

Back Upholstery Hardware	Recliner Back Upholstery Hardware
1101968	1101929
Screws w/ Washers (#10 x 7/8") (Pkg of 2)	Screws w/ Washers (#10 x 7/8") (Pkg of 8)

Seat Upholstery without Hardware

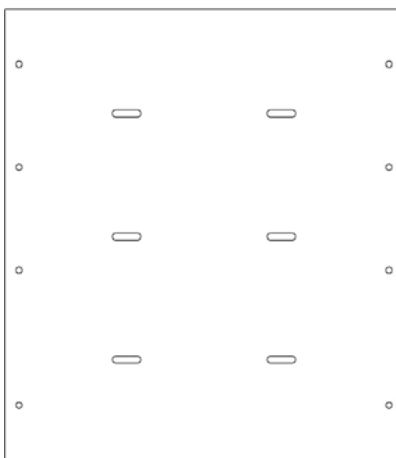

Seat Upholstery
Nylon/Vinyl

Seat Upholstery - Nylon

Width	10" Deep	12" Deep	14" Deep
10" Wide	1100668	N/A	N/A
12" Wide	1100669	1100673	N/A
14" Wide	N/A	1100674	1100675
16" Wide	N/A	N/A	1119547

NOTE: All nylon seat upholstery use U240 Black.


Seat Upholstery without Hardware

Seat Upholstery - Vinyl

Width	10" Deep	12" Deep	14" Deep
10" Wide	8881101969 See Note(s) A	N/A	N/A
12" Wide	8881101970 See Note(s) A	8881101974 See Note(s) A	N/A
14" Wide	N/A	8881101975 See Note(s) A	8881101976 See Note(s) A
16" Wide	N/A	N/A	8881119548 See Note(s) A
NOTE: A - Specify upholstery color when ordering, see category titled, "Upholstery Colors, Frame Finishes and Packaging."			


Seat Upholstery Hardware

Part Number	Description
1101929	Screws w/ Washers (#10 x 7/8") (Pkg of 8)

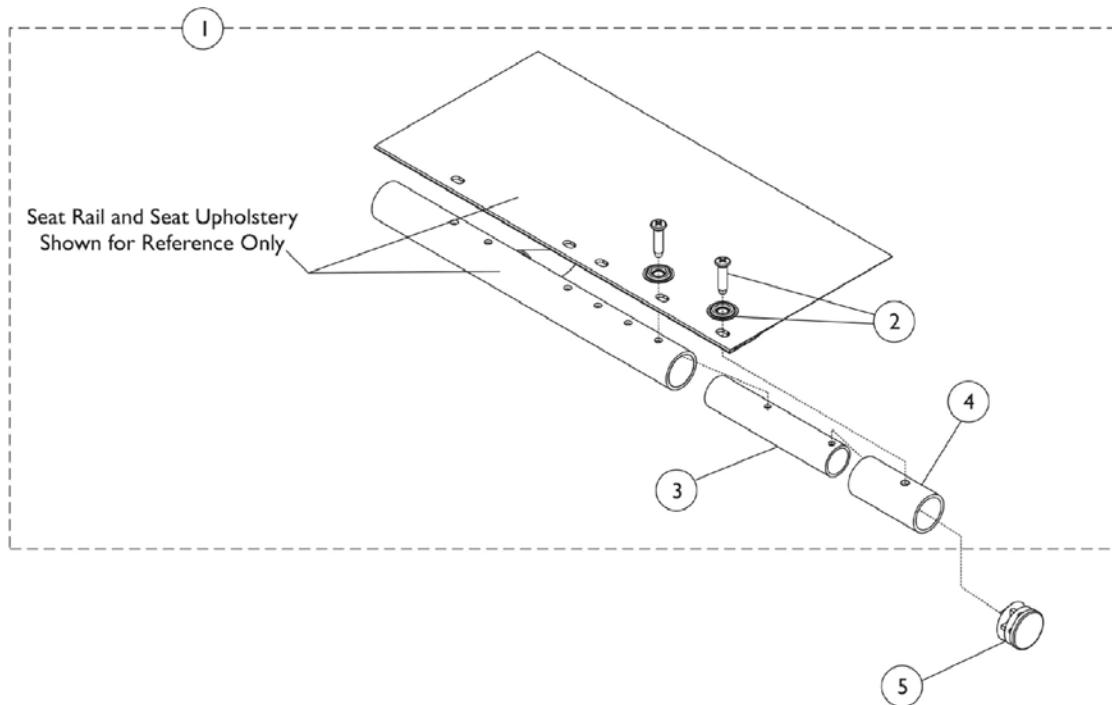

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1100972	Kit, Seat Extension (2")	1	1
2		1031095	Screws with Washers (#10-16 x 7/8")	4	2
3		1089278-NLA	NLA - Kit, Extension, Seat (2")	1	2
4		1100665	Tube, Seat Tension	1	2
5		1056328	Plug Button, Black (1")	1	2
NOTE: A - Seat extension kit includes items 2-4 but does not include a plug button.					


Casters - Front, Forks and Mounting Hardware

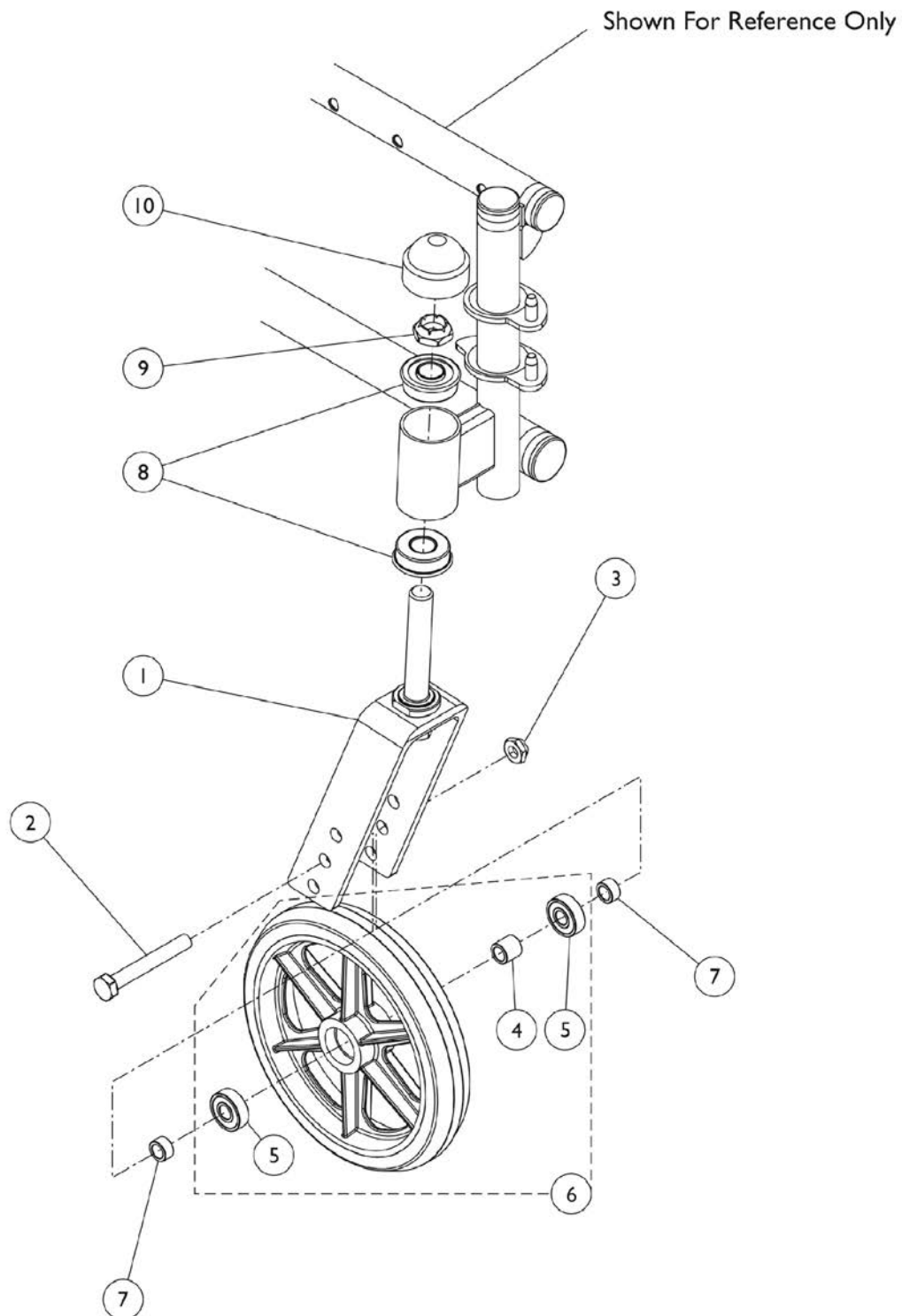

Casters - Front, Forks and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037420	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
2		40317X002	Screw, Hex Head (5/16-24 x 2-1/4")	1	1
3		60264X001	Locknut (5/16-24)	1	1
4		1026172	Spacer (5/16 x 7/16 x 15/32")	1	1
5		40117X026	Bearing, Flange (5/16")	1	2
6	A,D	1026956	Wheel Assembly, Composite Urethane, Light Grey (6" x 1")	1	1
6	A,E	1101023	Wheel Assembly, Dark Grey (6" x 1")	1	1
6	A,F	1034352	Kit, Wheel with Tire and Bearings Assembly (6") (Pair)	1	1
			Wheel Assembly, 3 Spoke Urethane w/ Bearings, Dark Grey (6" x 1")	2	1
7		1053202	Spacer (5/16 x 7/16 x 1/4")	1	2
8		1001801	Bearing, Flange (1/2")	1	2
9		1001979	Locknut (1/2-20)	1	1
10		1131189	Package, Head Tube Cap	10	1
10		1001843	Cap, Head Tube, Black	1	1
11	B,C	1061319	Hardware	1	1
NOTE: A - Includes bearings and bearing spacer. B - Not Shown. C - Includes items 2, 3, and 7. D - Model 1686. E - Model 1687. F - Model AT902.					

Rear Wheels

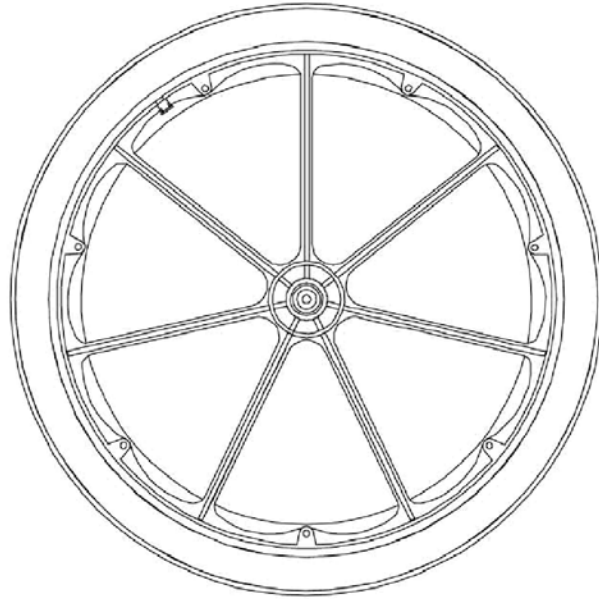

7 Spoke Composite Wheel


NOTE: 22" Rear Wheels not available on 10" or 12" Deep non-reclining chairs.

Rear Wheels

7 Spoke Composite Wheel

Wheel Style	Handrim Attaching Points	Single Wheel Assembly	Tire	Tube / Insert	Bearing	Bearing Spacer	Flat Free Rim Plug
Composite Pneumatic 1763BK (20x1-3/8")	7	1034126 See Note(s) B, A	1017519	1017520	40117X024	1015756	N/A
Composite Pneumatic with Flat Free 4763BK (20x1-3/8")	7	1034127 See Note(s) B, A	1017519	1023322	40117X024	1015756	1030936 See Note(s) C
Composite Urethane U1763BK (20 x 1-1/4")	7	1038555 See Note(s) B, A	1037774	N/A	40117X024	1015756	N/A
Composite Pneumatic 1764BK (22 x 1-3/8")	7	1027149 See Note(s) D, B, A	1018544	1018545	40117X024	1015756	N/A
Composite Pneumatic with Flat Free 4764BK (22 x 1-3/8")	7	1027150 See Note(s) D, B, A	1018544	1023323	40117X024	1015756	1030936 See Note(s) C
Composite Urethane U1764BK (22 x 1-1/4")	7	1032115 See Note(s) D, A	1028250	N/A	40117X024	1015756	N/A
NOTE: A - Does not include handrim. Includes tire, tube/insert and spacer. B - When ordering 20 and 22" wheels, a wheel lock shoe spacer may also be required. See section titled, "Wheel Locks." C - Wheels with flat free inserts have a plug which caps an opening in the rim. D - 22" rear wheels are not available on 10" or 12" deep non-reclining chairs.							

Rear Wheel Hardware

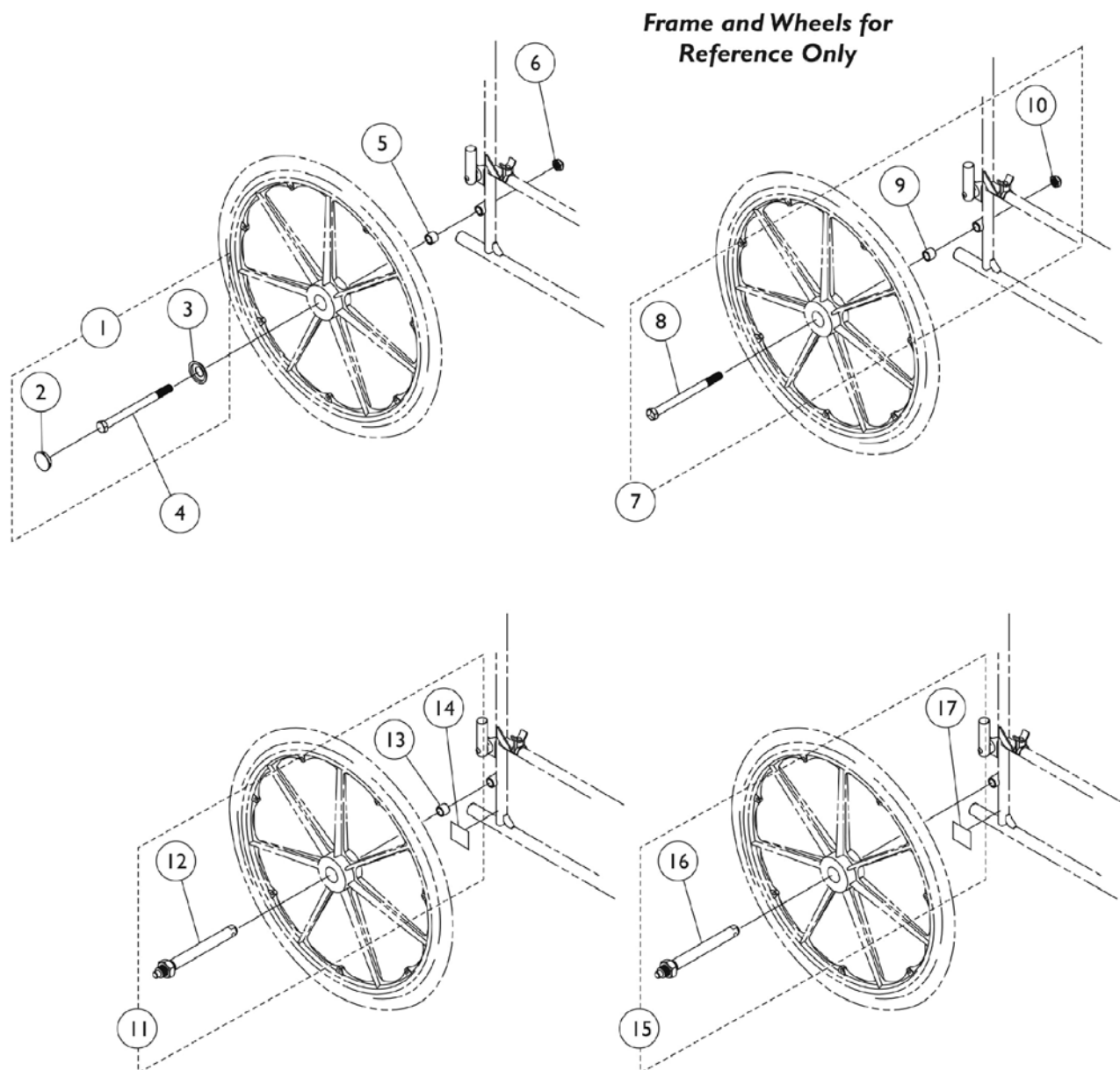

Rear Wheel Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1047943	Kit, Rear Wheel Hub Hardware	1	1
2			Cap, Hub, Black	1	1
3			Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
4	A	47203X025	Bolt, Hex Head Cap (7/16-20 x 4-3/4")	1	1
4	B	1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
5	A	1009857-NLA	NLA - Spacer (7/16 X 5/8 X 1/8")	1	1
5	B	1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
6		1045608	Locknut (7/16-20)	1	1
7	C	1034109-NLA	NLA - Kit, Rear Wheel Hardware, Conventional	1	1
8			Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
9			Spacer (7/16 x 5/8 x 1/2")	1	1
10			Locknut (7/16-20)	1	1
8		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
9		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
10		1045608	Locknut (7/16-20)	1	1
7	C	1110319	Kit, Rear Wheel Hardware, Space Saver	1	1
8			Bolt, Hex Head Cap (7/16-20 x 4-3/4")	1	1
9			Spacer (7/16 x 5/8 x 3/16")	1	1
10			Locknut (7/16-20)	1	1
11	B,D	1034113	Kit, Quick Release Rear Wheel Hardware, Conventional	1	1
12			Axle, Quick Release (7/16" x 5-7/16")	1	1
13			Spacer (7/16 x 5/8 x 1/2")	1	1
14			NLA - Label, Warning	1	1
15	A,F	1034114	Kit, Quick Release Rear Wheel Hardware, Space Saver	1	1
16			Axle, Quick Release (7/16" x 4-1/2")	1	1
17			NLA - Label, Warning	1	1
NOTE: A - For Space Saver Arms Chairs. B - For Conventional Arm Chairs. C - Kit includes items 8, 9, and 10. KIT is No Longer Available. Must order the components separately along with 1047943. D - Kit includes items 12, 13, and 14. E - Kit includes items 2 and 3. F - Kit includes items 16 and 17.					

Handrims and Attaching Hardware

As of 8/11/15 Chrome Handrims were replaced by Alum. Clear Anodized and Alum. Black 34P was replaced by Alum. Black Anodized Handrims.

Aluminum Handrims - Black 34P Finish

Wheel Style	Handrim Only	Attaching Points	Attaching Hardware Kit
1763BK / 4763BK / U1763	1020017	7	1025153 See Note(s) A
1764BK / 4763BK 4764BK / U1764	1020018	7	1025153 See Note(s) A

NOTE: A - Kit contains attaching hardware for (1) handrim.

As of 8/11/15 Chrome Handrims were replaced by Alum. Clear Anodized and Alum. Black 34P was replaced by Alum. Black Anodized Handrims.

Aluminum Handrims - Black Anodized Finish

Wheel Style	Handrim Only	Attaching Points	Attaching Hardware Kit
1763BK / 4763BK / U1763	1186217	7	1025153 See Note(s) A
1764BK / 4763BK 4764BK / U1764	1186218	7	1025153 See Note(s) A

NOTE: A - Kit contains attaching hardware for (1) handrim.

Steel Handrims - Chrome Plated

Wheel Style	Handrim Only	Attaching Points	Attaching Hardware Kit
1763BK / 4763BK / U1763BK	1019999	7	1025153 See Note(s) A
1764BK / 4764BK / U1764BK	1020000	7	1025153 See Note(s) A

NOTE: A - Kit contains attaching hardware for (1) handrim.

As of 8/11/15 Chrome Handrims were replaced by Alum. Clear Anodized and Alum. Black 34P was replaced by Alum. Black Anodized Handrims.


Aluminum Handrims - Clear Anodized

Wheel Style	Handrim Only	Attaching Points	Attaching Hardware Kit
1763BK / 4763BK / U1763	1183683	7	1025153 See Note(s) A
1764BK / 4763BK 4764BK / U1764	1183682	7	1025153 See Note(s) A

NOTE: A - Kit contains attaching hardware for (1) handrim.

Link-Pivot Style Wheel Locks

Conventional Arm (Before 8/14/03)

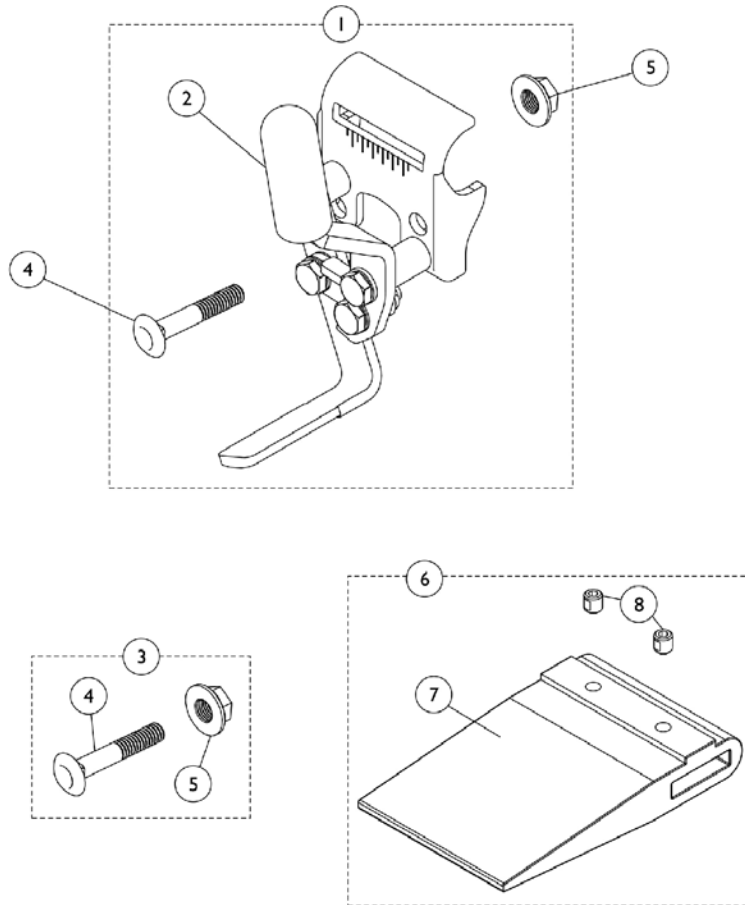


Link-Pivot Style Wheel Locks Conventional Arm (Before 8/14/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020001-NLA	NLA - Wheel Lock Assembly, Chrome - Right	1	1
1		1020002-NLA	NLA - Wheel Lock Assembly, Chrome - Left	1	1
1		1031782	Wheel Lock Assembly, Push to Lock, Adult - Right	1	1
1		1031783	Wheel Lock Assembly, Push to Lock, Adult - Left	1	1
1		1016846	Wheel Lock Assembly w/o Attaching Hardware, Chrome - Right	1	1
1		1016847	Wheel Lock Assembly w/o Attaching Hardware, Chrome - Left	1	1
1		1031796	Wheel Lock Assembly, Pull to Lock, Hemi - Right	1	1
1		1031797	Wheel Lock Assembly, Pull to Lock, Hemi - Left	1	1
2		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
3	A	1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
4			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
5			Nut, Hex Flang (1/4-20)	1	2
6	B,D	1037421	Kit, Wheel Lock Shoe Spacer, 20" Rear Wheels (Pair)	1	1
			Spacer, Wheel Lock Shoe (2-63/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
6	C,D	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	1	1
			Spacer, Wheel Lock Shoe (1-45/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
NOTE: A - Includes items 4 and 5. B - Used with 20" rear wheels on recliner chairs. C - Used with 20" rear wheels with 1" forward rear wheel positioner or 22" rear wheels on a 14" deep chair. D - Includes items 7 and 8.					

Link-Pivot Style Wheel Locks

Space Saver Arm (Before 8/14/03)



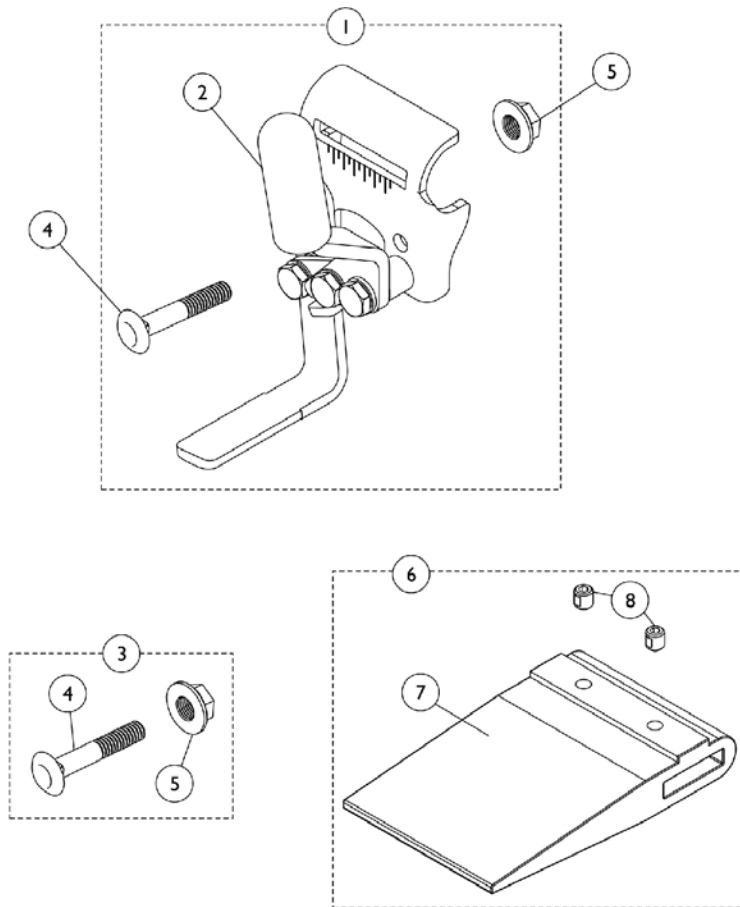


Link-Pivot Style Wheel Locks Space Saver Arm (Before 8/14/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1045861-NLA	NLA - Wheel Lock Assembly, Adult, Push to Lock, Black - Right	1	1
1	E	1045862-NLA	NLA - Wheel Lock Assembly, Adult, Push to Lock, Black - Left	1	1
1		1023493	Wheel Lock Assembly, Push to Lock - Right	1	1
1		1023494	Wheel Lock Assembly, Push / Lock - Left	1	1
1		1031792	Wheel Lock Assembly, Push to Lock, Adult - Right	1	1
1		1031793	Wheel Lock Assembly, Push to Lock, Adult - Left	1	1
1		1045857	Wheel Lock Assembly, Pull to Lock, Black - Right	1	1
1		1045858	Wheel Lock Assembly, Pull to Lock, Black - Left	1	1
1		1031794	Wheel Lock Assembly, Pull to Lock, Hemi - Right	1	1
1		1031795	Wheel Lock Assembly, Pull to Lock, Hemi - Left	1	1
2		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
3	A	1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
4			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
5			Nut, Hex Flang (1/4-20)	1	2
6	B,D	1037421	Kit, Wheel Lock Shoe Spacer, 20" Rear Wheels (Pair)	1	1
			Spacer, Wheel Lock Shoe (2-63/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
6	C,D	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	1	1
			Spacer, Wheel Lock Shoe (1-45/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
NOTE: A - Includes items 4 and 5. B - Used on 20" rear wheel for recliner chairs. C - Includes items 7 and 8. D - Used on 20" rear wheel with 1" forward rear wheel post or 22" rear wheel with 14" deep chair. E - No Longer Available. Use part #'s 1023493 and 1023494 as replacements.					

Disk Style Wheel Locks

Conventional & Space Saver Arm (After 8/14/03)

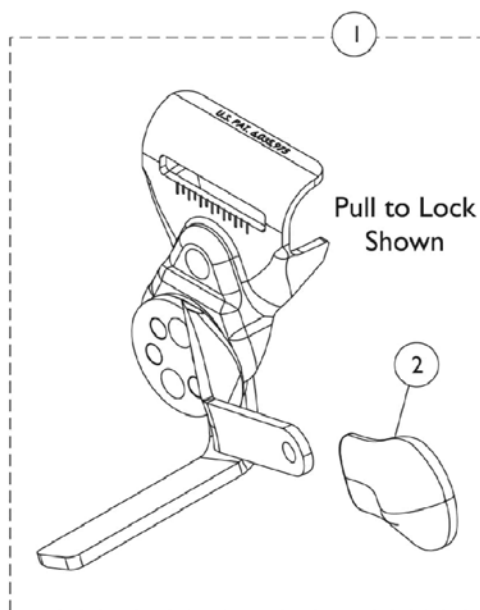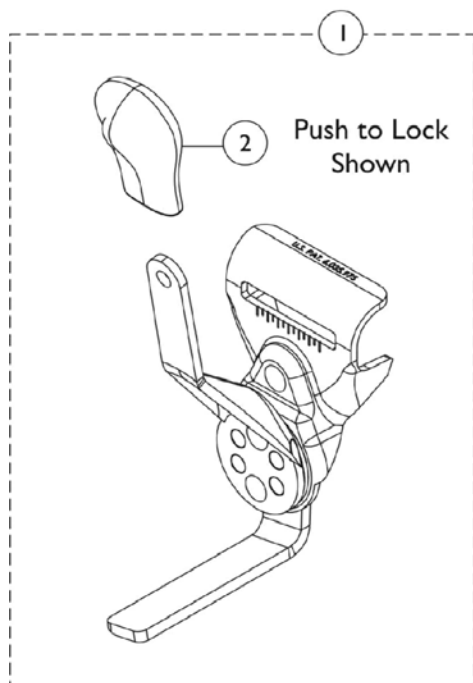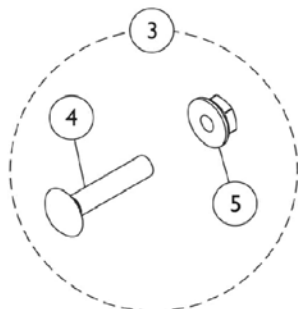



Disk Style Wheel Locks

Conventional & Space Saver Arm (After 8/14/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,C1,C2	1110227-NLA	NLA - Wheel Lock Assembly, Disk Pull to Lock, Black - Right	1	1
1	C,C1,C2	1110228-NLA	NLA - Wheel Lock Assembly, Disk Pull to Lock, Black - Left	1	1
1	C,C1,C2	1110225-NLA	NLA - Wheel Lock Assembly, Disk Push to Lock, Black - Right	1	1
1	C,C1,C2	1110226-NLA	NLA - Wheel Lock Assembly, Disk Push to Lock, Black - Left	1	1
2		1110160-NLA	NLA - Tip, Plastic Wheel Lock Handle, Black	1	1
3	A	1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
4			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
5			Nut, Hex Flang (1/4-20)	1	2
	B	1058900	Wheel Lock Extension Handle Assembly, Black (8")	1	1
	B,D	1101452	Wheel Lock Assembly, Push to Lock, Black - Right	1	1
	B,D	1101453	Wheel Lock Assembly, Push to Lock, Black - Left	1	1
	B,D	1101454	Wheel Lock Assembly, Pull to Lock, Black - Right	1	1
	B,D	1101455	Wheel Lock Assembly, Pull to Lock, Black - Left	1	1
	B,E	1111924	Wheel Lock Assembly, Push to Lock, Black - Right	1	1
	B,E	1111925	Wheel Lock Assembly, Push to Lock, Black - Left	1	1
	B,E	1111926	Wheel Lock Assembly, Pull to Lock, Black - Right	1	1
	B,E	1111927	Wheel Lock Assembly, Pull to Lock, Black - Left	1	1
NOTE: A - Includes items 3 and 4. B - Not shown. C - Disk Style Wheel Locks are no longer available as a Service Part. Must convert to Link-Pivot Style Wheel Locks listed. Order Right/Left side to match. For Conventional Arms use part #'s 1101452/1101453 (Push to Lock) or 1101454/1101455 (Pull to Lock). C1 - For Space Saver Arms use part #'s 1111924/1111925 (Push to Lock) or 1111926/1111927 (Pull to Lock). Can use existing attaching hardware. C2 - Additional Wheel Lock information can be found under categories titled "Wheel Locks - Link-Pivot Style - Conventional Arm, Builder Chairs (After 4/15/07)" or "Wheel Locks - Link-Pivot Style - Space Saver Arm, Builder Chairs (After 4/15/07)". D - Use for Conventional Arms. E - Use for Space Saver Arms.					

Link-Pivot Style Wheel Locks

Conventional Arm, Builder Chairs (After 4/15/07)

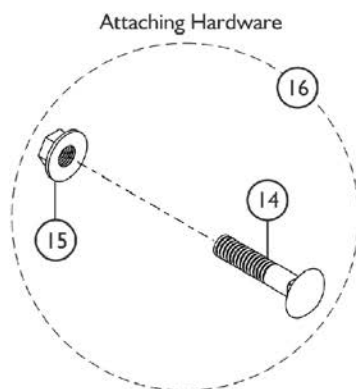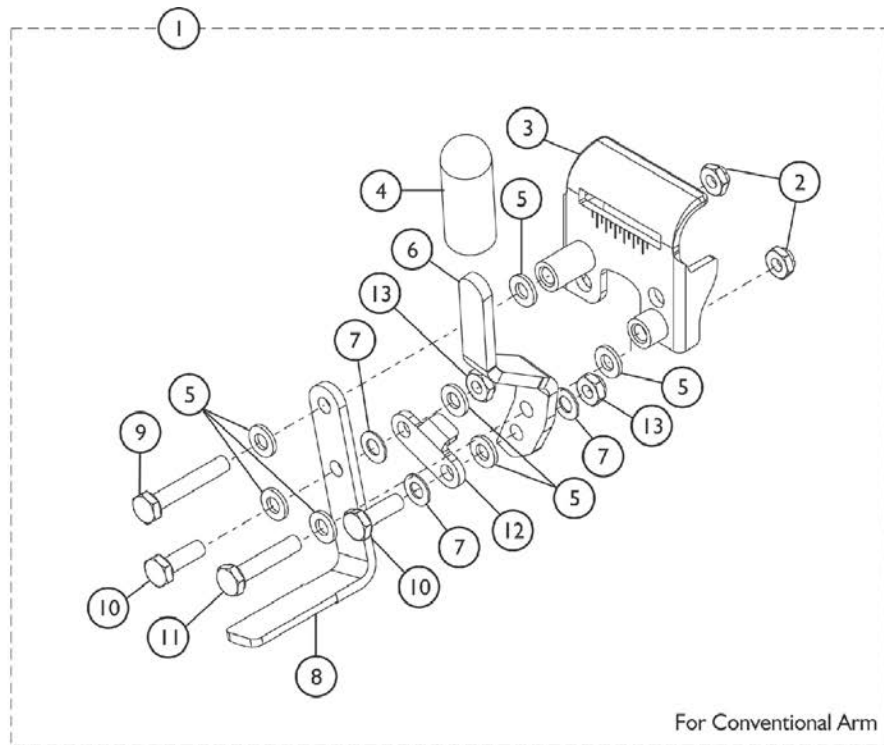


Link-Pivot Style Wheel Locks

Conventional Arm, Builder Chairs (After 4/15/07)


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1101452	Wheel Lock Assembly, Push to Lock, Black - Right	1	1
1	A	1101453	Wheel Lock Assembly, Push to Lock, Black - Left	1	1
1	A	1101454	Wheel Lock Assembly, Pull to Lock, Black - Right	1	1
1	A	1101455	Wheel Lock Assembly, Pull to Lock, Black - Left	1	1
2		1025195	Locknut (1/4-20)	1	1
3		1053284	Bracket, Mounting, Black - Right	1	1
3		1053285	Bracket, Mounting, Black - Left	1	1
4		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
5		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	7
6		1102134	Handle, Push/ Lock, Black - Right	1	1
6		1102135	Handle, Push/ Lock, Black - Left	1	1
7		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	3
8		1102133	Shoe, Wheel Lock	1	1
9		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
10		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	2
11		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
12		1102132-NLA	NLA - Link, Black	1	1
13		1025765	Locknut (1/4-28)	1	1
14		1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	2
15		1021403	Nut, Hex Flang (1/4-20)	1	1
16	B	1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
			Nut, Hex Flang (1/4-20)	1	2
	D	00078X021	Instruction Sheet, Wheel Lock	1	1
	C,D	1058899	Wheel Lock Extension Handle Assembly, Black (6")	2	1
NOTE: A - Includes Items 2 - 13. B - Includes Items 14 - 15. C - Includes: Extension Handle and Rubber Tip D - Not shown.					

Link-Pivot Style Wheel Locks

Space Saver Arm, Builder Chairs (After 4/15/07)

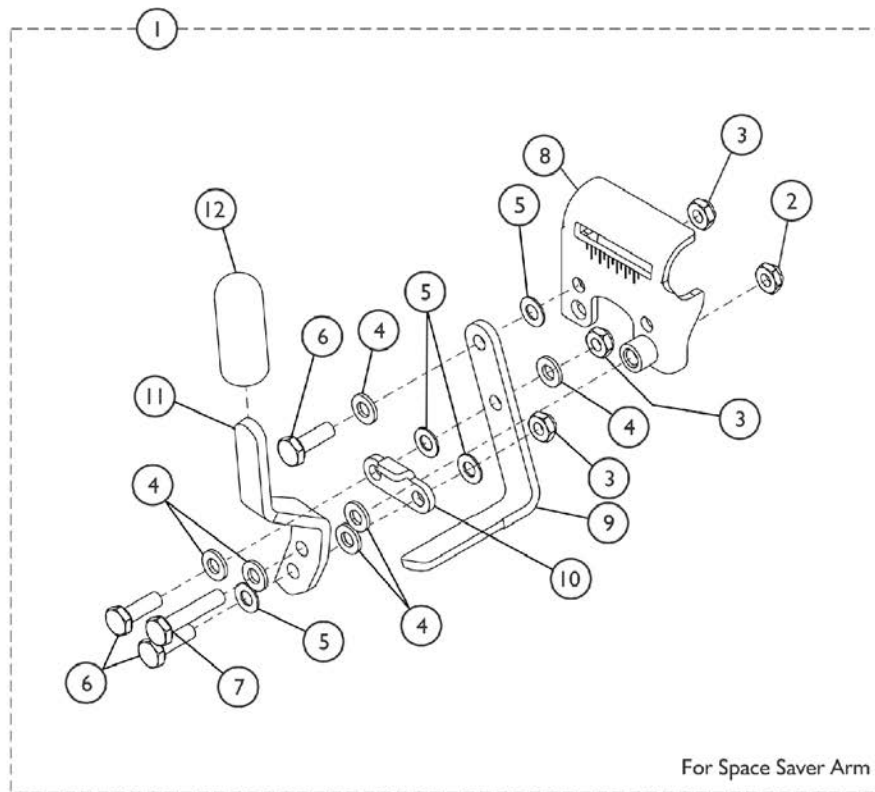

SSM-041A

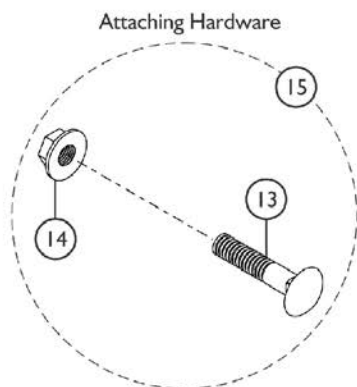


Link-Pivot Style Wheel Locks

Space Saver Arm, Builder Chairs (After 4/15/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1111924	Wheel Lock Assembly, Push to Lock, Black - Right	1	1
1	A	1111925	Wheel Lock Assembly, Push to Lock, Black - Left	1	1
1	A	1111926	Wheel Lock Assembly, Pull to Lock, Black - Right	1	1
1	A	1111927	Wheel Lock Assembly, Pull to Lock, Black - Left	1	1
2		1025195	Locknut (1/4-20)	1	1
3		1025765	Locknut (1/4-28)	1	3
4		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	6
5		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	4
6		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	3
7		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
9		1102133	Shoe, Wheel Lock	1	1
10		1102132	NLA - Link, Black	1	1
11		1102133	Shoe, Wheel Lock	1	1
12		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
13		1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
14		1021403	Nut, Hex Flang (1/4-20)	1	1
15	B	1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
			Nut, Hex Flang (1/4-20)	1	2
	D	00078X021	Instruction Sheet, Wheel Lock	1	1
	C,D	1058899	Wheel Lock Extension Handle Assembly, Black (6")	2	1
NOTE: A - Includes Items 2 - 12. B - Includes Items 13 and 14. C - Includes: Extension Handle and Rubber Tip D - Not shown.					

Footplate with Hardware Assembly - J932A/J933A

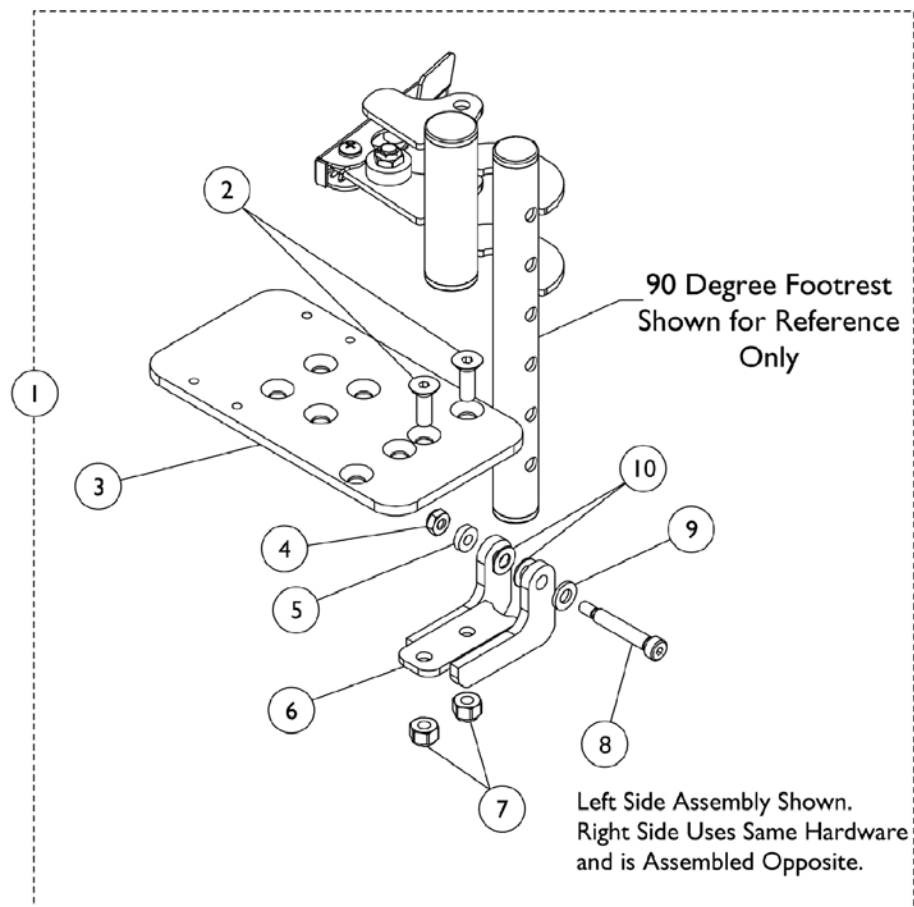


Footplate with Hardware Assembly - J932A/J933A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1102493	Footplate Assembly with Hardware, Black (10" W)	1	1
1	A,B	1102494	Footplate Assembly with Hardware, Black (12-14" W)	1	1
2		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1100988	Footplate, Black (3-1/4" W x 6" L) (10" W)	1	1
3		1100989	Footplate, Black (4-1/4" W x 6-1/2" L) (12" W)	1	1
4		1026035-NLA	NLA - Locknut (#10-24)	1	1
5		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
6		1101020	Block, Hinge	1	1
7		1025736	Locknut (1/4-20)	1	2
8		1000212	Screw, Shoulder Hex Socket (1/4 x 1-1/4", #10-24)	1	1
9		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	1
10		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	2

NOTE: A - Left side assembly shown. Right side uses same hardware and is assembled opposite.
 B - Includes items 2-10.


Footrest Support Assembly J932A

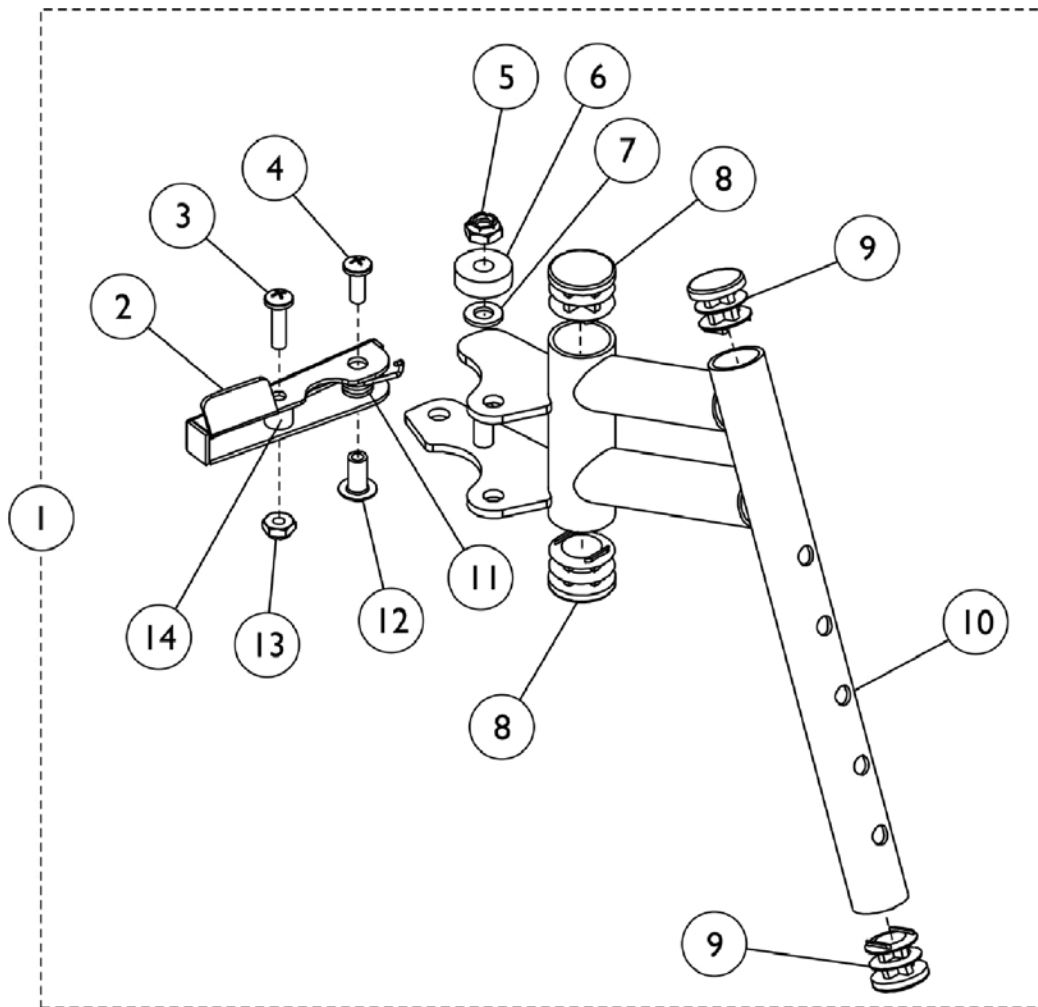

Footrest Support Assembly J932A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881101008	Footrest Support Assembly, Color - Right	1	1
1	A	8881101009	Footrest Support Assembly, Color - Left	1	1
1	B	1101010-NLA	NLA - Footrest Support Assembly, Chrome - Right	1	1
1	B	1101011-NLA	NLA - Footrest Support Assembly, Chrome - Left	1	1
2		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
2		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1025195	Locknut (1/4-20)	1	1
6		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
8		1027095	Plug Button, Black (7/8")	1	2
9		1027097	Plug Button, Black (3/4")	1	2
10		1100629-NLA	NLA - Tube, Footrest Support, 70 Degree, Chrome - Right	1	1
10		1100630-NLA	NLA - Tube, Footrest Support, 70 Degree, Chrome - Left	1	1
10	A	8881100625	Tube, Footrest Support, 70 Degree, Color - Right	1	1
10	A	8881100626	Tube, Footrest Support, 70 Degree, Color - Left	1	1
11		1028707	Spring, Release - Right	1	1
11		1028708	Spring, Release - Left	1	1
12		1028846-NLA	NLA - Rivnut (#10-32)	1	1
13		1025735	Locknut (#10-32)	1	1
14		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging." B - Chrome finish.					

Footrest Support Assembly J932A-14

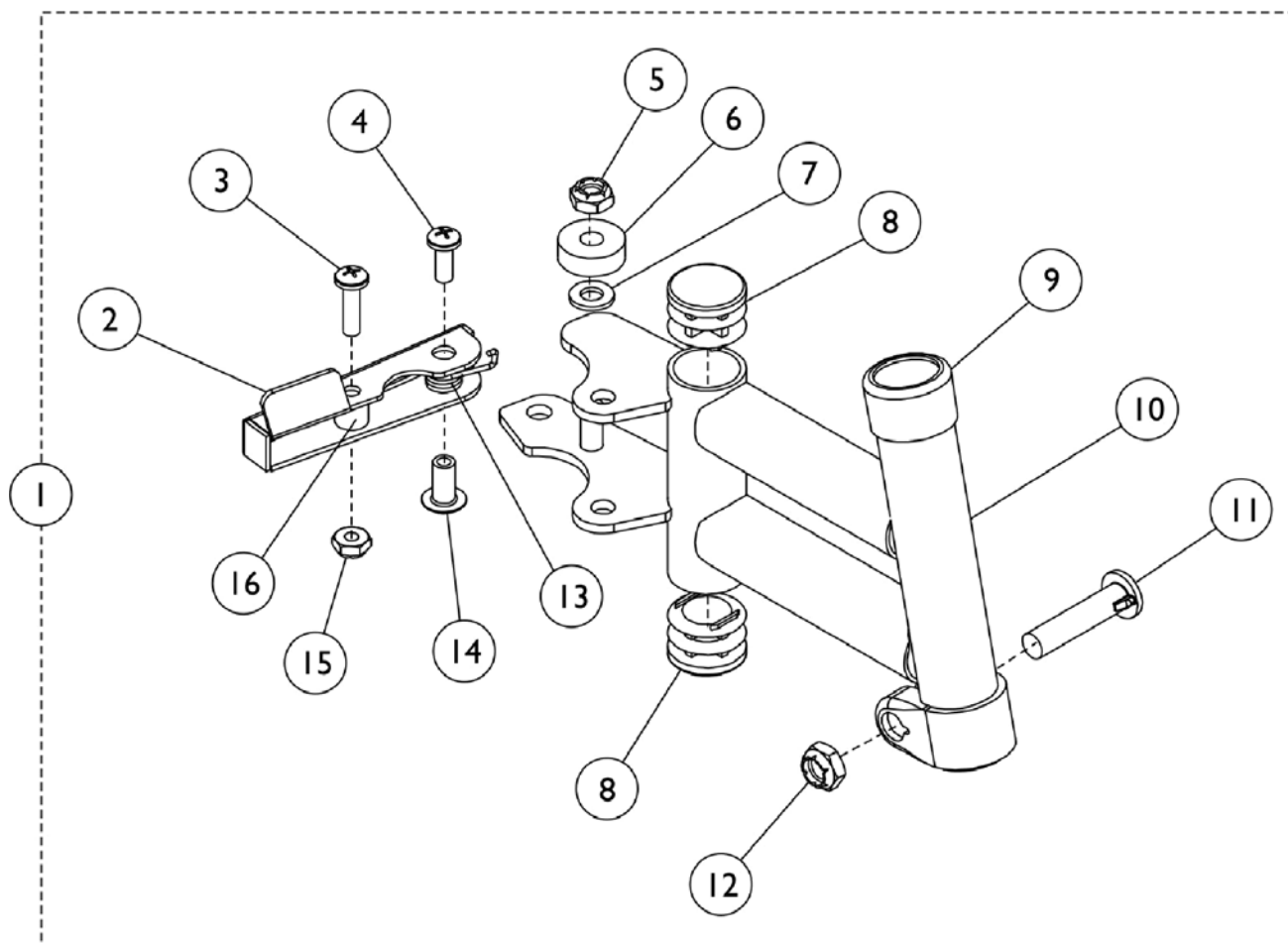


Footrest Support Assembly J932A-14

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881101006	Footrest Support Assembly, Color - Right	1	1
1	A,B	8881101007	Footrest Support Assembly, Color - Left	1	1
1	B	1101004-NLA	NLA - Footrest Support Assembly, Chrome - Right	1	1
1	B	1101005-NLA	NLA - Footrest Support Assembly, Chrome - Left	1	1
1	B	1133679	Footrest Support Assembly, Black - Right	1	1
1	B	1133680	Footrest Support Assembly, Black - Left	1	1
2		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
2		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1025195	Locknut (1/4-20)	1	1
6		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
8		1027095	Plug Button, Black (7/8")	1	2
9		15680X012	Collar, Anti-Rattle (7/8")	1	1
10		1101000-NLA	NLA - Tube, Footrest Support, Chrome - Right	1	1
10		1101001-NLA	NLA - Tube, Footrest Support, Chrome - Left	1	1
10	A	8881101002	Tube, Footrest Support, Color - Right	1	1
10	A	8881101003	Tube, Footrest Support, Color - Left	1	1
10		1133677	Tube, Footrest Support, Black - Right	1	1
10		1133678	Tube, Footrest Support, Black - Left	1	1
11		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
12		1025730	Locknut (5/16-24)	1	1
12		1107321	Nut, Low Cap (5/16"-24)	1	1
13		1028707	Spring, Release - Right	1	1
13		1028708	Spring, Release - Left	1	1
14		1028846-NLA	NLA - Rivnut (#10-32)	1	1
15		1025735	Locknut (#10-32)	1	1
16		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1

NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."
B - Includes items 2-16.

Footrest Support Assembly - J933A

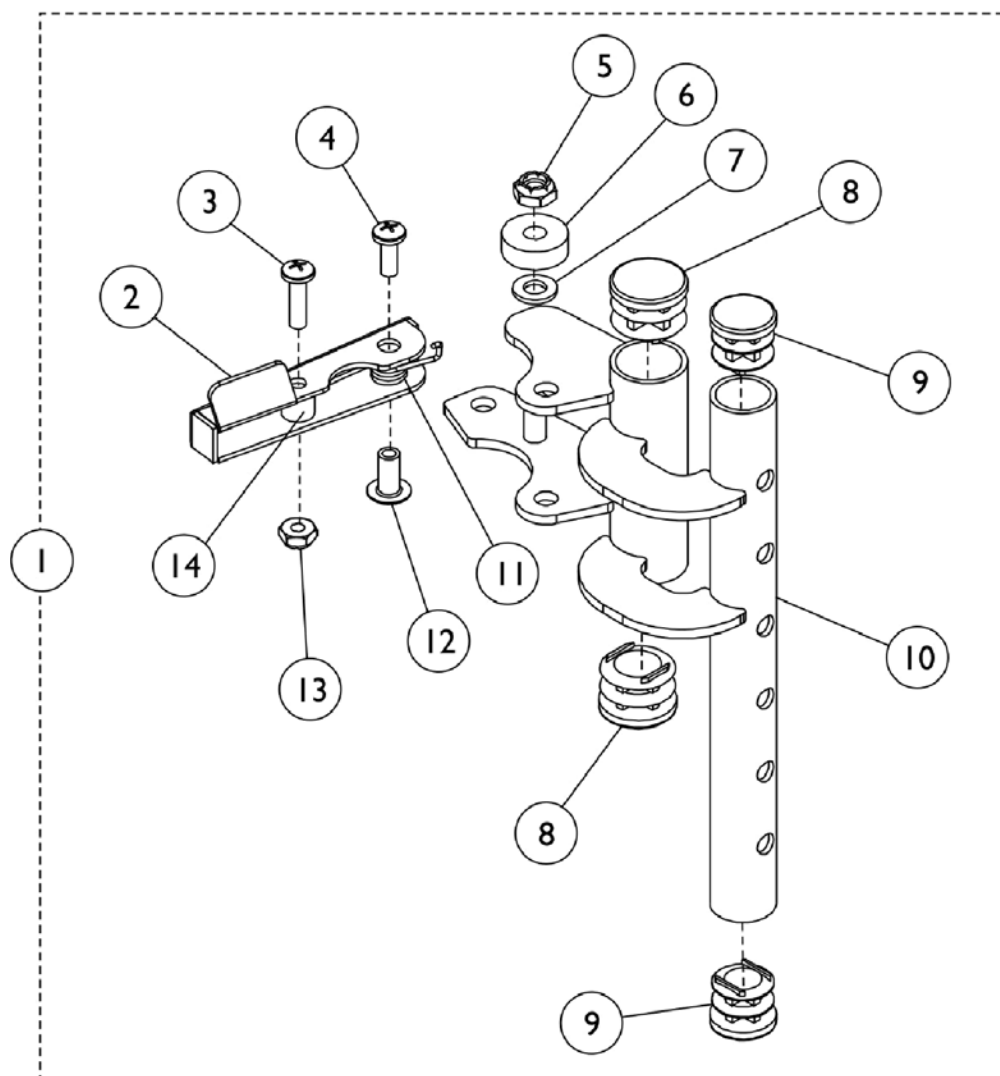


Footrest Support Assembly - J933A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881101012	Footrest Support Assembly, Color - Right	1	1
1	A,B	8881101013	Footrest Support Assembly, Color - Left	1	1
1	B	1101014	Footrest Support Assembly, Chrome - Right	1	1
1	B	1101015	Footrest Support Assembly, Chrome - Left	1	1
2		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
2		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1025195	Locknut (1/4-20)	1	1
6		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
8		1027095	Plug Button, Black (7/8")	1	2
9		1027097	Plug Button, Black (3/4")	1	2
10		1100631	Tube, Footrest Support, 90 Degree, Chrome - Right	1	1
10		1100632	Tube, Footrest Support, 90 Degree, Chrome - Left	1	1
10	A	8881100631	Tube, Footrest Support, 90 Degree, Color - Right	1	1
10	A	8881100632	Tube, Footrest Support, 90 Degree, Color - Right	1	1
11		1028707	Spring, Release - Right	1	1
11		1028708	Spring, Release - Left	1	1
12		1028846-NLA	NLA - Rivnut (#10-32)	1	1
13		1025735	Locknut (#10-32)	1	1
14		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging." B - Includes items 2-14.					

Legrest Support Assembly J934A - Chrome

NOTE: Front Riggings with model J934A are for 10-12" deep chairs.

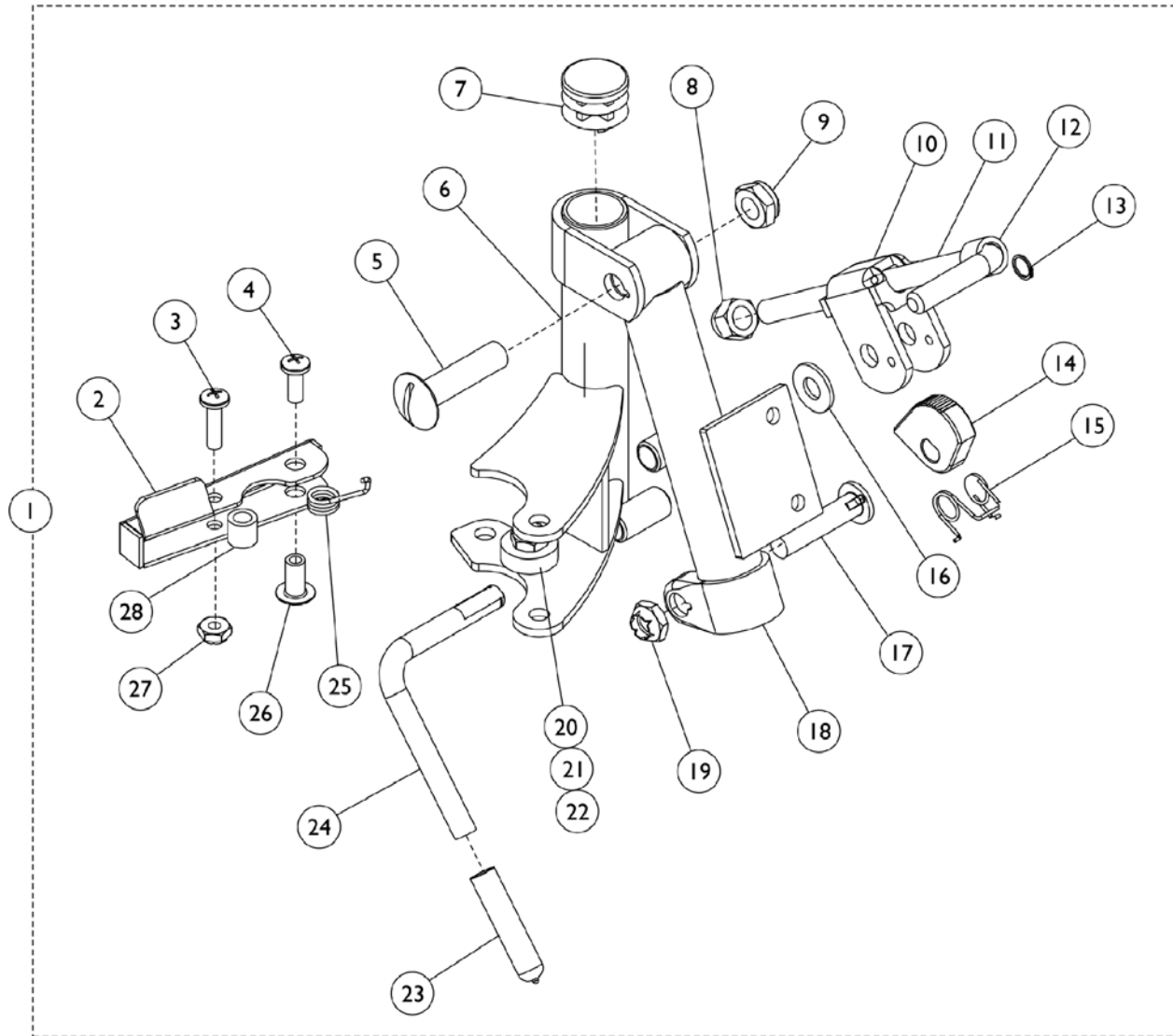

Legrest Support Assembly J934A - Chrome


NOTE: Front Riggings with model J934A are for 10-12" deep chairs.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1100658-NLA	NLA - Legrest Support Assembly, Chrome - Right	1	1
1	A	1100659-NLA	NLA - Legrest Support Assembly, Chrome - Left	1	1
2		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
2		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
3		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
4		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
5		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
6		1100652-NLA	NLA - Tube, Legrest Support, Hemi, Chrome - Right	1	1
6		1100653-NLA	NLA - Tube, Legrest Support, Hemi, Chrome - Left	1	1
7		1027095	Plug Button, Black (7/8")	1	1
8		10261X000	Nut, Acorn (5/16-24)	1	1
9		40363X000	Hex Nut (3/8"-16)	1	1
10		04528X002	Bracket, Release, Black	1	1
11		42326X001	Bar, Ratchet	1	1
12		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
13		1040831	Ring, Snap	1	1
14		1037125	Cam, Legrest	1	1
15		1037485	Spring, Release	1	1
16		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
17		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
18		1100644-NLA	NLA - Support, Elevating Legrest, Chrome - Right	1	1
18		1100645-NLA	NLA - Support, Elevating Legrest, Chrome - Left	1	1
19		1107321	Nut, Low Cap (5/16"-24)	1	1
20		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
21		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
22		1025195	Locknut (1/4-20)	1	1
23		1040060	Tip, Plastic, Black	1	1
24		1101031	Lever, Release - Right	1	1
24		1101032	Lever, Release - Right	1	1
25		1028707	Spring, Release - Right	1	1
25		1028708	Spring, Release - Left	1	1
26		1028846-NLA	NLA - Rivnut (#10-32)	1	1
27		1025735	Locknut (#10-32)	1	1
28		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1

NOTE: A - Includes items 2-28.

Legrest Support Assembly J934A - Black

NOTE: Front Riggings with model J934A are for 10-12" deep chairs.

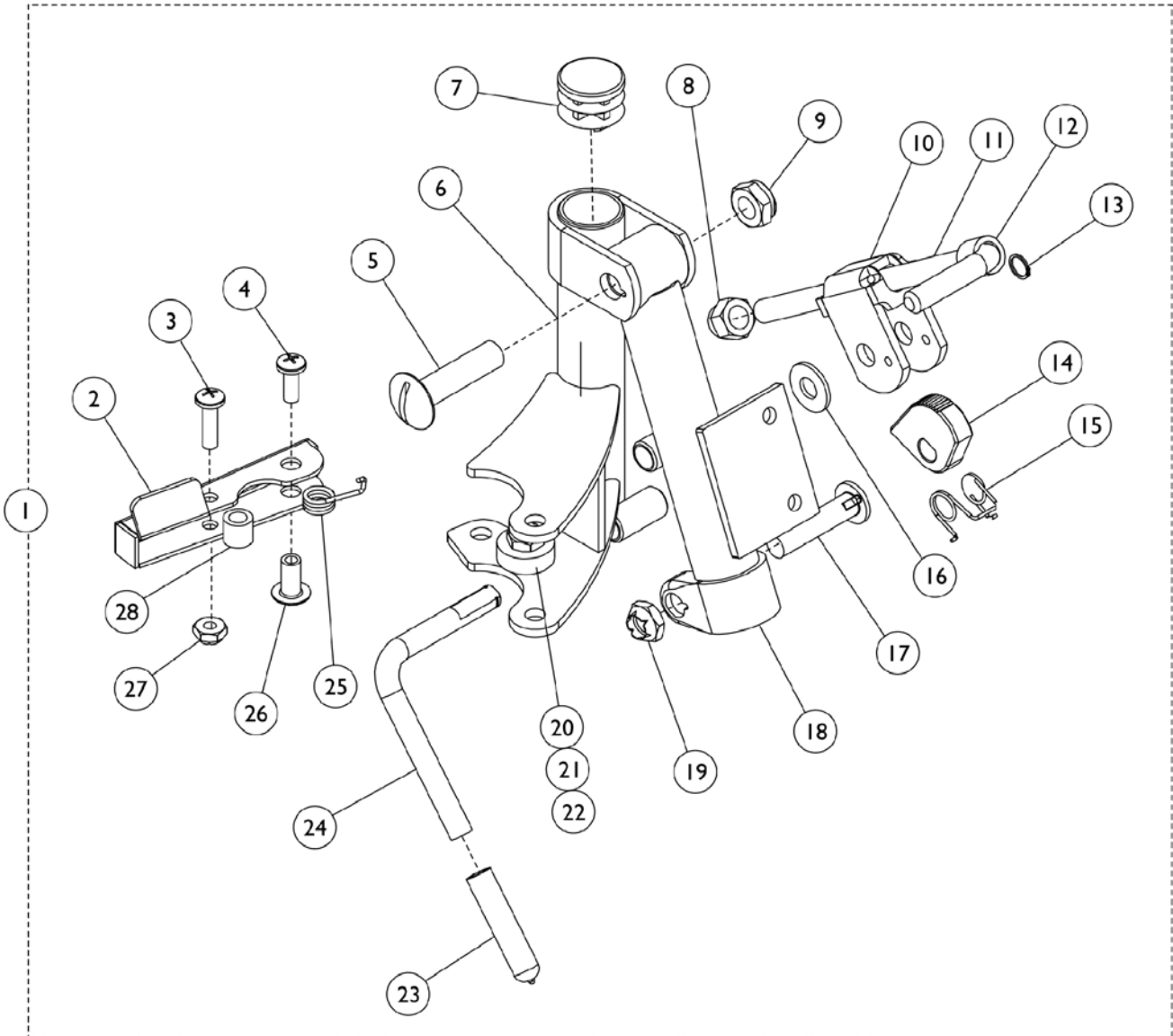


Legrest Support Assembly J934A - Black

NOTE: Front Riggings with model J934A are for 10-12" deep chairs.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1133654	Legrest Support Assembly, Black - Right	1	1
1	A	1133655	Legrest Support Assembly, Black - Left	1	1
2		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
2		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
3		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
4		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
5		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
6		1133650	Tube, Legrest Support, Black - Right	1	1
6		1133651	Tube, Legrest Support, Black - Left	1	1
7		1027095	Plug Button, Black (7/8")	1	1
8		10261X000	Nut, Acorn (5/16-24)	1	1
9		40363X000	Hex Nut (3/8"-16)	1	1
10		04528X002	Bracket, Release, Black	1	1
11		42326X001	Bar, Ratchet	1	1
12		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
13		1040831	Ring, Snap	1	1
14		1037125	Cam, Legrest	1	1
15		1037485	Spring, Release	1	1
16		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
17		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
18		1133652	Tube, Elevating Support, Black - Right	1	1
18		1133653	Tube, Elevating Support, Black - Left	1	1
19		1107321	Nut, Low Cap (5/16"-24)	1	1
20		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
21		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
22		1025195	Locknut (1/4-20)	1	1
23		1040060	Tip, Plastic, Black	1	1
24		1101031	Lever, Release - Right	1	1
24		1101032	Lever, Release - Right	1	1
25		1028707	Spring, Release - Right	1	1
25		1028708	Spring, Release - Left	1	1
26		1028846-NLA	NLA - Rivnut (#10-32)	1	1
27		1025735	Locknut (#10-32)	1	1
28		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1

NOTE: A - Includes items 2-28.


Legrest Support Assembly J934A-14 - Chrome

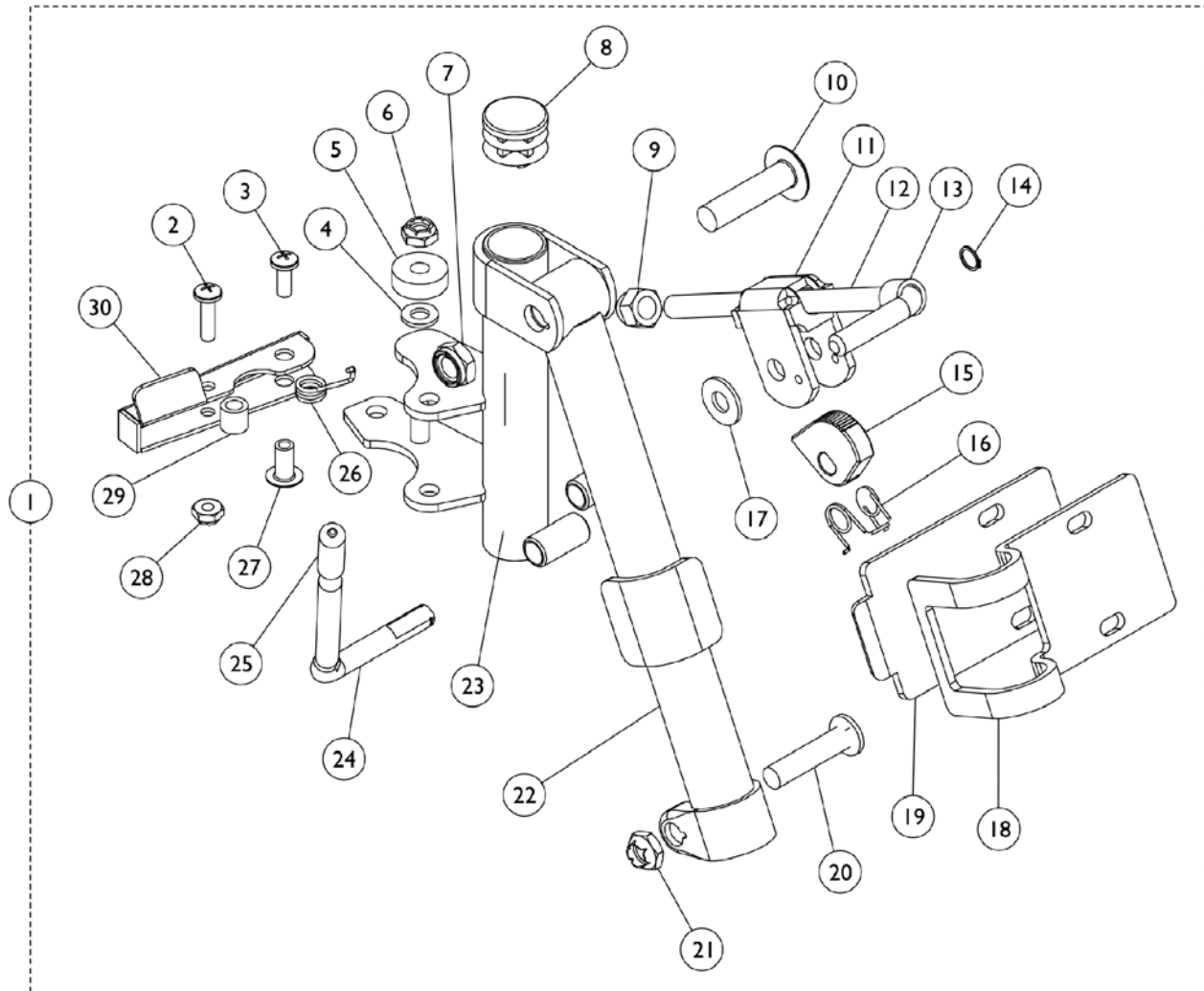

Legrest Support Assembly J934A-14 - Chrome

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1100996	Legrest Support Assembly, Black- Right	1	1
1	A	1100997	Legrest Support Assembly, Black- Left	1	1
2		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
5		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
6		1025195	Locknut (1/4-20)	1	1
7		40363X000	Hex Nut (3/8"-16)	1	1
8		1027095	Plug Button, Black (7/8")	1	1
9		10261X000	Nut, Acorn (5/16-24)	1	1
10		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
11		04528X002	Bracket, Release, Black	1	1
12		42326X001	Bar, Ratchet	1	1
13		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
14		1040831	Ring, Snap	1	1
15		1037125	Cam, Legrest	1	1
16		1037485	Spring, Release	1	1
17		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
18		47352X004	Clamp, Black	1	1
19		47352X003	Lock, Pad Plate, Legrest, Black	1	1
20		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
21		1107321	Nut, Low Cap (5/16"-24)	1	1
22		04526X002-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Right	1	1
22		04526X003-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Left	1	1
23		1131381	Support, Elevating Legrest, Black - Right	1	1
23		1131382	Support, Elevating Legrest, Black - Left	1	1
24		1037126	Handle, Release	1	1
25		1040060	Tip, Plastic, Black	1	1
26		1028707	Spring, Release - Right	1	1
26		1028708	Spring, Release - Left	1	1
27		1028846-NLA	NLA - Rivnut (#10-32)	1	1
28		1025735	Locknut (#10-32)	1	1
29		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
30	B	1031469-NLA	NLA - Handle, Release, Black - Right	1	1
30	C	1031470-NLA	NLA - Handle, Release, Black - Left	1	1
NOTE: A - Includes items 2-30. B - For use on left elevating legrest. C - For use on right elevating legrest.					
NOTE: Front Riggings with model J934-14 are for 14" deep chairs only.					


Legrest Support Assembly J934A-14 - Black

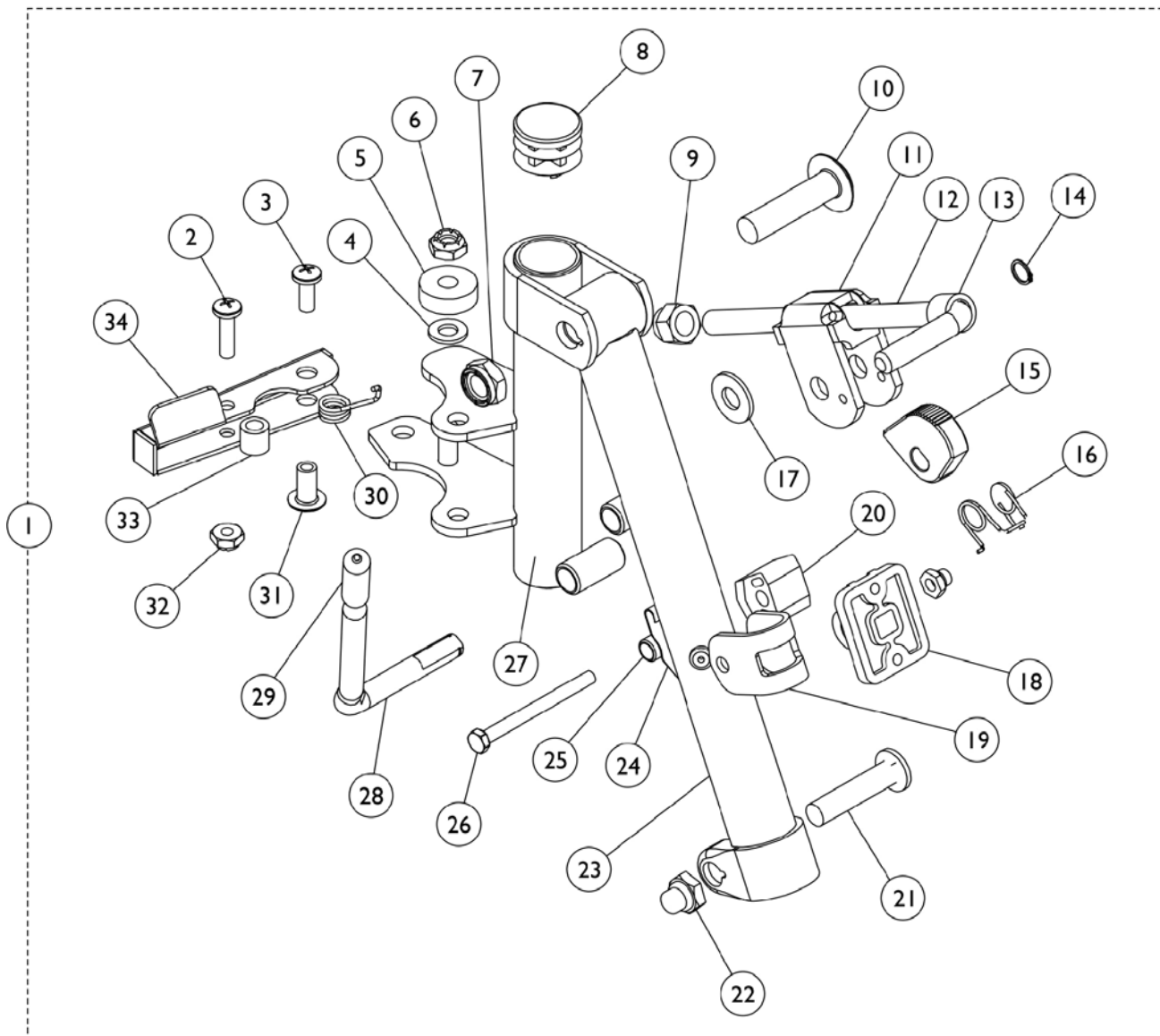

Legrest Support Assembly J934A-14 - Black

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1131383	Legrest Support Assembly, Black - Right	1	1
1	A	1131384	Legrest Support Assembly, Black - Left	1	1
2		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
5		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
6		1025195	Locknut (1/4-20)	1	1
7		40363X000	Hex Nut (3/8"-16)	1	1
8		1027095	Plug Button, Black (7/8")	1	1
9		10261X000	Nut, Acorn (5/16-24)	1	1
10		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
11		04528X002	Bracket, Release, Black	1	1
12		42326X001	Bar, Ratchet	1	1
13		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
14		1040831	Ring, Snap	1	1
15		1037125	Cam, Legrest	1	1
16		1037485	Spring, Release	1	1
17		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
18		04526X019	Plate, Legrest Pad	1	1
19		1110207	Bracket, Calf Pad Mounting, Black	1	1
20		1025133	Spacer, Legrest Pad, Black	1	1
21		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
22		1107321	Nut, Low Cap (5/16"-24)	1	1
23		1131379	Tube, Elevating Support, Black - Right	1	1
23		1131380	Tube, Elevating Support, Black - Left	1	1
24		48823X008	Spring, Leaf	1	1
25		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
26		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
27		1131381	Support, Elevating Legrest, Black - Right	1	1
27		1131382	Support, Elevating Legrest, Black - Left	1	1
28		1037126	Handle, Release	1	1
29		1040060	Tip, Plastic, Black	1	1
30		1028707	Spring, Release - Right	1	1
30		1028708	Spring, Release - Left	1	1
31		1028846-NLA	NLA - Rivnut (#10-32)	1	1
32		1025735	Locknut (#10-32)	1	1
33		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
34	B	1031469-NLA	NLA - Handle, Release, Black - Right	1	1
34	C	1031470-NLA	NLA - Handle, Release, Black - Left	1	1
NOTE: A - Includes items 2-30. B - For use on left elevating legrest. C - For use on right elevating legrest.					


Pivot Slide Tube with Footplate Assembly - J932A-14

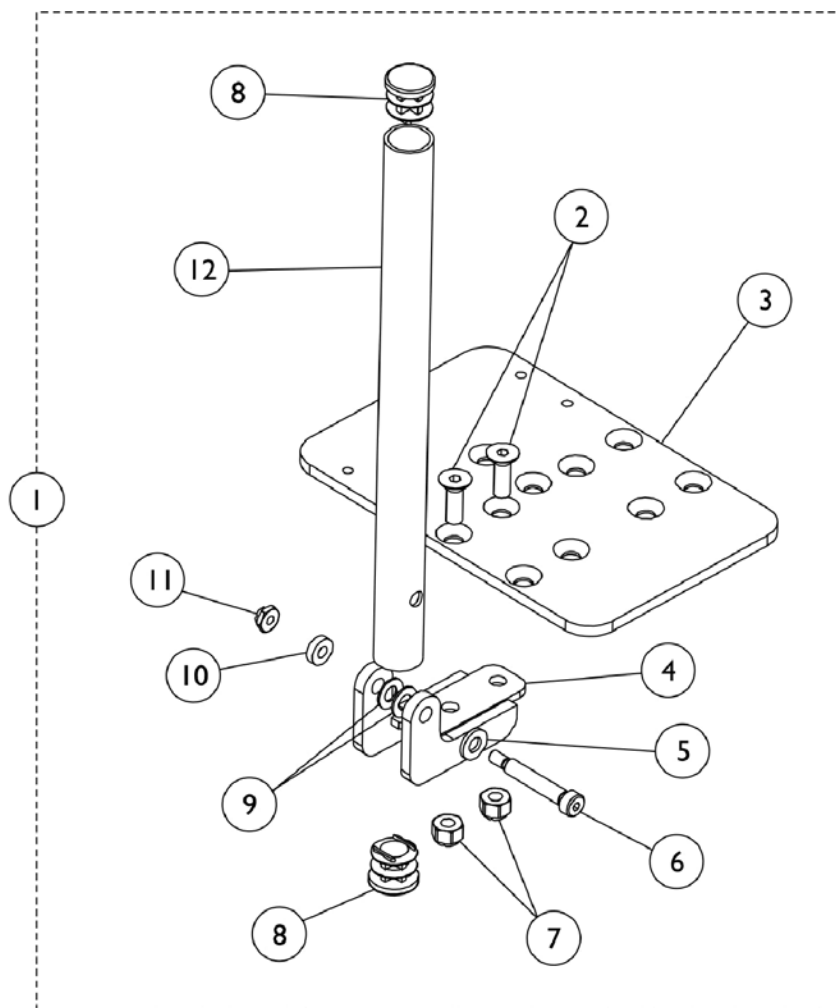

Pivot Slide Tube with Footplate Assembly - J932A-14

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,C	1102491	Pivot/Slide Tube w/ Footplate Assembly, Black - Right (12" Footplate)	1	1
1	A,B,C	1102492	Pivot/Slide Tube w/ Footplate Assembly, Black - Left (12" Footplate)	1	1
1	B,D	1102489-NLA	NLA - Pivot/Slide Tube w/ Footplate Assembly, Chrome - Right (12" Footplate)	1	1
1	B,D	1102490-NLA	NLA - Pivot/Slide Tube w/ Footplate Assembly, Chrome - Left (12" Footplate)	1	1
2		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1100988	Footplate, Black (3-1/4" W x 6" L) (10" W)	1	1
3		1100989	Footplate, Black (4-1/4" W x 6-1/2" L) (12" W)	1	1
4		1101020	Block, Hinge	1	1
5		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	1
6		1000212	Screw, Shoulder Hex Socket (1/4 x 1-1/4", #10-24)	1	1
7		1025736	Locknut (1/4-20)	1	2
8		1027097	Plug Button, Black (3/4")	1	2
9		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	2
10		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
11		1026035-NLA	NLA - Locknut (#10-24)	1	1
12	D	1102483-NLA	NLA - Tube, Slide, Chrome	1	1
12	A	1102484	Tube, Slide, Black	1	1
NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging." B - Includes items 2-12. C - Black D - Chrome					

Pivot Slide Tube with Footplate Assembly - J934A

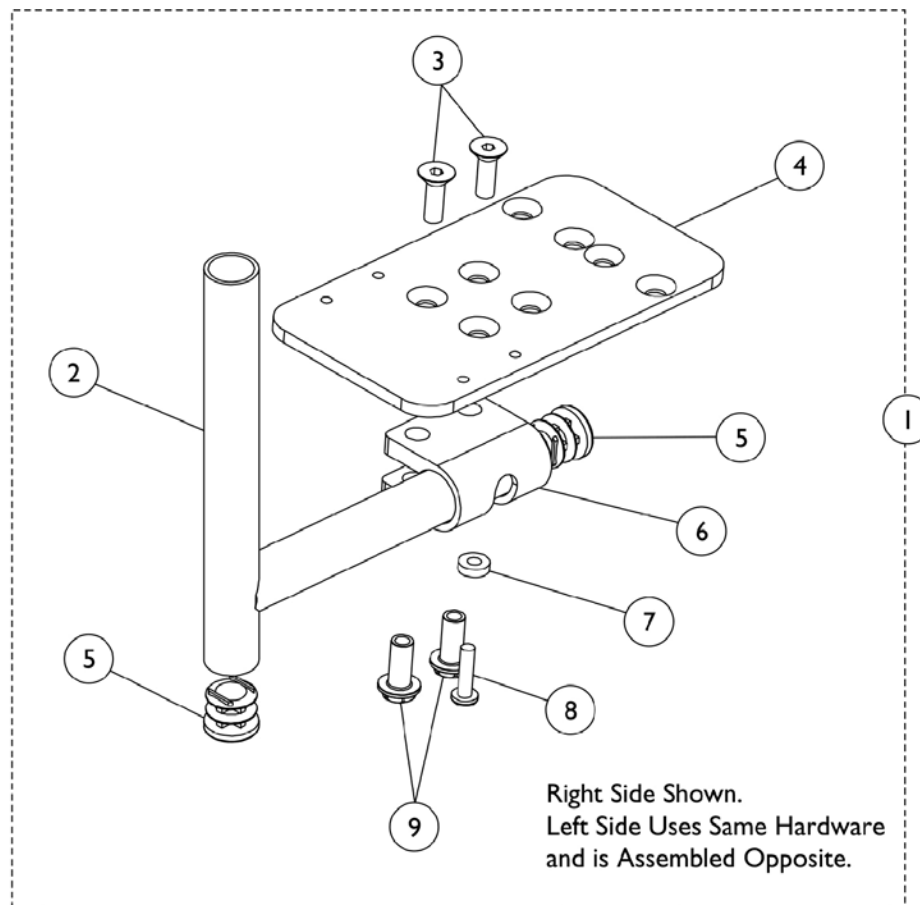


Pivot Slide Tube with Footplate Assembly - J934A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,E, G	1102495-NLA	NLA - Pivot/Slide Tube w/ Footplate and Hardware, Chrome (10" Footplate)	1	1
1	A,B,E, G	1135801	Pivot/Slide Tube w/ 10" Footplate and Hardware, Black (6 1/2")	1	1
1	A,C,E, G	1102497-NLA	NLA - Pivot/Slide Tube w/ Footplate and Hardware, Chrome (10" Footplate)	1	1
1	A,C,E, G	1133701	Pivot/Slide Tube w/ Footplate and Hardware, Black	1	1
1	A,D,E, G	1102499-NLA	NLA - Pivot/Slide Tube w/ Footplate and Hardware, Chrome (10" Footplate)	1	1
1	A,D,E, G	1135802	Pivot/Slide Tube w/ 10" Footplate and Hardware, Black (10 1/2")	1	1
1	B,F,G	1102496-NLA	NLA - Pivot/Slide Tube w/ Footplate and Hardware, Chrome (12" Footplate)	1	1
1	B,F,G	1135803	Pivot/Slide Tube w/ 12" Footplate and Hardware, Black (6 1/2")	1	1
1	C,F,G	1102498-NLA	NLA - Pivot/Slide Tube w/ Footplate and Hardware, Chrome (12" Footplate)	1	1
1	C,F,G	1133702	Pivot/Slide Tube w/ Footplate and Hardware, Black	1	1
1	D,F,G	1102500-NLA	NLA - Pivot/Slide Tube w/ Footplate and Hardware, Chrome (12" Footplate)	1	1
1	D,F,G	1135804	Pivot/Slide Tube w/ 12" Footplate and Hardware, Black (10 1/2")	1	1
2	B	1100663-NLA	NLA - Tube, Pivot/Slide, Chrome (6-1/2")	1	1
2	B	1135799	Tube, Pivot/Slide, Black (6-1/2")	1	1
2	C	1100664-NLA	NLA - Tube, Pivot/Slide, Chrome (8-1/2")	1	1
2	C	1133700	Tube, Pivot/Slide, Black (8-1/2")	1	1
2	D	1102011-NLA	NLA - Tube, Pivot/Slide, Chrome (10-1/2")	1	1
2	D	1135800	Tube, Pivot/Slide, Black (10-1/2")	1	1
3		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
4		1100988	Footplate, Black (3-1/4" W x 6" L) (10" W)	1	1
4		1100989	Footplate, Black (4-1/4" W x 6-1/2" L) (12" W)	1	1
5		1027097	Plug Button, Black (3/4")	1	2
6		1100991	Clamp, Footplate	1	1
7		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
8		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
9		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2

NOTE: A - Right side assembly shown. Left side uses same hardware and is assembled opposite.
 B - Knee-to-Heel range 7-10".
 C - Knee-to-Heel range 9-12".
 D - Knee-to-Heel range 11-14".
 E - Chair width 10".
 F - Chair width 12-14".
 G - Includes items 2-9.

Pivot Slide Tube with Footplate Assembly - J934A-14

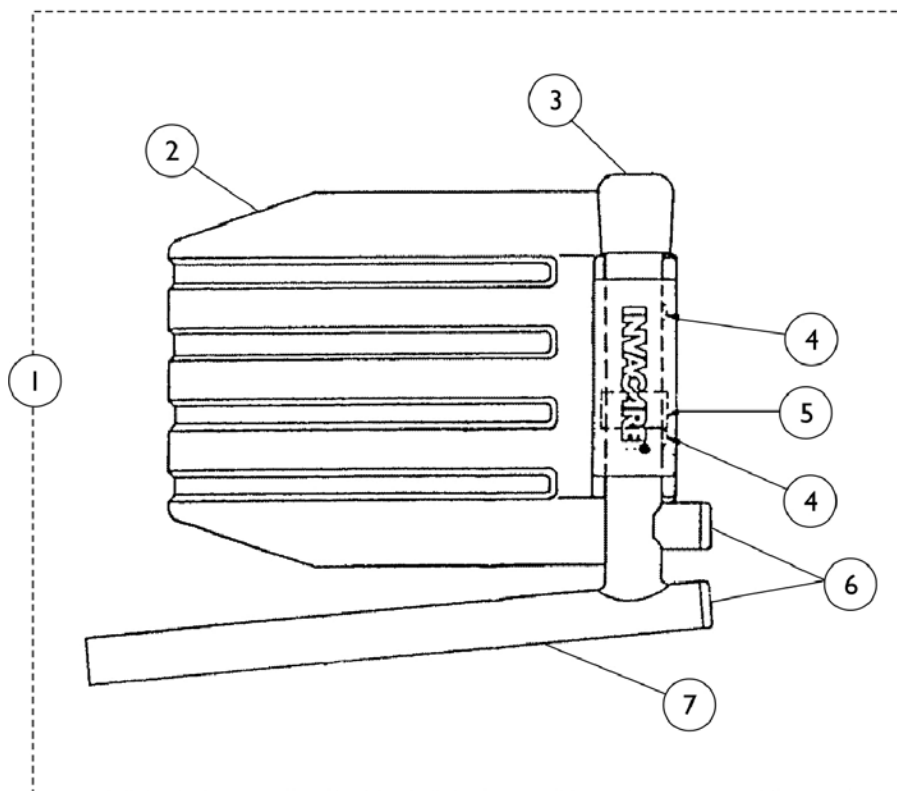

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1020234-NLA	NLA - Pivot/Slide Tube w/ Aluminum Footplate Assembly, Chrome - Right	1	1
1	A,B	1020235-NLA	NLA - Pivot/Slide Tube w/ Aluminum Footplate Assembly, Chrome - Left	1	1
2		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
2		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1016346	Cap, End, Tube, Black (7/8")	1	1
4		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
5		1006981	Spring, Tension	1	1
6		1027097	Plug Button, Black (3/4")	1	2
7		7500005-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Chrome	1	1
7		04470X013-NLA	NLA - Pivot/Slide Tube, Chrome	1	1

NOTE: A - 11-15" knee-to-heel range.
B - 14" chair width.

Calf Pads

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Calf Pads

Part Number	Description	Quantity Per Package	Quantity Per Assembly
8881192474 See Note(s) B, A, H, D	Legrest Pad - Right/Left	1	1
8881192473 See Note(s) A, H, B, E	Legrest Pad - Right/Left	1	1
88804530X003 See Note(s) C, F	Calf Pad Assembly - Right	1	1
88804530X004 See Note(s) C, F	Calf Pad Assembly - Left	1	1
1066591 See Note(s) G	Hardware, Legrest Pad	1	1
NOTE: A - Specify upholstery color when ordering, see category titled, "Upholstery Colors, Frame Finishes, and Packaging." B - For Model J934A. C - For Model J934A-14. D - 10" Wide Chairs. E - 12-14" Wide Chairs - 12" Deep. F - 14-16" Wide Chairs -14" Deep. G - Includes Screws and Instruction Sheet. H - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.			

Footrest & Legrest Accessories


Footrest and Legrest Accessories

Part Number	Description	Quantity Per Package	Quantity Per Assembly
1036099-NLA See Note(s) A	Heel Loops w/o Hardware (Pair)	2	1
1036100 See Note(s) B	Heel Loops w/o Hardware (Pair)	2	1
1035905 See Note(s) C	Heel Loop w/o Hardware (Pair) - Small	2	1
1036096 See Note(s) D	Hardware, Heel Loop (Pair)	2	1
1035926 See Note(s) E	Hardware, Heel Loops Aluminum Footplate (Pair)	2	1
1143050	Non-Removable Device	1	1

NOTE: A - Use on a 10" Wide Chair.
 B - J932A, J932A-14, J933A and J934A Front Riggings.
 C - J934A-14 Front Riggings.
 D - Use with Heel Loop #1036099 and 1036100.
 E - Use with Heel Loop #105905.

1" Forward Rear Wheel Positioner

14" Deep Non-Reclining Chairs Only

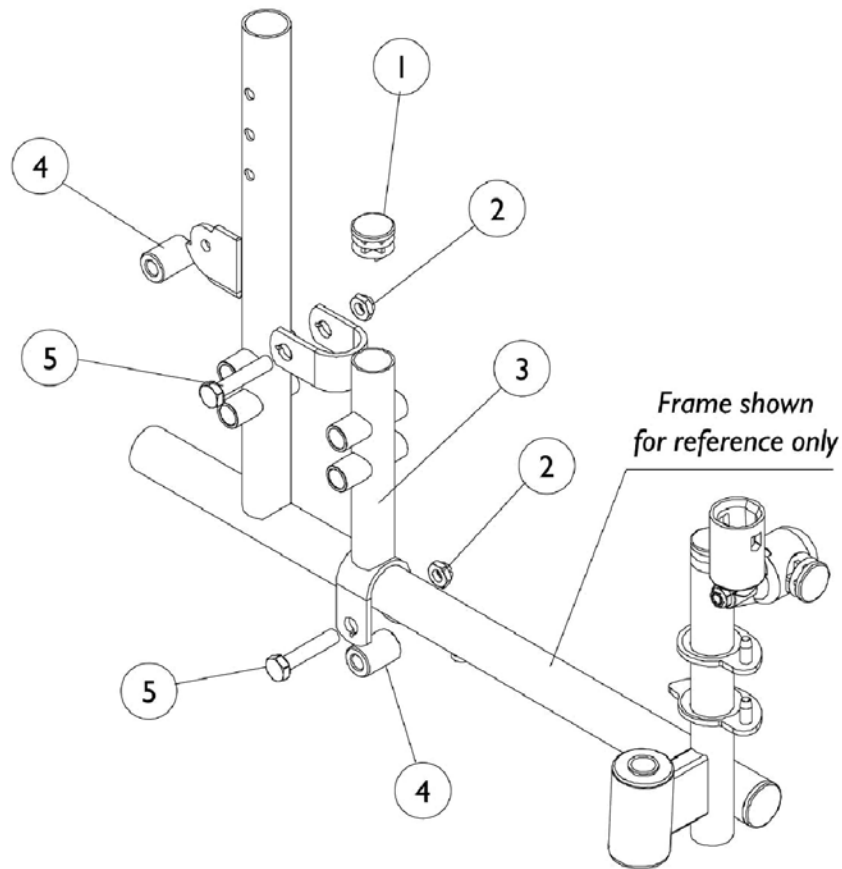

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	1
2		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	2
3	D	1101940-NLA	NLA - Bracket, Forward, Chrome - Right (1")	1	1
3	D	1101941-NLA	NLA - Bracket, Forward, Chrome - Left (1")	1	1
4		1102577	Spacer, Black (5/16 x 11/16 x 15/16")	1	2
5		11239X001	Screw, Hex Head (5/16-24 x 1-1/2")	1	2
	A,B,C	1101942	Hardware, Forward Option	2	1
NOTE: A - Not Shown B - Anti-Tippers must be used with 1" Forward Rear Wheel Positioner. See category titled, "Accessories." Wheel Lock Shoe Spacer is needed with rear wheels. See category titled, "Wheel Lock - Conventional Arms." C - Includes items 1, 2, 4, and 5. D - Item #3 is not available for color frame models.					

Solid Back and Solid Seat Inserts

Solid Back Inserts

Part Number	Chair Width X Height
8881036575	10" x 14"
See Note(s) A	
8881036580	12" x 14"
See Note(s) A	
8881036585	14" x 14"
See Note(s) A	
NOTE: A - Specify upholstery color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."	


Solid Seat Inserts

Chair Width	10" Deep	12" Deep	14" Deep	16" Deep
10" Wide	8881036544	8881036546	N/A	N/A
	See Note(s) A	See Note(s) A		
12" Wide	N/A	8881036547	N/A	N/A
		See Note(s) A		
14" Wide	N/A	N/A	N/A	8881119550
				See Note(s) A
16" Wide	N/A	N/A	8881036889-NLA	N/A
			See Note(s) A	
NOTE: A - Specify upholstery color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."				

Drop Hook Seat

Chair Width	12" Deep	14" Deep
14" Wide	8881036552	8881036550
	See Note(s) A	See Note(s) A
NOTE: A - Specify upholstery color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."		


I.V./ Oxygen Holder

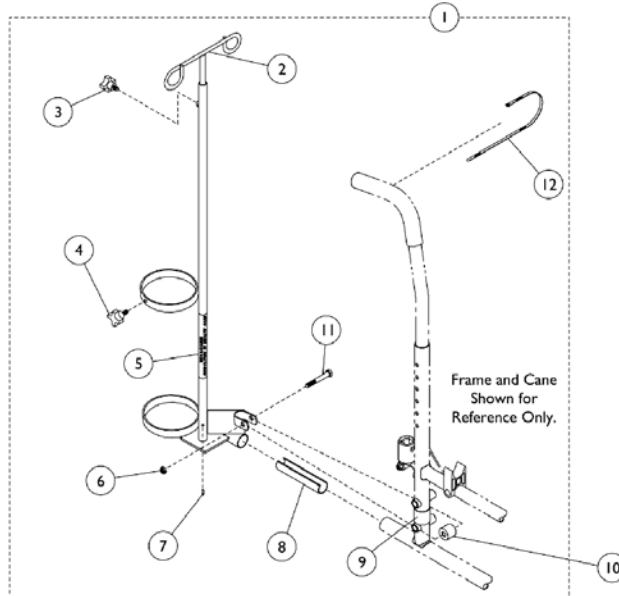

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1034217	Holder, I.V./Oxygen	1	1
2		1027468	Rod, I.V.	1	1
3		1027683	Knob, Adjustment, Black	1	1
4		1027684	Knob, Clamping, Black	1	1
5		1095497	Decal, Invacare Logo (1.6)	1	1
6		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
7		1027789	Pin, Roll	1	1
8		1023560	Insert, Guide Tube, Black (3-3/4" Long)	1	1
9		1023618	Insert, Guide Tube, Black (1" Long)	1	1
10		1102577	Spacer, Black (5/16 x 11/16 x 15/16")	1	1
11		11239X000-NLA	NLA - Screw, Hex Head (5/16-24 x 1-3/4")	1	1
12	C	98007X005	Tie Wrap (11-1/2" L)	1	1
		1027931	Instruction Sheet, I.V./Oxygen Holder	1	1

NOTE: A - Includes 2-9, 11 and 12.
 B - Items 8 and 9 may not be required for assembly.
 C - Not shown.

Oxygen Holder

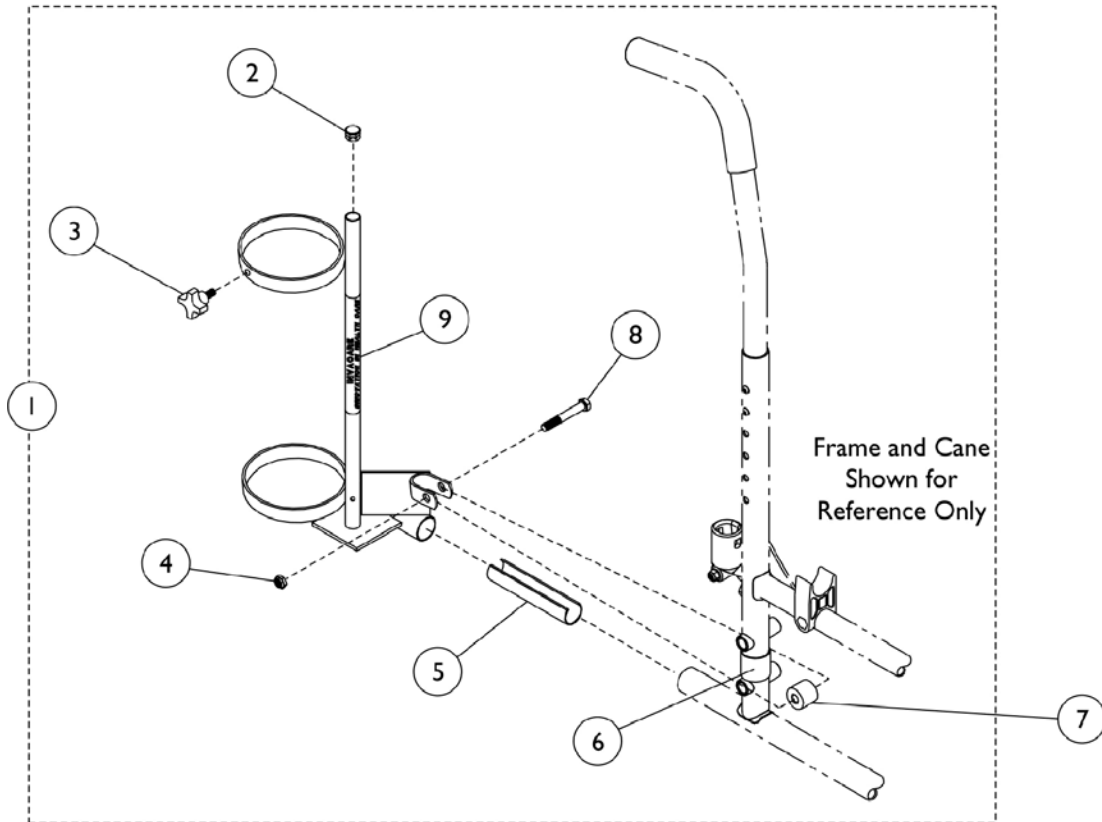


Oxygen Holder

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1034218-NLA	NLA - Holder, Oxygen	1	1
2		1017672	Plug Button, Grey (1/2")	1	1
3		1027684	Knob, Clamping, Black	1	1
4		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
5		1023560	Insert, Guide Tube, Black (3-3/4" Long)	1	1
6		1023618	Insert, Guide Tube, Black (1" Long)	1	1
7		1102577	Spacer, Black (5/16 x 11/16 x 15/16")	1	1
8		11239X000-NLA	NLA - Screw, Hex Head (5/16-24 x 1-3/4")	1	1
9	B	1095497	Decal, Invacare Logo (1.6)	1	1
		1027930	Instruction Sheet, Oxygen Holder	1	1
NOTE: A - Includes items 2-9 and an instruction sheet. Item 7 may not be required for assembly. B - Not shown.					


Stroller Handle

NOTE: For Back Canes and Hardware, see category titled, "Canes - Back."

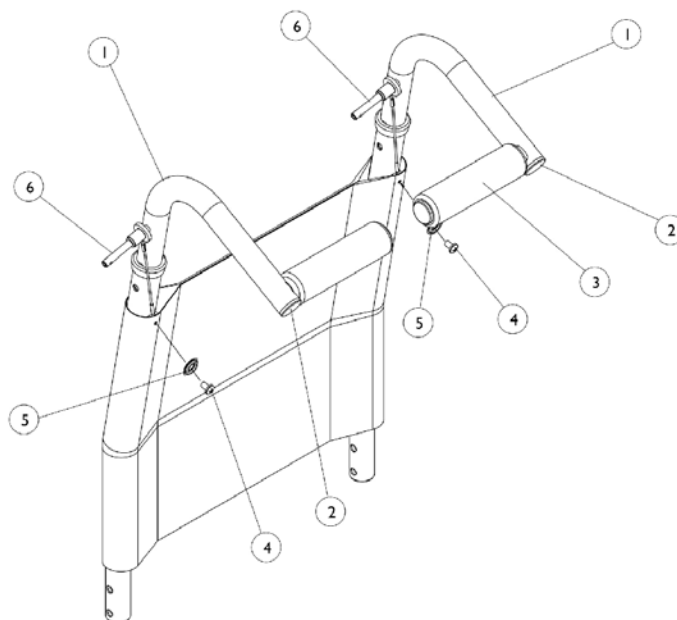

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1101954	Stroller Handle - Right	1	1
1		1101955	Stroller Handle - Left	1	1
2		1027095	Plug Button, Black (7/8")	1	6
3		1101930	Grip, Handle, Black	1	2
4		1026115-NLA	NLA - Screw, Phillips Pan Head (#10-32 x 5/16")	1	2
5		7930325-NLA	NLA - Washer (1/4 x 3/4 x 3/32")	25	2
6		1045470	Pin, Quick Release with Lanyard	1	2
	A,B	1101956	Stroller Handle Assembly - Right	1	1
	A,B	1101957	Stroller Handle Assembly - Left	1	1
	A,C	1102020	Hardware, Attaching	2	1
NOTE: A - Not Shown. B - Includes items 1 thru 3. C - Includes items 4 thru 6.					

Headrest and Push Handles - For Recliner Chairs Only - Chrome Finish

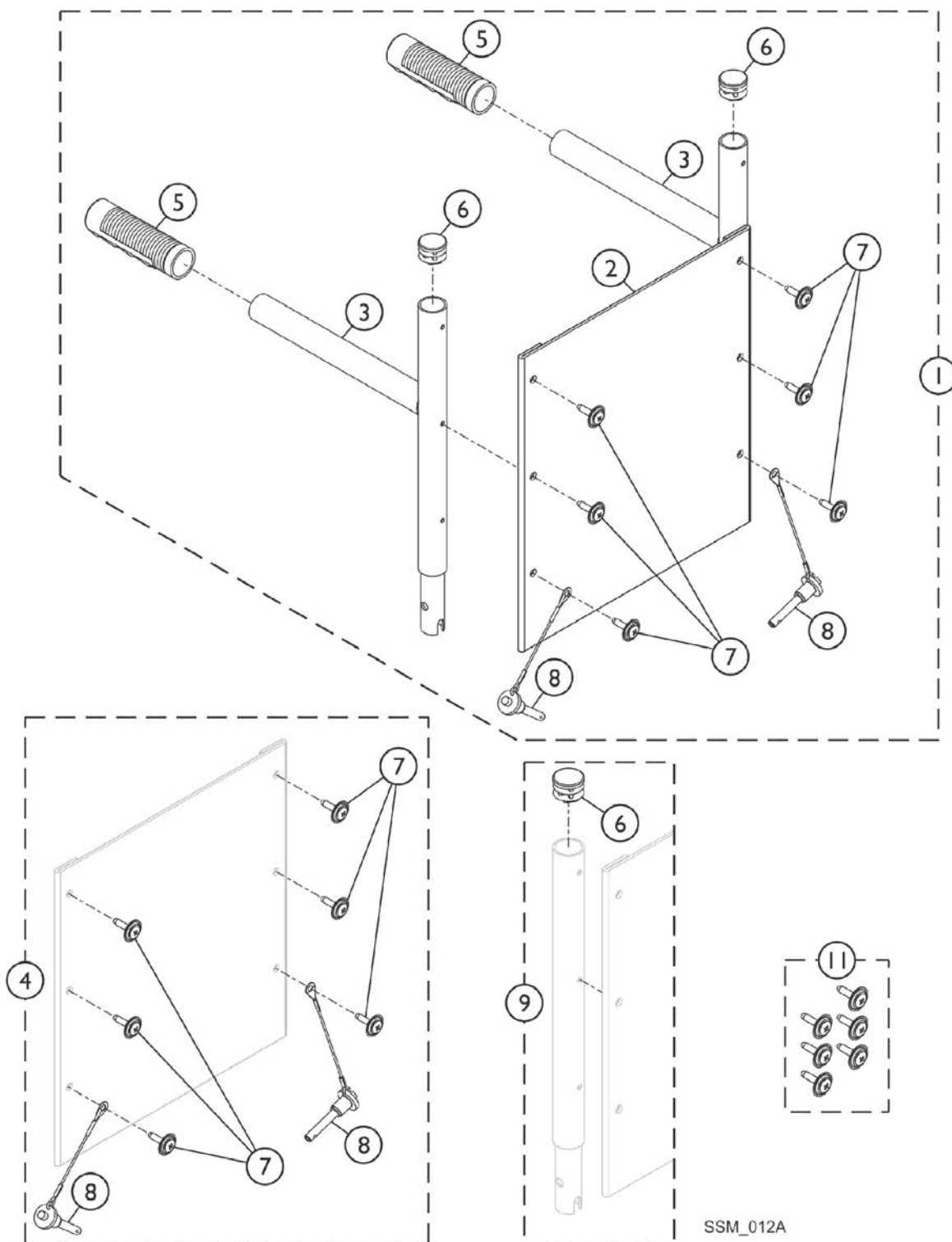


SSM_012A

Headrest and Push Handles - For Recliner Chairs Only - Chrome Finish

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1107053	Kit, Headrest with Handle Extension, Black (10" W)	1	1
1	A	1107052	Kit, Headrest with Handle Extension, Black (12" W)	1	1
1	A	1107051	Kit, Headrest with Handle Extension, Black (14" W)	1	1
2		1102536	Upholstery, Headrest (10" W)	1	1
2		1102535	Upholstery, Headrest (12" W)	1	1
2		1054627	Upholstery, Headrest, Recliner, Nylon, U240 (14" W)	1	1
2		1054628	Upholstery, Headrest, Recliner, Nylon, U240 (16" W)	1	1
3	B,F	1106218-NLA	NLA - Headrest Extension Tube Assembly with Handle, Chrome	1	2
4	C	1107050	Hardware, Attaching	1	1
5		1021261	Grip, Hand (6")	1	2
6		1027095	Plug Button, Black (7/8")	1	2
7		1076284	Screw w/ Washer (#10 x 7/8")	1	6
8		1045470	Pin, Quick Release with Lanyard	1	2
9	D	1054595-NLA	NLA - Headrest Extension Tube Assembly, Black	1	1
10	E	1102538	Cushion, Headrest (10" W)	1	1
10	E	1102537	Cushion, Headrest (12" W)	1	1
10	E	1054894	Cushion, Headrest (14" W)	1	1
10	E	1054896	Cushion, Headrest (16" W)	1	1
11		1030441	Package, Headrest Upholstery, Screws w/ Washers (#10 x 7/8")	6	1
NOTE: A - Includes items 2-4. B - Includes Headrest Extension Tube w/ Handle and items 5 and 6. C - Includes items 7 and 8. D - Includes Headrest Extension Tube and item 6. E - Not Shown F - No Longer Available. See Category titled "Accessories - Recliner Headrest & Push Handles - Color Finish" and order item #1 part number 8881182764 as a replacement.					

Headrest and Push Handles - For Recliner Chairs Only - Color Finish

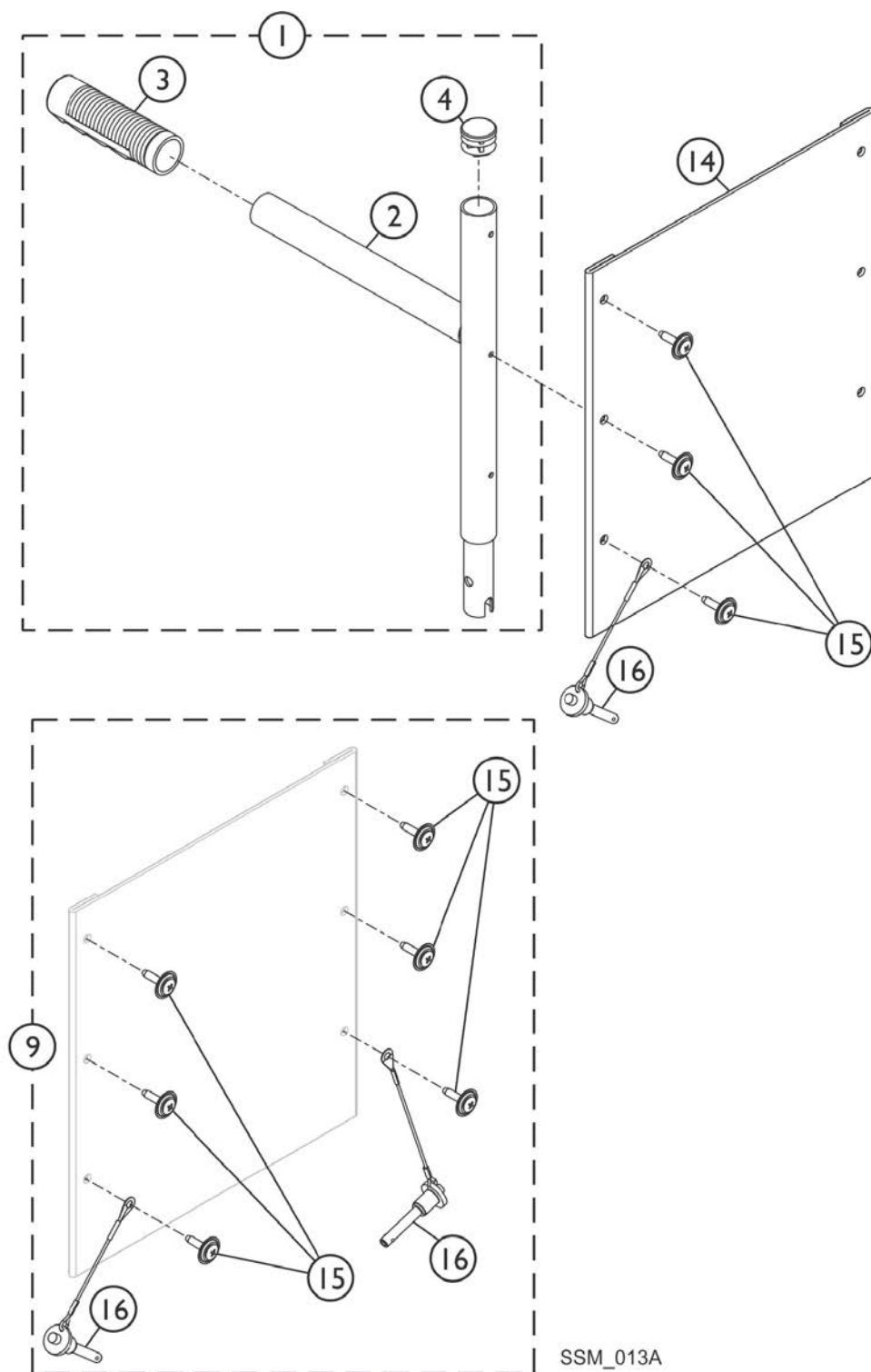

Headrest and Push Handles - For Recliner Chairs Only - Color Finish

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881182764	Kit, Headrest Extension Tube, with Handle, Color	1	1
2			Headrest Extension with Handle - Color	1	1
3			Grip, Hand (6")	1	1
4			Plug Button, Black (7/8")	1	1
9	B	1107050	Hardware, Attaching	1	1
3		1021261	Grip, Hand (6")	1	1
14	C	1102536	Upholstery, Headrest (10" W)	1	1
14	C	1102535	Upholstery, Headrest (12" W)	1	1
14		1054627	Upholstery, Headrest, Recliner, Nylon, U240 (14" W)	1	1
14		1054628	Upholstery, Headrest, Recliner, Nylon, U240 (16" W)	1	1
15		1076284	Screw w/ Washer (#10 x 7/8")	1	6
16		1045470	Pin, Quick Release with Lanyard	1	2
NOTE: A - Includes items 2-4. Specify 71P Silver Vein when ordering. B - Includes items 15 and 16. C - See category titled "Frame Finishes, Packaging, Literature and Upholstery Colors".					

Accessories

Miscellaneous Accessories

Part Number	Description	Quantity Per Package	Quantity Per Assembly
1101959	Anti-Tipper w/Hardware 20" Rear Wheel	1	1
1058136	Anti-Tipper w/Hardware 22" Rear Wheel	1	1
1102008	Positioning Straps - Airline Style Buckle	1	1
1102009	Positioning Straps Auto Style Buckle	1	1
1102007	Positioning Straps Hook & Loop Style Buckle	1	1
1101936-NLA	Bar, Adjustable, Spreader, Chrome (10 - 12" W)	1	1
1052597-NLA	Bar, Adjustable, Spreader, Chrome (14 - 15" W)	1	1
8881181933 See Note(s) A	Bar, Adjustable, Spreader, Color (10 - 12" W) See Note(s) A	1	1
8881181932 See Note(s) A	Bar, Adjustable, Spreader, Color (14 - 15" W) See Note(s) A	1	1
1038599	Wheel Lock Extension Handle Assembly, Black (6") (Pair)	2	1
1058900	Wheel Lock Extension Handle Assembly, Black (8")	1	2
NOTE: A - Specify 71P Silver Vein frame finish when ordering.			



Invacare Corporation

www.invacare.com

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Florida Technical Support
407-321-5630

Customer Service
800-333-6900

© 2022 Invacare Corporation, Elyria, Ohio.
Invacare, the Medallion Design and Yes, you
can. are trademarks; Jymni is a trademark of
Invacare Corporation.
(Revised 08/01/22)

Form No. 00-104