

Pronto® M6 **Power Wheelchair**



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 00-60. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

©2022 Invacare Corporation. Elyria, Ohio. Invacare, Yes, you can., the Medallion Design, and Pronto are registered trademarks of Invacare Corporation.

TABLE of CONTENTS

Description	Page Number
Accessories - Miscellaneous	31
Accessories - Power Wheelchair Tool Kit	33
Arms	10
Battery Harness - Cables & Charger	15
Electronics - Joystick - Controller & Cables	17
Frames - Front & Rear	4
Front Riggings - Attachment Kit - FRKTM6	20
Front Riggings - Footboard Assembly (After 11/05/01)	19
Front Riggings - Footboard Assembly (Before 11/06/01)	18
Front Riggings - Footrest - PW93	21
Front Riggings - Legrest - T94AA/C	23
Front Riggings - Legrest, Articulating - PAL4A	26
Front Riggings - Pivot Slide Tube & Footplates - T94AA/C and PAL4A	29
Motor & Gearbox Assembly	13
Packaging & Literature	3
Seat and Mounting Hardware	8
Shroud Covers	6
Wheels - Drive & Casters	12

Packaging and Literature


Note: When adjusting or replacing parts, please refer to your Owner's Manual for assistance.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1099761	Kit, Packaging Includes: Carton, Inserts and Foam.	1	1
	A	1098184-NLA	NLA - Owner's Manual, Pronto M6	1	1
	A	1102705	Paint, Touch-Up, Candy Red	1	1
	A	1101326-NLA	NLA - Paint, Touch-Up, 105P Midnight Blue	1	1
	A	1101326-NLA	NLA - Paint, Touch-Up, 105P Midnight Blue	1	1

NOTE: A - Not Shown

When adjusting or replacing parts, please refer to your Owner's Manual for assistance.


Frame Assembly, Front and Rear

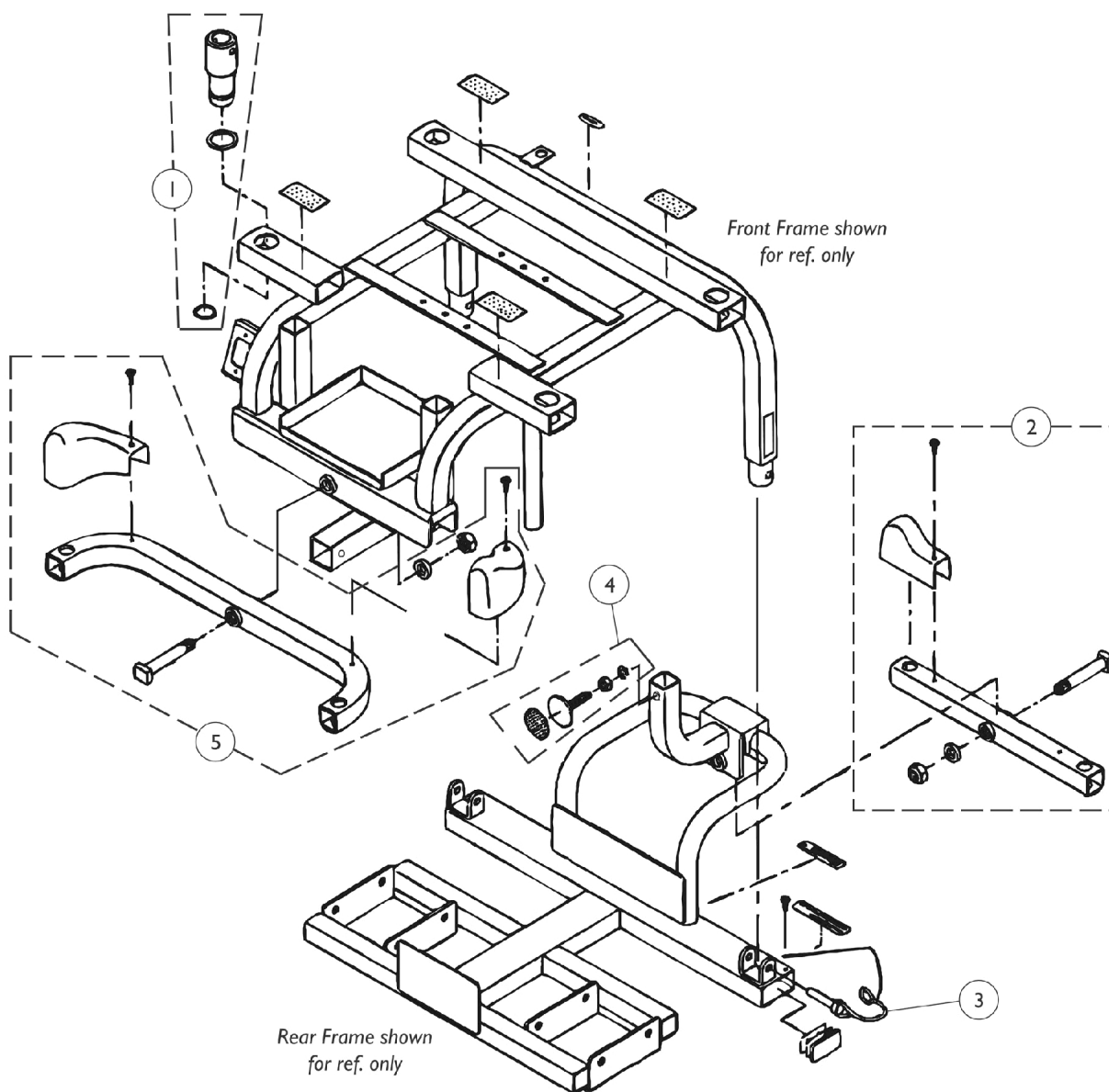

Frame Assembly, Front and Rear

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099721-NLA	NLA - Strut, Housing and Hardware Includes: 4 ea. Strut Housing, 4 ea. O-Ring, Rubber, 4 ea. Snap Ring	4	4
2		1099717-NLA	NLA - Axle Assembly (Rear) Includes: 1 ea. Axle, Rear 1 ea. Bolt, Square Head (M12 x 1.25 x 68) 1 ea. Spacer (M12 x 20 x 4.5) 1 ea. Locknut (M12 x 1.25 x 19) 2 ea. Shroud Cover, Rear Axle 2 ea. Screw, Phillips Pan Head (M4 x 12)	1	1
3		1099724-NLA	NLA - Hardware, Connecting Pin Includes: 4 ea. Connecting Pin, Frame 4 ea. Screw, Phillips Self Tap (M2 x 10)	4	4
4		1099722-NLA	NLA - Hardware, Adjustable Stop Rear Frame Includes: 1 ea. Washer, Spring (M8 x 13) 1 ea. Nut (M8 x 1.25 x 13) 1 ea. Screw, Set (M8 x 1.25 x 35) 1 ea. Fastener, Hook Adhesive Back	1	1
5		1099716-NLA	NLA - Axle Assembly (Front) Includes: 1 ea. Axle, Front 1 ea. Bolt, Square Head (M12 x 1.25 x 68) 1 ea. Spacer (12 x 20 x 4.5) 1 ea. Locknut (M12 x 1.25 x 19) 1 ea. Shroud Cover, Front Axle - Right 1 ea. Shroud Cover, Front Axle - Left 2 ea. Screw, Phillips Pan Head (M4 x 15)	1	1


Shroud Covers

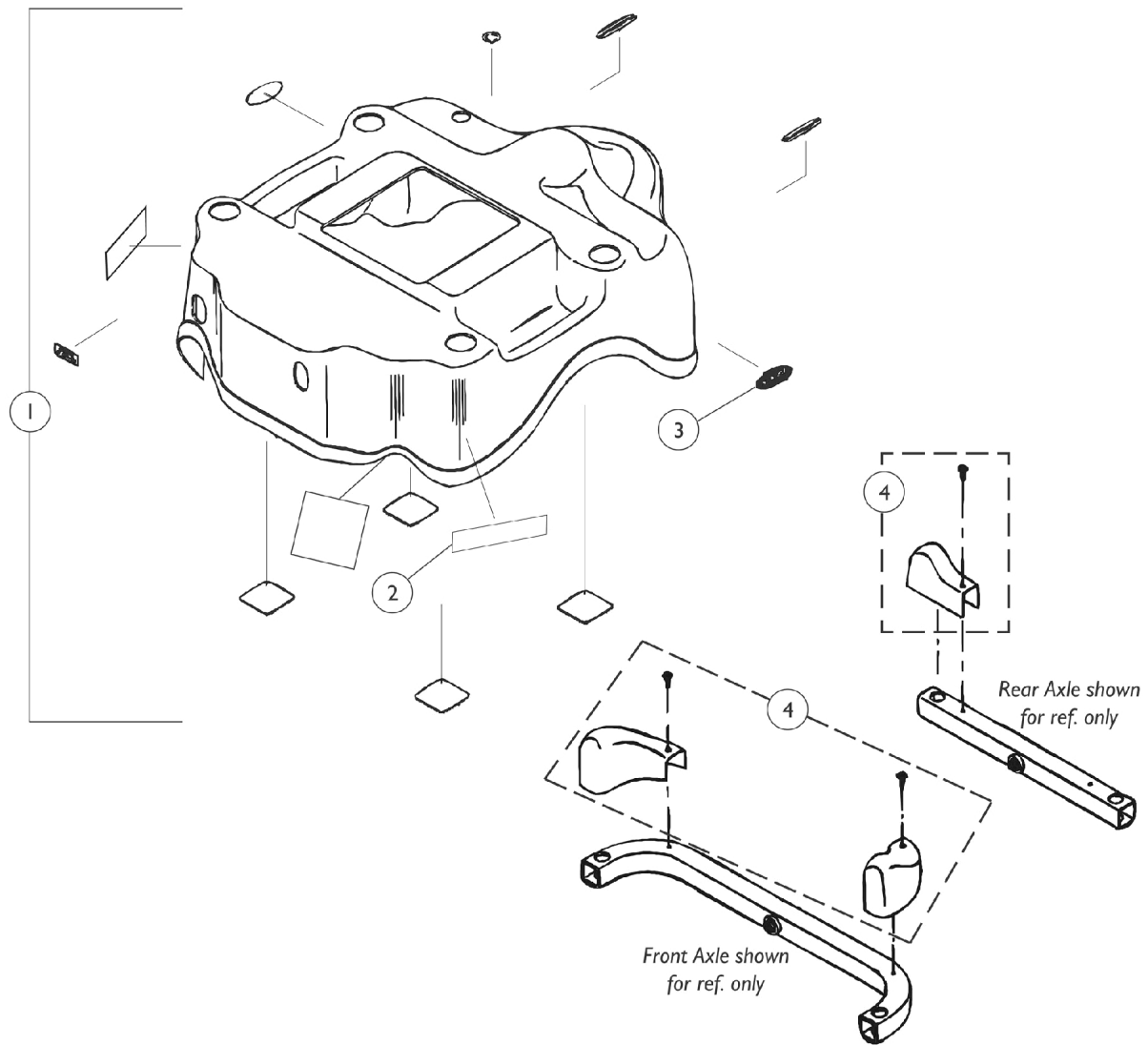

Shroud Covers

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099743-NLA	NLA - Shroud Cover Assembly with Decals, Red	1	1
1		1099745-NLA	NLA - Shroud Decal Assembly with Decals, Blue	1	1
2		1099774-NLA	NLA - Decal, Model - Pronto M6	1	2
3		1095489	Decal, Invacare 3-D Logo (2-1/2" L)	1	2
4		1099738-NLA	NLA - Kit, Shrouds w/ Screws, Front/Rear Axle Includes: (1) Shroud Cover, Front Axle - Right (1) Shroud Cover, Front Axle - Left (2) Shroud Cover, Rear Axle - Right/Left (4) Screw, Phillips Pan Head (M4 x 12)	4	4

Seat Assembly and Mounting Hardware


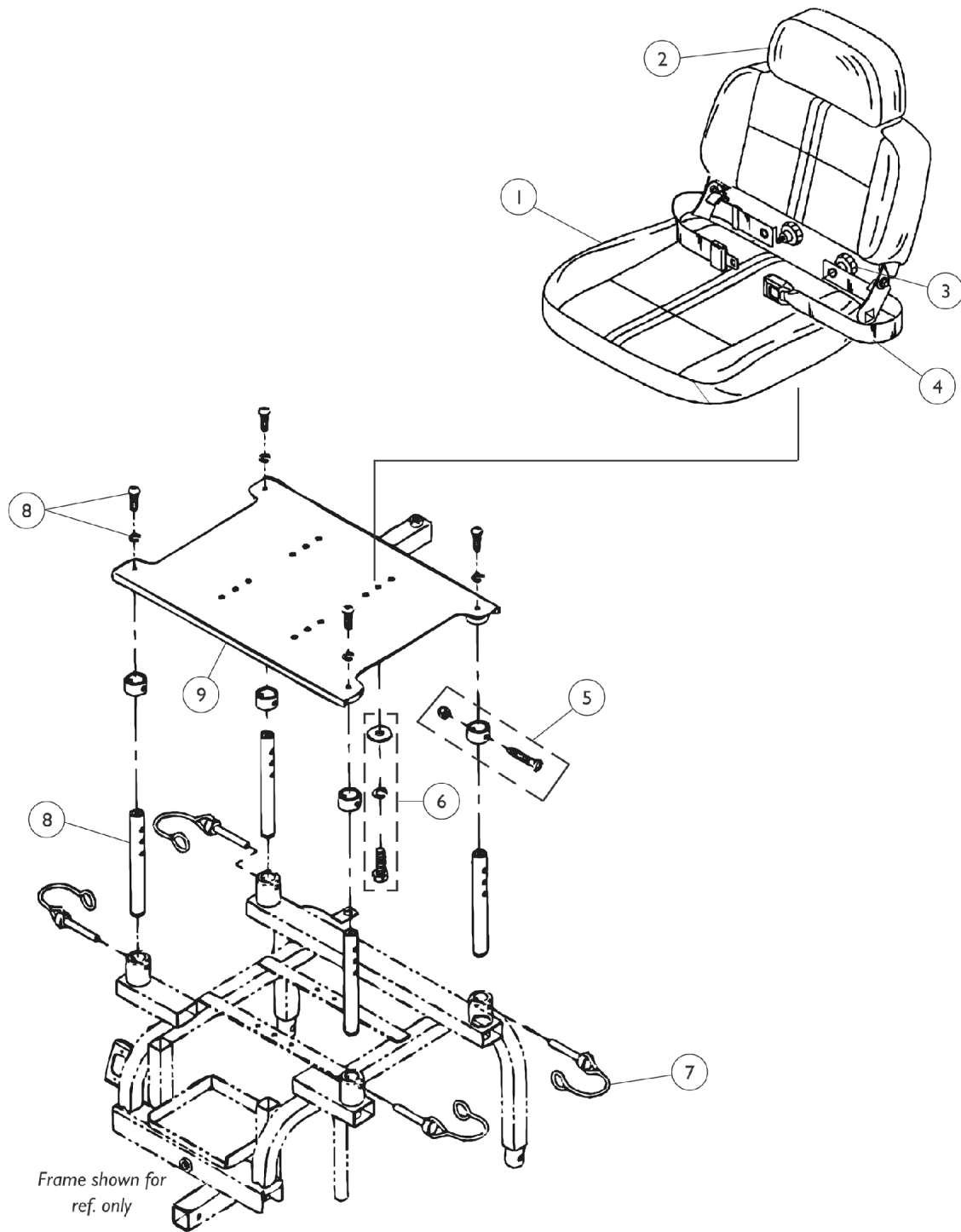


Seat Assembly and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1099755-NLA	NLA - Seat with Headrest Pillow, Gray Cloth - Before 3/01/01	1	1
1		1106210-NLA	NLA - Seat w/ Headrest Pillow, Vinyl, Grey - After 2/28/01	1	1
2		1099756-NLA	NLA - Pillow, Headrest, Gray Cloth (Before 3/01/01)	1	1
2		1106211-NLA	NLA - Headrest, Vinyl (Gray) - Lynx LX3/ LX3 Plus and Panther LX4/ MX4	1	1
3		1088768-NLA	NLA - Knob, Lower Arm Tightening	1	2
4		P715	Strap, Seat Positioning - All Models	1	1
5		1099737-NLA	NLA - Hardware, Seat Adjusting Collar Includes: 4 ea. Collar, Seat Adjusting 4 ea. Screw, Phillips Pan Head (M6 x 1.0 x 40) 4 ea. Locknut (M6 x 1.0 x 1.0)	4	4
6		1099758-NLA	NLA - Hardware, Seat Stand Mounting Includes: 4 ea. Washer (M8 x 22 x 2) 4 ea. Washer, Spring (M8 x 13) 4 ea. Screw, Hex Head Cap (M8 x 1.25 x 25)	4	4
7		1099763-NLA	NLA - Connecting Pin Hardware, Seat	4	4
8		1099740-NLA	NLA - Strut, Seat with Hardware Includes: 4 ea. Strut, Seat 4 ea. Washer, Spring (M8 x 13) 4 ea. Screw, Socket Head (M8 x 1.25 x 20)	4	4
9		1099757-NLA	NLA - Stand, Seat	1	1

NOTE: A - Cloth seats are NLA. As a replacement, order the vinyl seat with headrest pillow part number 1106210.


Arm Assembly

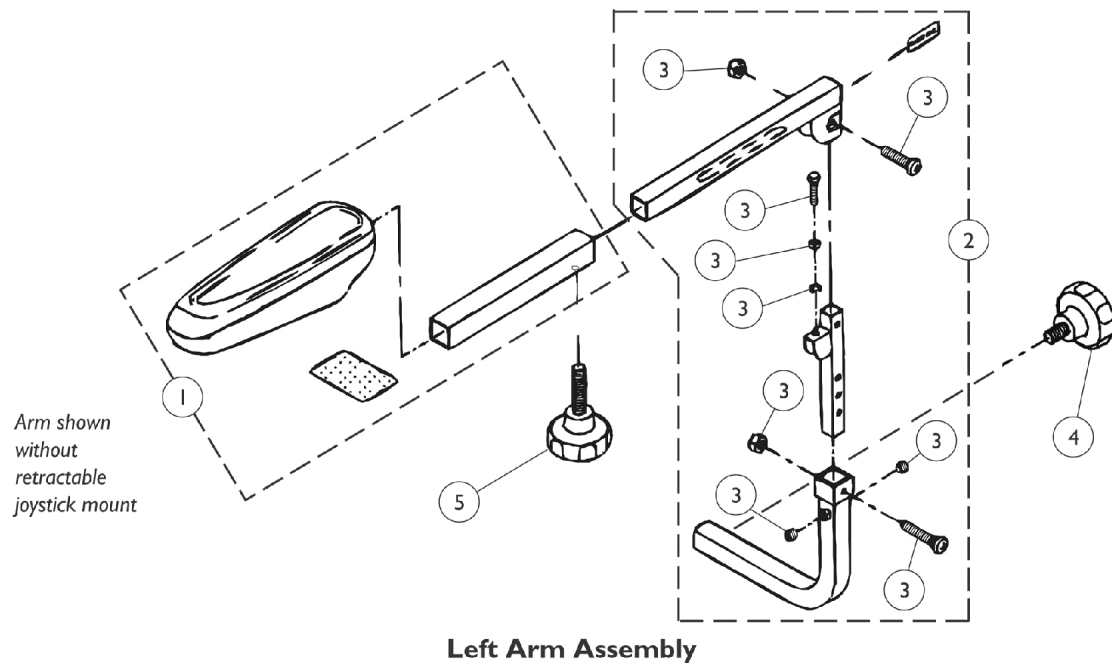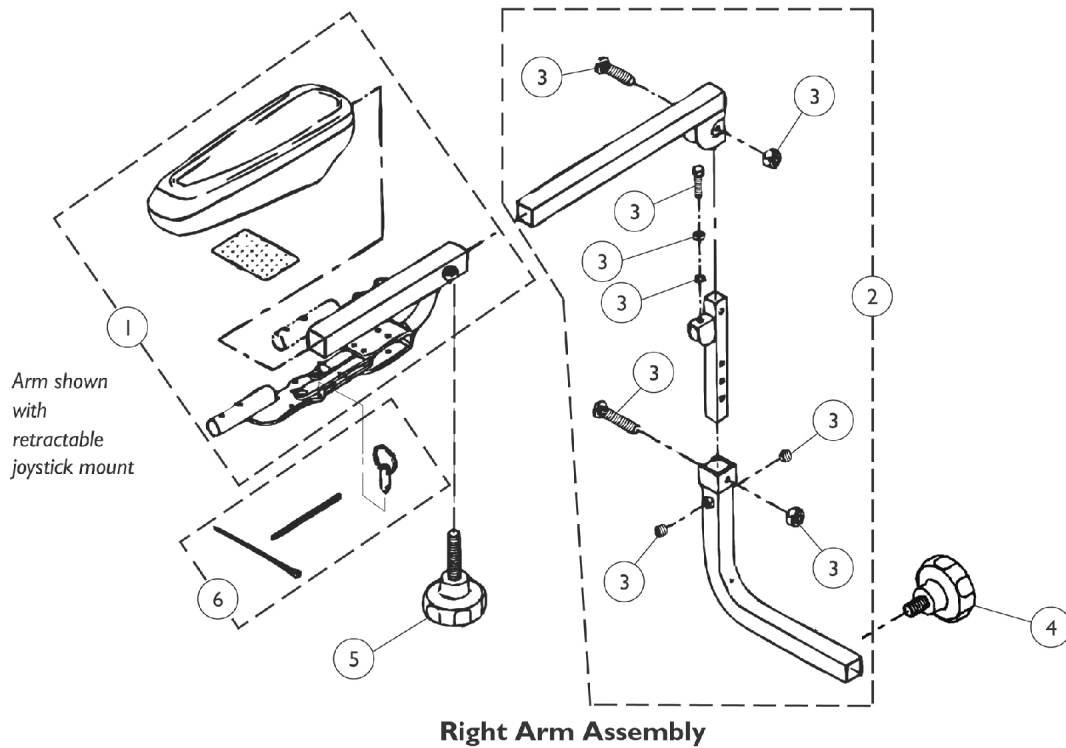

Arm Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099748-NLA	NLA - Pad, Arm with Arm Tube Sleeve and Retractable Joystick Mount - Right	1	1
1		1099749	Pad, Arm w/ Arm Tube Sleeve and Retractable Joystick Mount - Left	1	1
1		1099747-NLA	NLA - Pad, Arm w/ Arm Tube Sleeve - Right/Left	1	1
2		1099751-NLA	NLA - Arm Assembly without Arm Pad - Right	1	1
2		1099750-NLA	NLA - Arm Assembly without Arm Pad - Left	1	1
3		1099746-NLA	NLA - Arm Hardware Package (Pair) Includes: 4 ea. Screw, Nylok Socket (M8 x 1.25 x 8) 2 ea. Screw, Phillips Head (M8 x 1.25 x 40) 4 ea. Locknut (M8 x 1.25 x 13) 2 ea. Screw, Phillips Head (M8 x 1.25 x 32) 2 ea. Washer, Spring (M8 x 13) 2 ea. Nut (M8 x 1.25 x 13) 2 ea. Screw, Hex Head Cap (M8 x 1.25 x 35)	2	2
4	A	1088768-NLA	NLA - Knob, Lower Arm Tightening	1	1
5	B	1088769-NLA	NLA - Knob, Upper Arm Tightening	1	1
6		1099769-NLA	NLA - Hardware, Connecting Pin Retractable Joystick Mount Includes: 5 ea. Connecting Pin 5 ea. Chain 5 ea. Cable Tie	5	5
NOTE: A - Width adjustment. B - Depth adjustment.					

Drive Wheels and Casters

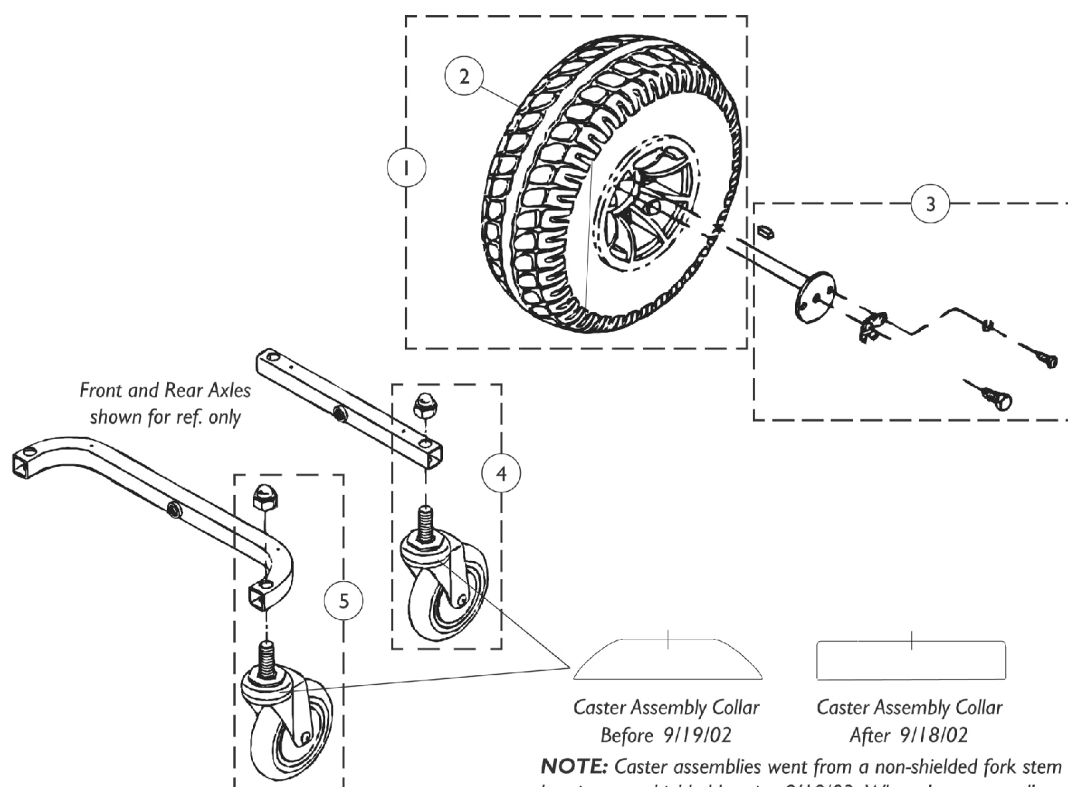


NOTE: Caster assemblies went from a non-shielded fork stem bearing to a shielded bearing 9/19/02. When the caster collar design looks similar to the diagram above made before 9/19/02, it is recommended that the casters be ordered in pairs.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099771-NLA	NLA - Wheel Assembly, Rear (10 x 3.50) - Black Rim	1	1
2		1088720-NLA	NLA - Tire w/ Foam Fill Insert, Rear, Grey (8" x 2")	1	1
3		1099739-NLA	NLA - Hardware, Rear Wheel Mounting Includes: 1 ea. Key 1 ea. Washer (M8 x 45 x 3) 1 ea. Washer, Locking Tab 2 ea. Washer, Spring (M6 x 10) 2 ea. Screw, Phillips Pan Head (M6 x 1.0 x 16) 1 ea. Screw, Hex Head Cap (M8 x 1.25 x 20)	1	1
4	A	1115709-NLA	NLA - Caster with Nut, Rear (5")	1	1
4	A	1115777-NLA	NLA - Caster with Nut, Rear (5")	2	1
5	A	1115710-NLA	NLA - Caster with Nut, Front (6")	1	1
5	A	1115778-NLA	NLA - Caster with Nut, Front (6")	2	1

NOTE: A - Casters are covered under warranty for a period of 1 year.


Motor and Gearbox Assembly

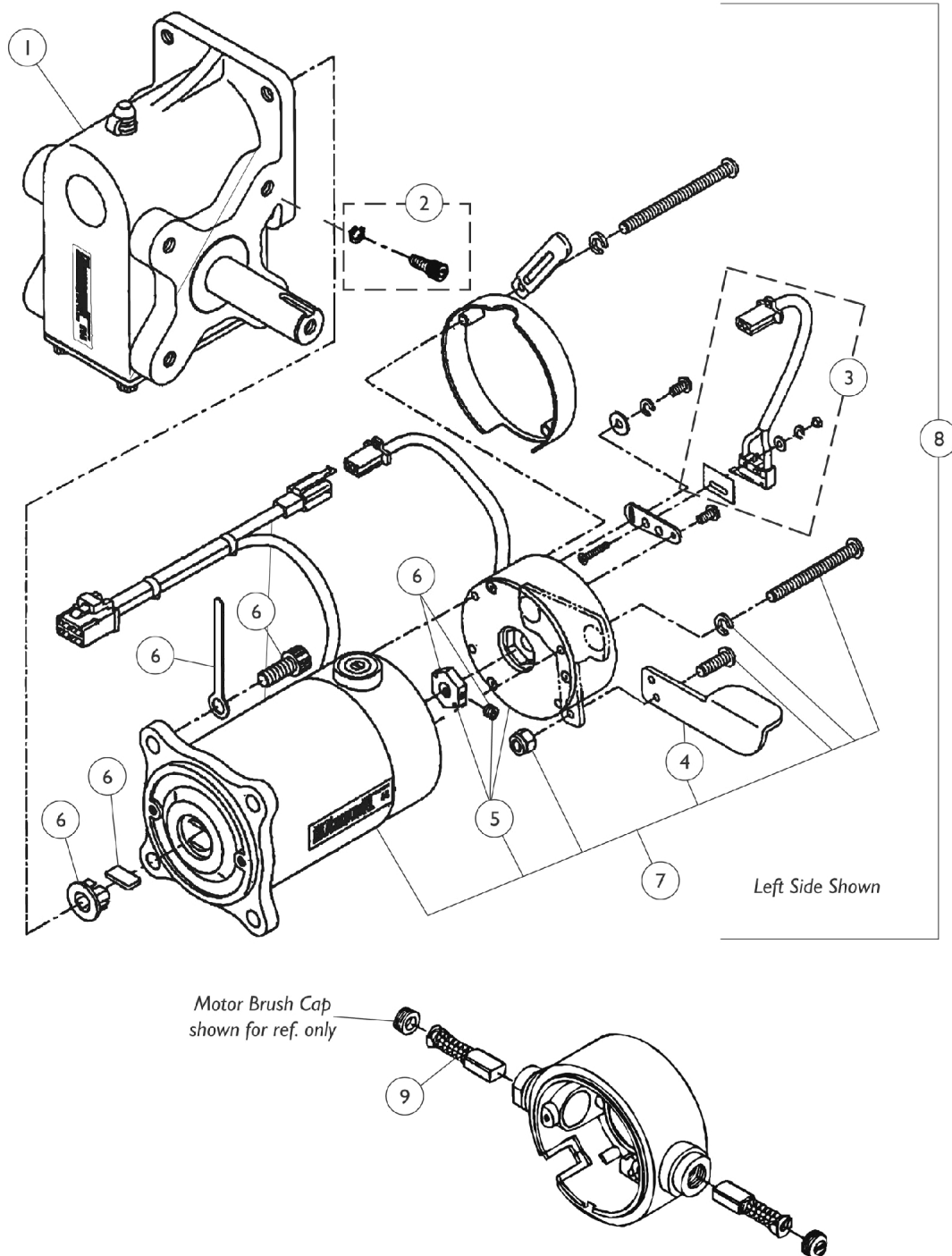

Motor and Gearbox Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099727-NLA	NLA - Gearbox - Right	1	1
1		1099726-NLA	NLA - Gearbox - Left	1	1
2		1099728-NLA	NLA - Hardware, Gearbox Mounting Includes: 8 ea. Washer, Spring (M8 x 13) 8 ea. Screw, Socket Head (M8 x 1.25 x 20)	8	8
3		1099733-NLA	NLA - Switch Assembly with Hardware Includes: 1 ea. Switch, 1 ea. Paper, Switch, 2 ea. Washer (2.2 x 5 x 0.5), 2 ea. Washer, Spring (2.1 x 4.3 x 0.7) 2 ea Nut (M2 x 0.4 x 6)	1	1
4		1099732-NLA	NLA - Handle, Free Wheel, Right	1	1
4		1099731-NLA	NLA - Handle, Free Wheel, Left	1	1
5		1099730-NLA	NLA - Brake, Motor with Axle	1	1
6		1099725-NLA	NLA - Package, Hardware, Motor/ Gearbox Includes: 1 ea. Plate, Motor Cable Tie Down 4 ea. Screw, Socket Head (M8 x 1.25 x 20) 1 ea. Plate, Bushing 1 ea. Bushing 1 ea. Driving Axle, Disc Brake with Set Screw	1	1
7		1099741-NLA	NLA - Motor/ Brake Assembly - Right	1	1
7		1099729-NLA	NLA - Motor/ Brake Assembly - Left	1	1
8		1099735-NLA	NLA - Motor/ Gearbox Assembly, Right	1	1
8		1099734-NLA	NLA - Motor/ Gearbox Assembly, Left	1	1
9		1099736-NLA	NLA - Brush, Motor (Pair)	2	2


Battery Harness, Cables and Charger

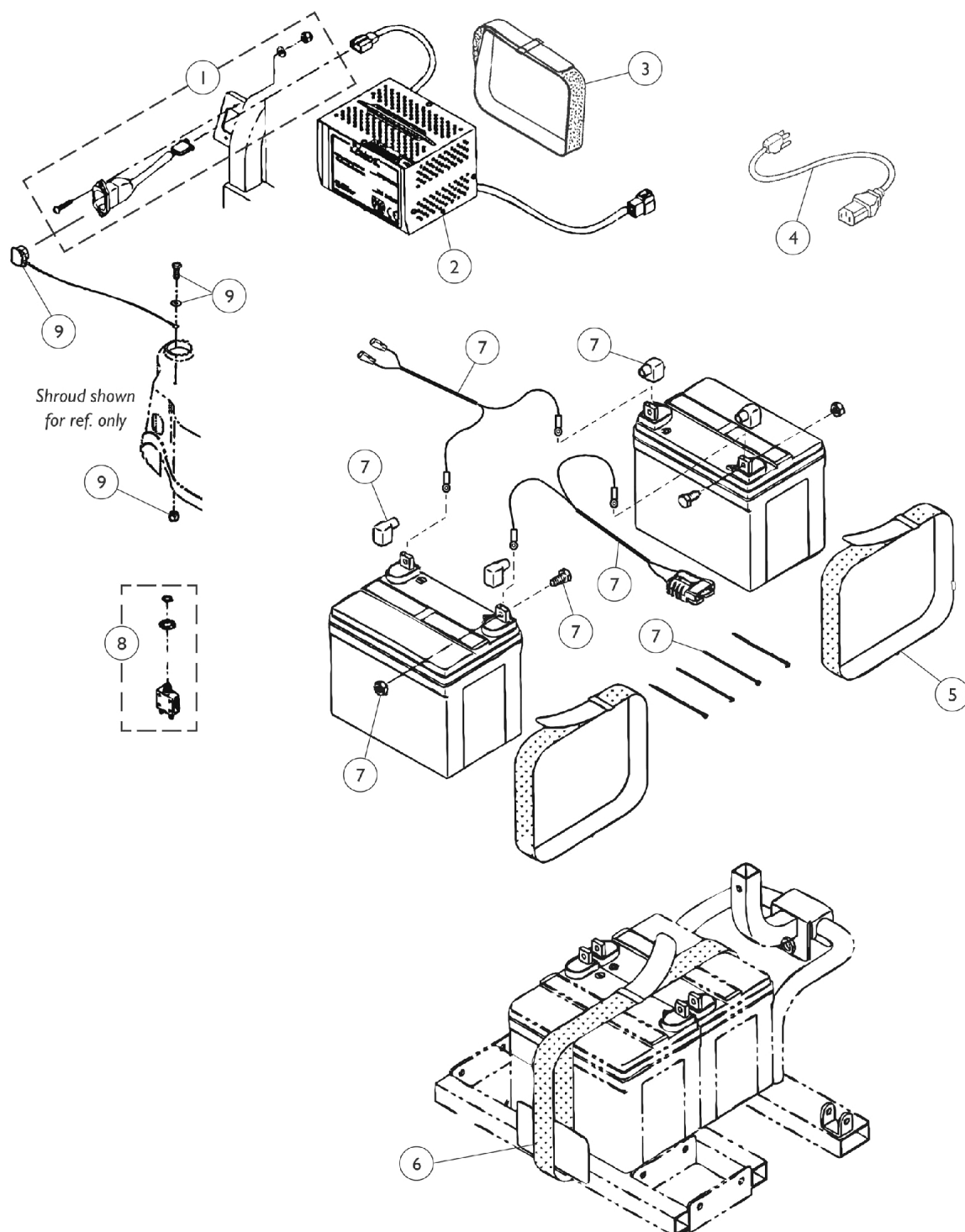

Battery Harness, Cables and Charger

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099754-NLA	NLA - Charging Cable, On Board with Hardware (Lynx SX3/ LX3/ LX3 Plus and Panther MX4) Includes: (1) Charging Cable, (2) Screw, Phillips Pan Head (M3 x 0.5 x 16) 2 ea. Washer (3.2 x 10 x 0.8) 2 ea. Locknut (M3 x 0.5 x 6) 2 ea. Cable Tie 1 ea. Tube, PVC (19 x 20 x 260L)	1	1
2	A	1099753-NLA	NLA - Charger, Battery, On-Board	1	1
2	B	1053161-NLA	NLA - Charger, Battery, 8 Amp, 115vac, Off Board Dual Mode Round Plug, Lester	1	1
3		1099744-NLA	NLA - Strap, Fastening - On-Board Charger	1	1
4		1088745-NLA	NLA - Cord, Charger, AC	1	1
5		1088742-NLA	NLA - Strap, Carrying, Battery	1	2
6		1099760-NLA	NLA - Strap, Fastening - Battery	1	1
7		1099752-NLA	NLA - Harness, Power Connector, Circuit Breaker Cables and Hardware Includes: 1 ea. Harness Power Connector 2 ea. Cover, Battery Terminal (Red) 2 ea. Cover, Battery Terminal (Black) 1 ea. Cable, Circuit Breaker (Red) 1 ea. Cable, Circuit Breaker (Black) 1 ea. Tube, Snake (10 x 110L) 1 ea. Tube, Snake (10 x 95L) 4 ea. Screw, Hex Head Cap (M6 x 1.0 x 12) 4 ea. Hexnut (M6 x 1.0 x 10) 4 ea. Cable Tie	1	1
8		1088748-NLA	NLA - Circuit Breaker (30 AMP) - Lynx SX3P/ SX3/ LX3/ LX3 Plus, Panther LX4 and Zoom-3	1	1
9		1099772-NLA	NLA - Charger Cable Cover Assembly Includes: 1 ea. Cover, Charger Cable 1 ea. Screw, Phillips Pan Head (M4 x 0.7 x 12) 1 ea. Washer (M4 x 9 x 1) 1 ea. Locknut (M4 x 0.7 x 7)	1	1
NOTE: A - (On-Board) Battery Charger #1099753 is no longer available. Use the (Off Board) Charger part #1053161 as a replacement. B - Use Battery Charger part number 1123249 as a replacement.					

Electronics

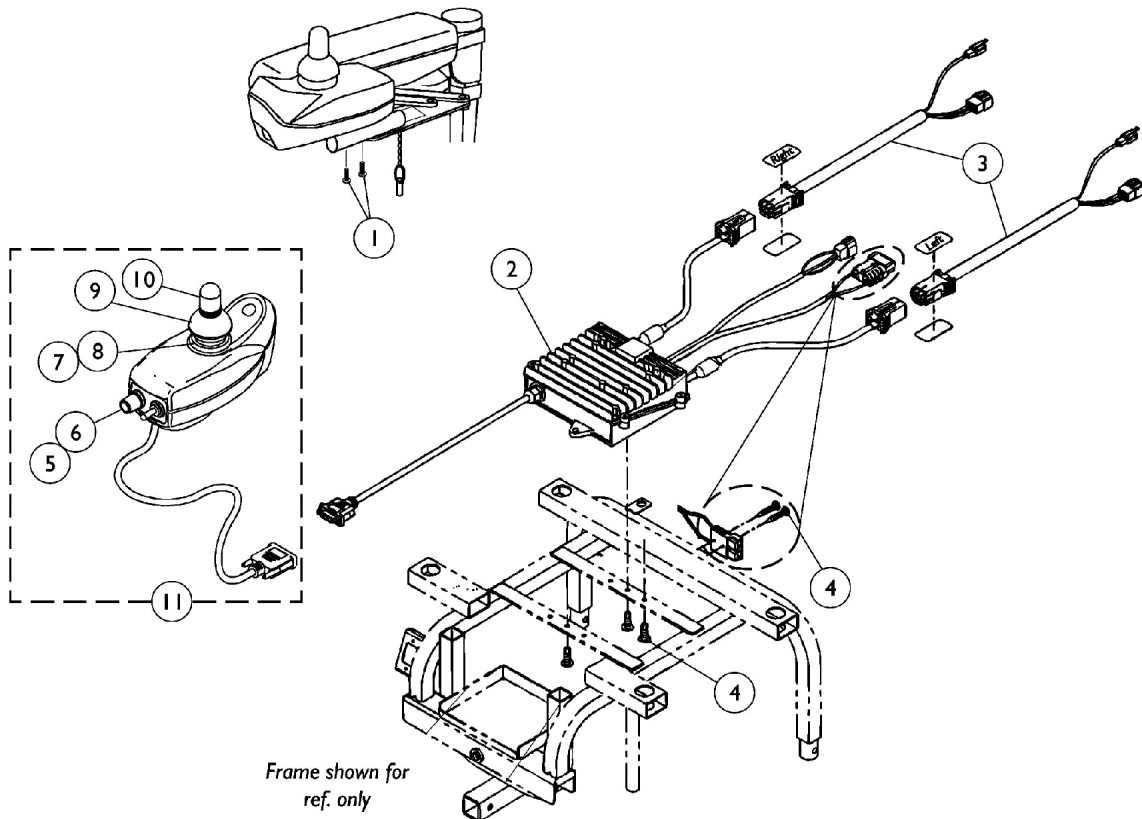


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099767-NLA	NLA - Screw, Phillips Pan Head (#10-32 x 1/2")	10	2
2		1098644-NLA	NLA - Controller, MKIV	1	1
3		1099742-NLA	NLA - Cable, Motor/ Brake w/ Right and Left Decals (Pair)	2	2
4	B	1099768-NLA	NLA - Controller Mounting Hardware	1	1
5		1040225-NLA	NLA - Potentiometer, Speed	1	1
6		1025516-NLA	NLA - Knob, Speed Control	1	1
7		50000X364-NLA	NLA - Boot, Joystick	1	1
8		1036764-NLA	NLA - Inductive, Joystick	1	1
9		1040218	Skirt, Joystick Rubber, Black	1	1
10		1040217	Knob, Joystick Rubber, Black	1	1
11	A	1092610-NLA	NLA - Joystick, RII	1	1

NOTE: A - To mount joystick on left side, see arm assembly page item 1.
 B - Includes: (3) Screw, Phillips Pan Head (#8-36 x 1/2"), (2) Screw, Phillips Pan Head (1/8 x 3/4").

Footboard Assembly


Before 11/06/01

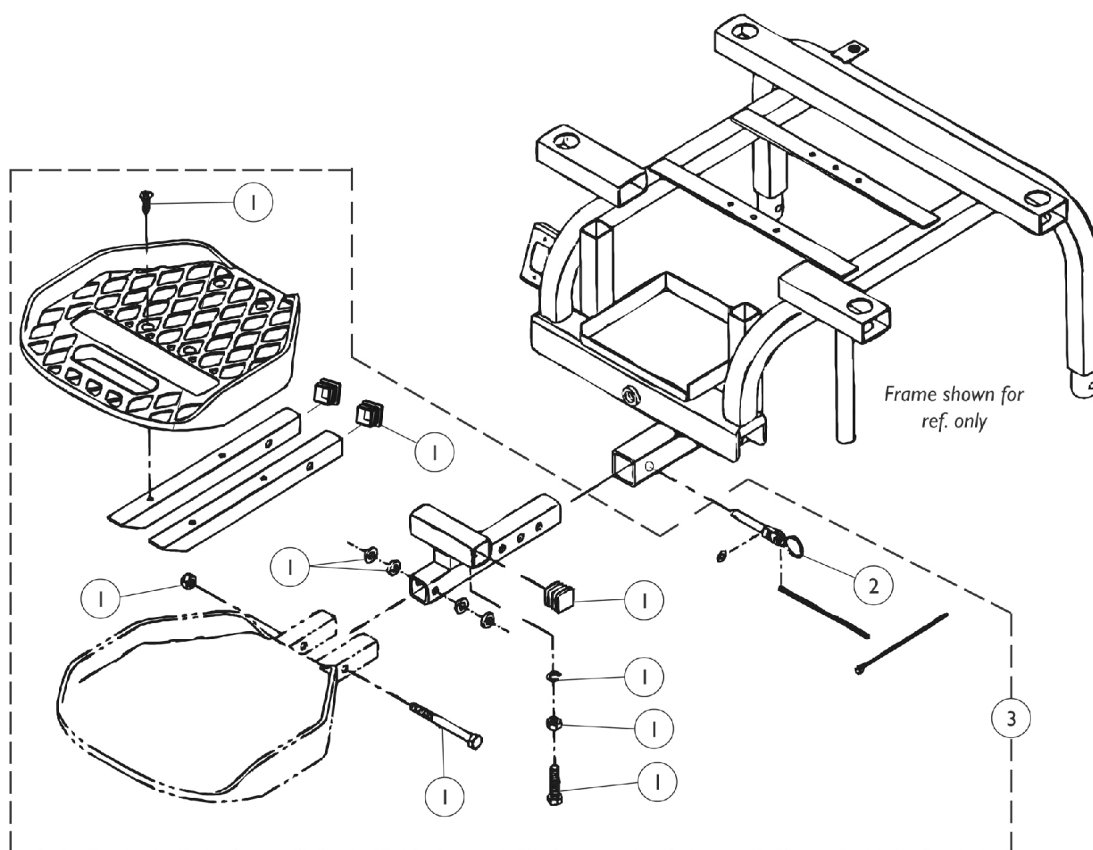

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099770-NLA	NLA - Footboard Hardware Includes: 4 ea. Screw, Phillips Head w/Patch (M6 x 1.0 x 20) 4 ea. Washer (8.2 x 15.8 x 3) 1 ea. Bolt, Hex Head Cap (M8 x 1.25 x 85) 1 ea. Locknut (M8 x 1.25 x 35) 2 ea. Washer, Spring (M8 x 13) 2 ea. Nut (M8 x 1.25 x 13) 2 ea. Screw, Hex Head Cap (M8 x 1.25 x 35) 2 ea. Plug Button, Square (3/4 x 3/4") 2 ea. Plug Button, Square (5/8 x 5/8")	1	1
2		1099715-NLA	NLA - Pin, Safety	4	1
3		1110986-NLA	NLA - Footboard Assembly	1	1


Footboard Assembly

After 11/05/01

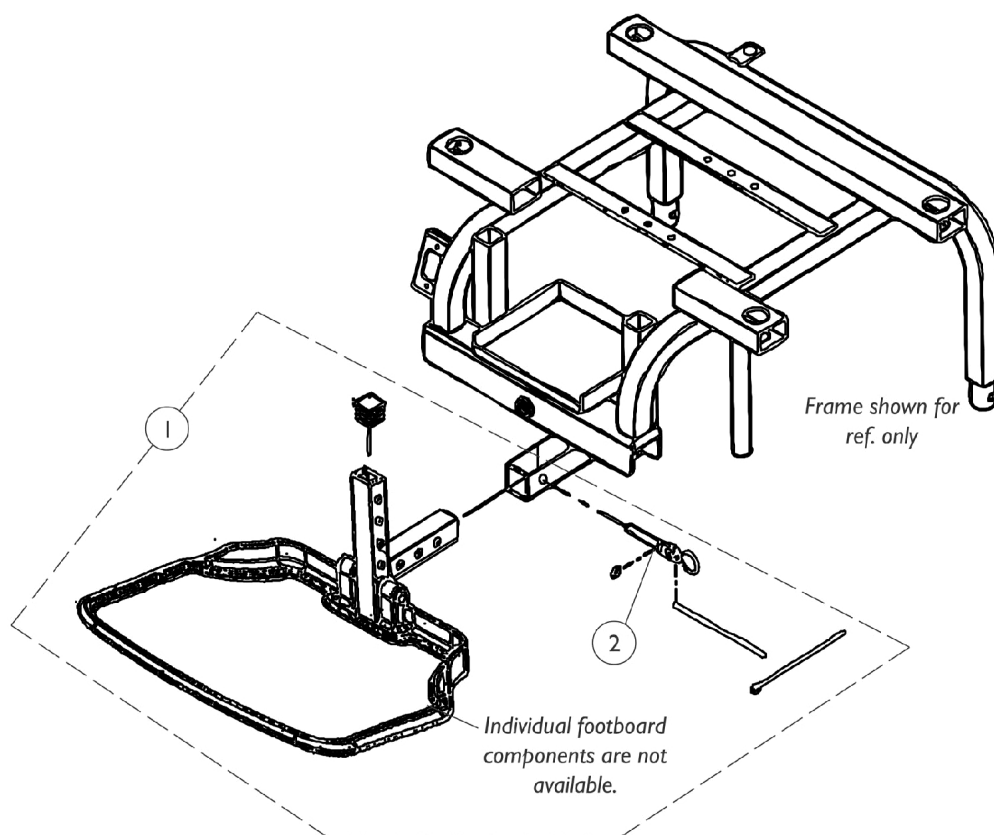

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099714-NLA	NLA - Footboard Assembly Includes: 1 ea. Footboard with Rubber Pad 1 ea. Adjustment Tube, Footboard 2 ea. Washer (8.2 x 15.8 x 3) 1 ea. Safety Pin 1 ea. Cable Tie 1 ea. Nut (M8 x 1.25 x 13) 1 ea. Screw, Hex Head Cap (M8 x 1.25 x 25) 2 ea. Plug Buttons, Ball 1 ea. Screw, Hex Head Cap (M8 x 1.25 x 60)	1	1
2		1099715-NLA	NLA - Pin, Safety	4	1

Front Riggngs Attachment Kit (FRKTM6)



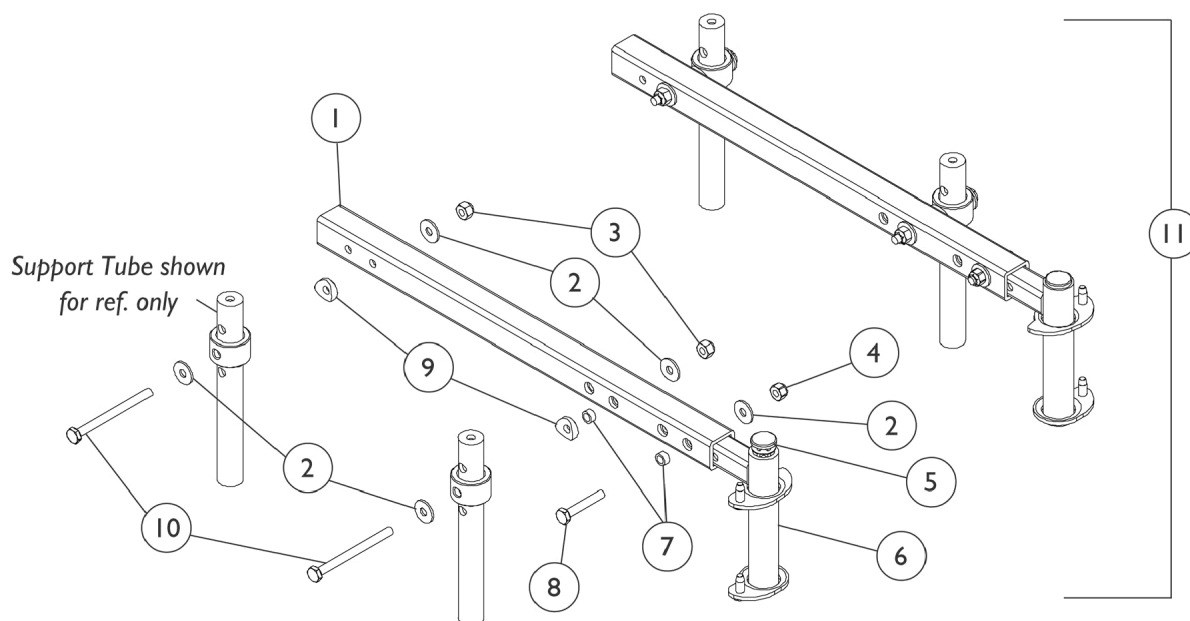


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1107807-NLA	NLA - Tube, Support, Front Rigging, Black	1	2
2		1026900	Washer (1/4 x 23/32 x 1/16")	1	10
3		7930345	Locknut (1/4-28)	25	4
4		7930333	Locknut (1/4-20)	25	2
5		1027097	Plug Button, Black (3/4")	1	2
6		1078020-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Right	1	1
6		1078021-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Left	1	1
7		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
8		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	2
9		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
10		1059374	Screw, Hex Head (1/4-28 x 3")	1	4
11		1107808-NLA	NLA - Kit, Front Rigging Attachment with Instruction Sheet	1	1
	A	1106633-NLA	NLA - Instruction Sheet, Front Rigging Attachment Kit	2	1

NOTE: A - Not Shown

PW93 Footrest

Heel loops are not included. See accessories page.

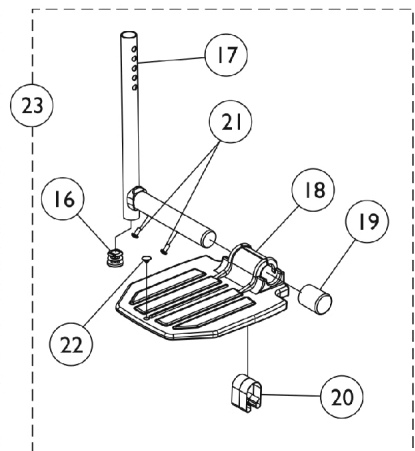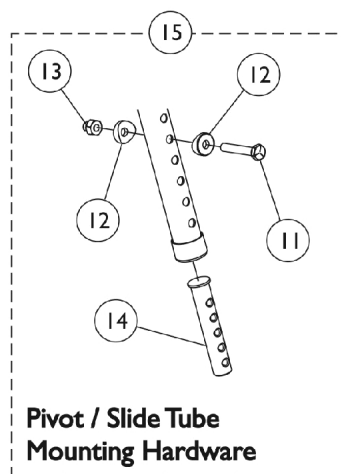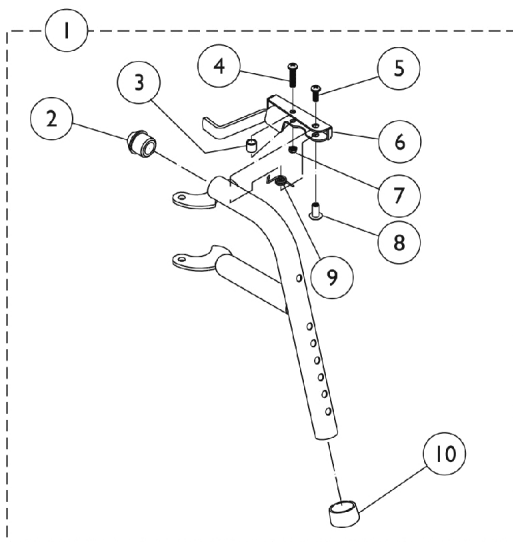



PW93 Footrest

Heel loops are not included. See accessories page.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881070776	Footrest Support Assembly, Adult, Color - Right	1	1
1	A	8881070777	Footrest Support Assembly, Adult, Color - Left	1	1
2		1020719	Anti-Rattle, Black	1	1
3		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
6		1035539	Handle, Release, Black - Right	1	1
6		1035540	Handle, Release, Black - Left	1	1
7		1025735	Locknut (#10-32)	1	1
8		1028846-NLA	NLA - Rivnut (#10-32)	1	1
9		1028707	Spring, Release - Right	1	1
9		1028708	Spring, Release - Left	1	1
10		15680X012	Collar, Anti-Rattle (7/8")	1	1
11		1070759	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
12		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
13		1044214	Nut, Hex with Nylon Cap (1/4-20)	1	1
14	B	1020534	Insert, Threaded	1	1
15		1074143	Kit, Slide Tube Mounting Hardware	1	1
16		1027097	Plug Button, Black (3/4")	1	1
17		1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
18		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
19		1016346	Cap, End, Tube, Black (7/8")	1	1
20		1006981	Spring, Tension	1	2
21		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	1
22		1070168	Plug, Push, Footplate	1	1
23		1074149	Pivot/Slide Tube Assembly w/ Aluminum Footplate, Black - Right	1	1
23		1074150	Pivot/Slide Tube Assembly w/ Aluminum Footplate, Black - Left	1	1
NOTE: A - Specify 24P black frame color when ordering. B - As of 8/15/03, the Threaded Insert was revised from 3 to 5 Holes. Threaded Insert is Backward Compatible and does not require ordering in Pairs.					

T94AA/C Legrest

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

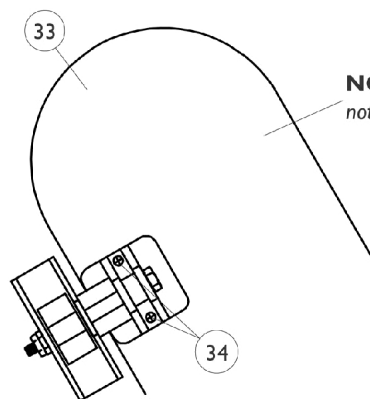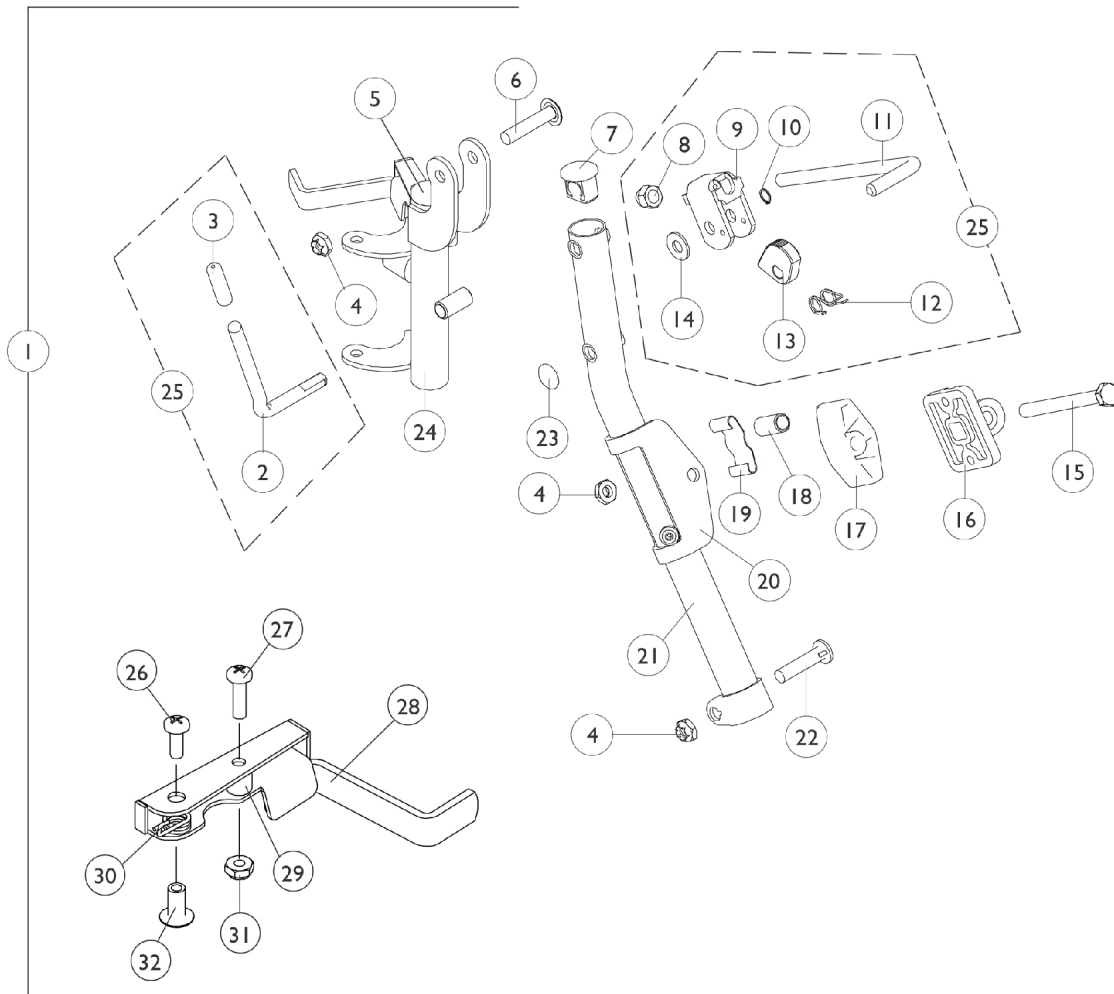

NOTE: Legrest pad and attaching hardware not included with support assembly.

T94AA/C Legrest

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,G	1031579-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Right	1	1
1	C,G	1031580-NLA	NLA - Legrest Support Assembly, Adult, Chrome - Left	1	1
2		1037126	Handle, Release	1	1
3		1040060	Tip, Plastic, Black	1	1
4		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	3
5		1027095	Plug Button, Black (7/8")	1	1
6		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
7		1095433	Plug Button, Black (7/8")	1	1
8		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
9		04528X002	Bracket, Release, Black	1	1
10		1040831	Ring, Snap	1	1
11		42326X001	Bar, Ratchet	1	1
12		1037485	Spring, Release	1	1
13		1037125	Cam, Legrest	1	1
14		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
15		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
16		04526X019	Plate, Legrest Pad	1	1
17		1091220	Spacer, Legrest	1	1
18		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
19		48823X008	Spring, Leaf	1	1
20		1025104	Bracket, Legrest, Chrome	1	1
21	A,E	1031358-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Right	1	1
21	A,E	1031359-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Left	1	1
21	F	1149473-NLA	NLA - Tube, Support, Legrest, Elevating, Silver Vein - Right	1	1
21	F	1149474-NLA	NLA - Tube, Support, Legrest, Elevating, Silver Vein - Left	1	1
22		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
23		1096773	Nut, Push Dome (5/16")	1	1
24		1031581-NLA	NLA - Legrest Support Tube Assembly, Adult, Black - Right	1	1
24		1031582	Legrest Support Tube Assembly, Adult, Black - Left	1	1
25	D	1046328-NLA	NLA - Kit, Ratchet	2	1
26		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
27		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
28		1035539	Handle, Release, Black - Right	1	1
28		1035540	Handle, Release, Black - Left	1	1
29		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
30		1028707	Spring, Release - Right	1	1
30		1028708	Spring, Release - Left	1	1




T94AA/C Legrest

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
31		1025735	Locknut (#10-32)	1	1
32		1028846-NLA	NLA - Rivnut (#10-32)	1	1
33	B,J	8881091700	Pad, Legrest	1	1
34	H	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
NOTE: A - Due to product improvement, item 5 must also be ordered. B - Specify U05 black light weight vinyl upholstery when ordering. C - Includes items 2-24. D - Includes items 2, 3 and 8-14. E - Before 7/6/07 F - After 7/5/07 G - After 7/5/07 Color changed from chrome to silver vein. It is recommended to order in pairs to match H - Includes Screws and Instruction Sheet. J - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.					

PAL4A Articulating Legrest

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

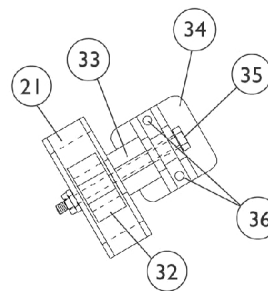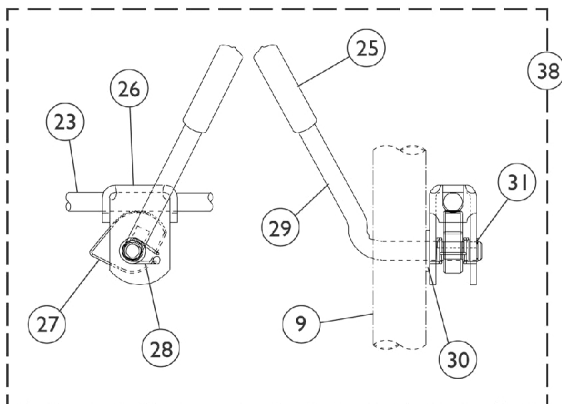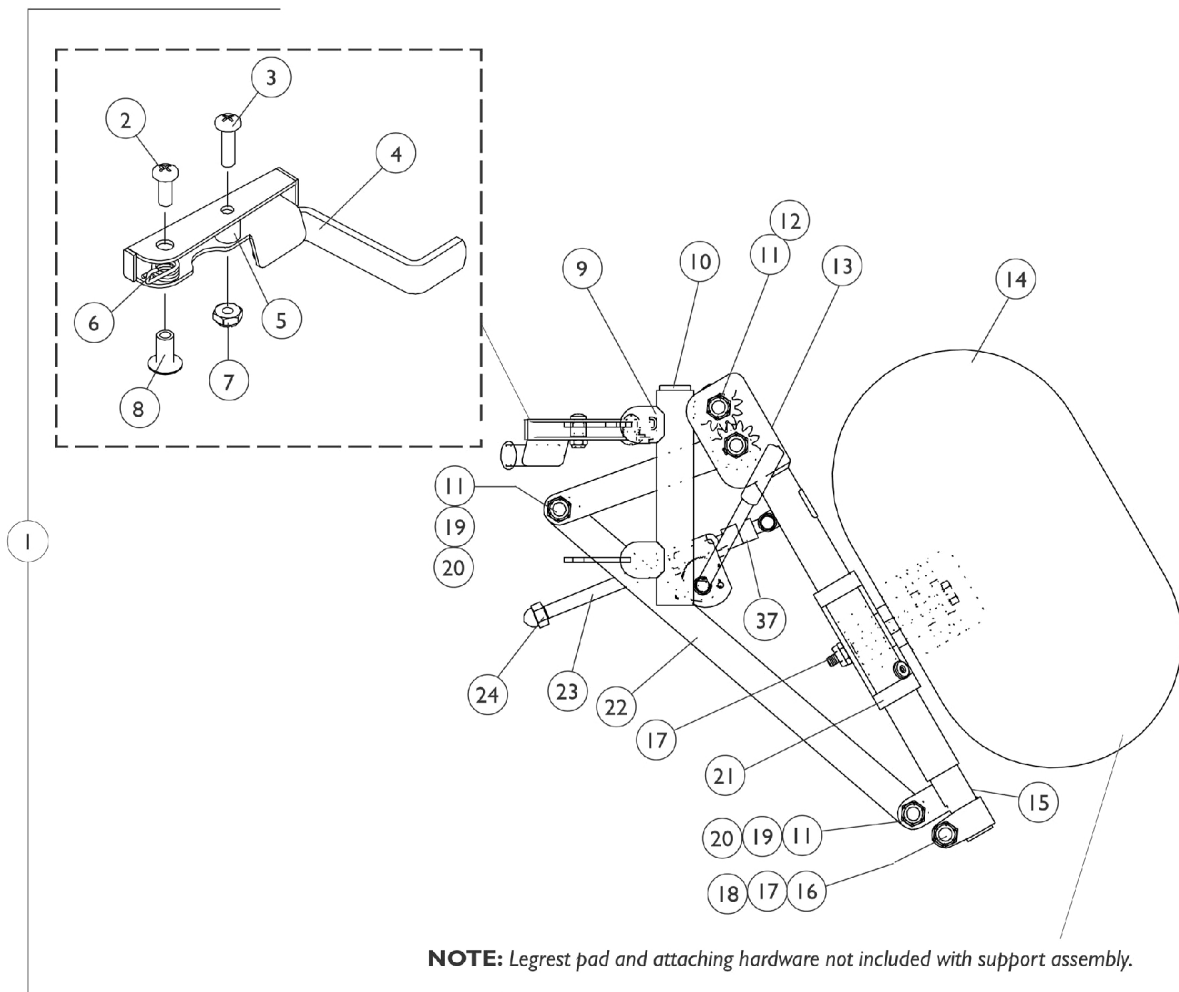

PAL4A Articulating Legrest

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1058482-NLA	NLA - Legrest Support Assembly, Articulating, Adult, Chrome - Right	1	1
1		1058483-NLA	NLA - Legrest Support Assembly, Articulating, Adult, Chrome - Left	1	1
2		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1035539	Handle, Release, Black - Right	1	1
4		1035540	Handle, Release, Black - Left	1	1
5		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
6		1028707	Spring, Release - Right	1	1
6		1028708	Spring, Release - Left	1	1
7		1025735	Locknut (#10-32)	1	1
8		1028846-NLA	NLA - Rivnut (#10-32)	1	1
9		1058478-NLA	NLA - Tube, Legrest Support - Right	1	1
9		1058479-NLA	NLA - Tube, Legrest Support - Left	1	1
10		1027097	Plug Button, Black (3/4")	1	1
11		40363X000	Hex Nut (3/8"-16)	1	3
12		1022854-NLA	NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)	1	1
13		1022877-NLA	NLA - Tube, Support and Drive Link, Chrome - Right	1	1
13		1022878-NLA	NLA - Tube, Support and Drive Link, Chrome - Left	1	1
14		8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
15		1022848	Tube, Slide, Articulating, Chrome - Right	1	1
15		1022849	Tube, Slide, Articulating, Chrome - Left	1	1
16		1027377	Washer (#20)	1	1
17		60264X001	Locknut (5/16-24)	1	1
18		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
19		1022866-NLA	NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	2
20		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
21		1025104	Bracket, Legrest, Chrome	1	1
22		1022846-NLA	NLA - Bar, Link, Black	1	1
23		1023811-NLA	NLA - Bar, Ratchet - Right	1	1
23		1023812-NLA	NLA - Bar, Ratchet - Left	1	1
24		10261X000	Nut, Acorn (5/16-24)	1	1
25		1040060	Tip, Plastic, Black	1	1
26		04528X002	Bracket, Release, Black	1	1
27		1037125	Cam, Legrest	1	1
28		1037485	Spring, Release	1	1
29		1037126	Handle, Release	1	1
30		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
31		1040831	Ring, Snap	1	1
32		48823X008	Spring, Leaf	1	1

PAL4A Articulating Legrest



Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
38	B	1046331-NLA	NLA - Kit, Articulating Legrest Ratchet Hardware (Pair)	2	1
33		1025133	Spacer, Legrest Pad, Black	1	1
34		04526X019	Plate, Legrest Pad	1	1
35		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
36		1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
37		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	2
38		1046331-NLA	NLA - Kit, Articulating Legrest Ratchet Hardware (Pair)	2	1
NOTE: A - Specify U67 black vinyl upholstery when ordering. B - Includes Screws and Instruction Sheet. C - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.					

Pivot Slide Tube and Footplates

T94AA/C and PAL4A

Heel loops are not included. See accessories page.

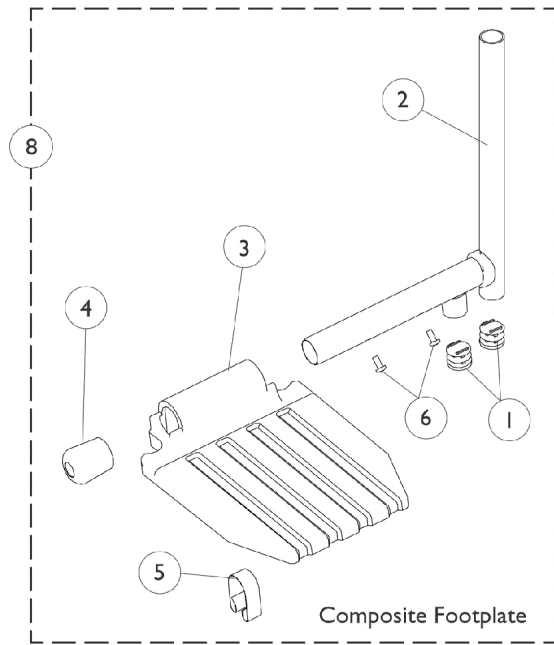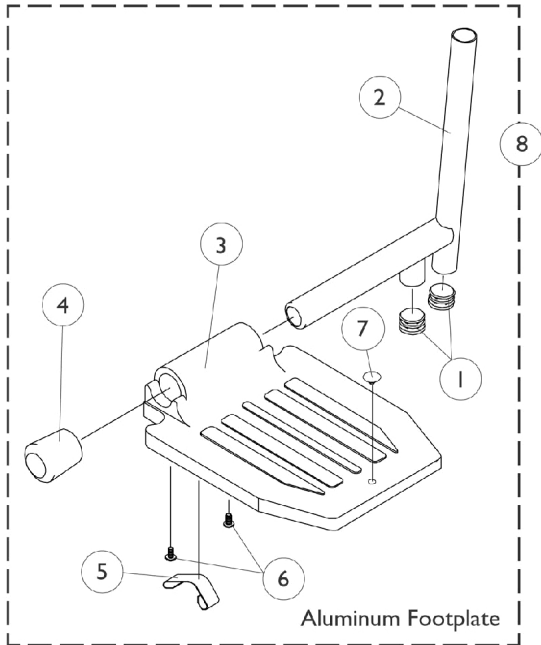


Pivot Slide Tube and Footplates T94AA/C and PAL4A

Heel loops are not included. See accessories page.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2	A	7500085-NLA	NLA - Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	D	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
3		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	C	1015178	Footplate, Composite, Black (6" W x 7-53/64" L)	1	1
4		1016346	Cap, End, Tube, Black (7/8")	1	1
5		1006981	Spring, Tension	1	1
6		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
7		1070168	Plug, Push, Footplate	1	1
8	B	1017237-NLA	NLA - Pivot/Slide Tube w/ Aluminium Footplate Assembly, Chrome - Right	1	1
8	B	1017238-NLA	NLA - Pivot/Slide Tube w/ Aluminium Footplate Assembly, Chrome - Left	1	1
8	C	1017233-NLA	NLA - Pivot/Slide Tube w/ Composite Footplate Assembly, Chrome - Right	1	1
8	C	1017234-NLA	NLA - Pivot/Slide Tube w/ Composite Footplate Assembly, Chrome - Left	1	1
8	D	1023907	Footplate w/Pivot Slide Tube - RH (18")	1	1
8	D	1023908	Footplate w/Pivot Slide Tube - LH (18")	1	1
NOTE: A - For T934A/C B - For T934A C - For T934C D - For PAL4A					

Miscellaneous Accessories

THE PRODUCTION OF THE P725 IS CURRENTLY SUSPENDED. CONTACT CUSTOMER SERVICE FOR ORDERING OPTIONS.

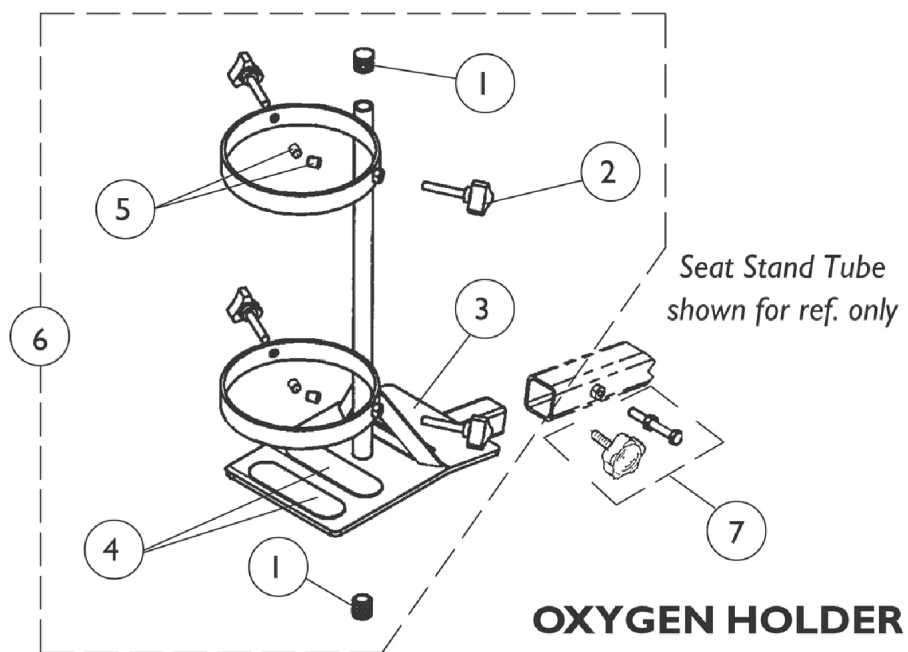

Miscellaneous Accessories

THE PRODUCTION OF THE P725 IS CURRENTLY SUSPENDED. CONTACT CUSTOMER SERVICE FOR ORDERING OPTIONS.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1035614	Plug Button, Black (5/8")	1	2
2		1027684	Knob, Clamping, Black	1	4
3		1080866	Decal, Oxygen Warning - English	1	1
		1090093-NLA	NLA - Decal, Oxygen Warning - French Canadian	1	1
		1092793-NLA	NLA - Decal, Oxygen Warning - Domestic Spanish	1	1
4		1072004	Pad, Non-Skid Adhesive Back	1	2
5	A	1086780-NLA	NLA - Package, Protective Cap	10	1
6	B	1091306-NLA	NLA - Oxygen Holder Assembly - Deluxe Seats	1	1
7		1088765-NLA	NLA - Fastener Hardware, Accessory Tube Includes: 1 ea. Tighten Knob 1 ea. Screw, Hex Cap (M10 x 25) 1 ea. Nut (M10 x 1.25 x 17)	1	1
	D	2000454-NLA	NLA - Bag, Poly (39" L x 33" W)	1	1
	D	1091305-NLA	NLA - Instruction Sheet, Oxygen Holder	1	1
	D	1083052-NLA	NLA - Carton, Shipper, Oxygen Holder	1	1
	D	P785-NLA	NLA - Basket, Rear - Lynx SX3P/ SX3 and LX3 Plus with Deluxe Seat	1	1
	C,D	P725	Kit, Crutch/Cane Holder	1	1
			Holder, Crutch/Cane - P725	1	1
			Cup, Crutch/Cane Holder, Black	1	1
			Strap, Hook and Loop - P725	1	1
			Plug Button, Black (7/8")	1	1
			Screw w/ Washer (#10 x 7/8")	1	2
	D	P710	Flag, Safety	1	1
	D	1035917-NLA	NLA - Heel Loop (Pair)	2	1
	D	1017621	Heel Loop - Right/Left	1	1
	D,E	1035907	Kit, Heel Loop Attaching Hardware (Pair)	2	1
NOTE: A - Four protective caps are included with the oxygen holder assembly, replacement caps are sold in packages of 10. B - Includes items 1-3, 5 and 6. Fastener hardware part number 1088765 is not included with the oxygen holder assembly. C - THE PRODUCTION OF THE P725 IS CURRENTLY SUSPENDED. CONTACT CUSTOMER SERVICE FOR ORDERING OPTIONS. D - Not Shown E - Consists of: 2 ea. Screw, Phillips Head (1/4-20 x 2-1/4") 2 ea. Spacer (1/4 x 11/32 x 1-3/4") 2 ea. Locknut (1/4-20)					

Power Wheelchair Tool Kit

I



**Multimeter
with Leads**



**Grounding
Wire**



**1" Test
Leads**



**Protractor/Angle
Locator**



**Invacare
Pen Light**



**Invacare
Tape Measure**



**Invacare
Phillips/Flat Head
Screwdriver**



MKIV Programmer



**Virtual Service
Charger Cable**

Power Wheelchair Tool Kit

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1125222-NLA	NLA - Kit, Power Wheelchair Tool Includes: 1 ea. Multimeter with Leads 1 ea. Locator, Protractor/Angle Magnetic Base 1 ea. Instructions, MKIV Programmer 1 ea. Wire, Grounding with Hand Strap and Clip 1 ea. Leads, 1" Test 1 ea. Light, Pen with Invacare Logo 1 ea. Tape Measure, Compact with Invacare Logo 1 ea. Screwdriver, Pen-Size Phillips/Flat Head 1 ea. Cable, Virtual Service Charger/Modem	1	1

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Customer Service
800-333-6900

©2022 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
(Revised 04/6/22)

Form No. 00-60