

Invacare 220 Scooter



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 04-044. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

©2018 Invacare Corporation, Elyria, Ohio. Invacare, Yes, you can. and the Medallion Design are registered trademarks of the Invacare Corporation.



TABLE of CONTENTS

Description	Page Number
Accessories - Front Basket	27
Arm - Arm Pad	10
Battery Box Kit	4
Caster - Front	12
Caster - Swing Arm	13
Electronics - Charger Kit	15
Electronics - Controller	21
Maintenance - Rear Preventative Kit	22
Maintenance - Tiller Kit - Lower	24
Maintenance - Tool Kit	26
Motor / Transaxle	6
Packaging & Literature	3
Shroud Kits	8
Tiller - Maintenance	18
Tiller - Tiller Assembly	16
Wheels - Drive (Rear)	20

Packaging & Literature

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		1128219-NLA	NLA - Kit, Packaging Includes: 10 ea. Foam Pieces, 2 ea. Poly Bags, 1 ea. Battery Carton, and 1 ea. Shipper Carton	1	1
		1129789-NLA	NLA - Kit, Packaging Padded Seat Includes: 1 ea. Insert, 3 ea. Foam Block, 1 ea. Poly Bag, and 1 ea. Shipper Carton	1	1
		1129790-NLA	NLA - Kit, Packaging Deluxe Seat Includes: 1 ea. Insert, 2 ea. Foam Blocks, 2 ea. Foam Rings, 2 ea. Bubble Wraps, 1 ea. Seat Post Carton, 1 ea. Poly Bag, and 1 ea. Shipper Carton	1	1
		1125053-NLA	NLA - Owner's Manual - Invacare 220 Scooter	1	1


Battery Box Kit

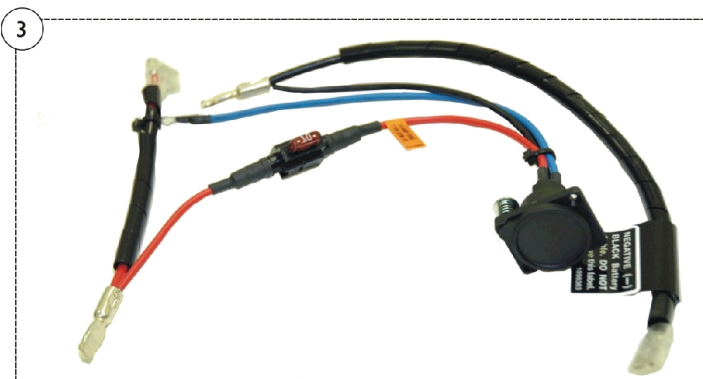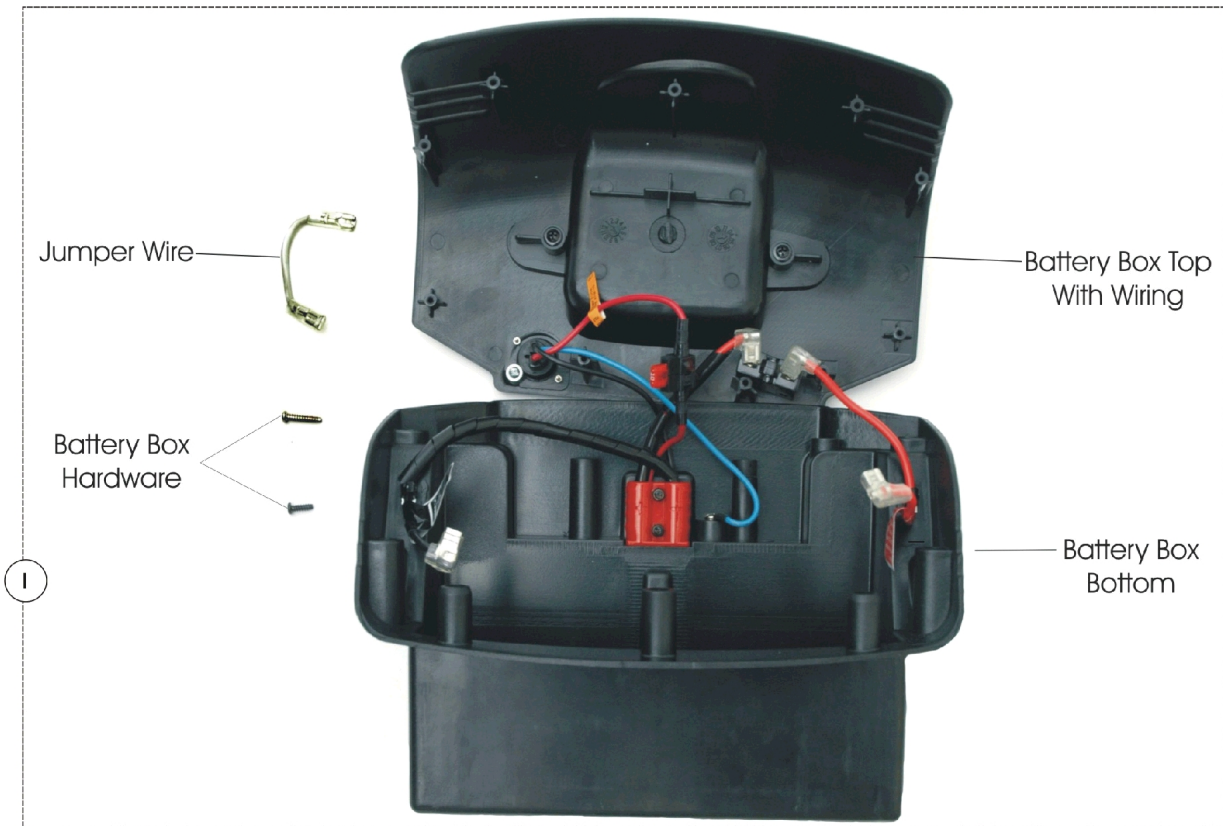

Battery Box Kit

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1128213-NLA	NLA - Kit, Battery Box Includes: 1 ea. Battery Box Top with Wiring, 1 ea. Battery Box Bottom, 1 ea. Battery Box Hardware, and 1 ea. Jumper Wire	1	1
2		1132327-NLA	NLA - Circuit Breakers	10	1
3		1132328-NLA	NLA - Harness, Charger Port with Cover	1	1
	A	12AH-2	Batteries, 12 volt -12 Amp Hour (Pair)	2	1
NOTE: A - Not Shown					

Motor and Transaxle

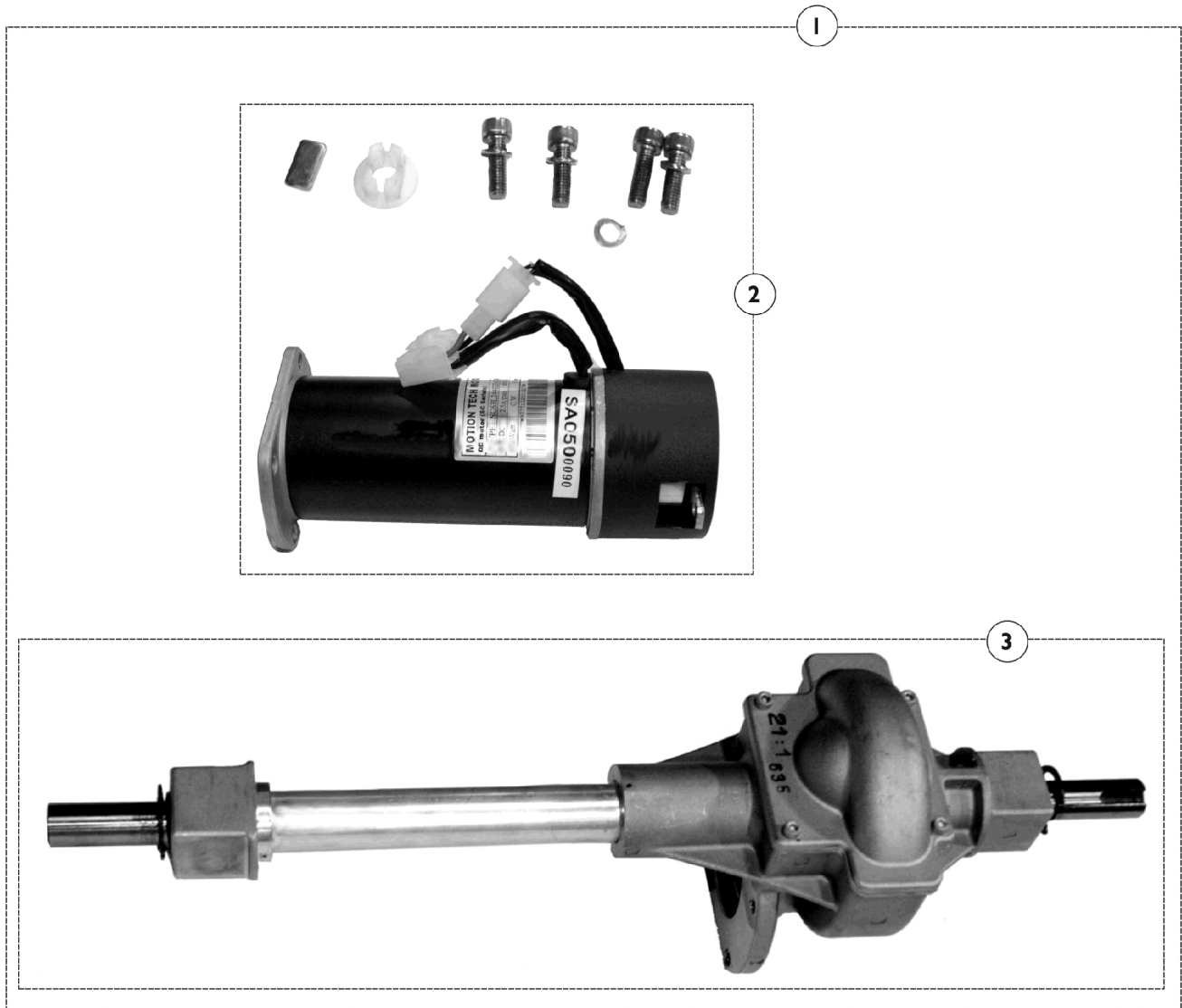

Motor and Transaxle

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1132324		1	1
NOTE: A - These parts cannot be purchased seperately.					

Shroud Kits



Shroud Kits

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1128210	Kit, Shroud Group with Hardware - Red	1	1
2			NLA - Kit, Foot Panel and Wheel Cover, Red	1	1
3			NLA - Kit, Footboard and Mat, Red	1	1
4			NLA - Kit, Rear Shroud and Handle with Hardware - Red	1	1
1	A	1128211	Kit, Shroud Group with Hardware - Blue	1	1
2			NLA - Kit, Foot Panel and Wheel Cover, Blue	1	1
3			NLA - Kit, Footboard and Mat, Blue	1	1
4			NLA - Kit, Rear Shroud and Handle with Hardware - Blue	1	1
5		1132317-NLA	NLA - Kit, Shroud Hardware	1	1
NOTE: A - Includes: Items 2-4					

Arm Pad



Arm Pad

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1128216-NLA	NLA - Kit, Arm Pad Includes: 2 ea. Arm Pads	1	1
2		1132326-NLA	NLA - Bolt, Seat/Arm with Washer (Pkg of 10)	10	1

Front Caster



Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		1128208-NLA	NLA - Kit, Front Caster Wheel Includes: 1 ea. Flat Free Wheel with Hub and Bearings, 1 ea. Wheel Axle (Hex Head), 2 ea. Washers, and 1 ea. Axle Nut	1	1

Swing Arm Caster



Swing Arm Caster

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1128214-NLA	NLA - Kit, Swing Arm Caster Includes: 2 ea. Solid Rear Wheel Casters with Bearings, 2 ea. Rear Caster Forks with Nuts, 4 ea. Head Tube Bearings, 2 ea. Head Tube Spacers, 2 ea. Axle Bolts with Nuts, 4 ea. Rear Caster Spacers, and 2 ea. Head Tube Caps	1	1
2		1132329-NLA	NLA - Kit, Caps, End, Black, Head Tube	10	1

Charger Kit



Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		1128217-NLA	NLA - Kit, Charger with Bag - Deluxe Seat Includes: 1 ea. Off Board Charger and 1 ea. Bag	1	1
		1130762	Kit, Charger with Bag - Standard Seat Includes: 1 ea. Off Board Charger and 1 ea. Bag	1	1

Tiller Assembly

Individual components are not sold separately unless indicated with a part number.



Tiller Assembly

Individual components are not sold separately unless indicated with a part number.


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1132311-NLA	NLA - Package, Foam Hand Grip	10	1
2		1132310-NLA	NLA - Package, Throttle Lever, Black	10	1
3-14		See Chart	Tiller Assembly	1	1
15		1132307-NLA	NLA - Ignition w/ Keys Assembly	1	1
16		1132312-NLA	NLA - Horn Assembly with Button	1	1
17		1132313-NLA	NLA - Cover, Speed Pot, Red	10	1
17		1132657-NLA	NLA - Cover, Speed Pot, Blue	10	1
18		1141002	Throttle Assembly	1	1

Item Number	Part Number	Description	Quantity Per Kit
3	1128206-NLA See Note(s) A	NLA - Kit, Full Tiller Assembly - Red	1
3	1128207-NLA See Note(s) B, A	NLA - Kit, Full Tiller Assembly - Blue	1
4	N/A	Foam Hand Grips	2
5	N/A	Speed Pot Cover (See Item #17 for Pkg of 10 in Red or Blue)	1
6	1126100	Key, Ignition	1
7	N/A	Plug Button for Hand Grip Ends	2
8	N/A	Throttle Lever Tip (See item #2 for Pkg of 10)	2
9	N/A	Complete Tiller Head - Red	1
9	N/A	Complete Tiller Head - Blue	1
10	N/A	Invacare Decal	1
11	N/A	HMV Decal	1
12	N/A	Screws, Tiller Head	2
13	N/A	Nuts, Tiller Head	2
14	1132309-NLA	NLA - Knob, Tiller Adjustment, Red	2
14	1132629-NLA	NLA - Package, Tiller Adjustment Knobs, Blue	2

NOTE: A - Includes: Items 4-14
B - No Longer Available.

Tiller Maintenance

Individual Componentets are not sold seperately unless indicated with a part number.

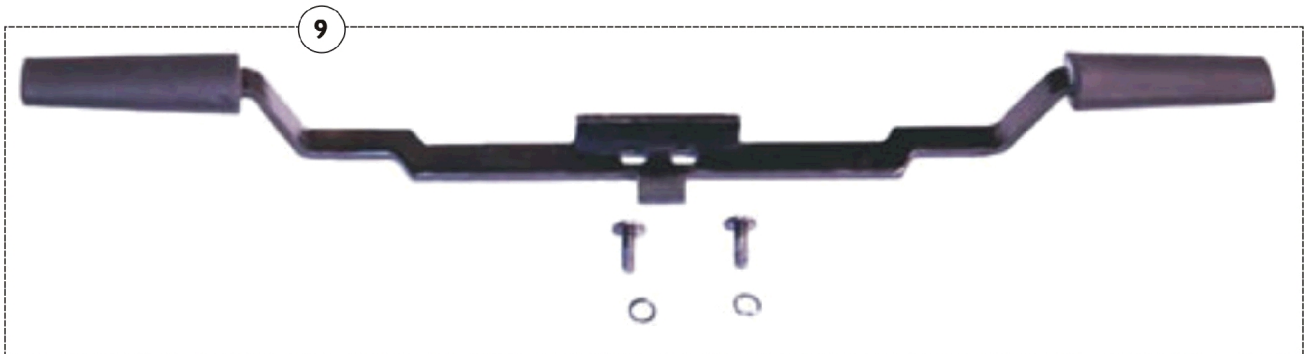


Tiller Maintenance

Individual Components are not sold separately unless indicated with a part number.

Item Number	Part Number	Description	Quantity Per Kit
1	1128205-NLA See Note(s) A	NLA - Kit, Tiller Maintenance Preventative Hardware - Red	1
1	1129787-NLA See Note(s) A	NLA - Kit, Tiller Maintenance Preventative Hardware - Blue	1
2	N/A See Note(s) B	Foam Hand Grips	2
3	1132312-NLA	NLA - Horn Assembly with Button	1
4	N/A See Note(s) B	Speed Pot Cover (Red or Blue)	1
5	N/A	Plug Buttons	2
6	N/A See Note(s) B	Throttle Lever Tips	2
7	1126100	Key, Ignition	2
8	1132309-NLA	NLA - Knob, Tiller Adjustment, Red	2
8	1132629-NLA	NLA - Package, Tiller Adjustment Knobs, Blue	2
9	1136805-NLA	Trottle Lever Assembly	1
NOTE: A - Includes: Items 2-8 B - To Order part available in quantities see section titled "Tiller Assembly".			

Drive Wheel - Rear

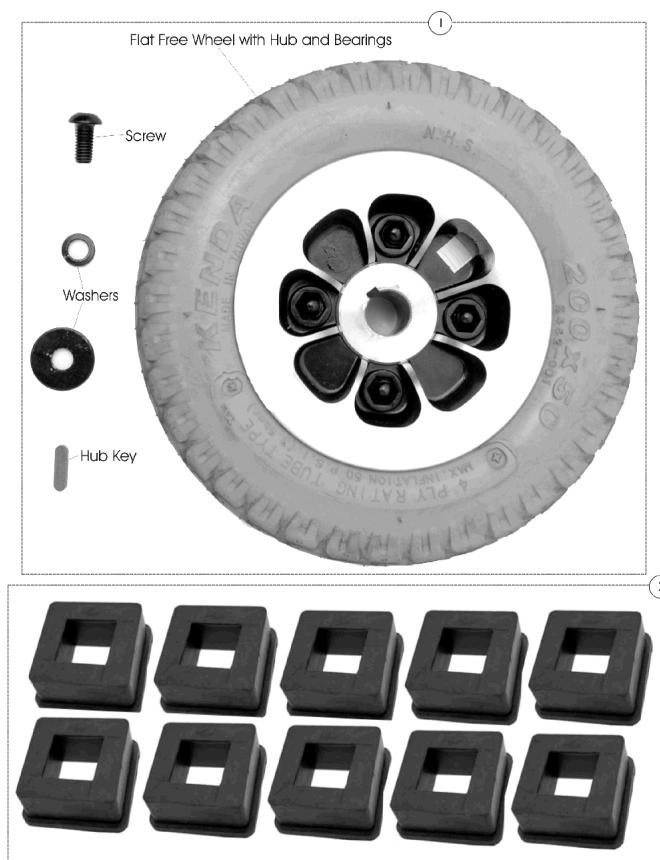

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1128209-NLA	NLA - Kit, Drive Wheel Includes: 1 ea. Flat Free Wheel with Hub and Bearings 1 ea. Hub Key, 1 ea. Screw, and 2 ea. Washers	1	1
2		1132325-NLA	NLA - Element, Rubber	10	1




Controller



Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1128292-NLA	NLA - Controller, Rhino (50 AMP)	1	1

NOTE: A - As of 12/8/06 the Controller has a bare metal finish. The function of the Controller remains the same.



Rear Maintenance Preventative Kit

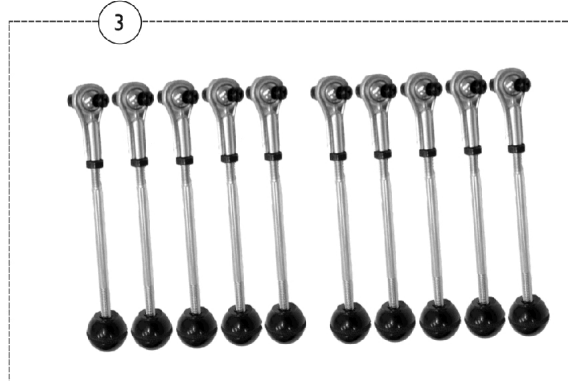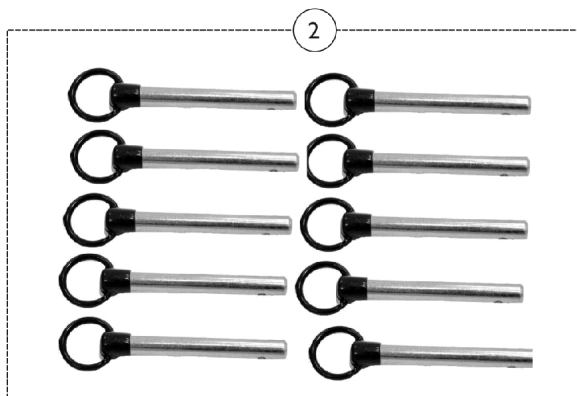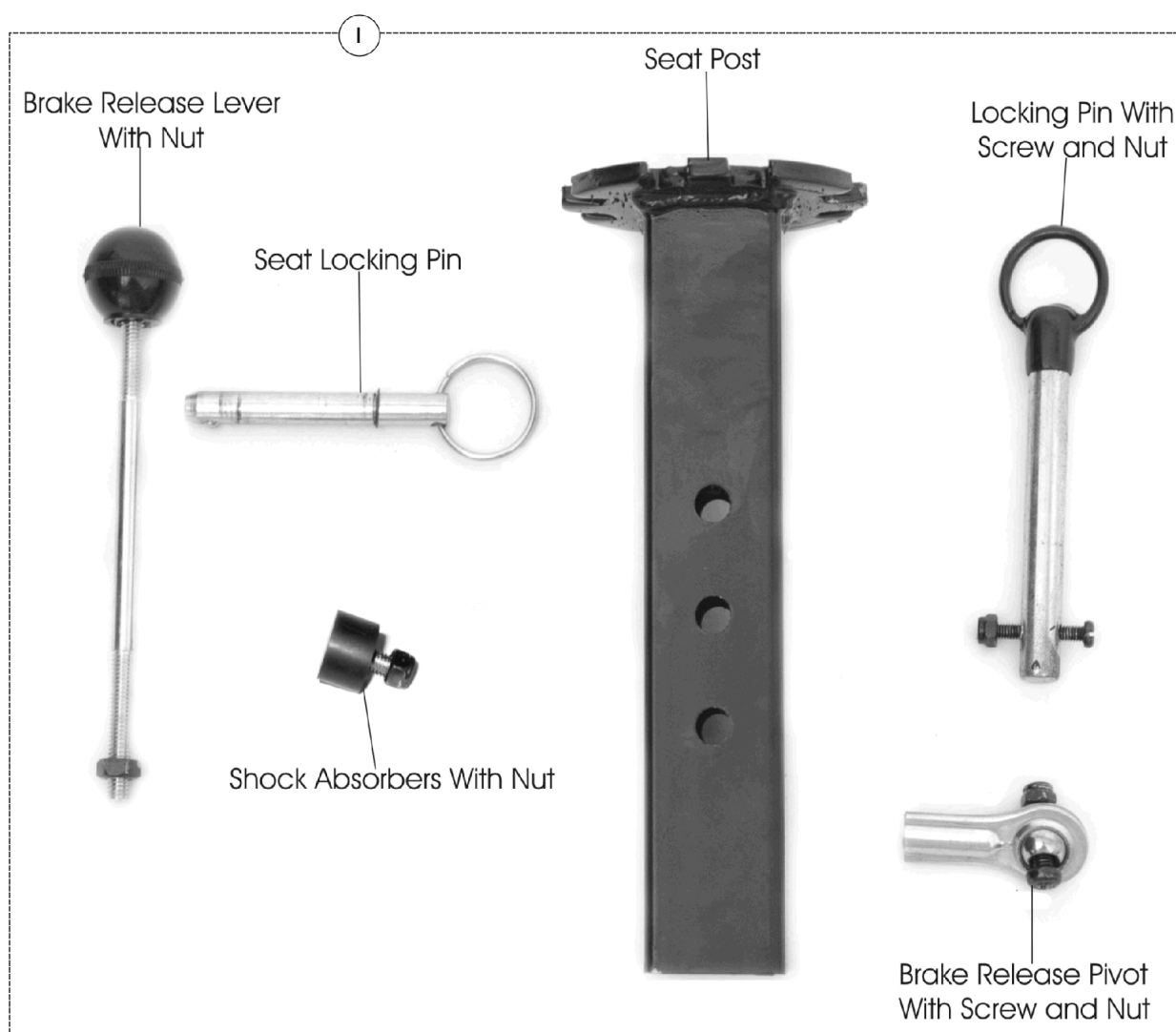

Rear Maintenance Preventative Kit

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1128212-NLA	NLA - Kit, Rear Preventative Maintenance Includes: 1 ea. Locking Pin with Screw & Nut, 1 ea. Seat Post, 2 ea. Seat Post Locking Pins, 2 ea. Brake Release Lever with Nut, 1 ea. Brake Release Pivot with Screw and Nut, and 4 ea. Shock Absorber with Nut	1	1
2		1132322-NLA	NLA - Pin, Seat Post Locking (Pkg of 10)	10	1
3		1132323-NLA	NLA - Package, Brake Release Lever w/ Nut	10	1

Tiller Maintenance Kit - Lower

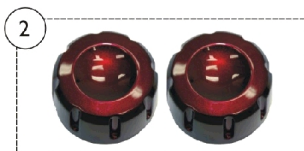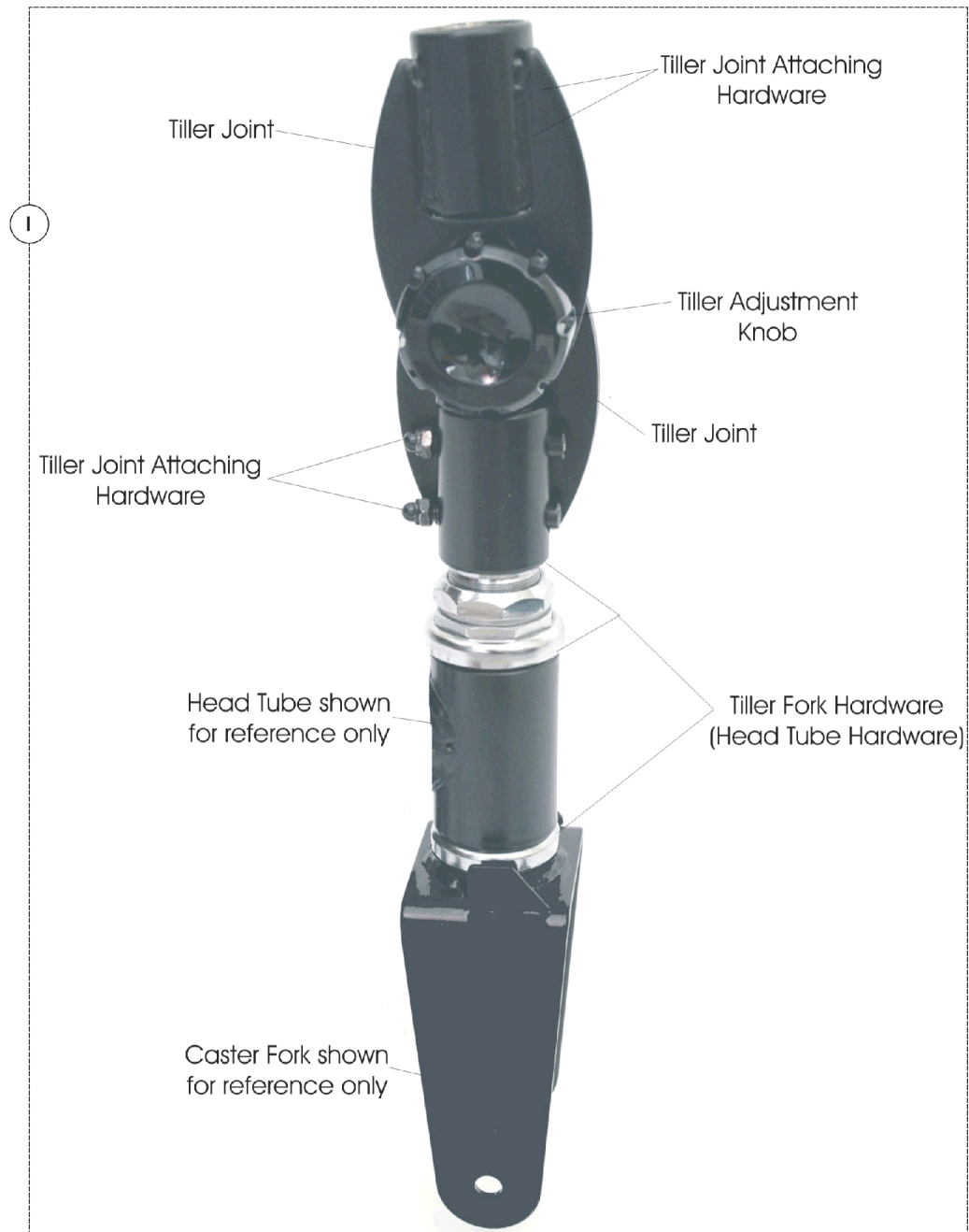

Tiller Maintenance Kit - Lower


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1128215-NLA	NLA - Kit, Lower Tiller Maintenance - Red Includes: 1 ea. Tiller Joint, 1 ea. Tiller Joint Attaching Hardware, 1 ea. Tiller Fork Hardware Group, and 2 ea. Tiller Adjustment Angle Knobs - Red	1	1
1		1129788-NLA	NLA - Kit, Lower Tiller Maintenance - Blue Includes: 1 ea. Tiller Joint, 1 ea. Tiller Joint Attaching Hardware, 1 ea. Tiller Fork Hardware Group, and 2 ea. Tiller Adjustment Angle Knobs - Blue	1	1
2		1132309-NLA	NLA - Knob, Tiller Adjustment, Red	1	1
2		1132629-NLA	NLA - Package, Tiller Adjustment Knobs, Blue	1	1

Tool Kit



Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1125223-NLA	NLA - Kit, Scooter Tool	1	1
			Strap, Ground with Wire and Clip	1	1
			Programmer, Remote (DS2K-PD)	1	1
			Leads, Test (1")	1	1
			Multimeter with Leads	1	1
			Screwdriver, Pen-Size	1	1
			Light, Pen-Size with Invacare Logo	1	1
			Bag, Zip-Lip	1	1
			Tape Measure, Compact	1	1

Front Basket

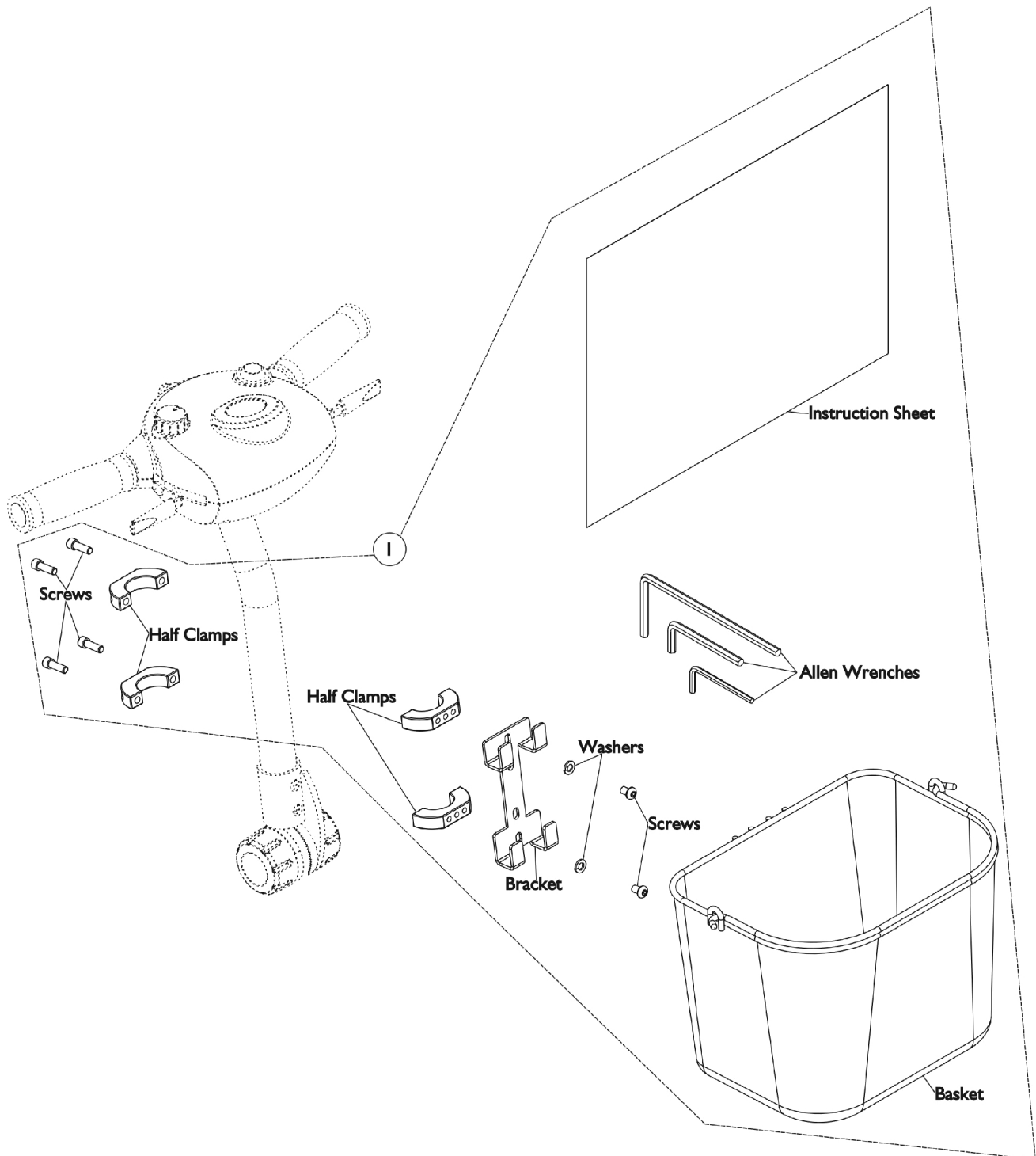

Front Basket

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		TBAS-NLA	Kit, Basket Includes: 1 ea. Instruction Sheet, 1 ea. Basket, 4 ea. Half Clamps, 1 ea. Bracket, 6 ea. Screws, 2 ea. Washers, and 3 ea. Allen Wrenches	1	1

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Customer Service
800-333-6900

©2018 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
(Revised 1/4/18)

Form No. 04-044