

HealthCraft Service Parts



Yes, you can.

Parts Catalogue Usage Guide

The information contained in this document is subject to change without notice. This catalogue contains service parts for Form No. 08-664C. If you have any questions regarding this catalogue, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-668-5324 or 1-905-890-8300

For Faxed In Orders Call:

1-800-668-5478 or

1-905-890-5244

For Technical Service Call:

1-800-668-5324 Ext. 2655

For Orders:

canadaorders@invacare.com

Literature: www.invacare.ca or 1-800-668-5324

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers


© 2020 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are registered trademarks of the Invacare Corporation. Information is subject to change without notification.

TABLE of CONTENTS


Description	Page Number
Advantage Pole	21
Advantage Rail	19
Assista-Rail	33
Assista-Rail with Assista-Cane	35
Dependa-Bar	29
Horizontal and Parallel Bar Systems	27
ModBar	39
ModPole	41
ModRail	37
PT Rail and Mast	31
Smart Rail	23
Smart Rail Hospital	25
SuperPole for Angled Ceilings	7
SuperPole G1	5
SuperPole with Safety Island / SuperPole with Safety Island Bariatric	13
SuperPole with SuperBar / SuperPole with SuperBar Heavy Duty	9
SuperPole with SuperTrapeze	11
SuperPole with SuperTray / SuperPole with SuperTray Heavy Duty	15
SuperPole, SuperPole Heavy Duty, SuperPole Ultra, SuperPole Stainless Steel	3
Uni-Fit Extender	17

SuperPole / SuperPole Heavy Duty / SuperPole Ultra / SuperPole Stainless Steel


**PARTS DIAGRAM: SUPERPOLE
/SUPERPOLE HEAVY DUTY**

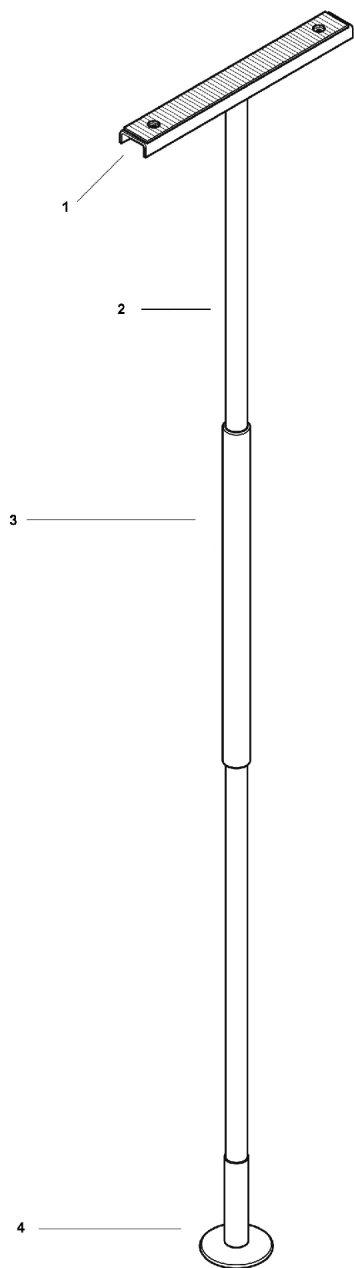

**PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE
ASSEMBLY**

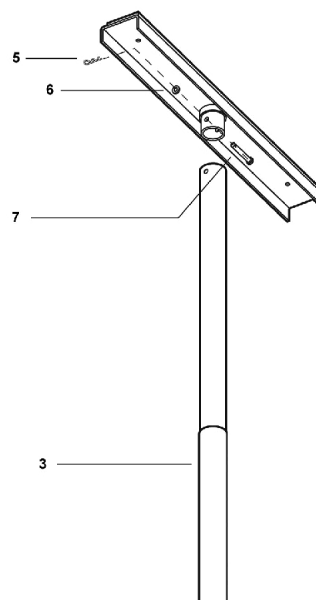

**PARTS DIAGRAM: POLE / EXTENSION SCREW /
BASE PLATE ASSEMBLY**

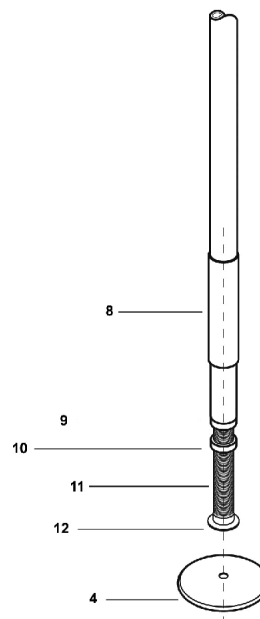


SuperPole / SuperPole Heavy Duty / SuperPole Ultra / SuperPole Stainless Steel


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	SP403	Ceiling Anchor Plate Assembly	1	1
1		SP403H DU	Top Plate, Ultra	1	1
1		SP403HD	Ceiling Anchor Plate Assembly, Heavy Duty	1	1
1		SP210	Top Plate, Stainless Steel	1	1
2		SP-P	Pole Assembly	1	1
2		SP-PHD	Pole Assembly, Heavy Duty	1	1
2		SP-PU	Pole Assembly, Ultra	1	1
2		SP-PHD-SS	Pole, Stainless Steel	1	1
3		GR-101	Grip, 20"	1	1
4		SP402	Base Plate Assembly	1	1
4		SP212-NLA	NLA - Base Plate Assembly, Stainless Steel	1	1
5		HW323	Hitch Pin	1	1
6		HW318	Washer, Nylon	1	1
7		HW302	Clevis Pin	1	1
8		SP401	Cover Sleeve, Plastic	1	1
8		SP207	Cover, Sleeve, Stainless Steel	1	1
9		SP105	Threaded Collar	1	1
9		SP105HD	Threaded Collar, Heavy Duty	1	1
9		SP105H DU	Threaded Collar, Ultra	1	1
10		SP172	Jam Nut	1	1
11		SP107	Extension Screw	1	1
12		SP145	Washer, Plastic	1	1
NOTE: A - Not available on the Stainless Steel SuperPole.					

SuperPole G1


PARTS DIAGRAM: SUPERPOLE G1

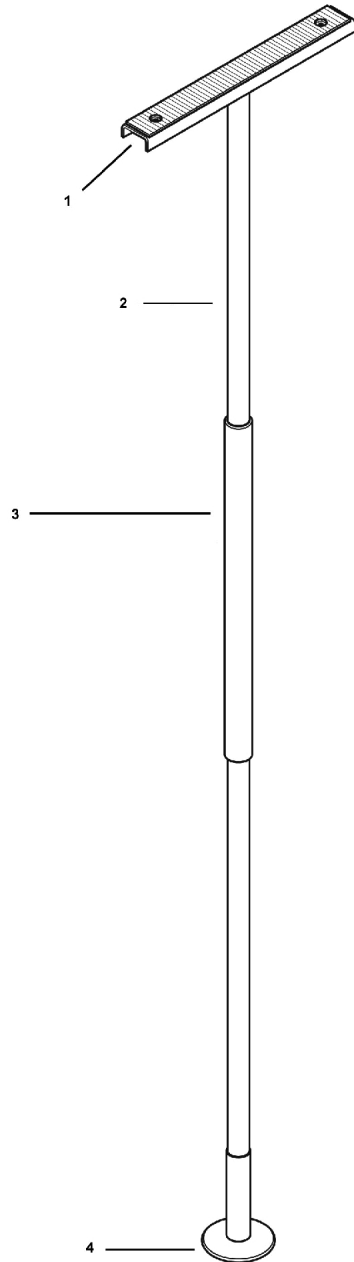

PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE ASSEMBLY

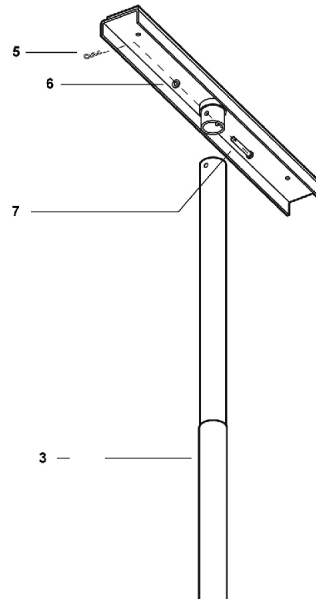

PARTS DIAGRAM: POLE / EXTENSION SCREW / BASE PLATE ASSEMBLY

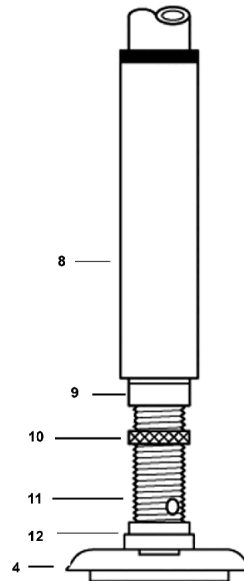


SuperPole G1

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
2		SP-P	Pole Assembly	1	1
3		GR-101	Grip, 20"	1	1
4		SP402G1	NLA - Base Plate Assembly	1	1
5		HW323	Hitch Pin	1	1
6		HW318	Washer, Nylon	1	1
7		HW302	Clevis Pin	1	1
8		SP401G1	NLA - Assembly, Cover Sleeve w/ Label	1	1
8		SP420G1	NLA - Assembly, Cover Sleeve and Bottom Bushing	--	--
9		SP105	Threaded Collar	1	1
10		SP172	Jam Nut	1	1
11		SP107G1	NLA - Extension Screw, G1	1	1
11		SP410G1	NLA - Assembly, Extension Screw w/ Jam Nut	1	1
12		SP419G1	NLA - Assembly, Bottom Bushing w/ Washer	1	1
Not Shown		SP-BXG1	NLA - Parts Box, SuperPole	1	1
Not Shown		SP500G1	NLA - Parts Pack, SuperPole	1	1
	A	SP421	SuperPole Conversion Kit	1	1
NOTE: A - Converts the Generation 1 SuperPole to a Generation 2 SuperPole. Conversion will now support SuperPole G2 Parts.					

SuperPole for Angled Ceilings

PARTS DIAGRAM: SUPERPOLE FOR ANGLED CEILINGS

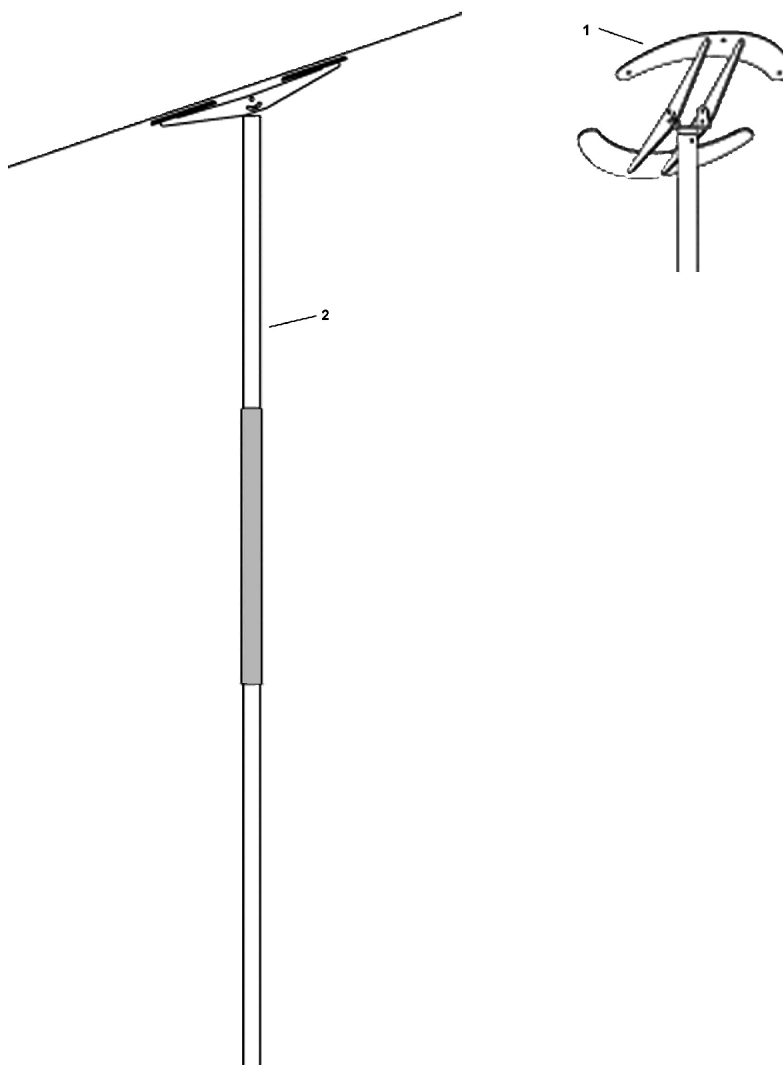


SuperPole for Angled Ceilings


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP198	Assembly, Angled Ceiling Plate w/ Bracket	1	1
2		SP-P	Pole Assembly	1	1
Not Shown		HW302	Clevis Pin	1	1
Not Shown		HW323	Hitch Pin	1	1
Not Shown		SP-ACP-BX-NLA	NLA - Parts Box, Angled Ceiling Plate	1	1
Not Shown		SP513	Parts Pack, Angled Ceiling Plate	1	1

SuperPole with SuperBar / SuperPole with SuperBar Heavy Duty


PARTS DIAGRAM: SUPERPOLE™ with SUPERBAR™

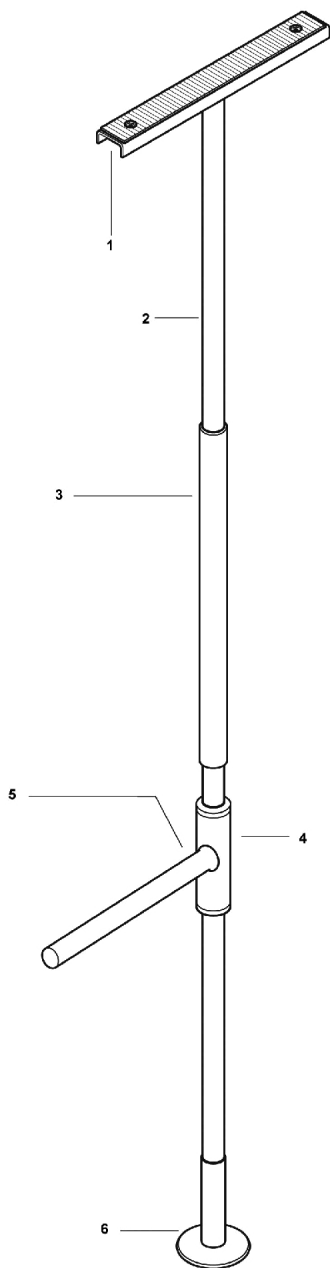


PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE ASSEMBLY

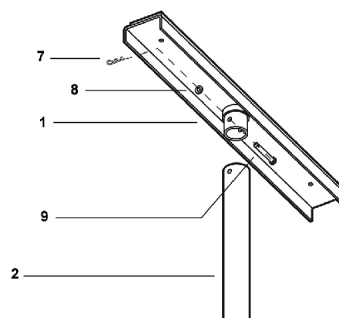

PARTS DIAGRAM: SUPERBAR

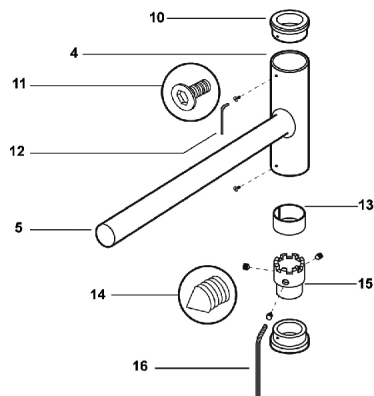

PARTS DIAGRAM: POLE / EXTENSION SCREW / BASE PLATE ASSEMBLY

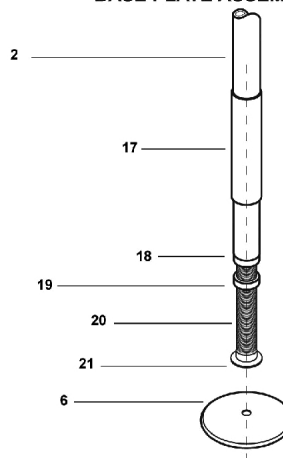


SuperPole with SuperBar / SuperPole with SuperBar Heavy Duty

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
1		SP403H DU	Top Plate, Ultra	1	1
1		SP403HD	Ceiling Anchor Plate Assembly, Heavy Duty	1	1
2		SP-P	Pole Assembly	1	1
2		SP-PHD	Pole Assembly, Heavy Duty	1	1
3	A	GR-101	Grip, 20"	1	1
4		SP404	SuperBar w/o the Castellated Collar	1	1
5		GR-100	Grip, 16"	1	1
6		SP402	Base Plate Assembly	1	1
7		HW323	Hitch Pin	1	1
8		HW318	Washer, Nylon	1	1
9		HW302	Clevis Pin	1	1
10		SP102	Pivot Bushing	1	2
11		HW100	Screw	1	2
12		HW300	Allen Key, Short Arm (3/32")	1	1
13		SP170	Spring Ring	1	1
14		HW112	Set Screws	1	4
15		SP411	Castellated Collar	1	2
16		HW320	Allen Key, Long Arm (5/32")	1	1
17		SP401	Cover Sleeve, Plastic	1	1
18		SP105	Threaded Collar	1	1
18		SP105HD	Threaded Collar, Heavy Duty	1	1
19		SP172	Jam Nut	1	1
20		SP107	Extension Screw	1	1
21		SP145	Washer, Plastic	1	1
Not Shown		SP-BX	Parts Box, SuperPole	1	1
Not Shown		SPB-BX	Parts Box, SuperBar	1	1
Not Shown		SPB-BXHD	Parts Box, SuperPole, Heavy Duty	1	1
Not Shown		SP501	Parts Pack, Super Bar	1	1


NOTE: A - Not available on Stainless Steel pole.

SuperPole with SuperTrapeze


PARTS DIAGRAM: SUPERPOLE™ with SUPERTRAPEZE™

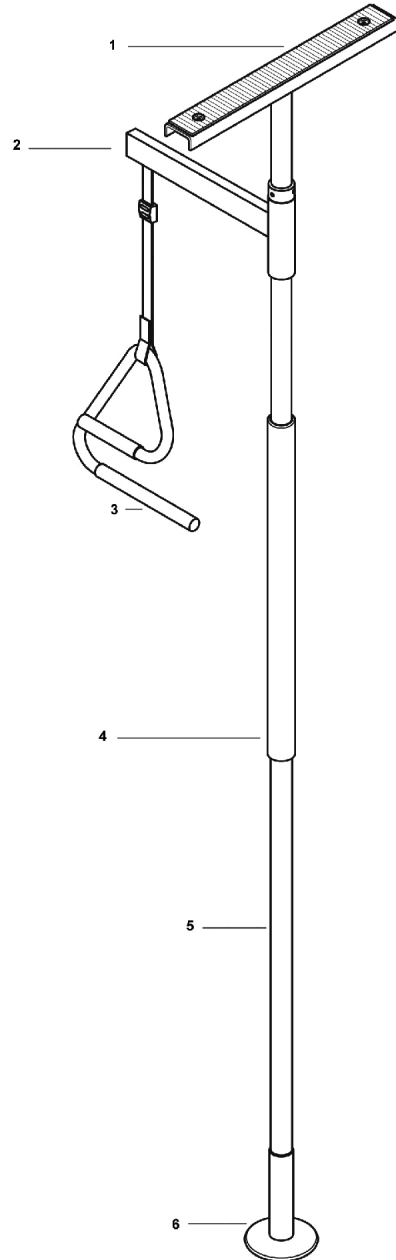


PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE ASSEMBLY

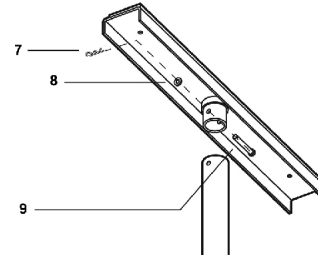

PARTS DIAGRAM: SUPERTRAPEZE™

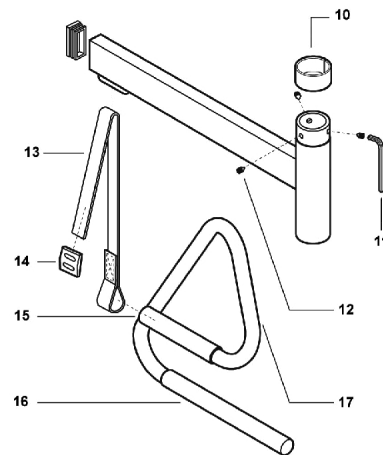

PARTS DIAGRAM: POLE / EXTENSION SCREW / BASE PLATE ASSEMBLY

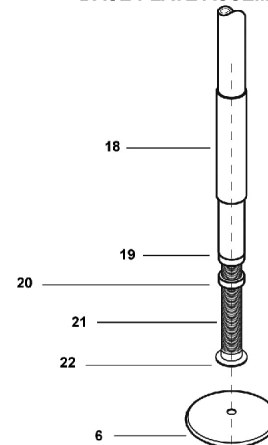


SuperPole with SuperTrapeze


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
2		SP408	Trapeze Arm w/ Strap and Buckle	1	1
3		STP-HS	Trapeze Handle w/ Strap	1	1
4		GR-101	Grip, 20"	1	1
5		SP-P	Pole Assembly	1	1
6		SP402	Base Plate Assembly	1	1
7		HW323	Hitch Pin	1	1
8		HW318	Washer, Nylon	1	1
9		HW302	Clevis Pin	1	1
10		SP170	Spring Ring	1	1
11		HW320	Allen Key, Long Arm (5/32")	1	1
12		HW112	Set Screws	1	3
13		SP125A	Strap	1	1
14		HW305	Buckle	1	1
15		GR-102	Grip, 6"	1	1
16		GR103	Grip, 10"	1	1
17		SP409	Trapeze Handle	1	1
18		SP401	Cover Sleeve, Plastic	1	1
19		SP105	Threaded Collar	1	1
19		SP105HD	Threaded Collar, Heavy Duty	1	1
20		SP172	Jam Nut	1	1
21		SP107	Extension Screw	1	1
22		SP145	Washer, Plastic	1	1
Not Shown		SP-BX	Parts Box, SuperPole	1	1
Not Shown		STP-BX	SuperTrapeze Parts Box	1	1
Not Shown		SP505	SuperTrapeze Parts Pack	1	1

SuperPole with Safety Island / SuperPole with Safety Island Bariatric


PARTS DIAGRAM: SUPERPOLE™ with SAFETY ISLAND

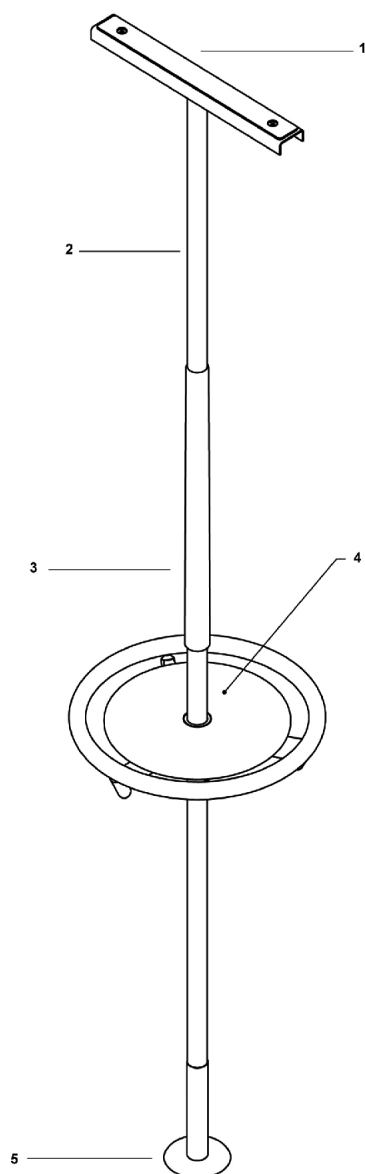


PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE ASSEMBLY

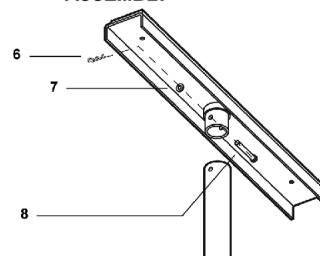

PARTS DIAGRAM: SAFETY ISLAND

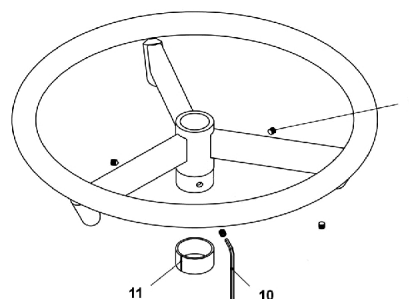

PARTS DIAGRAM: POLE / EXTENSION SCREW / BASE PLATE ASSEMBLY

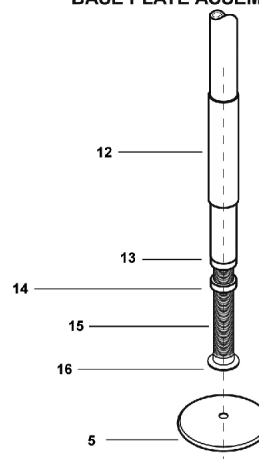


SuperPole with Safety Island / SuperPole with Safety Island Bariatric


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
1		SP403HD	Ceiling Anchor Plate Assembly, Heavy Duty	1	1
2		SP-P	Pole Assembly	1	1
2		SP-PHD	Pole Assembly, Heavy Duty	1	1
3		GR-101	Grip, 20"	1	1
4		SI121	Safety Island Assembly	1	1
5		SP402	Base Plate Assembly	1	1
6		HW323	Hitch Pin	1	1
7		HW318	Washer, Nylon	1	1
8		HW302	Clevis Pin	1	1
9		HW112	Set Screws	1	3
10		HW320	Allen Key, Long Arm (5/32")	1	1
11		SP170	Spring Ring	1	1
12		SP401	Cover Sleeve, Plastic	1	1
13		SP105	Threaded Collar	1	1
13		SP105HD	Threaded Collar, Heavy Duty	1	1
14		SP172	Jam Nut	1	1
15		SP107	Extension Screw	1	1
16		SP145	Washer, Plastic	1	1
Not Shown		SP-BX	Parts Box, SuperPole	1	1
Not Shown		SP-STI-BX	Parts Box, Safety Island	1	1
Not Shown		SP-BXHD	Parts Box, SuperPole Heavy Duty	1	1
Not Shown		SP-STI-BXHD	Parts Box, Safety Island Heavy Duty	1	1
Not Shown		SI500	Parts Pack, Safety Island	1	1
Not Shown		SP-500	Parts Pack, SuperPole	--	--

SuperPole with SuperTray / SuperPole with SuperTray Heavy Duty


PARTS DIAGRAM: SUPERPOLE™ with SUPERTRAY

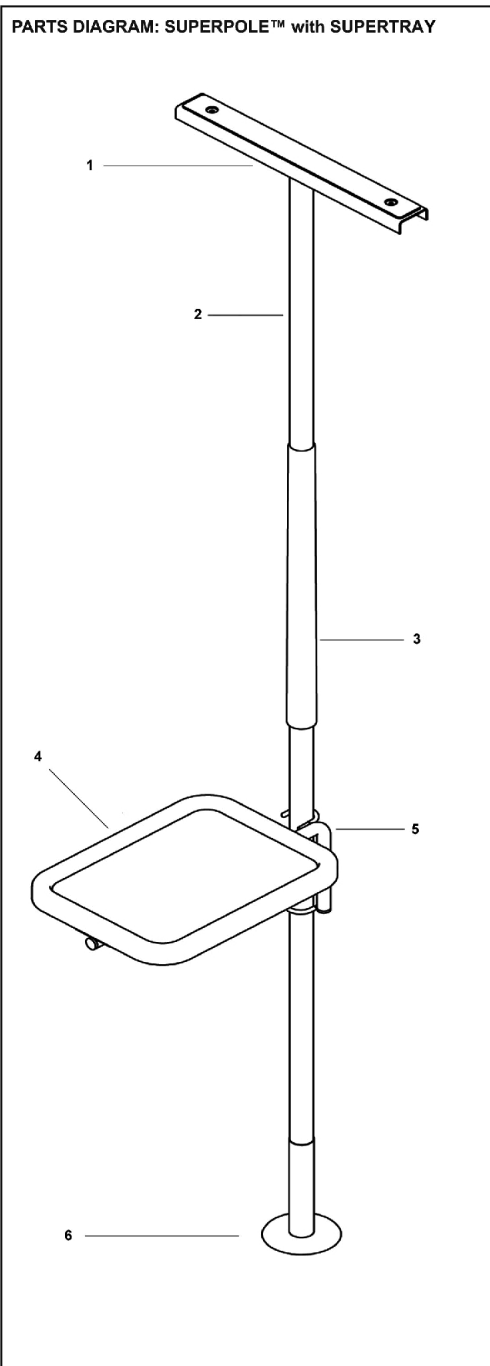


PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE ASSEMBLY

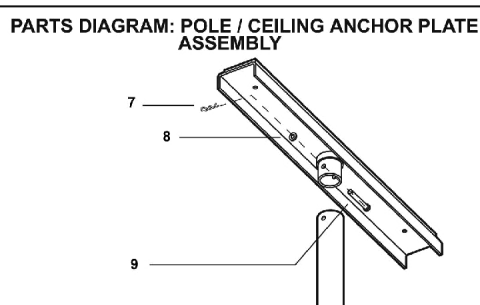

PARTS DIAGRAM: SUPERTRAY

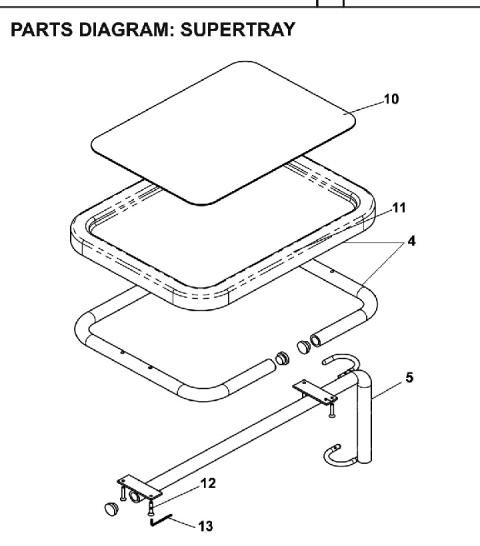

PARTS DIAGRAM: POLE / EXTENSION SCREW / BASE PLATE ASSEMBLY

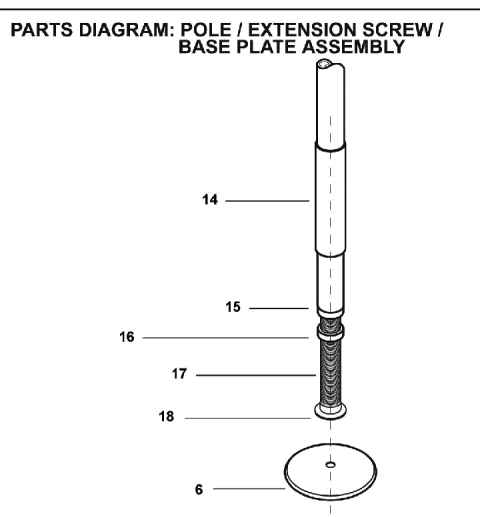


SuperPole with SuperTray / SuperPole with SuperTray Heavy Duty


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
1		SP403HD	Ceiling Anchor Plate Assembly, Heavy Duty	1	1
2		SP-P	Pole Assembly	1	1
2		SP-PHD	Pole Assembly, Heavy Duty	1	1
3		GR-101	Grip, 20"	1	1
4		SP406	Tray Frame	1	1
5		SP407	Tray Arm Assembly	1	1
6		SP402	Base Plate Assembly	1	1
7		HW323	Hitch Pin	1	1
8		HW318	Washer, Nylon	1	1
9		HW302	Clevis Pin	1	1
10		SP136	Rubber Tray Mat	1	1
11		SP405	Tray	1	1
12		HW102	Flat Screws	1	4
13		HW301	Hex Key, Short Arm (1/8")	1	1
14		SP401	Cover Sleeve, Plastic	1	1
15		SP105	Threaded Collar	1	1
15		SP105HD	Threaded Collar, Heavy Duty	1	1
16		SP172	Jam Nut	1	1
17		SP107	Extension Screw	1	1
18		SP145	Washer, Plastic	1	1
Not Shown		SP-BX	Parts Box, SuperPole	1	1
Not Shown		SPT-BX	SuperTray Parts Box	1	1
Not Shown		SP-BXHD	Parts Box, SuperPole Heavy Duty	1	1
Not Shown		SPT-BXHD	Parts Box, SuperTray Heavy Duty	1	1
Not Shown		SP506	Parts Pack, SuperTray	1	1
Not Shown		SP-500	Parts Pack, SuperPole	1	1

Uni-Fit Extender

PARTS DIAGRAM: UNIFIT EXTENDER

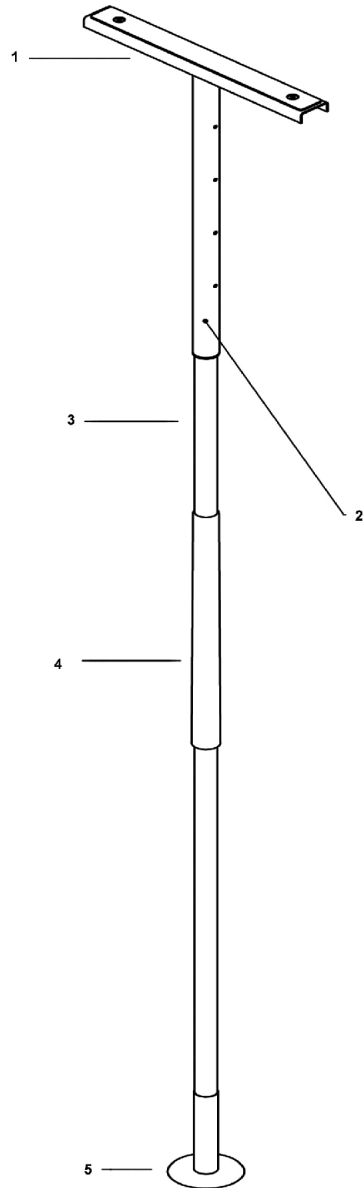


PARTS DIAGRAM: POLE / CEILING ANCHOR PLATE ASSEMBLY

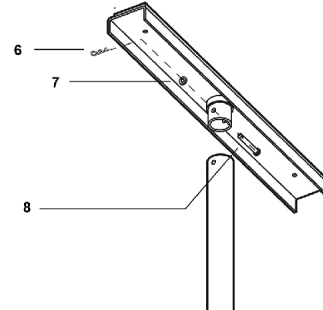

UNIFIT EXTENDER



PARTS DIAGRAM: POLE / EXTENSION SCREW / BASE PLATE ASSEMBLY

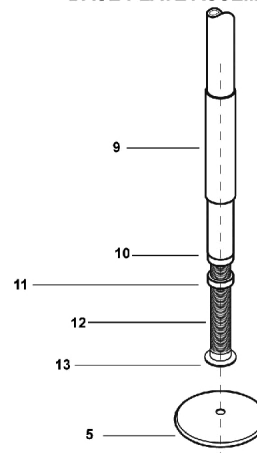


Uni-Fit Extender

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP403	Ceiling Anchor Plate Assembly	1	1
2		SP-UF	Extension Tube	1	1
3		SP-P	Pole Assembly	1	1
4		GR-101	Grip, 20"	1	1
5		SP402	Base Plate Assembly	1	1
6		HW323	Hitch Pin	1	1
7		HW318	Washer, Nylon	1	5
8		HW302	Clevis Pin	1	1
9		SP401	Cover Sleeve, Plastic	1	1
10		SP105	Threaded Collar	1	1
10		SP105HD	Threaded Collar, Heavy Duty	1	1
11		SP172	Jam Nut	1	1
12		SP107	Extension Screw	1	1
13		SP145	Washer, Plastic	1	1
Not Shown		SP-BX	Parts Box, SuperPole	1	1
Not Shown		SP507	Parts Pack, Uni-Fit Extender	1	1

Advantage Rail

PARTS DIAGRAM: ADVANTAGE RAIL

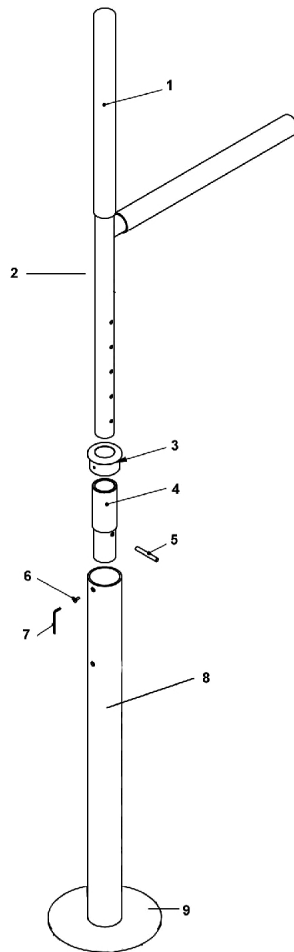


Advantage Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		GR-100	Grip, 16"	1	2
2		AR402	Assembly, T - Handle	1	1
3		AR105	Pivot Bushing	1	1
4		AR106	Pivot Sleeve	1	1
5		AR110	Pin, Shortened	1	1
6		HW100	Screw	1	1
7		HW300	Allen Key, Short Arm (3/32")	1	1
8		AR103C	Base Cover	1	1
9		AR400	Assembly, Mast	1	1
Not Shown		AR500	Parts Pack, Advantage Rail	1	1
Not Shown		AR-AP-CK	Convert Advantage Rail to Advantage Pole	1	1

Advantage Pole

PARTS DIAGRAM: ADVANTAGE POLE

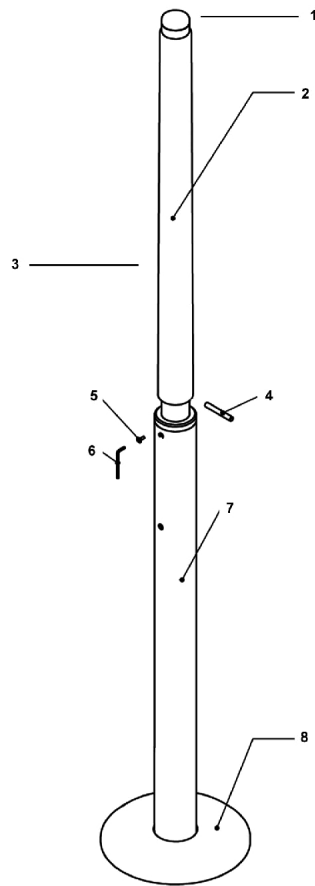


Advantage Pole

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		AR133	Plastic Plug	1	1
2		GR-101	Grip, 20"	1	1
3		AR405	Assembly, Pole, Advantage Pole	1	1
4		AR110	Pin, Shortened	1	1
5		HW100	Screw	1	1
6		HW300	Allen Key, Short Arm (3/32")	1	1
7		AR401	Assembly, Mast	1	1
8		AR103C	Base Cover	1	1
Not Shown		AR131	Pole Bushing, Top	1	1
Not Shown		AR501	Parts Pack, Advantage Pole	1	1
Not Shown		AR-AR-CK	Convert Advantage Pole to Advantage Rail	1	1

Smart Rail

PARTS DIAGRAM: SMART-RAIL

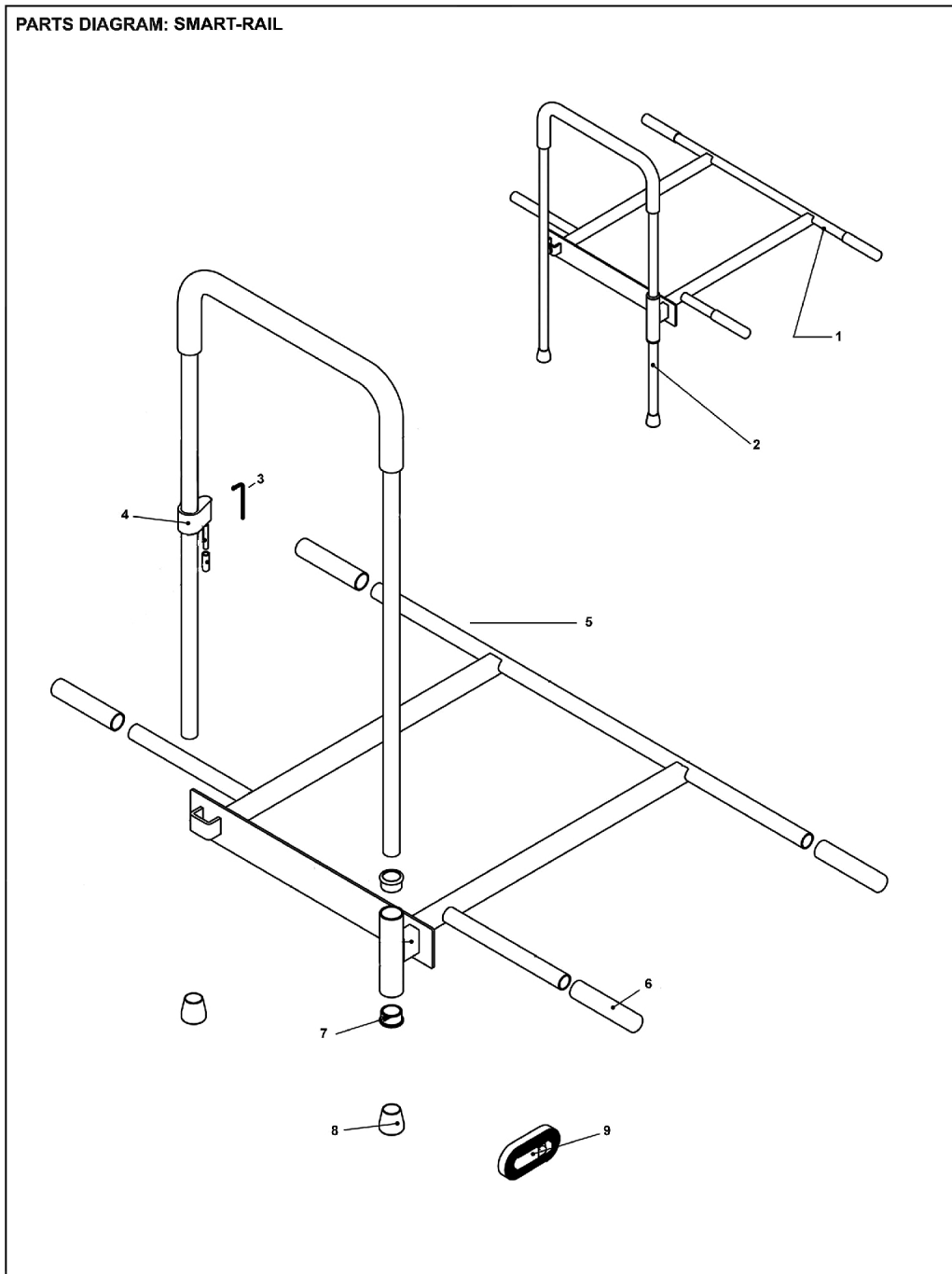


Smart Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SR403	Support Frame	1	1
2		SR402	Rail with Grip	1	1
3		HW301	Hex Key, Short Arm (1/8")	1	1
4		SR401	Latch Block	1	1
5		SR112	Support Frame	1	1
6		GR107	Traction Grips	1	4
7		SR109	Pivot Bushing	1	2
8		HW312	Cane Tips	1	2
9		SR400	Strap, Smart - Rail / Assista Rail	1	1
Not Shown		SR500	Parts Pack, Smart Rail	1	1

Smart Rail Hospital

PARTS DIAGRAM: SMART-RAIL HOSPITAL

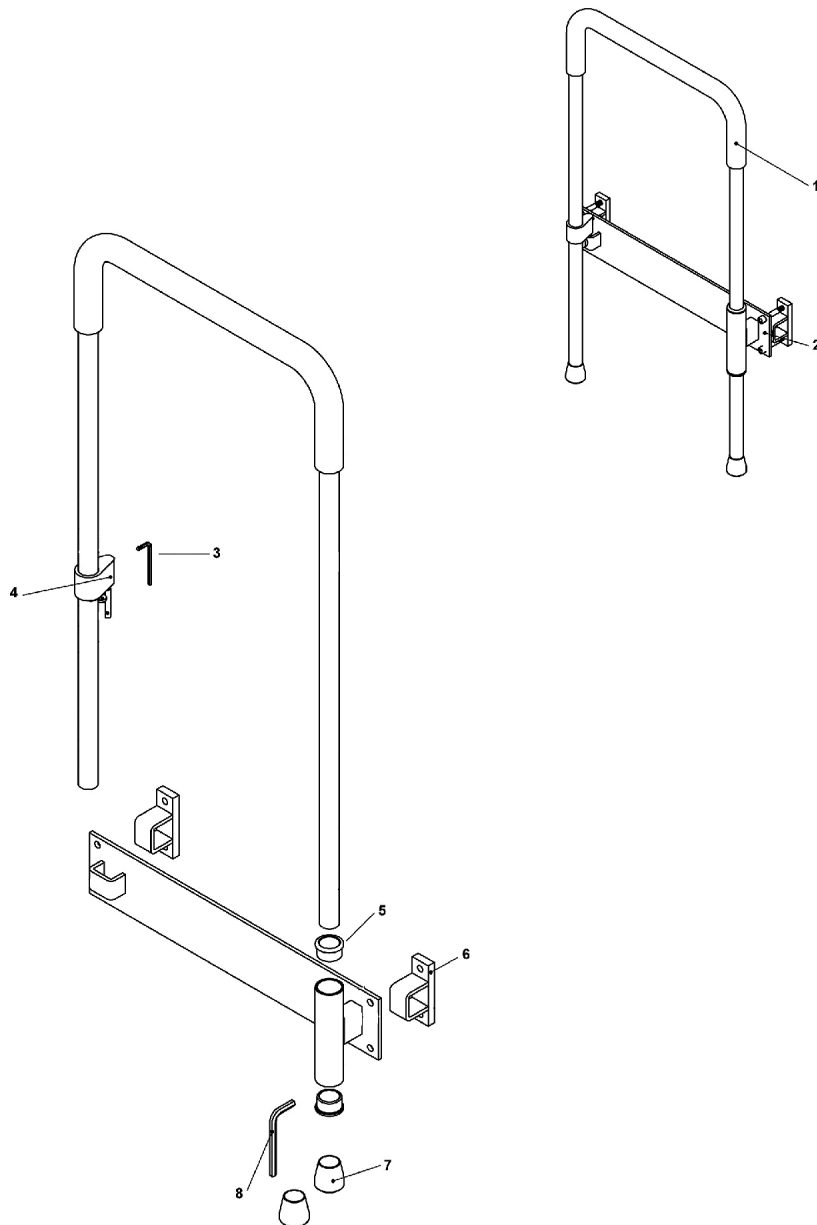


Smart Rail Hospital

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SR402	Rail with Grip	1	1
2		SR404	Frame Face Plate	1	1
3		HW301	Hex Key, Short Arm (1/8")	1	1
4		SR401	Latch Block	1	1
5		SR109	Pivot Bushing	1	2
6		SR201	Frame Clamps	1	2
7		HW312	Cane Tips	1	2
8		HW319	Hex Key, Short Arm (1/4")	1	1
Not Shown		SR501	Parts Pack, Smart Rail Hospital	1	1

Horizontal and Parallel Bar Systems

PARTS DIAGRAM: HORIZONTAL AND PARALLEL BAR SYSTEMS

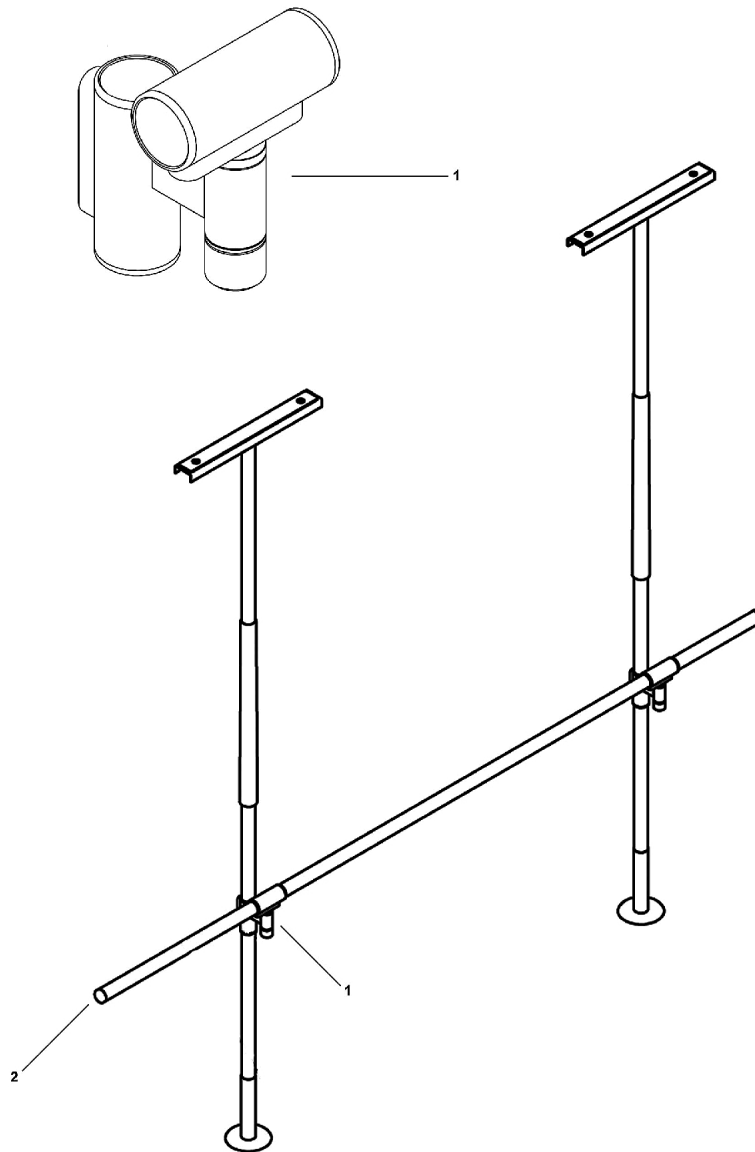


Horizontal and Parallel Bar Systems

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		PB204-2	Clamp Assembly	1	1
1		PB204-4	Clamp Assembly	1	1
2	A	PB203	Horizontal Rail, 100"	1	1
2	B	PB203	Horizontal Rail, 100"	1	2
Not Shown		PB202	Parts Pack, Parallel Bar	1	1
NOTE: A - For Horizontal Rail System. B - For Parallel Bar System.					

Dependa-Bar

PARTS DIAGRAM: DEPENDA-BAR

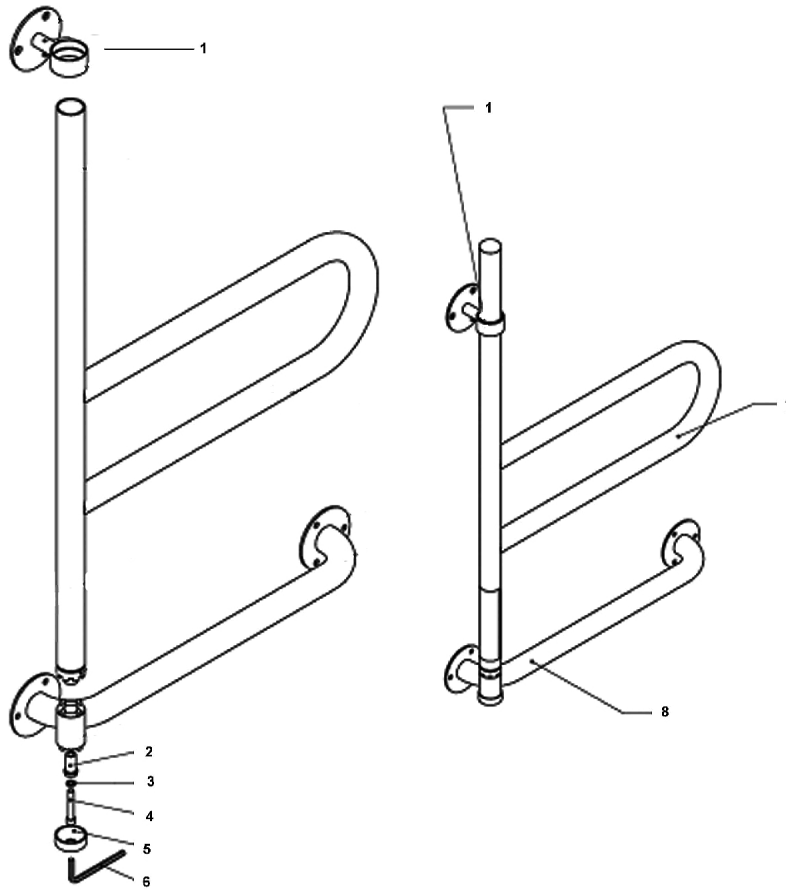


Dependa-Bar

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		DB204W	Top Flange, White	1	1
1		DB204	Top Flange, Stainless Steel	1	1
2		DB106	Spacer, Bottom Lift Limit	1	1
3		HW131	Washer, High Collar (1/4")	1	1
4		HW130	Screw, Socket Head Cap (1/4-20 x 1.5")	1	1
5		DB130	Cap, Vinyl (1.25" Punched)	1	1
6		HW321	Hex Key, Short Arm (3/16")	1	1
7		DB200W	Pivot Rail, White	1	1
7		DB200	Pivot Rail, Stainless Steel	1	1
8		DB201W	Bottom Rail, Stainless Steel, White (18")	1	1
8		DB203W	Bottom Rail, Stainless Steel, White (32")	1	1
8		DB201	Bottom Rail, Stainless Steel, Knurled (18")	1	1
8		DB203	Bottom Rail, Stainless Steel, Knurled (32")	1	1
Not Shown		DB500	Parts Pack, Dependa-Bar	1	1
Not Shown		DB501	Screw Sub Pack, Dependa-Bar	1	1

PT Rail and Mast

PARTS DIAGRAM: PT RAIL AND MAST

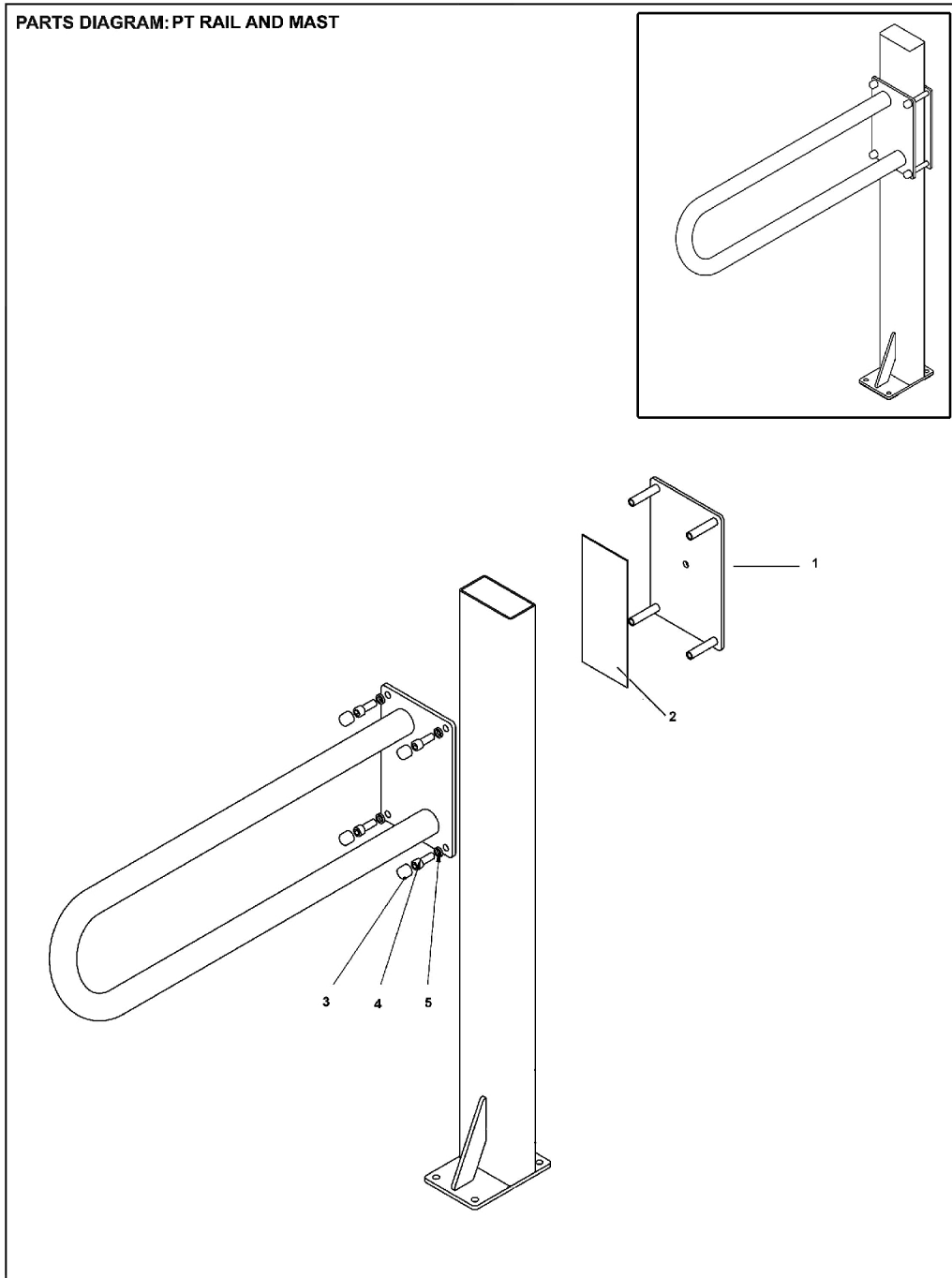


PT Rail and Mast

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		WR158PCW	Mast Clamp Plate, White (Attach PT Rail to Mast)	1	1
1		WR158PCP	Mast Clamp Plate, Platinum Grey Finish (Attach PT Rail to Mast)	1	1
1		WR158SS	Mast Clamp Plate, Stainless Steel (Attach PT Rail to Mast)	1	1
2		WR154	Mast Clamp Pad	1	1
3		HW513	Cap, White Vinyl	1	4
4		HW401	Screw (3/8-16 x 1")	1	4
5		HW402	Washer, Lock, High Collar (3/8")	1	4
Not Shown		WR500	Parts Pack, PT Rail	1	1
Not Shown		WR404	Parts Pack, WRM	1	1
Not Shown		PT-WRM-EMB	Wall Mast, Embedded	1	1
Not Shown		HW317	Hex Key, Short Arm (5/16")	1	1

Assista-Rail

PARTS DIAGRAM: ASSISTA-RAIL

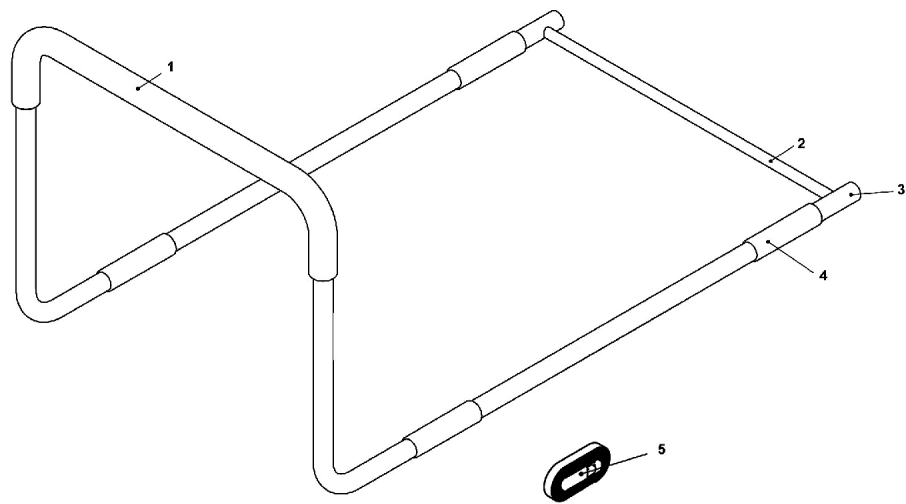


Assista-Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		GR108	Grip, Assista Rail Handle	1	1
2		AST106-C	Cross Bar	1	1
3		AST400	Assista-Rail	1	1
4		GR-111	Grips, Traction	1	4
5		SR400	Strap, Smart - Rail / Assista Rail	1	1
Not Shown		AST500	Parts Pack, Assista-Rail	1	1

Assista-Rail with Assista-Cane

PARTS DIAGRAM: ASSISTA-RAIL WITH ASSISTA-CANE

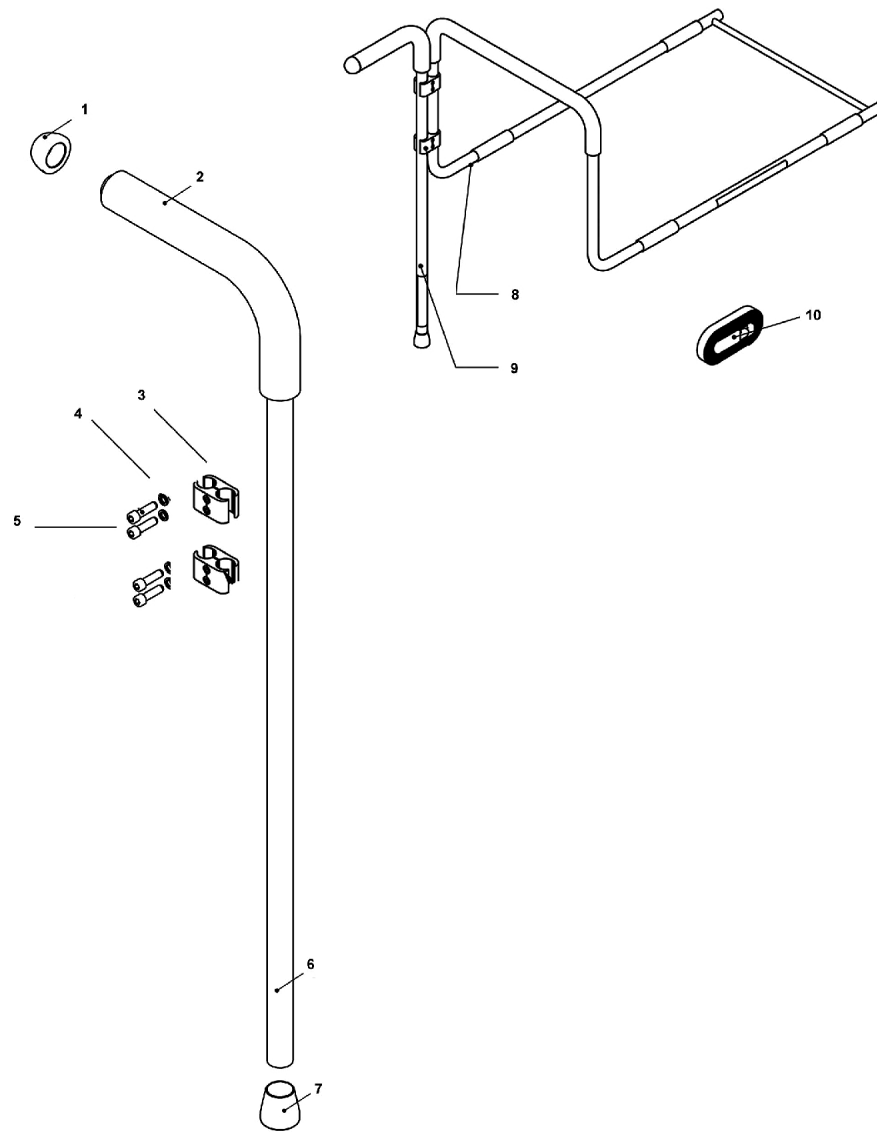


Assista-Rail with Assista-Cane

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		AST103	Cane Cap	1	1
2		GR110C	Grip, Cane	1	1
3		AST402	Safety Clamps	1	2
4		HW120	Washer, High Collar (1/4")	1	1
5		HW119	Screw, Socket Head Cap (1/4-20 X 7/8")	1	1
6		AST102	Cane	1	1
7		HW312	Cane Tips	1	1
8		AST400	Assista-Rail	1	1
9		AST401	Assista-Cane	1	1
10		SR400	Strap, Smart - Rail / Assista Rail	1	1
Not Shown		AST502	Parts Pack, Assista-Cane	1	1
Not Shown		AST504	Parts Pack, Assista-Cane Add On Kit	1	1
Not Shown		HW321	Hex Key, Short Arm (3/16")	1	1

ModRail


PARTS DIAGRAM: MODRAIL

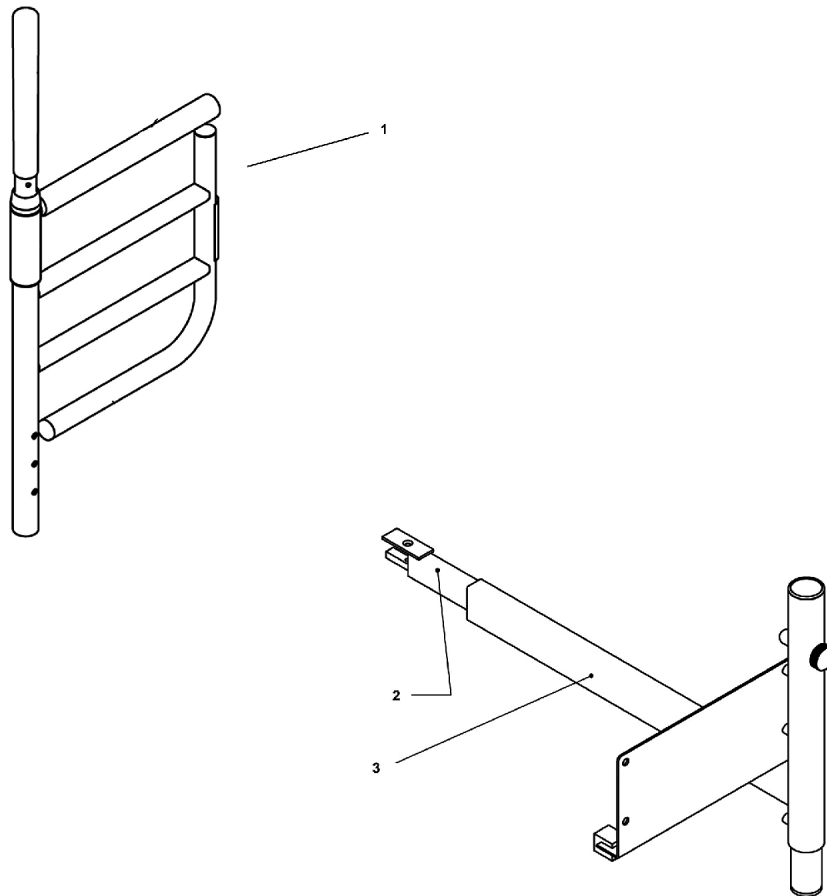

ModRail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		MR-S	ModRaik w/o Bracket	1	1
2		MR331	Cross Beam, Hospital Beds	1	1
2		MR341	Telescopic Cross Beam, Box Spring and Mattress Beds	1	1
3		MR-BK-HS	Bracket, ModRail Box Spring and Mattress Beds	1	1
3		MR-BK-INV	Bracket, ModRail Hospital Beds	1	1
Not Shown		MR530	Parts Pack, ModRail Hospital	1	1
Not Shown		MR540	Parts Pack, ModRail Box Spring and Mattress Beds	1	1
Not Shown		MR-CBE	Cross Beam Extender, ModRail	1	1

ModBar

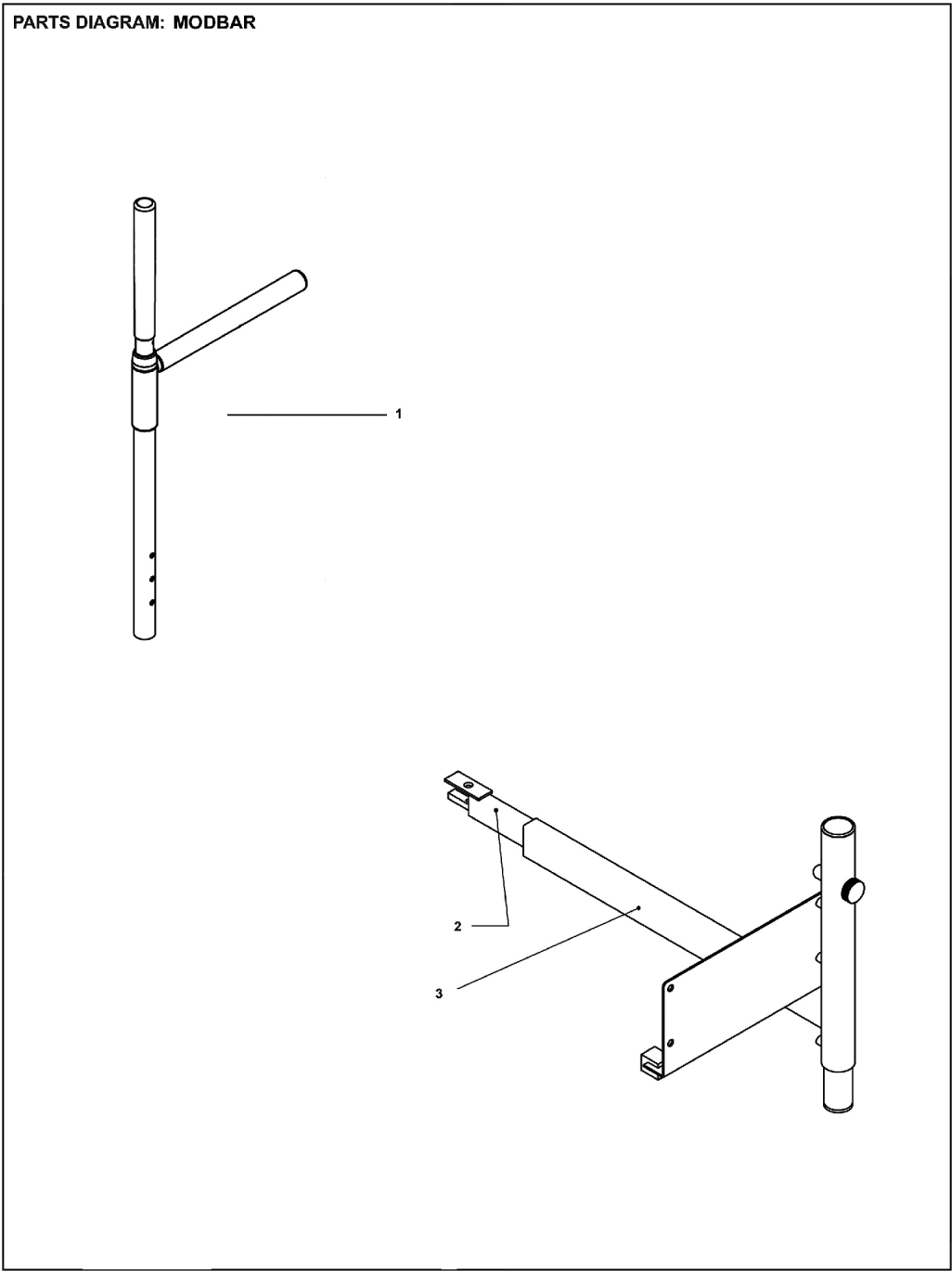


ModBar

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		MB-S	ModBar w/o Bracket	1	1
2		MR331	Cross Beam, Hospital Beds	1	1
2		MR341	Telescopic Cross Beam, Box Spring and Mattress Beds	1	1
3		MR-BK-HS	Bracket, ModRail Box Spring and Mattress Beds	1	1
3		MR-BK-INV	Bracket, ModRail Hospital Beds	1	1
Not Shown		MR530	Parts Pack, ModRail Hospital	1	1
Not Shown		MR540	Parts Pack, ModRail Box Spring and Mattress Beds	1	1
Not Shown		MR-CBE	Cross Beam Extender, ModRail	1	1

ModPole

PARTS DIAGRAM: MODPOLE

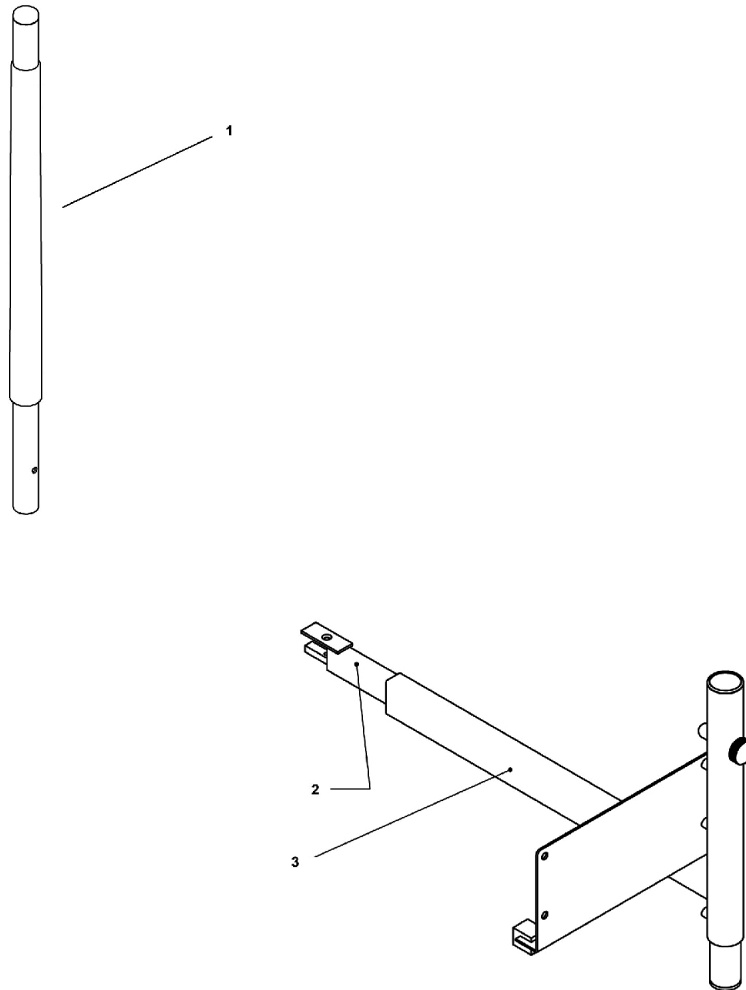

ModPole

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		MP-S	ModPole w/o Bracket	1	1
2		MR331	Cross Beam, Hospital Beds	1	1
2		MR341	Telescopic Cross Beam, Box Spring and Mattress Beds	1	1
3		MR-BK-HS	Bracket, ModRail Box Spring and Mattress Beds	1	1
3		MR-BK-INV	Bracket, ModRail Hospital Beds	1	1
Not Shown		MR530	Parts Pack, ModRail Hospital	1	1
Not Shown		MR540	Parts Pack, ModRail Box Spring and Mattress Beds	1	1
Not Shown		MR-CBE	Cross Beam Extender, ModRail	1	1

Invacare Canada L.P.

www.invacare.ca



570 Matheson Blvd E;8
Mississauga, Ontario,
L4Z 4G4 Canada
800-668-5324

Technical Support
8006685324 x2655
Customer Service
800-668-5324

Literature
800-668-5324
or www.invacare.ca

© 2020 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. (Revised 04/14/20)

Form No. 08-664C