

ICC - Lift IH1200, IH1300, IH1500, IH1600, IH1800, IH1900, 1100 Scale



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 09-077. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

**For Customer Service and Technical Support, Call:
1-800-668-2337**

**For Literature Requests, Call:
1-800-668-2337 or go to our website
www.invacare-cc.com**

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments
NOTE: = General information pertaining to that specific page
NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length
Washers Inside Diameter X Outside Diameter X Thickness
Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

® 2022 Invacare Corporation. Invacare, the Medallion Design and Yes, You can. are registered trademarks of the Invacare Corporation.

TABLE of CONTENTS

Description	Page Number
Base Assembly - 1200, 1300, 1500, 1600, 1800	4
Base Assembly Components - 1200, 1300, 1500, 1600, 1800	5
Battery and Battery Charger Assembly - 1200, 1300, 1500, 1600, 1800	10
Electrical Components - 1200, 1300, 1500, 1600, 1800	12
Electrical Covers - 1200, 1300, 1500, 1600, 1800	14
Mast and Ball Screw Assembly - 1200, 1300, 1500, 1600, 1800	7
Model 1100 - Scale Components	22
Model 1200 - Stretcher Assembly	16
Model 1300 - Seat Assembly	18
Model 1800 - C Base Seat Assembly	20
Model 1900 - K Style Base Assembly	31
Model 1900 - Mast Assembly	23
Model 1900 - Patient Seat Assembly	27
Model 1900S - Mast Assembly	25
Model 1900S - Patient Seat Assembly	29
Model Photos - 1200, 1300, 1800, 1900	3
Push Bar Assembly - 1200, 1300, 1500, 1600, 1800	9

Model Photos - 1200, 1300, 1800, 1900

ICCI Traverse Lift Models

Model 1200



Model 1300



Model 1800



Model 1900





SICC_048A

Base Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

Standard, Narrow and C-Base Lifts

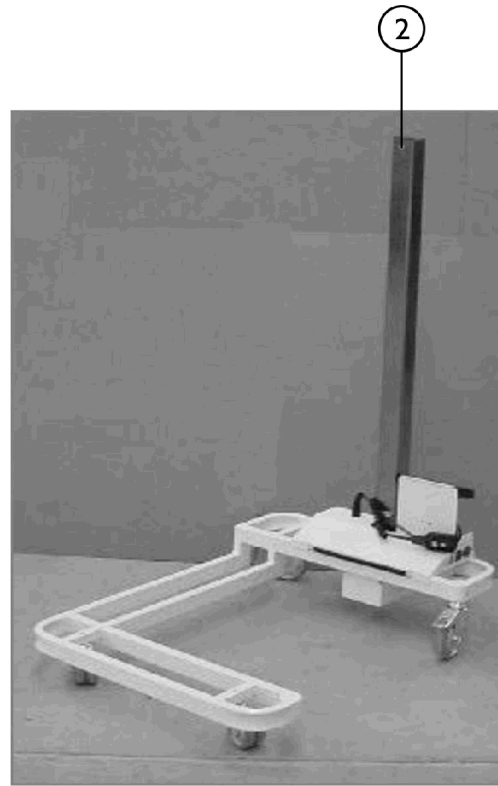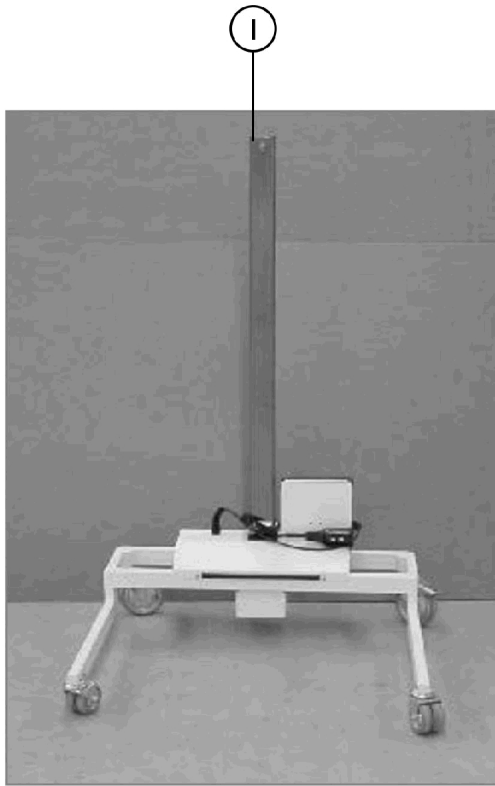

SICC_049A

Standard, Narrow and C-Base Lift Assemblies


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHTRV1000N-NL A	NLA - Narrow Lift Base Assembly	1	1
2	A	IHTRV1000C-NL A	NLA - C-Base Lift Assembly	1	1

NOTE: A - State color and Lift model when ordering.

Base Assembly Components

Model 1200, 1300, 1500, 1600, 1800 Service Parts

Standard, Narrow and C-Base Lift Assemblies

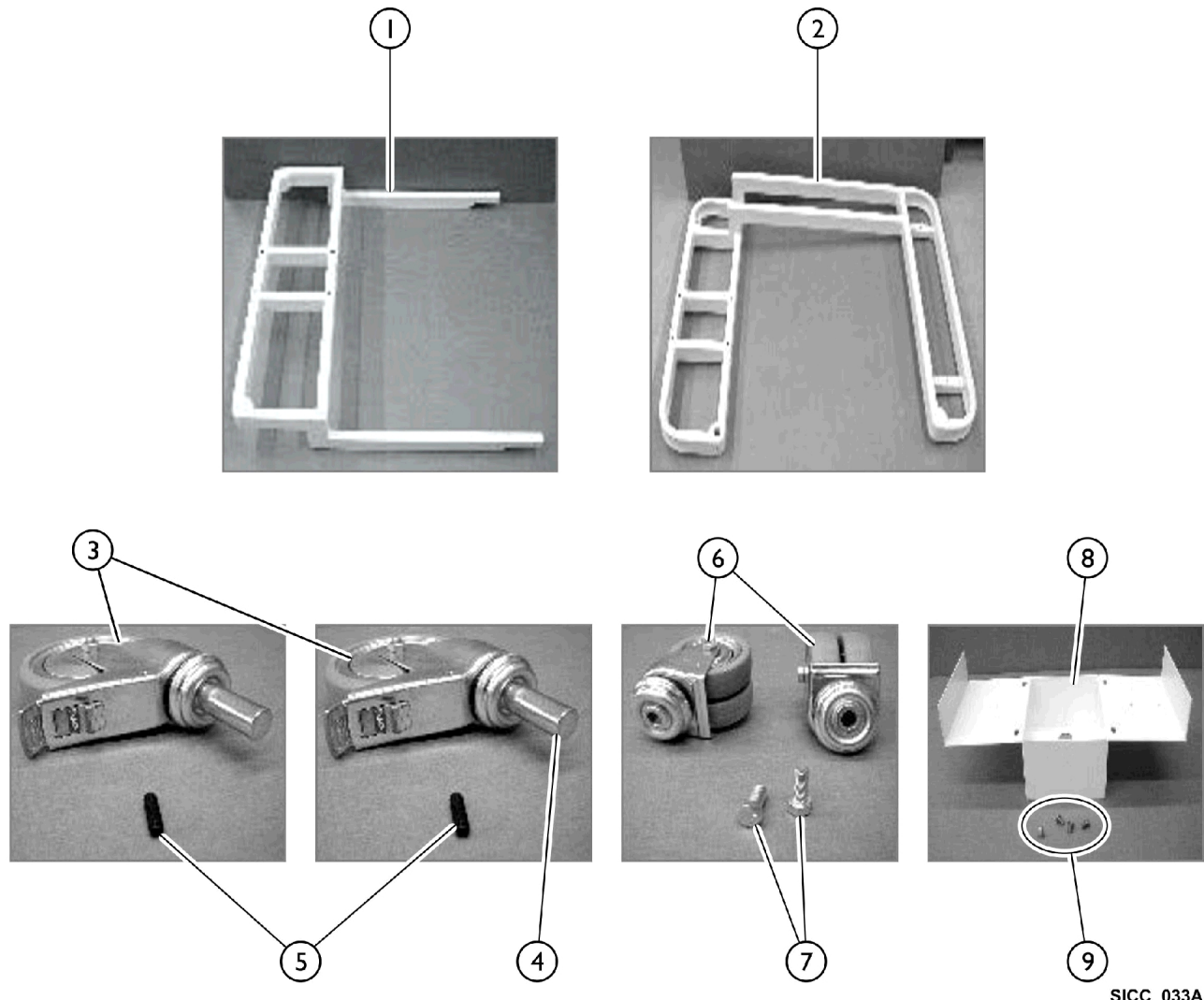

SICC_033A

Base Assembly Components


Model 1200, 1300, 1500, 1600, 1800 Service Parts

Standard, Narrow and C-Base Lift Assemblies

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		TRP1000A-NLA	NLA - Standard Base, Almond	1	1
1		TRP1000W	Standard Base, White	1	1
2		TRP1000CA-NLA	NLA - C-Base, Almond	1	1
2		TRP1000CW-NLA	NLA - C-Base, White	1	1
3		IHTRP1029-NLA	NLA - Caster, Brake - Base	1	1
4		IHTRP1028-NLA	NLA - Caster, Steering - Base	1	1
5		IHTRP1062B	Bolt, Caster (M8 x 25mm)	1	2
6		IHTRP1027-NLA	NLA - Caster, Front - Base	1	2
7		IHTRP1063-NLA	NLA - Bolt, Stainless, Front Caster (7/16-18 x 1")	1	2
8		IHTRP1001A-NLA	NLA - Motor Box, Almond	1	1
8		IHTRP1001W-NLA	NLA - Motor Box, White	1	1
9		IHTRP1065-NLA	NLA - Bolt, Motor Box, Stainless (1/4-20 x 3/8")	1	4

Mast and Ball Screw Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

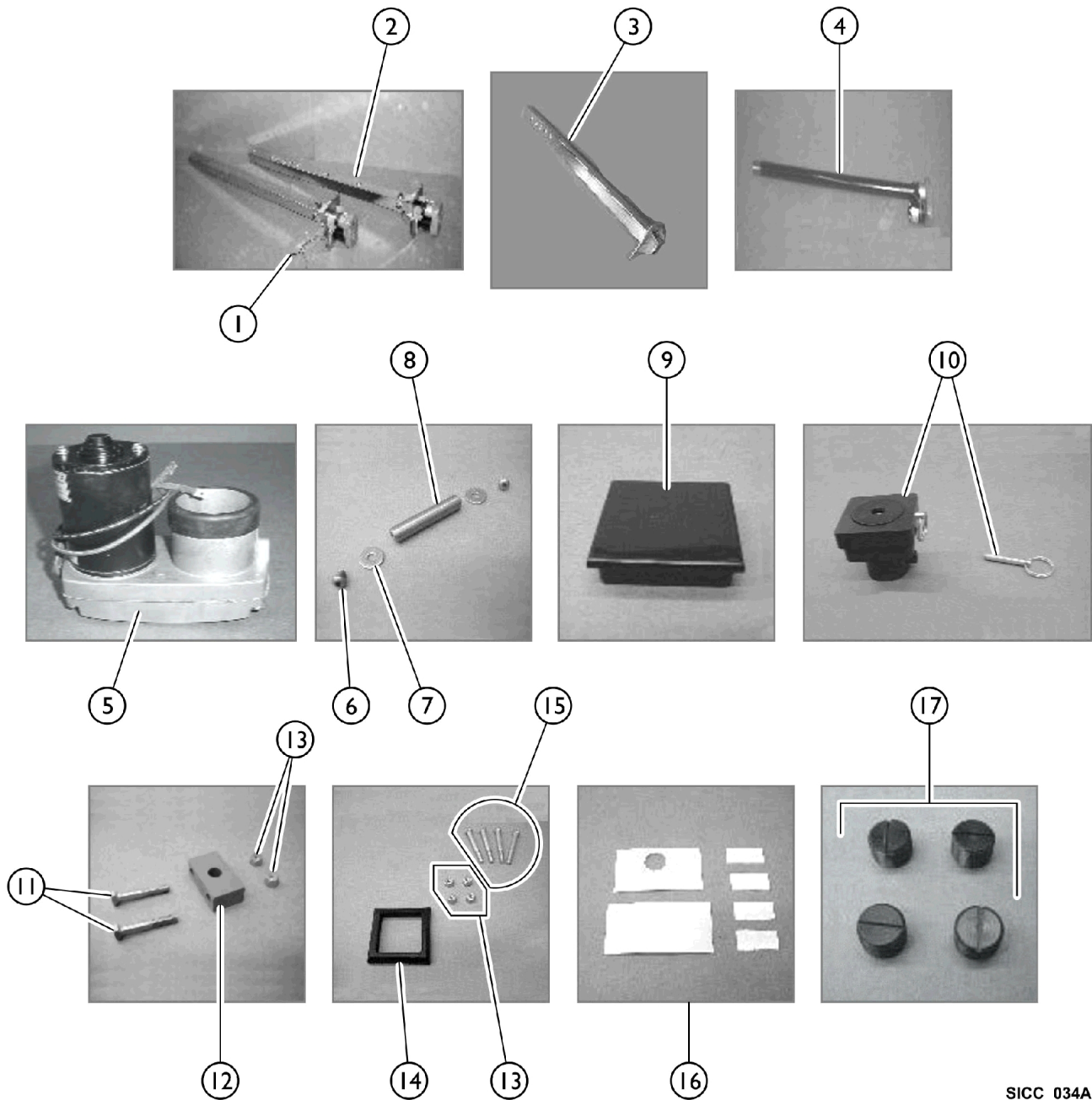

SICC_034A

Mast and Ball Screw Assembly



Model 1200, 1300, 1500, 1600, 1800 Service Parts

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHTRP1010-NLA	NLA - Outer Column Only	1	1
1		TRP1010-2R	Reverse Inner/Outer Column & Ball Screw Assembly (1500 & 1600 Lifts)	1	1
1		IHTRP1008R	Reverse Inner Column Only	1	1
2		TRP1010-2-NLA	NLA - Inner/Outer Column Ball and Screw Assembly	1	1
2		IHTRP1008-NLA	NLA - Inner Column Only	1	1
2		IHTRP1010-NLA	NLA - Outer Column Only	1	1
3		IHTRP1010C	Stationary Outer Column	1	1
4		TRP1030X	Ball Screw Assembly	1	1
5		IHTRP1820	Ball Screw Motor	1	1
6		IHTRP1065-NLA	NLA - Bolt, Motor Box, Stainless (1/4-20 x 3/8")	1	2
7		IHTRP1082-NLA	NLA - Washer	1	2
8		IHTRP1015-NLA	NLA - Lift Pin	1	1
9		IHTRP1019-NLA	NLA - Column Cap	1	1
10	A	IHTRP1007-NLA	NLA - Emergency Crank Cap	1	1
10	A	IHTRP1007D-NLA	NLA - Replacement Pin	1	1
11		IHTRP1061-NLA	NLA - Bolt, Trunion Block (5/16-18 x 2")	1	2
12		ITHRP1005	Trunion Block	1	1
13		IHTRP1070-NLA	NLA - Nut, Nylon (5/16-18)	1	6
14		IHTRP1016A-NLA	NLA - Column Seal	1	1
15		IHTRP1060-NLA	NLA - Bolt, Column Seal (5/16-18 x 2-3/4")	1	4
16		TRP1041KIT	Bearing Tape Kit	6	1
17		IHTRP1073-NLA	NLA - Bearing Screws	1	4

NOTE: A - Optional part.

Push Bar Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

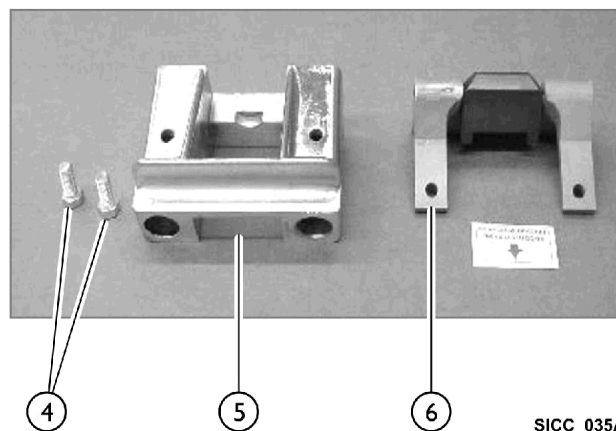
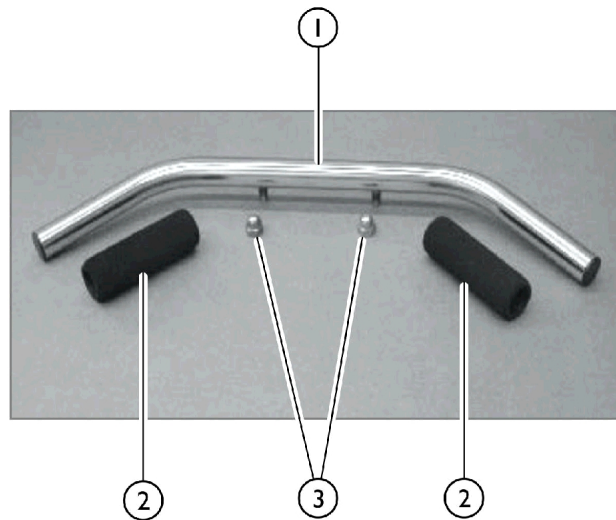


SICC_035A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHTRP1014C	Push Handle	1	1
2		TRP1036	Door Handle Grip	1	2
3		IHTRP1069	Nut, Acorn, SS (3/8-16)	1	2
4		IHTRP1067-NLA	NLA - Bolt, Push Bar (3/8-16 x 1")	1	2
5		IHTRP1012	Push Bar Bracket	1	1
6		IHTRP1013	Lock Out Bracket	1	1

Battery and Battery Charger Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

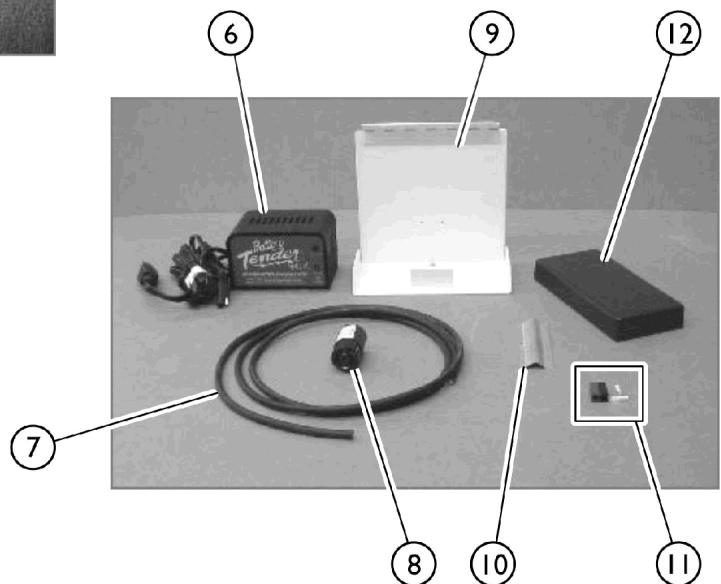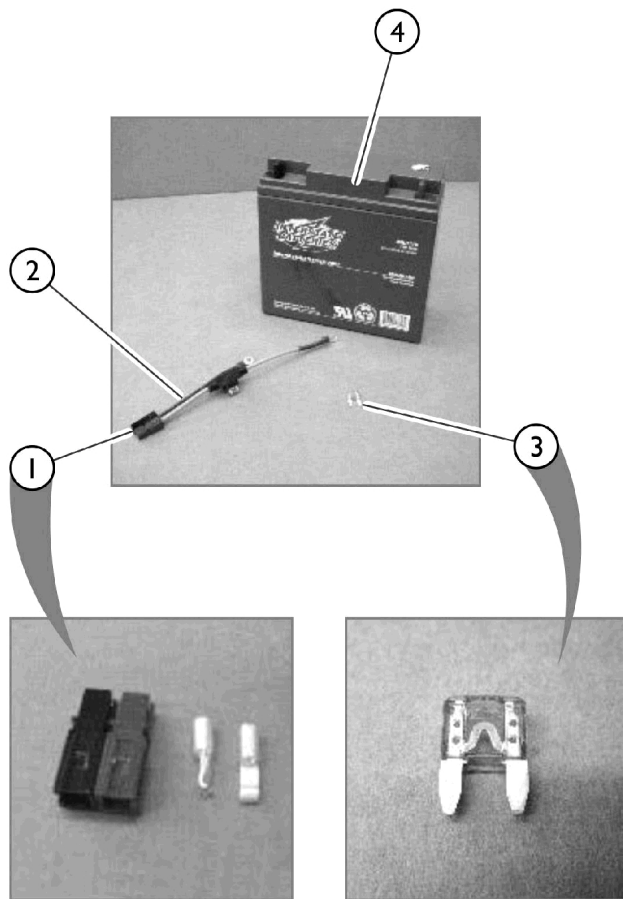


SICC_036A

Battery and Battery Charger Assembly

Model 1200, 1300, 1500, 1600, 1800 Service Parts

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHTRP1083	Wire Connectors - Red and Black	1	1
2		IHTRP1087-NLA	NLA - Battery Harness Assembly	1	1
3		IHTRP1085-NLA	NLA - 30 Amp Fuse (For Battery Harness)	1	1
4		IHTRP1051-NLA	NLA - Battery	1	1
5		IHTRP1052W-NLA	NLA - Battery Charger Assembly, White	1	1
5		IHTRP1052A-NLA	NLA - Battery Charger Assembly, Almond	1	1
5		IHTRP1052C-NLA	NLA - Battery Charger Assembly - 220V	1	1
6		IHTRP1052D	Battery Charger Only	1	1
7		IHTRP1033	Charger Cord - 6 ft.	1	1
8		IHTRP1033C	Hubble Plug	1	1
9		TRP1002XW-NLA	NLA - Battery Housing - White	1	1
10		IHTRP1003-NLA	NLA - Wire Guard	1	1
11	B	IHTRP1083	Wire Connectors - Red and Black	1	1
		IHTRP1083A-NLA	NLA - Wire Nut, Black	1	1
		IHTRP1083B-NLA	NLA - Wire Nut, Red	1	1
		IHTRP1083C-NLA	NLA - Contact, Loose (30 Amp)	1	1
12	A	IHTRP1011	Battery Cover	1	1
		IHTRP1040-NLA	NLA - Grip Tape	1	1
NOTE: A - Not Shown in Photos. B - Includes Wire Nut - Red, Wire Nut - Black, and Loose Contact.					

Electrical Components
Model 1200, 1300, 1500, 1600, 1800 Service Parts

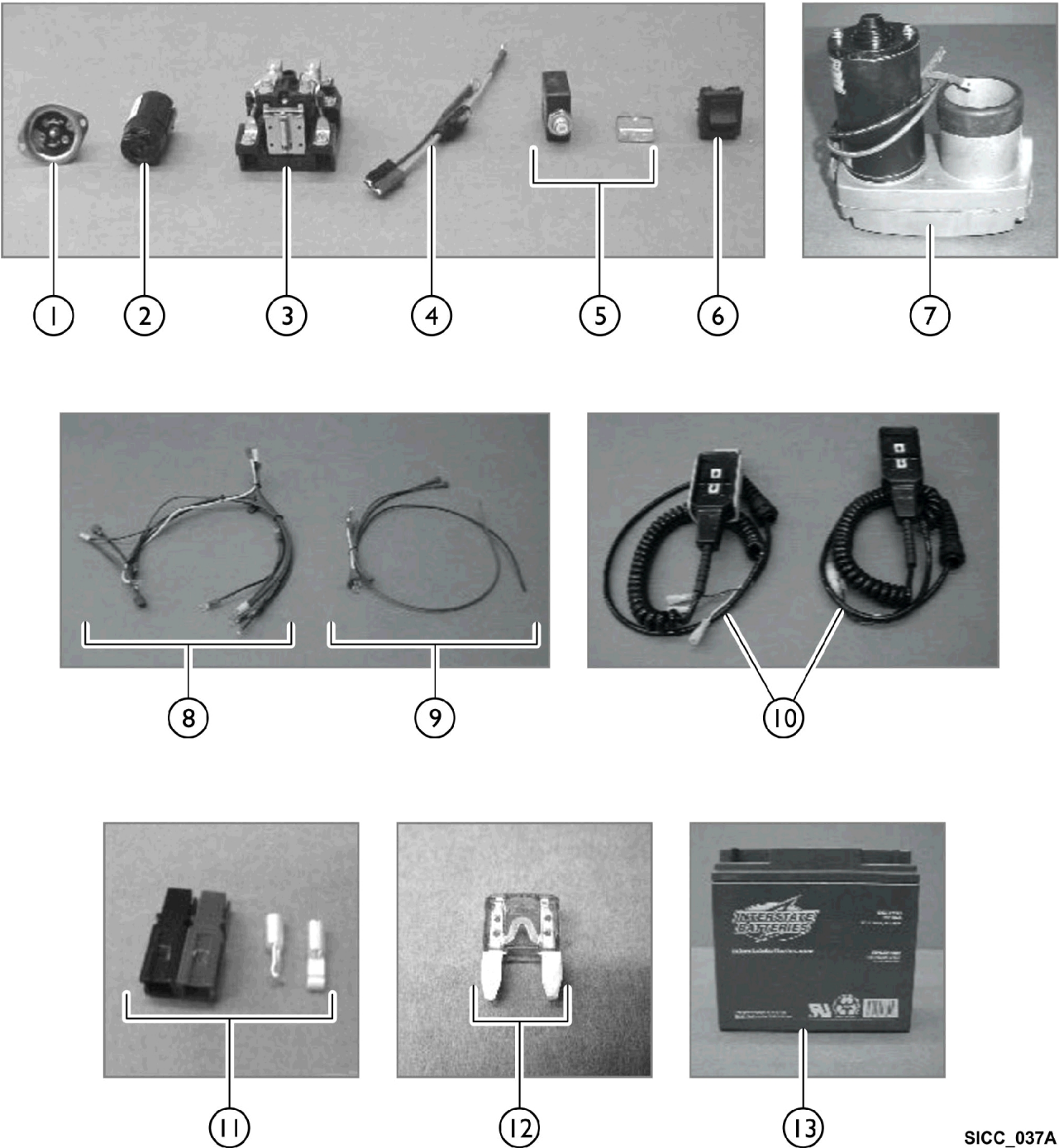

SICC_037A

Electrical Components


Model 1200, 1300, 1500, 1600, 1800 Service Parts

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHTRP1022C-NL A	NLA - Hubble Receiver	1	1
2		IHTRP1033C	Hubble Plug	1	1
3		IHTRP1023-NLA	NLA - Relay	1	1
4		IHTRP1087-NLA	NLA - Battery Harness Assembly	1	1
5	A	IHTRP1021A-NL A	NLA - 20A Circuit Relay (New Style)	1	1
5	A	IHTRP1021	20A Circuit Breaker (Old Style - Pollack)	1	1
6		IHTRP1901-NLA	NLA - Rocker Switch	1	1
7		IHTRP1820	Ball Screw Motor	1	1
8		IHTRP1024C-NL A	NLA - Wire Harness - Relays	1	1
9		IHTRP1025C-NL A	NLA - Wire Harness - Electrical Cover	1	1
10	B	IHTRP1020N-NL A	NLA - Hand Switch - New Style	1	1
11		IHTRP1083	Wire Connectors - Red and Black	1	1
		IHTRP1083A-NL A	NLA - Wire Nut, Black	1	1
		IHTRP1083B-NL A	NLA - Wire Nut, Red	1	1
		IHTRP1083C-NL A	NLA - Contact, Loose (30 Amp)	1	1
12		IHTRP1085-NLA	NLA - 30 Amp Fuse (For Battery Harness)	1	1
13		IHTRP1051-NLA	NLA - Battery	1	1
NOTE: A - Old Style is on the Right, New Style on the Left in the photo. B - Includes Red Wire Nut, Black Wire Nut, and Loose Contact.					



Electrical Covers

Model 1200, 1300, 1500, 1600, 1800 Service Parts

**Standard
Electrical Cover**

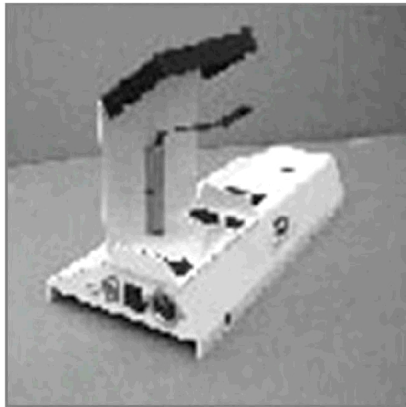

**Reverse
Electrical Cover**

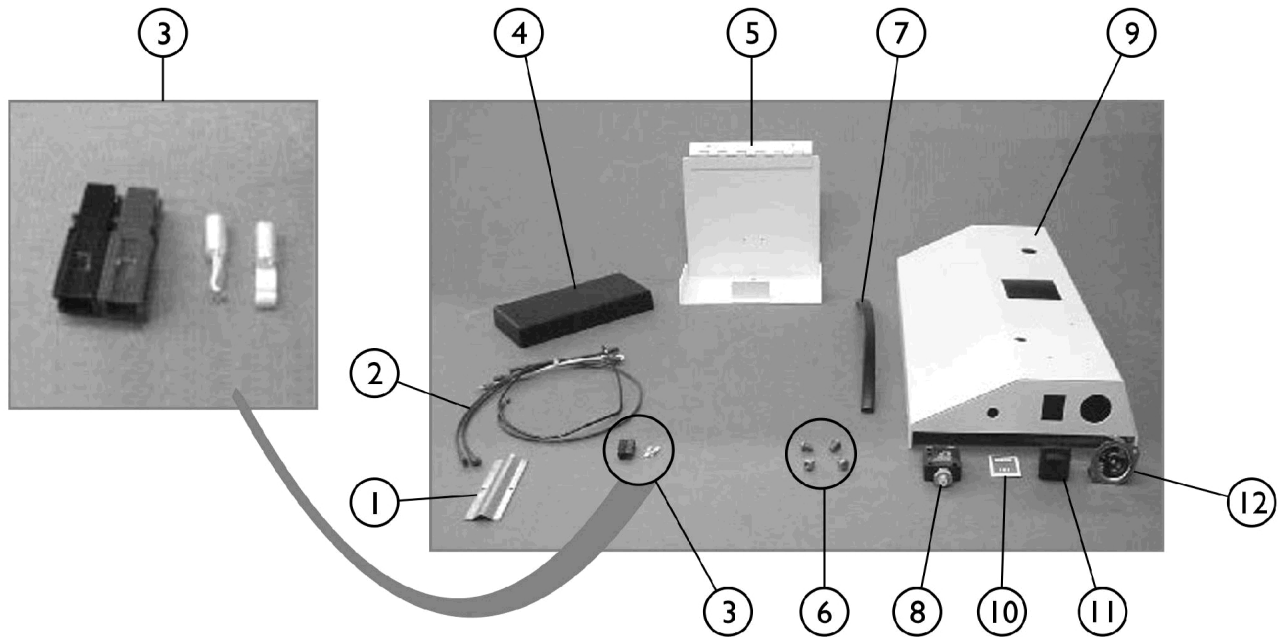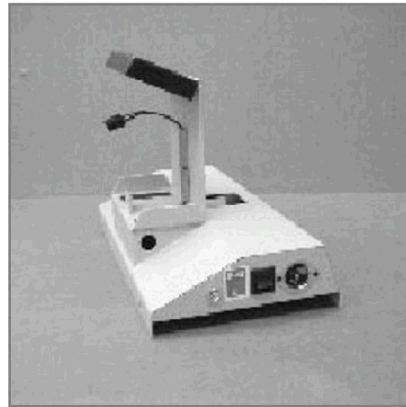


SICC_050A

Electrical Covers

Model 1200, 1300, 1500, 1600, 1800 Service Parts

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHTRP1003-NLA	NLA - Wire Guard	1	1
2		IHTRP1025C-NL A	NLA - Wire Harness - Electrical Cover	1	1
3		IHTRP1083	Wire Connectors - Red and Black	1	1
		IHTRP1083A-NL A	NLA - Wire Nut, Black	1	1
		IHTRP1083B-NL A	NLA - Wire Nut, Red	1	1
		IHTRP1083C-NL A	NLA - Contact, Loose (30 Amp)	1	1
4		IHTRP1011	Battery Cover	1	1
5		TRP1002XA	Battery Housing - Almond	1	1
5		TRP1002XW-NL A	NLA - Battery Housing - White	1	1
6		IHTRP1065-NLA	NLA - Bolt, Motor Box, Stainless (1/4-20 x 3/8")	1	4
7		IHTRP1038	NLA - Rubber Bumper Strip	1	1
8		IHTRP1021A-NL A	NLA - 20A Circuit Relay (New Style)	1	1
8		IHTRP1021	20A Circuit Breaker (Old Style - Pollack)	1	1
9		IHTRP1002A	Electrical Cover - Almond	1	1
9		IHTRP1002W-N LA	NLA - Electrical Cover - White	1	1
10		IHTRP1902-NLA	NLA - Label, Charge - Lift	1	1
11		IHTRP1901-NLA	NLA - Rocker Switch	1	1
12		IHTRP1022C-NL A	NLA - Hubble Receiver	1	1

Model 1200 - Stretcher Assembly

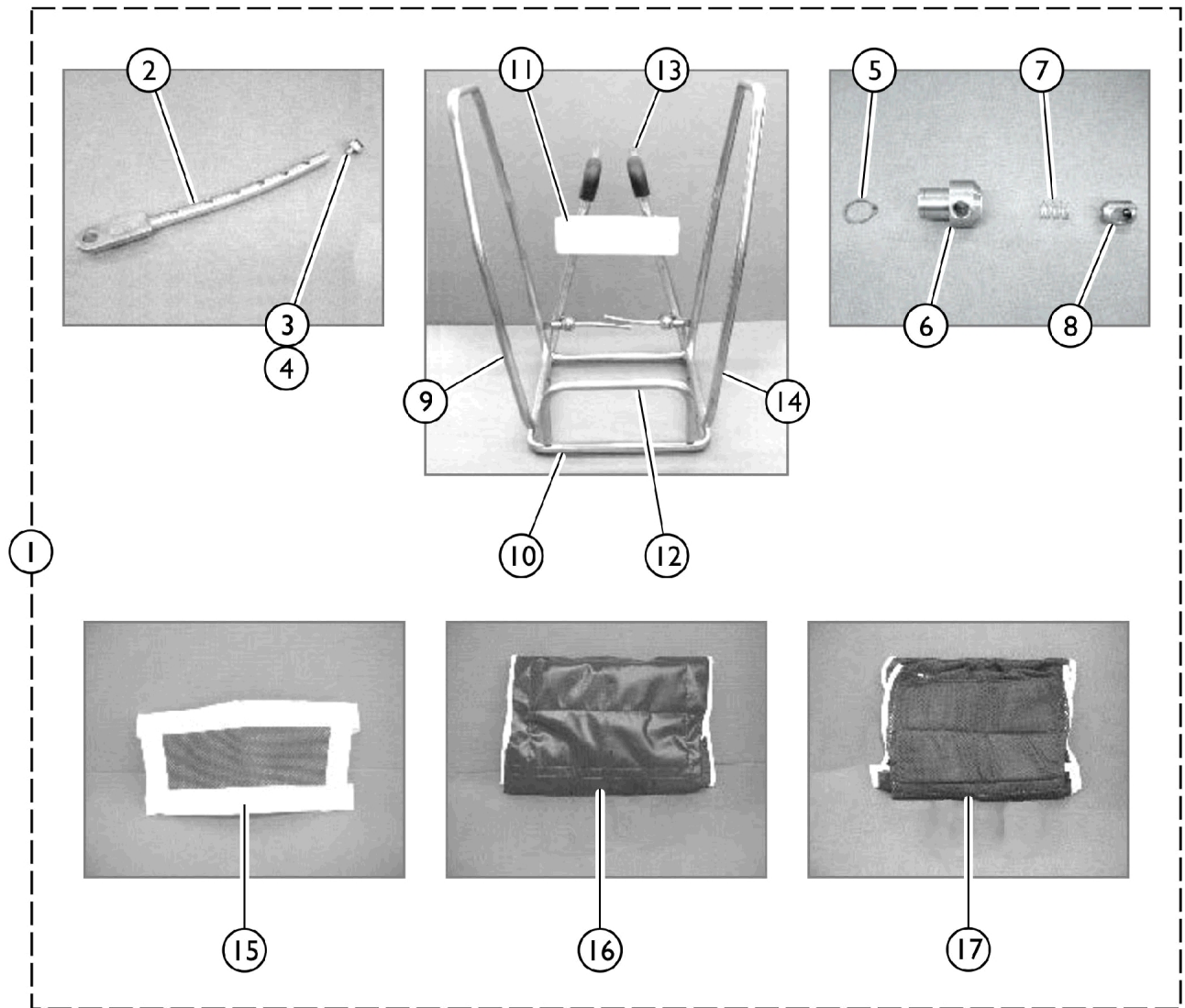


SICC_039A

Model 1200 - Stretcher Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHTRV1201	Stretcher Assembly w/ Cover (Ref. Only)	1	1
1	A	IHTRV1201N	Stretcher Assembly w/ Cover - Narrow (Ref. Only)	1	1
2		IHTRP1110-NLA	NLA - Adjusting Rod	1	1
3		IHTRP1112S-NL A	NLA - Adjusting Rod Bolt and Washer	1	1
4		TRP1112T-NLA	NLA - Washer, Adjustment	1	1
5		IHTRP1074-NLA	NLA - Snap Ring	1	1
6		IHTRP1108-NLA	NLA - Adjusting Rod Housing	1	1
7		IHTRP1173-NLA	NLA - Adjusting Rod Spring	1	1
8		IHTRP1109-NLA	NLA - Adjusting Rod Plunger	1	1
9		IHTRP1102R-NL A	NLA - Right Stretcher End Section	1	1
10		IHTRP1103-NLA	NLA - Stretcher Hanger Rod	1	1
11	B	TRV1305W-NLA	NLA - Stretcher Backrest, White	1	1
11	B	TRV1305A-NLA	NLA - Stretcher Backrest, Almond	1	1
12		IHTRP1202-NLA	NLA - Stretcher Body Rest - 12" Standard	1	1
13		IHTRP1034B-NL A	NLA - Tube, Foam (7/8" x 6")	1	1
14		IHTRP1102L-NL A	NLA - Left End Stretcher Section	1	1
15		IHTRV1213-NLA	NLA - Stretcher Back Pad	1	1
16		IHTRV1210-NLA	NLA - Blue Mesh Stretcher Cover	1	1
16		IHTRV1210N-NL A	NLA - Blue Mesh Stretcher Cover - Narrow	1	1
17		IHTRV1211-NLA	NLA - Solid Stretcher Cover	1	1
17		IHTRV1211N-NL A	NLA - Solid Stretcher Cover - Narrow	1	1
NOTE: A - Includes Items 2 thru 17. B - Specify color, almond or white when ordering.					


Model 1300 - Seat Assembly

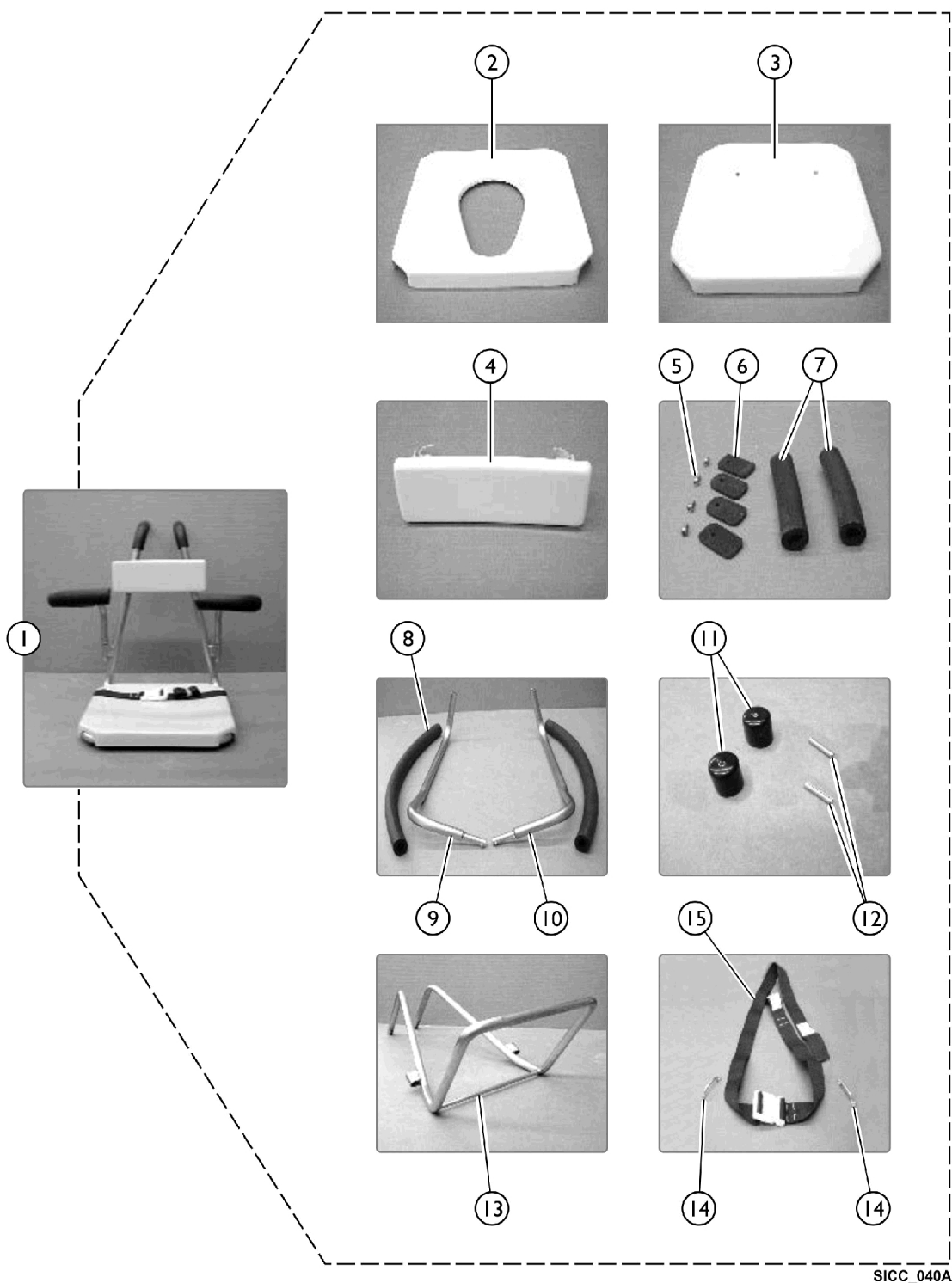

Model 1300 - Seat Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	IHTRV1301	Commode Seat Assembly (Ref. Only)	1	1
2		TRP1302A-NLA	NLA - Commode Seat - Almond	1	1
2		TRP1302W-NLA	NLA - Commode Seat - White	1	1
3		TRP1303A-NLA	NLA - Solid Seat - Almond	1	1
3		TRP1303W-NLA	NLA - Solid Seat - White	1	1
4	A	TRV1305W-NLA	NLA - Stretcher Backrest, White	1	1
4	A	TRV1305A-NLA	NLA - Stretcher Backrest, Almond	1	1
5		IHTRP1316-NLA	NLA - Screw, SS (1/4-20 x 1/2")	1	4
6		IHTRP1317-NLA	NLA - Plastic Lock Tab	1	4
7		IHTRP1034B-NL A	NLA - Tube, Foam (7/8" x 6")	1	2
8		IHTRP1034A-NL A	NLA - Tube, Foam (7/8" x 24")	1	2
9		IHTRP1304LC-N LA	NLA - Left Arm Rest, Seat Assembly	1	1
10		IHTRP1304RC-N LA	NLA - Right Arm Rest, Seat Assembly	1	1
11		IHTRP1037-NLA	NLA - End Cap (7/8")	1	2
12		IHTRP1308	Roll Pin (3/16" x 1")	1	2
13		IHTRP1315-NLA	NLA - Seal Hanger Rod	1	1
14		IHTRP9702-NLA	NLA - Footman Loop	1	1
15		IHPAR9721	Harness Strap	1	1
NOTE: A - Specify white or almond when ordering. B - Includes Items 2 thru 16.					

Model 1800 - C Base Seat Assembly

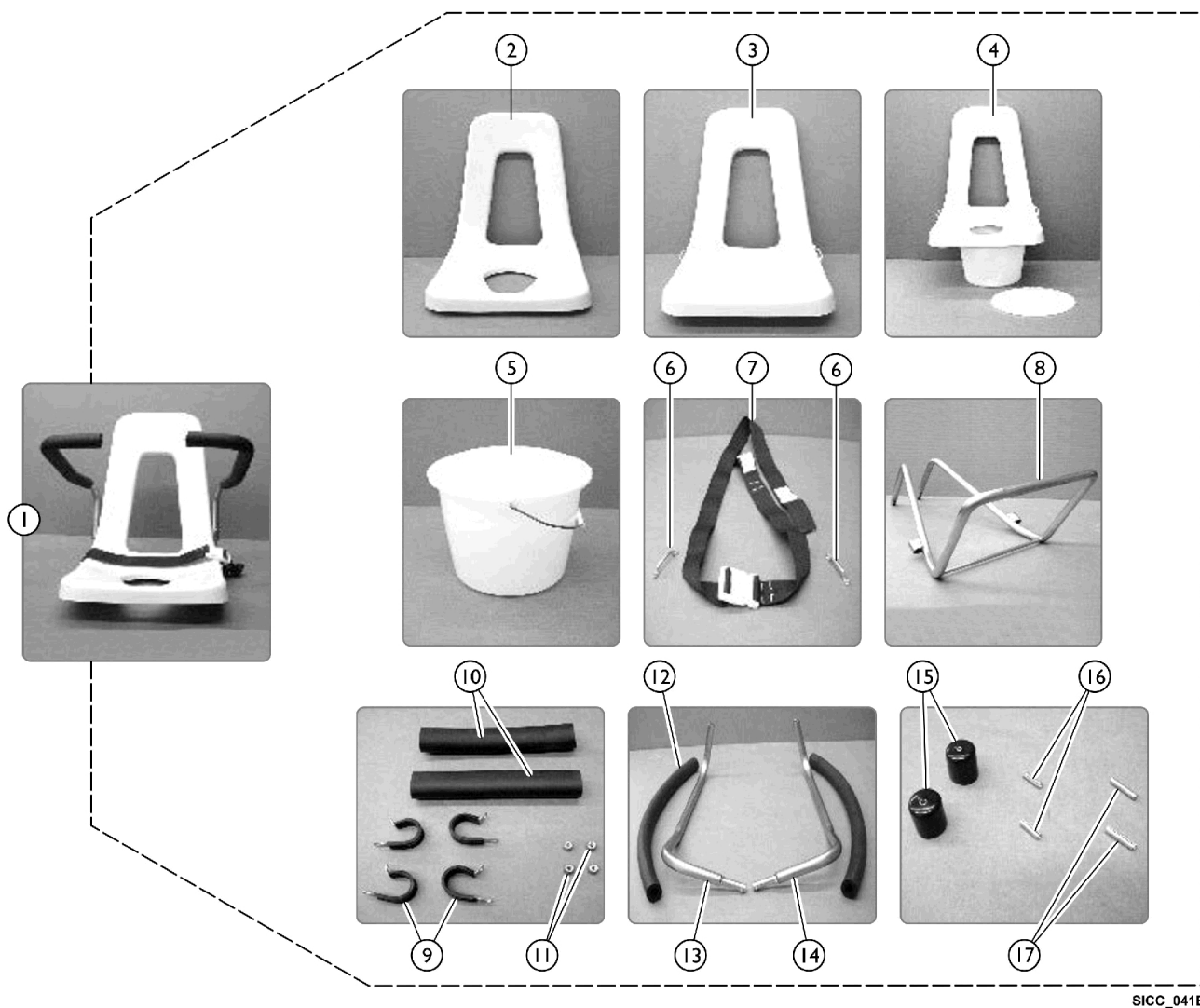


SICC_041B


Model 1800 - C Base Seat Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	IHTRV1801	Model 1800 Seat Assembly	1	1
2	C	TRP1802A-NLA	NLA - High Back Commode Seat - Almond	1	1
2	C	TRP1802W-NLA	NLA - High Back Commode Seat - White	1	1
3	C	TRV1803A-NLA	NLA - High Back Solid Seat - Almond	1	1
3	C	IHTRP1803W-NLA	NLA - High Back Solid Seat - White	1	1
4	C	IHTRP1802PA-NLA	NLA - High Back Solid Seat w/ Pail - Almond	1	1
4	C	IHTRP1802PW-NLA	NLA - High Back Solid Seat w/ Pail - White	1	1
5		IHTRP1804-NLA	NLA - Commode Pail Package	1	1
6		IHTRP9702-NLA	NLA - Footman Loop	1	1
7		IHPAR9721C	Harness Strap	1	1
8		IHTRP1302C-NLA	NLA - Seat Hanger Rod	1	1
9		IHTRP1806-NLA	NLA - C Clamp - 1" Tube	1	1
10		IHTRP1034C	Tube, Foam (7/8" x 8")	1	1
11		IHTRP1807-NLA	NLA - Locknut (1/4-20)	1	1
12		IHTRP1034A-NLA	NLA - Tube, Foam (7/8" x 24")	1	2
13		IHTRP1304LC-NLA	NLA - Left Arm Rest, Seat Assembly	1	1
14		IHTRP1304RC-NLA	NLA - Right Arm Rest, Seat Assembly	1	1
15		IHTRP1037-NLA	NLA - End Cap (7/8")	1	2
16		IHTRP1311	Roll Pin (3/16 x 3/4")	1	2
17		IHTRP1310-NLA	NLA - Roll Pin (3/16 x 1")	1	2
NOTE: A - Specify color and solid or commode when ordering. B - Includes Items 2 thru 17. C - Includes Items 6 and 7.					

Model 1100 - Scale Components

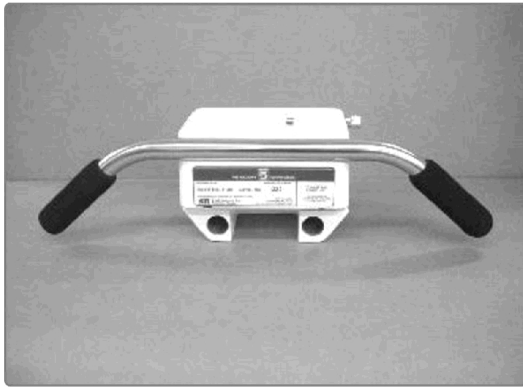

IHI 100 Scale

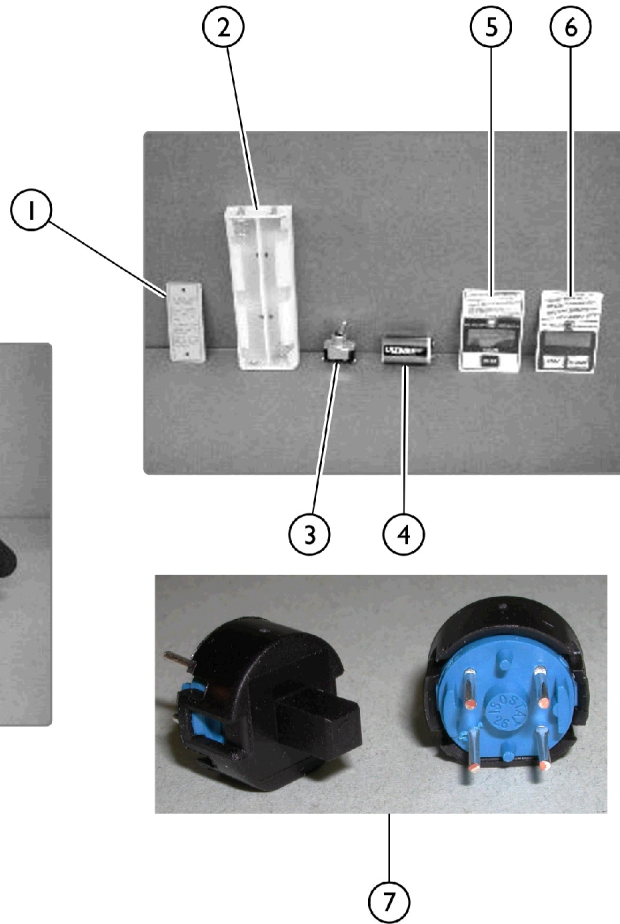


SICC_042B

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHPAR1173-NLA	NLA - Battery Cover, Scale	1	1
2		IHPAR1174-NLA	NLA - Battery Mount, C-Cell	1	1
3		IHPAR1175-NLA	NLA - Toggle Switch, Scale	1	1
4		IHPAR1176	Battery, Lithium - 9V	1	1
5		IHPAR1177-NLA	NLA - Label, One Button, Scale	1	1
6		IHPAR1178	Label, Two Button, Scale	1	1
7		IHPAR1179	Button, Zero-Out Scale	1	1

Model 1900 - Mast Assembly

Includes Labels, Shrouds, Battery, Controller and Actuator

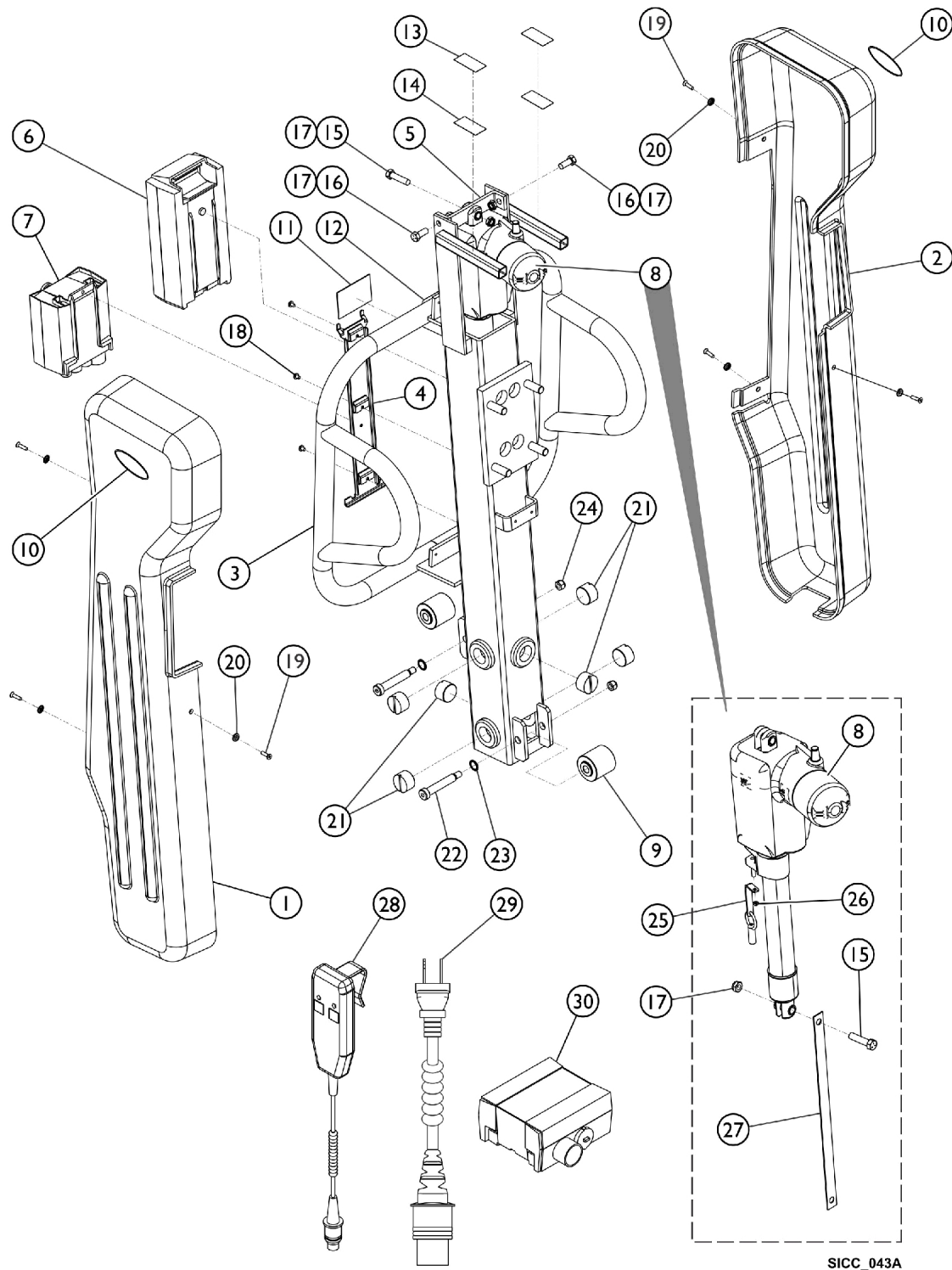

Model 1900 - Mast Assembly


Includes Labels, Shrouds, Battery, Controller and Actuator

Parts List

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147312-NLA	NLA - Shroud, Upper Left w/o Scale	1	1
2		1147298-NLA	NLA - Shroud, Upper Right w/o Scale	1	1
3		1147275	Bar, Lift Support, Mast	1	1
4		1078284	Bracket, Battery - Electric Only	1	1
5		1147293	NLA - Bracket, Bar, Cross	1	1
6		1078276	Battery Pack, Linak	1	2
7		1078275	Controller - Electric Only	1	1
8		1147306	Actuator, Mast	1	1
9		1147296	Roller, Mast, Outer	1	2
10		1090147-NLA	NLA - Decal, Invacare Medallion - Large	1	2
11		PAR9990-NLA	NLA - Label, Serial and Model Number	1	1
12		1111029-NLA	NLA - Decal, Capacity, Weight 350 lb. Canadian	1	1
13	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	--	--
14	A	1026082-NLA	NLA - Fastener, Dual Lock	--	--
15		1147365-NLA	NLA - Screw, Hex Head (3/8-16 x 1-1/2")	1	2
16		1147364-NLA	NLA - Screw, Hex Head (3/8-16 x 7/8")	1	2
17		40363X000	Hex Nut (3/8"-16)	1	4
18		1078980	Screw, Phillips Pan Head (#10-24 x 1/4")	1	3
19		IHPAR9911	Screw, Phillips, Flat Head (#10-24 x 3/4")	1	6
20		60105597	Washer, #10 Countersunk Finishing	1	6
21		1147292	Plug, Threaded - Plastic	1	6
22		1147363	Screw, Shoulder, Socket (1/8 x 2-1/2")	1	2
23		1154071-NLA	NLA - Washer, Bellville, Separated (17/32 x 3/4 x 1/32")	1	2
24		1089061-NLA	NLA - Locknut (3/8-16)	1	2
25		1154073-NLA	NLA - Bracket, L Shaped, Actuator	1	1
26		1147369-NLA	NLA - Locknut, Hex (M4 x .7)	1	1
27		1147297-NLA	NLA - Strapping, Actuator	1	1
28		1177604	Pendant, Two Channel w/ Hook	1	1
29		1079133	Cord, Power, USA	1	1
30	B	1078274	Charger, Electric Only w/o Power Cord	1	1
NOTE: A - Pieces must be cut from roll into 2" by 2" Squares (2 each). B - Power Cord not included with Charger.					

Model 1900S - Mast Assembly

Includes Labels, Shrouds, Battery, Controller, Actuator and Scale

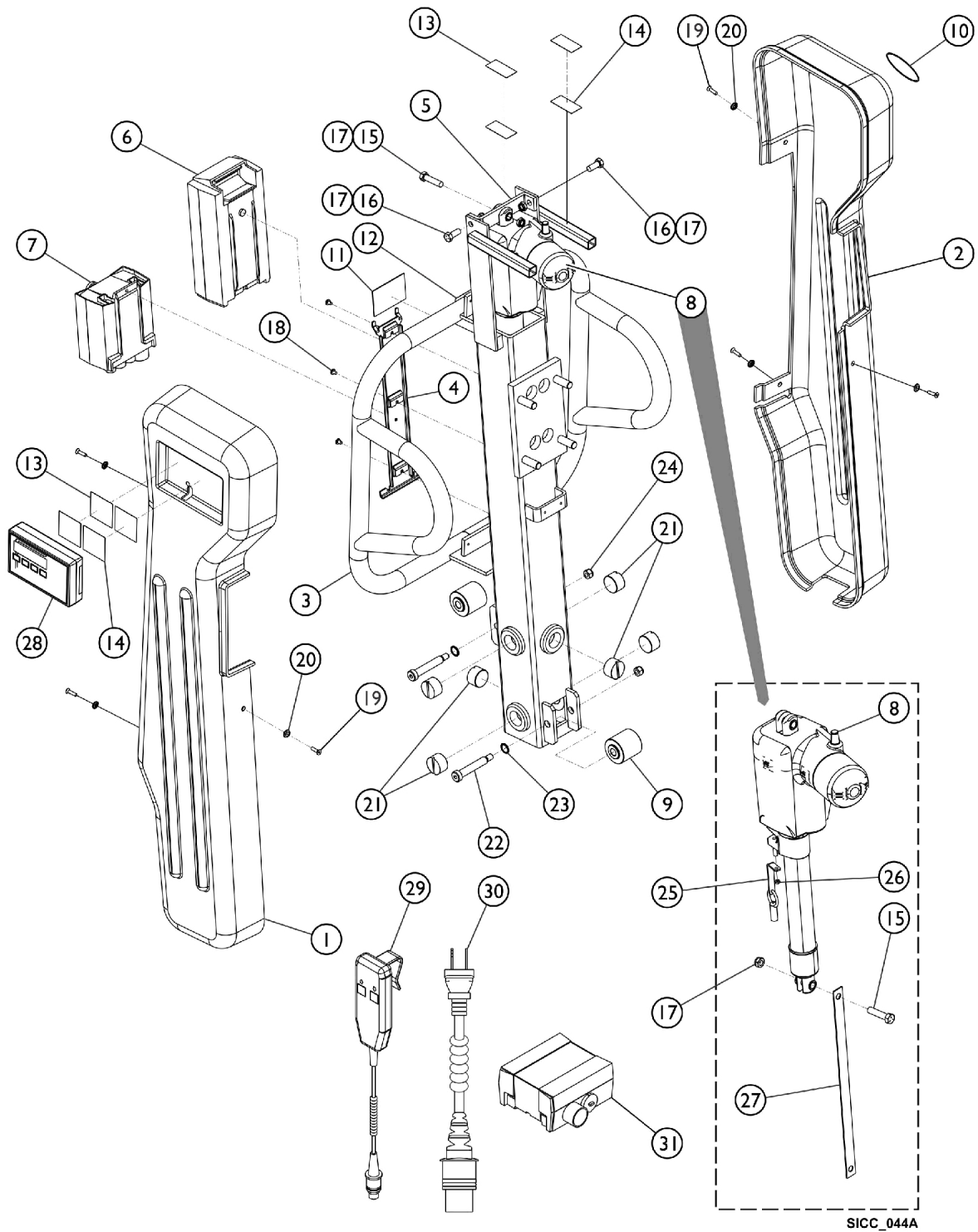

Model 1900S - Mast Assembly


Includes Labels, Shrouds, Battery, Controller, Actuator and Scale

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147299-NLA	NLA - Shroud, Upper Left w/ Scale	1	1
2		1147298-NLA	NLA - Shroud, Upper Right w/o Scale	1	1
3		1147275	Bar, Lift Support, Mast	1	1
4		1078284	Bracket, Battery - Electric Only	1	1
5		1147293-NLA	NLA - Bracket, Bar, Cross	1	1
6		1078276	Battery Pack, Linak	1	2
7		1078275	Controller - Electric Only	1	1
8		1147306	Actuator, Mast	1	1
9		1147296	Roller, Mast, Outer	1	2
10		1090147-NLA	NLA - Decal, Invacare Medallion - Large	1	2
11		PAR9990-NLA	NLA - Label, Serial and Model Number	1	1
12		1111029-NLA	NLA - Decal, Capacity, Weight 350 lb. Canadian	1	1
13	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
14	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1
15		1147365-NLA	NLA - Screw, Hex Head (3/8-16 x 1-1/2")	1	2
16		1147364-NLA	NLA - Screw, Hex Head (3/8-16 x 7/8")	1	2
17		40363X000	Hex Nut (3/8"-16)	1	4
18		1078980	Screw, Phillips Pan Head (#10-24 x 1/4")	1	3
19		IHPAR9911	Screw, Phillips, Flat Head (#10-24 x 3/4")	1	6
20		60105597	Washer, #10 Countersunk Finishing	1	6
21		1147292	Plug, Threaded - Plastic	1	6
22		1147363	Screw, Shoulder, Socket (1/8 x 2-1/2")	1	2
23		1154071-NLA	NLA - Washer, Bellville, Separated (17/32 x 3/4 x 1/32")	1	2
24		1089061-NLA	NLA - Locknut (3/8-16)	1	2
25		1154073-NLA	NLA - Bracket, L Shaped, Actuator	1	1
26		1147369-NLA	NLA - Locknut, Hex (M4 x .7)	1	1
27		1147297-NLA	NLA - Strapping, Actuator	1	1
28	B,C	1159620	Kit, Digital Scale, 1900S	1	1
			NLA - Digital Scale Assembly, 1900S	1	1
			Instruction Sheet, Digital Scale	1	1
			NLA - Hook, 2" Adhesive Back	1	1
		1159621	Kit, Load Cell and Connector	1	1
			Load Cell and Connector Assembly	1	1
			Instruction Sheet, Digital Scale	1	1
29		1177604	Pendant, Two Channel w/ Hook	1	1
30		1079133	Cord, Power, USA	1	1
31	D	1078274	Charger, Electric Only w/o Power Cord	1	1

NOTE: A - Pieces must be cut from roll into 2" by 2" Squares (2 each).
B - Includes Digital Scale, Instruction Sheet, and Hook.
C - If Lift produced prior to 7/1/07 then order part number 1159621 Load Cell and Connector Kit.
D - Power Cord not included with Charger.

Model 1900 - Patient Seat Assembly

Includes Patient Seat, Arms and Shroud

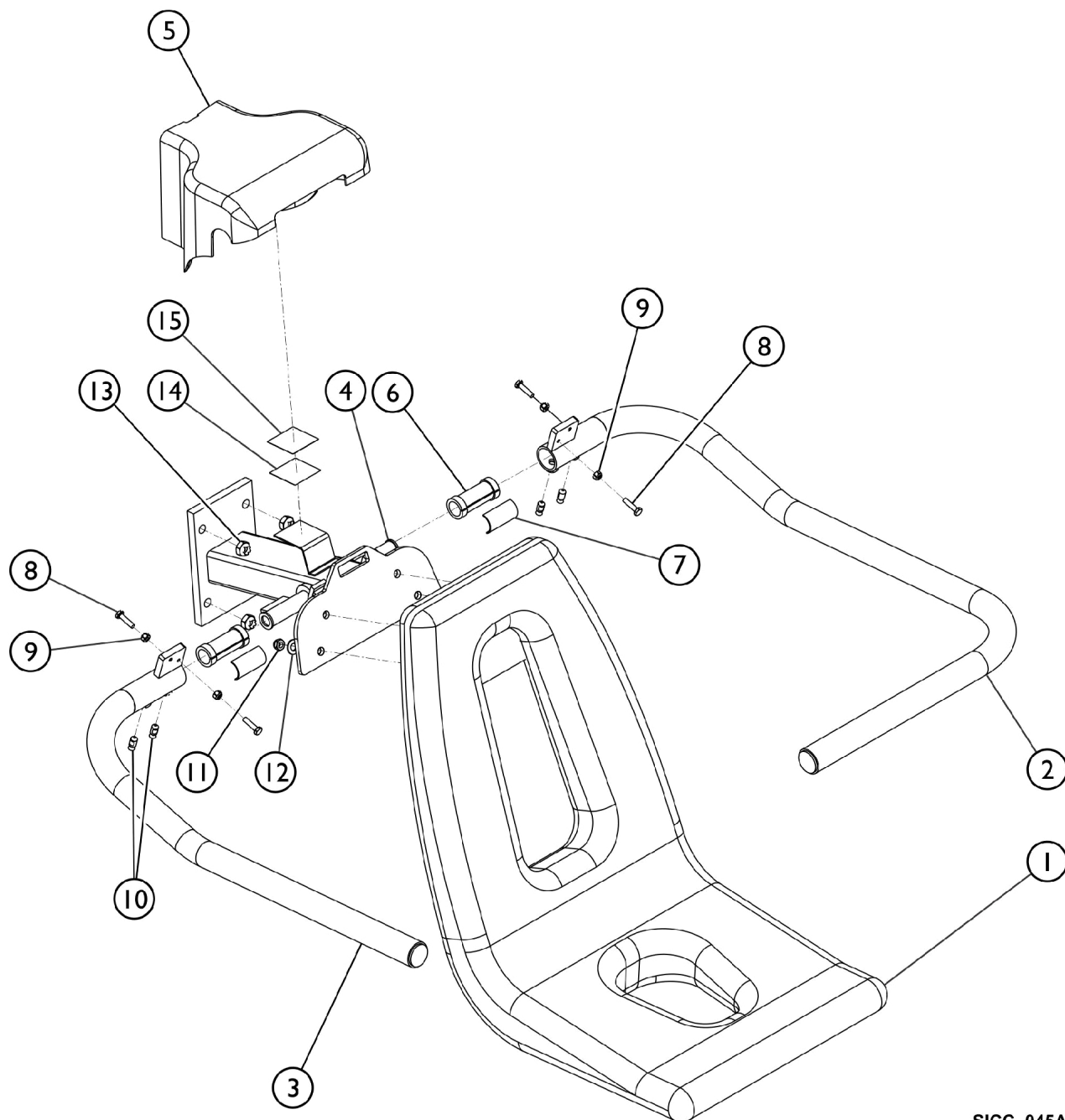

SICC_045A


Model 1900 - Patient Seat Assembly

Includes Patient Seat, Arms and Shroud

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147282	Seat, Patient, Lift	1	1
2		1147278	Arm, Seat, Lift - Right	1	1
3		1147277	Arm, Seat, Lift - Left	1	1
4		1147283-NLA	NLA - Support, Seat, Lift w/o Load Cell	1	1
5		1147304	Shroud, Seat, Arm	1	1
6		1147279-NLA	NLA - Bearing, Arm, Seat	1	2
7		1147372-NLA	NLA - Shoe, Brake, Bearing	1	2
8		PAR9954-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
9		IHTRP1807-NLA	NLA - Locknut (1/4-20)	1	4
10		1142191	Screw, Set, Socket, Cup Point (3/8-16 x 5/8")	1	4
11		40363X000	Hex Nut (3/8"-16)	1	4
12		1153346-NLA	NLA - Washer, Flat (13/32 x 13/16 x 1/16")	1	4
13		1095483-NLA	NLA - Locknut (1/2-13)	1	4
14	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
15	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1
	B	1147313	Belt, Lap, Seat	1	1
	B	1151805	Belt, Chest, Seat	1	1
NOTE: A - Cut item from roll into 2" by 2" square pieces (1 each). B - Item not shown in illustration.					

Model 1900S - Patient Seat Assembly

Includes Patient Seat, Arms, Shroud and Load Cell

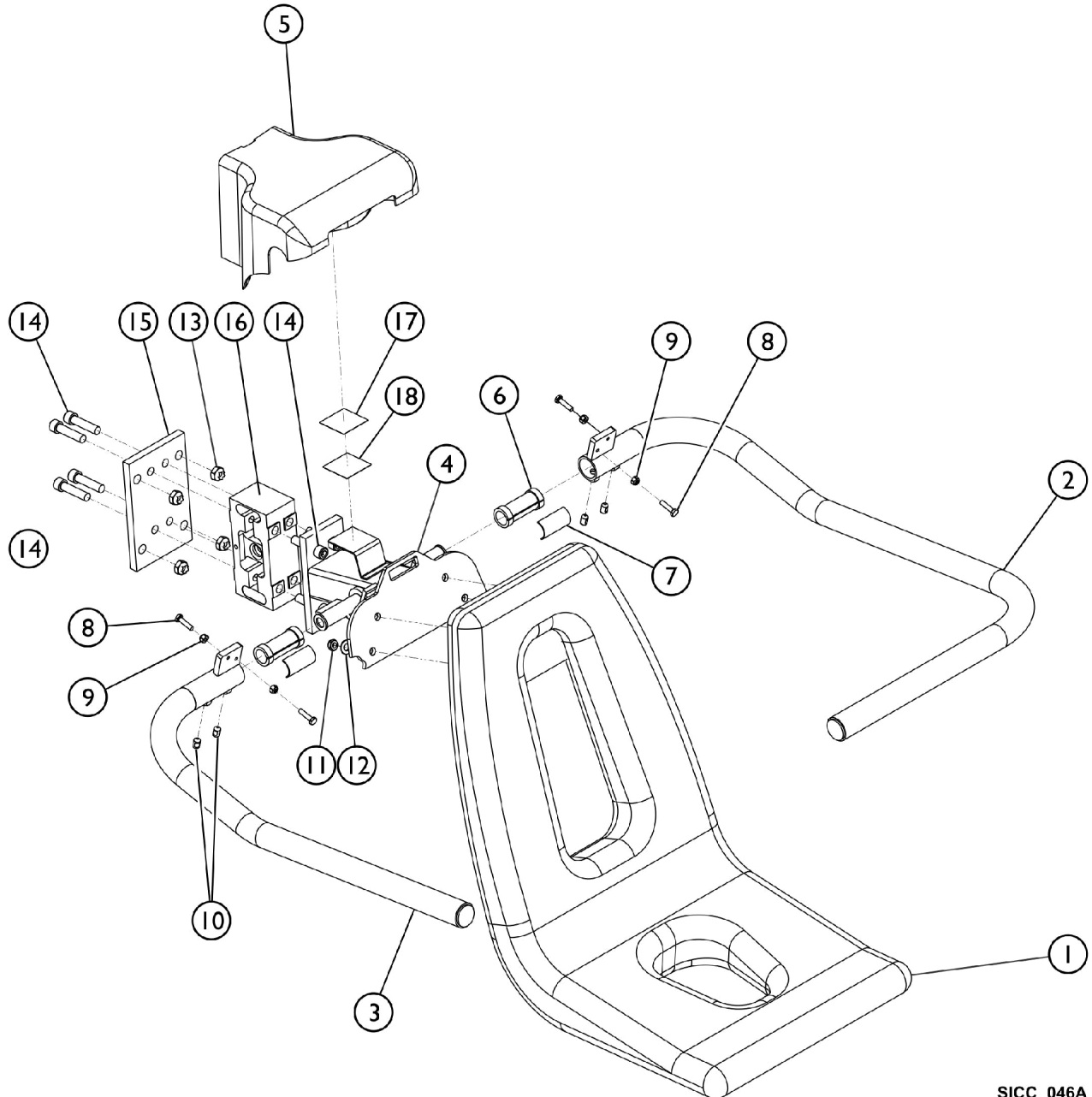

SICC_046A


Model 1900S - Patient Seat Assembly

Includes Patient Seat, Arms, Shroud and Load Cell

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147282	Seat, Patient, Lift	1	1
2		1147278	Arm, Seat, Lift - Right	1	1
3		1147277	Arm, Seat, Lift - Left	1	1
4		1147283-NLA	NLA - Support, Seat, Lift w/o Load Cell	1	1
5		1147304	Shroud, Seat, Arm	1	1
6		1147279-NLA	NLA - Bearing, Arm, Seat	1	2
7		1147372-NLA	NLA - Shoe, Brake, Bearing	1	2
8		PAR9954-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
9		IHTRP1807-NLA	NLA - Locknut (1/4-20)	1	4
10		1142191	Screw, Set, Socket, Cup Point (3/8-16 x 5/8")	1	4
11		40363X000	Hex Nut (3/8"-16)	1	4
12		1153346-NLA	NLA - Washer, Flat (13/32 x 13/16 x 1/16")	1	4
13		1095483-NLA	NLA - Locknut (1/2-13)	1	4
14		1147366-NLA	NLA - Screw, Cap Socket (M12 - 1.75 x 50mm)	1	8
15		1147307-NLA	NLA - Plate, Base, Load Cell	1	1
16	C	1159621	Kit, Load Cell and Connector	1	1
			Load Cell and Connector Assembly	1	1
			Instruction Sheet, Digital Scale	1	1
17	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
18	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1
	B	1147313	Belt, Lap, Seat	1	1
	B	1151805	Belt, Chest, Seat	1	1
NOTE: A - Cut item from roll into 2" by 2" square pieces (1 each). B - Item not shown in illustration. C - Part includes Load Cell and Connector and Instruction Sheet.					

Model 1900 - K Style Base Assembly

Includes K Style Base, Casters and Lower Shroud

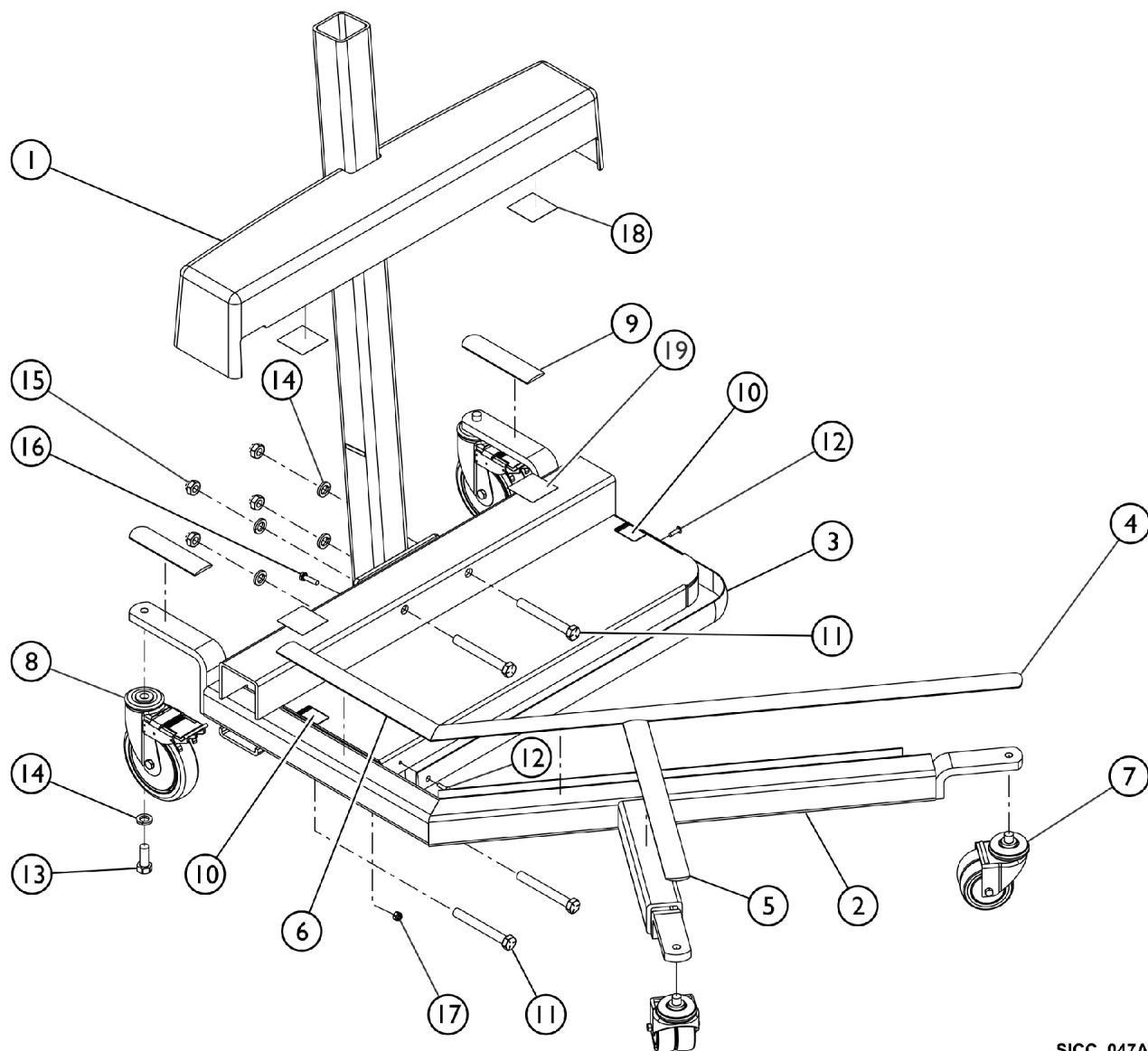

SICC_047A

Model 1900 - K Style Base Assembly

Includes K Style Base, Casters and Lower Shroud

Parts List

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147301	Shroud, Lower - Base	1	1
2		1147273	Frame Assembly, K Style - Lift	1	1
3		1147295-NLA	NLA - Trim, Strip, Pan	1	1
4		1147286-NLA	NLA - Trim, Strip (33")	1	1
5		1147287-NLA	NLA - Trim, Strip (15")	1	1
6		1147285-NLA	NLA - Trim, Strip (12")	1	1
7		1123560	Caster, Swivel Dual (3")	1	2
8		1123539	Caster Assembly, Swivel w/ Brake (5")	1	2
9		1147288-NLA	NLA - Trim, Strip (5")	1	2
10		1079203	Decal, Warning Pinch Point	1	2
11		1147361-NLA	NLA - Screw, Hex Head (1/2-13 x 4")	1	4
12		IHTRP1614-NLA	NLA - Rivet, Pop, Aluminum	1	2
13		1073884	Screw, Hex Head (1/2-13 x 1-1/4")	1	2
14		1014371	Lockwasher (1/2")	1	6
15		1095483-NLA	NLA - Locknut (1/2-13)	1	4
16		PAR9954-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
17		IHTRP1807-NLA	NLA - Locknut (1/4-20)	1	1
18	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
19	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1

NOTE: A - Cut item from roll into 2" by 1" pieces (2 each).



One Invacare Way
Elyria, OH USA
44035
800-333-6900

Customer/Technical Support
800-668-2337
Service Availability
Monday - Friday

Hours Available
8 AM - 6 PM EST

© 2022 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. (Revised 08/24/22)