
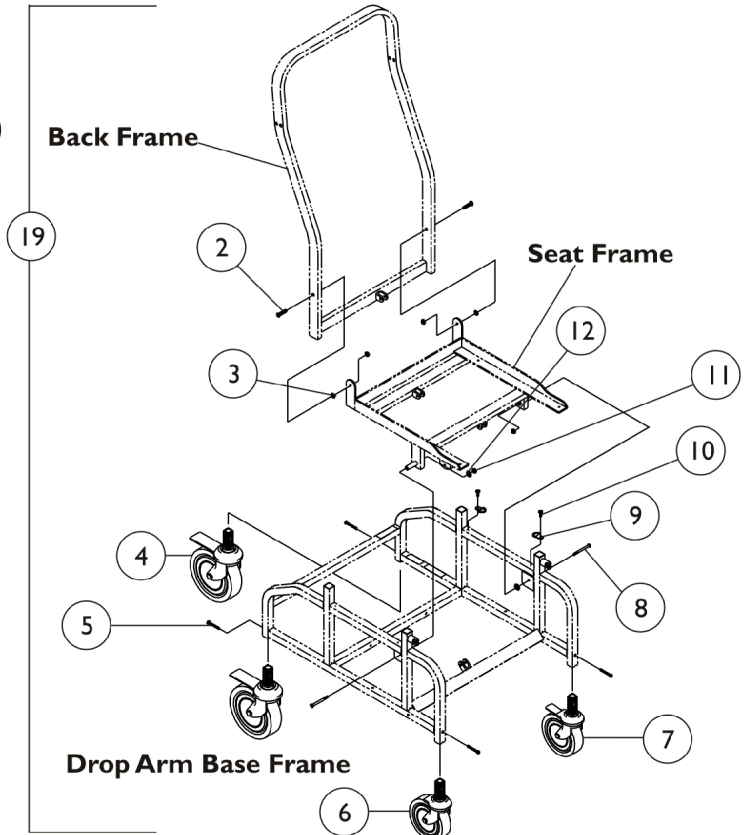


Base Frames and Hardware

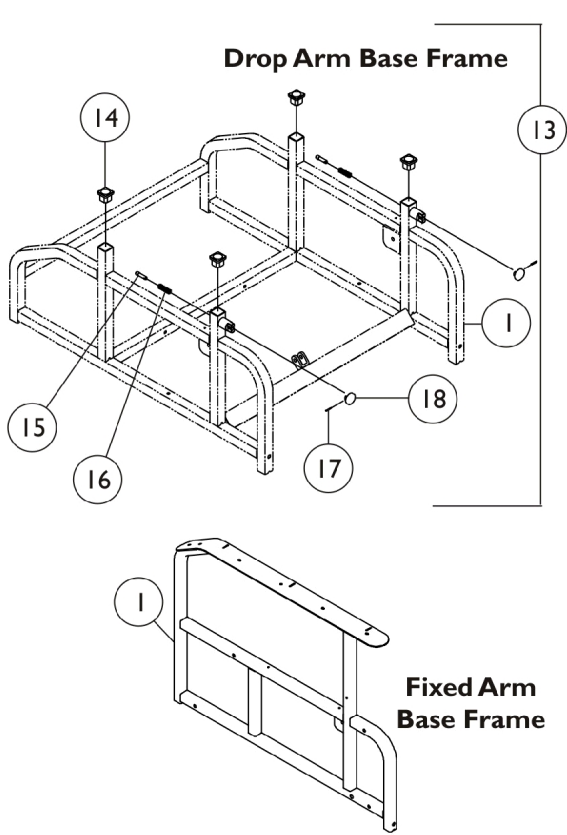

Base Frames and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1071225-NLA	NLA - Frame, Base, Fixed Arm, Grey	1	1
1	A	1069969-NLA	NLA - Frame, Base, Drop Arm, Grey	1	1
2		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	2
3		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
4		1069974-NLA	NLA - Caster with Brake - Rear (5")	1	2
5		1065215	Screw, Socket Button Head w/ Patch (5/16-18 x 3/4")	1	4
6		1069975-NLA	NLA - Caster, Swivel - Front (4")	1	1
7	A	1069973-NLA	NLA - Caster, Front Steering (4")	1	1
8		1037110	Screw, Hex Head (5/16-24 x 3-1/2")	1	2
9	A	1029407-NLA	NLA - Ring, "D"	1	2
10	A	1022597	Screw, Phillips Pan Head Tap (#10-32 x 3/8")	1	2
11		1026906-NLA	NLA - Locknut (5/16-24)	1	4
12		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
13	C	1071230-NLA	Kit, Drop Arm Frame Hardware	1	1
14			NLA - Glide, Plastic, Black	1	4
15			Pin, Locking	1	2
16			Spring, Arm Lock	1	2
17			NLA - Pin, Roll (5/64 x 5/8")	1	2
18			NLA - Knob	1	2
19	A,D	1071231-NLA	Kit, Comfort Plus Hardware	1	1
19	E	1073999-NLA	Kit, CP200 Only Hardware	1	1

NOTE: A - This is not standard equipment on the CP200 model.
C - Kit Includes Items 14-18.
D - Kit Includes Items 2-11.
E - Kit Includes Items 2-6, 8 and 11.