


# Ottoman Hardware

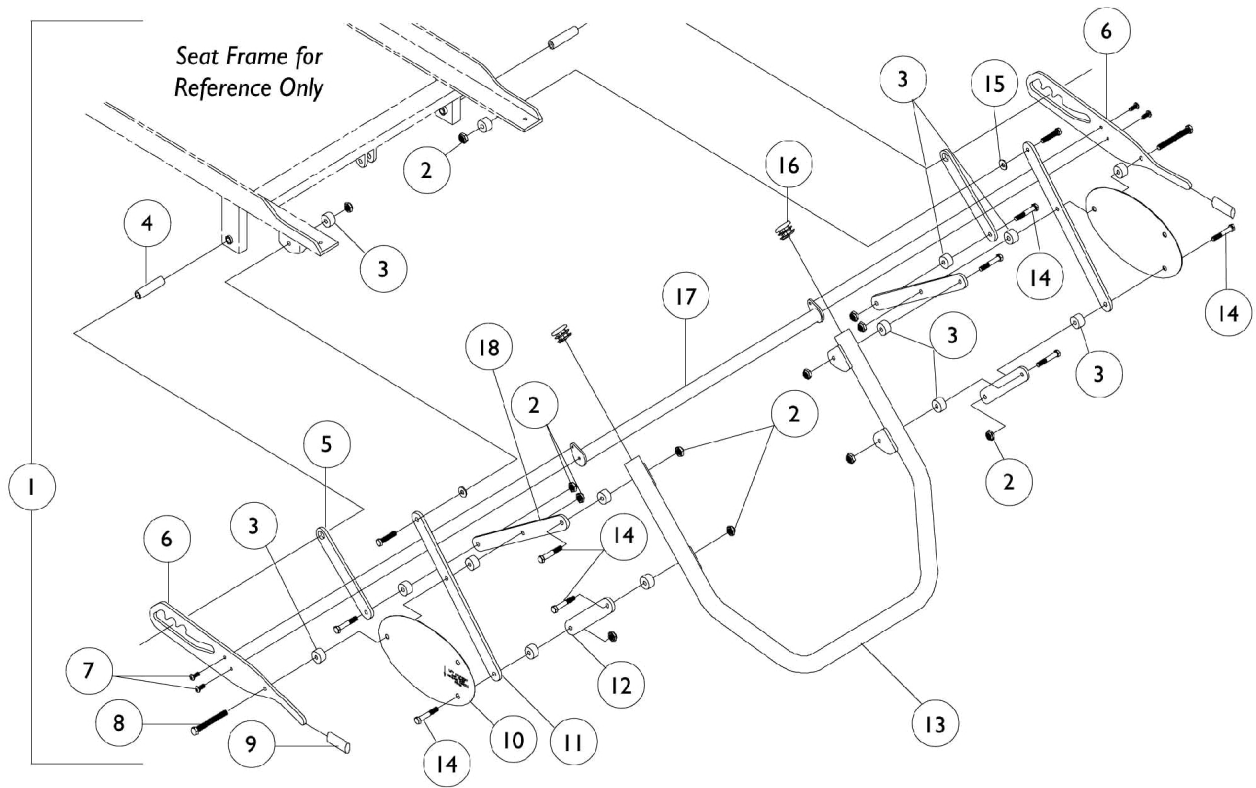

## Ottoman Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1071225-NLA	NLA - Frame, Base, Fixed Arm, Grey	1	1
2		7930333	Locknut (1/4-20)	25	1
3		1070001-NLA	NLA - Spacer, Nylon (1/4 X 3/4 X 7/16")	1	14
4		1069959-NLA	NLA - Bushing, Pivot	1	2
5		1069247-NLA	NLA - Link (5-1/8")	1	2
6		1070002-NLA	NLA - Ratchet Plate - Right	1	1
6		1070003-NLA	NLA - Ratchet Plate - Left	1	1
7		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	4
8		1024796-NLA	NLA - Screw, Hex Head (1/4-20 x 2")	1	2
9		1006490-NLA	NLA - Tip, Plastic, Black	1	2
10		1073640-NLA	NLA - Shield, Link - Right	1	1
10		1073641-NLA	NLA - Shield, Link - Left	1	1
11		1069248-NLA	NLA - Link (9-7/8")	1	2
12		1069250-NLA	NLA - Link (4-3/8")	1	2
13		1069971-NLA	NLA - Ottoman Weldment	1	1
14		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	10
15		1026106-NLA	NLA - Washer (1/4 x 5/8 x 1/16")	1	2
16		1029501	Plug Button, Black (1")	1	2
17		1069996-NLA	NLA - Connector, Ratchet	1	1
18		1069249-NLA	NLA - Link (8-1/2")	1	2

NOTE: A - Includes items 2-15