


I.V. Pole

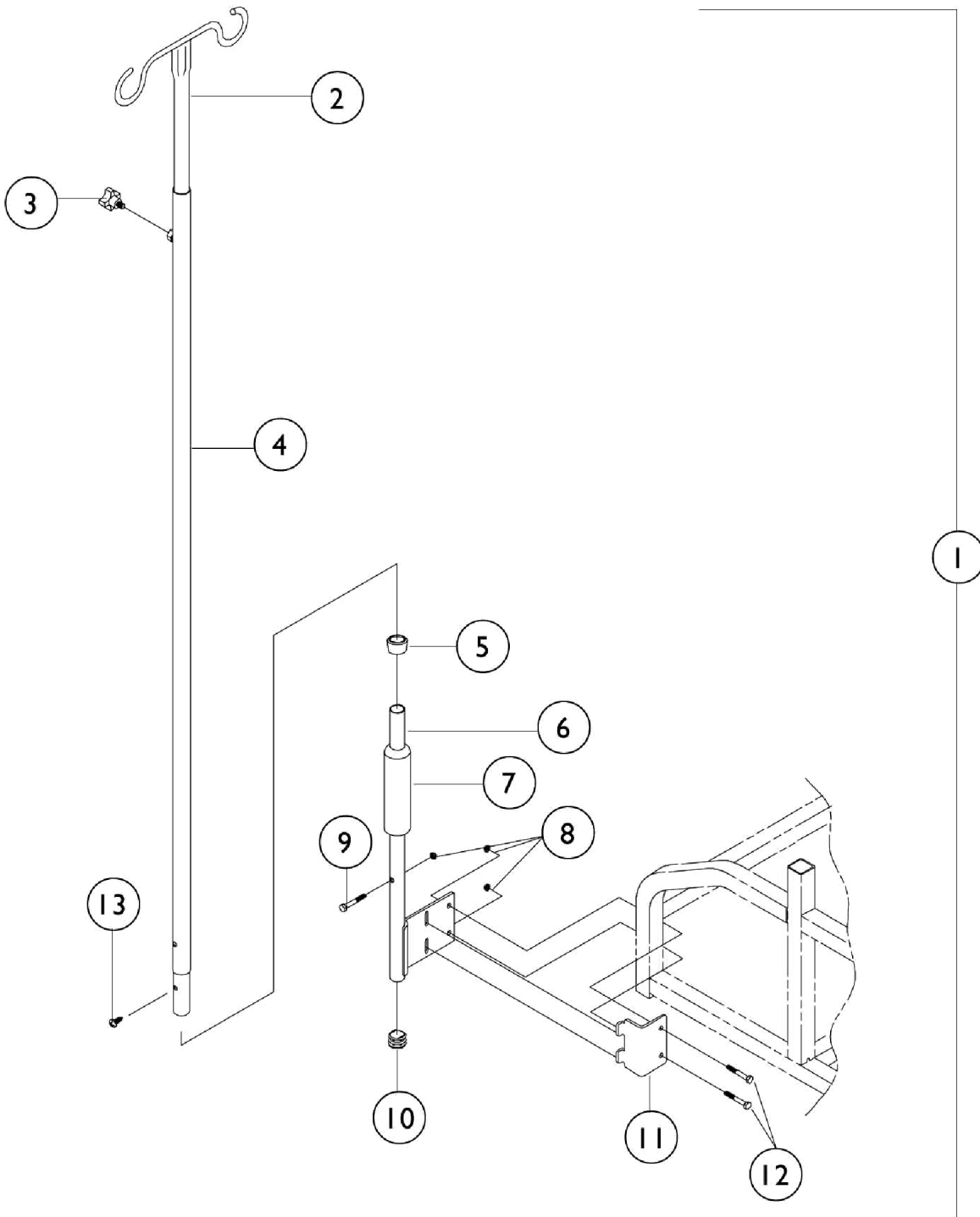

I.V. Pole

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1073160-NLA	NLA - I.V. Pole Assembly	1	1
2		04701X041	Rod, I.V., Recliner, Black	1	1
3		1027683	Knob, Adjustment, Black	1	1
4		1076521	Tube, Support, I.V. Rod, Black	1	1
5		15680X012	Collar, Anti-Rattle (7/8")	1	1
6		1071244	Bracket, I.V. Pole, Grey	1	1
7		1055398	Grip, Hand, Foam (4")	1	1
8		7930333	Locknut (1/4-20)	25	1
9		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
10		1027095	Plug Button, Black (7/8")	1	1
11		1073169-NLA	NLA - Bracket, I.V.	1	1
12		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
13		21243X000-NLA	NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	1
	A	1075648-NLA	NLA - Kit, Instruction Sheet, I.V. Rod	1	1

NOTE: A - Not Shown
B - Includes items 2-13