
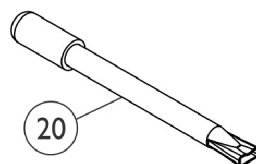
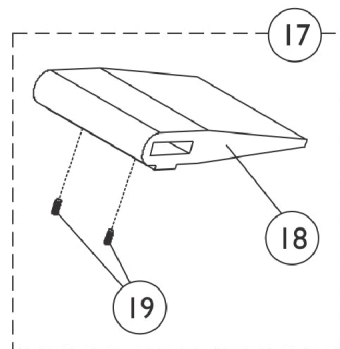
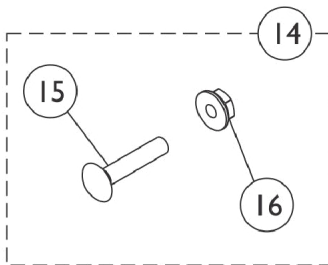


Link-Pivot Style Wheel Locks

Adult - Permanent Arms, Tracer EX and Tracer MG Permanent Arms

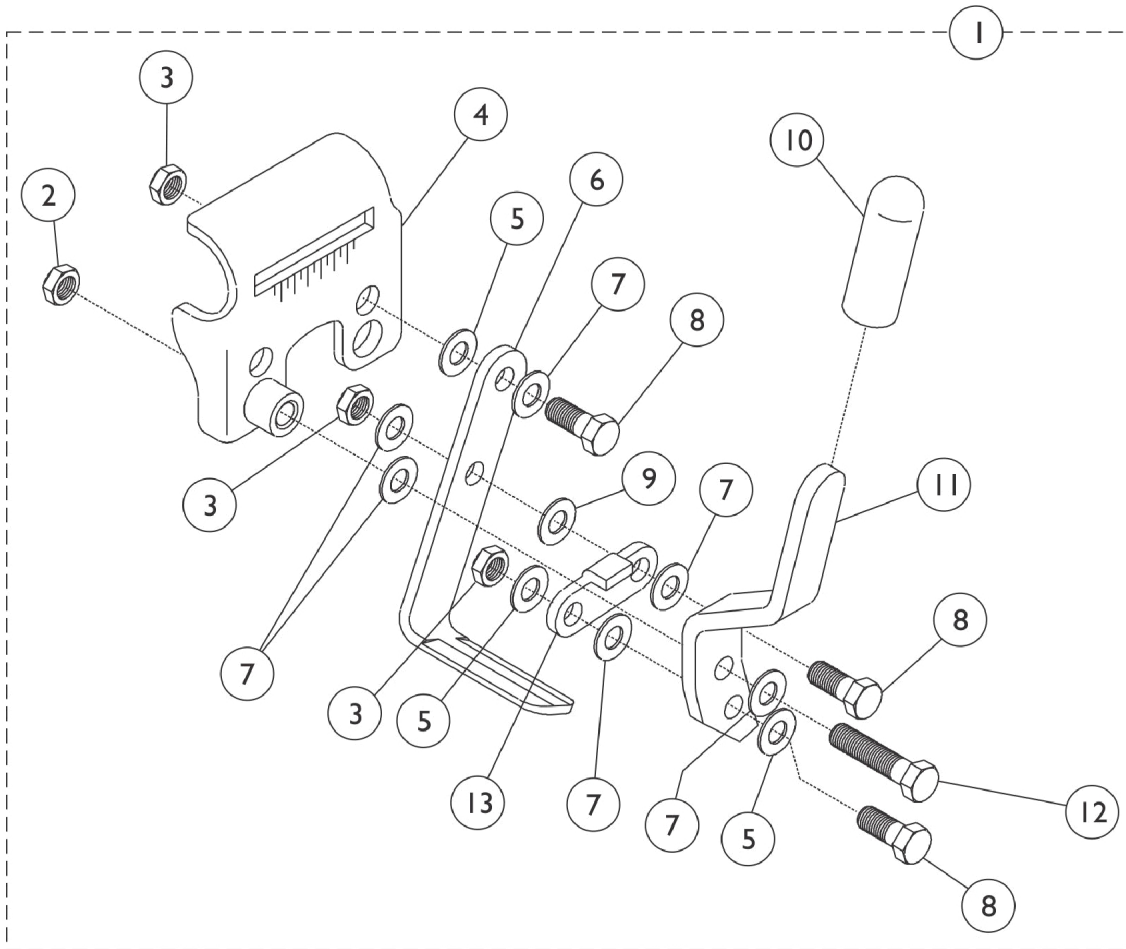

Link-Pivot Style Wheel Locks

Adult - Permanent Arms, Tracer EX and Tracer MG Permanent Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,H	1045861-NLA	NLA - Wheel Lock Assembly, Adult, Push to Lock, Black - Right	1	1
1	A,B,H	1045862-NLA	NLA - Wheel Lock Assembly, Adult, Push to Lock, Black - Left	1	1
1	B	1023493	Wheel Lock Assembly, Push to Lock - Right	1	1
1	B	1023494	Wheel Lock Assembly, Push / Lock - Left	1	1
1	A,C	1045862-NLA	NLA - Wheel Lock Assembly, Adult, Push to Lock, Black - Left	1	1
1	A,C	1045861-NLA	NLA - Wheel Lock Assembly, Adult, Push to Lock, Black - Right	1	1
2		60275X000	Locknut (1/4-20)	1	1
3		60320X000	Locknut (1/4-28)	1	1
4	D	1015445-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
4	D	1015446-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
4	E	1045843-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
4	E	1045844-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
5		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	1
6		1015443-NLA	NLA - Shoe	1	1
7		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	6
8		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	3
9		40157X004	Spacer, Nylon (1/4 x 7/16 x 1/8")	1	4
9		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	1
10		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
11		1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
11		1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
12		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
13		1016592-NLA	NLA - Link, Chrome	1	1
14	I	1034165	NLA - Hardware, Attaching Kit (Pair)	1	1
15			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
16			Locknut (1/4-20)	1	2
14		1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
15			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
16			Nut, Hex Flang (1/4-20)	1	2
17	F	1037421	Kit, Wheel Lock Shoe Spacer, 20" Rear Wheels (Pair)	1	1
18			Spacer, Wheel Lock Shoe (2-63/64")	1	2
19			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
17	F	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	1	1
18			Spacer, Wheel Lock Shoe (1-45/64")	1	2
19			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
20	G	1058900	Wheel Lock Extension Handle Assembly, Black (8")	1	1

Link-Pivot Style Wheel Locks

Adult - Permanent Arms, Tracer EX and Tracer MG Permanent Arms

NOTE: A - Includes items 2-13.
B - Used on Adult with Permanent Arms.
C - Used on Transport Models.
D - Before 5/94
E - After 4/94
F - These parts are NOT included with the Wheel Lock.
G - Includes: 1 each Extension Handle and Rubber Tip. Chrome is No Longer Available.
H - No Longer Available. Use part #'s 1023493 and 1023494 as replacements.
I - 1034165 in Zinc Finish is NLA. Use part number 1017677 in Black finish as a replacement.