


Footrest Support Assembly

TLXH3A/C, T943A/C, TLXH3S, T943S, TRH13, CGH13

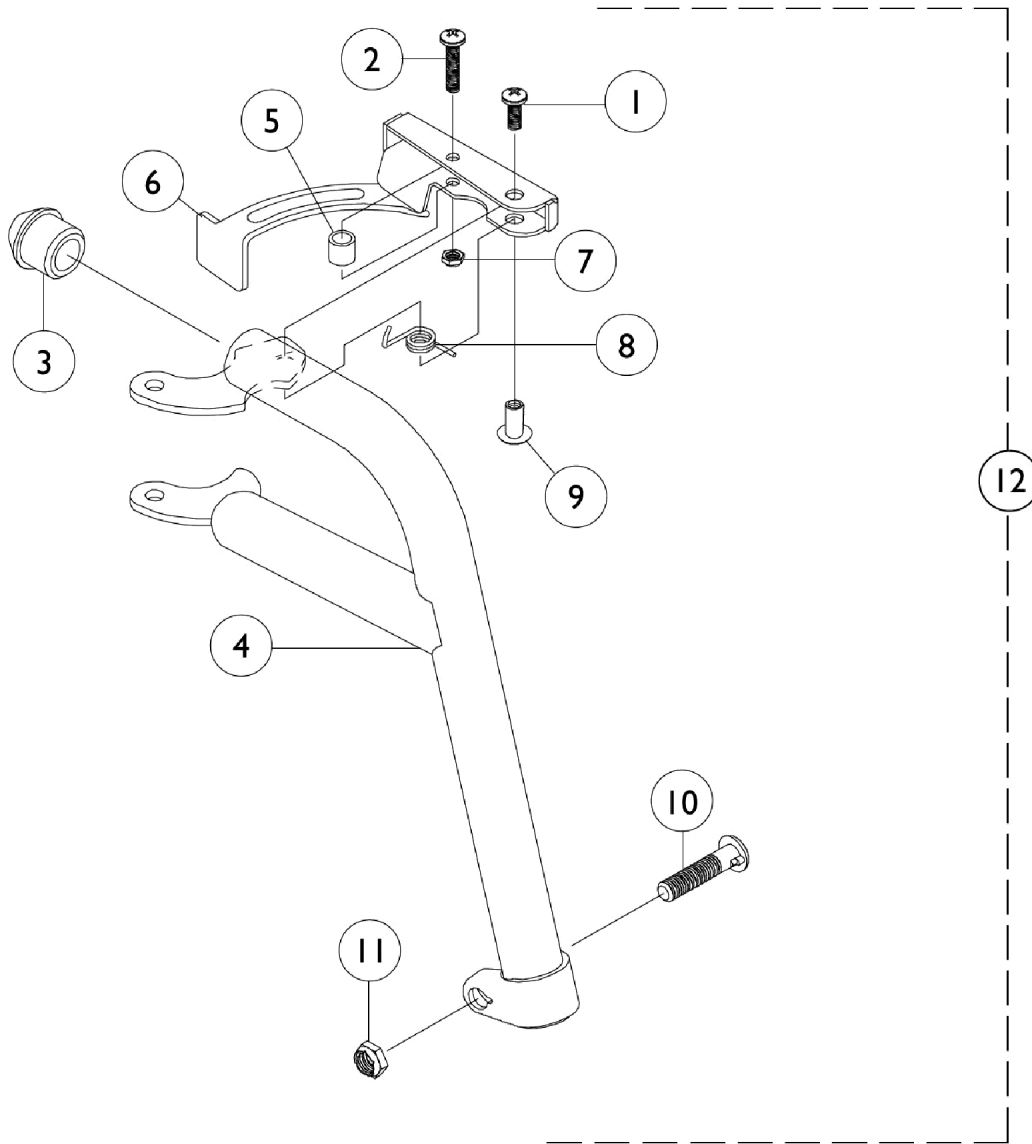

Footrest Support Assembly

TLXH3A/C, T943A/C, TLXH3S, T943S, TRH13, CGH13

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
2		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3		1020719	Anti-Rattle, Black	1	1
4		1020416-NLA	NLA - Tube, Support, Footrest, Chrome - Right	1	1
4		1020417-NLA	NLA - Tube, Support, Footrest, Chrome - Left	1	1
5		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
6		1023428-NLA	NLA - Handle, Release, Black- Right	1	1
6		1023431-NLA	NLA - Handle, Release, Black- Left	1	1
7		60279X003	Locknut (#10-32)	1	1
8		1008935	Spring, Release - Right	1	1
8		1008936	Spring, Release - Left	1	1
9		67005X000	Rivnut (#10-32)	1	1
10		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
11		1107321	Nut, Low Cap (5/16"-24)	1	1
12	A	See Chart	Item #12 Upper Support Assembly	1	1

NOTE: A - Includes items 1-11.

Item #12 Upper Support Assembly

Model Numbers	Chair Width	Support Assembly Right	Support Assembly Left
T943A, T943C, T943S TLXH3A, 5LXH3C, TLXH3S	14-18"	1030706-NLA	1030707-NLA
CGH13, TRH13	16-18"	1030706-NLA	1030707-NLA