


Footrest Support Assembly

TR13, CG13, T933A/C, TLX3A/C

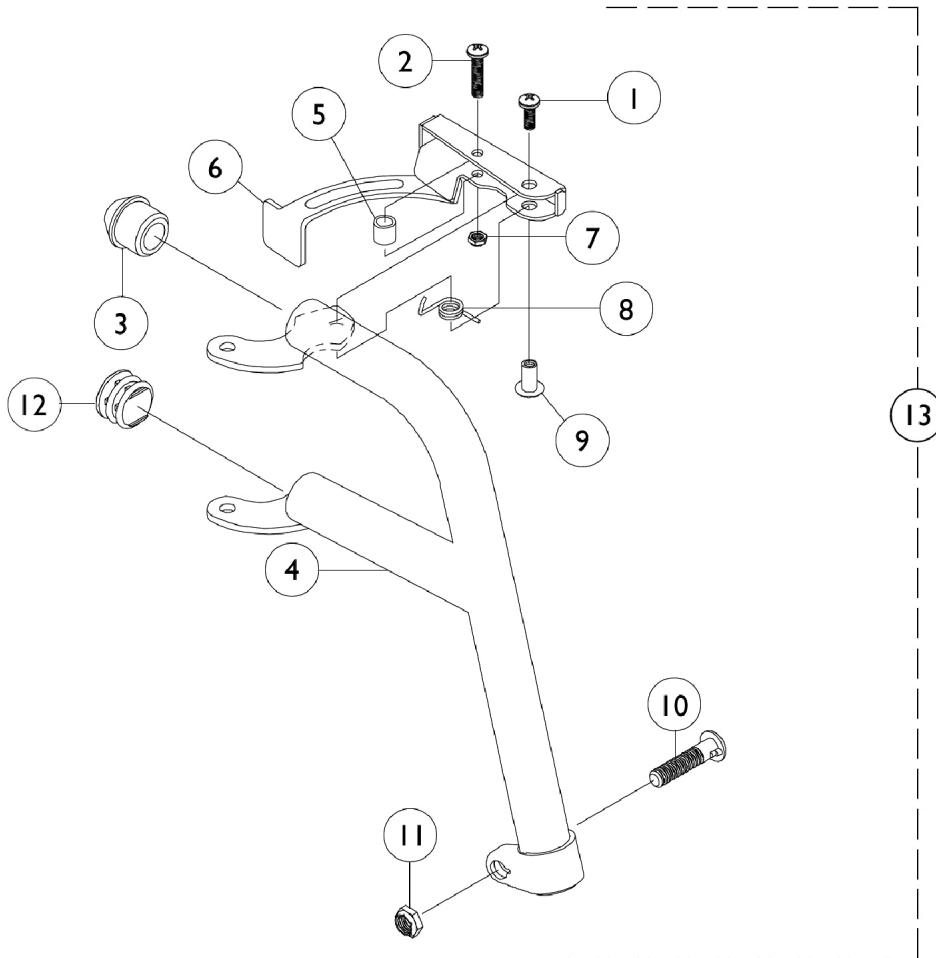

Footrest Support Assembly TR13, CG13, T933A/C, TLX3A/C

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
2		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3		1020719	Anti-Rattle, Black	1	1
4		1053170-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
4		1053171-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
5		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
6		1035539	Handle, Release, Black - Right	1	1
6		1035540	Handle, Release, Black - Left	1	1
7		60279X003	Locknut (#10-32)	1	1
8		1008935	Spring, Release - Right	1	1
8		1008936	Spring, Release - Left	1	1
9		67005X000	Rivnut (#10-32)	1	1
10		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
11		1107321	Nut, Low Cap (5/16"-24)	1	1
12		1027095	Plug Button, Black (7/8")	1	1
13	A	See Chart	Item #13 Upper Support Assembly	1	1

NOTE: A - Includes items 1-12.

Item #13 Upper Support Assembly

Model Numbers	Chair Width	Support Assembly Right	Support Assembly Left
CG13, TR13	14-20"	1053174-NLA	1053175-NLA
T93AA, T93AC,	16-22"	1110340	1110341
TLX3A, TLX3C	16-22"	See Note(s) B 1053174-NLA	See Note(s) B 1053175-NLA

NOTE: B - Color changed from Chrome to Black.