


# Legrest Support Assembly

T944A/C, T944S, TRH14, TRPH14, CGH14

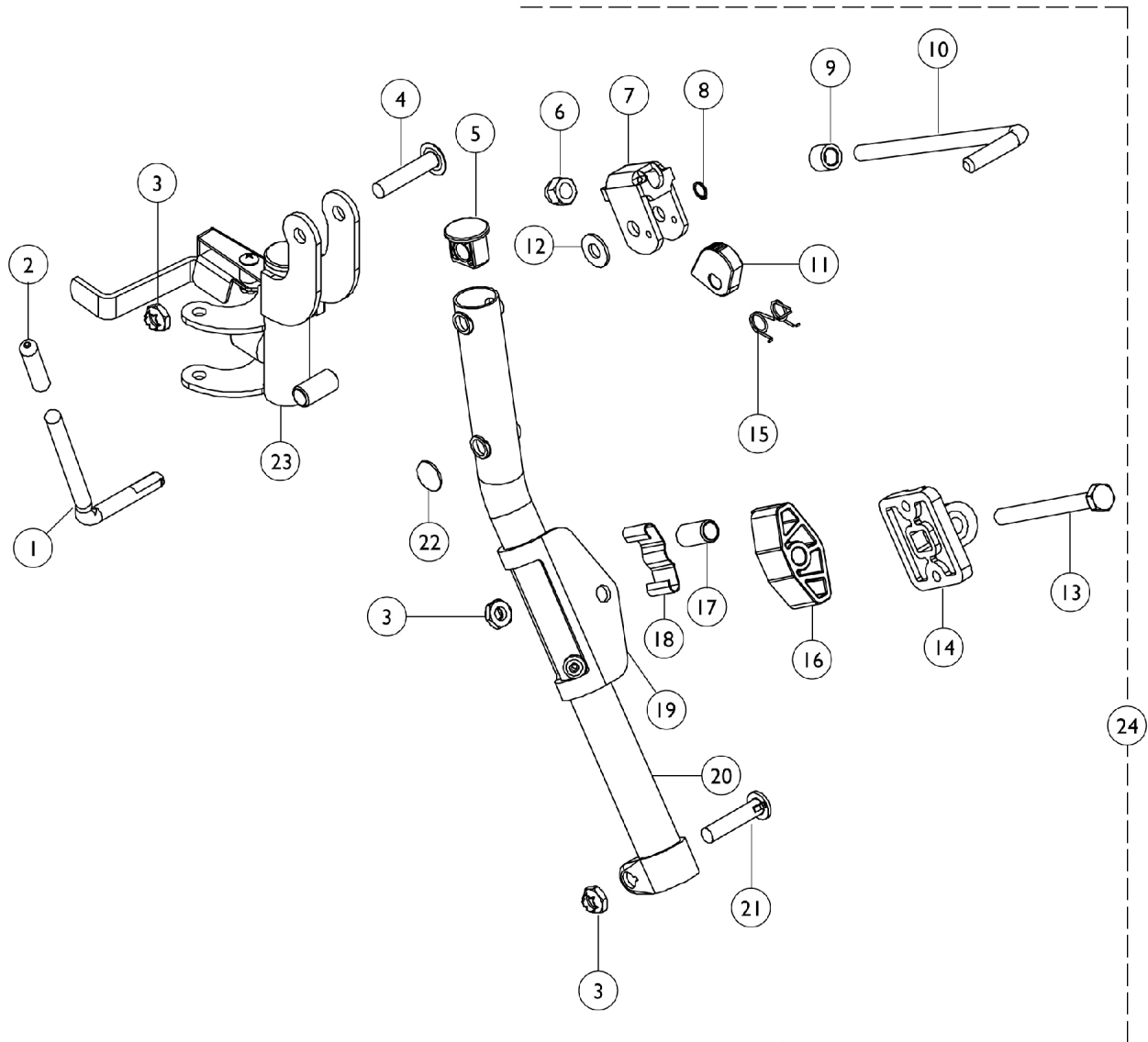

## Legrest Support Assembly

### T944A/C, T944S, TRH14, TRPH14, CGH14

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037126	Handle, Release	1	1
2		1040060	Tip, Plastic, Black	1	1
3		1107321	Nut, Low Cap (5/16"-24)	1	3
4		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
5		1095433	Plug Button, Black (7/8")	1	1
6		10261X000	Nut, Acorn (5/16-24)	1	1
7		04528X002	Bracket, Release, Black	1	1
8		1040831	Ring, Snap	1	1
9		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
10		42326X001	Bar, Ratchet	1	1
11		1037125	Cam, Legrest	1	1
12		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
13		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
14		04526X019	Plate, Legrest Pad	1	1
15		1037485	Spring, Release	1	1
16		1091220	Spacer, Legrest	1	1
17		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
18		48823X008	Spring, Leaf	1	1
19		1025104	Bracket, Legrest, Chrome	1	1
20	A	1095434-NLA	NLA - Tube, Support, Chrome - Right	1	1
20	A	1095435-NLA	NLA - Tube, Support, Chrome - Left	1	1
21		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
22		1096773	Nut, Push Dome (5/16")	1	1
23		1031356-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
23		1031357-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
	B,C	1046328-NLA	NLA - Kit, Ratchet	2	1
24	D	See Chart	Item #24 Upper Support Assembly	1	1
NOTE: A - Due to product improvement, item #5 must also be ordered. B - Includes items 1-2, 6-10, 12, and 15. C - Not shown D - Includes items 1-23. See chart titled, "Upper Support Assembly."					

### Item #24 Upper Support Assembly

#### Legrest

Prior Model Numbers	Chair Width	Support Assembly Right	Support Assembly Left
CGH14, TRH14 - Chrome	16-18"	1041822 See Note(s) B, A	1041823 See Note(s) B, A
T944A, T944C - Chrome	14-22"	1031360-NLA See Note(s) B, A	1031361-NLA See Note(s) A, B
NOTE: A - Without Calf Pads. B - Chrome			