


Legrest Support Assembly

TR14, TRP14, CG14, TLX4A/C, T934A/C

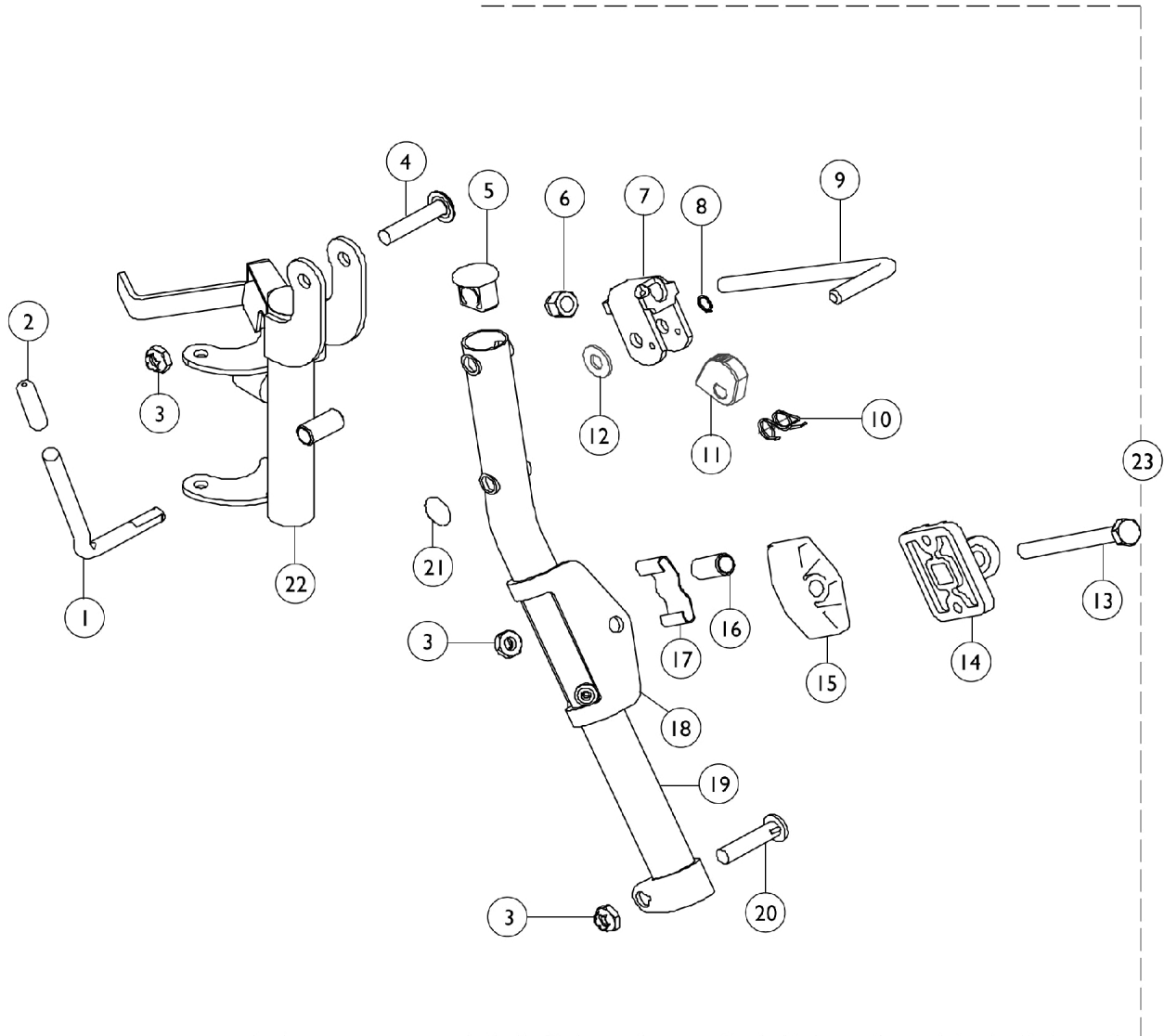

Legrest Support Assembly

TR14, TRP14, CG14, TLX4A/C, T934A/C

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037126	Handle, Release	1	1
2		1040060	Tip, Plastic, Black	1	1
3		1107321	Nut, Low Cap (5/16"-24)	1	3
4		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
5		1095433	Plug Button, Black (7/8")	1	1
6		10261X000	Nut, Acorn (5/16-24)	1	1
7		04528X002	Bracket, Release, Black	1	1
8		1040831	Ring, Snap	1	1
9		42326X001	Bar, Ratchet	1	1
10		1037485	Spring, Release	1	1
11		1037125	Cam, Legrest	1	1
12		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
13		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
14		04526X019	Plate, Legrest Pad	1	1
15		1091220	Spacer, Legrest	1	1
16		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
17		48823X008	Spring, Leaf	1	1
18		1025104	Bracket, Legrest, Chrome	1	1
19	A	1095434-NLA	NLA - Tube, Support, Chrome - Right	1	1
19	A	1095435-NLA	NLA - Tube, Support, Chrome - Left	1	1
20		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
21		1096773	Nut, Push Dome (5/16")	1	1
22		1031581-NLA	NLA - Legrest Support Tube Assembly, Adult, Black - Right	1	1
22		1031582	Legrest Support Tube Assembly, Adult, Black - Left	1	1
	B	1046328-NLA	NLA - Kit, Ratchet	2	1
23	C	See Chart	Item #23 Upper Support Assembly	1	1

NOTE: A - Due to product improvement, item #5 must also be ordered.
 B - Includes items 1-2, 6-10, and 12.
 C - Includes items 1-22.

Item #23 Upper Support Assembly

Legrest

Prior Model Numbers	Chair Width	Support Assembly Right	Support Assembly Left
CG14, TR14 - Black	14-18"	1041824 See Note(s) A, B	1041825 See Note(s) B, A
T934A, T934C, TLX4A, TLX4C, TRP14 - Black	16-22"	1041824 See Note(s) A	1041825 See Note(s) A

NOTE: A - Without Calf Pads.
 B - TR14 available for MGI International Accounts only.