

Aquatec Bath Safety Products

**Aquatec 90, Aquatec 900, Aquatec 9000
H720T, Marina Bath, Omega Commode**



Yes, you can.

Parts Catalogue Usage Guide

The information contained in this document is subject to change without notice. This catalogue contains service parts for Form No. 10-783C. If you have any questions regarding this catalogue, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-668-5324

For Faxed In Orders Call:

1-800-668-5478

For Technical Service Call:

1-800-668-5324

Literature Can Be Found at:

www.invacare.ca or

1-800-668-5324

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2016 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are registered trademarks of the Invacare

TABLE of CONTENTS

Description	Page Number
Aquatec 90 Toilet Seat Raiser	3
Aquatec 900 Toilet Seat Raiser	4
Aquatec 90000 Toilet Seat Raiser	6
H720T Glide About Commode	8
Marina Bath Board	11
Omega Commode	10


Aquatec 90 Toilet Seat Raiser



SPC_020A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		A16224	Lid w/ Hinge	1	1
2		16225	Clamps, Side (Pair)	2	1
3		16508	Pad, Non-Slip	1	4

Aquatec 900 Toilet Seat Raiser

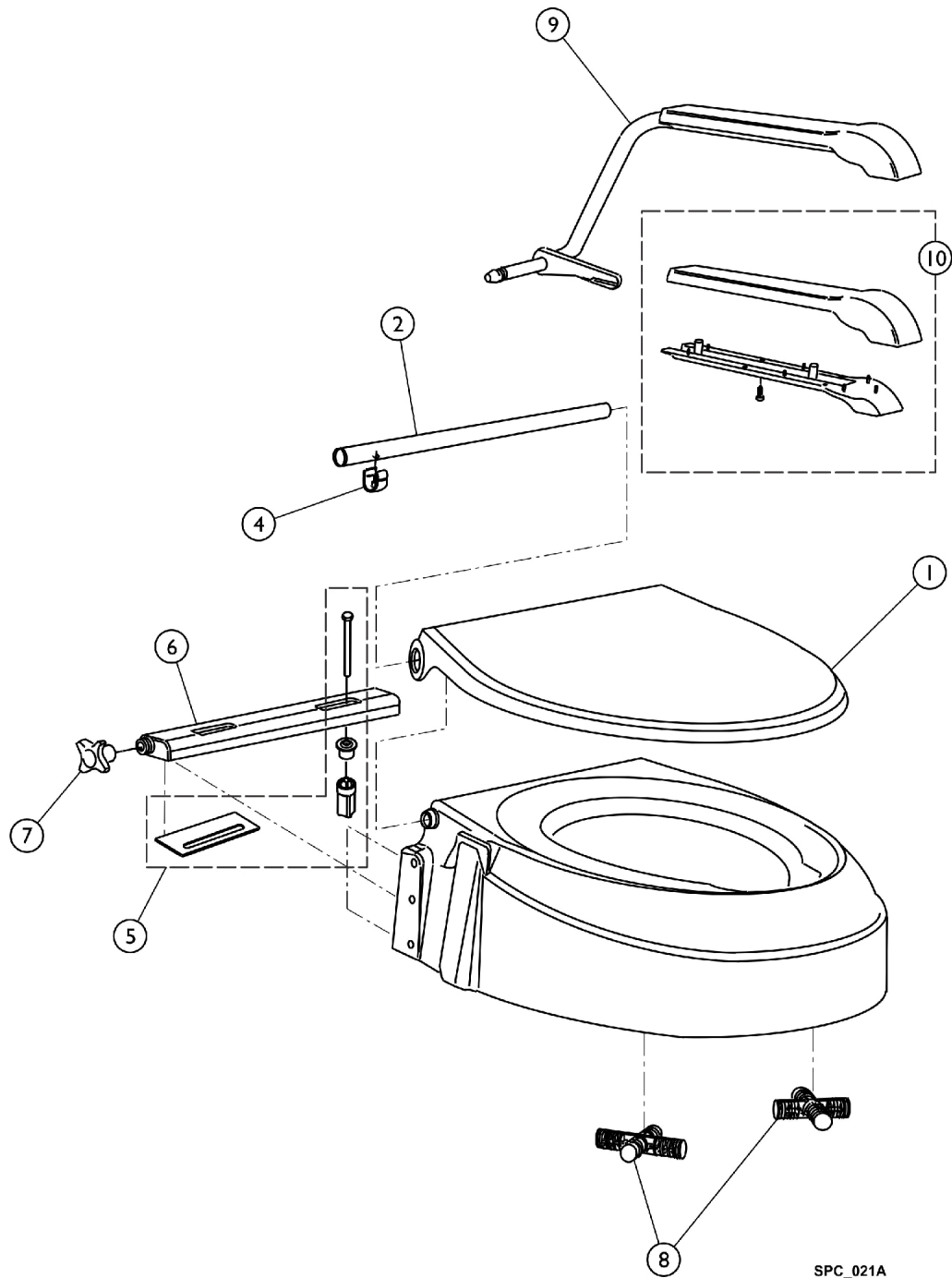


SPC_021A

Aquatec 900 Toilet Seat Raiser

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		A70170	Lid, Seat	1	1
2		72055	Tube, Rear, Lid	1	1
3		70048	Plug, Tube, Lid	1	2
4		70162	Clip, Tube, Lid	1	2
5		16565	Kit, Crossbar Installation, Lid	2	1
5		16582	Screw, Quick Tighten	1	2
5		1470909	Tape, Adhesive	1	2
6		70107	Crossbar, Seat	1	1
7		1470275	Knob, Four Point, Seat	1	2
8		70132	Cross, Attachment (Pair)	2	1
9		10105-10-2	Armrest (Pair)	2	1
10		A10158	Kit, Armrest Pad Upper/Lower with Screws	1	2

Aquatec 90000 Toilet Seat Raiser

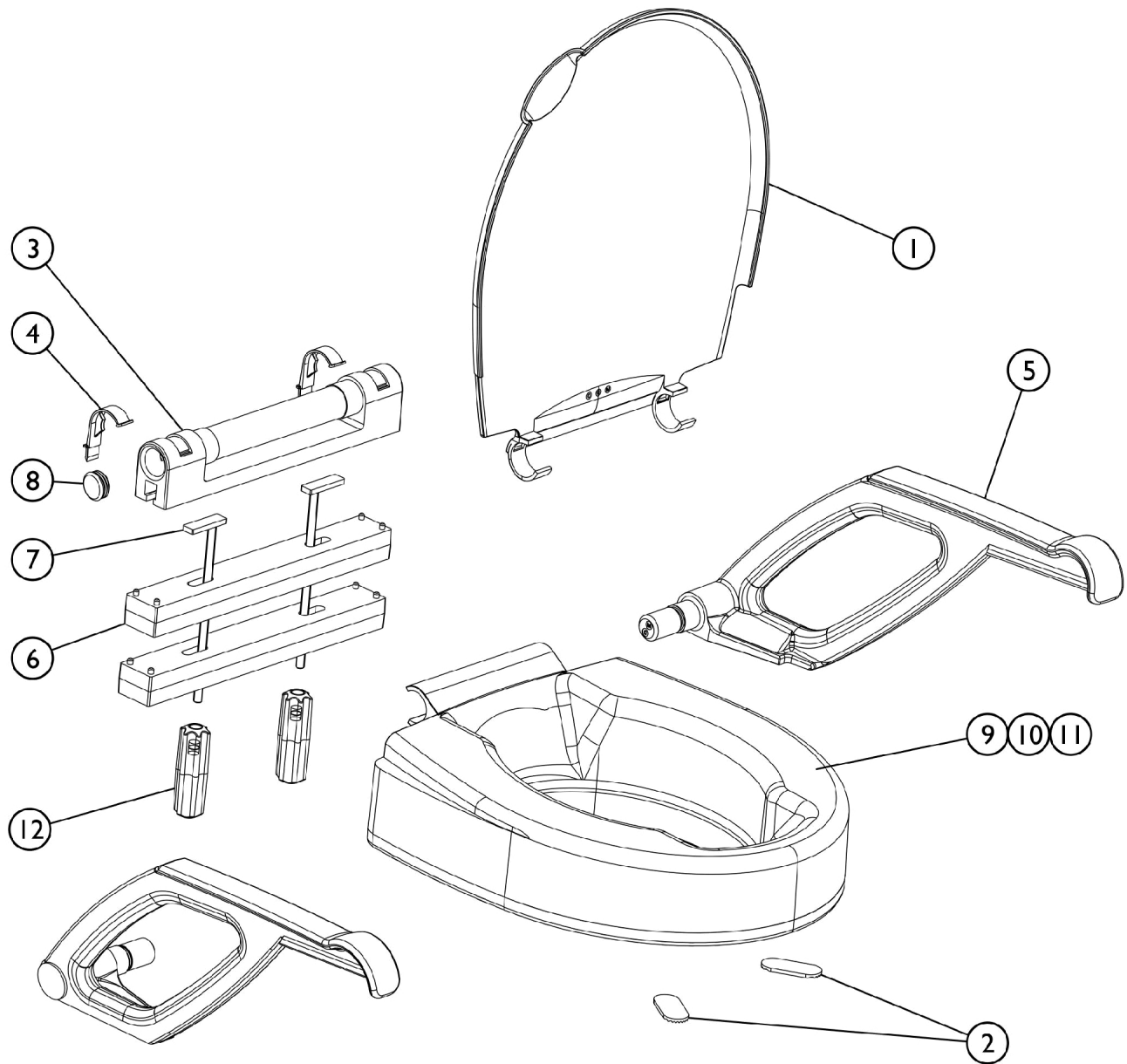


SPC_022A

Aquatec 90000 Toilet Seat Raiser

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		16796	Lid, Seat	1	1
2		16508	Pad, Non-Slip	1	2
3		16797	Hinge, Seat	1	1
4		16798	Clip, Hinge	2	1
5		16799	Armrests, Seat (Pair)	2	1
6		16800	Adapter, Height	1	2
7		1470794	Kit, Installation, Height Adapter	2	1
8		16802	Plug, Hinge	1	2
10		1470738	Base (2")	1	1
11		1470739	Base (4")	1	1
12		11073	Screws, Height Adapter	2	1
	A	1470825	Tilt Assembly, Height Adapter	1	1
NOTE: A - Not shown in illustration.					

H720T Glide About Commode

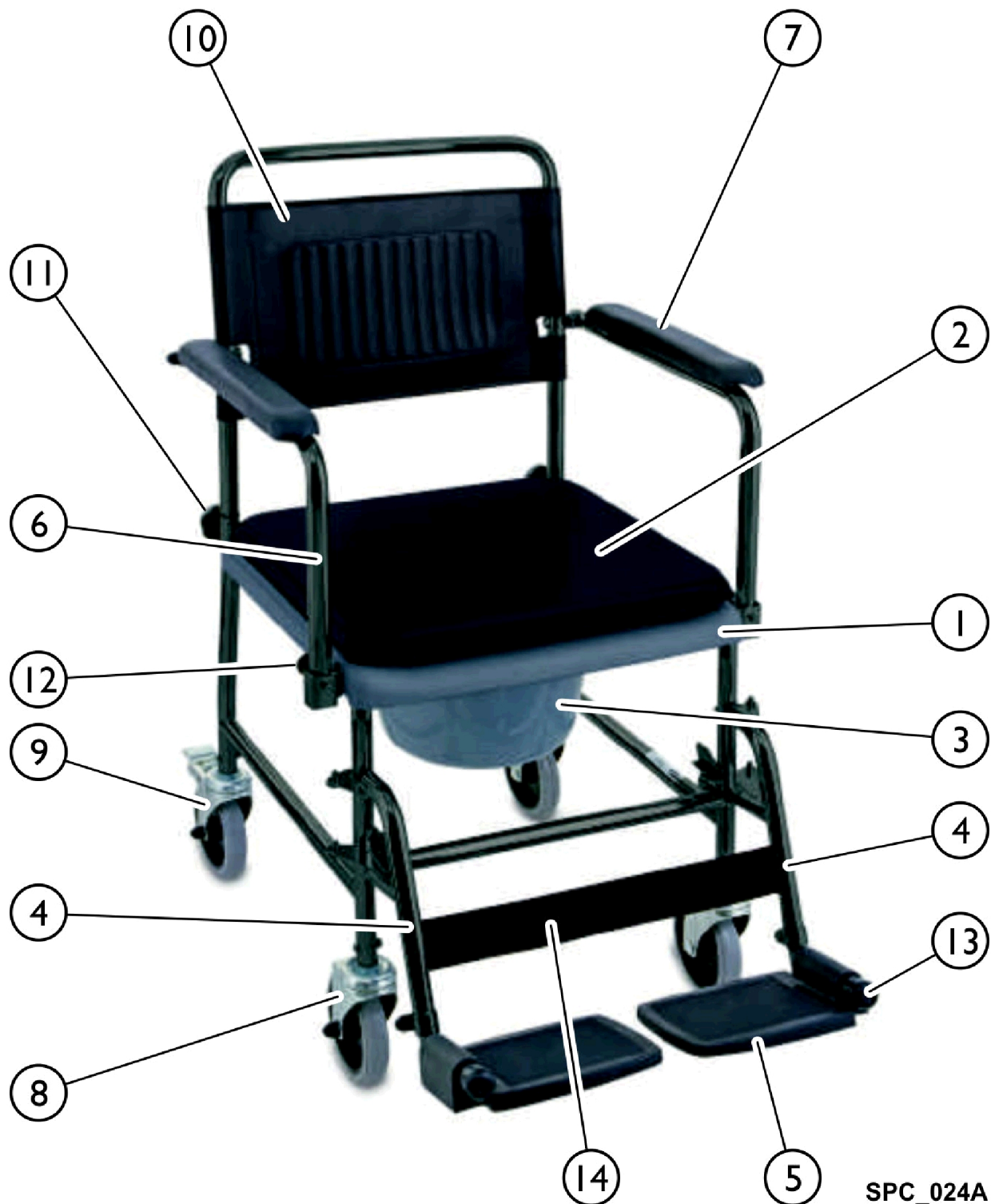


SPC_024A

H720T Glide About Commode

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1525864	Seat, Toilet w/ Holder, Gray	1	1
1		1525865	Seat, Toilet w/o Holder, Gray	1	1
2		1470947	Plate, Cover, Removable	1	1
2		1525866	Holder, Toilet Pan, Gray	1	1
3		1470948	Pan, Sanitary w/ Lid	1	1
4		1525868	Footrest, Right	1	1
4		1525867	Footrest, Left	1	1
5		1470952	Footplate, Left/Right	1	1
6		1525869	Armrest, Right, Gray	1	1
6		1525870	Armrest, Left, Gray	1	1
7		1470955	Pad, Armrest	1	2
8		1470956	Caster w/o Brake	1	2
9		1470957	Caster w/ Brake	1	2
10		1470958	Cover, Backrest	1	1
11		1470959	Knob, Backrest	1	2
12		1470960	Lock, Armrest	1	2
13		1470949	Tip, Rubber	1	2
14		1470962	Calf Strap	1	1


Omega Commode

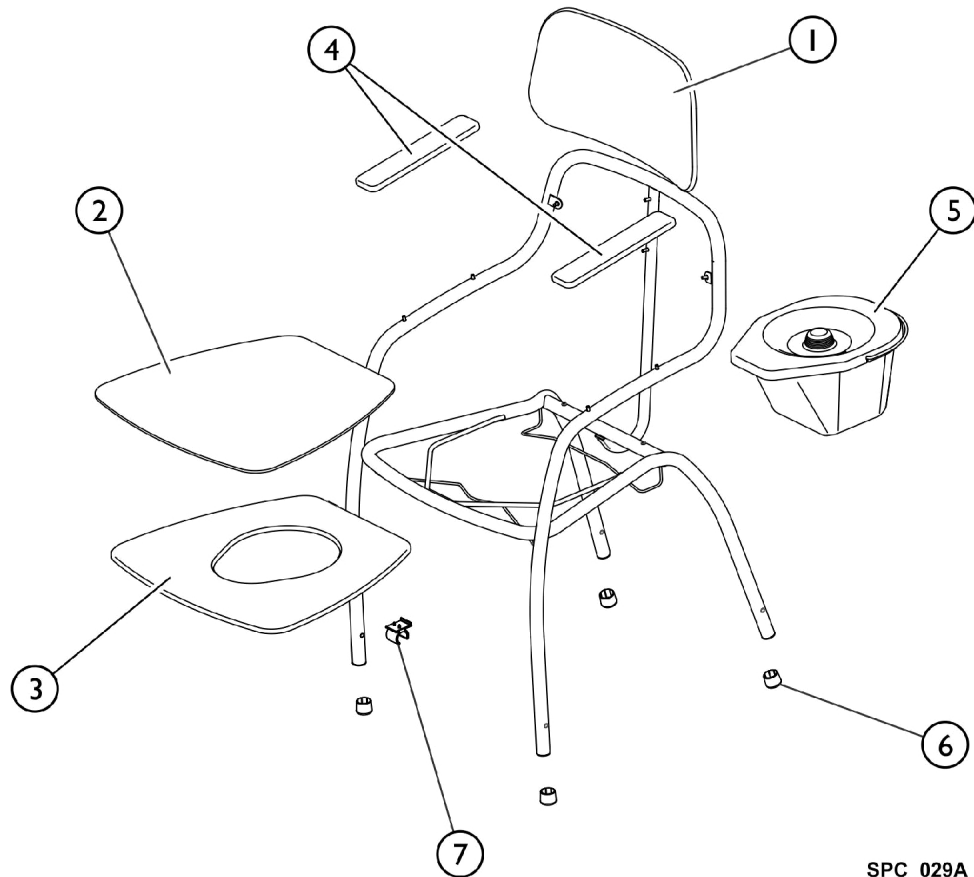

SPC_029A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1471004	Backrest, Dark Blue	1	1
2		1471005	Lid, Seat	1	1
3		1471007	Seat, Toilet	1	1
4		1471008	Pad, Armrest, Dark Blue (Pair)	2	1
5		1470991	Pan, Sanitary w/ Lid, Square	1	1
6		1470992	Tips, Rubber	1	4

Marina Bath Board

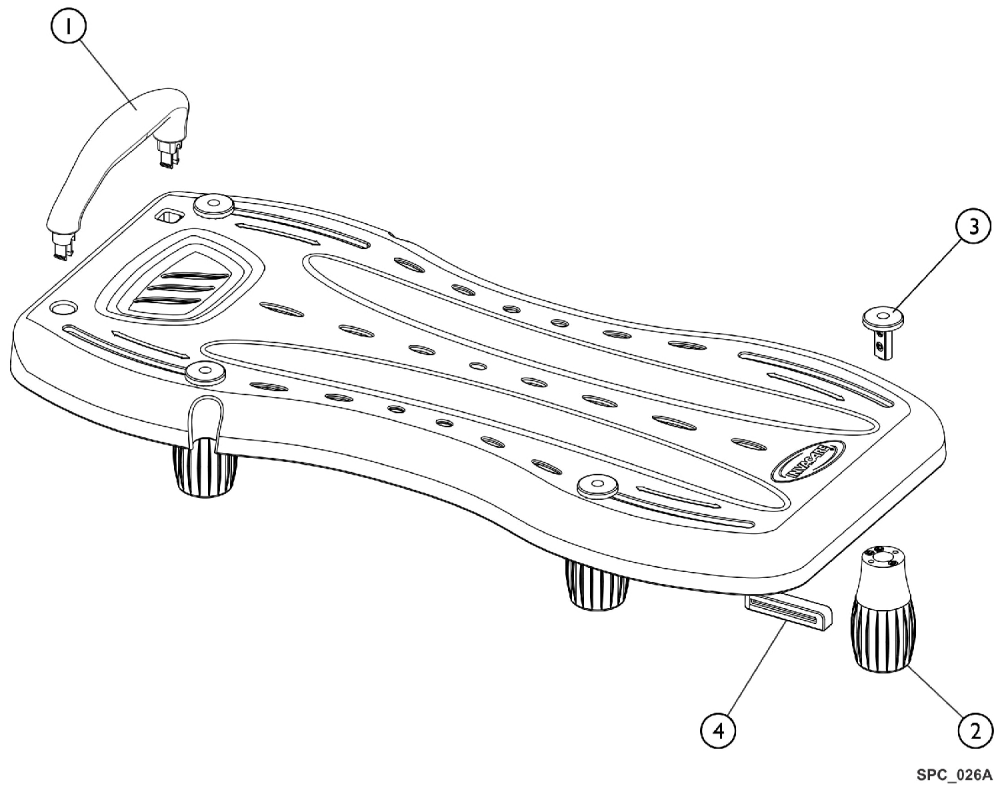

SPC_026A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1525942	Handle	1	1
2		1541331	Foot, Rubber	1	4
3		1525945	Screw, Rubber Foot	1	4
4		A11197	Protector, Slip	1	4

Invacare Corporation

www.invacare.ca



Yes, you can.

570 Matheson Blvd. #8
Mississauga, ON
L4Z 4G4
800-668-5324

Technical Support
800-668-5324
Customer Service
800-668-5324

For Faxed Orders
800-668-5478

© 2016 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
(Revised 04/19/16)

Form No. 10-783C