
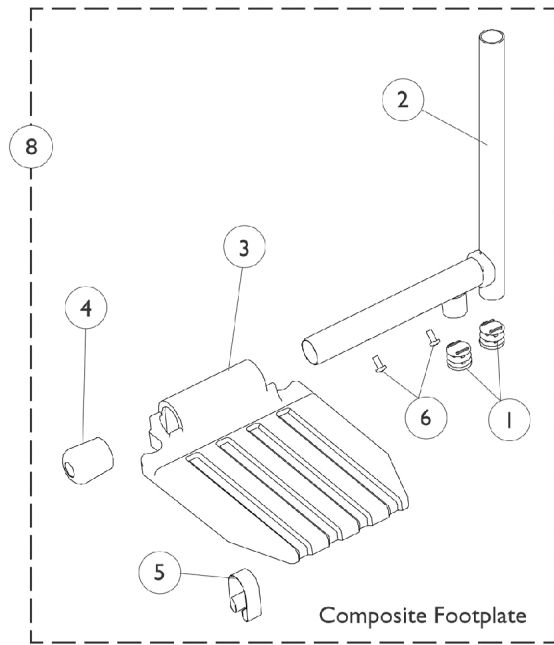


Pivot Slide Tube and Footplates

T94AA/C and PAL4A

Heel loops are not included. See accessories page.

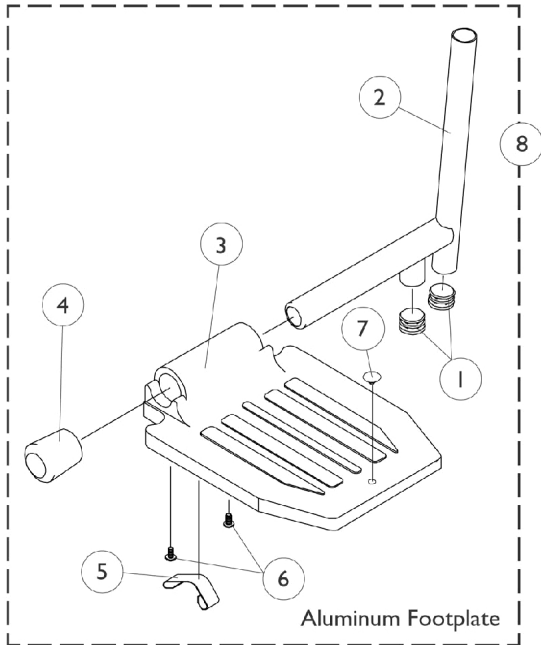

Pivot Slide Tube and Footplates

T94AA/C and PAL4A

Heel loops are not included. See accessories page.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2	A	7500085-NLA	NLA - Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	D	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
3		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	C	1015178	Footplate, Composite, Black (6" W x 7-53/64" L)	1	1
4		1016346	Cap, End, Tube, Black (7/8")	1	1
5		1006981	Spring, Tension	1	1
6		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
7		1070168	Plug, Push, Footplate	1	1
8	B	1017237-NLA	NLA - Pivot/Slide Tube w/ Aluminium Footplate Assembly, Chrome - Right	1	1
8	B	1017238-NLA	NLA - Pivot/Slide Tube w/ Aluminium Footplate Assembly, Chrome - Left	1	1
8	C	1017233-NLA	NLA - Pivot/Slide Tube w/ Composite Footplate Assembly, Chrome - Right	1	1
8	C	1017234-NLA	NLA - Pivot/Slide Tube w/ Composite Footplate Assembly, Chrome - Left	1	1
8	D	1023907	Footplate w/Pivot Slide Tube - RH (18")	1	1
8	D	1023908	Footplate w/Pivot Slide Tube - LH (18")	1	1
NOTE: A - For T934A/C B - For T934A C - For T934C D - For PAL4A					