
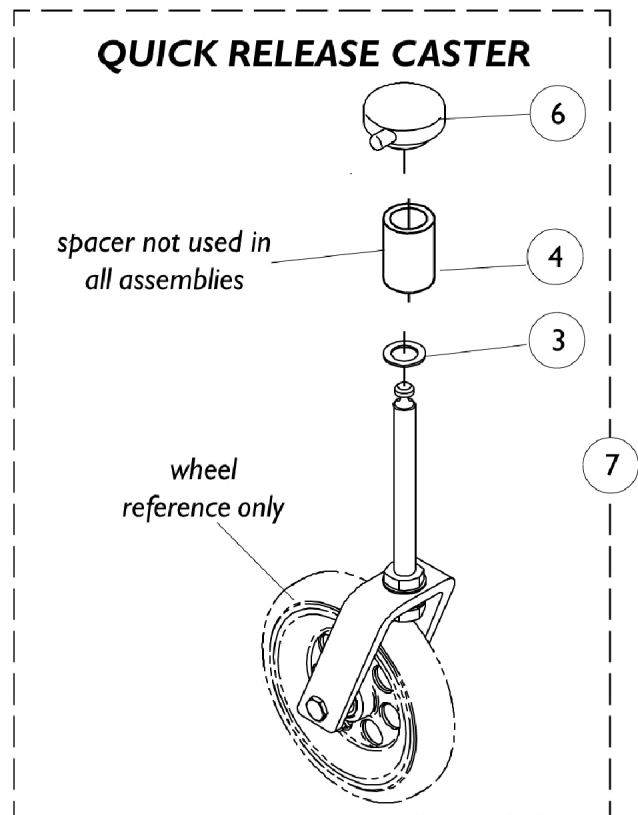


Caster Forks

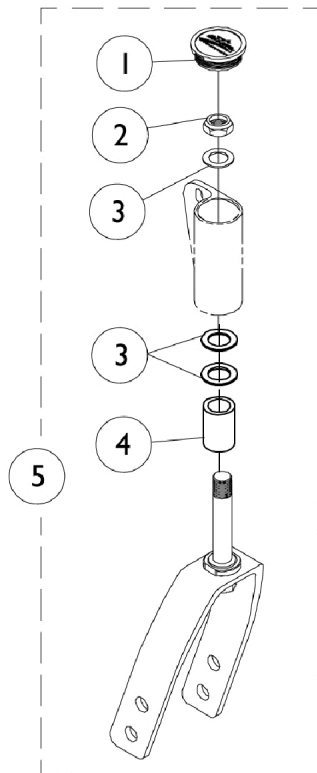

Caster Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1038353	Plug Button, Headtube, Black	1	1
2		1025737	Locknut (1/2-20)	1	1
3		1026174	Washer (1/2 x 7/8 x 3/64")	1	3
4		1046495	Spacer (1/2 x 3/4 x 1")	1	1
4		1026852	Spacer (1/2 x 3/4 x 2")	1	1
5		1065623	Kit, Fork w/ Hardware (3" Caster)	1	1
5		1065624	Kit, Fork w/ Hardware, 1" Extension (3 and 5" Caster)	1	1
5		1065625	Kit, Fork with Hardware and 2" Spacer (3" Caster)	1	1
5		1065626	Kit, Fork w/ Hardware (5" Caster)	1	1
5		1065627	Kit, Fork w/ Hardware, 1" Extension (5 and 6" Caster)	1	1
5		1065628	Kit, Fork w/ Hardware (6 and 6 x 1" Casters)	1	1
5		1065629	Kit, Fork w/ Hardware (6 x 2" and 8" Casters)	1	1
6		1042044-NLA	NLA - Cap, Head Tube, Quick Release Caster, Black	1	1
7		See Chart	#7 Quick Release Caster Fork Assembly	1	1

#7 Quick Release Caster Fork Assembly

(Includes Fork w/ Stem and items 3,4, and 6)

Fork Type	2-1/2" Caster Headtube	3" Caster Headtube
3" Quick Release Fork	1066391-NLA	1066384
3" Quick Release Fork w/ 1" Extension	1066392-NLA	1066385-NLA
3" Quick Release Fork w/ 2" Extension	1066393-NLA	1066386-NLA
5" Quick Release Fork	1066394-NLA	1066387-NLA
5" Quick Release Fork w/ 1" Extension	1066395-NLA	1066388-NLA
5" Quick Release Fork w/ 2" Extension	1066396-NLA	1066389-NLA
6" Quick Release Fork	1069162-NLA	1066390-NLA
6" Quick Release Fork 3-7/16" Offset	N/A	1066397-NLA