
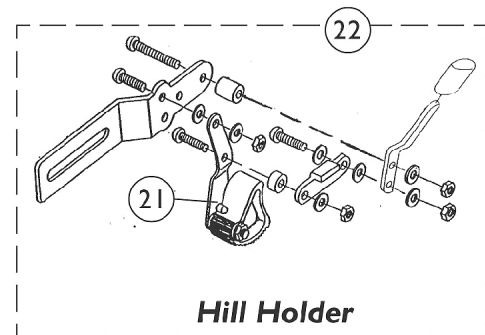
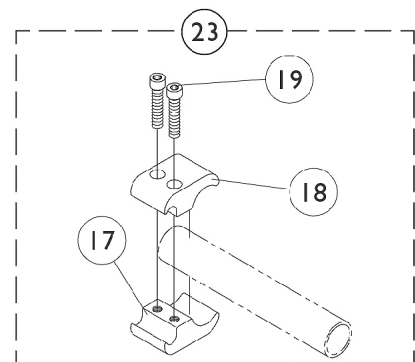


Wheel Lock & Hill Holder

Pull to Lock

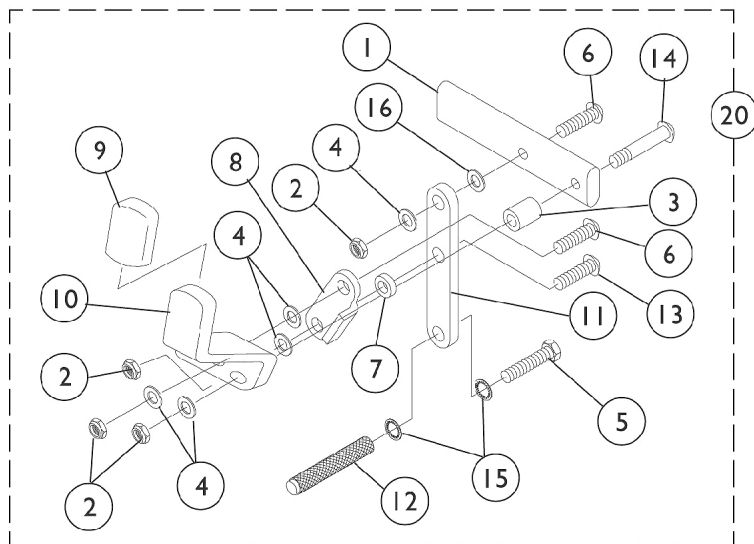

Wheel Lock & Hill Holder

Pull to Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028399-NLA	NLA - Strap, Wheel Lock	1	1
2		1025765	Locknut (1/4-28)	1	4
3		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	5
5	A	1055346-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	1
6		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
7		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
8		1018733-NLA	NLA - Link, Stop, Black	1	1
9		1006490	Tip, Plastic, Black	1	2
10		1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
10		1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
11		1029416-NLA	NLA - Decal, Black Lettering - Super Pro-T	1	1
12	A	1029420-NLA	NLA - Shoe, Wheel Lock	1	1
13		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
14		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
15		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	2
16		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
17		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
18		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
19		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
20	E	1038552-NLA	NLA - Wheel Lock, Clamp On, Pull to Lock - Right	1	1
20	E	1038553	Wheel Lock, Clamp On, Pull to Lock - Left	1	1
21		1026710	Hill Holder Assembly - Right	1	1
21		1026711-NLA	NLA - Hill Holder Assembly - Left	1	1
22	F	1026712	Hill Holder Wheel Lock Assembly - Right	1	1
			Bracket, Black	1	1
			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
			Locknut (1/4-28)	1	4
			NLA - Link, Stop, Black	1	1
			NLA - Handle, Wheel Lock, Black - Right	1	1
			Tip, Plastic, Black	1	1
			NLA - Spacer	1	1
			Hill Holder Assembly - Right	1	1
			NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
22	F	1026713	Hill Holder Wheel Lock Assembly - Left	1	1
			Bracket, Black	1	1
			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
			Locknut (1/4-28)	1	4
			Tip, Plastic, Black	1	1
			NLA - Handle, Wheel Lock, Black - Left	1	1

Wheel Lock & Hill Holder

Pull to Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
23 19	B,G	1068210-NLA	NLA - Link, Stop, Black	1	1
			NLA - Spacer	1	1
			NLA - Hill Holder Assembly - Left	1	1
			NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
			Kit, Wheel Lock Attaching Hardware (1" Tubing)	1	1
			Screw, Allen Head (1/4-20 x 1")	1	2
	C,D	1026352	Clamp, Half, Non-Threaded	1	1
			Clamp, Half, Threaded	1	1
			Kit, Wheel Lock Hardware	2	1
			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	2
			Washer (1/4 x 5/8 x 3/32")	1	2
			NLA - Clamp, Half, Threaded	1	2
			NLA - Clamp, Half, Non-Threaded	1	2
			Washer, Nylon (1/4 x 7/16 x 5/64")	1	2
			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
			NOTE: A - Order Part Number 1063507 (Wheel Lock Shoe Kit) for all Chairs Made Before 10/25/96. B - Wheel Lock hardware for item 20. Includes items 17-19 C - Not Shown D - Wheel Lock hardware for Hill Holder Wheel Locks (item 22) E - Includes items 1 - 16. F - Includes item 20 and 21. G - Includes item 17 - 19.		