


Frames and Hardware

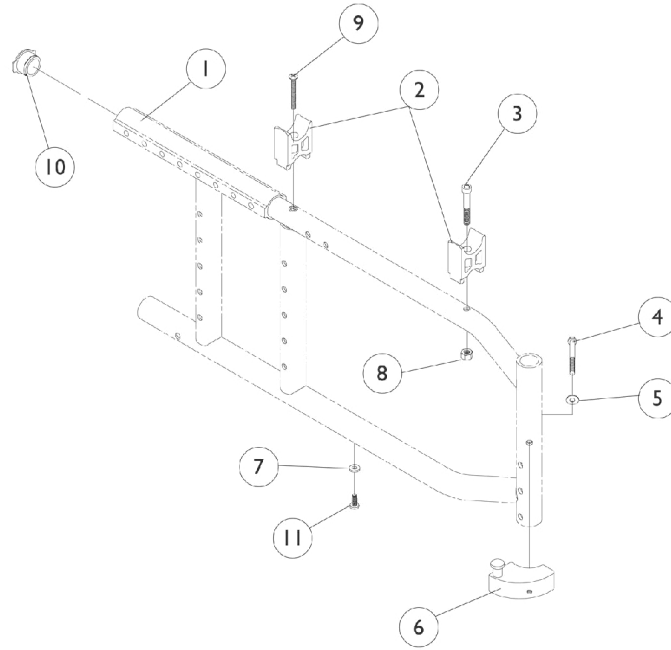

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881059138-NL A	NLA - Frame, Side, Color - Right (14" D)	1	1
1	A	8881059139-NL A	NLA - Frame, Side, Color - Left (14" D)	1	1
1	A	8881059140	Frame, Side, Color - Right (16" D)	1	1
1	A	8881059141	Frame, Side, Color - Left (16" D)	1	1
1	A	8881059142	Frame, Side, Color - Right (18" D)	1	1
1	A	8881059143	Frame, Side, Color - Left (18" D)	1	1
2		1037351-NLA	NLA - Guide, Seat	1	2
3		1059251-NLA	NLA - Screw, Socket Head (1/4-28 x 1-1/2")	1	2
4		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	1
5		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
6	B	1054571	Kit, Sector Block - Right	1	1
6	B	1054572	Kit, Sector Block - Left	1	1
7		1059111-NLA	NLA - Washer, Nylon (#10)	1	1
8		1065362	Locknut (1/4-28)	1	1
9		1025703-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1")	1	1
10		1061056-NLA	NLA - Plug Button, Black (1")	1	1
11		1028484	Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	1

NOTE: A - Specify frame color when ordering.
B - Includes Items 4 and 5.