



Cross braces and Hardware

Folding chairs

Modular Crossbraces

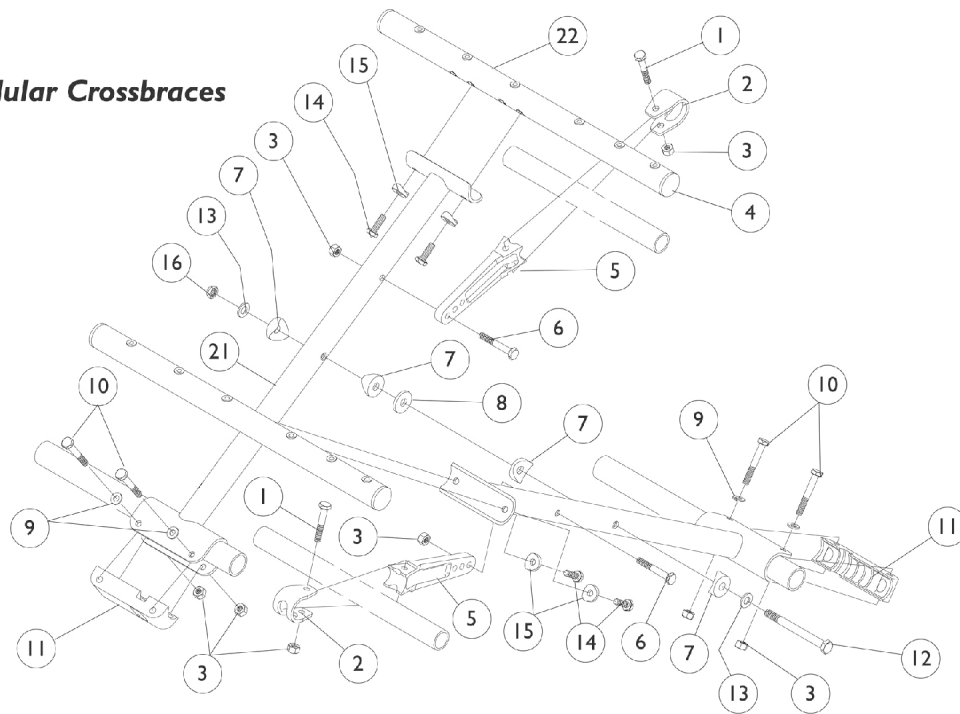

Crossbraces less Seat Rail

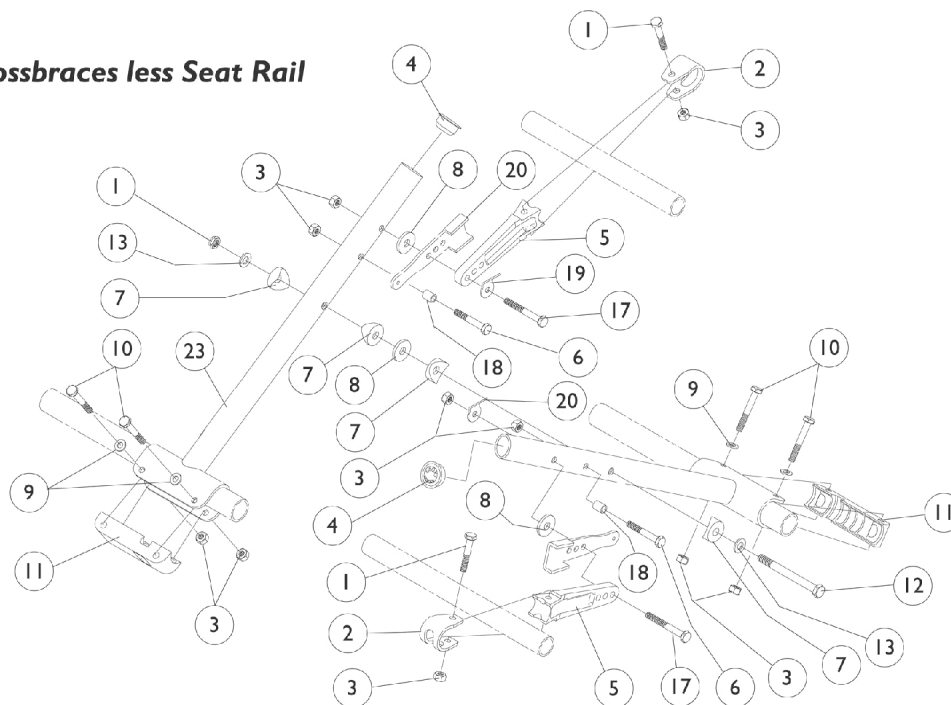

Cross braces and Hardware

Folding chairs

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
2		1012180	Bracket, Pivot, Black	1	2
3		1065362	Locknut (1/4-28)	1	8
4		1056328	Plug Button, Black (1")	1	4
5	B	1059105-NLA	NLA - Pivot Link	1	2
5	C,G	1014666-NLA	NLA - Link, Pivot, Black (14 - 16" W)	1	2
		7200188-NLA	NLA - Pivot Link w/ Hardware (14 - 16" W)	1	2
5	D	1014668	Link, Pivot, Black (17 - 18" W)	1	2
5	E	1014667	Link, Pivot, Black (20 - 22" W)	1	2
6		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	2
7		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
8		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	3
9		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
10		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	4
11		1012172-NLA	NLA - Clamp, Saddle	1	2
12		1025705-NLA	NLA - Screw, Hex Head (5/16-24 x 3-1/4")	1	1
13		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
14		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	2
15		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
16		1025730	Locknut (5/16-24)	1	1
17		1055349	Screw, Hex Head (1/4-28 x 2")	1	2
18		1026162	Spacer (1/4 x 3/8 x 11/32")	1	2
19		1026903-NLA	NLA - Tab, Tie Down	1	2
20	B	1063668-NLA	NLA - Stop, Pivot Link, Black	1	2
20	C	1063666-NLA	NLA - Stop, Pivot Link, Black	1	2
20	F	1063667-NLA	NLA - Stop, Pivot Link, Black	1	2
	A	1029412	Lock, Cord, Black	1	1
	A	1141633	Cord, Nylon (6-1/2 ft.)	1	1
21		See Chart	#21 Modular Crossbraces Less Seat Rail	1	1
22		See Chart	#22 Modular Seat Rails	1	1
23		See Chart	#23 Crossbraces Less Seat Rail	1	1

NOTE: A - Not Shown
 B - 12-13" wide chairs
 C - 14-16" wide chairs
 D - 17-19" wide chairs
 E - 20" wide chairs
 F - 17-20" wide chairs
 G - Pivot Link is no longer available as a service part individually. Order part #7200188 Pivot Link w/ Hardware (not shown) as a replacement. Includes items 2-4, 7 and 8.

#21 Modular Crossbraces Less Seat Rail

To change width on a chair with modular Crossbraces, order items 5 and 21.

Chair Width	12"	13"	14"	15"	16"	17"	18"
Right/Left	1061039	1061040-NLA	1061041	1061042	1061043	1061044	1061045
Chair Width	19"	20"					
Right/Left	1061037-NLA	1061038					

Cross braces and Hardware

Folding chairs

#22 Modular Seat Rails

Chair Depth	12"	13"	14"	15"	16"	17"	18"
Right	1061009-NLA	1061011-NLA	1061013-NLA	1061015-NLA	1061017-NLA	1061019-NLA	1061021
Left	1061010-NLA	1061012-NLA	1061014-NLA	1061016-NLA	1061018-NLA	1061020-NLA	1061022

#23 Crossbraces Less Seat Rail

Chair Width	12"	13"	14"	15"	16"	17"	18"
Right/Left	1063200-NLA	1063201-NLA	1063202-NLA	1063203-NLA	1063204	1063205	1063206