

Invacare® Comet/Comet HD **Canadian Power Scooter**



Yes, you can.

Parts Catalogue Usage Guide

The information contained in this document is subject to change without notice. This catalogue contains service parts for Form No. 11-010C. If you have any questions regarding this catalogue, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-668-5324

For Faxed In Orders Call:

1-800-668-5478

For Technical Service Call:

1-800-668-5324

Literature Can Be Found at:

www.invacare.ca or

1-800-668-5324

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2018 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

TABLE of CONTENTS


Description	Page Number
Brakes	16
Cable and Charger	19
Comet Scooter	3
Front Frame - Four Wheeler	5
Front Shroud	4
Front Wheels - Four Wheeler	11
Rear Frame	15
Rear Shroud	7
Rear Wheels and Propulsion	13
Seat and Seat Options	9
Steering	17

Comet Scooter




Invacare® **COMET**

Front Shroud

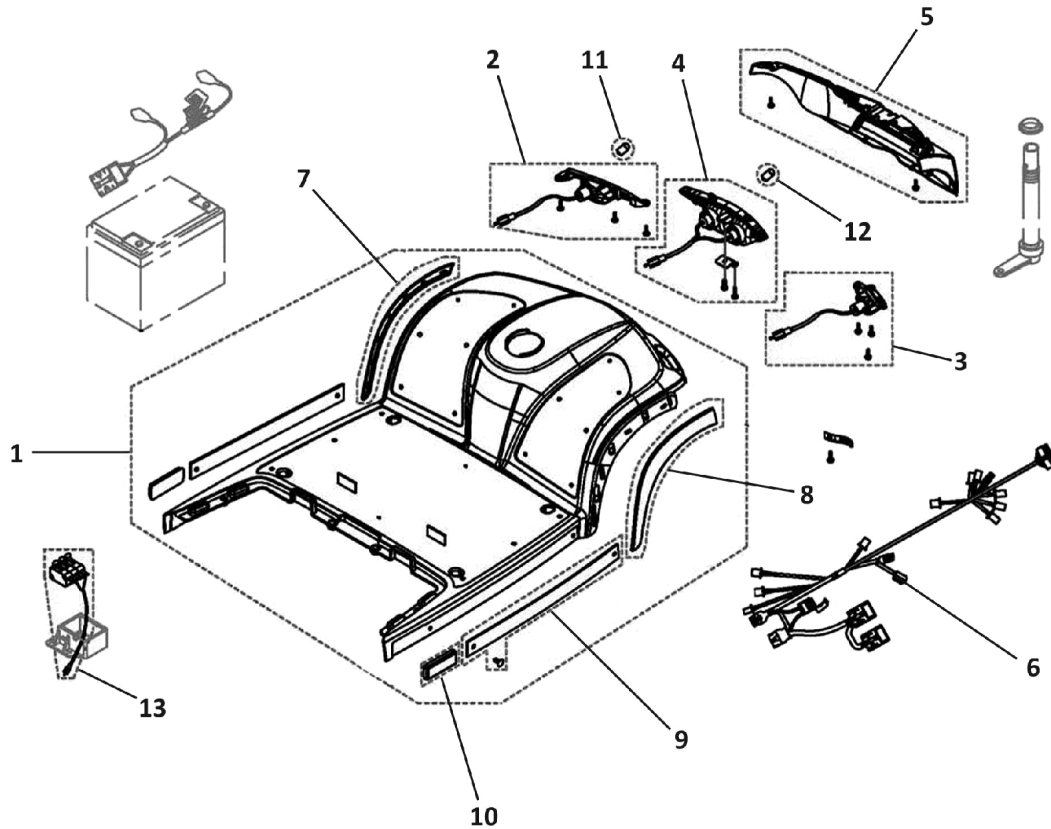

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521732	Shroud, Front, Silver	--	1
1		1521733	Shroud, Front, Blue	--	1
1		1530001	Shroud, Front, Red	--	1
1		1537792	Shroud, Front, Comet HD	--	1
2		1517733	Turn Signal - Left	--	1
3		1517734	Turn Signal - Right	--	1
4		1527473	Head Light, Complete	--	1
5		1521762	Cover Bumper, Front	--	1
6		1532911	Wire Harness	--	1
7		1521735	Moulding - Left	--	1
8		1521734	Moulding - Right	--	1
9		1521754	Moulding, Side	--	2
10		1517331	Reflector, Front Shroud	--	2
11		1522030	Bulb, Turn Signal (24V/10 W)	--	1
12		1527474	Bulb, Head Light (12V/15W)	--	1
13		1554030	Kit, Speed Limit Switch Box Upgrade (12km/h)	--	1


Front Frame - Four Wheeler

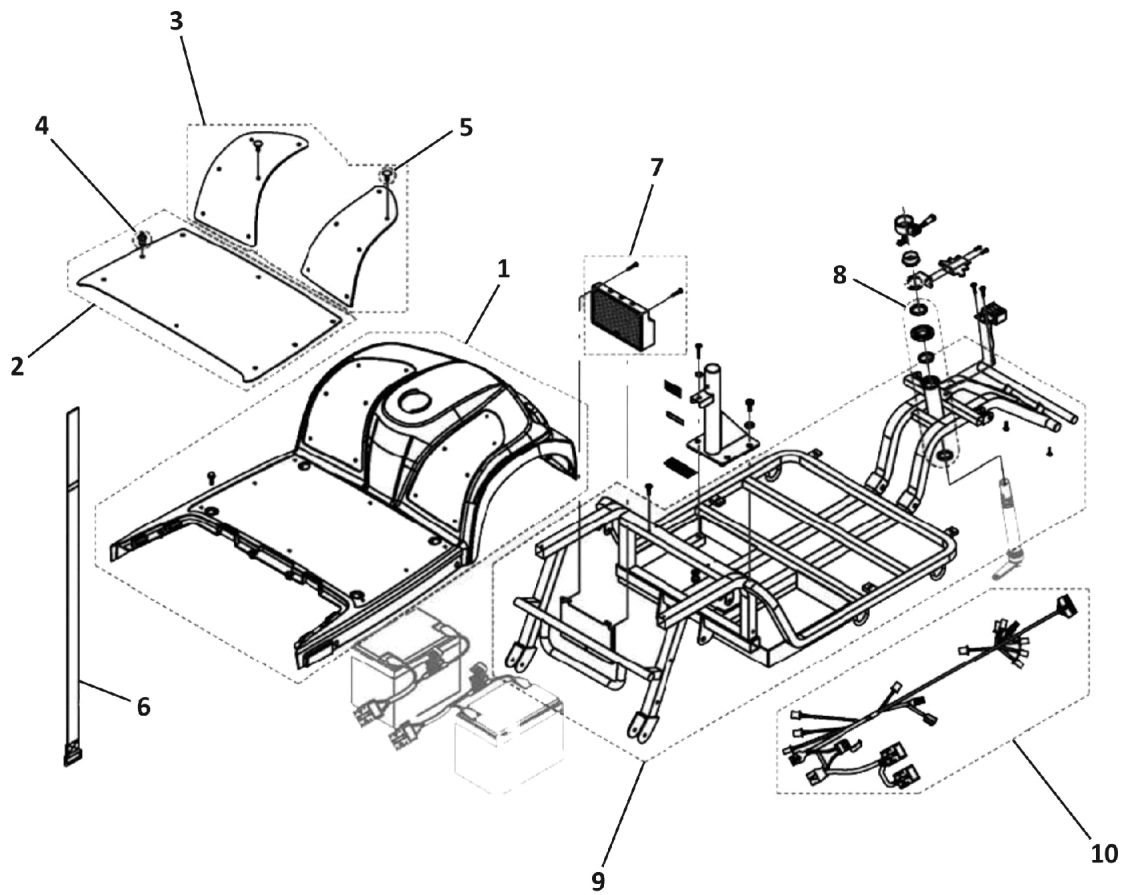

Front Frame - Four Wheeler

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521732	Shroud, Front, Silver	--	1
1		1521733	Shroud, Front, Blue	--	1
1		1537792	Shroud, Front, Comet HD	--	1
2	A	1521755	Pad, Leg Shield	--	1
3	B	1521756	Pad, Front - Right/Left	--	1
4		1505476	Push Rivet, Pad	--	8
5		1505479	Tree Rivet	--	12
6		1521758	Band, Battery, Complete	--	1
7	C	1517317	Controller, Rhino (110A)	--	1
7	D	1553399	Controller, Rhino (120A)	--	1
7	C	1521759	Controller, Rhino (160A)	--	1
7	D	1553400	Controller, Rhino (160A HD)	--	1
8		1505487	Kit, Bearing, Scooter Frame	--	1
9		1521760	Front	--	1
10		1532911	Wire Harness	--	1
NOTE: A - Includes item 4. B - Includes item 5. C - For scooters manufactured prior to June 2011. D - For scooters manufactured after June 2011.					


Rear Shroud

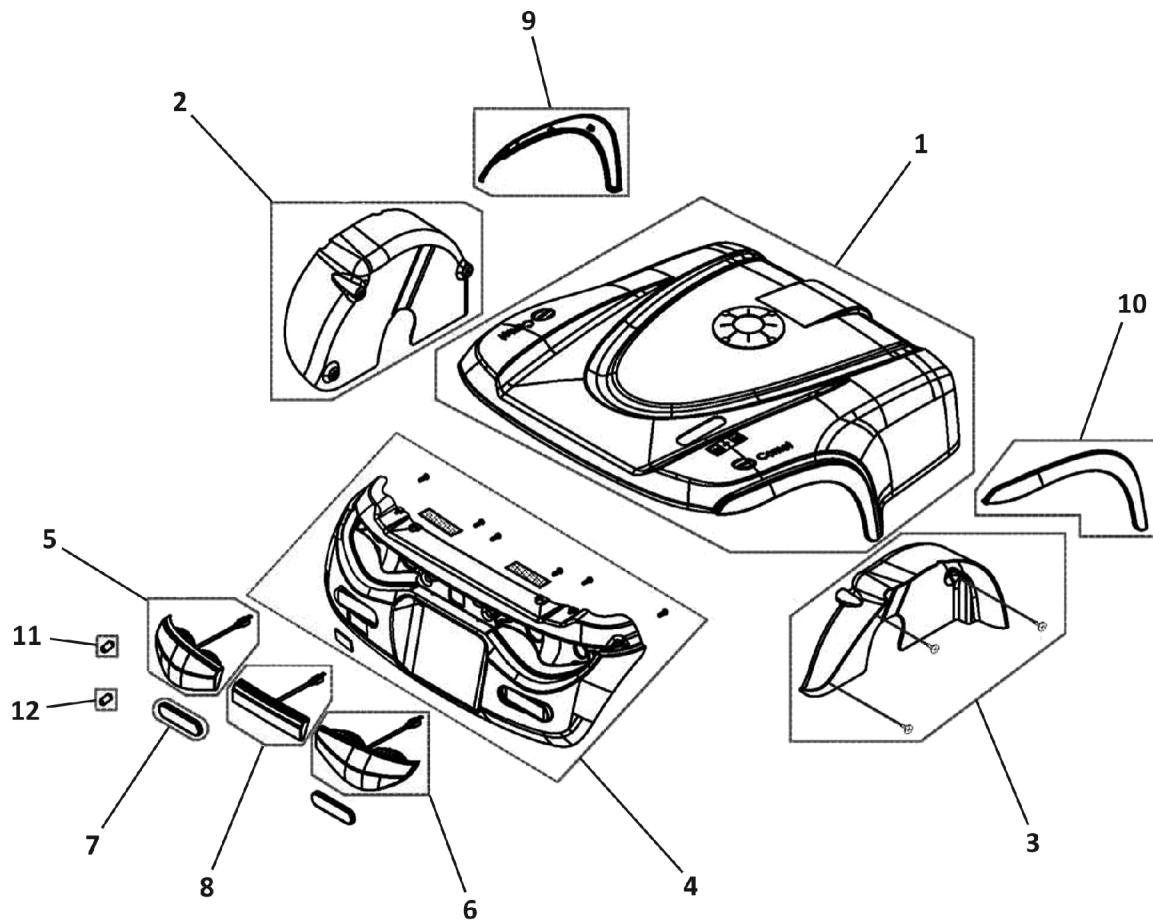

Rear Shroud

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521837	Rear Shroud, Silver	--	1
1		1521838	Rear Shroud, Blue	--	1
1		1530002	Shroud, Rear, Complete, Red	--	1
1		1537793	Shroud, Rear, Comet HD	--	1
2		1521862	Fender - Left	--	1
3		1521839	Fender - Right	--	1
4		1521683	Cover, Bumper, Rear	--	1
5		1518034	Turn Signal, Rear - Left	--	1
6		1518033	Turn Signal, Rear - Right	--	1
7		1521881	Reflector, Rear, E Mark	--	2
8		1518035	Neon Light, Complete	--	1
9		1521882	Cover, Rear - Left	--	1
10		1521883	Cover, Rear - Right	--	1
11		1522030	Bulb, Turn Signal (24V/10 W)	--	1
12		1522034	Bulb, Rear Light (24V/5W)	--	1


Seat and Seat Options

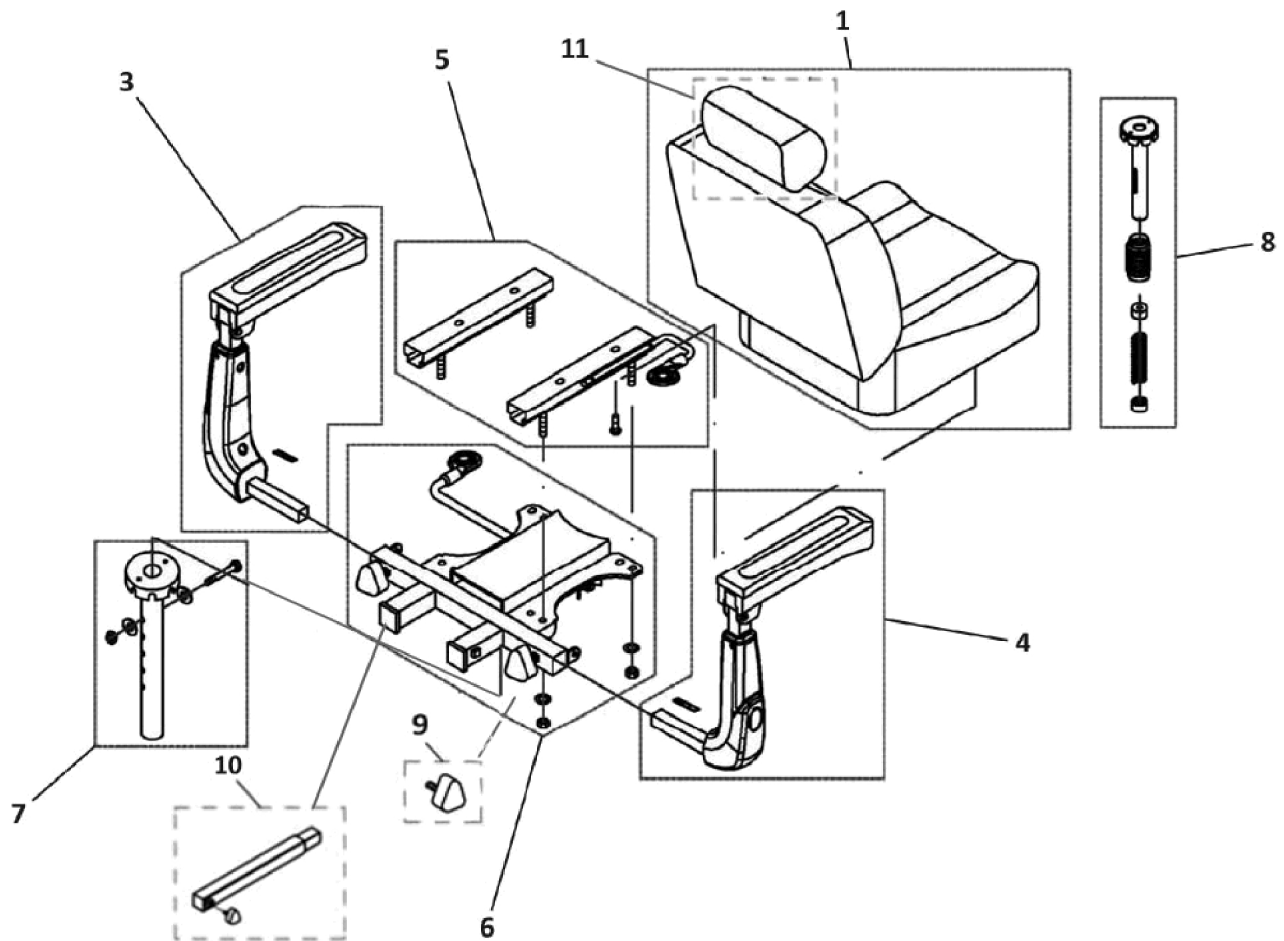

Seat and Seat Options

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521890	Seat, Standard	--	1
1		1586116	Seat, Standard w/ Double White Stitching	--	1
1		1533705	Seat, Comet HD (26")	--	1
1		1586115	Seat, Comet HD (26"), Double White Stitching	--	1
3		1521689	Armrest - Left	--	1
3		1533706	Armrest - Left, Comet HD	--	1
4		1521688	Armrest - Right	--	1
4		1533707	Armrest - Right, Comet HD	--	1
5		1521690	Slide Assembly	--	1
6		1505533	Seat Bracket Assembly	--	1
6		1533708	Seat Bracket Assembly, Comet HD	--	1
7		1521691	Seat Bracket, Standard	--	1
8		1541629	Seat Suspension (560 mm)	--	1
8		1541628	Seat Suspension (535 mm)	--	1
9		1507005	Knob, Tightening	--	1
10	A	1541502	Accessory Extension, Comet HD	--	1
11		1529380	Headrest (Only)	--	1
	B	1518240	Seat Cover, Comet - Black Only	--	1
NOTE: A - Additional Accessory, not included with Scooter, necessary to attach ACC140, ACC200, ACC210, ACC220, ACC230. B - Not Shown in illustration.					


Front Wheels - Four Wheeler

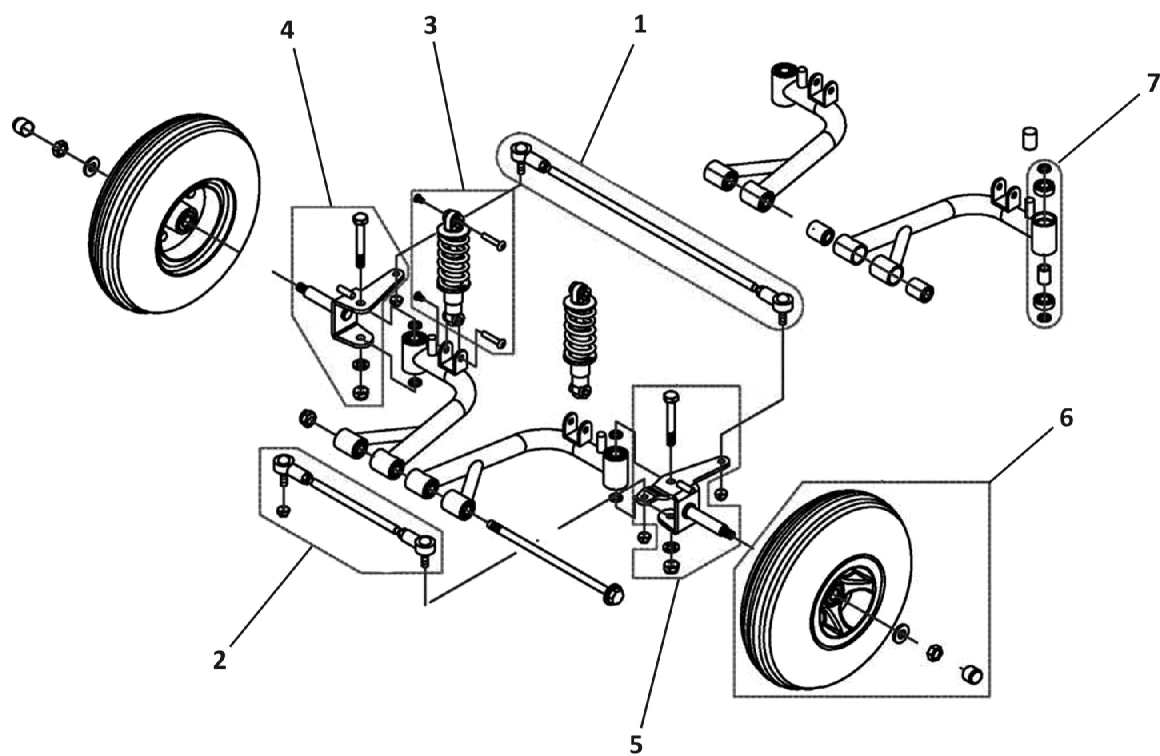

Front Wheels - Four Wheeler

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1518077	Rod Assembly, Front	--	1
2		1521994	Track Rod, Short	--	1
3		1521995	Shock Absorber, Front	--	2
4		1521996	Axle, Front - Left	--	1
5		1521997	Axle, Front - Right	--	1
6		1521999	Wheel, Front w/ Tube (13"), Complete Assembly - Grey	--	2
		1586120	Wheel, Front w/ Tube (13"), Complete Assembly - Black	--	2
		1527714	Tube (13")	--	4
		1527750	Tire, Front w/ Tube (13") - Grey	--	2
		1586118	Tire, Front w/ Tube (13") - Black	--	2
		1543159	Tire, Solid PU (13") - Grey	--	4
		1586119	Tire, Solid PU (13") - Black	--	4
7		1522000	Kit, Steering Bearing	--	2
		1527751	Kit, Tire Bearing	--	4
		1580174	Kit, Rim Front	--	2
		1509666	Kit, Mounting, Front	--	2


Rear Wheels and Propulsion

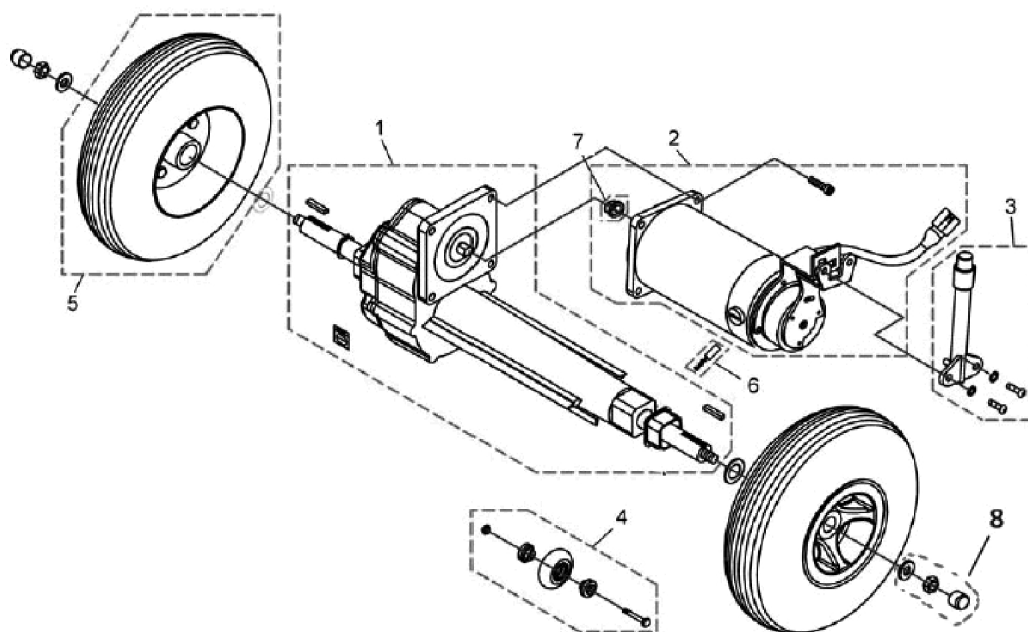

Rear Wheels and Propulsion


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1522013	Transaxle (15 km/h)	--	1
1		1533710	Transaxle (12 km/h), Comet HD	--	1
2		1553429	Motor (15 km/h)	--	1
2		1533850	Motor, Comet HD	--	1
3		1505672	Lever, Motor	--	1
4		1522021	Wheel, Anti-Tipper	--	2
5		1522020	Wheel, Rear w/ Tube (13"), Complete Assembly - Grey	--	2
		1586121	Wheel, Rear w/ Tube (13"), Complete Assembly - Black	--	2
		1527714	Tube (13")	--	2
		1527750	Tire, Front w/ Tube (13") - Grey	--	2
		1586118	Tire, Front w/ Tube (13") - Black	--	2
		1543159	Tire, Solid PU (13") - Grey	--	2
		1586119	Tire, Solid PU (13") - Black	--	2
		1533853	Brushes, Comet HD	--	4
		1527845	Brushes, Comet	--	4
		1533842	Guide Motor, Comet HD	--	1
8		1557459	Kit, Mounting Rear	--	4
		1505670	Washer, Plane	--	2
		1505669	Key (5 x 30)	--	2
		1527749	Kit, Rim Rear	--	2

Rear Frame

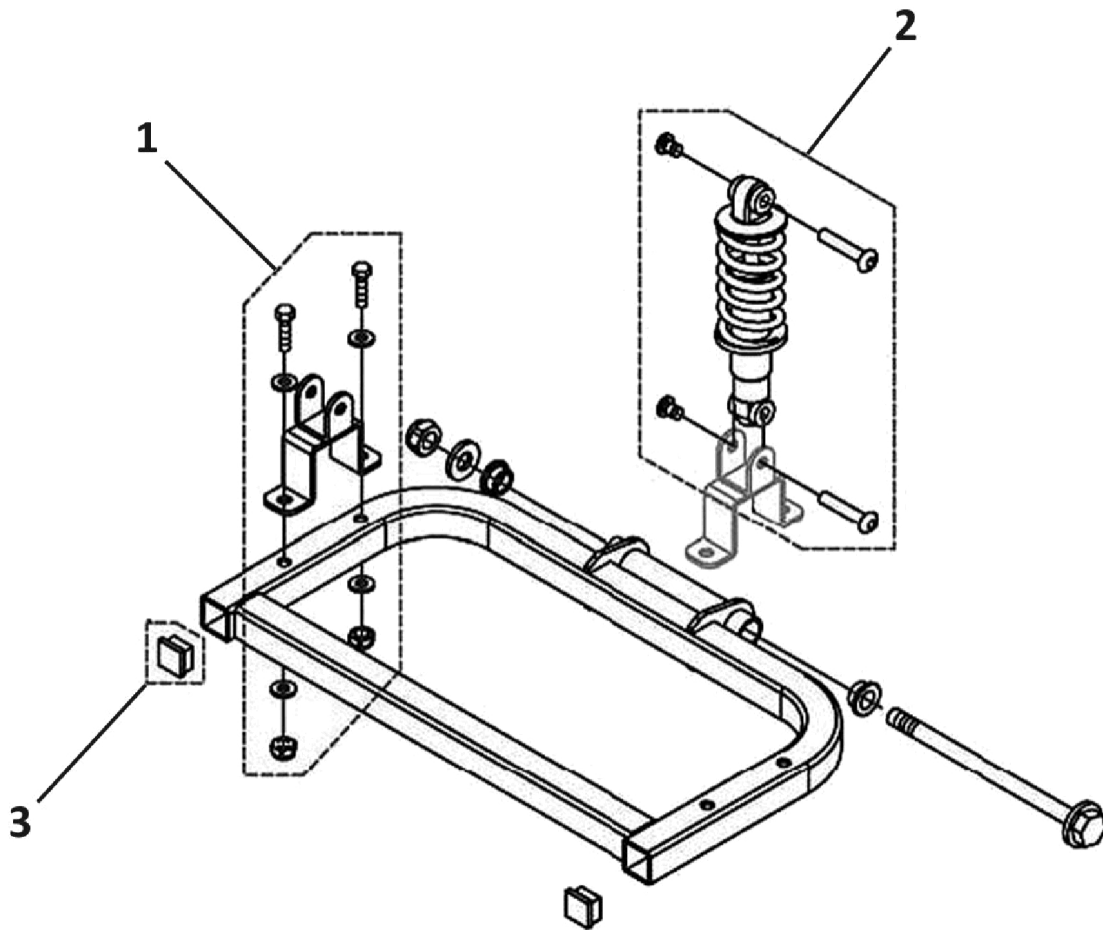


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521888	Holder Differential	--	2
1		1533846	Holder Differential, Comet HD	--	1
2		1521886	Shock Absorber (350 lbs.)	--	1
2		1521887	Shock Absorber (Optional)	--	1
2		1533847	Shock Absorber, Comet HD	--	1
3		1521597	Plug, Cane Bracket	--	2
5		1541616	Rear Frame, Comet HD	--	1

Brakes

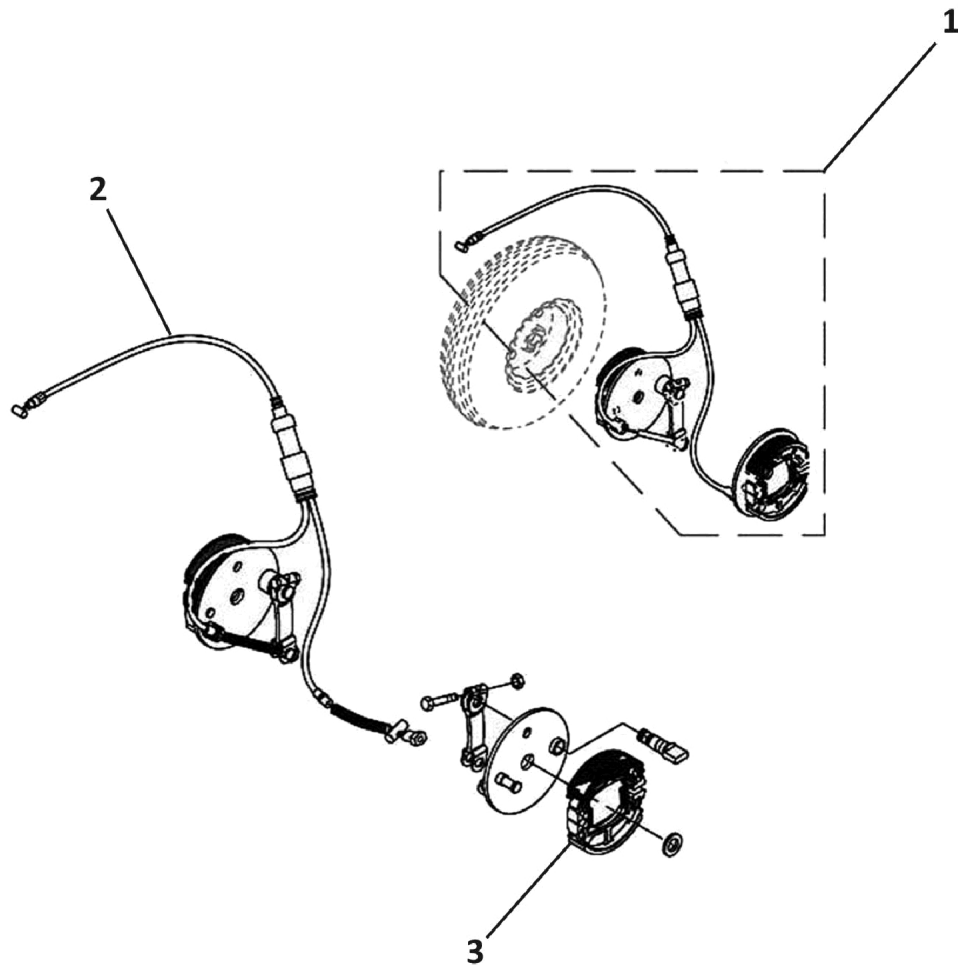

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521589	Brake Assembly, Four Wheel	--	1
2		1518422	Cable, Brake, Four Wheel	--	1
3		1521582	Shoe, Brake, Complete	--	1

Steering

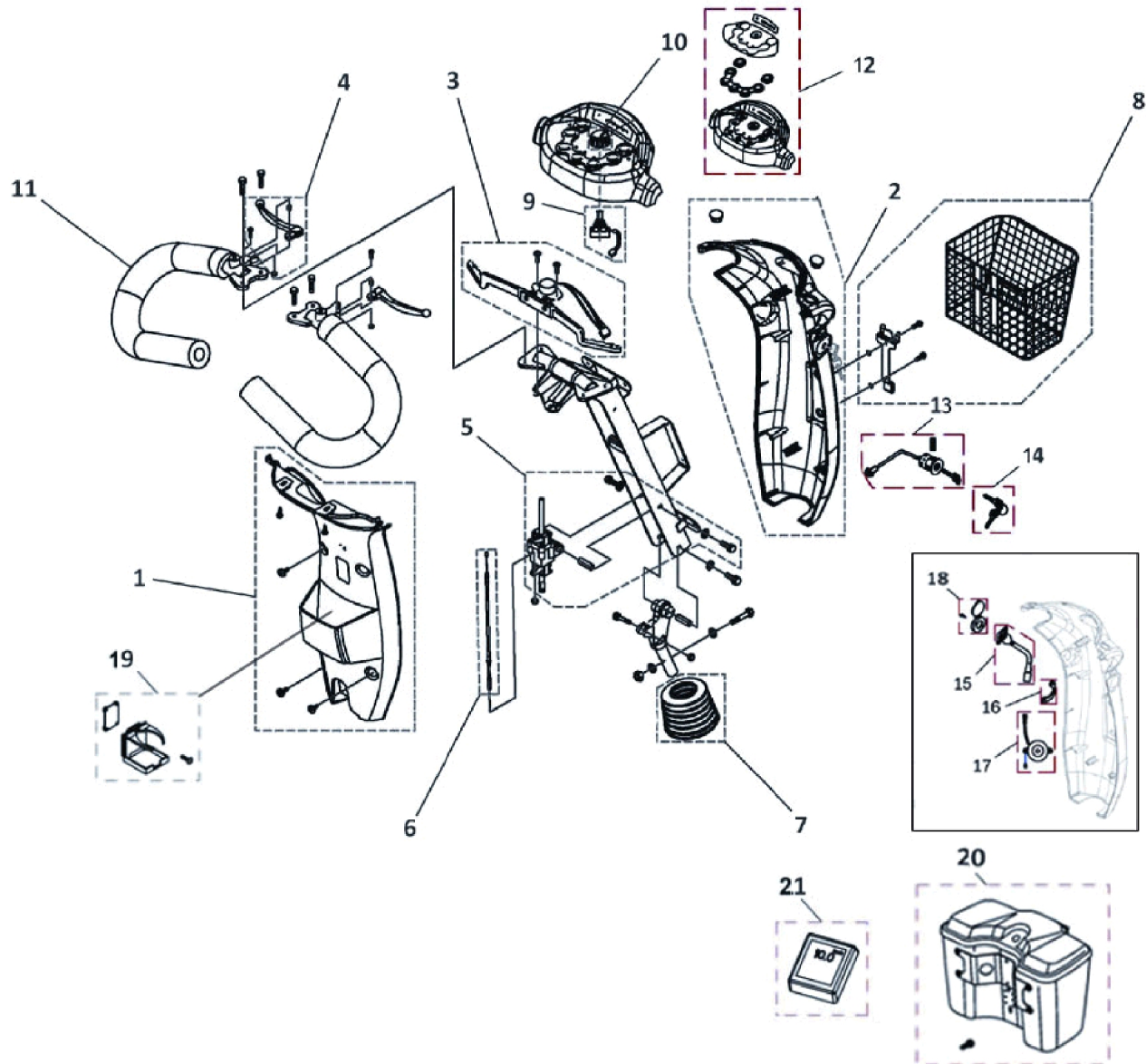


Steering

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1521725	Cover, Rear Steering	--	1
1		1537796	Cover, Front Steering, Slate Grey, Comet HD	--	2
1		1530006	Cover, Steering, Front, Complete, Red	--	1
2		1521726	Cover, Front Steering, Silver	--	1
2		1521727	Cover, Front Steering, Blue	--	1
3		1521575	Starter	--	1
4		1521578	Regulator Rod	--	2
5		1505279	Adjustable Steering Assembly	--	1
6		1521583	Cable, Starter to Adjustable Steering Assembly	--	1
7		1521586	Cover, Steering Joint	--	1
8		1521584	Basket, Front	--	1
9		1505460	Knob, Speed Control	--	1
10		1505462	Variable Resistor, Complete (30.0KO)	--	1
11		1527790	Foam Grip	--	2
12		1532908	Cover, Upper Dash	--	1
13		1505451	Ignition	--	1
14		1164848	Package, Ignition Key	10	1
15		1523543	Charger Port	--	1
16		1523544	Retaining Plate, Charger Port	--	1
17		1506153	Buzzer	--	1
18		1523534	Cover, Charger Port	--	1
19	B	1521594	Assembly, Drink Holder	--	1
20	B	1537710	Front Lock Box, Comet HD, Slate Grey	--	1
20	B	ACC270	Front Lock Box, Comet, Blue	--	1
20	B	ACC260	Front Lock Box, Comet, Silver	--	1
20	B	1551638	Front Lock Box, Comet, Red	--	1

NOTE: A - One each for left or right side.
B - Additional Accessory, not included with the Scooter.

Cable and Charger

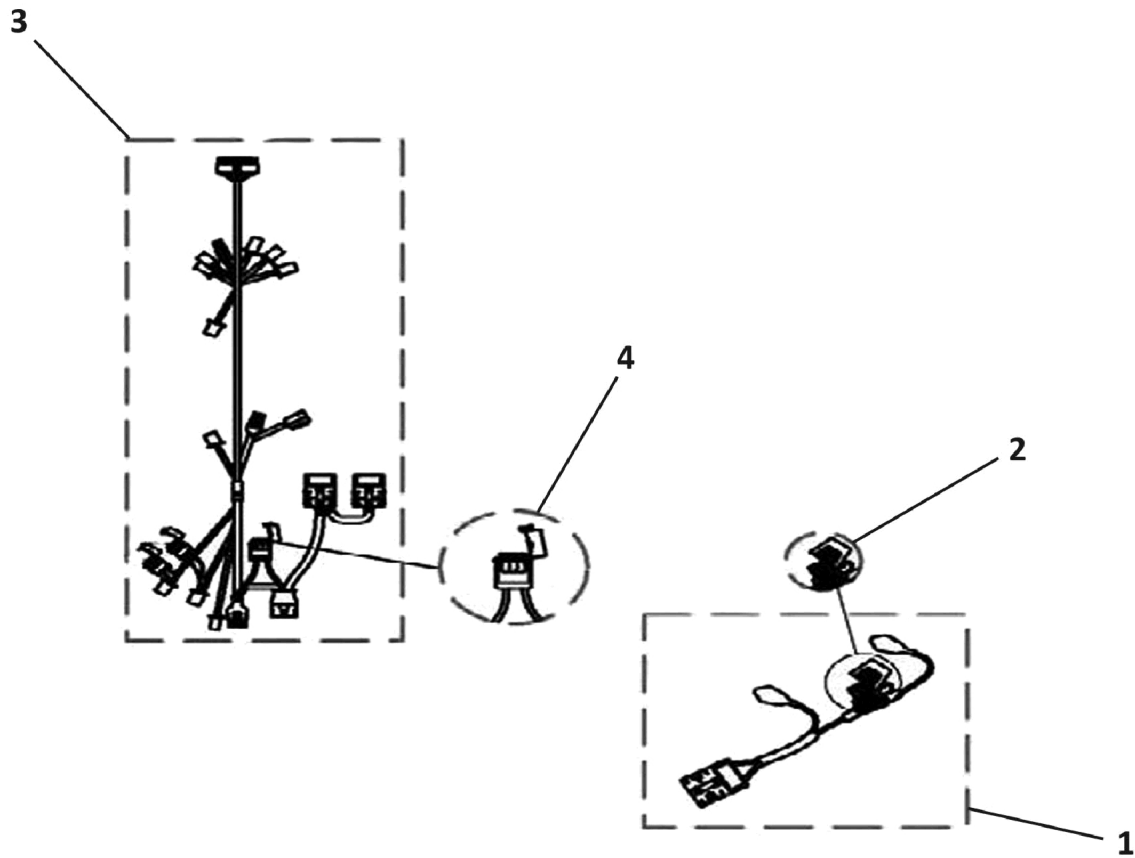

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521696	Wire, Battery Output	--	1
1		1537917	Wire, Battery Output, Comet HD	--	1
2		1505500	Fuse (70 Amp)	--	1
3		1532911	Wire Harness	--	1
4		1521577	Fuse (10 Amp)	--	1

Invacare Canada L.P.

www.invacare.ca



Yes, you can.

570 Matheson Blvd E;8
Mississauga, ON
L4Z 4G4
905-890-8300

Technical Support
800-668-5324
Customer Service
800-668-5324

Fax Orders
800-668-5478

© 2018 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
(Revised 3/14/18)

Form No. 11-010C