

Invacare® Pegasus **Canadian Power Scooter**



Yes, you can.

Parts Catalogue Usage Guide

The information contained in this document is subject to change without notice. This catalogue contains service parts for Form No. 11-011C. If you have any questions regarding this catalogue, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-668-5324

For Faxed In Orders Call:

1-800-668-5478

For Technical Service Call:

1-800-668-5324

Literature Can Be Found at:

www.invacare.ca or

1-800-668-5324

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2016 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

TABLE of CONTENTS


Description	Page Number
Brakes - Four Wheeler	16
Brakes - Three Wheeler	15
Electrical - Wires and Fuses	19
Frame - Three Wheeler	6
Front Frame - Four Wheeler	7
Front Shroud - Four Wheeler	5
Front Shroud - Three Wheeler	4
Front Wheel - Three Wheeler	12
Front Wheel and Steering - Four Wheeler	13
Pegasus Scooter	3
Rear Frame	10
Rear Shroud	9
Rear Wheels and Propulsion (12km/h)	14
Seat and Seat Options	11
Steering	17

Pegasus Scooter




Invacare® **PEGASUS**

Front Shroud - Three Wheeler

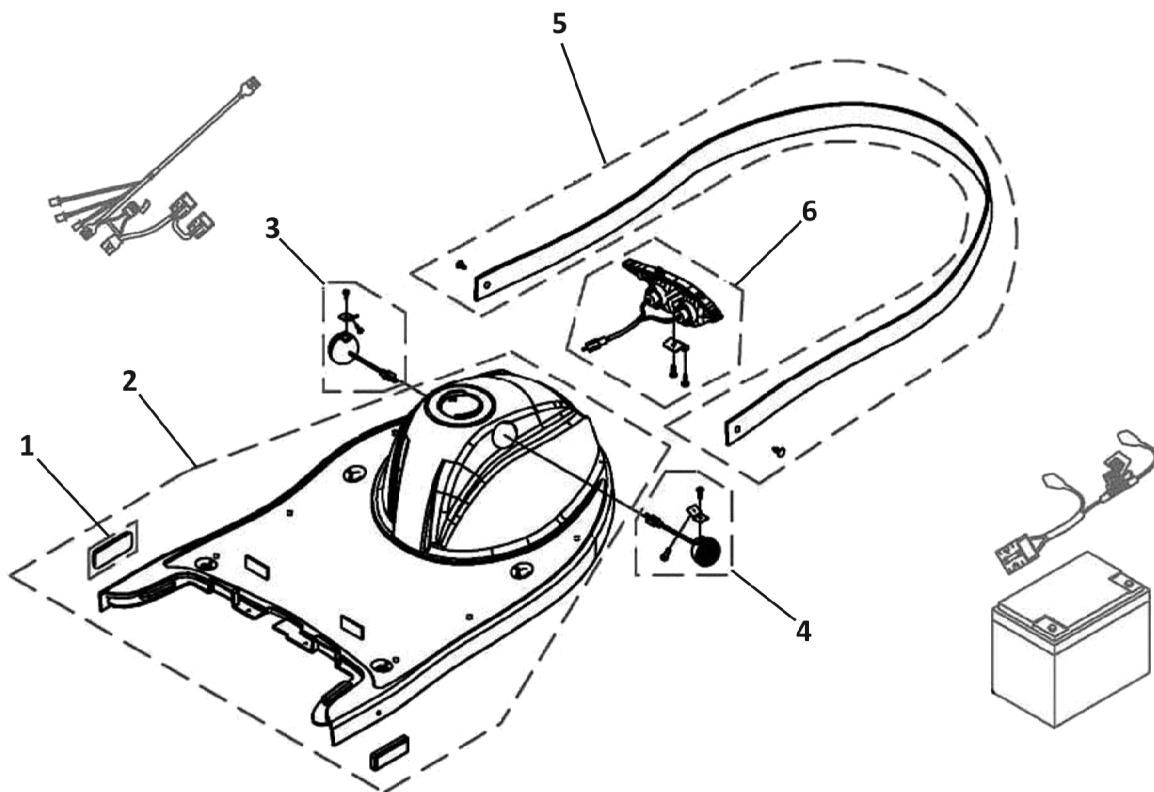


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1517331	Reflector, Front Shroud	--	2
2		1517882	Shroud, Three Wheel Front, Silver	--	1
2		1521655	Shroud, Three Wheel Front, Blue	--	1
3		1518016	Turn Signal - Left	--	1
4		1518017	Turn Signal - Right	--	1
5		1518012	Moulding, Front Shroud	--	1
6		1527473	Head Light, Complete	--	1

Front Shroud - Four Wheeler

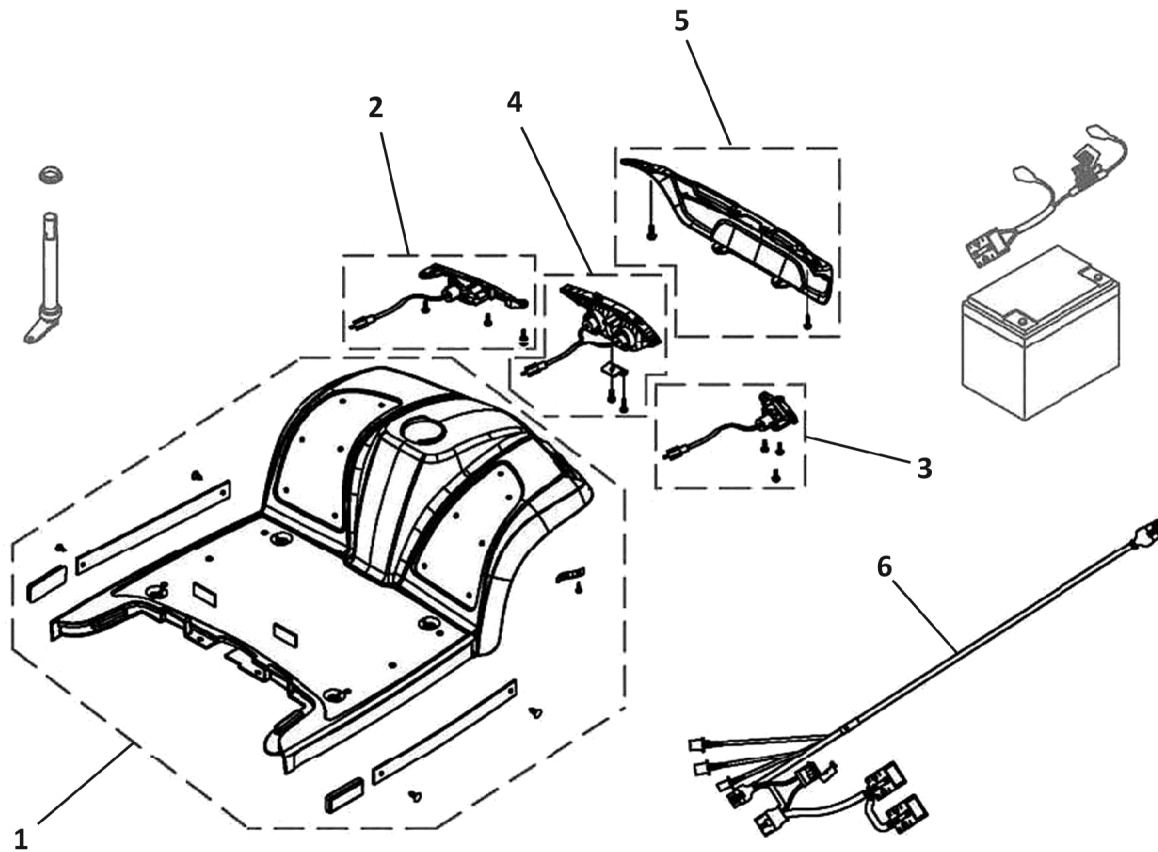


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1517290	Shroud, Four Wheel Front, Silver	--	1
1		1521654	Shroud, Four Wheel Front, Blue	--	1
1		1530000	Shroud, Four Wheel Front, Red	--	1
2		1517733	Turn Signal - Left	--	1
3		1517734	Turn Signal - Right	--	1
4		1527473	Head Light, Complete	--	1
5		1517881	Cover, Front Bumper	--	1

Frame - Three Wheeler

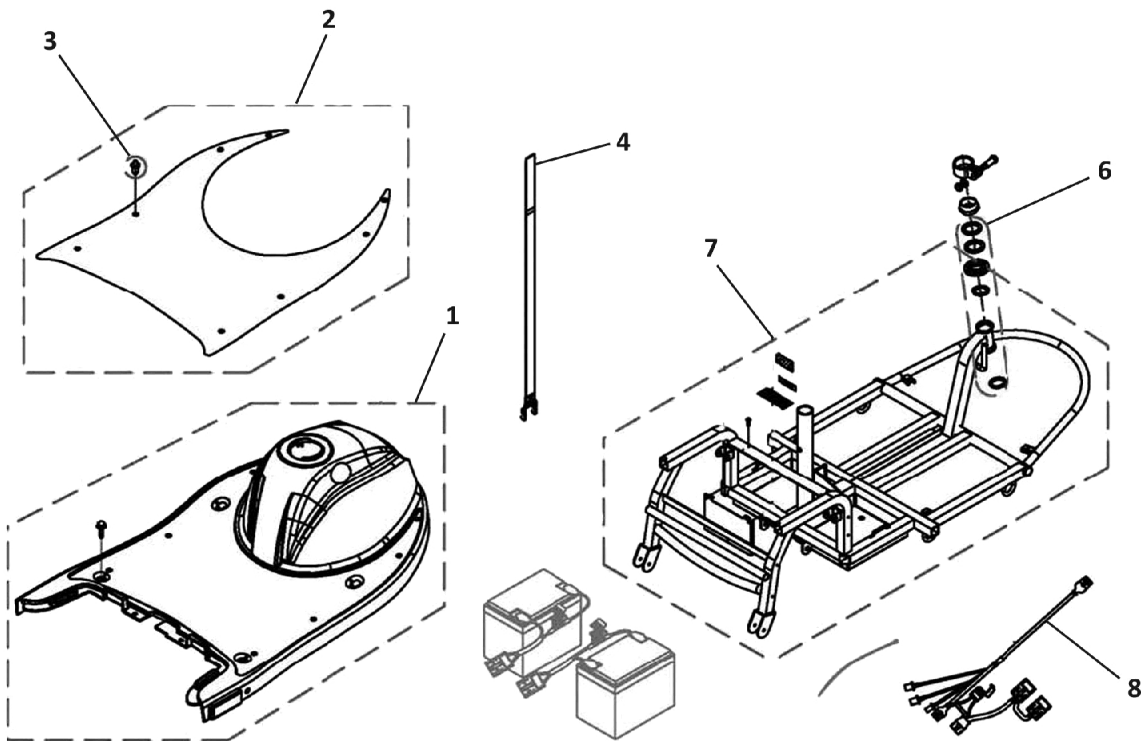

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1517882	Shroud, Three Wheel Front, Silver	--	1
1		1521655	Shroud, Three Wheel Front, Blue	--	1
2		1518014	Pad, Leg Shield	--	1
3		1505476	Push Rivet, Pad	--	8
4		1505482	Band, Battery, Complete	--	1
6		1505487	Bearing, Scooter Frame	--	1
7		1518015	Frame, Three Wheel Scooter	--	1


Front Frame - Four Wheeler

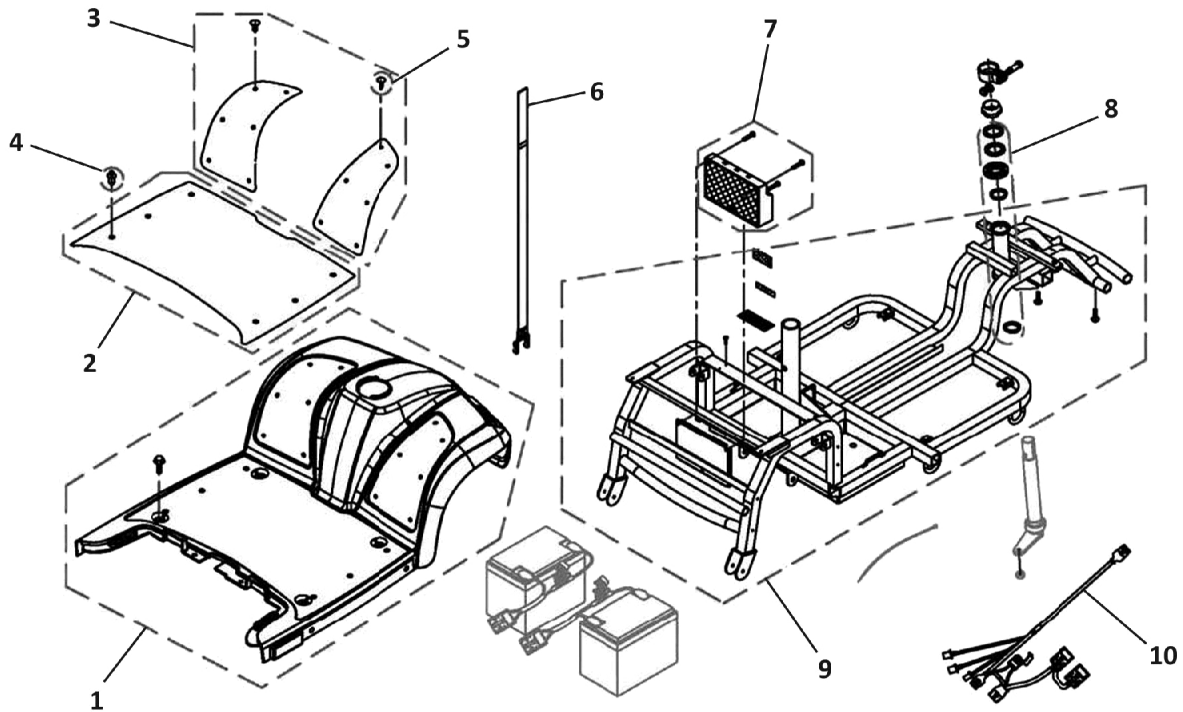

Front Frame - Four Wheeler


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1517290	Shroud, Four Wheel Front, Silver	--	1
1		1521654	Shroud, Four Wheel Front, Blue	--	1
2	A	1517291	Pad, Leg Shield	--	1
3	B	1517292	Pad, Front - Right/Left	--	1
4		1505476	Push Rivet, Pad	--	6
5		1505479	Tree Rivet	--	12
6		1505482	Band, Battery, Complete	--	1
7	C	1517316	Controller, Rhino (70A)	--	1
7	D	1553399	Controller, Rhino (120A)	--	1
8		1505487	Bearing, Scooter Frame	--	1
NOTE: A - Also includes item 4. B - Also includes item 5. C - For scooters manufactured prior to June 2011. D - For scooters manufactured after June 2011.					

Rear Shroud

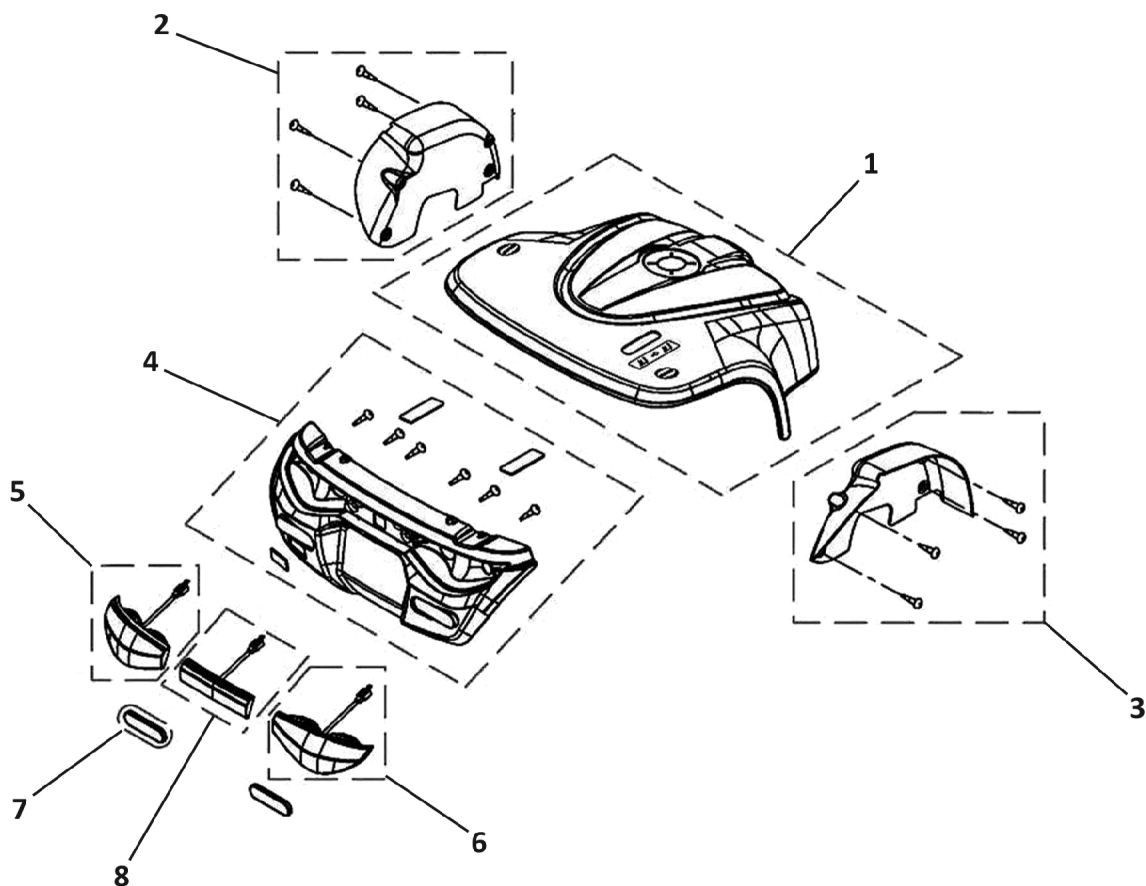


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1518037	Shroud, Rear, Complete, Silver	--	1
1		1521656	Shroud, Rear, Complete, Blue	--	1
1		1530003	Shroud, Rear, Complete, Red	--	1
2		1518030	Fender - Left	--	1
3		1518029	Fender - Right	--	1
4		1518032	Cover, Rear Bumper	--	1
5		1518034	Turn Signal, Rear - Left	--	1
6		1518033	Turn Signal, Rear - Right	--	1
7		1518036	Reflector, Rear, E-Mark	--	2
8		1518035	Neon Light, Complete	--	1

Rear Frame

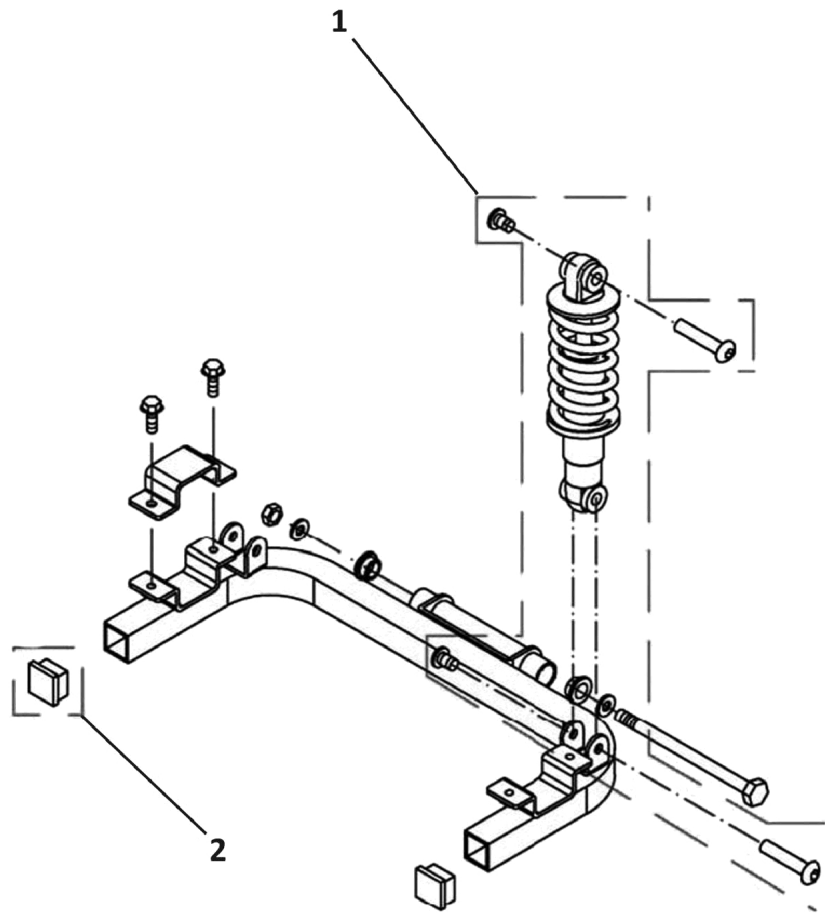


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521600	Shock Absorber	--	2
2		1521597	Plug, Cane Bracket	--	2

Seat and Seat Options

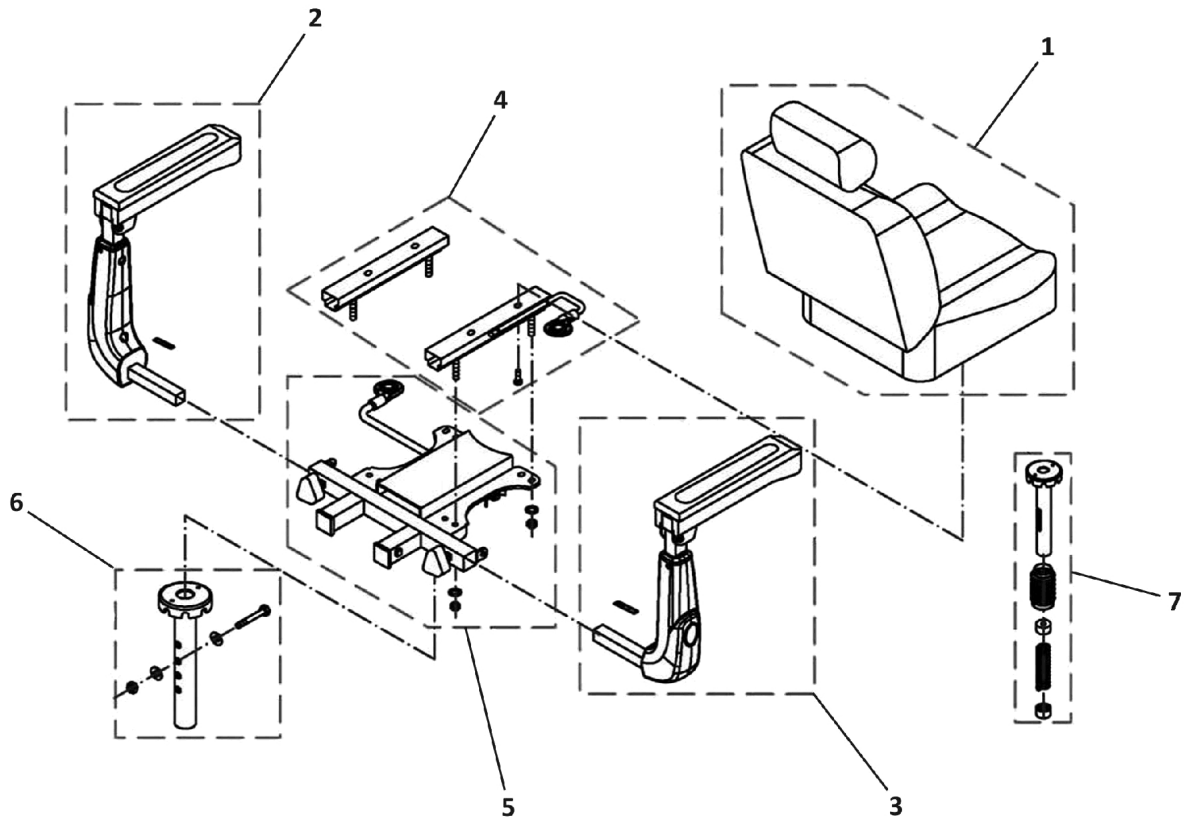


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521687	Standard Seat	--	1
1		1586114	Standard Seat w/ Double White Stitching	--	1
1		1529380	Headrest (Only)	--	1
2		1521689	Armrest - Left	--	1
3		1521688	Armrest - Right	--	1
4		1521690	Slide Assembly	--	1
5		1505533	Seat Bracket Assembly	--	1
6		1521691	Seat Bracket, Standard	--	1
7		1541629	Seat Suspension (560 mm)	--	1
7		1541628	Seat Suspension (535 mm)	--	1

Front Wheel - Three Wheeler

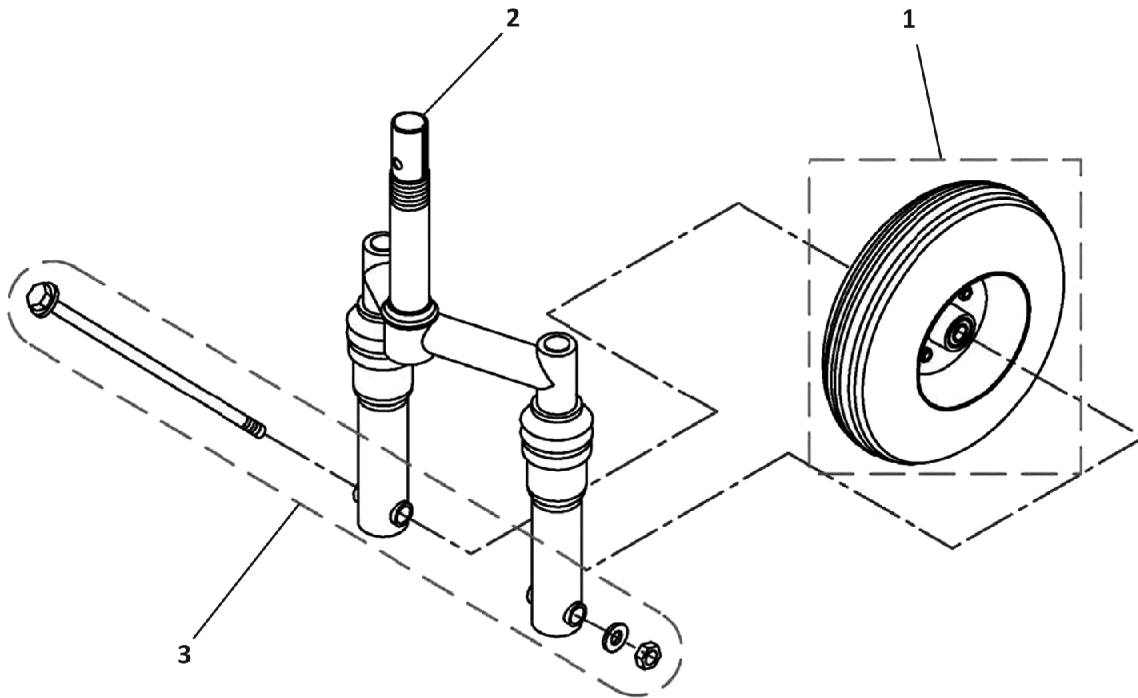


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521587	Wheel, Front, Complete	--	1
2		1518058	Fork Assembly, Complete	--	1
3		1518069	Axle, Front	--	1

Front Wheel and Steering - Four Wheeler

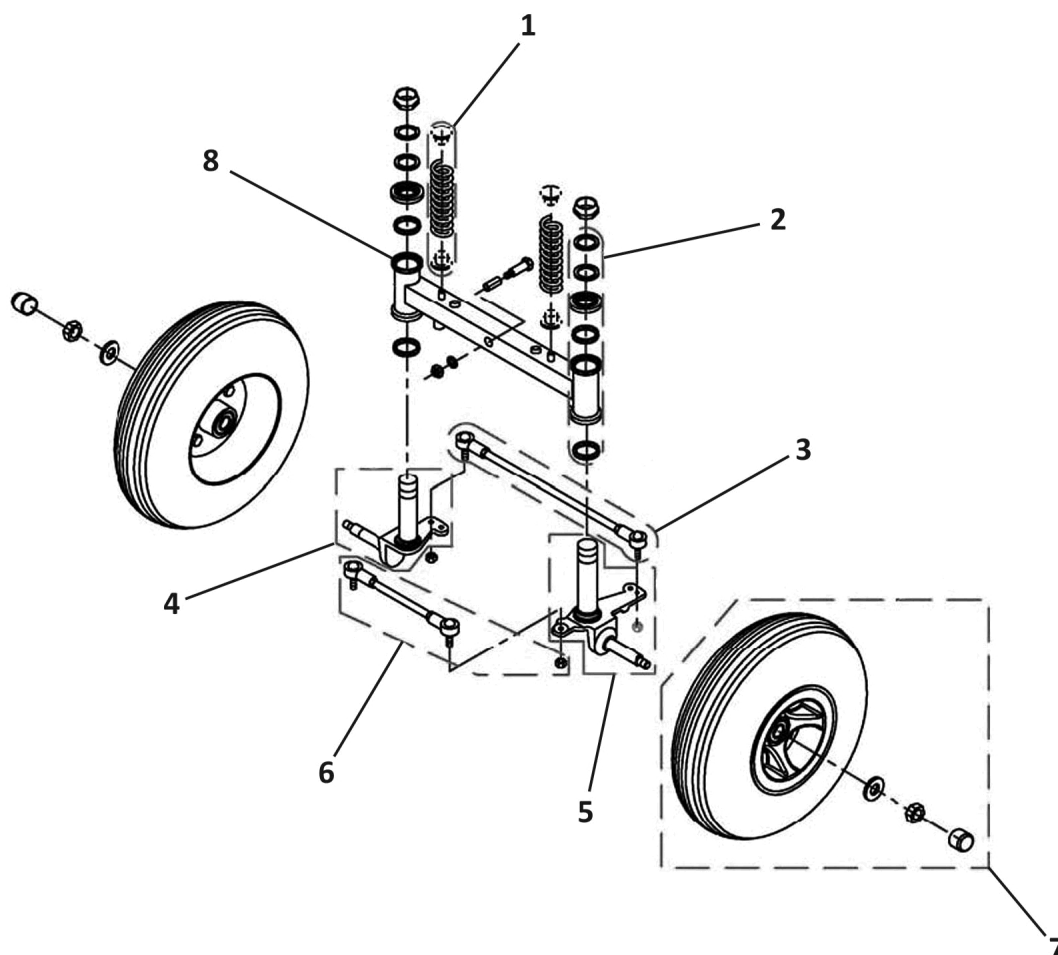


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1518196	Spring, Front Wheel	--	2
2		1505487	Bearing, Scooter Frame	--	1
3		1518077	Rod Assembly, Front	--	1
4		1518237	Axle, Front - Left	--	1
5		1518238	Axle, Front - Right	--	1
6		1518078	Rod Assembly, Rear	--	1
7		1518333	Wheel Assembly, Front	--	2
8		1518076	Swing Assembly	--	1

Rear Wheels and Propulsion (12km/h)

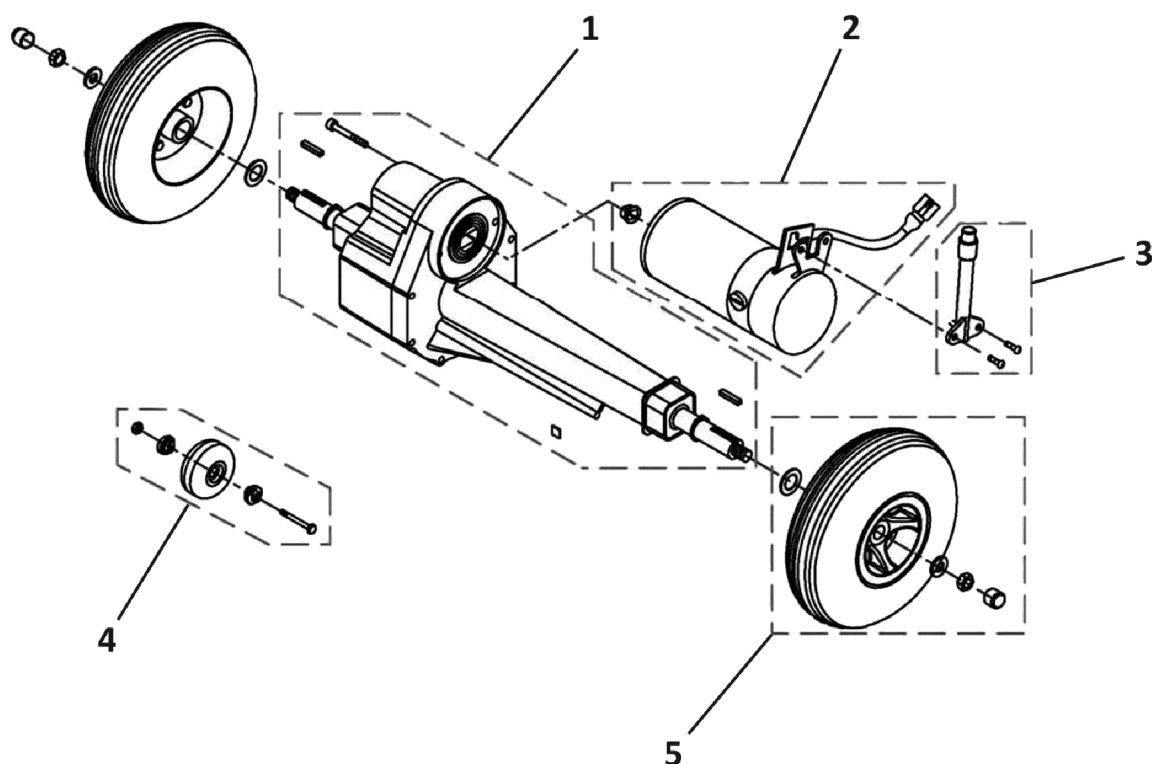


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521591	Transaxle (12 km/h)	--	1
2		1553428	Motor (12 km/h)	--	1
3		1505672	Lever, Motor	--	1
4		1505676	Wheel, Anti-Tip	--	2
5		1518378	Wheel Assembly, Rear	--	2

Brakes - Three Wheeler

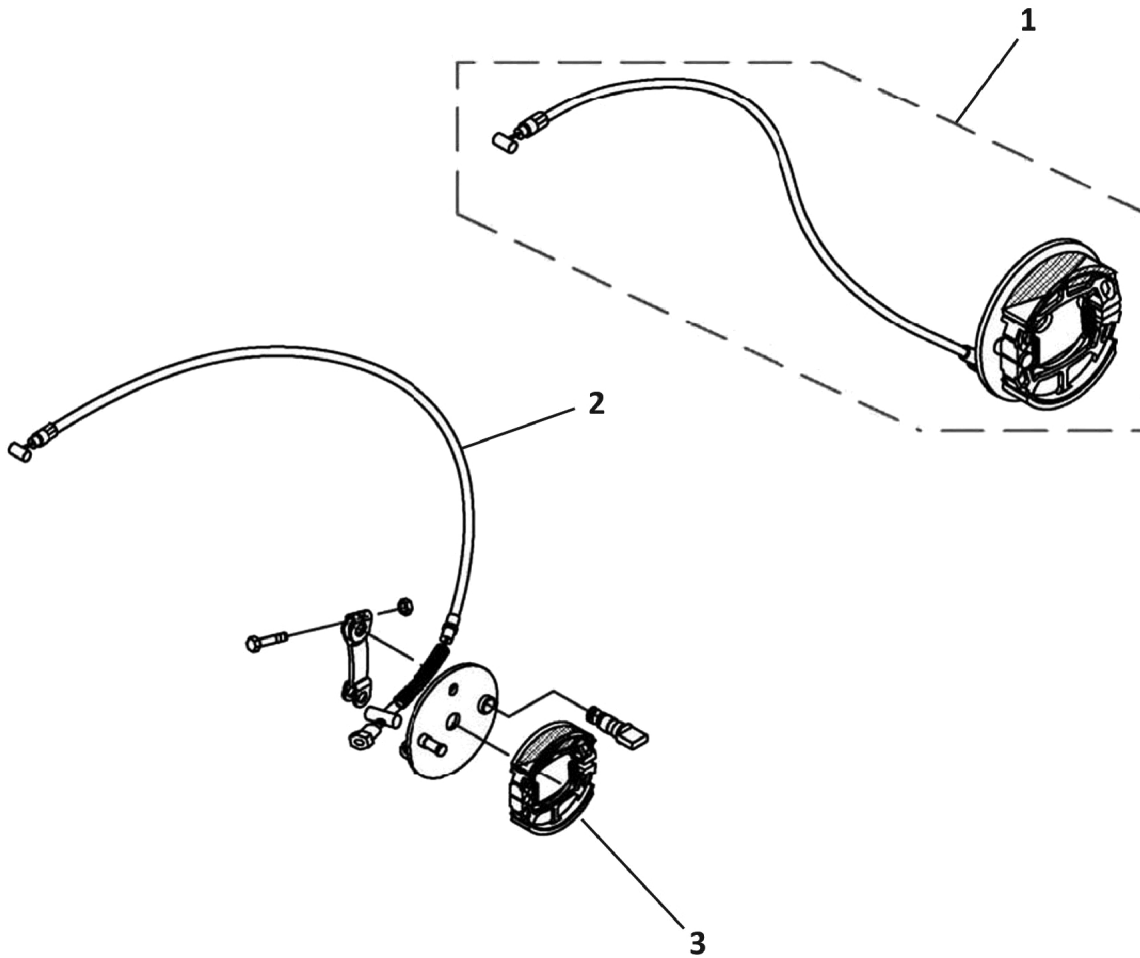


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521588	Brake Assembly, Three Wheel	--	1
2		1518419	Cable, Brake, Three Wheel	--	1
3		1521582	Shoe, Brake, Complete	--	1

Brakes - Four Wheeler

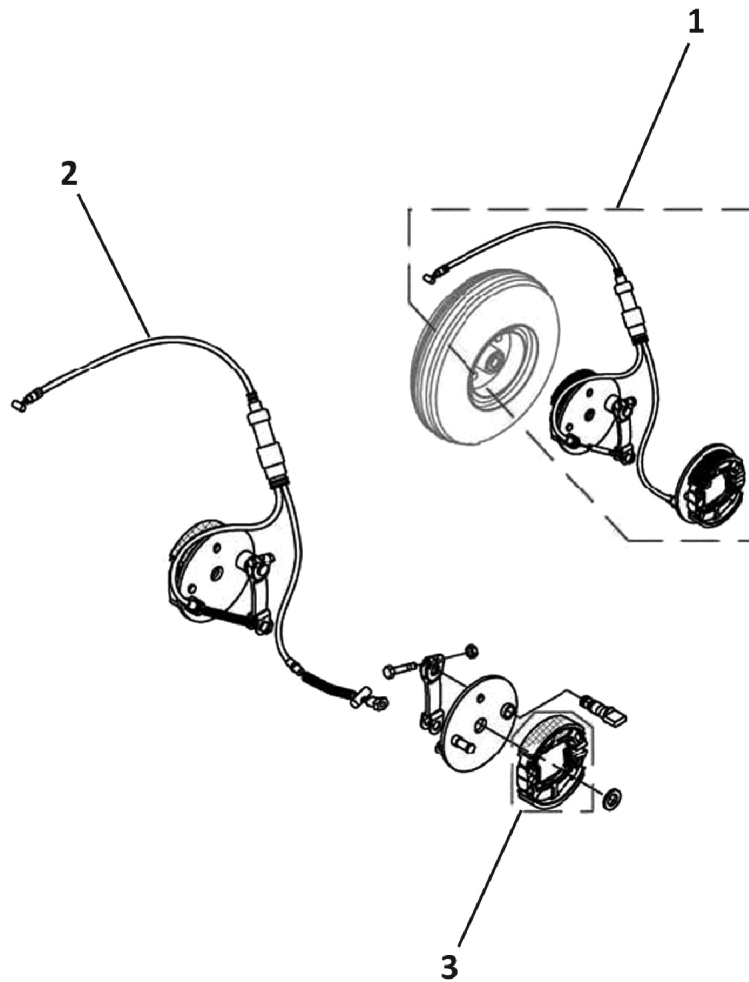

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521589	Brake Assembly, Four Wheel	--	1
2		1518422	Cable, Brake, Four Wheel	--	1
3		1521582	Shoe, Brake, Complete	--	1


Steering

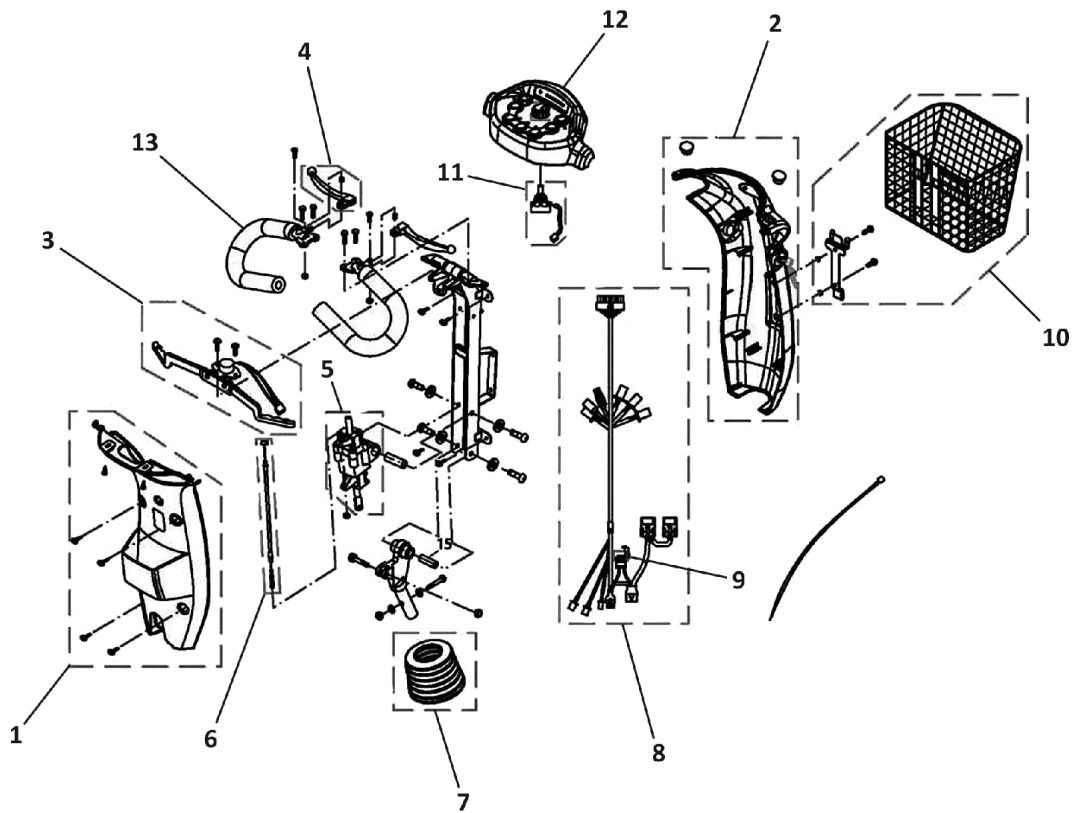

Steering

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521497	Cover, Steering Rear, Complete, Silver	--	1
1		1521660	Cover, Steering, Rear, Complete, Blue	--	1
1		1530006	Cover, Steering, Front, Complete, Red	--	1
2		1521495	Cover, Steering, Front, Complete, Silver	--	1
2		1521659	Cover, Steering, Front, Complete, Blue	--	1
3		1521575	Starter	--	1
4		1521578	Regulator Rod	--	1
5		1505279	Adjustable Steering Assembly	--	1
6		1521583	Cable, Starter to Adjustable Steering Assembly	--	1
7		1521586	Cover, Steering Joint	--	1
8		1521576	Wire Harness	--	1
9		1521577	Fuse (10 Amp)	--	1
10		1521584	Basket, Front	--	1
11		1505462	Variable Resistor, Complete (30.0KO)	--	1
12		1505460	Knob, Speed Control	--	1
13		1527790	Foam Grip	--	2
	A	1164848	Package, Ignition Key	10	1
NOTE: A - Not shown in the illustration.					

Electrical - Wires and Fuses

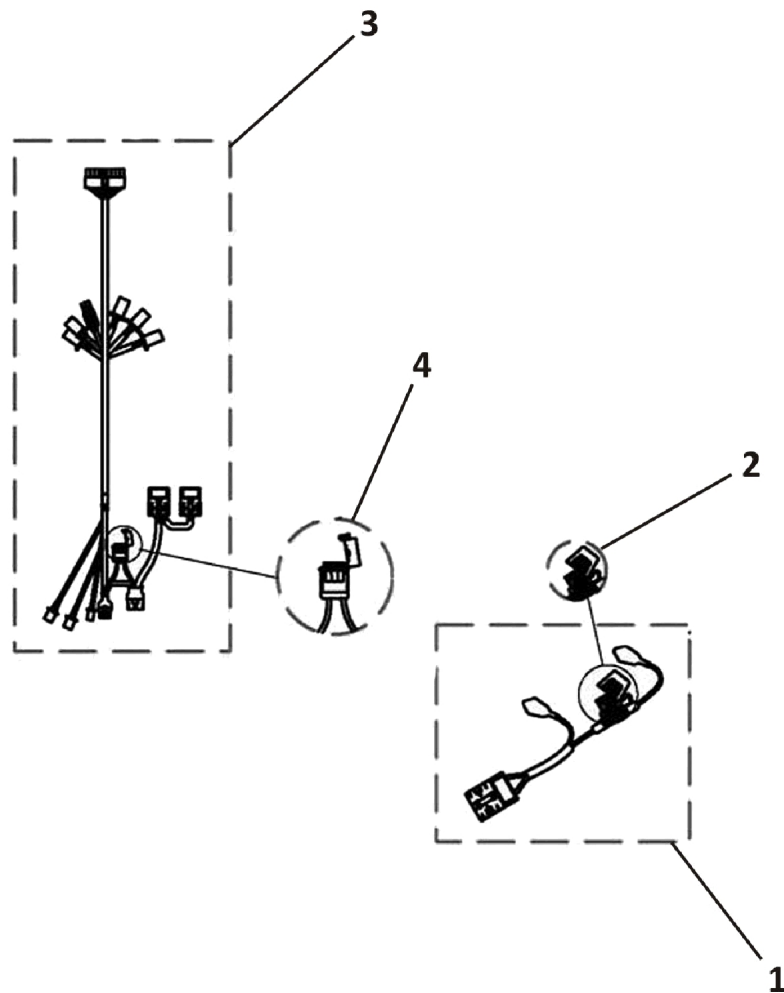

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1521696	Wire, Battery Output	--	1
2		1505500	Fuse (70 Amp)	--	1
3		1521576	Wire Harness	--	1
4		1521577	Fuse (10 Amp)	--	1

Invacare Canada L.P.

www.invacare.ca



Yes, you can.

570 Matheson Blvd E;8
Mississauga, ON,
L4Z 4G4
905-890-8300

Technical Support
800-668-5324
Customer Service
800-668-5324

For Fax Orders
800-668-5478

© 2016 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
(Revised 01/21/16)

Form No. 11-011C