


Axle and Axle Mounting Plates

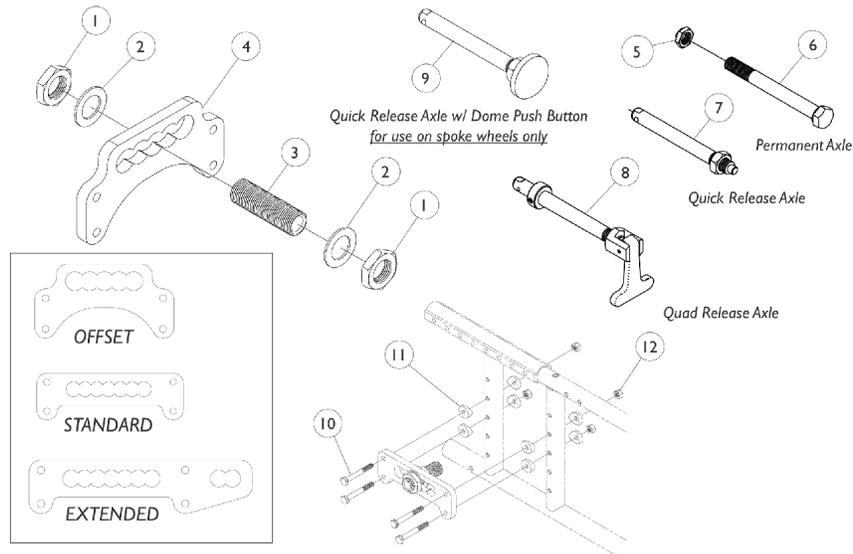

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1044945	Nut, Hex (3/4-16)	1	1
2		1025701	Washer, Steel (3/4 x 1-1/4 x 1/16")	1	2
3	D	1026178	Bushing, Axle (2-1/2")	1	1
3	E	1043488-NLA	NLA - Bushing, Axle (2-1/8")	1	1
4		1059099	Plate, Axle Mounting -Standard	1	1
4		1059101-NLA	NLA - Plate, Axle Mounting -Offset	1	1
4		1059103-NLA	NLA - Plate, Axle Mounting -Extended	1	1
5		1025737	Locknut (1/2-20)	1	1
6		1049227	Screw, Hex Head (1/2-20 x 5-1/4")	1	1
7	A	1026930-NLA	NLA - Axle, Quick Release (1/2 x 4-1/2")	1	1
7	B	1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	1
8		1043487-NLA	NLA - Axle, Quad Release	1	1
9	C	1068038-NLA	NLA - Axle, Quick Release (Dome)	1	1
9		1113049	Axle, Quick Release w/ Dome Button, Black	1	1
10		1055349	Screw, Hex Head (1/4-28 x 2")	1	4
11		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	8
12		1065362	Locknut (1/4-28)	1	4

NOTE: A - Chairs built Before 9/94
 B - Chairs built After 8/94
 C - Q.R. Axle (1068038) had an Anodized Dome finish. Q.R. Axle part no. 1113049, w/ Black dome button, can be used as a replacement but may want to replace in pairs due to aesthetics.
 D - Quick Release or Permanent Axles
 E - Quad Release Axles