


Axle Mounting Plate and Hardware

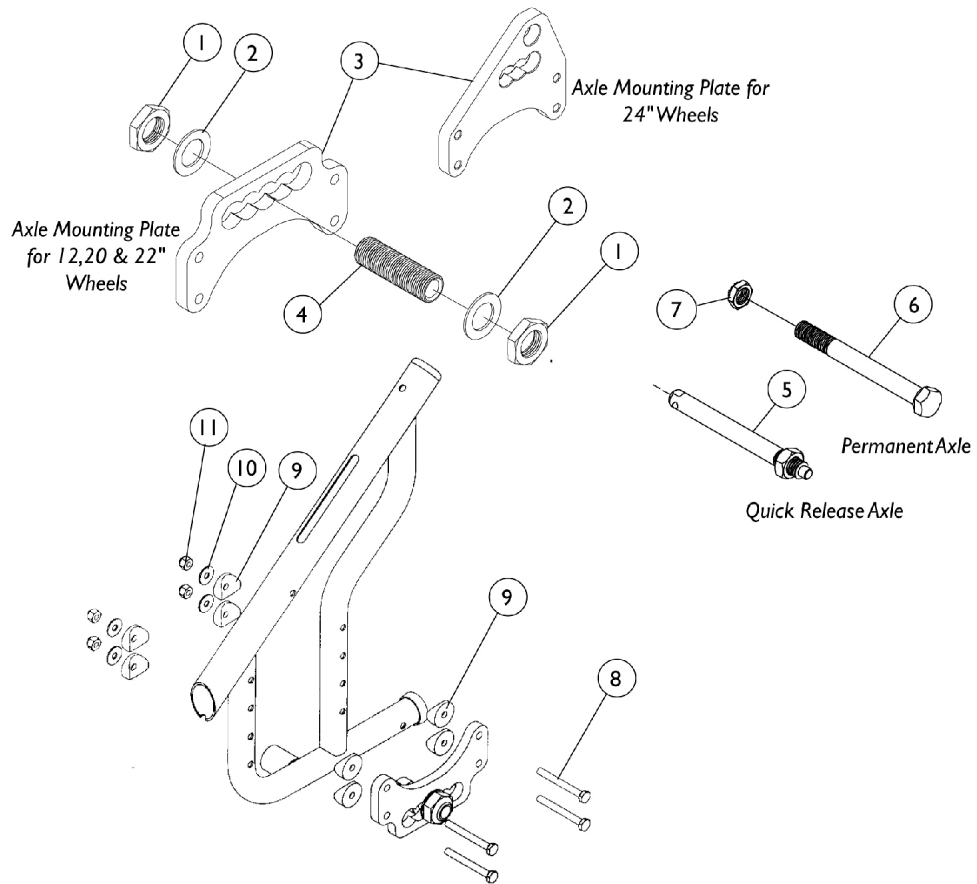

Axle Mounting Plate and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1044945	Nut, Hex (3/4-16)	1	1
2		1025701	Washer, Steel (3/4 x 1-1/4 x 1/16")	1	1
3	A	1059101-NLA	NLA - Plate, Axle Mounting -Offset	1	1
3	B	1106331-NLA	NLA - Plate, Axle Mounting	1	1
4		1026178	Bushing, Axle (2-1/2")	1	1
5		1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	1
6		1049227	Screw, Hex Head (1/2-20 x 5-1/4")	1	1
7		1025737	Locknut (1/2-20)	1	1
8		1055350	Screw, Hex Head (1/4-28 x 2-1/4")	1	8
9		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	16
10		1026900	Washer (1/4 x 23/32 x 1/16")	1	8
11		1065362	Locknut (1/4-28)	1	8

NOTE: A - 12",20", and 22" Rear Wheels
 B - 24" Rear Wheels