


Adjustable Angle Footplates with Pivot Tube - AT5543 (Chairs Built Before 6/01/03)

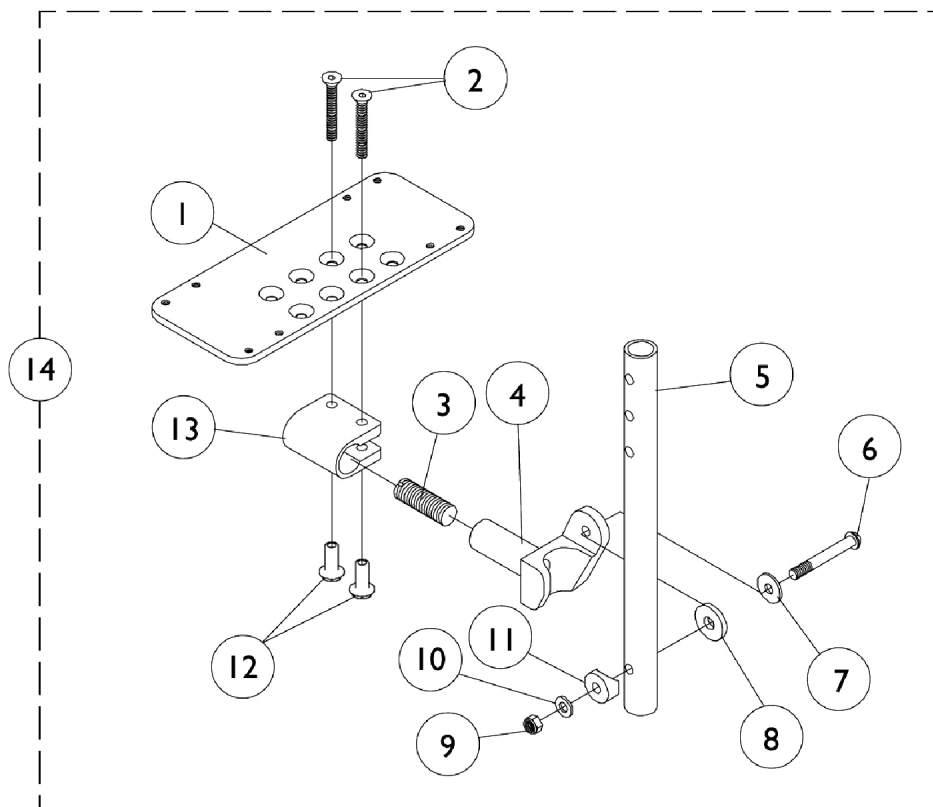

Adjustable Angle Footplates with Pivot Tube - AT5543 (Chairs Built Before 6/01/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1060425-NLA	NLA - Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
1	B	1060426-NLA	NLA - Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
1	B	1060427-NLA	NLA - Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
1	B	1062569-NLA	NLA - Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
1	B	1068617-NLA	NLA - Footplate, Adjustable Angle, XX-Large, Black (7-1/4" W x 8" L)	1	1
2	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
4		1052102	Hinge, Footplate - Right	1	1
4		1052103	Hinge, Footplate - Left	1	1
5		1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
5		1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
6		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
7		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
8		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
9		7930333	Locknut (1/4-20)	25	1
10	A	1026907	Washer (1/4 x 1/2 x 1/16")	1	1
11		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
12		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
13		1058517	Clamp, Footplate	1	1
14		See Chart	#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's	1	1
14		See Chart	#14 70 Degree Tapered	1	1
14		See Chart	#14 90 Degree and 70 Degree MFX	1	1
14		See Chart	#14 90 Degree and 70 Degree MFX	1	1

NOTE: A - Due to a hardware change, for chairs built Before 8/99 items 2, 12, and 13 must be ordered together.
B - Please refer to the charts in this category for footplate size application specific to front rigging type.

Adjustable Angle Footplates with Pivot Tube - AT5543

(Chairs Built Before 6/01/03)

#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's

Straight Slide Tube (9-7/8")	Right	Left
14-15" Wide (5-1/4" Footplate)	1060432	1060433
16-17" Wide (6-1/4" Footplate)	1062572	1062573
18-22" Wide (7-1/4" Footplate)	1068620	1068621
ALPT Straight Slide Tube (Aluminum 11-7/8")	Right	Left
14-15" Wide (5-1/4" Footplate)	1060440-NLA	1060441
16-17" Wide (6-1/4" Footplate)	1062578	1062579
18-22" Wide (7-1/4" Footplate)	1068622	1068623

Adjustable Angle Footplates with Pivot Tube - AT5543

(Chairs Built Before 6/01/03)

#14 70 Degree Tapered

Straight Pivot Tube (9-7/8" Long)	Right	Left
16" Wide (4-1/4" Footplate)	1060430	1060431
17-18" Wide (5-1/4" Footplate)	1060432	1060433
19-22" Wide (6-1/4" Footplate)	1062572	1062573
ALPT Straight Pivot Tube (11-7/8" Long)	Right	Left
16" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
17-18" Wide (5-1/4" Footplate)	1060440-NLA	1060441
19-22" Wide (6-1/4" Footplate)	1062578	1062579

#14 90 Degree and 70 Degree MFX

No Pivot Tube, Bolts Through Support	Right	Left
14-15" Wide (5-1/4" Footplate)	1060438	1060439
16-17" Wide (6-1/4" Footplate)	1062570	1062571
18-22" Wide (7-1/4" Footplate)	1068618	1068619