


Bilateral Contracture Footrest

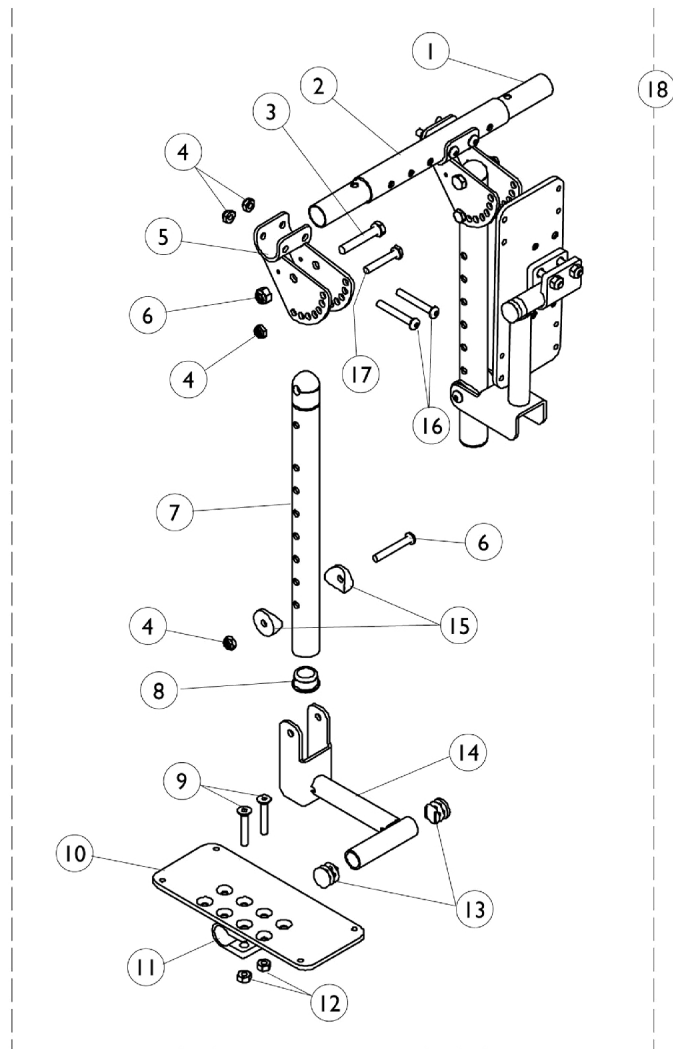

Bilateral Contracture Footrest

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Insert Tube	1	1
2		See Chart	#2 Outer Tube	1	1
3		1044939-NLA	NLA - Screw, Hex Head (5/16-18 x 1-3/4")	1	2
4		1025195	Locknut (1/4-20)	1	8
5		1093925-NLA	NLA - Bracket, Angle Adjustment, Black	1	2
6		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	2
7		1093923-NLA	NLA - Support, Contracture, Black	1	2
8		1056328	Plug Button, Black (1")	1	2
9		1060097-NLA	NLA - Screw, Socket Flat Head (1/4-28 x 1-1/2")	1	4
10		1060425-NLA	NLA - Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	2
10	A	1060426-NLA	NLA - Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	2
10	B	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	2
10	A	1060427-NLA	NLA - Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	2
10	B	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	2
10	A,C	1062569-NLA	NLA - Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	2
10	B	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	2
10	A	1068617-NLA	NLA - Footplate, Adjustable Angle, XX-Large, Black (7-1/4" W x 8" L)	1	2
10	B	1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	2
11		1058517	Clamp, Footplate	1	2
12		1025711-NLA	NLA - Nut, Hex (1/4-20)	1	4
13		1027097	Plug Button, Black (3/4")	1	4
14		1093926-NLA	NLA - Bracket, Footplate Mounting, Black - Right	1	1
14		1093927-NLA	NLA - Bracket, Footplate Mounting, Black - Left	1	1
15		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
16		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	4
17		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	2
18		See Chart	#18 Bilateral Contracture Footrest Kit	1	1

NOTE: A - Used before 9/2/08.

B - Used after 9/1/08.

C - Part #1062569 is No Longer Available. Order part #1113574 as a replacement. The footplate design was revised 9/2/08 with rounder corners and accessory holes chaged from eigh to four. It is recommended to replace footplate in pairs to match.

#1 Insert Tube

14" Width	15" Width	16" Width	17" Width	18" Width
1095315-NLA	1095316-NLA	1095317-NLA	1095318-NLA	1095319-NLA
19" Width	20" Width	21" Width	22" Width	

Bilateral Contracture Footrest

#1 Insert Tube

14" Width	15" Width	16" Width	17" Width	18" Width
1095320-NLA	1095321-NLA	1095322-NLA	1095323-NLA	

Bilateral Contracture Footrest

#2 Outer Tube

14" Width	15" Width	16" Width	17" Width	18" Width
1095330-NLA	1095331-NLA	1095332-NLA	1095333-NLA	1095334-NLA
19" Width	20" Width	21" Width	22" Width	
1095335-NLA	1095336-NLA	1095337-NLA	1095338-NLA	

#18 Bilateral Contracture Footrest Kit

14" Width	15" Width	16" Width	17" Width	18" Width
1095340-NLA	1095341-NLA	1095342-NLA	1095343-NLA	1095344-NLA
19" Width	20" Width	21" Width	22" Width	
1095345-NLA	1095346-NLA	1095347-NLA	1095348-NLA	