


Contracture Platform™ Footrest

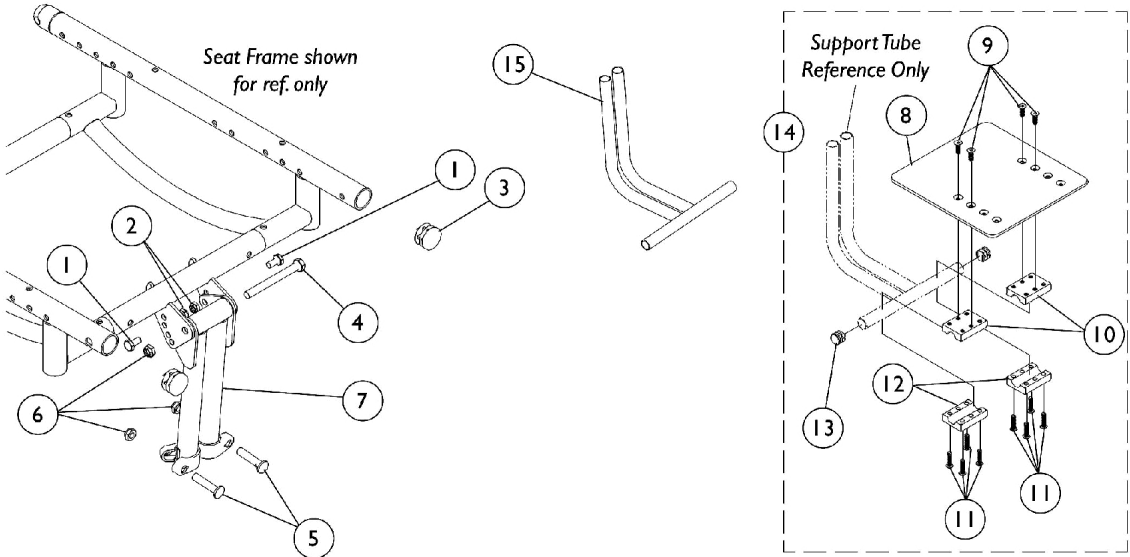

Contracture Platform™ Footrest

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1025752-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/2")	1	2
2		1025195	Locknut (1/4-20)	1	2
3		1029501	Plug Button, Black (1")	1	2
4		1028600-NLA	NLA - Screw, Hex Head (5/16-24 x 3")	1	1
5		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	2
6		1025730	Locknut (5/16-24)	1	3
		1108552-NLA	NLA - Hardware, Contracture Support Attaching	1	1
7		1052436-NLA	NLA - Support, Contracture Platform, Black, Long	1	1
8		1044076	Footboard	1	1
9		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	4
10		1027626-NLA	NLA - Half Clamp, Threaded	1	2
11		1009837	Screw, Socket Button Head (1/4-20 x 1")	1	8
12	E	1027627-NLA	NLA - Half Clamp, Non-Threaded	1	2
13		1027097	Plug Button, Black (3/4")	1	2
14		1052444-NLA	NLA - Footboard with Hardware	1	1
15	C,D	1052441	Tube, Support, Lower, 110 Degree, Black, Long	1	1
NOTE: A - Not Shown B - Includes items 1-6. C - Specify frame color when ordering. D - If "chrome" Support Tube is requested, it must be ordered as "special". Reference color part number when ordering. E - Includes items 8-13.					