


# Seat Frame Assembly & Back Canes

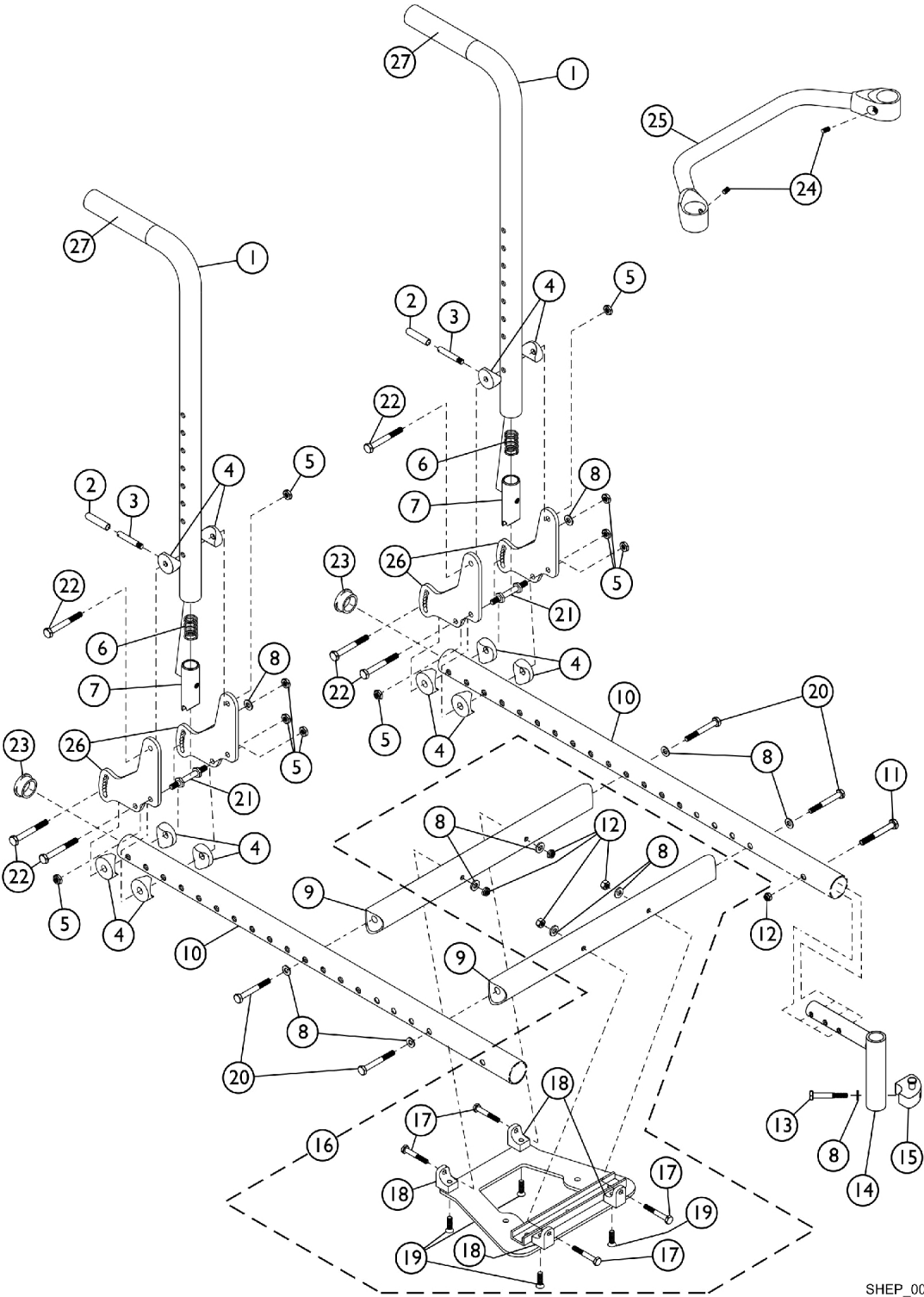

## Seat Frame Assembly & Back Canes

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
1	A	1059249	Cane, Back, Fixed Height, Black - Right/Left (20" H)	1	2	
2		1059271	Tip, Plastic, Black	1	2	
3		1058545	Lever, Actuator	1	2	
4		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	12	
5		1065362	Locknut (1/4-28)	1	10	
6		1011511	Spring, Compression (5/8 x 1 x 3/64")	1	2	
7		1058232	Plug, Latch	1	2	
8		1026907	Washer (1/4 x 1/2 x 1/16")	1	12	
9		See Chart	Item #9 Crossmember	1	2	
10		1082508	Rail, Seat, Black - Right	1	1	
10		1082509	Rail, Seat, Black - Left	1	1	
11		1074640	Screw, Allen Head (1/4-28 x 1-1/4")	1	2	
12		1025765	Locknut (1/4-28)	1	6	
13		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	2	
14		1086389	Tube, Support, Front Rigging Hanger, Black - Right	1	1	
14		1086390	Tube, Support, Front Rigging Hanger, Black - Left	1	1	
15		1054571	Kit, Sector Block - Right	1	1	
15		1054572	Kit, Sector Block - Left	1	1	
16		B	1073711	Upper Seat Plate Assembly and Mounting Hardware	1	1
17			1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	4
18			1071982	Bracket, L, Black	1	4
19			1074191-NLA	NLA - Screw, Socket Flat Head (1/4-28 x 1/2")	1	4
20			1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	4
21			1059127	Pin, Locking	1	2
22			1055349	Screw, Hex Head (1/4-28 x 2")	1	6
23			1027095	Plug Button, Black (7/8")	1	2
24			1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
25	See Chart		Item #25 Spreader Bar	1	1	
26	1059109		Plate, Backrest Mounting	1	4	
27	1061484	Grip, Handle, Foam, Black (5")	1	2		
	See Chart	Growth Kits	1	1		

NOTE: A - Item #27 Foam Handle Grip is not included.  
B - Includes the Upper Seat Plate Assembly and items 8, 12 and 17-19.

# Seat Frame Assembly & Back Canes

## Item #9 Crossmember

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1058329-NLA	1058330-NLA	1058331	1058332-NLA	1058333	1058334	1058335

## Item #25 Spreader Bar

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1042506-NLA	1042507-NLA	1042508-NLA	1042509-NLA	1042510	1042511	1042512

## Growth Kits

**Desired Chair Widths Shown - A Growth Kit contains items 8, 9, 20, 24, and 25. A Spreader Bar, Stroller Handles, Footplates, and Wheel Lock Extensions may also be needed.**

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1073863-NLA	1073864-NLA	1073865	1073866	1073867-NLA	1073868-NLA	1073869