


Base Frames & Hardware

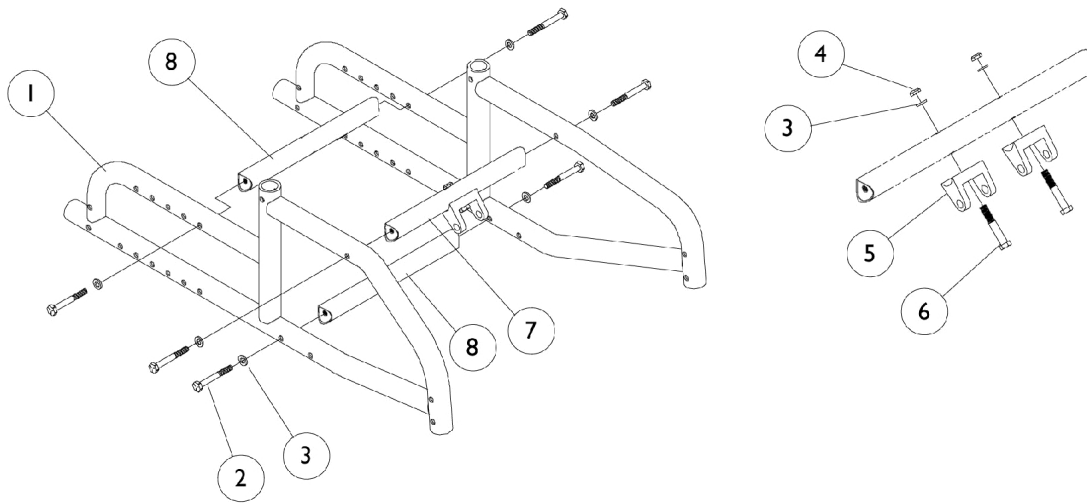

Base Frames & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881069533-NL A	NLA - Frame, Standard, Color - Right (24" L)	1	1
1	A	8881069534-NL A	NLA - Frame, Standard, Color - Left (24" L)	1	1
1	A	8881069535-NL A	NLA - Frame, Large, Color - Right (28" L)	1	1
1	A	8881069536-NL A	NLA - Frame, Large, Color - Left (28" L)	1	1
2		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	6
3		1026907	Washer (1/4 x 1/2 x 1/16")	1	8
4		1025765	Locknut (1/4-28)	1	2
5		1071841-NLA	NLA - Bracket, Cylinder Mounting, Black	1	2
6		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	4
7		See Chart	#7 Crossmember	1	1
8		See Chart	#8 Crossmember	1	1
	B	See Chart	Growth Kits	1	1

NOTE: A - Specify frame color when ordering.
B - Includes items 2, 3, 7 and 8.

#7 Crossmember

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1071183	1071184	1071185	1071186	1071187	1071188	1071189-NLA

#8 Crossmember

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1058329-NLA	1058330-NLA	1058331	1058332-NLA	1058333	1058334	1058335

Growth Kits

Desired Chair Width Shown - Growth kits contain items 2,3,7,8 along with the required Seat Pivot Crossmember and Hardware. A Spreader Bar, Stroller Handles, Footplates, and Wheel Lock Extensions may also be needed.

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1076267-NLA	1076268-NLA	1076269	1076270	1076271	1076272	1076273