


Seat Pivot & Lower Seat Plate Assembly

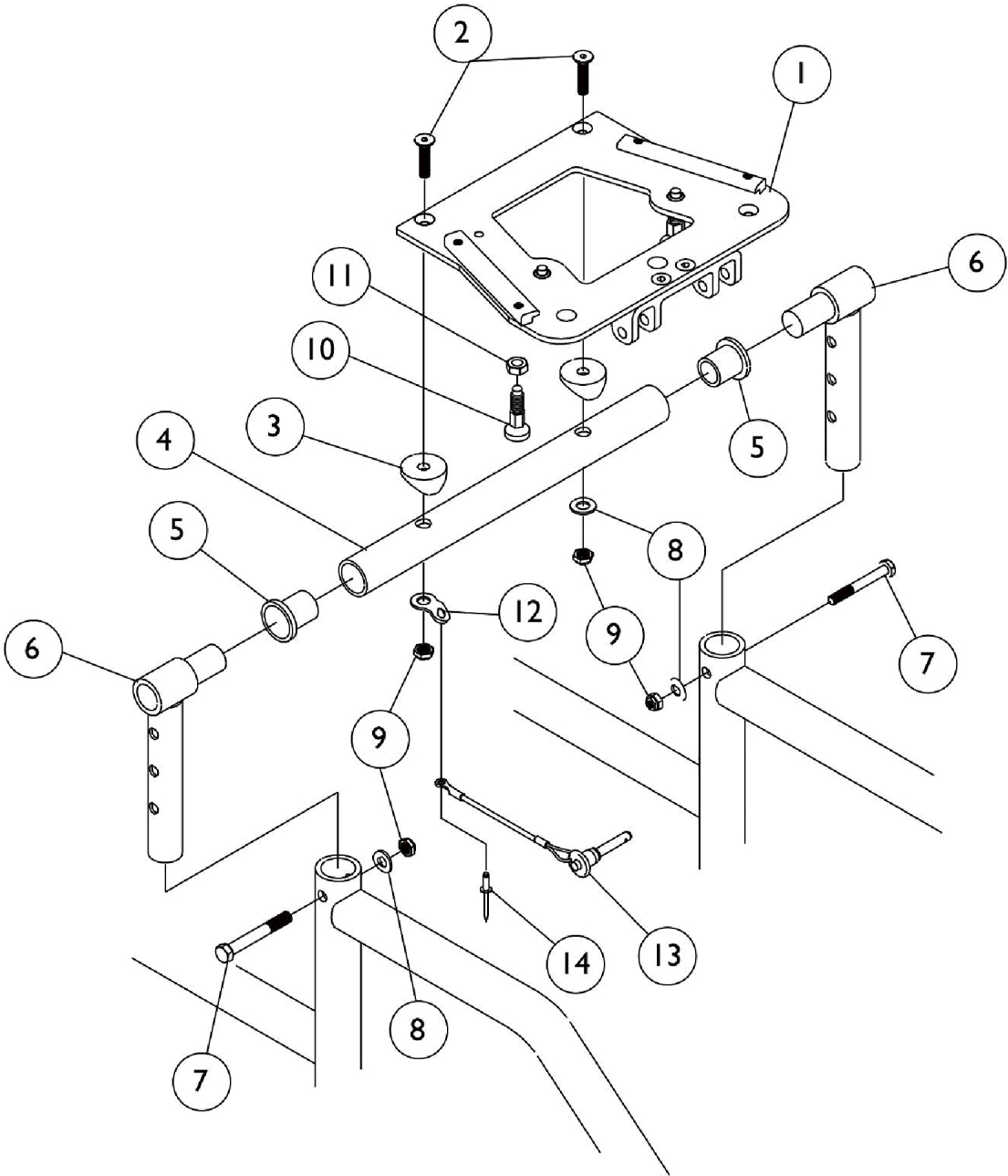

Seat Pivot & Lower Seat Plate Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1073713	Plate, Seat, Lower	1	1
2		1055332-NLA	NLA - Screw, Socket Flat Head (1/4 -28 x 1-3/4")	1	2
3		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
4		See Chart	#4 Pivot Crossmember	1	1
5		1071166-NLA	NLA - Bushing, Pivot	1	2
6		1071162-NLA	NLA - Pivot, Seat Frame	1	2
7		1055310-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/4")	1	2
8		1002104-NLA	NLA - Washer, Flat (1/4 x 1/2 x 1/16")	1	3
9		1065362	Locknut (1/4-28)	1	2
10		1080085-NLA	NLA - Pin, Spring Loaded Plunger	1	2
11		1073708-NLA	NLA - Nut, Hex Jam (1/2-13)	1	2
12		1026903-NLA	NLA - Tab, Tie Down	1	1
13		1045470	Pin, Quick Release with Lanyard	1	1
14		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	1

NOTE: A - Includes Items 10 and 11

#4 Pivot Crossmember

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1073572-NLA	1073573-NLA	1073574-NLA	1073575-NLA	1073576-NLA	1073577-NLA	1073578-NLA