
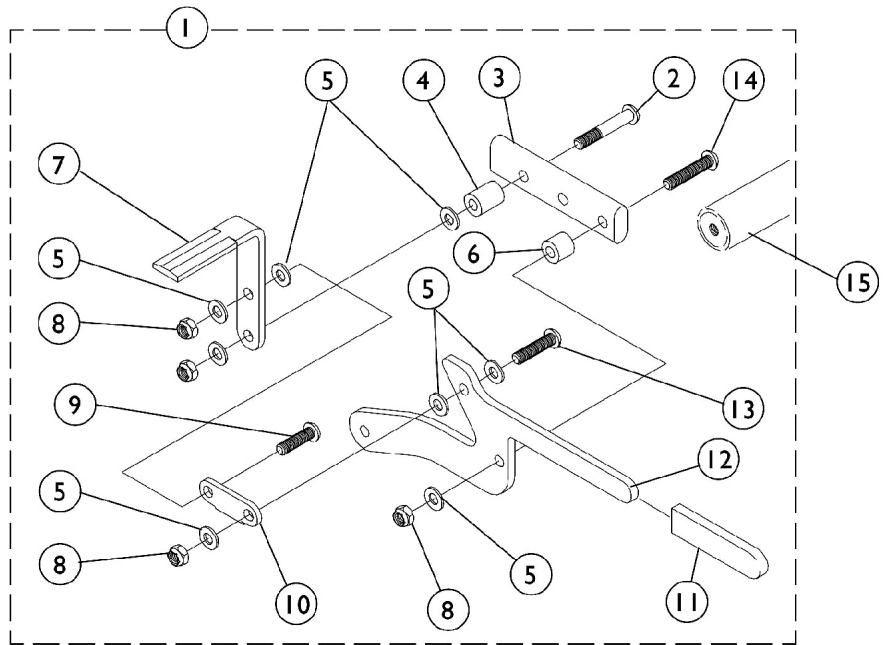
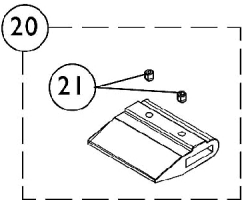
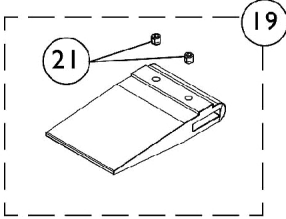


Attendant Operated Wheel Lock

12" Rear Wheels Only

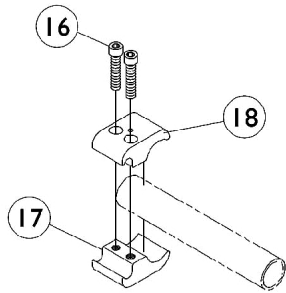

Attendant Operated Wheel Lock 12" Rear Wheels Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1073842	Wheel Lock Assembly - Right	1	1
1	A	1073873	Wheel Lock Assembly - Left	1	1
2		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
3		1028399-NLA	NLA - Strap, Wheel Lock	1	1
4		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
5		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	8
6		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
7		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
8		1025765	Locknut (1/4-28)	1	4
9		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
10		1018733-NLA	NLA - Link, Stop, Black	1	1
11		1075507	Tip, Plastic, Black	1	1
12		1073674-NLA	NLA - Lever, Wheel Lock, Black	1	1
13		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
14		1055307-NLA	NLA - Screw, Allen Button Head (1/4-28 x 1")	1	1
15		See Chart	#15 Wheel Lock Crossmember	1	1
16		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
17		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
18		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
19	B	1037421	Kit, Wheel Lock Shoe Spacer, 20" Rear Wheels (Pair)	2	1
			Spacer, Wheel Lock Shoe (2-63/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
20	C	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	2	1
			Spacer, Wheel Lock Shoe (1-45/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
21		1008438	Screw, Set Socket Cup Point (#8-32 x 3/16")	1	2

NOTE: A - Includes items 2-14
B - 20" Rear Wheels
C - 22" Rear Wheels

#15 Wheel Lock Crossmember

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1071155-NLA	1071156-NLA	1071157	1071158	1071159	1071160	1071161