


Adjustable Stroller Handles (Before 5/30/08)

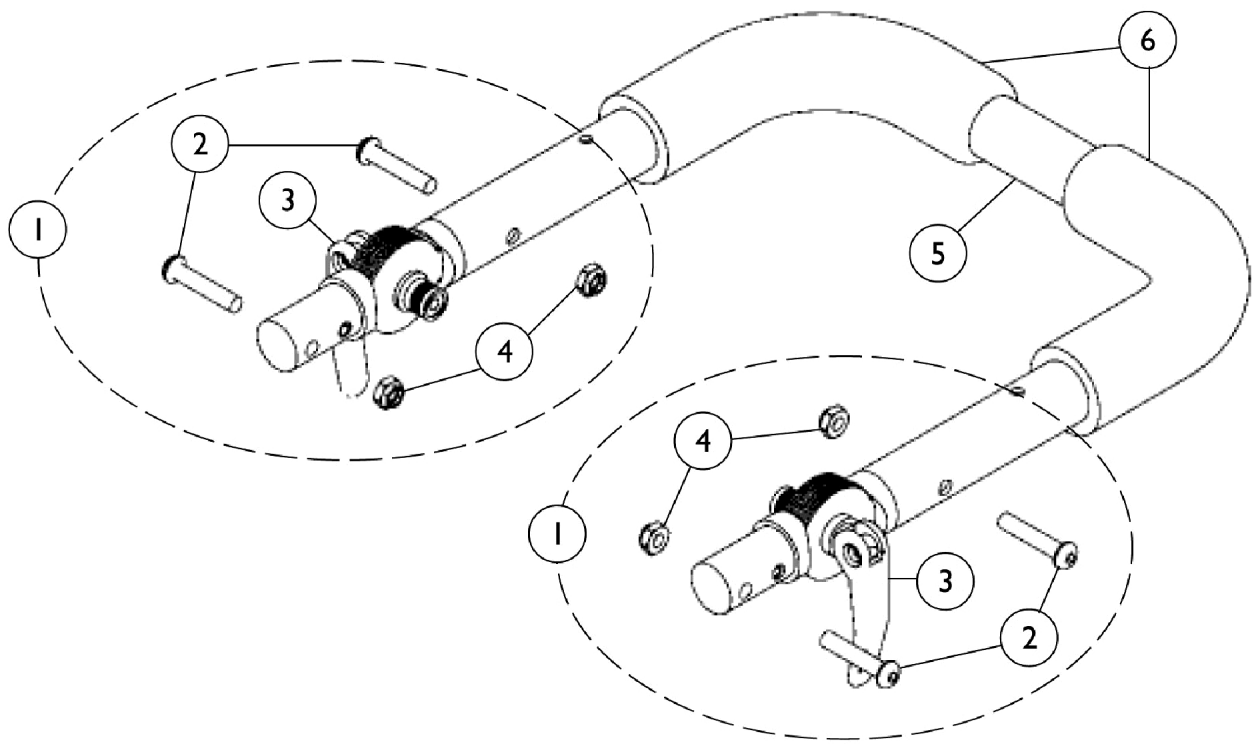

Adjustable Stroller Handles (Before 5/30/08)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1114689-NLA	NLA - Hardware, Stroller Handle Hinge (1 Pair)	2	1
2		1055309	Screw, Socket Button Head (1/4-28 x 1-1/4")	1	4
3		1112539	Folding Hinge Assembly	1	2
4		1025765	Locknut (1/4-28)	1	4
5		See Chart	#5 Adjustable Stroller Handles	1	1
6	C	1032245	Grip, Foam Hand (6-7/8" L)	1	2
6	B	1133698	Grip, Foam Hand (9" L)	1	2

NOTE: A - Includes items 2-4.
 B - 12-16" Wide Chairs
 C - 10-11" Wide Chairs

Item #5 Push Handle with Grip

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1082618-NLA	1082619-NLA	1082620-NLA	1082621-NLA	1082622-NLA	1082623	1082624