

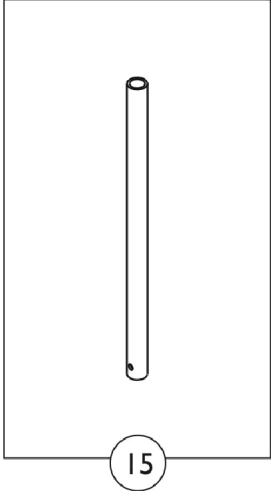
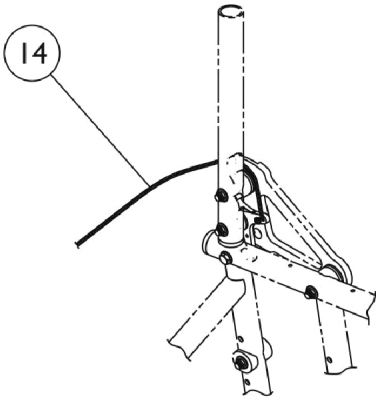


Rear Upright Tube Mounting Hardware & Back Canes

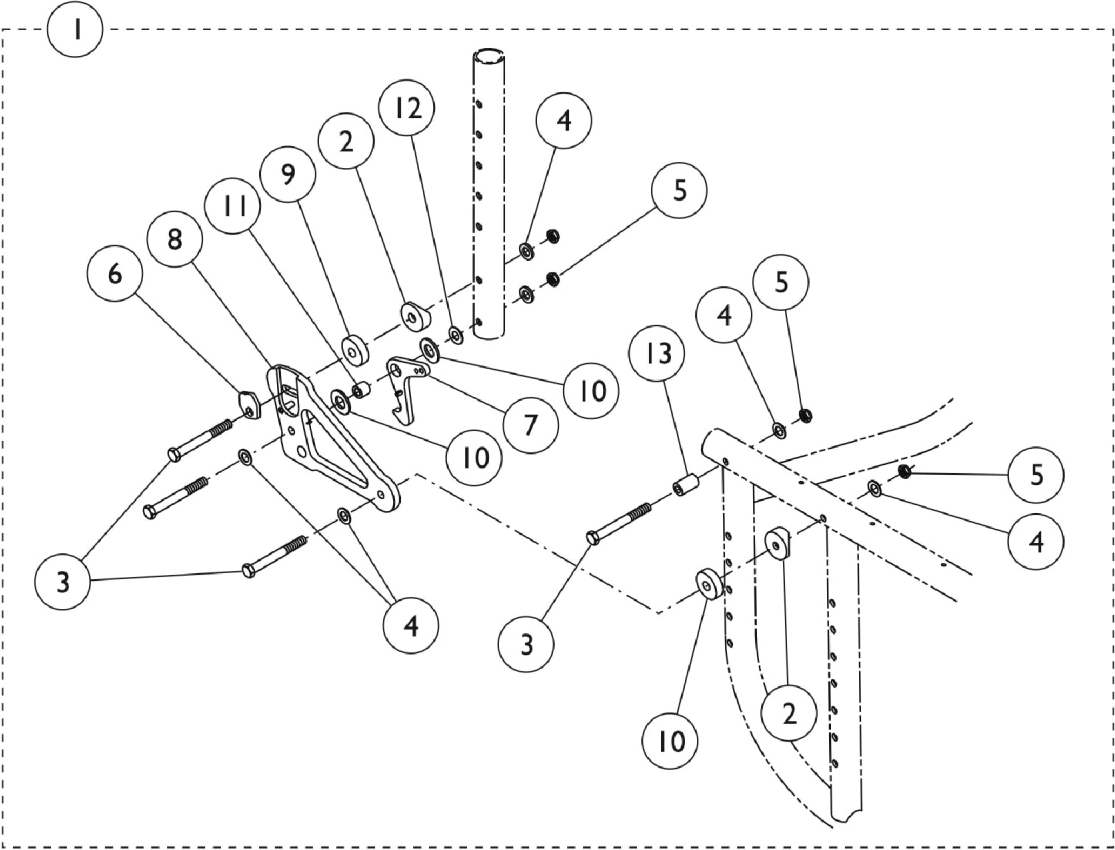

Adjustable Back Cane w/o Push Handle

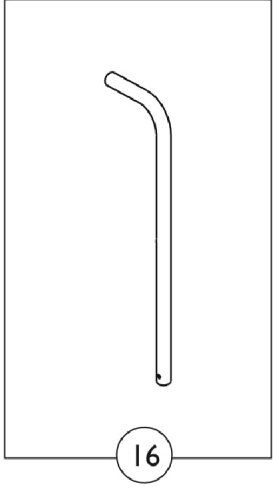

Straight Cane w/ Push Handle

Rear Upright Tube Mounting Hardware & Back Canes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1041149-NLA	NLA - Kit, Backrest Hardware (Pair)	1	1
2		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	2
3		1055349	Screw, Hex Head (1/4-28 x 2")	1	4
4		1026907	Washer (1/4 x 1/2 x 1/16")	1	16
5		1025765	Locknut (1/4-28)	1	4
6		1039232-NLA	NLA - Cam, Adjustment	1	1
7		1081354-NLA	NLA - Latch, Backrest - Right	1	1
7		1081355-NLA	NLA - Latch, Backrest - Left	1	1
8		1079920-NLA	NLA - Plate, Backrest Support -Right	1	1
8		1079921-NLA	NLA - Plate, Backrest Support -Left	1	1
9		1040776-NLA	NLA - Spacer, Nylon (1/4 x 1 x 1/4")	1	2
10		1012371	Washer, Nylon (3/8 x 7/8 x 1/16")	1	1
11		1026162	Spacer (1/4 x 3/8 x 11/32")	1	1
12		1025728	Washer (1/4 x 1/2 x 1/32")	1	1
13		1039233	Spacer (1/4 x 7/16 x 5/8")	1	1
14		1141633	Cord, Nylon (6-1/2 ft.)	1	1
15		1026971	Back Canes, Adjustable w/o Push Handle (8-11" H)	2	1
			Plug Button, Black (3/4")	1	2
			NLA - Tube, Insert, Adjustable Back (8 - 11" H)	1	2
			Spring, Torsion w/ Button	1	2
			Bushing, Pivot	1	2
15		1026972	Back Canes, Adjustable w/o Push Handle (10-14"/16-20" H)	2	1
			Plug Button, Black (3/4")	1	2
			NLA - Tube, Insert, Adjustable Back (10 - 20" H)	1	2
			Spring, Torsion w/ Button	1	2
			Bushing, Pivot	1	2
16		1054773-NLA	NLA - Cane, Back, Straight w/ Push Handle and Hardware (10-14" H)	1	1
			Grip, Foam (8")	1	1
			Plug Button, Black (3/4")	1	1
			NLA - Back Cane w/ Push Handle, Straight (10-14" H)	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			NLA - Locknut (1/4-20)	1	1
16		1054774-NLA	NLA - Cane, Back, Straight w/ Push Handle and Hardware (12-16" H)	1	1
			Grip, Foam (8")	1	1
			Plug Button, Black (3/4")	1	1
			NLA - Back Cane w/ Push Handle, Straight (12-16" H)	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			NLA - Locknut (1/4-20)	1	1
16		1027850-NLA	NLA - Cane, Back, Straight w/ Push Handle and Hardware (12-16" and 14-18" H)	1	1
			Grip, Foam (8")	1	1
			NLA - Cane, Back, Straight, Black - Right/Left	1	1
			Screw w/ Washer (#10 x 7/8")	1	1

Rear Upright Tube Mounting Hardware & Back Canes

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
16			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1	
			NLA - Locknut (1/4-20)	1	1	
		1044031-NLA	NLA - Back Cane w/ Hardware, Straight w/ Push Handle (14-18" and 16-20" H)	1	1	
			NLA - Back Cane, Straight w/ Push Handle	1	1	
			Grip, Foam (8")	1	1	
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1	
			NLA - Locknut (1/4-20)	1	1	
		B,C	1038784-NLA	NLA - Pin, Roll (3/16 x 1/2")	1	1
		B,C	1038785-NLA	NLA - Pin, Roll (1/8 x 3/8")	1	1
		B,C	1039231-NLA	NLA - Latch, Black	1	1
		B,C	1039234-NLA	NLA - Spring, Leaf	1	1
		B	1039235-NLA	NLA - Plate, Backrest Support, Right	1	1
		B	1039236-NLA	NLA - Plate, Backrest Support, Left	1	1
NOTE: A - Includes Items 2 thru 13. B - Not Shown. C - Before 5/7/98 only.						