



Axles & Camber Bars

Before 11/25/96

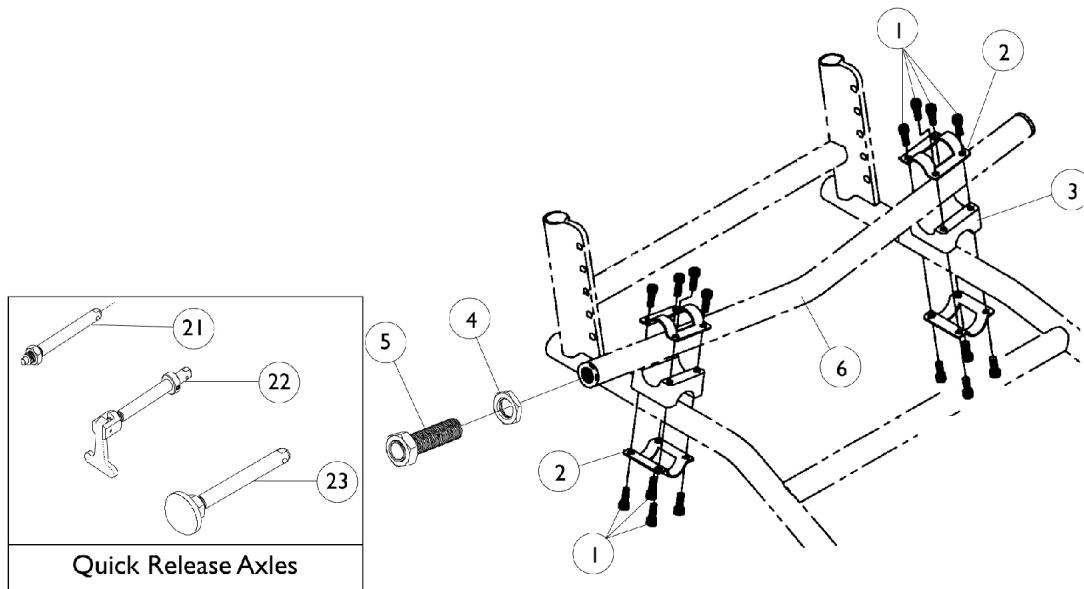

After 11/24/96

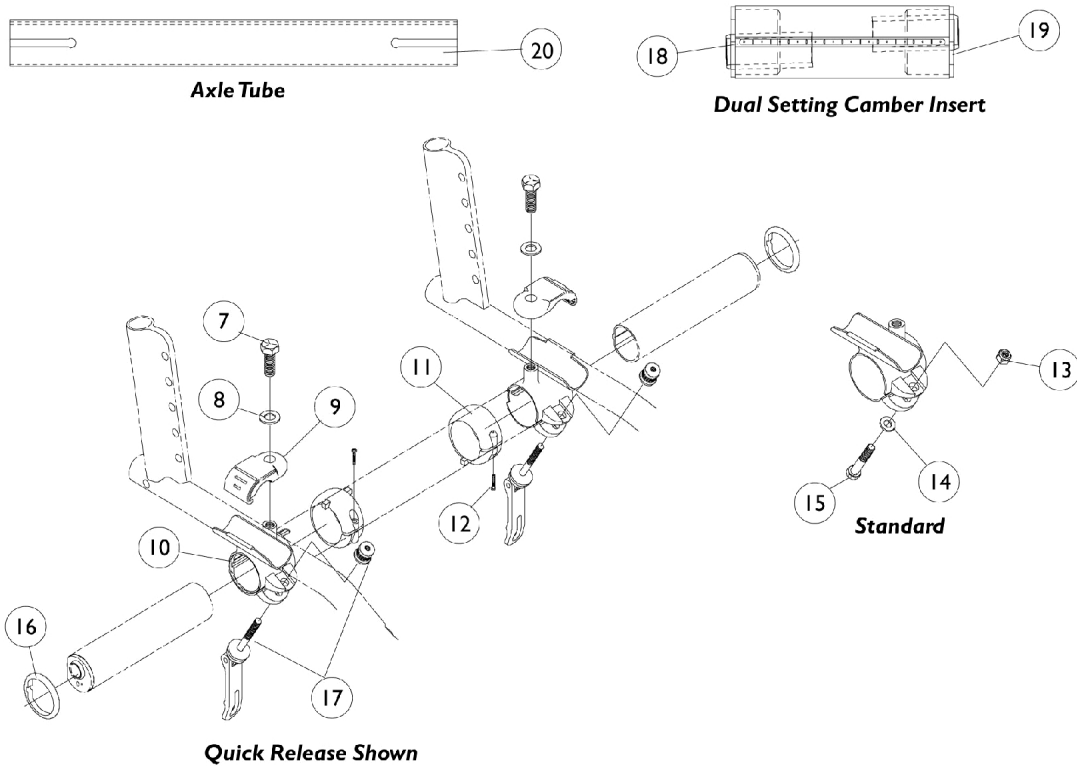

Axles & Camber Bars

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028481-NLA	NLA - Screw, Socket Head (1/4-20 x 1/2")	1	16
2		1046298-NLA	NLA - Half Clamp	1	4
3		1046235-NLA	NLA - Spacer, Camber Tube	1	2
4		1046399-NLA	NLA - Hex Jam Nut (5/8-24)	1	2
5		1048768-NLA	NLA - Bushing, Axle, Quick Release	1	2
5		1049335-NLA	NLA - Bushing, Axle, Quad Release	1	2
6		See Chart	Camber Bars	1	1
7		1065939-NLA	NLA - Screw, Hex Head (3/8-16 x 1")	1	2
8		1065940-NLA	NLA - Washer (3/8 x 3/4 x 5/64")	1	2
9		1062890-NLA	NLA - Frame, Clamp, Black	1	2
10		1062875-NLA	NLA - Clamp, Axle Tube - Right	1	1
10		1062876-NLA	NLA - Clamp, Axle Tube - Left	1	1
11		1065886	Ring, Toe-In/Toe-Out Adjusting	1	2
12		1066489-NLA	NLA - Screw, Thread Forming (M4 x 16mm)	1	2
13		1065362	Locknut (1/4-28)	1	2
14		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
15		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
16		1065887-NLA	NLA - Ring, Indexing	1	2
17		1064265-NLA	NLA - Pin, Quick Release	1	2
18		1062949-NLA	NLA - Decal, Indexing	1	2
19		See Chart	Dual Setting Camber Insert	1	1
20		See Chart	Axle Tubes	1	1
21		1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	1
22		1043487-NLA	NLA - Axle, Quad Release	1	1
23	A	1068038	NLA - Axle, Quick Release (Dome)	1	1
23	B	1113049	Axle, Quick Release w/ Dome Button, Black	1	1
23	C	1065303	Axle, Quick Release w/ Flat Button, Black	1	1
NOTE: A - Q.R. Axle (1068038) had an Anodized Dome finish. Q.R. Axle part no. 1113049, w/ Black dome button, can be used as a replacement but may want to replace in pairs due to aesthetics. B - Used on all Spoke Rear Wheels except SPIN24, SPIN25, and SPIN26 C - Used on SPIN24, SPIN25, and SPIN26 Spoke Rear Wheels					

Axles & Camber Bars

Camber Tube Hardware Kits

Kit Part Number	Description
1067258 See Note(s) A	Standard Hardware Kit
1067259 See Note(s) B	Quick Release Hardware Kit
NOTE: A - Includes items 7-16 B - Includes items 7-12,16,17	

#6 Camber Bar Before 11/25/96

Camber Angle	Part Number
0 degrees	1046237-NLA
3 degrees	1046238-NLA
6 degrees	1046239-NLA
9 degrees	1046240-NLA
12 degrees	1046241-NLA

Axles & Camber Bars

#19 Dual Setting Camber Insert

Camber Angle Dual Settings	Right	Left
0 - 3 degrees	1066219-NLA	1066229-NLA
0 - 6 degrees	1066220-NLA	1066230-NLA
0 - 9 degrees	1066221-NLA	1066231-NLA
0 - 12 degrees	1066222-NLA	1066232-NLA
3 - 6 degrees	1066223-NLA	1066233-NLA
3 - 9 degrees	1066224-NLA	1066234-NLA
3 - 12 degrees	1066225-NLA	1066235-NLA
6 - 9 degrees	1066226-NLA	1066236-NLA
6 - 12 degrees	1066227-NLA	1066237-NLA
9 - 12 degrees	1066228-NLA	1066238-NLA

#20 Axle Tubes

		Chair Width			
12"	13"	14"	15"	16"	17"
1065990-NLA	1065991-NLA	1065992-NLA	1065993-NLA	1065994-NLA	1065995-NLA
18"	19"	20"	21"	22"	
1065996-NLA	1065997-NLA	1065998-NLA	1083015-NLA	1083016-NLA	