


# Hill Holder Wheel Locks & Attaching Hardware

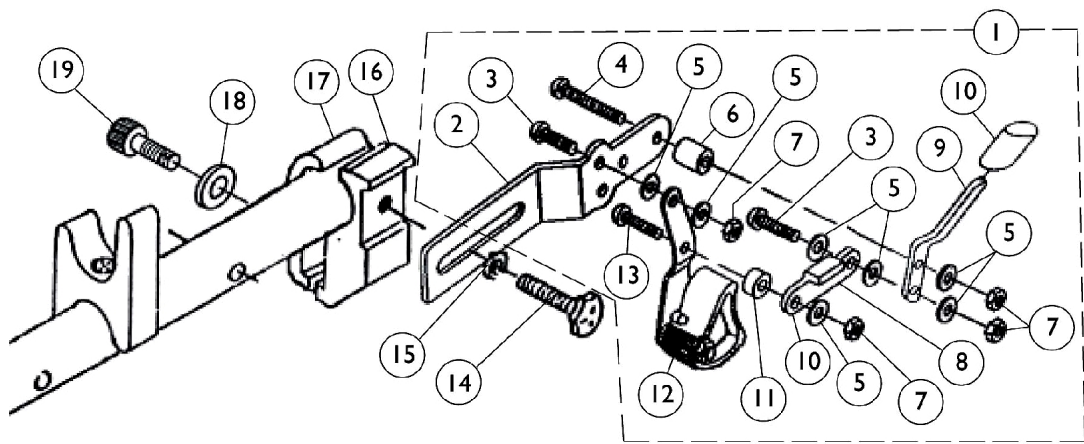

## Hill Holder Wheel Locks & Attaching Hardware

| Item Number                                                                | Note | Part Number | Description                                       | Quantity Per |          |
|----------------------------------------------------------------------------|------|-------------|---------------------------------------------------|--------------|----------|
|                                                                            |      |             |                                                   | Package      | Assembly |
| 1                                                                          | A    | 1026712     | Hill Holder Wheel Lock Assembly - Right           | 1            | 1        |
| 2                                                                          |      |             | Bracket, Black                                    | 1            | 1        |
| 3                                                                          |      |             | Screw, Socket Button Head (1/4-28 x 5/8")         | 1            | 2        |
| 4                                                                          |      |             | NLA - Screw, Socket Button Head (1/4-28 x 1-1/4") | 1            | 1        |
| 5                                                                          |      |             | Washer, Nylon (1/4 x 1/2 x 1/32")                 | 1            | 7        |
| 6                                                                          |      |             | NLA - Spacer (1/4 x 1/2 x 1/2")                   | 1            | 1        |
| 7                                                                          |      |             | Locknut (1/4-28)                                  | 1            | 4        |
| 8                                                                          |      |             | NLA - Link, Stop, Black                           | 1            | 1        |
| 9                                                                          |      |             | NLA - Handle, Wheel Lock, Black - Right           | 1            | 1        |
| 10                                                                         |      |             | Tip, Plastic, Black                               | 1            | 1        |
| 11                                                                         |      |             | NLA - Spacer                                      | 1            | 1        |
| 12                                                                         |      |             | Hill Holder Assembly - Right                      | 1            | 1        |
| 13                                                                         |      |             | NLA - Screw, Socket Button Head (1/4-28 x 3/4")   | 1            | 1        |
| 1                                                                          | A    | 1026713     | Hill Holder Wheel Lock Assembly - Left            | 1            | 1        |
| 2                                                                          |      |             | Bracket, Black                                    | 1            | 1        |
| 3                                                                          |      |             | Screw, Socket Button Head (1/4-28 x 5/8")         | 1            | 2        |
| 4                                                                          |      |             | NLA - Screw, Socket Button Head (1/4-28 x 1-1/4") | 1            | 1        |
| 5                                                                          |      |             | Washer, Nylon (1/4 x 1/2 x 1/32")                 | 1            | 7        |
| 6                                                                          |      |             | NLA - Spacer (1/4 x 1/2 x 1/2")                   | 1            | 1        |
| 7                                                                          |      |             | Locknut (1/4-28)                                  | 1            | 4        |
| 8                                                                          |      |             | Tip, Plastic, Black                               | 1            | 1        |
| 9                                                                          |      |             | NLA - Handle, Wheel Lock, Black - Left            | 1            | 1        |
| 10                                                                         |      |             | NLA - Link, Stop, Black                           | 1            | 1        |
| 11                                                                         |      |             | NLA - Spacer                                      | 1            | 1        |
| 12                                                                         |      |             | NLA - Hill Holder Assembly - Left                 | 1            | 1        |
| 13                                                                         | B,C  | 1026352     | NLA - Screw, Socket Button Head (1/4-28 x 3/4")   | 1            | 1        |
|                                                                            |      |             | Kit, Wheel Lock Hardware                          | 2            | 1        |
| 14                                                                         |      |             | NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")          | 1            | 2        |
| 15                                                                         |      |             | Washer (1/4 x 5/8 x 3/32")                        | 1            | 2        |
| 16                                                                         |      |             | NLA - Clamp, Half, Threaded                       | 1            | 2        |
| 17                                                                         |      |             | NLA - Clamp, Half, Non-Threaded                   | 1            | 2        |
| 18                                                                         |      |             | Washer, Nylon (1/4 x 7/16 x 5/64")                | 1            | 2        |
| 19                                                                         |      |             | Screw, Socket Head (1/4-20 x 1-1/4")              | 1            | 2        |
| NOTE: A - Includes items 2-13<br>B - Not Shown<br>C - Includes items 14-19 |      |             |                                                   |              |          |