
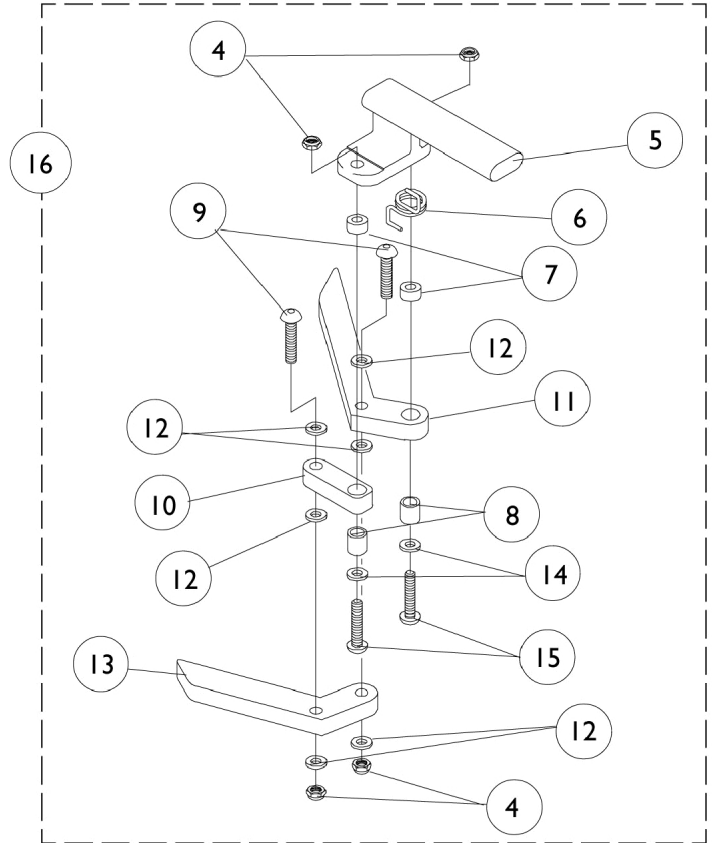


# Undermount Wheel Locks

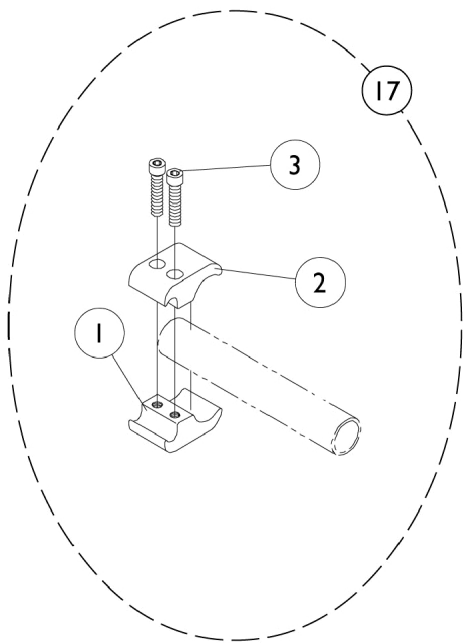

## Undermount Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
2		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
3		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
4		1025765	Locknut (1/4-28)	1	4
5		1037120-NLA	NLA - Bracket, Mounting, Black - Right	1	1
5		1037121-NLA	NLA - Bracket, Mounting, Black - Left	1	1
6		1036519-NLA	NLA - Spring, Torsion - Right	1	1
6		1036520-NLA	NLA - Spring, Torsion - Left	1	1
7		1026104-NLA	NLA - Spacer	1	2
8		1005844-NLA	NLA - Bushing	1	2
9		1055307-NLA	NLA - Screw, Allen Button Head (1/4-28 x 1")	1	2
10		1025188-NLA	NLA - Link, Wheel Lock, Black	1	1
11		1024962-NLA	NLA - Handle, Wheel Lock, Black	1	1
12		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	6
13		1024961-NLA	NLA - Shoe, Wheel Lock	1	1
14		1025728	Washer (1/4 x 1/2 x 1/32")	1	2
15		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	2
16	A	1036647-NLA	NLA - Wheel Lock, Undermount - Right	1	1
16	A	1036648-NLA	NLA - Wheel Lock, Undermount - Left	1	1
17	B	1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
			NLA - Clamp, Wheel Lock, Threaded (1")	1	2
			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4

NOTE: A - Includes items 4 - 15.  
B - Includes items 1 - 3.