


One Piece Adjustable Angle Footplate

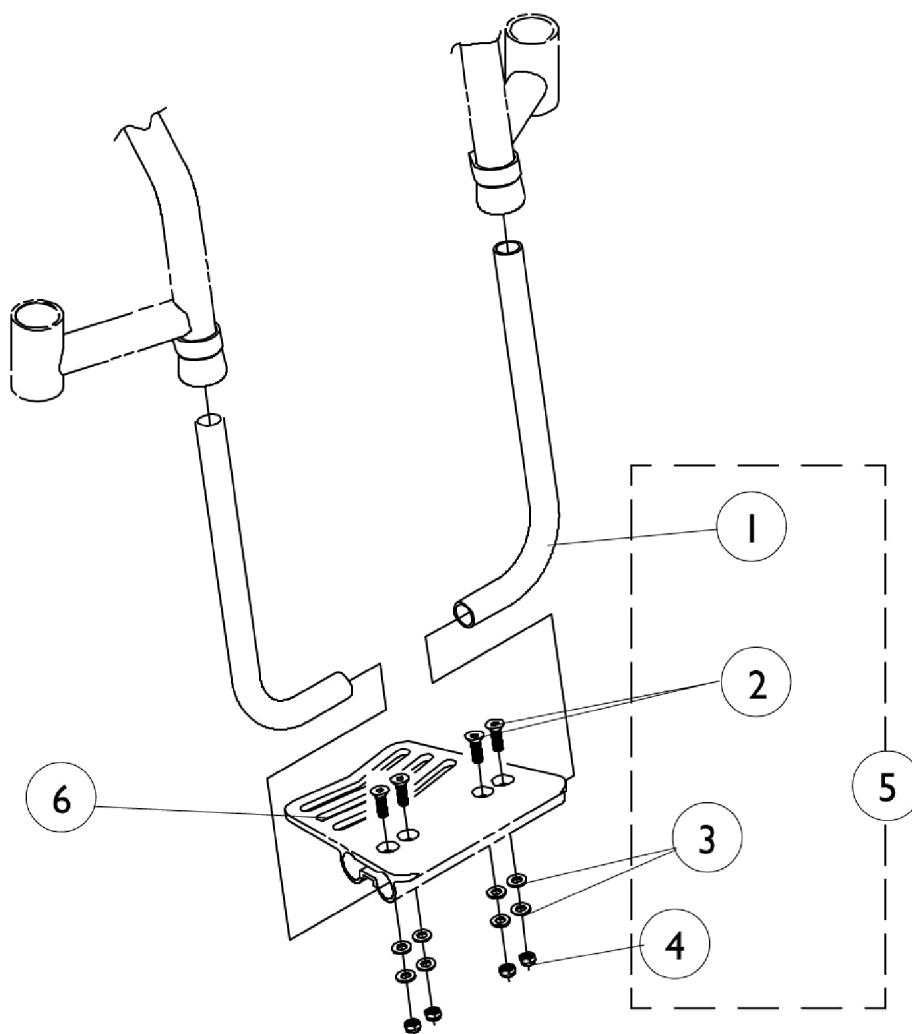

One Piece Adjustable Angle Footplate

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1046409-NLA	NLA - Tube, Support, Footplate, Black	1	2
1	B	1056296-NLA	NLA - Support Tube, Footplate	1	2
2		1046421-NLA	NLA - Screw, Socket Flat Head (1/4-28 x 1")	1	4
3		1013971	Washer, Nylon (1/4 x 7/16 x 5/64")	1	8
4		1025765	Locknut (1/4-28)	1	4
5	A,C	1046420-NLA	NLA - Hardware, Footplate Mounting	1	1
5	B,C	1058667-NLA	NLA - Hardware, Footplate Mounting	1	1
6		See Chart	#6 Footplate - One Piece Adjustable Angle	1	1
NOTE: A - Tapered Front Frame B - "V" Front Frame C - Includes items 1-4					

#6 One Piece Adjustable Angle Footplates

Chair Width	Tapered Front Frame	"V" Front Frame
12-13"	1046460-NLA	1046461-NLA
14"	1046461-NLA	1046462
15"	1046462	1046463-NLA
16"	1046463-NLA	1046464-NLA
17"	1046464-NLA	1046465-NLA
18"	1046465-NLA	1046466
19-22"	1046466	1046466