
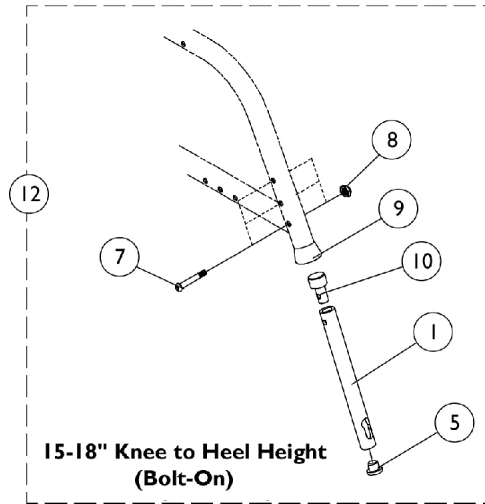


Footrest Extension Tubes

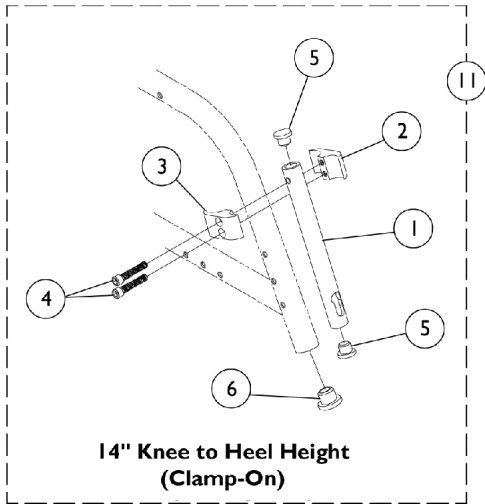

Footrest Extension Tubes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1069004-NLA	NLA - Tube, Footrest Extension - Right	1	1
1	B	1069003-NLA	NLA - Tube, Footrest Extension - Left	1	1
1	C	1069006-NLA	NLA - Tube, Footrest Extension - Right	1	1
1	C	1069005-NLA	NLA - Tube, Footrest Extension - Left	1	1
2		1043712	Clamp, Half, Threaded	1	1
3		1043711	Clamp, Half, Non-Threaded	1	1
4		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
5		1068500-NLA	NLA - Plug Button, Black (3/4")	1	2
6		1056328	Plug Button, Black (1")	1	1
7		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
8		1025765	Locknut (1/4-28)	1	1
9		1026124	Collar, Anti-Rattle	1	1
10		1068502-NLA	NLA - Plug, Extension Tube	1	1
11	D	1068982-NLA	NLA - Footrest Extension Tube Assembly (Pair)	2	1
12	E	1068929-NLA	NLA - Footrest Extension Tube Assembly (Pair)	2	1
12	F	1068930-NLA	NLA - Footrest Extension Tube Assembly (Pair)	2	1
	A	1013870-NLA	NLA - Anti-Rattle (1")	1	2
	A	1024786	Bushing, Pivot	1	2

NOTE: A - Not Shown
 B - 14-16" Knee to Heel Height
 C - 16-18" Knee to Heel Height
 D - 14" Knee to Heel
 E - 15-16" Knee to Heel
 F - 16-18" Knee to Heel