


Wheel Lock - 24" Wheel (Before 2/98)

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

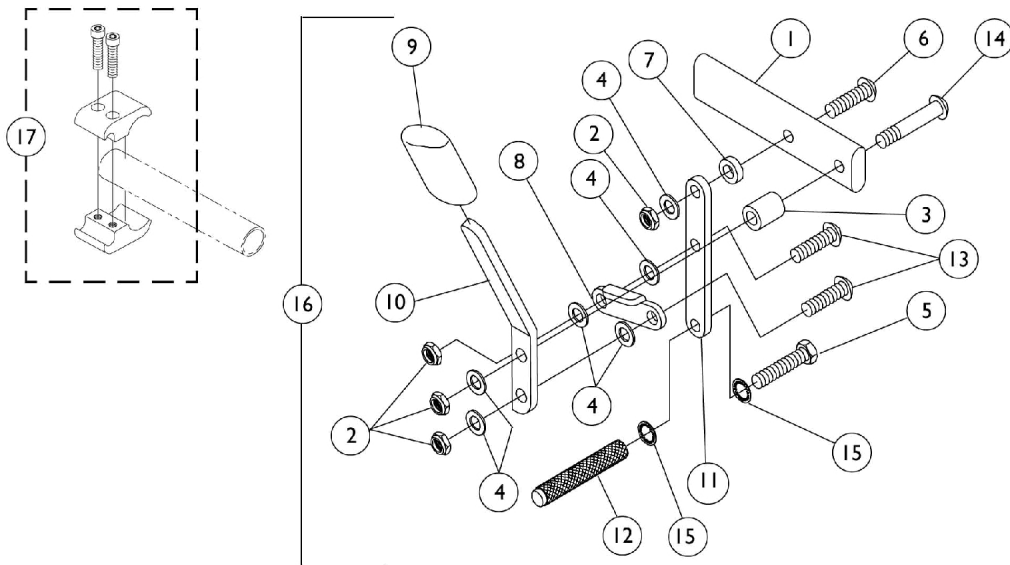

Wheel Lock - 24" Wheel (Before 2/98)

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028399-NLA	NLA - Strap, Wheel Lock	1	1
2		7930334-NLA	NLA - Locknut (1/4-28)	25	1
3		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
4		1026103-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	1	6
5		1055346-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	1
6		1055306-NLA	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
7		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
8		1018733-NLA	NLA - Link, Stop, Black	1	1
9		1006490-NLA	NLA - Tip, Plastic, Black	1	1
10		1038344-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
10		1038345-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
11		1028416-NLA	NLA - Link, Wheel Lock, Black	1	1
12		1029420-NLA	NLA - Shoe, Wheel Lock	1	1
13		1055304-NLA	NLA - Screw, Socket Button Head (1/4-28 x 5/8")	1	2
14		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
15		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	1
16		1029423	Wheel Lock, High Mount, Push to Lock - Right	1	1
16		1029424	Wheel Lock, High Mount, Push to Lock - Left	1	1
17		1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
			NLA - Clamp, Wheel Lock, Threaded (1")	1	2
			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4
	A,B	1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	2	1
	A,B	1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	2	1
	A,B	1027268	Screw, Allen Head (1/4-20 x 1")	4	1

NOTE: A - Included with Item 17.
B - Not Shown.

NOTE: If a complete assembly is desired, order the new style (After 1/99) to assure the upgrades are received.