


# Wheel Lock - 24" Wheel (From 2/98 to 2/99)

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

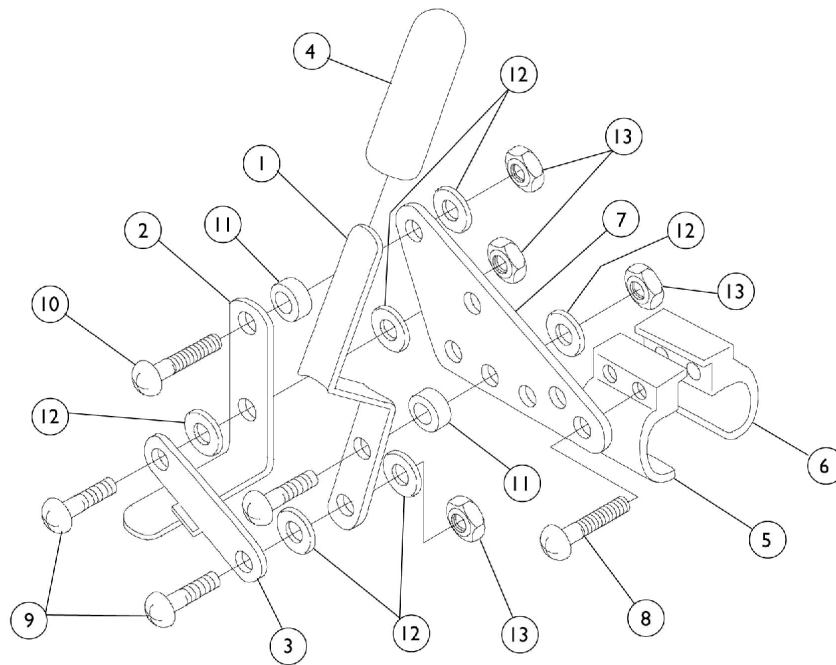

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		BM42	Handle, Black	1	1
2		440585	Shoe	1	1
3		440586	Link	1	1
4		B412M-NLA	NLA - Handle, Tip, Black	1	1
4		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
5		1108197-NLA	Clamp, Threaded - Right	1	1
5		1108198	Clamp, Threaded - Left	1	1
6		1072398-NLA	NLA - Clamp, Non-threaded	1	1
7		1072394-NLA	NLA - Plate	1	1
8		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
9		1108181-NLA	NLA - Screw, Socket Button Head (#10-32 x 5/8")	1	2
10		1003645	Screw, Socket Button Head (#10-32 x 3/4")	1	2
11		1108184-NLA	NLA - Spacer, Nylon (3/16 x 3/16 x 5/16")	1	2
12		1026103-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	1	8
13		1025735	Locknut (#10-32)	1	4

NOTE: If a complete assembly is desired, order the new style (After 1/99) to assure the upgrades are received.